stryker®


**System 6**  
**Battery Packs**  
REF 6215 (large)  
REF 6212 (small)

**Instructions For Use**

**R<sub>x</sub> ONLY**

**CE** 0197

ENGLISH	SVENSKA	ΕΛΛΗΝΙΚΑ
ESPAÑOL	DANSK	日本語
DEUTSCH	SUOMI	中文
FRANÇAIS	PORTUGUÊS	한국어
ITALIANO	NORSK	
NEDERLANDS	POLSKI	

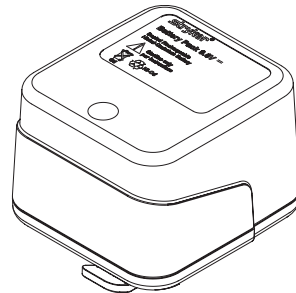## Intended Use

These rechargeable battery packs are intended for use with Stryker System 6 handpieces only. Charge these battery packs on the System 6 Battery Charger REF 6110-120 configured with the System 6 charger module REF 6110-625 only.

## User/Patient Safety\*

### WARNINGS:

- Only trained and experienced health care professionals should use this equipment. Before using any system component or any component compatible with this system, read and understand the instructions. Pay special attention to WARNING information. Become familiar with the system components prior to use. Failure to comply may result in patient and/or operating room staff injury.
- Upon initial receipt and before each use, inspect all components for damage. Damage may include bent contacts and cracks in the housing. DO NOT charge or use if damage is apparent. Failure to comply may result in patient and/or operating room staff injury.
- DO NOT use this equipment in the presence of a mixture consisting of a flammable anesthetic and air or oxygen or nitrous oxide.

## Instructions

### NOTES:

- Recharge battery packs before each surgery, even if they have not been used, to ensure the maximum running time.
  - The charge cycle takes approximately 5 to 60 minutes.
  - The expected run time for each battery is 2.5 to 10 minutes depending on demand procedure utilization.
  - Only use battery packs that have been charged within the last 48 hours.
  - Store battery packs on the charger where they will be maintained fully charged until they are sterilized. DO NOT store battery packs in handpieces where they will discharge. Discharge will occur even though the handpiece is not running.
1. To properly install the battery pack into a handpiece, read and understand the appropriate Stryker handpiece instructions for use.
  2. To properly charge the battery pack, read and understand the instructions for use provided with the Stryker System 6 Battery Charger REF 6110-120.

\*DO NOT service this equipment. If service is required, return to Stryker. For more information, contact your Stryker sales representative or call Stryker customer service at 1-800-253-3210. Outside the US, contact your nearest Stryker subsidiary.

## Cleaning and Sterilization Recommendations\*

**WARNING:** Clean and sterilize battery packs before first and every use.

### CAUTIONS:

- DO NOT immerse battery packs in liquid.
  - DO NOT use disinfectants with pH levels higher than 10.5. See the disinfectant's Material Safety Data Sheet to verify the pH range. Failure to comply may cause the housing material to crack.
  - DO NOT sterilize a battery pack if damage is apparent (cracks or other damage).
  - DO NOT leave battery packs in a hot sterilizer for more than the prescribed time as this will shorten battery pack life.
  - After sterilization, ALWAYS allow batteries and battery packs to cool for at least one hour before recharging. Failure to comply may result in damage and a shortened battery life.
  - Dry battery packs before placing them in the charger to avoid damage to the battery packs and charger.
1. Remove the battery pack from the handpiece.
  2. Inspect the battery pack housing for cracks or damage.

3. Using a sponge, wipe the battery pack clean with an enzymatic, hospital grade disinfectant having a pH range of 6.5 to 10.5.
4. Rinse the battery pack thoroughly under running water.
5. Wipe the battery pack dry with a lint-free towel.
6. To obtain optimal performance and help prevent damage, perform one of the following sterilization procedures:

#### “Flash” Autoclave:

- Gravity displacement sterilizer
- 270-272 °F (132-134 °C)
- Unwrapped in an instrument tray. Ensure nothing touches the terminals.
- 3-minute exposure time; no dry time.

#### Hi Vac:

- Pre-vacuumed sterilizer
- 270-272 °F (132-134 °C)
- Wrapped or unwrapped
- 4-minute exposure time

#### ETO:

- 100% ETO
- 120-135 °F (49-57 °C)
- Wrapped in an instrument tray or fully perforated sterilization box.
- 1-hour exposure time; 8-hour minimum aeration time.

#### STERRAD Gas Plasma:

- Perform sterilization per the manufacturer's user instructions.

*\*Sterilization validation based on the Association for the Advancement of Medical Instrumentation (AAMI) protocol.*

**NOTE:** Battery packs sterilized using the ETO method will run approximately 20% less time than battery packs sterilized by the “Flash” method.

STERRAD® is a registered trademark of Advanced Sterilization Products, a Johnson & Johnson company.

## Specifications\*

**Models:** REF 6215 Large Battery Pack, REF 6212 Small Battery Pack

**Electrical:**

	Large Battery Pack	Small Battery Pack
Charge	2.0 A	1.0 A
Output	9.6 V $\overline{---}$	9.6 V $\overline{---}$

**Duty Cycle:** See the recommended duty cycle information supplied with each handpiece.

**Enclosure Protection:** IPX0 Ordinary Equipment

**Environmental Conditions:** See the operating, storage, and transportation specifications supplied with each handpiece.

\*Specifications are approximate and may vary from unit to unit or as a result of power fluctuations.

## Symbol Definitions

 <p>NICKEL-CADMIUM rechargeable battery. Must be recycled or disposed of properly.</p> <p>Ni-Cd</p>	 <p>Recycle or dispose of properly.</p> <p>Ni-Cd</p>
--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------

## Disposal/Recycle Information

This battery pack contains Nickel-Cadmium. Check with your local environmental protection agency for special disposal restrictions.

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部件名称 (Component Name)	有毒或有害物质和元素 (Toxic or Hazardous Substances and Elements)					
	铅 (Pb)	汞 (Hg)	镉 (Cd)	六价铬 (Cr <sup>6+</sup> )	多溴联苯 (PBB)	多溴二苯醚 (PBDE)
印制电路板组件和接线 (PCB Assembly and Wiring)	X	O	X	O	O	O
电池外壳 (Battery Housing)	O	O	O	O	O	O
集装组件 (Cluster Assembly)	X	O	X	O	O	O
电池触点 (Battery Contacts)	X	O	O	O	O	O
相关金属零部件 (Associated Metal Parts)	O	O	O	O	O	O
相关塑料零部件 (Associated Plastic Parts)	O	O	O	O	O	O
泡沫塑料/橡胶零部件 (Foam/Rubber Parts)	O	O	O	O	O	O
<p>○：表示该有害有毒物质在该部件所有均质材料中的含量在 SJ/T11363-2006 标准规定的限量要求以下。            X：表示该有害有毒物质至少在该部件的某一均质材料中的含量超出 SJ/T11363-2006 标准规定的限量要求。            (企业可在此处，根据实际情况对上表中打“X”的技术原因进行进一步说明。)</p>						
<p>O: Indicates that this toxic or hazardous substance contained in all the homogenous materials for this part is below the limit requirement in SJ/T11363-2006.            X: Indicates that this toxic or hazardous substance contained in at least one of the homogenous materials used for this part is above the limit requirement in SJ/T11363-2006.            (Enterprises may further provide in this box technical explanation for marking "X" based on their actual conditions.)</p>						

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