

Anesthesia Workstation Bumper Installation

60-0064 Rev B



This guide is CONFIDENTIAL and designed only for Omnicell Technical personnel and/or designated representatives.

This guide and accompanying software and/or hardware described in it are protected under copyright laws and may not be copied, wholly or in part, without the express written consent of Omnicell, Inc. The same proprietary and copyright notices must be attached to any permitted copies as were attached to the original documents.

Omnicell, Inc.
1201 Charleston Road
Mountain View, CA 94043
(650) 251-6100
www.omnicell.com

Omnicell and the Omnicell design mark, OmniBuyer, OmniCenter, OmniRx, OmniSupplier, Pandora, PandoraVIA, SafetyMed, SafetyStock, and Sure-Med are registered trademarks. Anesthesia TT, Anesthesia Workstation, Anywhere RN, Executive Advisor, FlexBin, Medication Surveillance, OmniDispenser, OmniLinkRx, OmniScanner, OmniTrack, Omni TT, Open Touch, OptiFlex, OptiFlex MobileTrack, Point-to-Point Medication Safety, ProServ1, SecureVault, See & Touch, SinglePointe, TempCheck, Touch & Go, vSuite, and WorkflowRx are trademarks of Omnicell, Inc. in the United States and internationally. All other trademarks and trade names are the property of their respective owners.

Copyright 2011 Omnicell, Inc. All rights reserved.

Table of Contents

Bumper Installation	1-1
Overview	1-1
Kit/Parts	1-1
Procedures	1-2
 Documentation Feedback	 FB-1

Bumper Installation

Overview

In order to avoid damage to the bumpers on the frame of the Anesthesia Workstation (AWS), Manufacturing will now ship the bumpers as a kit (P/N: 15-7069). The quarter-turn fasteners needed to secure the bumpers in place will be installed by Manufacturing before shipment. In the field, feel for the quarter-turn fasteners below the frame of the Workstation. Secure the bumpers on top accordingly and tighten the fasteners by hand. The following steps detail the exact location of the fasteners.



Note: In order to obtain a clear view, pictures were taken with the frame on its side. However, in the field, do not attempt to roll the Workstation onto its side.

Kit/Parts

- Bumpers Kit (P/N: 15-7069)
 - 57-7071 Bumper, Front
 - 57-7072 Bumper, Rear
 - 57-7073 Bumper, Right
 - 57-7074 Bumper, Left



Figure 1-1. Bumper Kit

Procedures

1. Place the front bumper on the front of the frame.
2. Engage the 11 quarter-turn fasteners on the underside of the front of the workstation with the corresponding retainers in the bumpers.



Figure 1-2. Front bumper

3. Tighten the fasteners to secure the front bumper.
4. Place the rear bumper on the frame.
5. Engage the eight quarter-turn fasteners installed on the rear, bottom of the frame with the retainers in the bumpers.

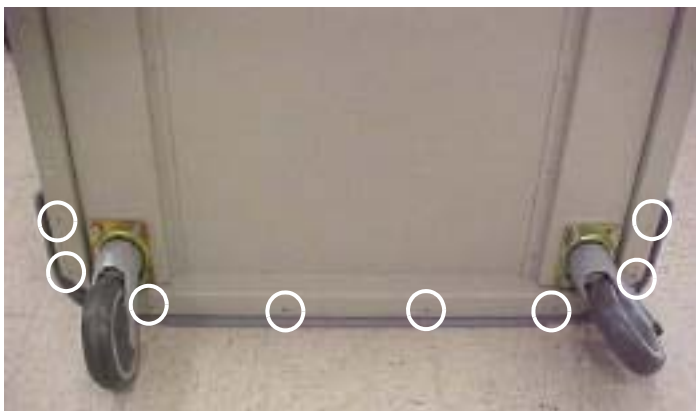


Figure 1-3. Rear bumper

6. Tighten the fasteners to secure the rear bumper to the frame.

7. Secure the left and the right bumper respectively, by engaging the fasteners with the retainers in the bumpers.



Figure 1-4. Side bumpers

8. Tighten the three quarter-turn fasteners installed on either side of the frame.

Documentation Feedback

This document is designed to provide relevant technical information to those responsible for the implementation, service, and support of Omnicell products. The Documentation team needs your input, so we can continue to improve our publications.

Sending Comments to the Technical Documentation Team

Did this document meet your needs? If so, please let us know what we're doing right. If not, please provide specific feedback. E-mail or fax your feedback as follows:

- E-mail: **Documentation Requests** e-mail group at **documentationrequests@omnicell.com** (specify the document title or PN).
- Fax: Send this page, along with your feedback, to **(650) 251-6266**, attention: Documentation.

This document is designed to provide relevant technical information to Omnicell personnel responsible for the implementation, service, and support of Omnicell Automation Systems.

Feedback Form

Name:	E-mail:
Dept./Title:	Phone:
Feedback:	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	
<hr/>	

