

System 2000 Rhapsody, Primo, Harmonie



WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the material safety data sheet of the disinfectant.

To prevent cross-contamination, always follow the disinfection instructions in the Instructions For Use.

To avoid eye or skin irritation, never disinfect in the presence of a patient.

Caregiver obligations

Between patients

Clean and disinfect System 2000 bath before every use.

Clean and disinfect with disinfection unit (Still Bath and Hydrosound System)

NOTE

For disinfection of the Hydromassage system start on page 2.

Remove visual residue

1. Raise the bathtub to convenient working position.
2. Make sure all functions are turned off.
3. Open the drain.
4. Detach and place all accessories at the bottom of the bathtub.
5. Shower and rinse the bathtub and the accessories with water to remove visible residue. Turn the accessories and rinse again.

Clean

6. Put on protective gloves and eye wear.
7. Lift out the disinfection spray handle.
8. Press the disinfection button.
9. Spray disinfectant into the bathtub until the flow meter shows correct mixing value according to the Instructions For Use. Adjust if needed.
10. Spray disinfectant in the drain. Scrub the drain with a brush.
11. Close the drain.
12. Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.
13. Spray into the surface overflow outlet.
14. Press the disinfection button to turn it off.
15. Scrub the surfaces of the bathtub, loose parts and accessories thoroughly with a soft bristle brush.
16. Open the drain.
17. Rinse all parts clean with plenty of water with a temperature of 25 °C (77 °F).
18. Soak a cloth in disinfectant and wipe all other contact areas: the outside of the bathtub, metal sensory knobs and hand control.
19. Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.

Disinfect

20. Spray disinfectant in the drain.
21. Close the drain.
22. Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.
23. **Allow appropriate contact time according to the instructions of the disinfectant.**
24. Push the disinfection button to end the disinfection cycle.

Automatic Disinfection only:
If the preset disinfection time has not elapsed other functions are disabled for at least 2 minutes.
25. Put back the disinfection spray handle.
26. Open the drain.
27. Push the button for shower.
28. Rinse all parts clean with water having a temperature of 25 °C (77 °F).
29. Soak a cloth in disinfectant and wipe all other contact areas: the outside of the bathtub, metal sensory knobs and hand control.
30. **Allow appropriate contact time according to the instructions of the disinfectant.**
31. Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.
32. Leave to air dry.

Clean and disinfect with a spray bottle

Remove visual residue

1. Raise the bathtub to convenient working position.
2. Make sure all functions are turned off.
3. Open the drain.
4. Detach and place all accessories at the bottom of the bathtub.
5. Shower and rinse the bathtub and the accessories with water to remove visible residue. Turn the accessories and rinse again.

Clean

6. Put on protective gloves and eye wear.
7. Spray disinfectant in the drain. Scrub the drain with a brush.
8. Close the drain.
9. Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.
10. Spray into the surface overflow outlet.
11. Scrub the surfaces of the bathtub, loose parts and accessories thoroughly with a soft bristle brush.
12. Open the drain.
13. Rinse all parts with plenty of water with a temperature of 25 °C (77 °F).
14. Soak a cloth in disinfectant and wipe all other contact areas: the outside of the bathtub, metal sensory knobs and hand control.

15. Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.

Disinfect

16. Use a spray bottle and spray disinfectant in the drain.
17. Close the drain.
18. Spray disinfectant on all internal bathtub surfaces and on all sides of the accessories.
19. **Allow appropriate contact time according to the instructions of the disinfectant.**
20. Open the drain.
21. Push the button for shower.
22. Rinse all parts clean with water having a temperature of 25 °C (77 °F).
23. Soak a cloth in disinfectant and wipe all other contact areas: the outside of the bathtub, metal sensory knobs and hand control.
24. **Allow appropriate contact time according to the instructions of the disinfectant.**
25. Wet a new cloth with water and remove the disinfectant. Occasionally rinse the cloth in running water.
26. Leave to air dry.

Clean and disinfect the Hydromassage System

With automatic disinfection

1. Raise the bathtub to convenient working position.
2. Close the drain.
3. Unscrew the screws and remove the suction inlet screen. Clean the suction inlet screen separately.
4. Press the disinfection button to start the disinfection cycle. Disinfectant comes out from the Hydromassage system nozzle. The disinfection cycle ends with the Hydromassage system being rinsed with water.
5. Spray disinfectant into the surface overflow outlet.
6. Continue with the remaining disinfection instructions according to *Clean and disinfect with disinfection unit (Still bath and Hydrosound System)*, on page 1.
7. **Allow appropriate contact time according to the instructions of the disinfectant.**
8. Press the Shower button and rinse the surface overflow outlet.
9. After completed disinfection, attach the suction inlet screen with the screws and tighten them gently.

Without automatic disinfection

1. Raise the bathtub to convenient working position.
2. Open the drain.
3. Unscrew the screws and remove the suction inlet screen. Clean the suction inlet screen separately.
4. Pull out the turnable Hydromassage system nozzle if a treatment hose is going to be connected.
5. Insert the treatment hose, if any, in the Hydromassage system outlet.
6. Spray disinfectant into the suction inlet until disinfectant comes out of the Hydromassage system nozzle/treatment hose.
7. Spray disinfectant into the surface overflow outlet.
8. **Allow appropriate contact time according to the instructions of the disinfectant.**
9. Press the disinfection button to turn it off.
10. Press the shower button and connect the shower handle to the suction inlet.
11. Turn the Hydromassage system knob to its highest intensity.
12. Press the trigger and rinse until the water comes out **clear** from the Hydromassage system nozzle/treatment hose.
13. Remove the treatment hose, if any.
14. Put back the nozzle into the Hydromassage system outlet.
15. Rinse the surface overflow outlet with water.
16. Continue with the remaining disinfection instructions according to *Clean and disinfect with disinfection unit (Still bath and Hydrosound System)*, on page 1.
17. After completed disinfection, attach the suction inlet screen with the screws and tighten them gently.