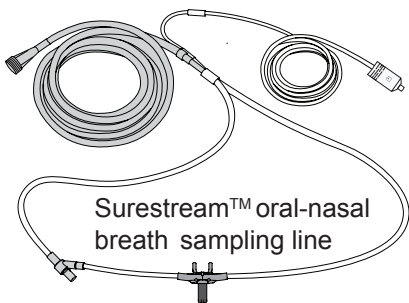
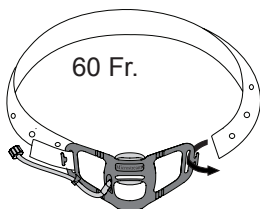
Surestream™

ETCO₂ SAMPLING

Smart SureLine Guardian™ Directions for Use

Smart
SureLine
Guardian™
bite block

Sampling lines: 7 ft / 2 m or 14 ft / 4 m

US Patents: www.covidien.com/patents.

© 2015 Oridion Medical 1987 Ltd.
All rights reserved.

PN 012575F

Oridion®

Surestream™ EtCO₂ Sampling Smart SureLine Guardian™

Intended Use

The Smart SureLine Guardian™ set is intended to sample oral and nasal CO₂ and administer supplemental oxygen via the nose and mouth, for patients who can wear a 60 fr. bite block, during upper endoscopy type procedures. **The set is intended for single patient use only.**

Contents:

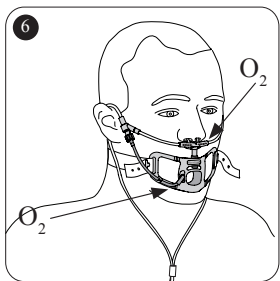
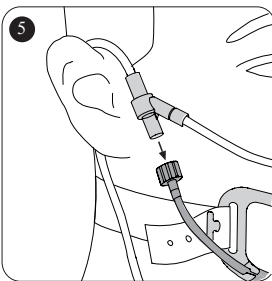
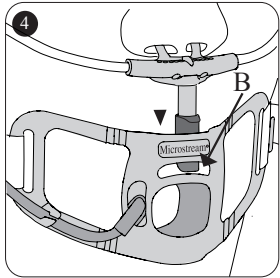
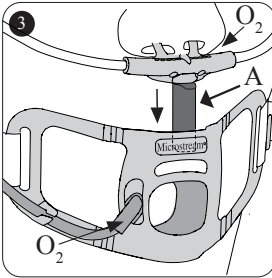
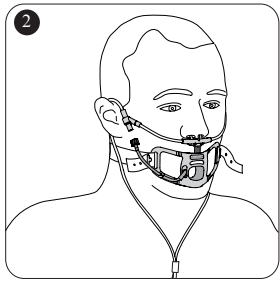
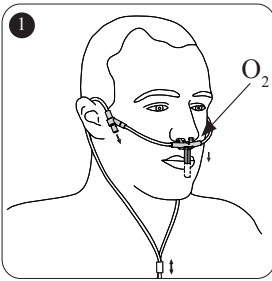
25 Smart SureLine Guardian™ kits that contain:

- 1 Smart SureLine Guardian™ bite block
 - 1 Surestream™ oral/nasal breath sampling line with O₂ delivery
- Note: A hook and loop strap (PN 012542) is available for use instead of the rubber strap, for enhanced securing of the Guardian™ on the patient. See insert for usage instructions.

Instructions for Use

See the sketches below for instructions for use. Please note:

- Connect the sampling line to the monitor by inserting the sampling line connector clockwise into the monitor CO₂ port and turning the connector until it can no longer be turned, to ensure that it is connected securely to the monitor. This will assure that there is no leak of gases during measurement at the connection point and that measurement accuracy is not compromised.
- Place the oral/nasal sampling line on patient. Ensure that the unattached end of the O₂ connector on the right side of the sampling line is facing downward (see sketch 1). Adjust length of oral prong (A in sketch 3) until fully extended (sketch 1).
- Place the Smart SureLine Guardian™ bite block on the patient after the oral/nasal sampling line is in place. Ensure that the strap is tightened and that it remains under the O₂ and CO₂ lines during the procedure. Once both parts are in place, adjust the length of the oral prong of the sampling line so that it rests in the channel of the Smart SureLine Guardian™ bite block (B in sketch 4) and is visible through the window. (This is to accommodate different patient face sizes.) Please note that the oral prong may need to be repositioned to keep it in the channel during the procedure.
- The oral/nasal sampling line includes an O₂ luer connector. Attach this connector to the Smart SureLine Guardian™ bite block as required (see sketch 5). If sampling line and Smart SureLine Guardian™ bite block are not connected via the O₂ luer, 100% of the O₂ flow will be delivered to the patient's nares. If sampling line and Smart SureLine Guardian™ bite block are connected via the O₂ luer (as recommended), O₂ will simultaneously be delivered to the patient's nares and mouth, with 80% of the O₂ flow going to the patient's mouth and 20% dispersed through the O₂ holes below the patient's nares.
- After the Smart SureLine Guardian™ bite block is removed from the patient at the end of the procedure, CO₂ measurement and O₂ delivery can continue during recovery with the sampling line alone. O₂ delivery should be <5L/min when being delivered only through the nasal cannula.

Warning

- Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Cautions

- Securely connect all components. Loose connections or failure to correctly insert the oral prong into the Smart SureLine Guardian™ channel may cause an inaccurate measurement of respiratory gases. The oral prong must remain in the channel throughout the procedure.
- Ensure that tubing is not stretched during use.
- Sedation may cause hypoventilation and CO₂ waveform distortion or disappearance. Waveform attenuation or disappearance is an indicator that the status of the patient's airway should be assessed.
- Partial blockage of the oral airway due to endoscope positioning may cause periods of low readings and rounded waveforms. The occurrence will be more pronounced with high oxygen delivery levels.
- If O₂ is delivered at flow rates greater than 10 l/min, the patient's exhaled CO₂ may be diluted, producing attenuated waveforms and EtCO₂ values.
- If CO₂ insufflation is performed during CO₂ monitoring, the EtCO₂ values will accordingly rise very significantly and this may result in device alarms and abnormally high waveforms until the CO₂ is evacuated from the patient.
- Expect delay times for the following products to be longer than the standard by the given times: Long (4m) Smart SureLine Guardian™ - 3 seconds.
- The Guardian is recommended for use with oxygen provided at up to 10l/min, or 5l/min with the sampling line alone. At higher levels of oxygen provision, dilution of CO₂ readings may occur, leading to lower CO₂ values.

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 Single patient use	For use with a single patient
Rx ONLY	Rx Only
 Consult instructions for use	Consult instructions for use
 Caution, consult accompanying documents	Caution, consult accompanying documents
 -4°F -20°C 131°F 55°C Temperature limitation	Storage temperature
 PHT DEHP	DEHP
 This product does not contain natural rubber latex	Does not contain natural rubber latex
 Fire hazard	Fire Hazard
 0482	CE Mark

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Oridion®, Microstream®, FilterLine®, Smart CapnoLine®, CapnoLine®, Smart CapnoLine Guardian™, Surestream™, and Smart SureLine Guardian™ are trademarks or registered trademarks of Oridion Medical 1987 Ltd.™* and ®* are trademarks of their respective owners.

Oridion Medical 1987 Ltd.
7 Hamarpe St.
P.O. Box 45025
Jerusalem 9777407, Israel
US Toll Free: 1-888-674-3466
1-888-ORIDION

Obelis S.A.
Bd Général Wahis 53
1030 Brussels
Belgium

www.oridion.com