## **Nebulizer Preventative Maintenance**

## **Preventive Maintenance Checklist**

## Jet Nebulizer

- 1. Inspect exterior of equipment for damaged or missing hardware.
- 2. Inspect the power cord, strain relief and plug/s for any signs of damage.
- 3. Turn unit off, open user accessible covers and inspect unit for damage.
- 4. Clean unit interior components and exterior with vacuum or compressed air.
- 5. Inspect interior for signs of corrosion or missing hardware. Repair as required.
- 6. Inspect electrical components for signs of excessive heat or deterioration.
- 7. Inspect water jet assembly including orifice. Check for water leakage.
- 8. Check gaskets and 0-rings.
- 9. Verify correct operation of all buttons, controls, displays and/or indicators.
- 10. Verify correct operation of unit in all functional modalities.
- 11. Clean exterior of unit including all accessories, cables, controls and displays.
- 12. Change air intake filter when it is gray or dirty.

## Ultrasonic Nebulizer:

- 1. Inspect exterior of equipment for damaged or missing hardware.
- 2. Inspect the power cord, strain relief and plug/s for any signs of damage.
- 3. Turn unit off, open user accessible covers and inspect unit for damage.
- 4. Clean unit interior components and exterior with vacuum or compressed air.
- 5. Inspect interior for signs of corrosion or missing hardware. Repair as required.
- 6. Inspect electrical components for signs of excessive heat or deterioration.
- 7. Inspect oxygen inlet orifice for restrictions.
- 8. Inspect water jet assembly including orifice. Check for water leakage.
- 9. Check gaskets and 0-rings.
- 10. Verify correct ultrasonic output for sharp water cone and misting.
- 11. Verify correct operation of low-water cut-off circuit.
- 12. Tune RF output if needed.
- 13. Verify correct operation of all buttons, controls, displays and/or indicators.
- 14. Verify correct operation of unit in all functional modalities.
- 15. Clean exterior of unit including all accessories, cables, controls and displays.
- 16. Clean air filter.
- 17. Clean nebulizer chamber and lid daily or between each patient.
- 18. Clean hoses and tubes daily.