

F-30.xx

Service Manual

NOTE This manual is related to the following Equipment:

Suction Pump F-30.xx







F-30.xx

Service Manual

All equipment and accessories should be in good working order prior to use.

It is the responsibility of each facility's engineering department to test equipments and "routine basis" to determine if the devices are functioning within specifications.

No device or accessory should be used if a malfunction is suspected.

The installation, operation, search for breakdowns and the repair of equipment must only be performed by trained personnel⁽¹⁾ who have sufficient knowledge about the equipment, spare parts, and the information supplied.



BEFORE STARTING TO INSPECT THE DEVICE, MAKE SURE IT IS NOT CONNECTED TO THE ELECTRIC POWER SUPPLY NETWORK.

DO NOT USE THE APPLIANCE IN THE PRESENCE OF INFLAMMABLE AND/OR EXPLOSIVE LIQUIDS, GASES OR MIXTURES.

DO NOT LEAVE THE APPLIANCE EXPOSED TO ATMOSPHERIC AGENTS OR HEAT SOURCES.

USING THE APPLIANCE UNDER DIFFERENT CONDITIONS THAN THOSE SPECIFIED IN THIS HANDBOOK MAY PREJUDICE ITS SAFETY AND ITS TECHNICAL PARAMETERS

DELIVERY AND UNCRATING OF UNIT

Inspect equipment and shipping crate immediately upon receipt. If any damage is apparent, you should both report it to the trucking delivery person and contact the transportation company immediately.

Make notes of any damage on the Bill Of Lading. Retain all shipping materials for inspection. Any claims for damage must start at the receiving point. Do not return goods without written authorization.

When submitting a claim for shipping damage, request that the carrier inspect the shipping container and equipment.

Check packing slip carefully and make sure all materials have been received as indicated on the packing ticket.

An educated (qualified) person has sufficient knowledge or experience to allow him to avoid the dangers presented by electricity, whereas an informed (trained) person is sufficiently informed or works supervised by qualified persons, so as to allow him to avoid the dangers presented by electricity (e.g. operators or maintenance staff).



F-30.xx

Service Manual

TECHNICAL SPECIFICATIONS

LENGTH	mm	450
WIDTH	mm	150
HEIGHT	mm	280
WEIGHT	kg	4,00
POWER SUPPLY		230V – 50/60 Hz
MAX VACUUM	bar	- 0,75
MAX FLOW – WITH FILTER	l/mins	10
MAX FLOW -DIRECT MEASUREMENT ON ASPIRATION VENT	l/mins	40
FUSES	Α	2x1,25

ACCESSORIES INCLUDED

CODE N° 05.0001 OR 05.0002 05.00552 05.0010 05.0016 DESCRIPTION JAR 1000 ML OR 2000 ML SET OF TUBES ASPIRATION PROBE CH 14 (DISPOSABLE) ANTIBACTERIAL/HYDROPHOBIC FILTER (DISPOSABLE)



F-30.xx

Service Manual

OPERATING

The appliance is a portable SUCTION UNIT FOR EMERGENCY USE, MINOR SURGERY AND aimed at freeing the airways from organic secretions.

OPERATIONAL TEMPERATURE: BETWEEN 0° AND 50° RECOMMENDED RECHARGE TEMPERATURE: BETWEEN 15° AND 30° OPERATIONAL HUMIDITY: BETWEEN 20 AND 90% RU

BEYOND 2500 METERS OF HEIGHT, THE ASPIRATION FLOW MAY DECREASE.

THE FOLLOWING STEPS ARE POSSIBLE ONLY IF THE APPLIANCE IS COMPLETELY RECHARGED

1.TIGHTLY SCREW THE LID ON THE JAR TO GUARANTEE A PERFECT SEAL

2.CONNECT THE SHORT TUBE TO FILTER 1 ON THE "ASPIRATION" 2 VENT

3.CONNECT THE OTHER END OF THE TUBE TO THE **''VACUUM'' 3** VENT WITH THE FLOAT ON THE JAR'S LID

4.CONNECT THE LONG TUBE **4** TO THE OTHER "**PATIENT**" **5** VENT ON THE JAR'S LID (VENT WITHOUT FLOAT)

5.CONNECT THE OTHER END OF THE LONG TUBE **4** TO THE PROBE CONNECTOR **6** AND EVENTUALLY THE SUPPLIED PROBE

7.PRESS THE ON/OFF SWITCH 7

8.ASPIRATION INTENSITY CAN BE VARIED WITH THE RELEVANT CONTROL ON THE FRONT PANEL OF THE APPLIANCE **8** AND CAN BE CHECKED ON THE APPLIANCE FRONT PANEL **9** VACUOMETER.

9.IT IS RECOMMENDED TO CARRY OUT A FIRST ASPIRATION CYCLE WITH AT LEAST 500 ML OF WATER ONLY.

10. THE ASPIRATED FLUID WILL FLOW IN THE RELEVANT JAR. SHOULD THE AMOUNT OF FLUID EXCEED THE JAR'S CAPACITY, THE OVERFLOW VALVE WILL STOP ASPIRATION.

11.IMMEDIATELY SWITCH OFF THE APPLIANCE BY SETTING THE **ON/OFF 7** SWITCH ON THE "**OFF**" POSITION.





F-30.xx

Service Manual

PERIODIC MAINTENANCE CHECK

SUCTION UNIT WAS DESIGNED TO KEEP MAINTENANCE TO A MINIMUM. NEVERTHELESS IT IS NECESSARY TO CARRY OUT A DAILY CHECK BEFORE USING THE APPLIANCE.

BEFORE CARRYING OUT ANY CHECK DUE TO MALFUNCTIONING OR ANOMALIES, CALL THE FAZZINI S.R.L. TECHNICAL SERVICE OR PERSONNEL QUALIFIED BY FAZZINI S.R.L.

BATTERY CAN BE CHANGED ONLY BY TRAINED PEOPLE

FAZZINI S.R.L. DOES NOT OFFER ANY WARRANTY ON APPLIANCE THAT UPON INSPECTION BY THE TECHNICAL SERVICE SHOWS SIGNS OF TAMPERING.

THE OPERATOR MUST EXCLUSIVELY REPLACE THE SUPPLIED ACCESSORIES IF NECESSARY IN LINE WITH THE APPLIANCE'S USE INTENSITY

ASPIRATION TUBES SHOULD BE STERILIZED AFTER EACH USE. IN CASE OF REPAIR, THE USER SHOULD CALL FAZZINI S.R.L. TRANSPORTATION COSTS MUST ALWAYS BE AGREED WITH FAZZINI S.R.L. BEFORE SHIPPING FAILED PIECES.

SHOULD ANY FLUID LEAK IN THE APPLIANCE, CALL THE TECHNICAL SERVICE OF FAZZINI S.R.L. FOR MAINTENANCE. DO NOT OPEN THE UNIT.

STORAGE

THE SUCTION UNIT MUST BE STORED IN A DRY LOCATION, PROTECTED FROM ATMOSPHERIC AGENTS STORAGE TEMPERATURE: BETWEEN -10° AND + 55° RELATIVE HUMIDITY BETWEEN 20 AND 80%






F-30.xx

Service Manual

Legenda F-30.xx

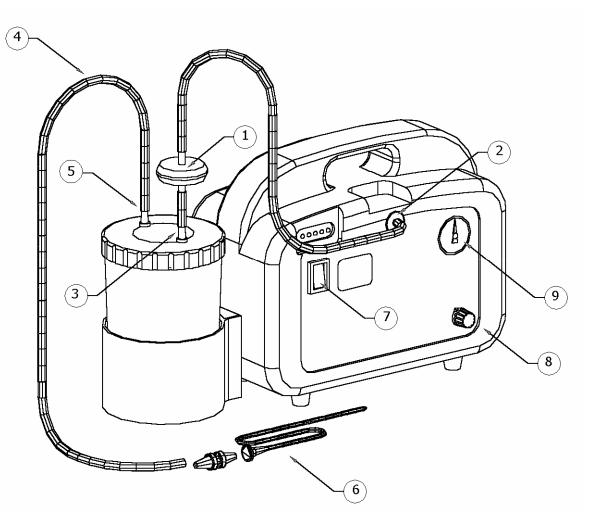


Filter
Vacuum drive
Jar drive to the filter
Disposable tube to the patient
Jar drive to the patient
Suction catheter CH 14
Main switch
Vacuum regulator
Vacuum gauge



F-30.xx

Service Manual

General assembly – F-31.xx

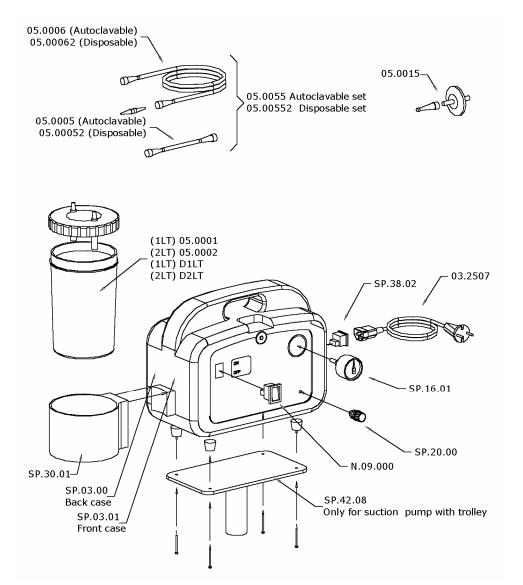

Doc : Service Manual - F-30.doc Rev. 2 / 12 APR 2006 Page 7 of 10




F-30.xx

Service Manual

Functional layout

Main

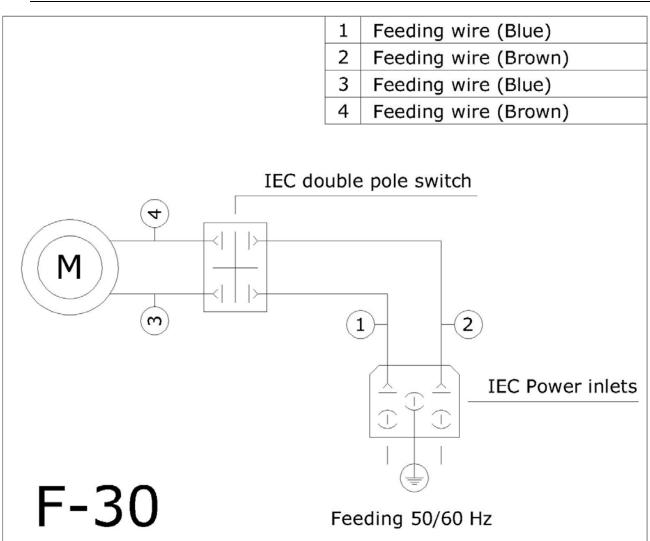




F-30.xx

Service Manual

Pneumatic circuit

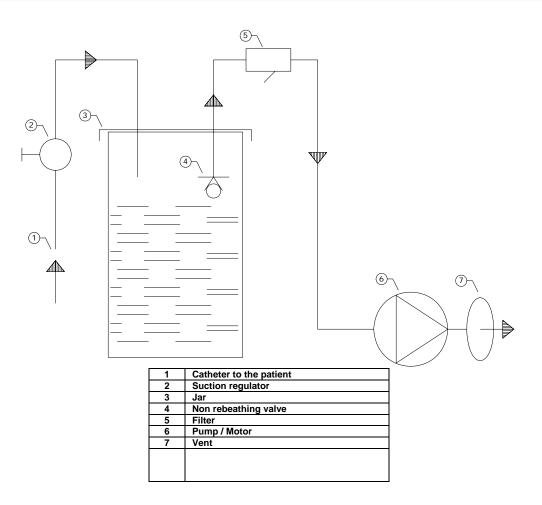



F-30.xx

Service Manual

Troobleshooting

	Situation		Possible Cause	Solution
1	Equipment will not turn on	а	Disconnected or loose cord	Check connections and reconnect unit
		b	Faulty wall outlet	Use another wall outlet
		с	Fuses are damaged	Check the fuses inside SP.38.02 If fuses are damaged check the electrical insulation on the electrical apparatus
		d	Main rocker switch is damaged	In ON position, check the presence of the light ; if no light is detected replace with a spare switch
		e	Internal wiring connection are not completed	Open the equipment and verify internal electrical wiring (see functional layout)
	Situation		Possible Cause	Solution
2	Suction fault	а	Vacuum regulator do not work properly	Turn the manifold SP.21
		b	Internal tubing are not properly connected	Open the equipment and verify internal tubing connection
		с	Internal tubing are choked	Open the equipment and verify internal tubing bends
		d	Pump fault	Open the equipment ,disconnect the tubing and check the pump