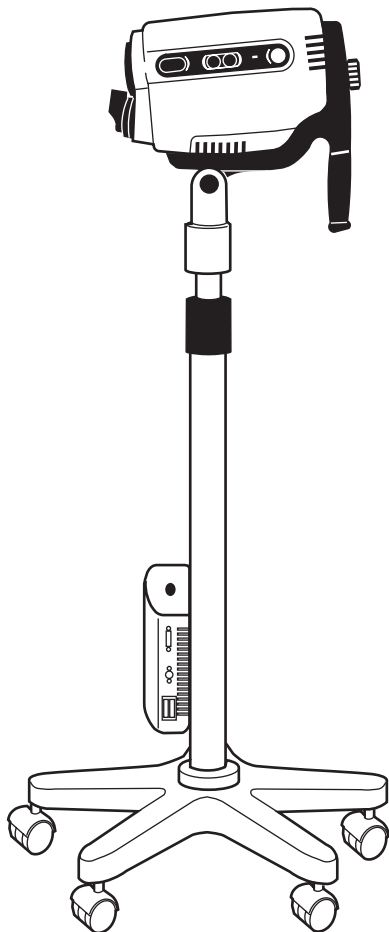
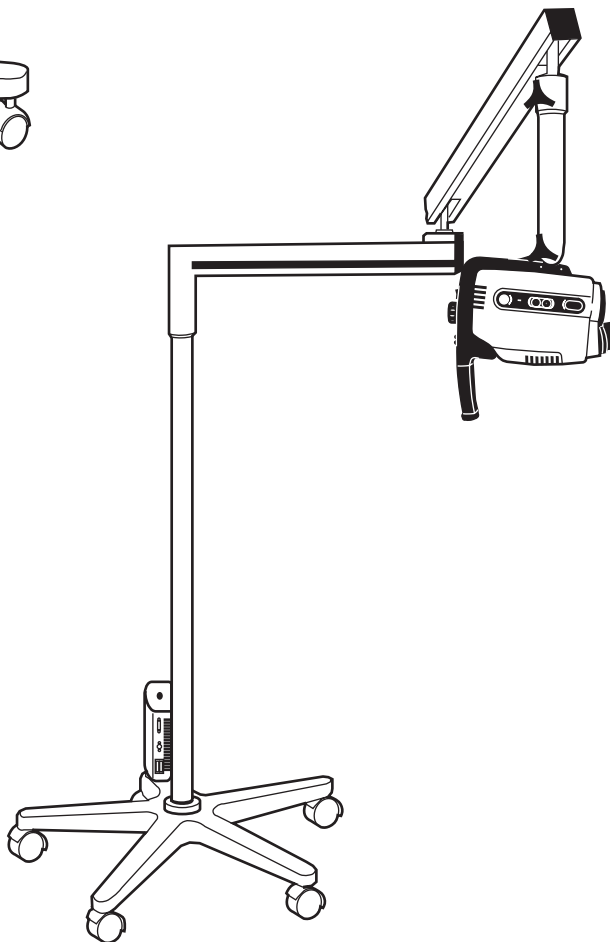
Video Colposcope

Models:
88002/88004/88006/89002/89004/89006

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Español (Spanish)101

Table of Contents

Conventions	2
General Precautions.....	2
Symbols.....	3
Summary of Warnings and Cautions.....	4
Components	5
Video Colposcope	5
Optional Accessories.....	5
Nomenclature	7
Video Colposcope Front View	7
Video Colposcope Side View	7
Video Colposcope Back View.....	8
Video Colposcope Bottom View	8
Vertical Pole Stand	9
Swing Arm Stand	9
Monitor.....	10
Connection Diagrams	11
Connecting Video Colposcope with Monitor.....	11
Connecting Video Colposcope with VCR/Video Printer	11
Preparation for Use.....	12
General Precautions.....	12
Assembly of Video Colposcope	12
Vertical Pole Stand Assembly	12

Swing Arm Stand Assembly	13
Setting Up Video Colposcope	14
Operation	15
Power Switch.....	15
Lamp Ignition.....	15
Focus and Zoom Controls	15
Green Filter Controls.....	15
Illumination Beam Director	15
Vertical Height Adjustment Ring	15
Swing Arm Height Adjustment.....	15
Positioning Video Colposcope	15
Remote Video Functions	15
Maintenance	16
Video Colposcope Cleaning	16
Disinfecting Solutions.....	16
Illumination Lens Cleaning.....	16
Camera Lens Cleaning	16
Replacing Lamp	17
Replacing Fuses.....	19
Service Information	19
Troubleshooting	20
Specifications	21

Thank you for purchasing the Welch Allyn Video Colposcope. Follow the operation and maintenance instructions found in this manual and your video colposcope will provide you with many years of reliable service. Please read these instructions thoroughly before attempting to use your new video colposcope.

CAUTION: Federal law restricts sale of this device to, or to the order of, a physician or other appropriately licensed medical professional.

WARNING: Users of this equipment should be thoroughly trained in the appropriate medical procedures. Furthermore, they should take the time to read and understand these instructions before performing any procedure. They should also read and understand the instructions for any other equipment used in conjunction with the video colposcope (i.e. electrosurgical generators). Failure to do so may result in injury to the patient and/or damage to the video colposcope.

IMPORTANT: The material outlined in this manual should be reviewed and understood prior to operation of the equipment.

NOTE: This equipment has been tested and complies with emissions standard EN60601-1-2: 1993, CISPR Publication 11:1990/EN55011: 1991 for Radiated and Conducted Emissions, Group 1, Class B.

This equipment also has been tested and complies with susceptibility standards to electrostatic discharge (ESD), radiated electromagnetic fields, fast transient bursts, and conducted surge interference. The procedures and criteria for conformance are contained in IEC801-2, 801-3, 801-4, 801-5, and EN60601-1-2: 1993.

In addition, this equipment has been tested and complies with EN6000-3-2 (1st Edition) covering harmonic disturbances in supply systems caused by household appliances and similar electrical equipment, Part 2, and EN 6000-3-3 (1st Edition) covering voltage fluctuation disturbances in supply systems caused by household appliances and similar electrical equipment Part 3.

INDICATIONS FOR USE: For examination of the tissues of the vagina, cervix, and external genitalia, to investigate, by means of magnification, abnormal cervical cytology or suspicious lesions of the lower female genital tract. Also used for corresponding biopsy and treatment, when indicated.

NOTE: This device complies with the guidelines of the American Conference of Governmental Industrial Hygienists threshold limits values for ultraviolet radiation for exposure time and distance consistent with the intended use of this device.

Conventions

WARNING: Warnings alert the user to the possibility of serious injury, death, or other adverse reaction associated with the use or misuse of the device.

CAUTION: Cautions indicate a potentially hazardous situation that, if not avoided, may result in minor or moderate injury and/or damage to the equipment. They also alert against unsafe practices.

NOTE: Notes provide supplemental information to the text and indicate a potentially hazardous situation, which, if not avoided, may result in property damage. They also highlight important information on the use of this equipment.

General Precautions

- For safety, the video colposcope should only be coupled to a grounded 110-120 VAC hospital-grade outlet (220 - 240 volt, 50 cycle international).
- The video colposcope should not be operated in the presence of flammable or explosive gases or chemicals, or installed in areas where these materials are commonly used.
- To avoid overheating, the video colposcope should not be positioned closer than 6" to any wall.
- Keep all liquids away from electrical equipment to avoid the possibility of shock and instrument damage.
- Occasionally inspect the power cord for signs of cuts, abrasions or dents.
- The video colposcope should never be stored or operated in areas where it could get wet or could be exposed to any environmental conditions like extreme temperature or humidity, direct sunlight, dust, etc.
- The lamp is extremely bright. DO NOT stare directly into illumination lens when the lamp is lit.
- All service to the video colposcope must be performed by Welch Allyn or by an authorized repair center.
- Video colposcope user should adhere to the operating conditions found in this manual. Otherwise, instrument damage may occur and/or operator/patient safety may be compromised.
- There are no user servicable parts (other than the lamp and fuses) in this unit or in its accessories. Any attempt to disassemble and/or repair this unit will result in voiding of the warranty.

Symbols

On: Power: Connects to the low voltage supply.

Off: Power: Disconnects from the low voltage supply.

Caution: Consult user's manual for additional information.

Warning: Consult user's manual for additional information.

Warning: High temperatures

Type B Equipment

Risk of fire. Replace fuses as marked.

Warning: High-intensity light

Warning: Power supply of unit is energized whenever power cord is plugged in.

The CE mark on this product indicates it has been tested to and conforms with the provisions noted in the 93/42/EEC Medical Device Directive.

Authorized European Representative:
European Regulatory Manager
Welch Allyn, Ltd.
Kells Road, Navan,
County Meath, Republic of Ireland
Tel 353 46 79060
Fax 353 46 27128

Summary of Warnings and Cautions

WARNING: All video monitors and peripheral equipment used with this instrument must be IEC 601-1 (or UL 2601) listed to comply with medical electrical safety guidelines. If not IEC 601-1(UL-2601) listed, the monitor/peripheral equipment should be positioned outside of the 6-foot patient contact area. Please contact Welch Allyn for more information.

WARNING: Total system risk current should not exceed 50 μ A. An isolation transformer is required if the total system risk current exceeds 50 μ A when accessories are interconnected.

CAUTION: The video colposcope is cooled via a vent fan located in the back of the unit. The fan draws air from the front of and beneath the video colposcope and exhausts the air out the back of the video colposcope. Verify that the unit is no less than 6" from a wall.

WARNING: DO NOT use a converter adapter that will convert the three-prong AC plug to a two-prong line plug. The power supply in the video colposcope will not be properly grounded and electric shock might result.

CAUTION: DO NOT clean illumination lens with alcohol.
DO NOT touch optical or illumination lenses except as described in Maintenance section of this manual.

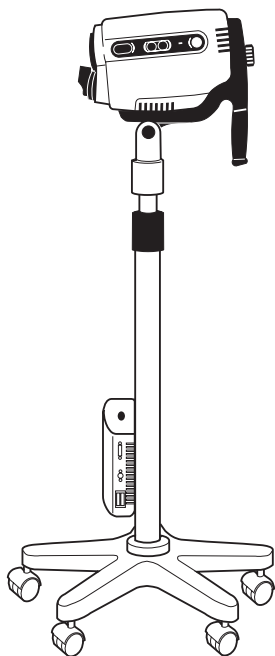
WARNING: The lamp operates at a high temperature. **DO NOT** attempt to remove the lamp before allowing it to cool. Allow at least five minutes for the lamp to cool before replacing. Replace with Welch Allyn lamp #09800 only.

CAUTION: The colposcope can be damaged if the unit is transported while holding the handle. The unit should be transported by grasping the pole.

Components


Video Colposcope

88002 / 88004 / 88006

Video colposcope and vertical stand with vertical stand hardware kit:

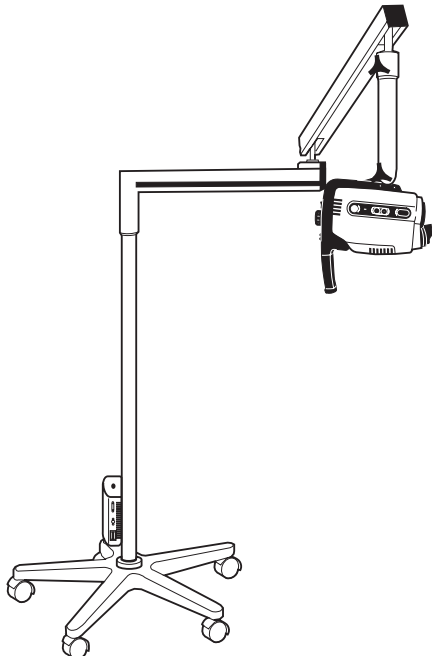
- Allen wrench
- Allen bolt
- Mounting washer
- Spacer

89002 / 89004 / 89006


Video colposcope and swing arm stand with swing arm hardware kit:

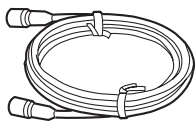- Allen wrench
- Bolt

88600

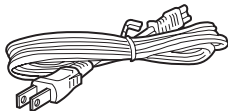
S-Video Cable

761076-0


Power Cord

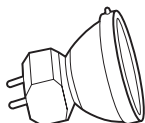Optional Accessories

09800

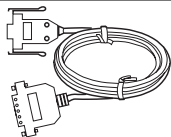Replacement Lamp


88500

RS-232 Interface Cable


488307-9

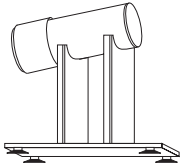Replacement Fuse


88010

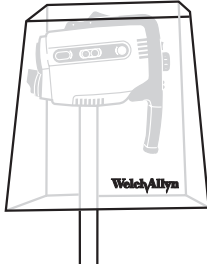Cervical Model

88040

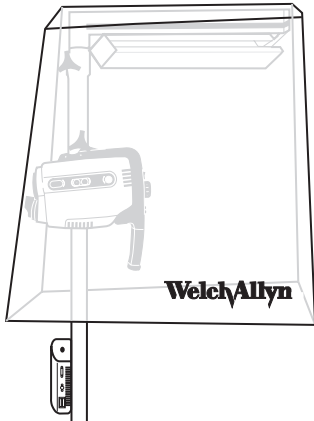Dust Cover for Vertical Stand


89040

Dust Cover for Swing Arm

88030

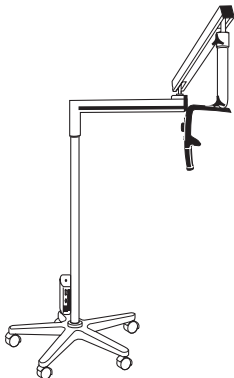Vertical Stand with Base Only


89030

Swing Arm Stand with Base Only

Note: Only accessories and components indicated in this manual are to be used with the Welch Allyn Video Colposcope system.


Nomenclature

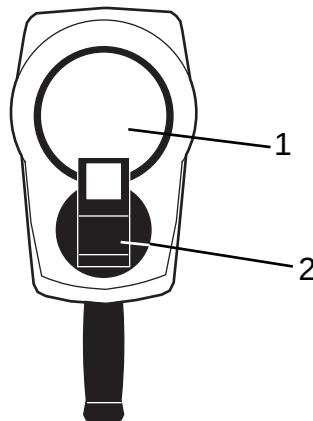Video Colposcope Front View


Name	Function
1. Objective lens	The lens that establishes the magnification and field of view.
2. Illumination beam director	Directs lamp beam.

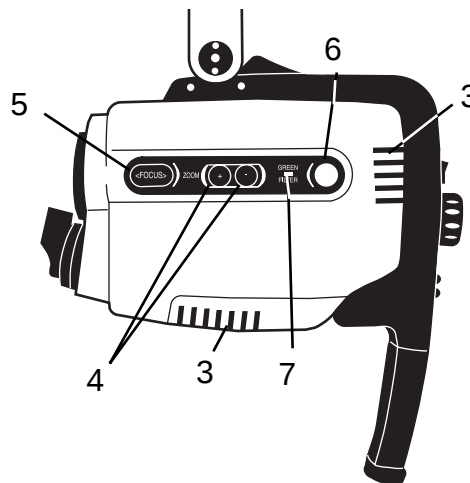Video Colposcope Side View


Name	Function
3. Ventilation slots	Allow air to enter and leave unit, cooling internal components.
4. Zoom control button	Pressing + or - increases or decreases magnification accordingly.
5. Focus control button	Pressing < or > adjusts the focus.
6. Green filter button	Pressing turns the electronic green filter on or off.
7. LED indicator	Illuminates if the electronic green filter is on. Flashes during activation or deactivation.

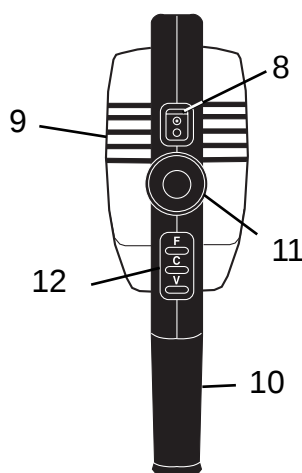Video Colposcope Back View


Name	Function
8. Power switch	Power control for the video colposcope.
9. Ventilation slots	Allow air to enter and leave unit, cooling internal components.
10. Video colposcope handle	For positioning the colposcope in the proper up/down, left/right position.
11. Attachment knob	Fastens handle to video colposcope.
12. Remote video function buttons	Allow use of remote video printer functions.

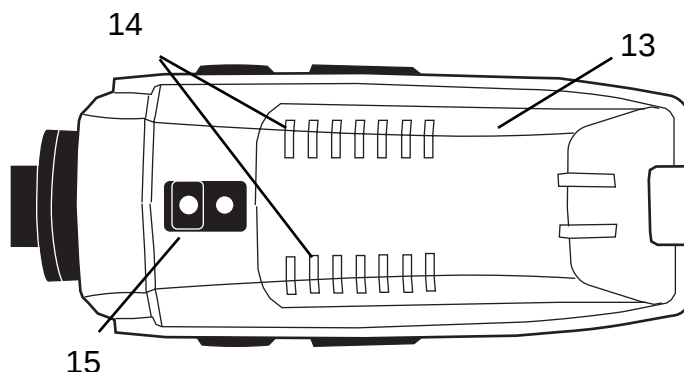Video Colposcope Bottom View

Name	Function
13. Lamp access door	Removes for lamp replacement. (Note: If door is not properly closed, interlock power switches will not activate and the lamp will not start.)
14. Ventilation slots	Allow air to enter and leave unit, cooling internal components.
15. Mounting piece	Allows the video colposcope to attach to the stand. On the swing arm model, the mounting piece is located at the top of the colposcope. On the vertical stand model, the mounting piece is located at the bottom of the colposcope.

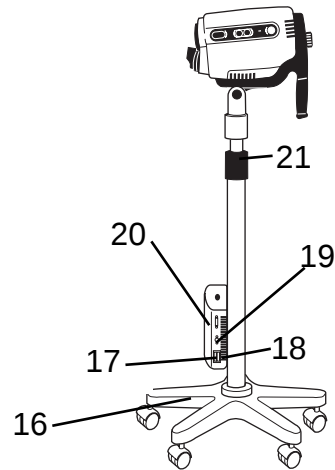Vertical Pole Stand


Name

- 16. Vertical rolling base stand
- 17. Fuse drawer
- 18. Power supply cord receptacle
- 19. S-Video output
- 20. RS232 interface cable output
- 21. Height adjustment ring

Function

- Allows mobility with 5-caster base, including two locking casters.
- Opens for replacement of fuses.
- Couples with power cord, which should be plugged into a 110-120 VAC hospital-grade outlet (220-240 volt, 50 cycle international).
- Outputs Y/C (S-Video).
- Couples with RS232 interface cable.
- Allows vertical movement and locking of stand at desired height; vertical height adjustment 36"-46" (91.4cm to 116.8cm).

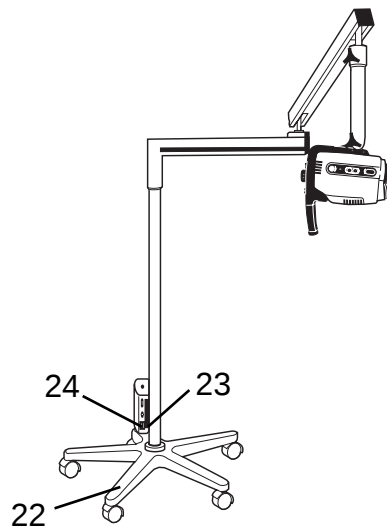Swing Arm Stand

Name

- 22. Swing arm rolling base stand
- 23. Power supply cord receptacle
- 24. Fuse drawer

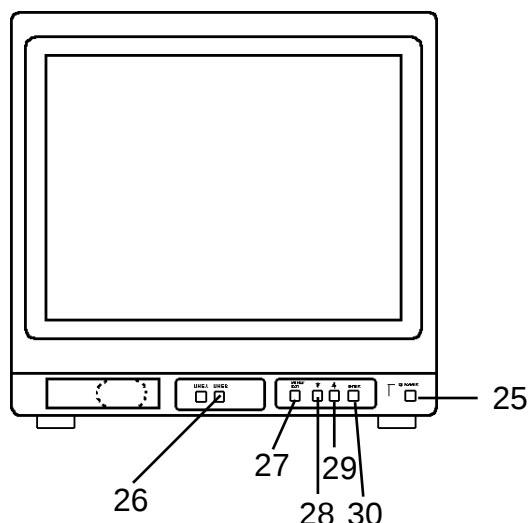
Function

- Allows overhead positioning of the instrument with 5-caster weighted base, including two locking casters; vertical height 29.5" to 49.5" (74.9cm to 125.7cm).
- Couples with power cord, which should be plugged into a 110-120 VAC hospital-grade outlet (220-240 volt, 50 cycle international).
- Opens for replacement of fuses.


Monitor

NOTE: The following is a brief overview of the nomenclature of the Sony PVM-14N1U Video Monitor. For detailed instructions, please refer to the operation manual that is provided by the manufacturer.

Front Panel of Monitor

Back Panel of Monitor

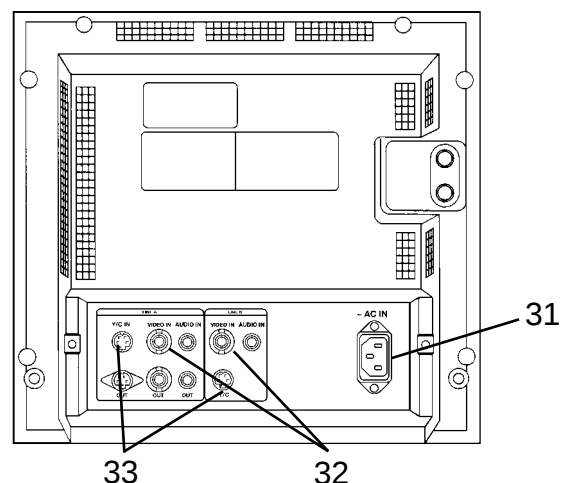
PLEASE REFERENCE YOUR SONY MONITOR MANUAL FOR FURTHER INFORMATION.

Name	Function
25. Power Switch and Indicator	Press to turn the monitor on. The indicator lights in green. To turn the power off, press again.
26. Line A/Line B (input select) Buttons	Press Line A if the signal is fed through a Line A connector on the back of the monitor. Press Line B if the signal is fed through a Line B connector on the back of the monitor.
27. Menu/Exit	Press to make the on-screen menu appear. Press again to leave the on-screen menu. The following parameters can be modified using the on-screen menu system: Volume - No function. Contrast - Adjust the contrast of the screen using the arrow buttons. Brightness - Adjust the brightness of the screen using the arrow buttons. Chroma - Adjust the color intensity of the video signal using the arrow buttons. Phase - Adjust the phase of the video signals using the arrow buttons. Color Select - Select the color system of the current input signals. Language - Select the menu language. User memory - Select options for storing and recalling menu settings.
28. Down Arrow	In item selection menus, moves the cursor downward. In adjustment and setting menus, decreases the value of the selected item or moves downward through item selections.
29. Up Arrow	In item selection menus, moves the cursor upward. In adjustment and setting menus, increases the value of the selected item or moves upward through item selections.
30. Enter	Chooses the selected item. Is active only during the first level of on-screen selections.
31. Power supply	Couples with power cord, which should be plugged into a 110-120 VAC hospital grade cord receptacle outlet (220-240 volt, 50 cycles for international).
32. Line A/Line B Video In	Input connectors for composite video.
33. Line A/Line B Y/C In	Input connectors for high resolution S-Video


NOTE: This monitor is shipped with default picture settings. Any change of the on-screen controls may result in poor picture quality.

Connection Diagrams

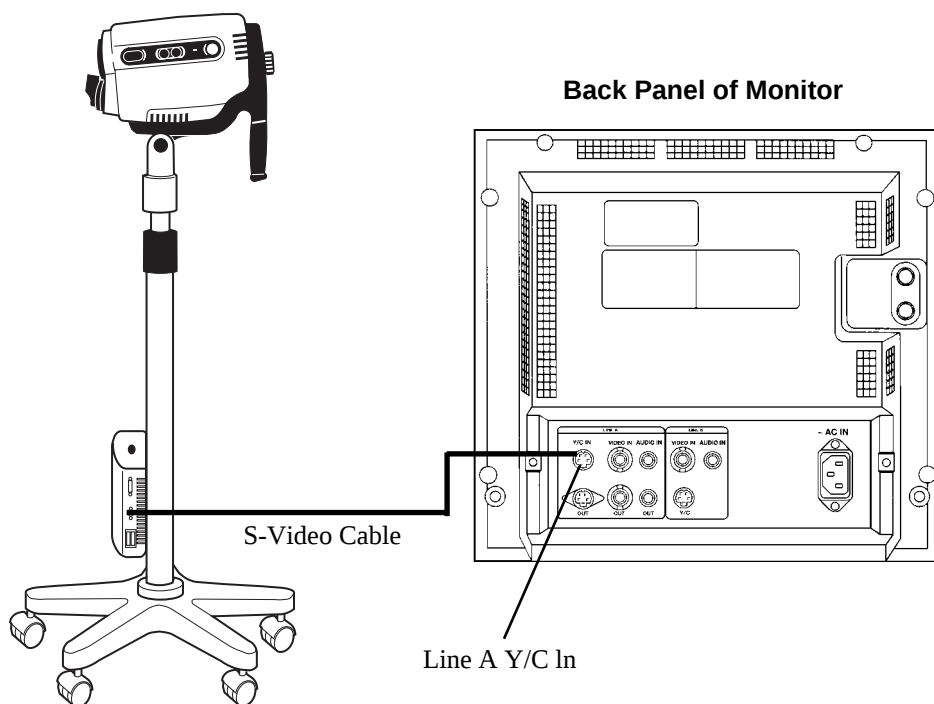Connecting Video Colposcope with Monitor

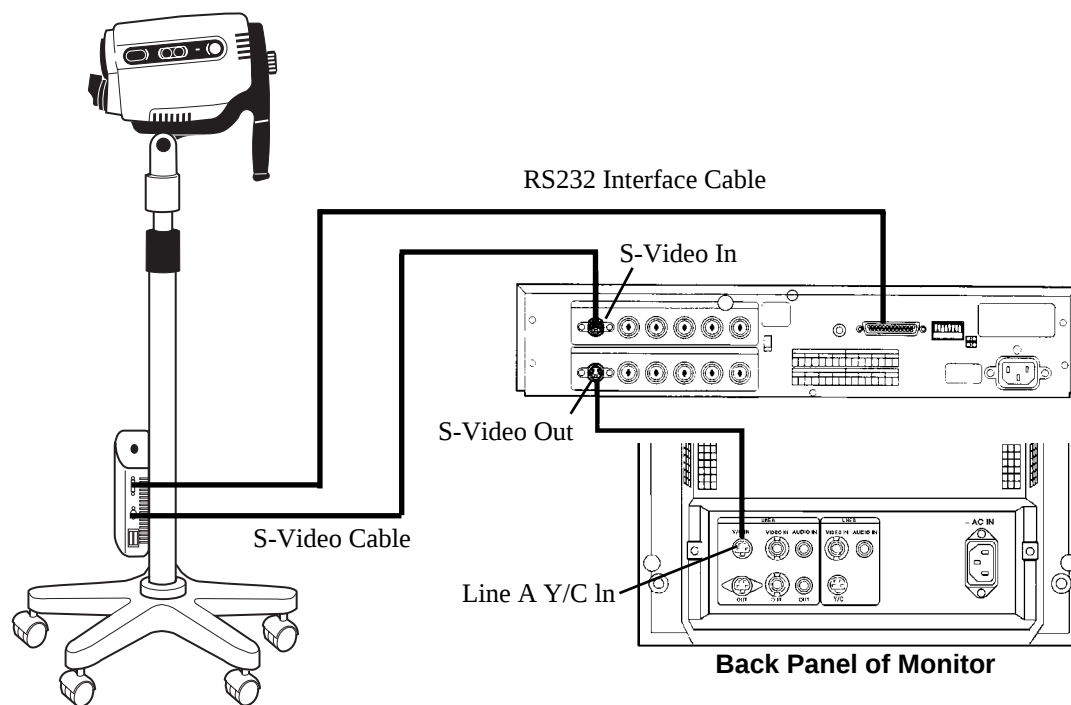Connecting Video Colposcope with VCR/Video Printer

WARNING: All video monitors and peripheral equipment used with this instrument must be IEC 601-1 (or UL 2601) listed to comply with medical electrical safety guidelines. If not IEC 601-1(UL-2601) listed, the monitor/peripheral equipment should be positioned outside of the 6-foot patient contact area. Please contact Welch Allyn for more information.

Preparation For Use

General Precautions

Make sure the unit is always grounded and secure during use. Do not disable power cord ground connection. Grounding reliability is achieved only when power cord is connected to a hospital-grade receptacle. Inspect the electrical plug and cord routinely. Do not use if damaged. Do not use the video colposcope in the presence of any flammable anesthetics.

Do not open the video colposcope housing. An electrical shock hazard exists due to high voltage. There are no user serviceable parts inside the video colposcope, except the lamp.

NOTE: Opening of the video colposcope housing by an unauthorized repair facility will void the product warranty.

WARNING: Total system risk current should not exceed 50 μ A. An isolation transformer is required if the total system risk current exceeds 50 μ A when accessories are interconnected.

CAUTION: The video colposcope is cooled via a vent fan located in the back of the unit. The fan draws air from beneath the video colposcope and exhausts the air out the back of the video colposcope. Verify that the unit is no less than 6" from a wall.

WARNING: DO NOT use a converter adapter that will convert the three-prong AC plug to a two-prong line plug. The power supply in the video colposcope will not be properly grounded and electric shock might result.

CAUTION: The colposcope can be damaged if the unit is transported while holding the handle. The unit should be transported by grasping the pole.

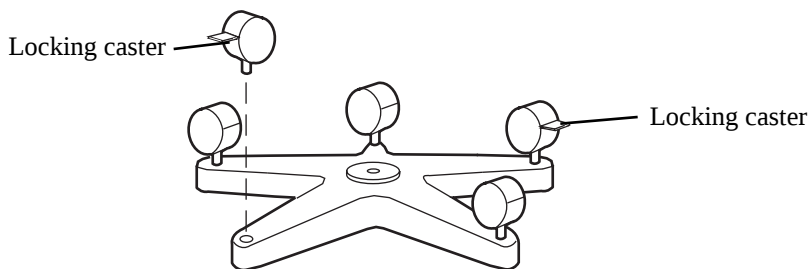
Assembly of Video Colposcope

Before initial set up of the video colposcope, check all components received against the parts list of components (see Components section of this manual) to verify a complete set. If parts are missing, please notify Welch Allyn. Review the Nomenclature, Preparation for Use, Operation, and Maintenance sections to become familiar with the equipment.

Vertical Pole Stand Assembly



The video colposcope stand is shipped unassembled. Minimal assembly is required.

1. To assemble the stand, remove the stand and base parts from their cartons.
2. Place base upside down on the floor and insert casters into holes on the bottom of the base. (Do not place two locking casters next to each other.)

3. Place the base on the floor with casters down. Lock the two locking casters.

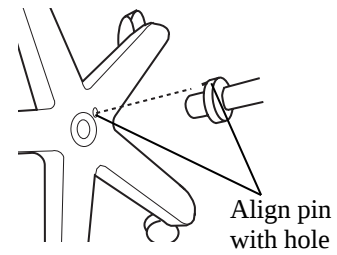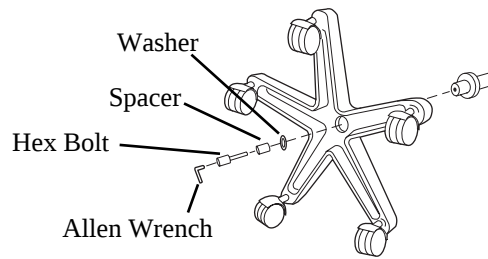- Place pole into the base, aligning the pin on the flange with the pin hole in the base. Place bolt, spacer, and washer, oriented as shown, into the pole. Tighten securely with the enclosed Allen wrench. (The assembly may need to be tilted.)


- Before using the video colposcope, refer to “Setting Up Video Colposcope” section of this manual.

Swing Arm Stand Assembly


The video colposcope stand is shipped unassembled. Minimal assembly is required.

NOTE: BASE WEIGHS 58 LBS. YOU MAY REQUIRE ASSISTANCE TO LIFT IT.

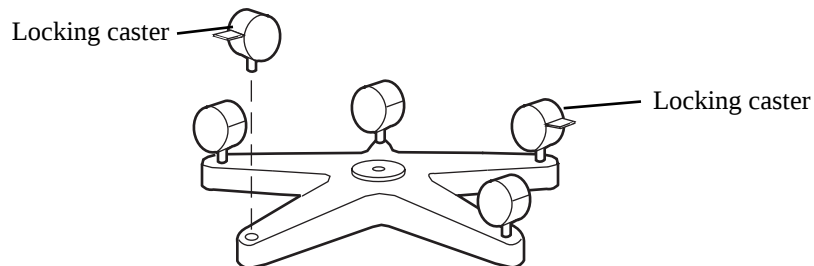
- To assemble the stand, remove the stand and base parts from the cartons.
- Place base upside down on the floor and insert casters into holes on the bottom of the base. (Do not place two locking casters next to each other.)

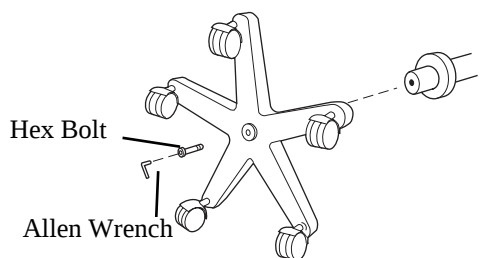
- Place the base on the floor with casters down. Lock the two locking casters.
- Place the vertical pole section of the assembly into the base, aligning the pin on the flange with the pin hole in the base.
- Locate the packaged bolt.
- Place the bolt through the bottom of the pole. (The assembly may need to be tilted.)

Align pin with hole

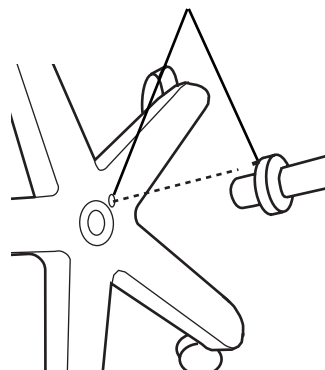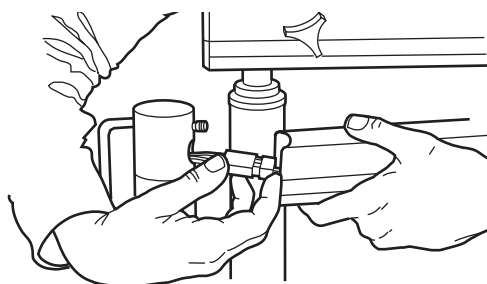
- Tighten securely with the enclosed Allen wrench.
- Remove the bolt from the arm of the colposcope assembly and insert it through the hole near the top of the pole.


- Position the pole and the arm of the remaining colposcope assembly in close proximity.
- Connect the two wire harnesses via their connectors. (This may require assistance for proper assembly.)

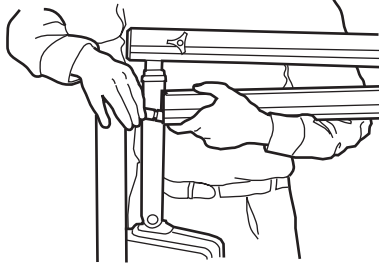11. Bring the colposcope swing arm assembly to the pole while at the same time pushing the electrical connectors into the colposcope arm.

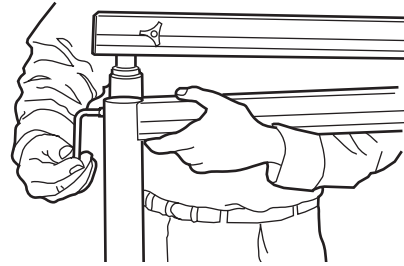
12. Tighten the bolt securely with the enclosed Allen wrench. Be sure not to pinch any of the electrical wires between the vertical pole and arm of the colposcope assembly.

13. Before using the video colposcope, refer to Setting Up Video Colposcope section of this manual.

Setting Up Video Colposcope

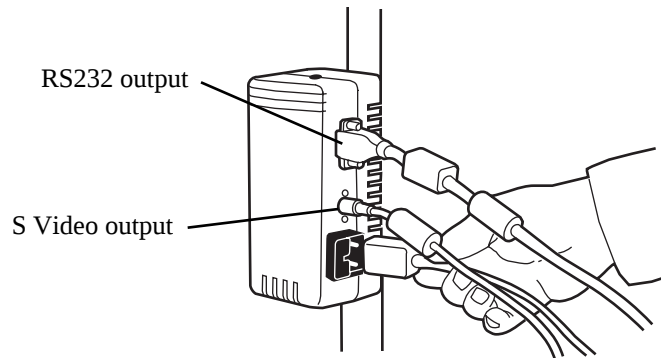1. Connect the power cord to the power cord receptacle in the power supply located on the video colposcope stand.
2. Plug the other end of the power cord into a properly grounded 110-120 volt AC outlet (220-240 volt, 50 cycle for international).
3. Connect the S-Video cable to the S-Video output on the side of the power supply located on the video colposcope stand.
4. Connect the other end of the S-Video cable into the Line C connector Y/C (S-Video) on the back of the monitor.
5. Plug the monitor into a properly grounded 110-120 volt AC outlet (220-240 volt, 50 cycle for international).

Operation

Power Switch

With the power cord connected to a properly grounded outlet, and the S-video cable connected to the video colposcope and monitor, activate the power switch on the handle of the video colposcope.

Lamp Ignition

Once the video colposcope's power switch has been activated, the lamp will ignite. The lamp requires approximately 11 seconds to warm up.

Focus and Zoom Controls

Once the power switch has been activated, the video colposcope will execute a setup procedure, ending in the low magnification setting, or 4.5x. To obtain coarse focus, position the distal end of the video colposcope approximately 300mm from the target. Adjust the video colposcope by moving the stand (for vertical stand), or the arm (for the swing arm stand), until the picture is in focus. Press the + on one of the zoom controls until maximum magnification is obtained, and readjust the position of the stand or the swing arm until the picture is in focus. Fine focus, in either direction, is provided by the focus controls once coarse focus has been achieved. After coarse focus is achieved at maximum magnification, zoom control can be set to a desired lower magnification by pressing either zoom control. Focus will be maintained throughout the entire magnification range once these steps have been completed if the video colposcope or target is not moved.

Note: The setup (nominal) setting of the focus controls can be recovered by pressing one of the zoom + controls for 4 seconds after maximum magnification has been achieved.

Green Filter Controls

Activate the electronic green filter by pressing the green button located on either side of the video colposcope. Full activation of the green filter occurs in less than 10 seconds. Deactivated the filter by pressing the green filter button again. The green filter LED indicator flashes during activation and deactivation and remains on after activation is complete.

Illumination Beam Director

The illumination beam director can be rotated up to 45° clockwise or counterclockwise to better illuminate the examination area. To rotate the director, grasp the flat sides and twist in either direction as desired. The illumination beam director moves with some resistance. This helps to hold it securely in the desired position. The center position can be recovered by returning the director to the detent position.

Vertical Height Adjustment Ring (Vertical Colposcope Model Only)

The height of the vertical stand can be adjusted by rotating the black height adjustment ring, located on the pole of the stand, counterclockwise. Once the ring has been loosened, adjust the pole to the desired height and then tighten the ring again by rotating it clockwise.

Swing Arm Height Adjustment (Swing Arm Colposcope Model Only)

Adjust the height of the swing arm stand by loosening the height adjustment knob located on the arm of the stand. The swing arm can then be moved to the desired position. After moving the arm, tighten knob securely by rotating clockwise. The desired drag can be obtained by adjusting this knob either clockwise or counterclockwise.

Positioning Video Colposcope

The video colposcope can be angled up and down and left and right. To change the up/down tilt angle, loosen the knob located on the tilt axis. The video colposcope can then be tilted as necessary. To change the horizontal angle, loosen the knob located on the support shaft. The drag of either knob can be set by loosening or tightening as necessary.

Remote Video Functions

The handle of the video colposcope contains three buttons that control three video functions. These functions, provided by a remote video printer, are Freeze, Print, and Toggle. (See Connection Diagram section of this manual for connecting the printer to the video colposcope.)

1. Pressing the freeze ("F") button stores a new image in printer memory. The frozen image is displayed on the monitor.
2. Pressing the copy ("C") button prints the image displayed on the monitor. If live video is displayed, the image visible when the button was pressed will be printed.
3. Pressing the video ("V") button changes the display from printer memory (frozen image) to live video, or from live video to printer memory (frozen image), depending on the current mode.

Maintenance

Video Colposcope Cleaning

Turn the power switch off and unplug the power cord from the electrical outlet prior to cleaning.

The video colposcope housing can be wiped down with a cloth dampened slightly with a mild solution of disinfectant. Be careful not to allow the plug prongs to get wet. Also be careful not to allow the camera lens or lamp lens to get wet.

Disinfecting Solutions

The disinfecting solutions listed below are safe for cleaning the housing of the video colposcope if used according to the manufacturer's instructions for cleaning and disinfecting, and in accordance with procedures detailed in the cleaning section of this manual.

- Cidex, Cidex Plus
- 70% isopropyl alcohol
- 10% mild bleach solution
- Metricide
- 10% Wescodyne
- Banicide
- Wavecide-01

Note: References to brand names are not endorsements of their efficacy as disinfecting solutions. However, tests have shown these solutions to be compatible with Welch Allyn colposcopes, providing the manufacturers' directions are followed. Do not use any other solutions unless a sample has been sent to Welch Allyn for compatibility testing.

IMPORTANT

Be careful not to allow solution to drip into the air vents.

DO NOT spray solution into the air vents.

DO NOT sterilize any part of the unit.

DO NOT IMMERSE any part of the unit in cleaning solutions.

Illumination Lens Cleaning

The lamp lens may be cleaned with a cloth dampened slightly with warm water and mild detergent.

CAUTION: DO NOT clean illumination lens with alcohol. **DO NOT** touch optical or illumination lenses except as described in Maintenance section of this manual.

Camera Lens Cleaning

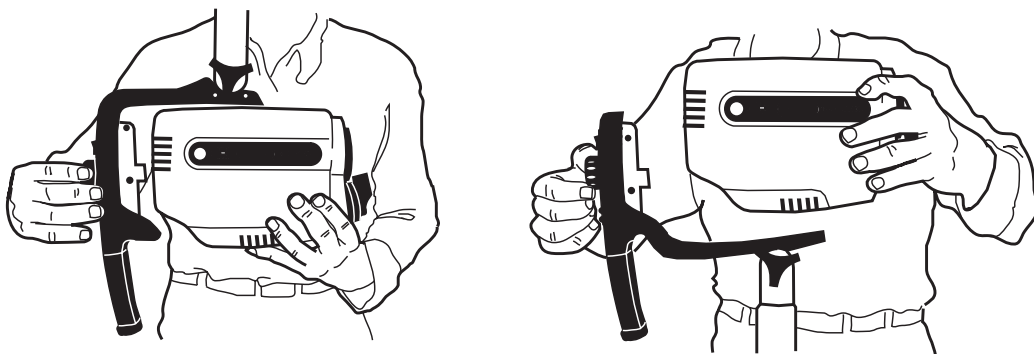
The camera lens may be cleaned with isopropyl alcohol or any commercial lens cleaner.

Replacing Lamp


WARNING: The lamp operates at a high temperature. DO NOT attempt to remove the lamp before allowing it to cool. Allow at least five minutes for the lamp to cool before replacing. Replace with Welch Allyn lamp #09800 only.


1. Turn the power switch off and unplug the power cord from the electrical outlet.
2. Holding the video colposcope as shown, turn the knob located at the back of the handle counterclockwise and unscrew and remove the video colposcope from the handle.

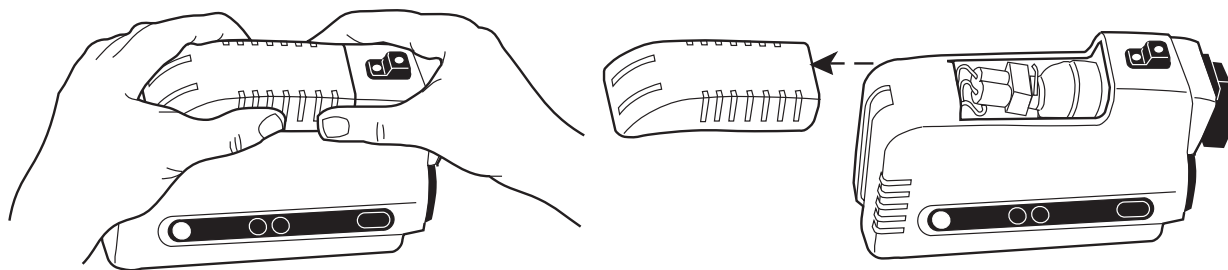
3. Place the unit on a suitable work surface. Slide the lamp access door away from the housing as shown until the door is completely removed.

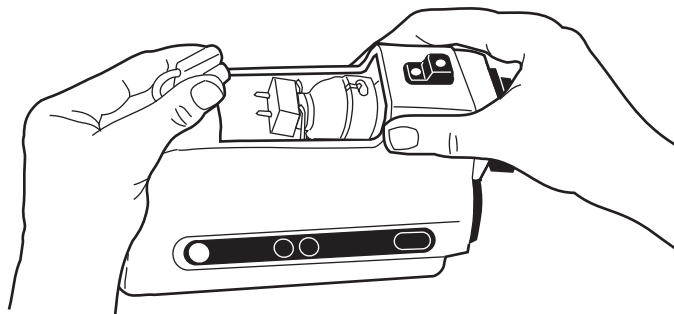4. Unplug the lamp from the electrical connector.

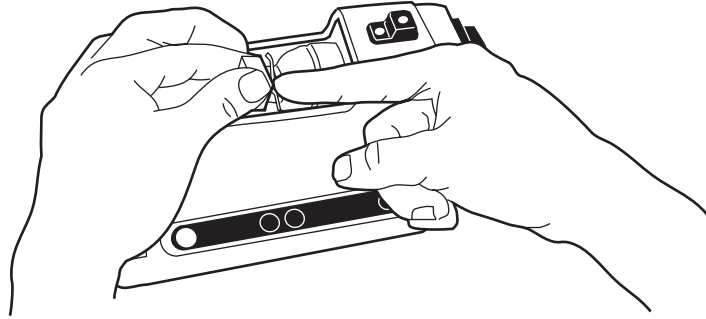5. Push the retainer spring toward the back of the colposcope and remove the lamp from its housing.


6. Remove a Welch Allyn replacement lamp # 09800 from its package. Do not touch the lamp itself or the interior reflective surface of the lamp. Skin oils will cause premature lamp failure. Hold lamp by the outside of the reflector or the connector only. Remove any grease or fingerprints with a clean cotton swab moistened with alcohol. Do not leave any lint on the lamp.

7. Holding the retainer spring as in step 5, slide the new lamp into the lamp housing so that the lamp's alignment pin engages the alignment slot in the lamp holder. (Make sure lamp is properly seated and snaps into place.)

8. Plug the electrical connector onto the new lamp.

9. Slide the lamp access door securely back into place, reversing the process described in step 3 above.

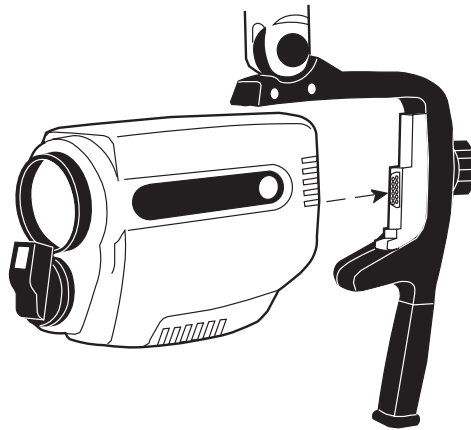10. Place the video colposcope back into the handle, making sure the power connector is engaged.


11. Tighten the knob by rotating clockwise until snug.

12. Plug the power cord back into the electrical outlet.

Replacing Fuses

Two fuses are located in slots adjacent to the power supply cord receptacle on the side of the power supply housing. This housing is attached to the pole of the video colposcope stand.

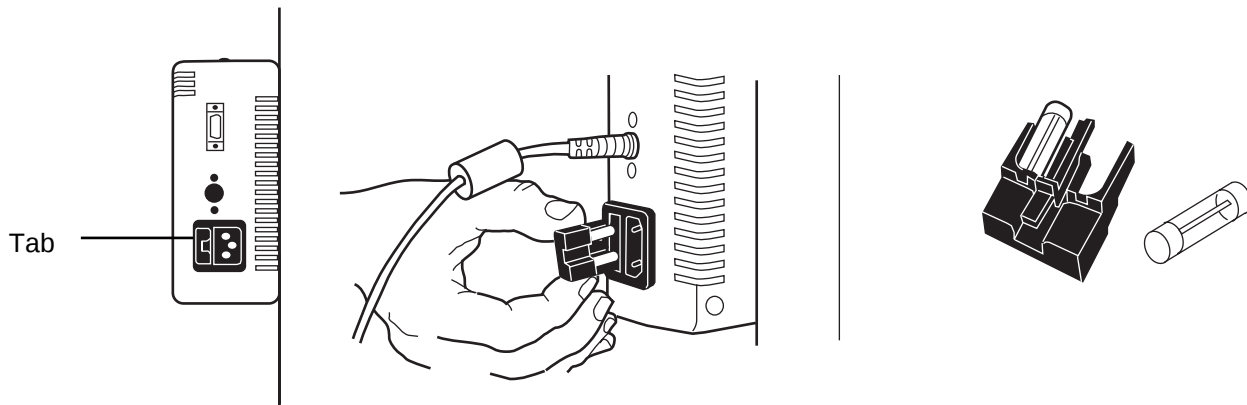1. To replace a blown fuse, remove the fuse holder by pressing the tab and pulling the holder out of the power supply
2. Pull out and remove the blown fuse from the fuse holder.

3. Replace with new fuses # T1.00L-250V 1A Time Lag/Low Breaking Capacity (Welch Allyn Part # 488307-9).
4. Reinsert the fuse holder by pressing until it snaps into place.

Service Information

Customers in North America should return instruments requiring service to the Welch Allyn Technical Service Department listed below, or to an authorized Welch Allyn distributor.

Technical Service Department
Welch Allyn, Inc.
4341 State Street Road
Skaneateles Falls, NY 13153-0220
U.S.A.
Telephone: 1-800-535-6663
Fax: 315-685-4653

Customers outside of North America should return their units to a local, authorized Welch Allyn distributor, or to their nearest Welch Allyn service center.

Welch Allyn, Inc., U.S.A. - 4341 State Street Road, Skaneateles Falls, NY 13153-0220, Tel: 800-535-6663, Fax: 315-685-4653
Welch Allyn, Ltd., Canada - 160 Matheson Blvd. E., Unit #3, Mississauga, Canada L4Z 1V4, Tel: 905-890-0004, Fax: 905-890-0008
Welch Allyn, Ltd., UK - Aston Abbots, Buckinghamshire, England HP22 4ND, Tel: 011-44-1296-682140, Fax: 011-44-1296-682104
Welch Allyn, GmbH, Germany - Zollerstrasse 2-4, D72417 Jungingen, Germany, Tel: 011-49-7477-9271-73, Fax: 011-49-7477-9271-93
Welch Allyn, Ltd., Singapore - 6001 Beach Road #21-09, Golden Mile Tower, Singapore 199589, Tel: 011-65-291-0882, Fax: 011-65-291-5780
Welch Allyn, Ltd., Australia - PO Box 864, Ground Floor, 18-20 Orion Road, Lane Cove, NSW 2066, Australia, Tel: 011-61-294-183-155, Fax: 011-61-294-183-650

The part and serial numbers are located on the housing of the colposcope near the fan. Remove the colposcope from the handle to locate these numbers.

Troubleshooting

Condition	Check	Action
Power does not come on.	Power cord	Check connections at power supply and wall outlet.
	Attachment of unit to handle	Check colposcope unit's alignment with handle.
No image on monitor.	Attachment of unit to handle and door to unit	Make sure lamp access door and handle are properly attached.
	Fuses	Remove fuse panel & replace blown fuse with T1.00L 250V 1A time lag (Welch Allyn part # 488307-9).
	Wall outlet	Plug the power cord into a wall outlet known to work.
	Monitor/other peripheral devices	Make sure power is on for all devices.
	Cable connections	Make sure all video cables are connected properly.
	Monitor input selection	Make sure correct input selection is made on the front panel.
Lamp will not light.	Lamp housing	Make sure lamp assembly is installed properly.
	Lamp	Replace lamp.
	Lamp access door	Make sure lamp access door is properly closed.
Printer does not respond properly to handle buttons.	Power cord and printer	Unplug power cord from wall outlet, then plug cord back in to outlet. At the same time, turn printer off and then on again.
	Printer	Make sure printer is on and is properly connected.

Specifications

Item	Specification	Technical Data
Power requirements	Voltage Frequency Current	115 VAC domestic 220 -240 VAC international 50/60 Hz 1.0 Amp
Dimensions	Vertical stand Swing arm stand Optical centerline to swing arm bottom Colposcope HxWxD (excluding handles)	Low: 36.0", high: 46.0" (91.4 cm" to 116.8cm) Low: 29.5", high: 49.5" (74.9cm to 125.7cm) 15.0" (38.1 cm) 8.25" x 5.75" x 3.37" (21.0cm x 14.6cm x 8.6cm)
Weight	Colposcope Vertical stand Swing arm stand	3.1 lbs (1.4 kg) 25 lbs (11.3 kg) 80 lbs (36.3 kg)
Focal length	300mm	
Magnification	4.5x - 25x (typ.)	
Field of view	66mm - 14mm	
Depth of field	112mm - 5mm	
Operating environment	Ambient temperature Relative humidity Atmospheric pressure	50° F (+10° C) to 104° F (+40° C) 95% 70kPa to 110kPa
Illumination	Lamp type Lamp life Brightness adjustment Lamp Lamp voltage	21 Watt metal halide arc 750 hrs @ 1 hr per start Automatic electronic shutter Welch Allyn part no. 09800 60 Volts

Item	Specification	Technical Data
Cooling method	Forced air via fan	
Color system	Color mosaic CCD	
Video output	Y/C (S-Video)	
Transport/Storage environment	Ambient temperature Relative humidity	-40° F (-40° C) to 122° F (+50° C) 95% max.
Output devices	VCR Monitor Thermal or video printer	
Compliance	Listed Listed Listed Certified Conformance Explosion proofing	UL2601-1 CAN/CSA C22.2 No.601.1 IEC601-1 IEC601-1-2 CE Use with potentially flammable surroundings is prohibited.
Fuses	T1.00L-250V (Welch Allyn part # 488307-9)	1 Amp time lag, low breaking capacity
Classification	Electro medical equipment Degree of protection against electrical shock	Class I equipment Type B
Patents	This product is covered by the following patents: 5,083,059; 5,117,154; 5,138,228; 5,144,201; 5,291,100; D391,360. Patents pending.	

The CE mark on this product indicates it has been tested to and conforms with the provisions noted in the 93/42/EEC Medical Device Directive.

Authorized European Representative:
European Regulatory Manager
Welch Allyn, Ltd.
Kells Road, Navan,
County Meath, Republic of Ireland
Tel 353 46 79060
Fax 353 46 27128

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