




User Manual



95248
(01/2010)



bioMérieux, Inc.
Box 15969
Durham, North Carolina 27704-0969 / USA
Tel. (1) 800-682-2666

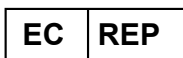

bioMérieux® SA
au capital de 12 029 370 €
673 620 399 RCS LYON
69280 Marcy l'Etoile / France
tél. 33 (0)4 78 87 20 00 / fax 33 (0)4 78 87 20 90
<http://www.biomerieux.com>

Algeria

bioMérieux Algérie EURL

Algérie Business Center
Les Pins Maritimes - Mohammadia
Alger
tel. (213) 21 89 14 81
fax (213) 21 89 14 82

Argentina

bioMérieux Argentina

Av. Congreso 1745
C1428BUE
Capital Federal Buenos Aires
tel. (54) 11 5555 6800
fax (54) 11 5555 6888

Australia

bioMérieux Australia P/L

Unit 25 - Parkview Business Centre
1, Maitland Place
Baulkham Hills NSW 2153
tel. (61) 2 8852 4700
fax (61) 2 8852 4777

Austria

bioMérieux Austria GmbH

Eduard-Kittenberger-Gasse 97
Top 3
A-1230 Wien
tel. (43) 186 50 650
fax (43) 186 50 661

Belgium

bioMérieux Benelux s.a./n.v.

Media Square
18-19 Place des Carabiniers
Bruxelles 1030
tel. (32) 2 743 01 70
fax (32) 2 733 55 97

Brazil

bioMérieux Brasil SA

Estrada Do Mapuá
491 Taquara - Jacarepaguá
CEP 22710 261
Rio de Janeiro RJ
tel. (55) 21 2444 1400
fax (55) 21 2445 6025

Canada

bioMérieux Canada, Inc.

7815, Henri-Bourassa West
Saint Laurent, QC
H4S 1P7
tel. (1) 514 336 7321
fax (1) 514 807 0015

Chile

bioMérieux Chile S.A.

Seminario 131
Providencia
Santiago
tel. (56) 2634 20 92
fax (56) 2634 20 93

China

bioMérieux China Limited

Room 1601-02B & 10
Est Ocean Centre
n° 24A Jiang Guo Men Nei Street
100004 Beijing
tel. (86) 10 6515 6963
fax (86) 10 6515 6993

bioMérieux China Limited

Room 2605, South Tower,
World Trade Center
371-375 Huan Shi Dong East Road
510095 Guangzhou
tel. (86) 20 8762 7010
fax (86) 20 8762 7015

Colombia

bioMérieux Colombia Ltda

Avenida 15 No. 100-43
Piso 2
Bogotá, D.C.
tel. (57) 1 520 0080
fax (57) 1 520 0088
(57) 1 520 0831

Czech Republic

bioMérieux CZ s.r.o.

Business Park Kosice
Jinonická 80
158 00 Praha 5
tel. (420) 2 57 290 623
(420) 2 57 290 232
fax (420) 2 57 290 964

Denmark

bioMérieux Danmark Aps

Smedeholm 13C
2730 Herlev
tel. (45) 70 10 84 00
fax (45) 70 10 84 01

Finland

bioMérieux Suomi Oy

Konulantie 47 C
FI-00390 Helsinki
tel. (358) 9 8545 6000
fax (358) 9 8545 6045

France

bioMérieux SA

69280 Marcy l'Etoile
tel. (33) (0)4 78 87 20 00
fax (33) (0)4 78 87 20 90
<http://www.biomerieux.com>

Germany

bioMérieux Deutschland GmbH

Weberstrasse 8
D 72622 Nürtingen
tel. (49) 7022 30070
fax (49) 7022 36110

Greece

bioMérieux Hellas S.A.

Papanikoli 70
15232 Halandri
Athens
tel. (30) 2 10 81 72 400
fax (30) 2 10 68 00 880

Hungary

bioMérieux Hungária Kft.

Fóto út. 56 (5. emelet)
H-1047 Budapest
tel. (36) 1 231 3050
fax (36) 1 231 3059

India

bioMérieux India Pvt. Ltd

A-32, Mohan Co-Operative Ind. Estate
New Delhi 110 024
tel. (91) 11 42 09 88 00
fax (91) 11 24 64 88 30

Indonesia

Representation Office**bioMérieux Indonesia**

Enseval Building
Kawasan Industri Pulo Gadung -
Jl. Pulo - Lentut No. 10
Jakarta Timur 13920
tel. (62) 21 461 51 11
fax (62) 21 460 41 07

Italy

bioMérieux Italia S.p.A.

Via Fiume Bianco, 56
00144 Roma
tel. (39) 06 523 081
fax (39) 06 523 08240

Ivory Coast

bioMérieux Afrique Occidentale

08 BP 2634
Avenue Joseph Blohorn
Abidjan 08
tel. (225) 22 40 93 93/22 40 41 40
fax (225) 22 40 93 94

Japan

Sysmex bioMérieux, Ltd.

Osaki Central Tower 8F
1-2-2 Osaki Shinagawa-ku
Tokyo 141-0032
tel. (81) 3 6834 2666
fax (81) 3 6834 2667

Korea

bioMérieux Korea Co., Ltd.

1st & 2nd Floor, Yoosung Building
830-67 Yeoksam-dong,
Kangnam-gu
Séoul 135-080
tel. (82) 2 2188 4700
fax (82) 2 547 6263

Mexico

bioMérieux México SA de CV

Chihuahua 88, col. Progreso
México 01080, D.F.
tel. (52) 55 5481 9550
fax (52) 55 5616 2245

Netherlands (The)

bioMérieux Benelux BV

Boseind 15
P.O. Box 23
5280 AA Bostel
tel. (31) 411 65 48 88
fax (31) 411 65 48 73

New Zealand

bioMérieux New Zealand Ltd.
C/- Logical Freight Solutions
12C Rennie Drive, Airport Oaks
Auckland
tel. (64) 9 918 6354
fax (64) 9 918 6355

Norway

bioMérieux Norge AS
Økernveien 145
N-0513, Oslo
tel. (47) 23 37 55 50
fax (47) 23 37 55 51

Philippines (The)

Representation Office
bioMérieux Philippines
11th Floor, Pearlbank Centre
146 Valero Street, Salcedo Village
1227 Makati City
tel. (632) 817 7741
fax (632) 812 0896

Poland

bioMérieux Polska Sp. Z.o.o.
Ul. Zeromskiego 17
01-882 Warsaw
tel. (48) 22 569 85 00
fax (48) 22 569 85 54

Portugal

bioMérieux Portugal, Lda.
Av. 25 de Abril de 1974, nº 23-3º
2795-197 LINDA-A-VELHA
tel. (351) 21 415 23 50
fax (351) 21 418 32 67

Russia

o.o.o. bioMérieux
Derbenevskaya ul. 20, str. 11
115 114 Moscow
tel. (7) 495 221 10 79
fax (7) 495 221 10 79

Singapore

bioMérieux Singaporete. Ltd.
11 Biopolis Way, Helios, Block 11
#10-03 Singapore 138667
tel. (65) 6513 9554
fax (65) 6478 9501

South Africa

bioMérieux South Africa Pty
7 Malibongwe Drive
Randburg 2125
tel. (27) 11 801 91 10
fax (27) 11 791 24 19

Spain

bioMérieux España S.A.
Manual Tovar, 45-47
28034 Madrid
tel. (34) 91 358 11 42
fax (34) 91 358 06 29

Sweden

bioMérieux Sverige AB
Hantverkarsvägen 15
436 33 Askim
tel. (46) 31 68 84 90
fax (46) 31 68 48 48

Switzerland

bioMérieux Suisse s.a.
51, avenue Blanc
Case postale 2150
1211 Genève 2
tel. (41) 22 906 57 60
fax (41) 22 906 57 42

Taiwan

Representation Office
bioMérieux China Limited
Taiwan Branch
RM 608, No. 6-3 Ching Cheng Street
Taipei 105
tel. (886) 2 2545 2250
fax (886) 2 2545 0959

Thailand

bioMérieux Thailand Ltd
3195/9 Vibulthani Tower, 4th Floor
Rama IV Road, Klongton, Klongtoey
Bangkok 10110
tel. (66) 2 661 56 44
fax (66) 2 661 56 45

Turkey

bioMérieux Diagnostik A.S.
Değirmen Sok. Nida Plaza Kat:6
34742 Kozyatağı-Istanbul
tel. (90) 216 444 00 83
fax (90) 216 373 16 63

United Kingdom

bioMérieux UK Ltd
Grafton Way, Basingstoke
Hampshire RG22 6HY
tel. (44) 1256 461881
fax (44) 1256 816863

USA

bioMérieux, Inc.
100 Rodolphe Street
Durham NC 27712
tel. (1) 919 620 2000

Vietnam

Representation Office
bioMérieux Vietnam
Room 4A, 4th Floor
Green House Building
62A Pham Ngoc Thach Street, Ward 6
District 3
Ho Chi Minh City
tel. (84) 88 209 906
fax (84) 88 209 905



This product and its documentation complies with the In Vitro Diagnostic Medical Device Directive 98/79/EC.

Liability Disclaimer

bioMérieux, Inc. makes no express or implied warranty regarding this manual, its quality, performance, or appropriate use regarding any type of specific procedure.

Furthermore, this manual may be modified by bioMérieux without notice and without implying any obligation or liability on the part of the company.

Intellectual Property

bioMérieux, the blue logo, BacT/ALERT, BacT/LINK and MB/BacT are used, pending, and/or registered trademarks of bioMérieux in the USA and other countries.

CLSI is a registered trademark of Clinical and Laboratory Standards Institute, Inc.

PSC and Quicksan are registered trademarks of PSC, Inc.

Zip is a registered trademark of Iomega Corporation.

© 2010 bioMérieux, Inc. All rights reserved.

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, or translated into any language (human or computer) in any form, or by any means whatsoever, without the prior express written permission of bioMérieux, Inc.

Warranty

bioMérieux, Inc., ("Seller") warrants the BacT/ALERT® instrument (the "Instrument") to the original purchaser for a period of one (1) year after date of installation against defects in material and workmanship and defects arising from failure to conform to specifications applicable on the date of installation. Seller further agrees to correct, either by repair, or, at its election, by replacement, any such defect found on examination to have occurred, under normal use and service, during such one-year period, provided Seller is promptly notified in writing upon discovery of such defect.

Seller shall not be liable under this Warranty for any defect arising from abuse of the system, failure to operate and maintain the system in accordance with the documentation included with the Instrument, including repair service, alteration or modification of the system by any person other than service personnel of bioMérieux, Inc., or use of modified, changed, or previously used disposables.

The warranty of Seller set forth above and the obligations and liabilities of Seller thereunder are exclusive and in lieu of all other remedies or warranties, express or implied, arising by law or otherwise, with respect to the system delivered hereunder (including without limitation any obligation of Seller with respect to merchantability, fitness for particular purpose and non-infringement). In no event shall Seller be liable for incidental or consequential damages, however arising and whether or not occasioned by Seller's negligence.

This Warranty shall not be extended or altered except by written instrument signed by Seller.

All of the product elements in the Seller's Instrument and the total Instrument are warranted to be new or equivalent to new for the full product warranty period of one year. Disposables and replacement items with a normal life expectancy of less than one (1) year, such as batteries and bulbs, are excluded from this warranty.

STANDARD SYMBOLS

The following table presents symbols that may appear in the instructions for use or on the instrument, package inserts, or packaging.



CE-Marking of Conformity



Consult Instructions for Use



Use by



Manufacturer



Date of manufacture



Contains sufficient for $\langle n \rangle$ tests



Keep dry



Fragile, handle with care



Caution, consult accompanying documents



Biological risks



Electric shock warning



Radiation warning



Potential pinch-point warning



Laser



Temperature limitation



Upper limit of temperature



Lower limit of temperature



In Vitro Diagnostic Medical Device



Batch code



Authorized Representative in the European Community



Catalog number



Serial Number



Do not reuse



Recyclable



Separate collection for waste electrical and electronic equipment

Standard Symbols



Very toxic



Corrosive



Sodium azide



Irritant



Positive control



Negative control



Keep away from sunlight



Protect from light



This way up




Do not stack



Humidity limitation



Fuse

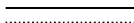

Direct current






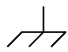







	Alternating current
	Both direct and alternating current
	Three-phase alternating current
	Earth (ground) terminal
	Protective conductor terminal
	Frame or chassis terminal
	Equipotentiality
	ON (supply)
	OFF (supply)
	ON (only for a component of the system equipment)
	OFF (only for a component of the system equipment)
	Equipment protected throughout by double insulation or reinforced insulation (Equivalent to Class II of IEC 536)
	Potential tip over/crush hazard

TABLE OF CONTENTS

Standard Symbols	vi
List of Figures	xiii
List of Tables	xvii
HOW TO USE THIS MANUAL	1-1
Intended Audience	1-2
<i>Specimen/Sample</i>	1-2
Purpose of the BacT/ALERT® 3D System	1-2
Additional Supplies	1-3
Purpose of This Manual	1-4
Manual Organization	1-4
Chapter Organization	1-6
Finding Topics	1-6
Typographic and Usage Conventions	1-7
<i>Name and Titles</i>	1-7
<i>Press</i>	1-7
<i>Procedural Steps</i>	1-7
<i>References</i>	1-7
<i>Select</i>	1-8
<i>User Input</i>	1-8
Warnings, Cautions, and Information	1-8
SYSTEM OVERVIEW	2-1
Introduction	2-2
<i>Hardware Configuration</i>	2-2
<i>Software Configuration Options</i>	2-2
<i>21 CFR Part 11 and HIPAA</i>	2-3
<i>Theory Of Operation</i>	2-4
Principle of Detection	2-4
<i>Electrical Warnings</i>	2-4
<i>Electrical Grounding</i>	2-6
<i>Electrical and Electronic Recycling</i>	2-7
<i>Fuse Replacement</i>	2-7
BacT/ALERT® 3D Hardware	2-8
<i>Mycobacteria Drawers (Clinical Use)</i>	2-8
<i>Controller Module</i>	2-9
Operator Panel	2-9
Barcode Reader Aperture	2-9

Backup Drive	2-10
Keyboard Drawer	2-10
Quick Reference Card	2-10
Keyboard	2-10
Accessing the Controller Module Keyboard	2-11
Printer Port	2-13
Audible Alarm Speaker	2-14
External Speaker Port	2-14
Power Connector	2-14
Power Switch	2-14
Display Switch Assembly	2-14
CPU 1 Port	2-14
CPU 2 Port	2-14
Module Ports (6)	2-14
Monitor Port	2-14
Modem Port	2-14
LIS Port (Diagnostic Port)	2-14
COMM Port	2-15
<i>Controller Module Specifications</i>	<i>2-15</i>
Electrical Power Services Requirements	2-15
Power Consumed in Watts	2-15
Heat Dissipated	2-15
Sound Emission	2-15
Instrument Dimensions	2-15
<i>Controller Module Environmental Requirements</i>	<i>2-15</i>
Operating Temperature Range	2-15
Storage Temperature Range	2-15
Operating Humidity Range	2-15
Storage Humidity Range	2-15
Maximum Operating and Storage Altitude	2-16
Pollution degree 2 in accordance with IEC 664	2-16
Overvoltage Category II per IEC 664	2-16
<i>Combination Module</i>	<i>2-16</i>
Operator Panel	2-17
Barcode Reader	2-17
Keyboard	2-18
UPS	2-18
Backup Drive	2-18
Drawer	2-18
Rack	2-18
Cell	2-18
Cell Indicator Lamp/Cell Indicator Light	2-18
Cell Flag	2-18
Thermometer (Not Shown)	2-19
Drawer Indicator Light — Yellow	2-19
Drawer Indicator Light — Green	2-19
Printer Port	2-19
External Speaker Port	2-19
Power Connector	2-19

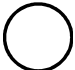Power Switch	2-19
Display Switch Assembly	2-20
CPU 1 Port	2-20
CPU 2 Port	2-20
Module Port (3)	2-20
Modem Port	2-20
LIS Port	2-20
Barcode Scanner Port	2-20
Keyboard Port	2-20
Monitor Port	2-20
COMM Port	2-20
<i>Combination Module Specifications</i>	<i>2-20</i>
Electrical Power Services Requirements	2-20
Power Consumed in Watts	2-21
Heat Dissipated	2-21
Combination Module Dimensions	2-21
Sound Emission	2-21
<i>Combination Module Environmental Requirements</i>	<i>2-21</i>
Operating Temperature Range	2-21
Storage Temperature Range	2-21
Operating Humidity Range	2-21
Storage Humidity Range	2-21
Maximum Operating and Storage Altitude	2-21
Pollution degree 2 in accordance with IEC 664 (Clinical Use)	2-21
Overvoltage Category II per IEC 664 (Clinical Use)	2-21
<i>Incubation Module</i>	<i>2-22</i>
Drawer	2-22
Rack	2-22
Cell	2-22
Cell Indicator Lamp/Cell Indicator Light	2-23
Cell Flag	2-23
Drawer Indicator Light — Yellow	2-23
Drawer Indicator Light — Green	2-23
Controller Module Port	2-24
Power Connector	2-24
Power Switch	2-24
<i>Incubation Module Specifications</i>	<i>2-25</i>
Electrical Power Services Requirements	2-25
Power Consumed in Watts	2-25
Heat Dissipated	2-25
Instrument Dimensions	2-25
Sound Emission	2-25
<i>Incubation Module Environmental Requirements</i>	<i>2-25</i>
Operating Temperature Range	2-25
Storage Temperature Range	2-25
Operating Humidity Range	2-25
Storage Humidity Range	2-25
Maximum Operating and Storage Altitude	2-25
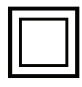
<i>Instrument Installation and Setup (Clinical Use)</i>	2-26
<i>Instrument Installation and Setup (INDUSTRY Use)</i>	2-26
BacT/ALERT® 3D Software	2-27
<i>Operator Panel Screens</i>	2-27
<i>Common Screen Elements</i>	2-28
Icon	2-28
Button	2-28
Common System Buttons	2-28
Scroll Button	2-29
Slider Switch	2-29
Anchor Display/Scroll Buttons	2-29
Text Entry Field	2-30
BASIC FUNCTIONS (CLINICAL USE)	3-1
Introduction	3-2
Monitoring the System	3-3
<i>Main Screen Introduction</i>	3-3
Background Color	3-4
Instrument Icon	3-4
Bottle Count Table	3-5
<i>Viewing Faults</i>	3-6
<i>Viewing the Cell Status Screen</i>	3-7
Understanding the View Cell Status Screen Display	3-8
Text/Data Entry	3-9
<i>Common Text Fields and Field Limit</i>	3-9
<i>Using the Barcode Scanner to Enter Data</i>	3-10
<i>Manually Entering Text into a Data Entry Field (Keyboard)</i>	3-10
Loading Bottles	3-11
<i>Loading Bottles</i>	3-11
<i>Changing the Maximum Test Time — Individual Bottles</i>	3-15
<i>Handling Anonymous Bottles</i>	3-16
Unloading Bottles	3-16
<i>Unloading Bottles</i>	3-17
<i>Handling Unconfirmed Positive Bottles (False Positives)</i>	3-19
Accessing the Setup Screen Function Buttons	3-20
<i>Accessing the Setup Screen</i>	3-20
Inactivity Timeout for all Setup Screens	3-21
<i>Setup Screen Function Buttons</i>	3-21
Viewing and Printing	3-23
<i>Introduction</i>	3-23
Viewing Bottle Data	3-23
Viewing/Printing Reports	3-23
Viewing/Printing Graphs	3-23

Using the Print Screen Function	3-24
<i>Viewing, Printing, and Exporting Test Data</i>	3-24
<i>Viewing and Printing Bottle Graphs</i>	3-31
<i>Display Bottle Readings</i>	3-34
Sending/Requesting LIS Information	3-39
<i>Sending Results to the LIS</i>	3-39
<i>Requesting Information from the LIS</i>	3-39
BASIC FUNCTIONS (INDUSTRY USE)	4-1
Introduction	4-2
Monitoring the System	4-3
<i>Main Screen Introduction</i>	4-3
Background Color	4-4
Instrument Icon	4-4
Bottle Count Table	4-5
<i>Viewing Faults</i>	4-6
<i>Viewing the Cell Status Screen</i>	4-7
Understanding the View Cell Status Screen Display	4-8
Text/Data Entry	4-9
<i>Common Text Fields and Field Limits</i>	4-9
<i>Using the Barcode Scanner to Enter Data</i>	4-10
<i>Manually Entering Text into a Data Entry Field (Keyboard)</i>	4-10
Loading Bottles	4-11
<i>Loading Bottles</i>	4-11
<i>Changing the Maximum Test Time — Individual Bottles</i>	4-14
<i>Handling Anonymous Bottles</i>	4-15
Unloading Bottles	4-16
<i>Unloading Bottles</i>	4-16
<i>Handling Unconfirmed Positive Bottles (False Positives)</i>	4-19
Accessing the Setup Screen Function Buttons	4-20
<i>Accessing the Setup Screen</i>	4-20
Inactivity Timeout for all Setup Screens	4-21
<i>Setup Screen Function Buttons</i>	4-21
Viewing and Printing	4-23
<i>Introduction</i>	4-23
Viewing Bottle Data	4-23
Viewing/Printing Reports	4-23
Viewing/Printing Graphs	4-23
Using the Print Screen Function	4-24
<i>Viewing, Printing, and Exporting Test Data</i>	4-24
<i>Viewing and Printing Bottle Graphs</i>	4-31
<i>Display Bottle Readings</i>	4-34

Sending/Requesting LIS Information	4-39
<i>Sending Results to the LIS</i>	4-39
<i>Requesting Information from the LIS</i>	4-39
EDITING TEST DATA (CLINICAL USE)	5-1
Viewing/Editing Bottle Data	5-2
<i>Introduction</i>	5-2
<i>Selecting Bottles Using the Edit Cell Contents Button</i>	5-3
<i>Selecting Bottles Using the Select Bottle to Edit/Graph Button</i>	5-4
<i>Editing Bottle Details Using the Edit Bottle Detail Screen.</i>	5-5
Edit Bottle ID Field	5-6
View Accession Number Field	5-6
View Hospital ID Field.	5-6
View Patient First Name Field	5-6
View Patient Last Name Field.	5-6
Edit Load Status Slidebar Switch	5-7
Edit Maximum Test Time Scroll Buttons.	5-7
Edit Bottle Type Scroll Button.	5-7
View Cell Location Icon	5-8
View First Loaded Time Icon	5-8
View Last Unloaded Time Icon.	5-8
View Time of Last Bottle Reading Icon.	5-8
View Test Time Icon	5-8
View Test Result Icon	5-9
Edit Test Result Button.	5-9
Graph Bottle Readings Button	5-10
View Algorithm/Polynomial Icon	5-10
View How Determined/Positivity Index Icon	5-11
Editing Data Relationships	5-12
<i>Introduction</i>	5-12
<i>Initiating the Edit Data Relationships Function</i>	5-12
<i>Editing Bottle ID to Accession Number Relationships.</i>	5-14
Attaching Bottle IDs without an Accession Number to an Accession Number .5-14	
Moving a Bottle ID Association from one Accession Number to Another 5-15	
<i>Editing Accession Number to Hospital ID Relationships.</i>	5-16
Attaching Accession Numbers without a Hospital ID to a Hospital ID 5-16	
Moving an Accession Number Association from one Hospital ID to Another . . 5-17	
<i>Editing Hospital ID to Patient Name Relationships.</i>	5-17
EDITING TEST DATA (INDUSTRY USE)	6-1
Viewing/Editing Bottle Data	6-2
<i>Introduction</i>	6-2
<i>Selecting Bottles Using the Edit Cell Contents Button</i>	6-3
<i>Selecting Bottles Using the Select Bottle to Edit/Graph Button</i>	6-4
<i>Editing Bottle Details Using the Edit Bottle Detail Screen.</i>	6-5

Edit Bottle ID Field	6-6
View Sample ID Field	6-6
View User Defined 1	6-6
View User Defined 2	6-6
View User Defined 3	6-6
Edit Load Status Sidebar Switch	6-7
Edit Maximum Test Time Scroll Buttons	6-7
Edit Bottle Type Scroll Button	6-7
View Cell Location Icon	6-8
View First Loaded Time Icon	6-8
View Last Unloaded Time Icon	6-8
View Time of Last Bottle Reading Icon	6-8
View Test Time Icon	6-8
View Test Result Icon	6-9
Edit Test Result Button	6-9
Graph Bottle Readings Button	6-10
View Algorithm/Polynomial Icon	6-10
View How Determined/Positivity Index Icon	6-11
Editing Data Relationships	6-12
<i>Introduction</i>	6-12
<i>Initiating the Edit Data Relationships Function</i>	6-12
<i>Editing Bottle ID to Sample ID Relationships</i>	6-14
Attaching Bottle IDs without a Sample ID to a Sample ID	6-14
Moving a Bottle ID Association from one Sample ID to Another	6-15
<i>Editing Sample ID to User Defined 3 Relationships</i>	6-16
Attaching Sample IDs without a User Defined 3 to a User Defined 3	6-16
Moving a Sample ID Association from One User Defined 3 to Another	6-17
<i>Editing User Defined 3 to User Defined 1/User Defined 2 Relationships</i>	6-17
SOFTWARE CONFIGURATION	7-1
Setting the Maximum Test Time	7-2
Setting the Audible Alarms	7-3
<i>Priority of Alarms</i>	7-5
<i>Terminating an Instrument Alarm</i>	7-5
Changing the System Password	7-5
Initiating Manual Backup	7-7
Configuring Report Screens	7-9
<i>Introduction</i>	7-9
<i>Entering Report Labels</i>	7-10
<i>Configuring Report Contents</i>	7-13
Load Report Configuration Screen (Clinical Use)	7-16
Status Report Configuration Screen (Clinical Use)	7-17
Unload Report Configuration Screen (Clinical Use)	7-18
Load Report Configuration Screen (INDUSTRY Use)	7-19
Status Report Configuration Screen (INDUSTRY Use)	7-20

Unload Report Configuration Screen (INDUSTRY Use)	7-21
<i>Viewing and Printing Calibration Data</i>	7-21
<i>Viewing and Printing Calibration History</i>	7-25
SYSTEM MAINTENANCE	8-1
Hardware Maintenance	8-2
<i>Preventative Maintenance</i>	8-2
<i>Safety Precautions and Procedures</i>	8-2
General Precautions	8-3
Spill Cleanup	8-3
Disinfection Procedure for Spills Onto the Instrument (Clinical Use)	8-4
Disinfection Procedure for Spills Onto the Instrument (INDUSTRY Use)	8-5
Disinfection Procedure for Spills Within the Instrument	8-5
<i>Using the Keyboard in Place of the Operator Panel</i>	8-8
<i>UPS On/Off Button Location</i>	8-9
Controller Module	8-9
Combination Module	8-10
<i>Controller/Combination Module Reboot/Shutdown (BacT/ALERT® 3D Select and SelectLink)</i>	8-10
Shutdown Method 1	8-11
Shutdown Method 2	8-11
Shutdown Method 3	8-12
Shutdown Method 4	8-13
<i>Controller or Combination Module Startup</i>	8-13
<i>Full System Shutdown (BacT/ALERT® 3D Signature)</i>	8-13
<i>Full System Startup (BacT/ALERT® 3D Signature)</i>	8-14
<i>Set Up an External Monitor</i>	8-14
<i>Restore Internal Monitor Settings</i>	8-14
Software Maintenance	8-15
<i>Restarting an Incubation Module or the Combination Module Drawers</i>	8-15
<i>Setting and Formatting the System Date and Time</i>	8-17
<i>Enabling and Disabling Modules, Drawers, Racks, and Cells</i>	8-18
Relocating Bottles	8-21
<i>Adjusting an Incubation or Combination Module's Temperature</i>	8-21
Checking an Incubation or Combination Module's Temperature	8-21
Setting the Optimal Temperature for an Incubation or Combination Module	8-24
Calibrating an Incubation or Combination Module's Temperature	8-25
<i>Calibrating an Instrument Cell</i>	8-25
Locating a Cell Which Failed Calibration	8-26
Viewing a Cell's Readings and/or Calibrating a Cell	8-26
<i>Viewing Incubation Module Information</i>	8-29
SYSTEM TROUBLESHOOTING	9-1
Introduction	9-2

Fault Codes	9-3
<i>Instrument Fault Codes</i>	9-3
<i>Instrument Status Codes</i>	9-24
<i>Operator Error Codes</i>	9-27
<i>Bottle Problems</i>	9-39
<i>User Output Device Problems</i>	9-39
21 CFR PART 11 MODE	10-1
Log In/Out of System — 21 CFR Part 11 Mode	10-2
<i>Logging In for the First Time</i>	10-2
<i>Logging In</i>	10-5
<i>Inactivity Timeout</i>	10-7
<i>Login Errors</i>	10-7
<i>Change Password</i>	10-8
<i>Change Password Errors</i>	10-9
<i>Logging Out</i>	10-12
Configuring Users — 21 CFR Part 11 Mode	10-12
<i>Adding a User</i>	10-13
<i>Deleting a User</i>	10-15
Deleting More Than One User	10-16
<i>Clearing a User Password</i>	10-16
Clearing More Than One User Password	10-17
Audit Trail	10-18
<i>Accessing the Audit Trail</i>	10-20
MYCOBACTERIAL TESTING (CLINICAL USE)	11-1
System Description	11-2
<i>Intended Use</i>	11-2
<i>Overview</i>	11-2
<i>BacT/ALERT® 3D Instrument</i>	11-3
Circulation Fan	11-3
Rocker Clamp	11-3
Installation Procedures and Special Requirements	11-4
Barcodes	11-4
Load Bottles	11-5
Unload Positives and Negatives	11-5
Set Maximum Test Time	11-5
Positive Detection Algorithm	11-5
Bottle Status	11-5
System Startup	11-6
<i>General Information</i>	11-6
MB Drawer Configuration	11-6
Limitations of the Test	11-7
Service and Maintenance	11-7

<i>Theory of Operation</i>	11-7
Principle of Detection	11-7
<i>Safety Features</i>	11-8
General Precautions	11-9
Spill Cleanup	11-9
Disinfection Procedure for Spills Within/Onto the Instrument	11-9
Remove Drawer	11-10
<i>Device Comparison for Recovery of Mycobacteria</i>	11-10
INTERNATIONAL CHARACTER ENTRY	A-1
International Character Entry	A-2
Clinical Use	A-3
INDUSTRY Use	A-3
<i>Entering International Characters</i>	A-4
BOTTLE QUALITY CONTROL	B-1
BacT/ALERT® Culture Bottles	B-2
<i>Introduction</i>	B-2
Flip Cap	B-3
Stopper/Seal	B-3
Bottle	B-3
Volume Designations (Clinical Use)	B-3
Volume Designations (INDUSTRY Use)	B-3
Barcode	B-3
Sensor	B-4
Limitations of the Test	B-4
<i>Clinical Use</i>	B-4
<i>INDUSTRY Use</i>	B-4
<i>Quality Control of Growth Performance</i>	B-4
Blood Culture Bottles (SA, SN, FA, FN, and PF)	B-5
Mycobacteria Culture Bottles (MB and MP)	B-5
Culture Bottles (iAST, iNST, and iLYM)	B-5
BEST PRACTICES	C-1
Best Practices for Preventing False Positives	C-2
<i>Preventing False Positives — User</i>	C-2
High Amount of White Blood Cells (Clinical Use)	C-2
Blood (or Sample) Volume too High	C-2
Large Bottle Loading and Unloading Events	C-3
Temperature Changes in the Environment	C-3
Anonymous Bottle Loading of BacT/ALERT® FA and PF Bottles	C-4
Bottle Not Completely Loaded into the Cell	C-4
<i>Preventing False Positives — Instrument</i>	C-5
Cell Noise	C-5
High Initial Value Readings	C-7
False Positive MP Process Bottles Flagged by Delta Algorithm (Clinical Use)	C-7

GLOSSARY.....	GLOSSARY-1
INDEX	INDEX-1

LIST OF FIGURES

Figure 1-1:	<i>Procedure Icon</i>	1-7
Figure 2-1:	<i>Electrical Grounding Requirements</i>	2-7
Figure 2-2:	<i>Instrument Front View</i>	2-9
Figure 2-3:	<i>Removing the Barcode Reader Through the Barcode Reader Aperture</i>	2-10
Figure 2-4:	<i>Accessing the Keyboard and Reference Card</i>	2-12
Figure 2-5:	<i>Back View Controller Module</i>	2-13
Figure 2-6:	<i>Front View Combination Module</i>	2-16
Figure 2-7:	<i>Back View Combination Module</i>	2-17
Figure 2-8:	<i>Cell Flag (Clip Removed)</i>	2-19
Figure 2-9:	<i>Front View Incubation Module (Right-Handed Configuration)</i>	2-22
Figure 2-10:	<i>Back View Incubation Module</i>	2-24
Figure 2-11:	<i>Button Examples</i>	2-28
Figure 2-12:	<i>Scroll Button</i>	2-29
Figure 2-13:	<i>Slider Switch</i>	2-29
Figure 2-14:	<i>Anchor Display/Scroll Buttons</i>	2-30
Figure 3-1:	<i>Main Screen</i>	3-3
Figure 3-2:	<i>Combination Module with an Additional Incubation Module</i>	3-5
Figure 3-3:	<i>Controller Module with One Incubation Module</i>	3-5
Figure 3-4:	<i>Bottle Count Table/Unload Buttons</i>	3-6
Figure 3-5:	<i>View Cell Status Screen</i>	3-7
Figure 3-6:	<i>Disabled Cell and Drawer</i>	3-9
Figure 3-7:	<i>Main Screen — Load Mode</i>	3-12
Figure 3-8:	<i>Change Maximum Test Time Screen</i>	3-15
Figure 3-9:	<i>Main Screen — Unload Mode</i>	3-17
Figure 3-10:	<i>Setup Screen</i>	3-20
Figure 3-11:	<i>Report Selection Screen</i>	3-24
Figure 3-12:	<i>Sample Report Screen</i>	3-26
Figure 3-13:	<i>Find Text Screen</i>	3-29
Figure 3-14:	<i>Save to File Screen</i>	3-30
Figure 3-15:	<i>Select Bottle to Edit/Graph Screen</i>	3-32
Figure 3-16:	<i>Graph Bottle Readings Screen</i>	3-33
Figure 3-17:	<i>Bottle Readings Screen</i>	3-35

Figure 3-18: Find Text Screen	3-37
Figure 3-19: Save to File Screen	3-38
Figure 4-1: Main Screen	4-3
Figure 4-2: Combination Module with an Additional Incubation Module	4-5
Figure 4-3: Controller Module with One Incubation Module	4-5
Figure 4-4: Bottle Count Table/Unload Buttons	4-6
Figure 4-5: View Cell Status Screen	4-7
Figure 4-6: Disabled Cell and Drawer	4-9
Figure 4-7: Main Screen — Load Mode	4-12
Figure 4-8: Change Maximum Test Time Screen	4-15
Figure 4-9: Main Screen — Unload Mode	4-17
Figure 4-10: Setup Screen	4-20
Figure 4-11: Report Selection Screen	4-24
Figure 4-12: Sample Report Screen	4-26
Figure 4-13: Find Text Screen	4-29
Figure 4-14: Save to File Screen	4-30
Figure 4-15: Select Bottle to Edit/Graph Screen	4-32
Figure 4-16: Graph Bottle Readings Screen	4-33
Figure 4-17: Bottle Readings Screen	4-35
Figure 4-18: Find Text Screen	4-37
Figure 4-19: Save to File Screen	4-38
Figure 5-1: Edit Cell Contents Screen	5-3
Figure 5-2: Select Bottle to Edit/Graph Screen	5-4
Figure 5-3: Edit Bottle Detail Screen	5-5
Figure 5-4: Edit Test Result Screen	5-9
Figure 5-5: Edit Data Relationships Screen	5-13
Figure 5-6: Edit Bottle ID to Accession Number Relationships Screen	5-14
Figure 5-7: Edit Accession Number to Hospital ID Relationships Screen	5-16
Figure 5-8: Edit Hospital ID to Patient Name Relationships Screen	5-18
Figure 6-1: Edit Cell Contents Screen	6-3
Figure 6-2: Select Bottle to Edit/Graph Screen	6-4
Figure 6-3: Edit Bottle Detail Screen	6-5
Figure 6-4: Edit Test Result Screen	6-9
Figure 6-5: Edit Data Relationships Screen	6-13

Figure 6-6:	<i>Edit Bottle ID to Sample ID Relationships Screen</i>	6-14
Figure 6-7:	<i>Edit Sample ID to User Defined 3 Relationships Screen</i>	6-16
Figure 6-8:	<i>Edit User Defined 3 to User Defined 1/User Defined 2 Relationships Screen</i>	6-18
Figure 7-1:	<i>Set Maximum Test Time Screen</i>	7-2
Figure 7-2:	<i>Setting the Audible Alarm Options Screen</i>	7-4
Figure 7-3:	<i>Change Password Screen</i>	7-6
Figure 7-4:	<i>Padlock Icon (Full Open position)</i>	7-6
Figure 7-5:	<i>Padlock Icon (Half Open Position)</i>	7-7
Figure 7-6:	<i>Padlock Icon (Closed Position)</i>	7-7
Figure 7-7:	<i>Backup Management Screen</i>	7-8
Figure 7-8:	<i>Backup In Progress Icon</i>	7-9
Figure 7-9:	<i>Report Selection Screen</i>	7-10
Figure 7-10:	<i>Report Label Entry Screen</i>	7-11
Figure 7-11:	<i>Report Selection Screen</i>	7-14
Figure 7-12:	<i>Load Report Configuration Screen (Clinical Use)</i>	7-16
Figure 7-13:	<i>Status Report Configuration Screen (Clinical Use)</i>	7-17
Figure 7-14:	<i>Unload Report Configuration Screen (Clinical Use)</i>	7-18
Figure 7-15:	<i>Load Report Configuration Screen (INDUSTRY Use)</i>	7-19
Figure 7-16:	<i>Status Report Configuration Screen (INDUSTRY Use)</i>	7-20
Figure 7-17:	<i>Unload Report Configuration Screen (INDUSTRY Use)</i>	7-21
Figure 7-18:	<i>Report Selection Screen</i>	7-22
Figure 7-19:	<i>Cell Calibration Report</i>	7-23
Figure 7-20:	<i>Calibration Report</i>	7-24
Figure 7-21:	<i>Report Selection Screen</i>	7-25
Figure 7-22:	<i>Calibration History Screen</i>	7-26
Figure 8-1:	<i>Front Access UPS Orientation</i>	8-10
Figure 8-2:	<i>Exit Software Button</i>	8-12
Figure 8-3:	<i>View Cell Status Screen</i>	8-16
Figure 8-4:	<i>Set Date/Time Screen</i>	8-17
Figure 8-5:	<i>Enable/Disable Module, Drawer, Rack, or Cell Screen</i>	8-19
Figure 8-6:	<i>Reference Thermometer (Drawer B — Right-hand Configuration)</i>	8-22
Figure 8-7:	<i>Digital Thermometer (Drawer A — Left-hand Configuration)</i>	8-22
Figure 8-8:	<i>Calibrate Module Temperature Screen</i>	8-23
Figure 8-9:	<i>Calibrate Cell Screen</i>	8-27

Figure 8-10: View Incubation Module Information Screen	8-30
Figure 9-1: Instrument Icon with Fault Codes	9-3
Figure 9-2: Instrument Status Code 710	9-24
Figure 9-3: Operator Error Code 911	9-27
Figure 9-4: QuickScan® 6000 Reset Barcode	9-40
Figure 9-5: Honeywell Reset Barcode	9-40
Figure 9-6: Red Informational Warning Screen	9-42
Figure 10-1: Main Screen While Logged Out — 21 CFR Part 11 Mode	10-2
Figure 10-2: User Login Screen	10-3
Figure 10-3: User Login Screen with Change Password Fields	10-4
Figure 10-4: Main Screen While Logged In — 21 CFR Part 11 Mode	10-5
Figure 10-5: User Login Screen	10-6
Figure 10-6: User Login Screen with Wrong Password Alert	10-8
Figure 10-7: User Login Screen with Change Password Fields	10-9
Figure 10-8: User Login Screen with Change Password Error	10-10
Figure 10-9: User Login Screen with Wrong Password Alert	10-11
Figure 10-10: User Configuration Screen	10-13
Figure 10-11: Add User Screen	10-14
Figure 10-12: Delete User Screen	10-15
Figure 10-13: Clear Password Screen	10-17
Figure 11-1: Instrument Icon for MB-Configured System	11-3
Figure 11-2: Bottle Count Table and Unload Buttons	11-6
Figure 11-3: Metabolic Pathway for Mycobacterial CO ₂	11-8
Figure B-1: Typical BacT/ALERT® Culture Bottle	B-2
Figure C-1: Example Cell Graph	C-4
Figure C-2: Cell Graph Example — Electronics	C-5
Figure C-3: Cell Graph Example — Reverts to Normal	C-6
Figure C-4: Cell Graph Example — Cyclic	C-6
Figure C-5: Cell Graph Example — High Initial Value Reading	C-7

LIST OF TABLES

<i>Table 2-1: Common System Buttons</i>	2-28
<i>Table 5-1: Bottle Specific Algorithms</i>	5-10
<i>Table 5-2: Status Determination Codes</i>	5-11
<i>Table 6-1: Bottle Specific Algorithms</i>	6-10
<i>Table 6-2: Status Determination Codes</i>	6-11
<i>Table 7-1: Report Field Descriptions</i>	7-12
<i>Table 10-1: Audit Trail Events</i>	10-18
<i>Table 11-1: Device Comparison for Recovery of Mycobacteria</i>	11-10
<i>Table A-1: International Character Table</i>	A-2
<i>Table C-1: Bottle Volumes</i>	C-2

About This Chapter

This chapter gives you important information about the BacT/ALERT® 3D system and how to use this manual. It is recommended that you read this chapter first.

IMPORTANT: *Read this manual carefully before you attempt to operate the BacT/ALERT® 3D system.*

Chapter Contents

Intended Audience • 1-2
<i>Specimen/Sample • 1-2</i>
Purpose of the BacT/ALERT® 3D System • 1-2
Additional Supplies • 1-3
Purpose of This Manual • 1-4
Manual Organization • 1-4
Chapter Organization • 1-6
Finding Topics • 1-6
Typographic and Usage Conventions • 1-7
<i>Name and Titles • 1-7</i>
<i>Press • 1-7</i>
<i>Procedural Steps • 1-7</i>
<i>References • 1-7</i>
<i>Select • 1-8</i>
<i>User Input • 1-8</i>
Warnings, Cautions, and Information • 1-8

Intended Audience

The BacT/ALERT® 3D system and this manual are intended for laboratory use by trained, professional, clinical and INDUSTRY users.

Most material in the manual applies to both sets of users. If any information in the manual is intended for clinical use only or industry use only, it is marked *Clinical Use* or *INDUSTRY Use*.

Specimen/Sample

In this manual, you will find references to both “specimen” and “sample.” “Specimen” refers to Clinical Use, and “sample” refers to INDUSTRY Use.

Purpose of the BacT/ALERT® 3D System

The BacT/ALERT® 3D Microbial Detection System is a totally automated test system capable of incubating, agitating, and continuously monitoring aerobic and anaerobic media inoculated with:

- patient specimens suspected of having bacteremia, fungemia, and/or mycobacteremia (Clinical Use),
- or*
- samples to be monitored for bacterial or fungal contamination (INDUSTRY Use).



CAUTION: bioMérieux shall not be liable as to any defect arising from abuse of the instrument, failure to operate and maintain the instrument in accordance with the User Manual, operation of the instrument by a person who has not been trained in its operation by bioMérieux, repair, service, alteration or modification of the instrument by any person other than service personnel of bioMérieux, or modification, change or reuse of the disposables supplied by bioMérieux for use in the instrument.



CAUTION: This BacT/ALERT® 3D User Manual is only intended for use with B.30 Software or higher. The software version B.30 (or higher) is displayed at the bottom of the instrument icon on the Main screen.



CAUTION: All figures depicting operator screens are examples only. Actual screens may differ to the extent they are affected by the actual data entered by the operator, or actual data transmitted to the instrument over the DBMS or LIS interface, or actual data generated by the instrument.



CAUTION: Regarding section "Entering Report Labels," the user is solely responsible for the choice of customized report label text and for validating that the intended label text appears in all associated reports. *bioMérieux shall not be liable for any consequences resulting from misinterpretation of customized report labels.*



CAUTION: (INDUSTRY Use) The BacT/ALERT[®] system is marketed for use in the detection of microorganisms in blood and other normally sterile body fluids, and in the detection of microorganisms in other sample matrices and for additional specific indications for use, as specified in the package inserts, user manuals, and labeling of the specific components of the BacT/ALERT[®] system. Customers who use the BacT/ALERT[®] system in testing of sample types or for indications other than those described in the applicable package inserts and user manuals do so at their own risk. Performance characteristics for the BacT/ALERT[®] system for any use outside the labeling, package insert, or user manual have not been established.

Additional Supplies

Contact bioMérieux or your local vendor for laboratory supplies and accessories.

Purpose of This Manual

This manual focuses on the BacT/ALERT® 3D software application and how you use it in your workflow. It contains step-by-step procedures for using your BacT/ALERT® 3D system.

By using these procedures, you can perform all the functions required to operate your system, including:

- accessing the BacT/ALERT® 3D software
- system monitoring
- entering data (where applicable)
- loading and unloading bottles
- viewing and printing data (printing with the BacT/ALERT® 3D Signature is performed at the data management computer)
- LIS interaction with the BacT/ALERT® 3D SelectLink
- accessing the Setup screen

Note: *Screens and figures are intended for illustrative purposes only and are not to be construed as representations of actual test data, results, or components. Screens and components are not shown to scale.*

Manual Organization

This manual is organized by chapters. Chapters are organized according to the order of menu commands in the software application.

Chapter 1, How To Use This Manual — Provides an introduction to the BacT/ALERT® 3D system and how to use this manual. It is recommended that you read this chapter first.

Chapter 2, System Overview — This chapter gives you a complete description of the different BacT/ALERT® 3D hardware and software configurations available. Also, the operator panel screens are listed along with a description of common screen elements.

Chapter 3, Basic Functions (Clinical Use) — Introduces you to the Main screen and shows you how to perform basic functions (ex. enter data, view faults, view the Cell Status screen, load/unload bottles, etc.). This chapter also introduces you to the Setup screen and the associated function buttons.

Chapter 4, Basic Functions (INDUSTRY Use) — Introduces you to the Main screen and shows you how to perform basic functions (ex. enter data, view faults, view the Cell Status screen, load/unload bottles, etc.). This chapter also introduces you to the Setup screen and the associated function buttons.

Chapter 5, Editing Test Data (Clinical Use) — Explains how to access bottle data for loaded and unloaded bottles using the Edit Bottle Detail screen. This chapter shows you how to view and edit bottle data.

Chapter 6, Editing Test Data (INDUSTRY Use) — Explains how to access bottle data for loaded and unloaded bottles using the Edit Bottle Detail screen. This chapter shows you how to view and edit bottle data.

Chapter 7, Software Configuration — Explains how to configure the software for your specific needs. This chapter shows you how to set the maximum test time, set the audible alarms, change the system password, initiate a manual backup, configure report screens, and view and print calibration data.

Chapter 8, System Maintenance — This chapter provides you with procedures on how to perform maintenance on the BacT/ALERT® 3D hardware and software.

Chapter 9, System Troubleshooting — Describes the different types of instrument fault, instrument status and operator error codes, as well as bottle and user output device problems, you may encounter when using the instrument. Cause(s) and solutions(s) for each type of fault/error/problem are also listed.

Chapter 10, 21 CFR Part 11 Mode — This chapter explains how to log in and out of the instrument while in 21 CFR Part 11 mode, as well as instructions for configuring users (ex. adding or deleting a user, or clearing a user password). This chapter also describes the events recorded in the audit trail and how to retrieve the audit trail.

Chapter 11, Mycobacterial Testing (Clinical Use) — This chapter provides you with a complete description of the of the BacT/ALERT® 3D instrument when it is configured for Mycobacterial (MB) functions.

Appendix A, International Character Entry — Explains how to enter international characters on the BacT/ALERT® 3D instrument.

Appendix B, Bottle Quality Control — This chapter provides you with a description of the BacT/ALERT® 3D culture bottle along with a quality control procedure.

Glossary — An alphabetized list of frequently used terms along with a definition for each term.

Chapter Organization

All chapters include the following:

- *About This Chapter* — Brief description of the chapter's content and purpose.
- *Chapter Contents* — A table of contents for the chapter.
- *Descriptions and/or Procedures* — Chapters contain descriptions and procedures appropriate to their subject matter. See the Manual Organization section in this chapter for more information.
- *Background Information*, where applicable and useful.

Finding Topics

This manual uses several methods to help you find information and keep your bearings.

Table of Contents — Located at the front of the manual. It contains the titles of all chapters/appendices and their sections, and the page number of each title and section.

List of Figures — Located at the front of the manual. It contains a list of all figures in the manual and the page number of each figure.

List of Tables — Located at the front of the manual. It contains a list of all tables in the manual and the page number of each table.

Chapter Contents — Located at the front of each chapter. It lists all sections in the chapter and their page numbers.

Page Headers — Located at the top of each page. There are two parts to a header: the chapter title and the primary section title.

Page Footers — Located at the bottom of each page. There are three parts to a footer: the manual's title, the chapter's part number, and the page number.

Index — Located at the back of the manual. It contains topical entries and their page numbers.

Typographic and Usage Conventions

Name and Titles

Button, icon and field names are in Proper Case, bold.

Example: Press the **Select Maximum Test Time** button.

The names of windows and screens are in Proper Case, but are not bolded.

Example: The Set Maximum Test Time screen...

Press

This manual uses the word “press” to refer to pressing a button on the Controller Module Operator Panel.

Example: Press the **Load Bottles** button.

We also use the word “press” to refer to pressing a key on the keyboard in order to initiate action in the firmware.

Keyboard entries are in Proper Case and bolded (ex. **Ctrl**). If two keys are to be pressed simultaneously, they will be separated with a plus sign (ex. press **Ctrl + Alt + Delete**).

Procedural Steps

Steps in procedures are sequentially numbered. A bullet list in a step indicates options.

Sections that contain procedures are denoted by the Procedure icon in the margin.

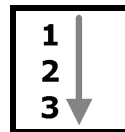


Figure 1-1: Procedure Icon

References

References to chapter and section titles in this manual are in Proper Case.

Example: See [Chapter 7, Software Configuration](#).

References to other manuals are in Proper Case and italic.

Example: See the *BacT/ALERT® 3D Service Manual*.

Select

The word “select” is generally used for selecting user interface navigation.

Example: Select the appropriate bottle type from the **Media Type** scroll button.

User Input

Instructions for user input begin with the word “type” or “enter.” This manual uses bold for literal user input and italic for placeholders.

Example of a Literal User Input: Enter the Software Exit Password **24313124**.

In this example, you are to type exactly what you see on the page (**24313124** in this example).

Example of a Placeholder: Enter your *password* before you...

In this example, you are to type your assigned password.

Warnings, Cautions, and Information

This manual uses different types of symbols to alert you to important information. Symbols and their associated information are labeled in text where they occur and set off from surrounding paragraphs, as shown in the following examples.

WARNING



Warning is a statement that alerts the user to the possibility of injury, death, or other serious adverse reactions associated with the use or misuse of a device.



CAUTION: Caution is a statement that alerts the user to the possibility of a problem with the device associated with its use or misuse. Such problems include device malfunction, device failure, damage to the device, or damage to other property. Where applicable, a caution statement may include a precaution that should be taken to avoid the hazard.

IMPORTANT: *Important relates to content presented in this manual. It is used to reinforce the importance of your understanding or remembering something.*

Note: *Note supplies additional information about a topic.*

About This Chapter

This chapter gives you a complete description of the different BacT/ALERT® 3D hardware and software configurations available. Also, the operator panel screens are listed along with a description of common screen elements.

Note: *Information in this chapter that is intended for clinical use only or industry use only is marked Clinical Use or INDUSTRY Use.*

Chapter Contents

Introduction • 2-2
<i>Hardware Configuration • 2-2</i>
<i>Software Configuration Options • 2-2</i>
<i>21 CFR Part 11 and HIPAA • 2-3</i>
<i>Theory Of Operation • 2-4</i>
<i>Electrical Warnings • 2-4</i>
<i>Electrical Grounding • 2-6</i>
<i>Electrical and Electronic Recycling • 2-7</i>
<i>Fuse Replacement • 2-7</i>
BacT/ALERT® 3D Hardware • 2-8
<i>Mycobacteria Drawers (Clinical Use) • 2-8</i>
<i>Controller Module • 2-9</i>
<i>Controller Module Specifications • 2-15</i>
<i>Controller Module Environmental Requirements • 2-15</i>
<i>Combination Module • 2-16</i>
<i>Combination Module Specifications • 2-20</i>
<i>Combination Module Environmental Requirements • 2-21</i>
<i>Incubation Module • 2-22</i>
<i>Incubation Module Specifications • 2-25</i>
<i>Incubation Module Environmental Requirements • 2-25</i>
<i>Instrument Installation and Setup (Clinical Use) • 2-26</i>
<i>Instrument Installation and Setup (INDUSTRY Use) • 2-26</i>
BacT/ALERT® 3D Software • 2-27
<i>Operator Panel Screens • 2-27</i>
<i>Common Screen Elements • 2-28</i>

Introduction

The BacT/ALERT® 3D is the next generation of BacT/ALERT® instrumentation with comparable sensitivity and specificity to the BacT/ALERT® Classic systems. The system is non-invasive. A touch-activated operator panel allows for a text-free user interface to direct rapid loading and unloading of individual test samples.

Once placed in the unit, handling of a specimen/sample bottle is not required until a result is obtained. Immediately upon detection, positive results are indicated visually on the unit's monitor and, if desired, by an audible beep. If no microbial growth is present after a specified time, a specimen/sample is determined to be negative. The system will also indicate the negative samples that are ready for removal when prompted. Because the system handles bottles individually, testing of new specimens/samples may begin at any time. The system also utilizes barcode technology to assist in specimen/sample and data tracking.

The disposable culture bottles contain a liquid emulsion sensor that is monitored continuously using solid-state photodetectors. In addition, the bottles contain media and atmosphere which promote the recovery of a wide variety of microorganisms without venting.

Hardware Configuration

The BacT/ALERT® 3D utilizes a flexible modular design in one of two hardware configurations.

- A Controller Module that directs 1 to 6 Incubation Modules (each with 240 cells for bottle monitoring).
- A Combination Module that either stands alone (120 cells for bottle monitoring), or directs up to 3 Incubation Modules.

Software Configuration Options

Both the Controller and Combination Modules supervise the reading of the sensors and contain decision-making algorithms to determine which specimens/samples are positive or negative. Both modules can be arranged in one of three BacT/ALERT® 3D software configurations:

- **BacT/ALERT® 3D Select configuration** — The BacT/ALERT® 3D Controller or Combination Module is not connected to either a bioMérieux data management computer or a Laboratory Information System (LIS). Limited data management is available through the system.

- **BacT/ALERT® 3D SelectLink configuration** — The BacT/ALERT® 3D Controller or Combination module is connected directly to an LIS. Limited data management is available through the system.
- **BacT/ALERT® 3D Signature configuration** — The BacT/ALERT® 3D Controller or Combination Module is connected directly to a bioMérieux data management computer. Data generated by the BacT/ALERT® 3D is managed by extensive data management software produced by bioMérieux. This data management software provides a high level of flexibility. Data can easily be stored for a definable length of time, edited, queried, sorted, and reported. Predefined queries and reports are available, or they can be customized by the user. A detailed description of the data management software is found in the appropriate data management system operator manual.

The software configuration is listed at the top of all screens (except for the Edit Cell Contents or View Cell Contents screen). The background color will also indicate the configuration unless there is an error condition or a loaded positive bottle (see [Main Screen Introduction in Chapter 3](#), Clinical Use, and [Main Screen Introduction in Chapter 4](#), INDUSTRY Use):

- Blue — BacT/ALERT® 3D Select configuration
- Green — BacT/ALERT® 3D SelectLink configuration
- Gray — BacT/ALERT® 3D Signature configuration

21 CFR Part 11 and HIPAA

The BacT/ALERT® 3D Version B.30 or higher product release provides compatibility with 21 CFR Part 11 and Health Insurance and Portability and Accountability Act (HIPAA) requirements.

When installed, the BacT/ALERT® 3D instrument can be configured to operate in 21 CFR Part 11 mode. If 21 CFR Part 11 mode is enabled, you will need to enter a user name and password to access all functions available to the user. See [Log In/Out of System — 21 CFR Part 11 Mode in Chapter 10](#).

To meet HIPAA requirements, a password is required to view and print test data. These functions include access to the **Print**, **Report Label Entry**, **Report Configuration**, **Calibration Report**, and **Calibration History** buttons. These buttons are accessed via a Report Selection screen (see [Figure 3-11](#) for Clinical Use or [Figure 4-11](#) for INDUSTRY Use). See the topic, [Configuring Report Screens in Chapter 7](#) for further information.

Theory Of Operation

Principle of Detection

If microorganisms are present in the test sample, carbon dioxide is produced as the microorganisms metabolize the substrates in the culture medium. When growth of the microorganisms produces CO₂, the color of the sensor in the bottom of each culture bottle changes from dark to light.

A light-emitting diode (LED) projects light onto the sensor. The light reflected is measured by a photodetector. As more CO₂ is generated, more light is reflected. This information is compared to the initial sensor reading. If there is a high initial CO₂ content, an unusually high rate of CO₂ production, and/or a sustained production of CO₂, the sample is determined to be positive.

Mycobacterial growth in the BacT/ALERT® MP Bottles may also be determined positive by the delta or a slow sustained change of CO₂ production (Clinical Use).

If the CO₂ level does not change significantly after a specified number of days at optimal conditions, the sample is determined to be negative.



CAUTION: Unloading or manipulating the bottles when not indicated by the system may interfere with critical bottle readings.

Electrical Warnings

The BacT/ALERT® 3D has been designed and tested in accordance with the standards listed below and has been supplied in a safe condition. A CB Certification and Construction File have been established for the apparatus.

- UL 61010-1 (2nd Edition, 2001), *Safety Requirements for Electrical Equipment for Measurement, Control, and Laboratory Use; Part 1: General Requirements*
- IEC 61010-1 (2nd Edition, 2001), *Safety Requirements for Electrical Equipment for Measurement, Control and Laboratory Use*
- IEC 61010-2-081 (1st Edition, 2001), *Safety Requirements for Electrical Equipment for Measurement, Control, and Laboratory Use – Part 2-081: Particular requirements for automatic and semi-automatic laboratory equipment for analysis and other purposes*
- CAN/CSA-C22.2 No. 1010.1-92, *Safety Requirements for Electrical Equipment for Measurement, Control and Laboratory Use*

WARNING



The user must adhere to the following warnings to ensure safe operation and to maintain the apparatus in a safe condition:

- Ensure that the BacT/ALERT[®] 3D instrument is configured for the correct voltage at the instrument's power entry port before turning on.
- Intentional interruption of the protective conductor inside or outside the apparatus, or disconnection of the protective ground terminal, is prohibited.
- Disconnect the apparatus from all voltage sources before it is opened for any adjustment, replacement, maintenance, or repair.
- Do not perform any adjustments, maintenance, or repairs of the opened apparatus while under voltage. If this is unavoidable, maintenance must be carried out only by a skilled person who is aware of the hazard involved.
- Whenever it is likely that the BacT/ALERT[®] 3D module has been impaired, it should be rendered inoperative and secured against any unintended operation by disconnecting the power cord. If the presence of moisture is evident, turn off the machine at the breaker before removing the power cord and fuse.
- Do not forcibly remove the Zip[®] disk from the instrument. Forcibly removing the Zip[®] disk may cause damage to the Zip[®] disk or Zip[®] drive and may cause the system to lock up.

WARNING

Contact your local bioMérieux Representative if any of the following conditions occur:

- **System shows visible damage.**
- **System fails to perform intended measurements.**
- **System has been subjected to storage under unfavorable conditions (ex. above 90% humidity, extreme temperatures, dusty environment, prolonged storage).**
- **System has been subjected to severe transport stresses.**

Electrical Grounding

An electrical ground is required for this instrument. Before installing the instrument, ensure that a grounded wall receptacle is available for each module, which is factory equipped with a power supply cord that has the appropriate grounding plug. It must be plugged into a mating grounding type wall receptacle in accordance with the National Electrical Code and applicable local codes and ordinances for this type of installation (see [Figure 2-1](#))..

WARNING

Do not remove the power cord's ground prong under any circumstances.

WARNING

Do not insert any object, other than a Zip® disk, into the Zip® drive under any circumstances.

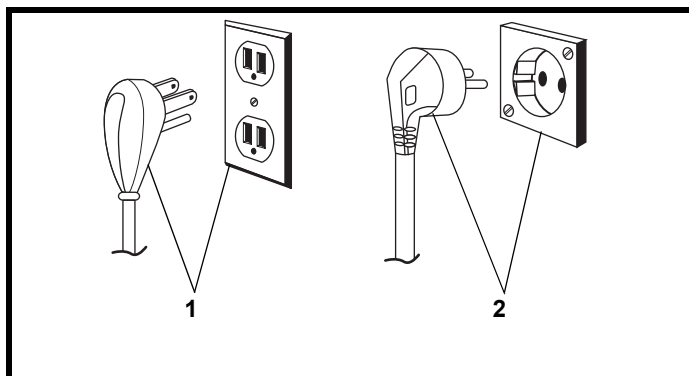


Figure 2-1: Electrical Grounding Requirements

1 — US Standard

2 — 220 Volt - European

Electrical and Electronic Recycling

WARNING



This statement only applies to European countries with regard to the waste electrical and electronic equipment European directive:

You can play an important role in contributing to reuse, recycling and other forms of recovery of waste electrical and electronic equipment. Sorting this type of waste significantly reduces potential negative effects on the environment and human health as a result of the presence of hazardous substances in electrical and electronic equipment.

At the end of the life cycle of this product, do not dispose of the product as unsorted municipal waste, even if it is decontaminated. It is imperative that you contact bioMérieux to assure for its appropriate disposal.

Fuse Replacement

There are no user-serviceable fuses in the BacT/ALERT® 3D system. Contact your local bioMérieux Instrument Service Representative.

BacT/ALERT® 3D Hardware

Each BacT/ALERT® 3D system includes either a Controller Module with at least one Incubation Module or a Combination Module. Each Controller Module can direct up to six Incubation Modules. Each Combination Module contains two incubation drawers in a right or left-handed configuration and can direct up to three additional Incubation Modules.

Incubation Modules are available in right and left-handed configurations. A right-handed Incubation Module is designed for placement on the right side of the Controller or Combination Module and allows bottle access from the left of an open drawer. Incubation Modules contain four drawers, each with the capacity to hold up to 60 culture bottles. An Incubation Module can be configured so one or more drawers are empty, allowing a 60, 120, 180, or 240-bottle capacity.

Mycobacteria Drawers (Clinical Use)

Mycobacteria (MB) and non-MB drawers can co-exist in the same Incubation Module. MB drawers may be provided directly by bioMérieux or may be converted from a non-MB configuration in the field. This is accomplished by disengaging the agitation mechanism so the three racks within each drawer remain immobile.

Each MB drawer handle has a red MB label also displaying the Incubation Module number and letter corresponding to that drawer. The MB drawer faces on the Instrument icon of the Main screen are also labeled.

Note: *The BacT/ALERT® 3D software must also be configured to activate the MB status of each drawer.*

Controller Module

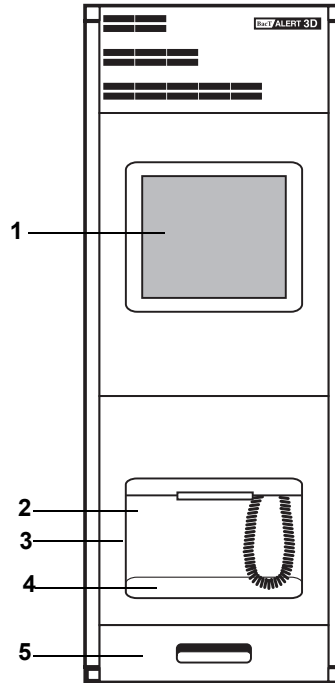


Figure 2-2: Instrument Front View

- 1 — Operator Panel**
- 2 — Barcode Reader**
- 3 — Backup Drive (Located in Barcode Reader Aperture)**
- 4 — Barcode Reader Strip**
- 5 — Keyboard Drawer**

Operator Panel

Displays bottle and system information. Includes a touchscreen overlay so that the operator may input selections and data by touching the screen.

Barcode Reader Aperture

Houses a removable Barcode Reader that is used to scan bottle barcode labels to identify bottles and accession/sample barcodes when loading or unloading. The reader can be removed, if desired.

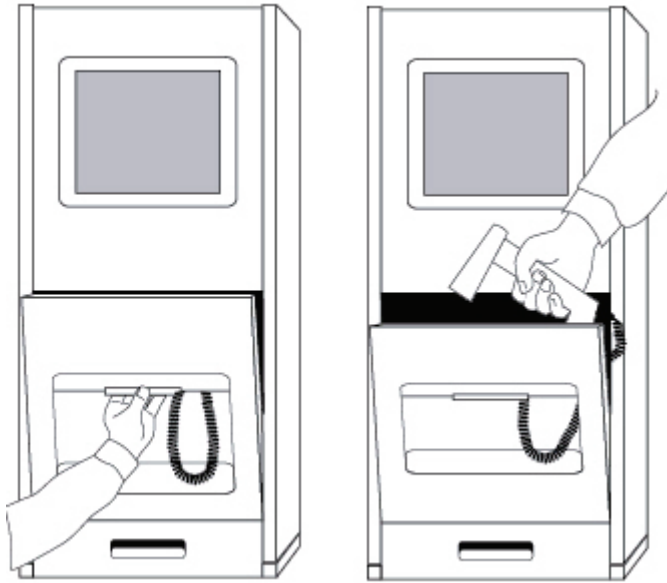


Figure 2-3: Removing the Barcode Reader Through the Barcode Reader Aperture

Backup Drive

Used for system backups and software upgrades. May be either a Zip® Drive or a USB Port, depending on system configuration. Located in the Barcode Reader Aperture.

Keyboard Drawer

Houses a pullout working surface, Quick Reference Card, and keyboard.

Quick Reference Card

Describes specific error codes. Located in the keyboard drawer beneath the Barcode Reader Aperture. Located at the bottom of the Controller Module.

Keyboard

Provides an alternate means of input. Serves as a backup input device should the Touchscreen or Barcode Reader fail.

Accessing the Controller Module Keyboard



There are two different designs of keyboard drawers, Design A and Design B (see [Figure 2-4](#)).

To access the keyboard in a drawer of Design A:

- 1) Pull the keyboard drawer on the bottom of the Controller Module face out as far as it will go. A cover is exposed which includes a pocket for the Reference Card.
- 2) Grasp the sides of the exposed cover nearest the face of the Controller Module.
- 3) Press the metal latches on each side of the drawer to extend fully.
- 4) Extend the drawer fully, and swivel the cover up.
- 5) Remove the keyboard, close the cover, and rest the keyboard on the working surface.

To access the keyboard in a drawer of Design B:

- 1) Pull the keyboard drawer on the bottom of the Controller Module face out as far as it will go. A cover is exposed which includes a pocket for the Reference Card.
- 2) Grasp the sides of the exposed cover nearest you.
- 3) Press the metal latches on each side of the drawer to extend fully.
- 4) Swivel the cover up from the front to expose the keyboard.
- 5) Remove the keyboard, close the cover, and rest the keyboard on the working surface.

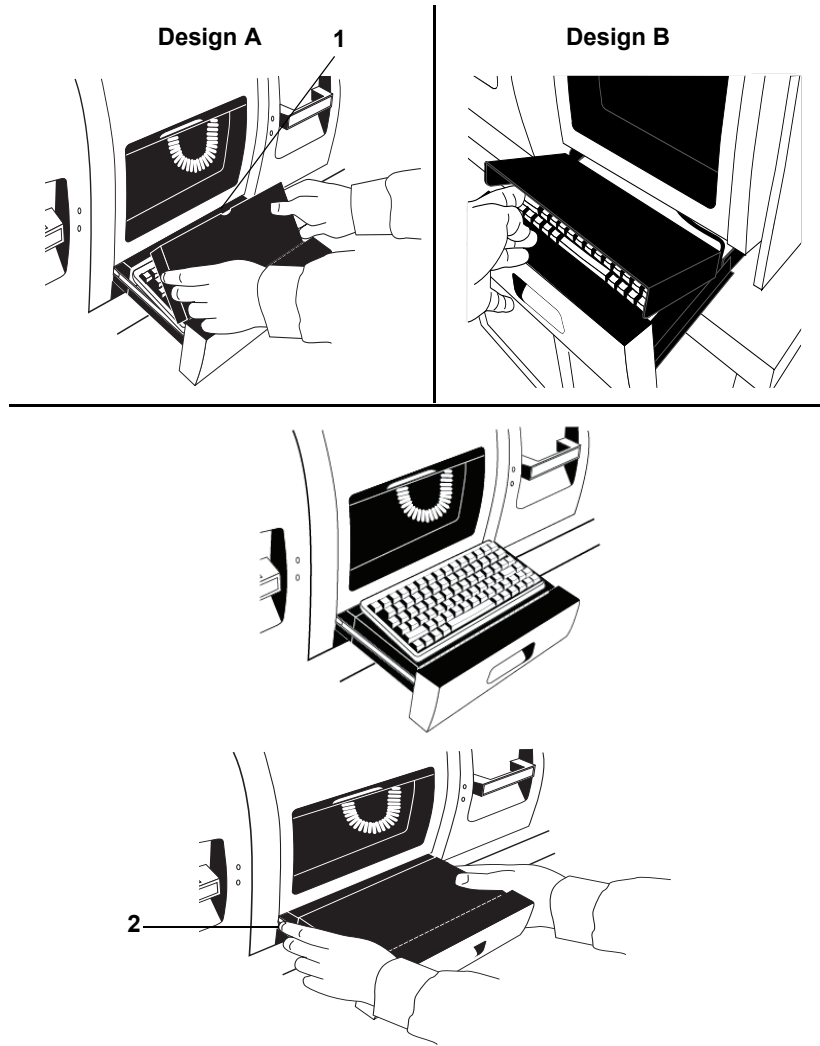


Figure 2-4: Accessing the Keyboard and Reference Card

1 — Notch for cord

2 — Press latch on each side to close

When keyboard entry is complete, the keyboard may be returned to its original position beneath the cover plate. Close the drawer by depressing both latches on each side of the drawer while pushing it closed.

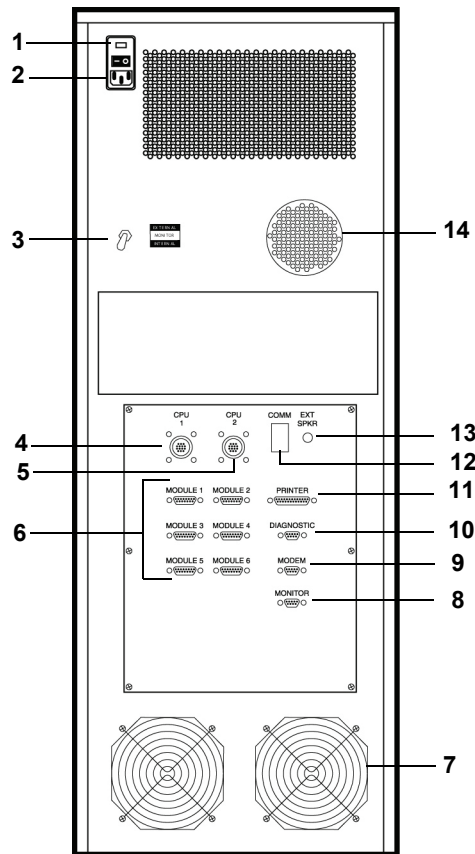


Figure 2-5: Back View Controller Module

- | | |
|-----------------------------|---------------------------------|
| 1 — Power Switch | 8 — Monitor Port |
| 2 — Power Connector | 9 — Modem Port |
| 3 — Display Switch Assembly | 10 — LIS Port (Diagnostic Port) |
| 4 — CPU 1 Port | 11 — Printer Port |
| 5 — CPU 2 Port | 12 — Comm Port |
| 6 — Module Ports (6) | 13 — External Speaker Port |
| 7 — Fans | 14 — Audible Alarm Speaker |

Printer Port

Interface port (parallel) for producing hard copy reports when the instrument is not in the BacT/ALERT® Signature configuration.

Audible Alarm Speaker

Specified by the user to flag instrument failures, operator errors, and positive bottle results.

External Speaker Port

Port for connecting external speakers that are available for order as a separate kit.

Power Connector

Connector for alternating current (AC) power cord.

Power Switch

Turns the AC Power to the instrument On and Off.

Display Switch Assembly

Used to select the internal or external display device.

CPU 1 Port

Used to connect the bioMérieux data management computer to the Controller Module (or Combination Module) or to connect two Controller Modules together.

CPU 2 Port

Used to connect two Controller Modules (or Combination Modules) together.

Module Ports (6)

Connects an Incubation Module to the Controller Module.

Monitor Port

Used to connect an external monitor, if necessary.

Modem Port

Used to connect an external modem to the Controller Module. Allows remote diagnosis of instrument problems. The external modem box should be in the "OFF" position when not in use by a bioMérieux Representative.

LIS Port (Diagnostic Port)

Used to connect the Controller Module to a Laboratory Information System (LIS). See [Figure 2-5, Back View Controller Module, on page 2-13](#).

COMM Port

Reserved for future use.

Controller Module Specifications

Electrical Power Services Requirements

- 100/120 Volts (50-60 Hz)
- 220/240 Volts (50-60 Hz)

Power Consumed in Watts

- @115 Volts (72 Watts typical)
- @230 Volts (72 Watts typical)

Heat Dissipated

- 245 BTU/Hr. 2840 maximum

Sound Emission

- 46.4 dB

Instrument Dimensions

- Width — 14 inches (35.6 cm)
- Height — 36 inches (91.4 cm)
- Depth — 24.3 inches (61.7 cm)
- Weight — 91 pounds (57.2 Kg)

Controller Module Environmental Requirements

Operating Temperature Range

- 10°C to 30°C (50°F to 86°F)

Storage Temperature Range

- -17°C to 57°C (0°F to 135°F)

Operating Humidity Range

- 10% to 90% relative humidity, non-condensing.

Storage Humidity Range

- 10% to 90% relative humidity, non-condensing.

Maximum Operating and Storage Altitude

- 6562 feet (2000 meters)

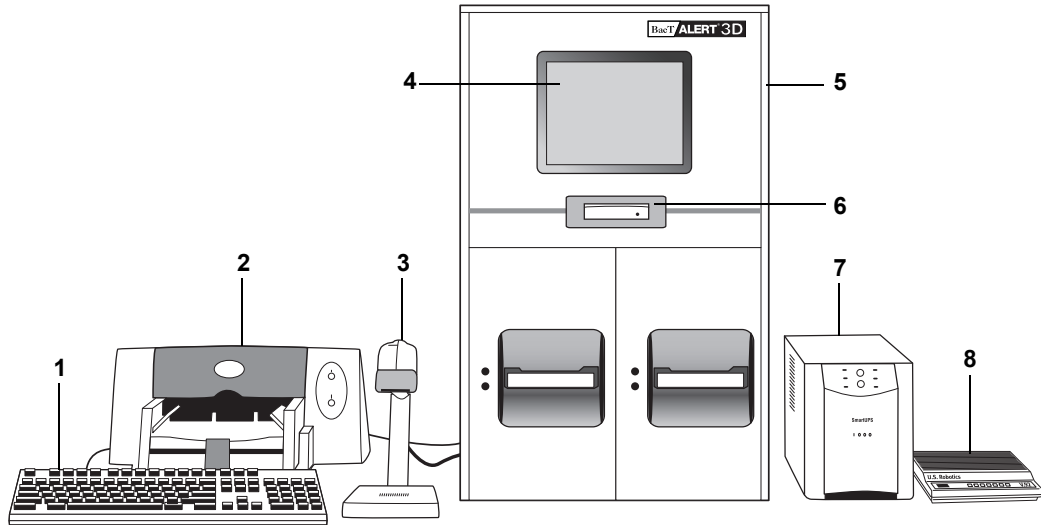
*Pollution degree 2 in accordance with IEC 664**Overvoltage Category II per IEC 664***Combination Module**

Figure 2-6: Front View Combination Module

- 1 — Keyboard**
- 2 — Printer**
- 3 — Barcode Reader**
- 4 — Operator Panel**
- 5 — Combination Module**
- 6 — Backup Drive**
- 7 — UPS**
- 8 — Modem**

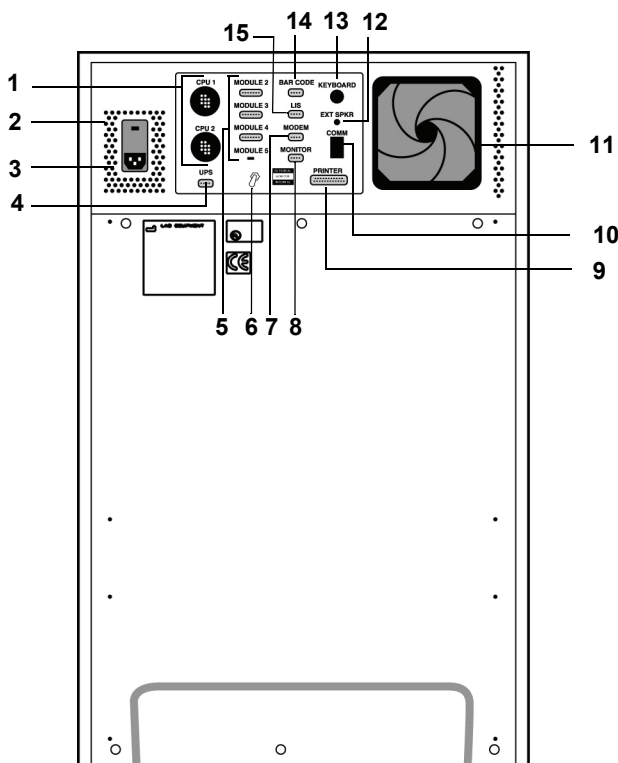


Figure 2-7: Back View Combination Module

- | | |
|-----------------------------|----------------------------|
| 1 — CPU 1 and CPU 2 Ports | 9 — Printer Port |
| 2 — Power Switch | 10 — COMM Port |
| 3 — Power Connector | 11 — Fan |
| 4 — UPS Port | 12 — External Speaker Port |
| 5 — Module 2 – 5 Ports | 13 — Keyboard Port |
| 6 — Display Switch Assembly | 14 — Barcode Reader Port |
| 7 — Modem Port | 15 — LIS Port |
| 8 — Monitor Port | |

Operator Panel

Provides a way of displaying information. Includes a touchscreen overlay so that the operator may input selections and data by touching the screen.

Barcode Reader

The Barcode Reader is external to the Combination Module. The Barcode Reader, used to scan bottle barcode labels to identify bottles and accession/

sample barcodes when loading or unloading, is on a stand separate from the Combination Module.

Keyboard

The keyboard is external to the Combination Module. It provides an alternate means of input. Serves as a backup input device should the Touchscreen or Barcode Reader fail.

UPS

The UPS is external to the Combination Module.

Backup Drive

The Backup Drive allows system backups to be made to either a Zip® disk or USB Flash Drive.

Drawer

Labeled A or B in the Combination Module. Each drawer contains 3 racks for a drawer capacity of 60 culture bottles.

Rack

Each rack contains 20 cells.

Cell

Cells are designated with a number from 1 to 60. Each cell holds and monitors one culture bottle.

Cell Indicator Lamp/Cell Indicator Light

Located next to each cell in the rack.

- Illuminates to indicate which cells are available for loading after the operator selects the Load Bottles function.
- Illuminates to indicate which cells should be unloaded when the operator selects one of the Unload Bottles functions.
- Blinks slowly or rapidly to indicate whether a bottle load or unload was performed correctly or incorrectly, respectively.

Cell Flag

Secures bottles in cells and aids in cell diagnostics and bottle loading and unloading determinations.

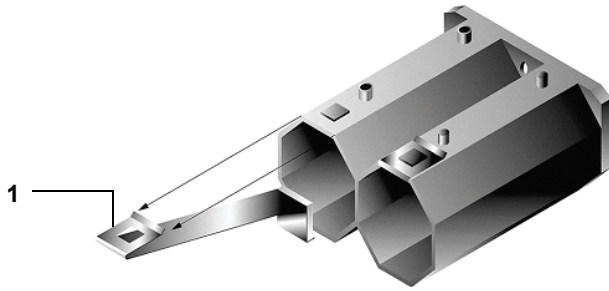


Figure 2-8: Cell Flag (Clip Removed)

1 — Clip

Thermometer (Not Shown)

Located in back of drawer.

Drawer Indicator Light — Yellow

- Illuminates when the drawer is open and turns off when the drawer is closed.
- Flashes if the drawer remains open too long or if an error condition is associated with the drawer.

Drawer Indicator Light — Green

- Illuminates if the drawer or its cells are involved in an operation the user has selected.
- Flashes in conjunction with the yellow indicator if a drawer is left open too long.

Printer Port

Interface port (parallel) for producing hard copy reports when the instrument is not in BacT/ALERT® 3D Signature configuration.

External Speaker Port

Port for connecting external speakers. External speakers are available for order as a separate kit.

Power Connector

Connector for Alternating Current (AC) power cord.

Power Switch

Turns the AC power to the Combination Module on and off.

Display Switch Assembly

Used to select the internal or external display device.

CPU 1 Port

Used to connect the bioMérieux data management computer to the Combination Module or to connect to a Controller Module.

CPU 2 Port

Used to connect to a Controller Module (or Combination Module).

Module Port (3)

Connects an Incubation Module to the Combination Module.

Modem Port

Used to connect an external modem to the Combination Module. Allows remote diagnosis of instrument problems. The external modem box should be in the "OFF" position when not in use by a bioMérieux Representative.

LIS Port

Used to connect the Combination Module to a Laboratory Information System (LIS).

Barcode Scanner Port

Used to connect the barcode scanner.

Keyboard Port

Used to connect the keyboard.

Monitor Port

Used to connect an external monitor, if necessary.

COMM Port

Reserved for future use.

Combination Module Specifications

Electrical Power Services Requirements

- 100/120 Volts (50/60 Hz)
- 220/240 Volts (50/60 Hz)

Power Consumed in Watts

- @115 Volts — 256 Watts (idle), 640 Watts typical
- @230 Volts — 265 Watts (idle), 640 Watts typical

Heat Dissipated

- 904 BTU/Hr. 2840 maximum

Combination Module Dimensions

- Width — 19.5 inches (49.6 cm)
- Height — 30.8 inches (78.1 cm)
- Depth — 24.5 inches (62.2 cm)
- Unloaded Weight — 200 pounds (90.7 Kg)
- Loaded Weight — 216.5 (98.2Kg)

Sound Emission

- 55.5 dB

Combination Module Environmental Requirements*Operating Temperature Range*

- 10°C to 30°C (50°F to 86°F)

Storage Temperature Range

- -17°C to 57°C (0°F to 135°F)

Operating Humidity Range

- 10% to 90% relative humidity, non-condensing.

Storage Humidity Range

- 10% to 90% relative humidity, non-condensing.

Maximum Operating and Storage Altitude

- 6562 feet (2000 meters)

*Pollution degree 2 in accordance with IEC 664 (Clinical Use)**Overvoltage Category II per IEC 664 (Clinical Use)*

Incubation Module

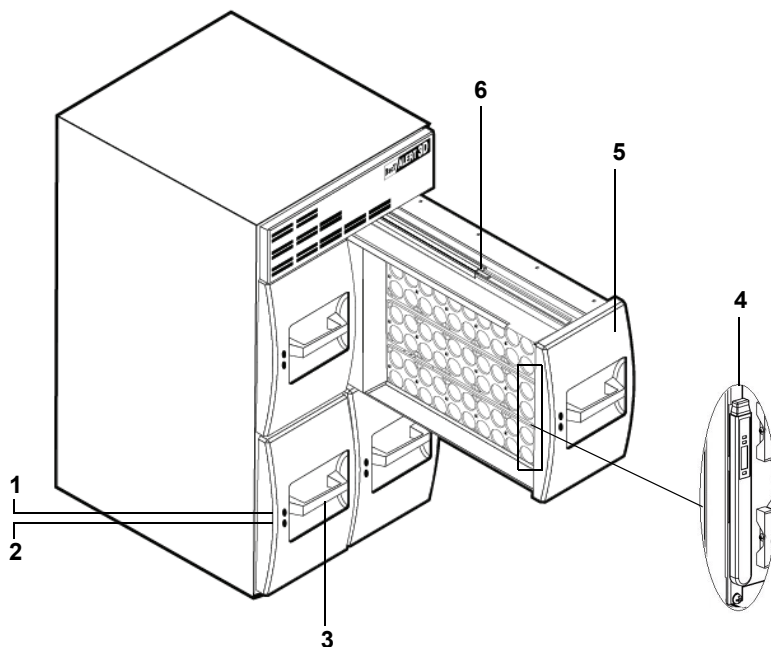


Figure 2-9: Front View Incubation Module (Right-Handed Configuration)

- 1 — Green Indicator Lamp**
- 2 — Yellow Indicator Lamp**
- 3 — Drawer Label**
- 4 — Digital Thermometer (located on the inner front panel of drawer B in right-hand module and drawer A of left-hand module)**
- 5 — Drawer**
- 6 — Drawer Release Latch**

Drawer

Labeled A, B, C, or D in each Incubation Module. Contains 3 racks for a drawer capacity of 60 culture bottles.

Rack

Each rack contains 20 cells.

Cell

Designated with a number from 1 to 60. Each cell holds and monitors one culture bottle.

Cell Indicator Lamp/Cell Indicator Light

Located next to each cell in the rack.

- Illuminates to indicate which cells are available for loading after the operator selects the Load Bottles function.
- Illuminates to indicate which cells should be unloaded when the operator selects one of the Unload Bottles functions.
- Blinks slowly or rapidly to indicate whether a bottle load or unload was performed correctly or incorrectly respectively.

Cell Flag

Secures bottles in cells and aids in cell diagnostics and bottle loading and unloading determinations (see [Figure 2-8, Cell Flag \(Clip Removed\)](#), on [page 2-19](#)).

Drawer Indicator Light — Yellow

- Illuminates when the drawer is open and turns off when the drawer is closed.
- Flashes if the drawer remains open too long or if an error condition is associated with the drawer.

Drawer Indicator Light — Green

- Illuminates if the drawer or its cells are involved in an operation the user has selected.
- Flashes in conjunction with the yellow indicator if a drawer is left open too long.

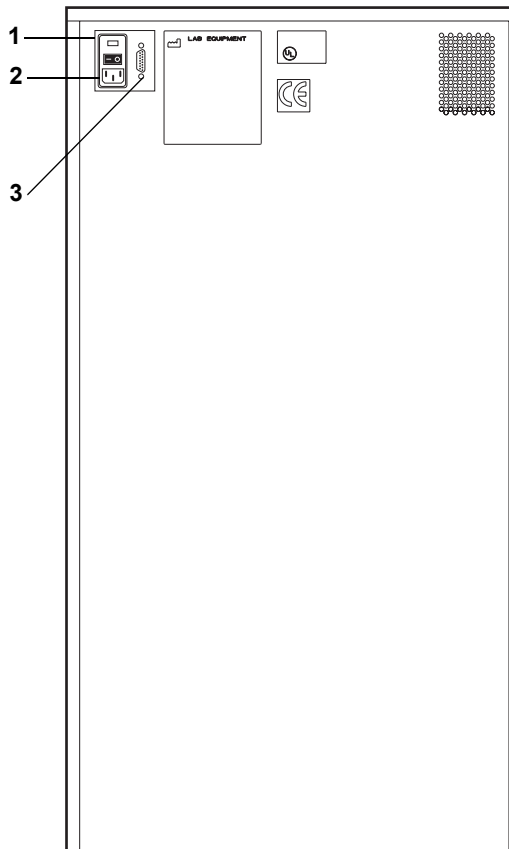


Figure 2-10: Back View Incubation Module

- 1 — Power Switch**
- 2 — Power Connector**
- 3 — Controller Module Port**

Controller Module Port

Connects the Controller Module to an Incubation Module.

Power Connector

Connector for AC power cord.

Power Switch

Turns the AC power of the Incubation Module On and Off.

Incubation Module Specifications

Electrical Power Services Requirements

- 100/120 Volts (50/60Hz)
- 220/240 Volts (50-60 Hz)

Power Consumed in Watts

- @115 Volts — 256 Watts (idle), 640 Watts typical
- @230 Volts — 265 Watts (idle) 640 Watts typical

Heat Dissipated

- 904 BTU/Hr. 2840 maximum

Instrument Dimensions

- Width — 19.5 inches (49.6 cm)
- Height — 36 inches (91.4 cm)
- Depth — 24.3 inches (61.7 cm)
- Unloaded Weight — 262 pounds (118.8 Kg)
- Loaded Weight — 307 (133.8 Kg)

Sound Emission

54.7 dB

Incubation Module Environmental Requirements

Operating Temperature Range

- 10°C to 30°C (50°F to 86°F)

Storage Temperature Range

- -17°C to 57°C (0°F to 135°F)

Operating Humidity Range

- 10% to 90% relative humidity, non-condensing.

Storage Humidity Range

- 10% to 90% relative humidity, non-condensing.

Maximum Operating and Storage Altitude

- 6562 feet (2000 meters)

Instrument Installation and Setup (Clinical Use)



CAUTION: Initial installation and setup of the BacT/ALERT® 3D Microbial Detection System, or of any added Incubation Module(s) is to be performed only by a bioMérieux Service Representative.

WARNING



Clinical Use: The BacT/ALERT® 3D has been designed to minimize risks associated with MB testing. However, to further reduce the risks of accidental exposure to infectious agents, additional precautions should be taken. It is strongly recommended that the instrument be placed in a laboratory used for the routine culture of *M. tuberculosis*. For activities involving the propagation and manipulation of *M. tuberculosis* or *Mycobacterium* species grown in culture, Biosafety Level 3 Practice, Containment Equipment, and Facilities are required as recommended by CDC and NIH guidelines.

At a minimum, the instrument should be placed in a contained environment with controlled access which has a tuberculosis exposure control plan. The locations should have surfaces which can be easily decontaminated using an appropriate topical disinfectant. The instrument must not be placed in an open corridor or hallway that is accessible to the general public or the patient population.

If it is necessary to move or store the BacT/ALERT® 3D System or one of its modules, contact bioMérieux Service for assistance.

Instrument Installation and Setup (INDUSTRY Use)



CAUTION: Initial installation and setup of the BacT/ALERT® 3D Microbial Detection System, or of any added Incubation Module(s), is to be performed only by your local bioMérieux Representative.

If it is necessary to move or store the BacT/ALERT® 3D System or one of its modules, contact your local bioMérieux Representative for assistance.

BacT/ALERT® 3D Software

Operator Panel Screens

The Main screen is normally visible on the Operator Panel, but other screens may be displayed to perform a variety of functions. Each screen has a screen ID number in the upper left-hand corner to cross-reference it to the following descriptions and instructions in this Manual. When selected, the Setup screen replaces the Main screen.

Screen ID Numbers

- 1.0 Main screen
- 1.1 View Cell Status screen
- 1.2 Change Maximum Test Time screen
- 1.3 User Login screen (21 CFR Part 11 mode only)
- 2.0 Setup screen
- 2.1 Set Date/Time screen
- 2.2 Enable/Disable Module, Drawer, Rack, or Cell screen
- 2.3 Calibrate Module Temperature screen
- 2.4 Calibrate Cell screen
- 2.4.1 Cell Calibration Report Screen
- 2.7 Set Maximum Test Time screen
- 2.8 Set Audible Alarm Options screen
- 2.9 Change Password screen
- 2.11 Select Bottle to Edit/Graph screen
- 2.11.1 Edit Bottle Detail screen
- 2.11.1.1 Edit Test Result
- 2.11.2 Graph Bottle Readings screen
- 2.11.2.1 Bottle Readings screen
- 2.12 Edit Cell Contents screen (2.11.1, 2.11.1.1, 2.11.2 and 2.11.2.1 can also be accessed from this screen)
- 2.13 View Incubation Module Information screen
- 2.14 Report Label Entry screen *
- 2.15 Report Configuration screen *
- 2.15.1 Report screen *
- 2.16 Backup Management screen
- 2.17 Edit Data Relationships screens
- 2.19 Viewing and Printing Calibration data screen
- 2.20 Bottle Type Customization screen
- 2.21 Report Selection screen*
- 2.21.1 Calibration History screen
- 2.23 User Configuration screen (21 CFR Part 11 mode only)
- 2.23.1 Add User screen (21 CFR Part 11 mode only)
- 2.23.2 Delete User screen (21 CFR Part 11 mode only)

- 2.23.3 Change User Password screen (21 CFR Part 11 mode only)
- * Screen's availability dependent upon BacT/ALERT® 3D software configuration.

Common Screen Elements

Icon

- Graphic symbols used instead of text to convey information and concepts in the Operator Panel.

Button

- Appear as rectangular shapes which can be pressed to input choices or activate functions.
- Button function is indicated by icon displayed on button face.
- If a button is gray, then it is disabled and its associated function is unavailable.

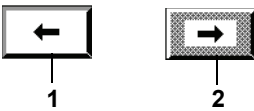


Figure 2-11: Button Examples

- 1 — Enabled Button
- 2 — Disabled Button

Common System Buttons

Table 2-1: Common System Buttons

	Check button – Accept changes, save data or entries on that particular screen.
	Cancel button – Discard changes or entries on that particular screen to keep the original values.
	Previous Screen Button – Return to the previous screen.
	Next Screen Button – Move to the next available screen.

Scroll Button

- Used for numeric entry and symbolic or media type selection.
- Consists of two small buttons and an area between them to display the current value.
- Touching the top or bottom button of the scroll button displays the next higher or lower value.
- Adjacent scroll buttons provide entry for multi-digit values.

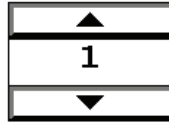


Figure 2-12: Scroll Button

Slidebar Switch

- Turns a function On or Off. The specific function is represented by a separate icon next to the slidebar.
- Touching the right half of the icon moves the slidebar to the right, indicating an On selection (1).
- Touching the left half of the icon moves the slidebar to the left, indicating an Off selection (0).



Figure 2-13: Slidebar Switch

Anchor Display/Scroll Buttons

- **Anchor Display** top or bottom buttons — Control whether the display stays anchored on the first or last line of the diagnostic output.
- **Line** scroll up or down buttons — Move the display area one line at a time (up or down). This function can also be performed by pressing the ↑ or ↓ key on the keyboard.
- **Page** scroll up or down buttons — Move the display area one page (up or down). This function can also be performed by pressing the **Page Up** or **Page Down** key on the keyboard.
- **Home/End** scroll buttons — Position the display area to the first or last line of the diagnostic output. This function can also be performed by pressing the **Home** or **End** key on the keyboard.

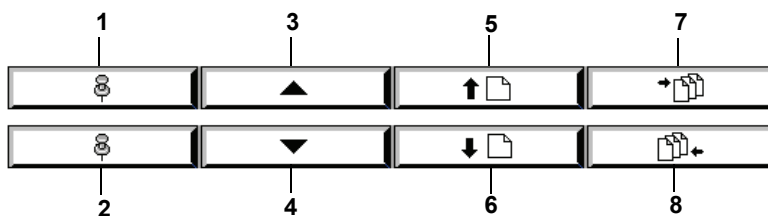


Figure 2-14: Anchor Display/Scroll Buttons

- 1 — Anchor Display Top Button
- 2 — Anchor Display Bottom Button
- 3 — Line Scroll Up Button
- 4 — Line Scroll Down Button
- 5 — Page Scroll Up Button
- 6 — Page Scroll Down Button
- 7 — Home Scroll Button
- 8 — End Scroll Button

Text Entry Field

A text entry field appears as a rectangular box that allows the user to enter text either manually via the keyboard or by scanning a barcode (see [Text/Data Entry in Chapter 3](#)). The **Bottle ID** field on the Main screen is an example of a text field (see [Figure 3-1, Main Screen](#)).

About This Chapter

This chapter introduces you to the Main screen and provides you with procedures on how to perform daily basic functions for clinical use.

Chapter Contents

- Introduction • 3-2
- Monitoring the System • 3-3
 - Main Screen Introduction • 3-3*
 - Viewing Faults • 3-6*
 - Viewing the Cell Status Screen • 3-7*
- Text/Data Entry • 3-9
 - Common Text Fields and Field Limit • 3-9*
 - Using the Barcode Scanner to Enter Data • 3-10*
 - Manually Entering Text into a Data Entry Field (Keyboard) • 3-10*
- Loading Bottles • 3-11
 - Loading Bottles • 3-11*
 - Changing the Maximum Test Time — Individual Bottles • 3-15*
 - Handling Anonymous Bottles • 3-16*
- Unloading Bottles • 3-16
 - Unloading Bottles • 3-17*
 - Handling Unconfirmed Positive Bottles (False Positives) • 3-19*
- Accessing the Setup Screen Function Buttons • 3-20
 - Accessing the Setup Screen • 3-20*
 - Setup Screen Function Buttons • 3-21*
- Viewing and Printing • 3-23
 - Introduction • 3-23*
 - Viewing, Printing, and Exporting Test Data • 3-24*
 - Viewing and Printing Bottle Graphs • 3-31*
 - Display Bottle Readings • 3-34*
- Sending/Requesting LIS Information • 3-39
 - Sending Results to the LIS • 3-39*
 - Requesting Information from the LIS • 3-39*

Introduction

The basic functions are those tasks that may be performed during daily workflow. They include:

- Monitoring system
- Entering data (where applicable)
- Loading and unloading bottles
- Viewing and printing data (printing with the BacT/ALERT® 3D Signature is performed at the data management computer.)
- LIS interaction with the BacT/ALERT® 3D SelectLink
- Accessing the Setup screen

Monitoring the System

Main Screen Introduction

The BacT/ALERT® 3D system can be monitored from the Main screen (see [Figure 3-1](#)).

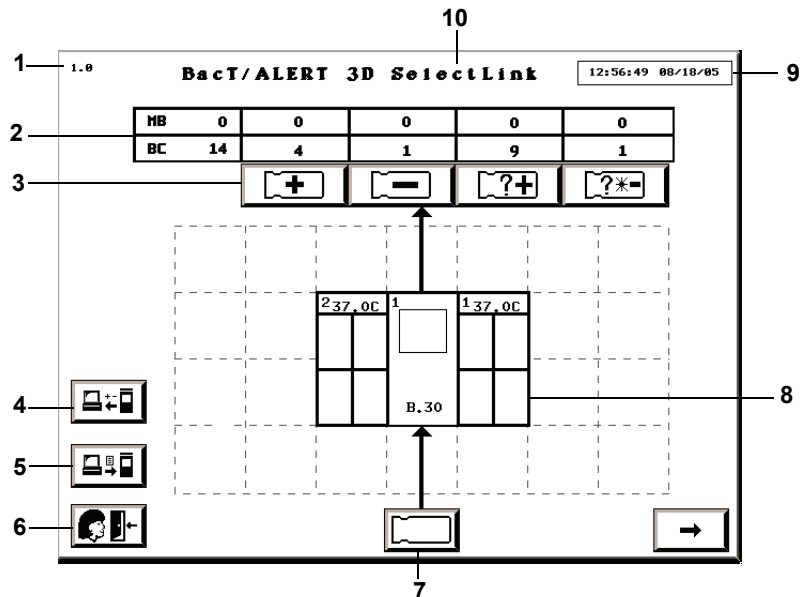


Figure 3-1: Main Screen

- 1 — Screen ID Number
- 2 — Bottle Count Table
- 3 — Unload Buttons
- 4 — Manual Send Test Results button (SelectLink only)
- 5 — Manual Request Test Orders button (SelectLink only)
- 6 — Logout button (21 CFR Part 11 mode only)
- 7 — Load Bottles button
- 8 — Instrument Icon
- 9 — Current Date/Time
- 10 — Software Configuration



CAUTION: If the current date/time displayed at the top of the screen does not advance, contact bioMérieux Customer Service immediately.

Background Color

The default background color is determined by the software configuration (see [Software Configuration Options in Chapter 2](#)). The following conditions will override default background colors:

- A yellow screen indicates that the instrument has detected a positive bottle.
- A red screen indicates that an instrument fault has occurred. Touching the screen or pressing any key on the keyboard returns a red screen to yellow or the configuration default color depending on whether positive bottles are present. The fault code will remain on the screen until the error is cleared.

Instrument Icon

The following information is indicated on the **Instrument** icon:

- System ID numbers are assigned to both the Controller/Combination Module and Incubation Modules.
- Combination Module has 2 System ID numbers.
- The programmed optimal temperature (°C) is displayed for each Incubation Module.
- Software version number for the BacT/ALERT® 3D Controller/Combination Module.
- Disabled or uninstalled components appear diagonally striped in gray on the screen.
- An entire rack will appear diagonally striped if only one of its cells is disabled.

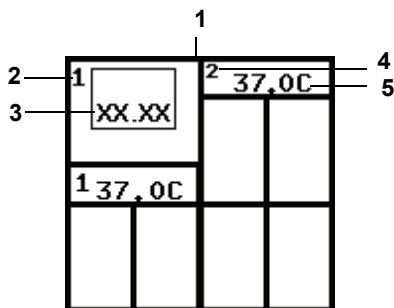


Figure 3-2: Combination Module with an Additional Incubation Module

- 1 — Combination Module Instrument Icon
- 2 — Combination Module ID
- 3 — Software Version
- 4 — Incubation Module ID
- 5 — Optimal Temp (°C)

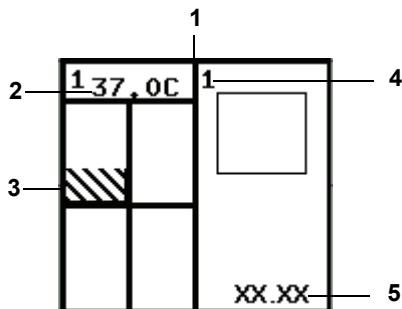


Figure 3-3: Controller Module with One Incubation Module

- 1 — Controller Module Instrument Icon
- 2 — Optimal Temp (°C)
- 3 — Disabled Rack or Cell
- 4 — Controller ID
- 5 — Software Version

Bottle Count Table

Located just above the **Instrument** icon are the **Unload** buttons and a bottle count table indicating the number of bottles of each type currently loaded in the instrument.

1 —	MB	1	1	0	0	0
	BC	14	4	1	9	1
			+	-	?+	?*-
			4	6	8	10

Figure 3-4: Bottle Count Table/Unload Buttons

- 1 — Total number of mycobacteria (MB) bottles loaded in the system.
- 2 — Total number of blood or sterile fluid culture (BC) bottles loaded in the system.
- 3 — Total number of identified bottles with a positive test status.
- 4 — Unload Positive Identified Bottles Button
- 5 — Total number of bottles (identified and anonymous) with a negative test status.
- 6 — Unload Negative Bottles Button
- 7 — Total number of anonymous bottles with a positive test status.
- 8 — Unload Positive Anonymous Bottles Button
- 9 — Total number of anonymous bottles with a negative-to-date or negative test status.
- 10 — Unload Anonymous Negative or Negative-to-Date Bottles Button

Viewing Faults



Instrument faults are reported using a numeric code within a red diamond shape. Fault codes are displayed on the **Instrument** icon where the fault condition exists.

Note: Only high priority codes are displayed on the **Instrument** icon.

- 1) If the code appears on the Controller Module or the top half of the Combination Module, then move to [Step 4](#). If the fault code appears on an Incubation Module or bottom half of the Combination Module, then continue to [Step 2](#).
- 2) Touch the Incubation or Combination Module on the **Instrument** icon that contains the fault code.
- 3) The View Cell Status screen appears (see [Viewing the Cell Status Screen on page 3-7](#) and [Figure 3-5](#)).
 - Drawer faults appear at the top of the screen.
 - Rack faults appear at the left end of the rack display.

- Cell faults appear in the cell display.
- 4) For a complete list and description of Instrument Fault Codes, see the topic, [Instrument Fault Codes in Chapter 9](#).

Viewing the Cell Status Screen



The View Cell Status screen is continuously updated with changes that occur while the screen is active, such as the loading/unloading of bottles, new test results, and the error status of the drawer, as indicated by fault codes appearing or disappearing. To view the cell status screen:

- 1) Display the View Cell Status screen by touching the appropriate Incubation Module on the Instrument icon (see [Figure 3-1, Main Screen, on page 3-3](#)).

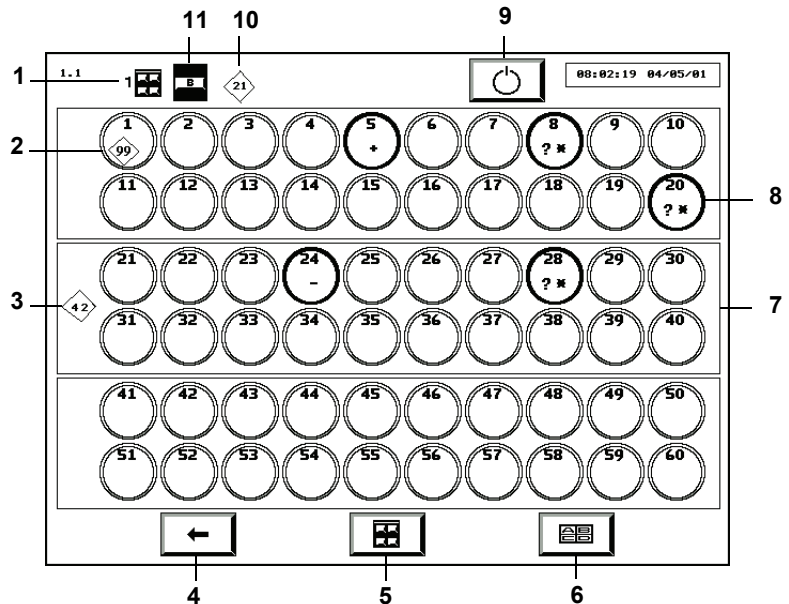


Figure 3-5: View Cell Status Screen

- | | |
|----------------------------------------|--------------------------------------|
| 1 — Incubation Module Indicator | 6 — Drawer Selection Button |
| 2 — Cell Fault Code | 7 — Rack |
| 3 — Rack Fault Code | 8 — Cell |
| 4 — Previous Screen Button | 9 — Restart Incubation Module Button |
| 5 — Incubation Module Selection Button | 10 — Drawer Fault Code |
| | 11 — Drawer Indicator |

- 2) If necessary, use the Incubation Module Selection and Drawer Selection buttons to display the desired drawer.

Note: *If a drawer is not installed, it is not displayed.*

- 3) Press the **Previous Screen** button to return to the Main screen.

Understanding the View Cell Status Screen Display

For each cell, the cell identification number appears at the top of the circle.

A hollow circle indicates an empty cell, and a solid circle indicates a loaded cell. The color of the solid circle indicates the bottle status or a cell that is pending quality control check.

- Black — negative-to-date bottle
- Green — negative bottle
- Yellow — positive bottle
- White — cell is pending quality control check

Loaded cells also contain symbols to indicate the bottle's status:

- | | |
|-----|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| + | Positive |
| — | Negative |
| * | Negative-to-date |
| ~ + | Critical determination in progress. (Represents a bottle that is presently undergoing a critical determination as to whether it will turn positive or remain negative or negative-to-date.) |

Note: *Bottles with a critical determination in progress status will be temporarily removed from the bottle count table on the Main screen.*

Cells loaded with an anonymous bottle contain a ?. If a loaded cell contains no ?, then the bottle is identified.

Disabled racks and cells are indicated by gray diagonal stripes within their borders. Disabled drawers are indicated by gray diagonal stripes within the drawer indicator (see [Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#)).

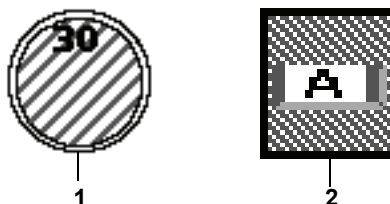


Figure 3-6: Disabled Cell and Drawer






1 — Disabled Cell

2 — Disabled Drawer

Note: If a drawer is disabled, its racks and cells are disabled and turn gray.

Text/Data Entry

Common Text Fields and Field Limit

 <input type="text"/>	Bottle ID field
 <input type="text"/>	Patient First Name field — may contain up to 20 characters
 <input type="text"/>	Patient Last Name field — may contain up to 31 characters
 <input type="text"/>	Accession Number field — may contain up to 16 characters
 <input type="text"/>	Hospital ID field — may contain up to 22 characters

Note: The field length and initial character type (alpha, numeric, or other) can be configured for the **Accession Number** field. The initial character type can be configured for the **Bottle ID** field. To configure these fields, contact your local bioMérieux Representative.

Note: The **Hospital ID**, **Patient First Name**, and **Patient Last Name** fields do not display with the BacT/ALERT® 3D Signature configuration.

Note: All fields except the **Bottle ID** field may be hidden or disabled regardless of software configuration. To hide or disable a field, contact your local bioMérieux Representative.

Note: To change the order of the patient fields, contact your local bioMérieux Representative.

Using the Barcode Scanner to Enter Data



To scan a bottle or accession barcode:

- 1) Before scanning the barcode, touch the desired field to place focus on that field. The field should turn white indicating focus.
- 2) Rotate the bottle so the bottle ID or accession number barcode is on top.
- 3) Place the bottle over the barcode strip located in the Barcode Reader Aperture below the Control Panel of the Controller Module (see [Figure 2-2, Instrument Front View](#)) or at the base of the barcode reader stand with the Combination Module (see [Figure 2-6, Front View Combination Module](#)).
- 4) There will be two short beeps when the bottle ID is successfully scanned into the **Bottle ID** field.
- 5) If the **Accession Number** field is enabled, there will be three short beeps when the accession number is successfully scanned into the **Accession Number** field.

If an Operator Error occurs, a series of beeps will alert the operator to view the Operator Panel.

If the barcode is not read:

- 1) Verify the appropriate field has focus.
- 2) Move the bottle away from the barcode strip and initiate another scan.

Note: The **Hospital ID** field is a keyboard-entry field only.

Manually Entering Text into a Data Entry Field (Keyboard)



Field text, where applicable, can be entered using the keyboard (see [Figure 2-4, Accessing the Keyboard and Reference Card](#)). If a barcode label cannot be scanned successfully, the bottle ID or accession number can also be entered using the keyboard.

See Appendix A for International character entry instructions.

Note: Before entering text into a field, touch the desired field to place focus on that field. The field should turn white indicating focus.

- 1) Using the keyboard, enter the desired text.
- 2) Press the **Tab** key to move focus to the next field.

Note: *If desired, change the cursor location using the keyboard keys. Keys that provide cursor positioning and editing functions are as follows:*

LEFT ARROW	Moves cursor left one position
RIGHT ARROW	Moves cursor right one position
HOME	Moves cursor to the start of the text field
END	Moves cursor to one position past end of text
DELETE	Deletes the character at the current cursor position
BACKSPACE	Deletes the character at the current cursor position and moves the cursor left one position

Note: *Entering text does not over-write existing text located to the right of the insertion point.*

Note: *Entered text defaults to all-uppercase. To change the default, contact your local bioMérieux Representative.*


Loading Bottles



CAUTION: In order to preserve test data integrity, handle only one bottle at a time. Completely load a bottle according to this procedure before proceeding to the next bottle.

Loading Bottles



- 1) From the Main screen (see [Figure 3-1, Main Screen, on page 3-3](#)), press the **Load Bottles** button ().

The Load Mode screen appears.

1.0 BacT/ALERT 3D SelectLink 13:04:22 08/18/05

MB	1	1	0	0	0
BC	14	4	1	9	1

1. [Icon]

2. 7.8 [Icon]

3. BACT/ALERT SA [Icon]

4. SABB477W 11082005 4824538547

5. JANE

6. DOE

7. B.30

8. [Icon]

Figure 3-7: Main Screen — Load Mode

- | | |
|-------------------------------------|-------------------------------|
| 1 — Load Bottles Icon | 5 — Patient First Name Field* |
| 2 — Change Maximum Test Time Button | 6 — Accession Number Field |
| 3 — Bottle Type Scroll Button | 7 — Patient Last Name Field* |
| 4 — Bottle ID Field | 8 — Hospital ID Field* |

*Available with Select and SelectLink only.

The number of available cells appears at the bottom of each drawer on the **Instrument** icon.

Green indicators on Incubation or Combination Module drawers illuminate drawers that contain available cells.



CAUTION: Inspect each bottle and sensor before loading:

If the sensor is yellow, treat the bottle as a positive culture. If the bottle is cracked, do not load the bottle.

- 2) Verify the **Bottle ID** field appears white, then scan or manually enter the bottle ID (see [Text/Data Entry on page 3-9](#)).

If the field is left blank when a bottle is loaded, then the bottle is considered anonymously loaded (see [Handling Anonymous Bottles on page 3-16](#)).

- 3) Verify the correct bottle type is displayed on the **Bottle Type** scroll button.

If the **Bottle ID** field contains data from a generic label, the bottle type can be manually entered using the **Bottle Type** scroll button before inserting the bottle to ensure proper testing of the bottle. The instrument will also beep continuously to alert the operator that the bottle type needs to be manually entered. The audible alert can be disabled by calling bioMérieux for assistance. See [Figure 3-7, Main Screen — Load Mode, on page 3-12](#).



CAUTION: For best results, manually enter the bottle type when “GENERIC” is the displayed bottle type. Otherwise, observe the following:

When a bottle is loaded with a generic or anonymous bottle type, care must be taken to load BacT/ALERT® MP bottles into drawers labeled MB. The maximum test time assigned to a generic bottle loaded into an MB drawer is the same time assigned to the BacT/ALERT® MP media type in the Set Maximum Test Time screen (see [Setting the Maximum Test Time in Chapter 7](#)).

All non-MB bottles must be loaded into drawers labeled BC and the maximum test time is set to the time specified for the Unknown bottle type.

BacT/ALERT® MB Bottles should never be loaded with a Generic media type or loaded anonymously. BacT/ALERT® MB must be displayed on the Bottle Type scroll button before loading a BacT/ALERT® MB bottle.

- 4) If the **Accession Number** field is enabled and blank, then continue to [Step 5](#). If the **Accession Number** field is disabled, then go to [Step 7](#).
- 5) Verify the **Accession Number** field appears white, then scan or manually enter the *accession number*.
- 6) If the fields are displayed and enabled, then manually enter the following in the order listed: *Hospital ID*, *Patient First Name*, and *Patient Last Name*.

- To change the order of the **Patient First Name** and **Patient Last Name** fields, contact your local bioMérieux Representative.
- Entries to the **Patient First Name** and **Patient Last Name** fields cannot be made without the **Hospital ID** entry.

Note: The **Hospital ID** field is a keyboard-entry only field.

- 7) The default maximum test time is displayed above the Change Maximum Test Time button. The scanned bottle's maximum test time can be adjusted if desired. See [Changing the Maximum Test Time — Individual Bottles on page 3-15](#).
- 8) If all drawers are closed, slowly open a drawer with an illuminated indicator. Available cells will have an illuminated cell indicator light.
- 9) Insert the bottle, sensor first, into a cell with an illuminated cell indicator light.

WARNING



An erroneous test result (ex. false negative or false positive) could occur if a bottle is not fully seated into a cell. When inserting a bottle, ensure the bottle is fully seated into the cell. See [Appendix C, Best Practices](#) for additional information on preventing false positives.

- 10) The cell indicator light blinks slowly to acknowledge the bottle is loaded.
- 11) Verify that all text fields clear before proceeding.
- 12) Repeat [Step 2](#) through [Step 11](#) for each remaining bottle. Limit the bottle load time to two minutes in one area to control entry of room temperature bottles into racks. Close the drawer to allow temperature to equilibrate before loading in that area again. Load bottles into different drawers (ex., If you have four incubators, use the drawers in all four incubators). See [Appendix C, Best Practices](#) for loading bottles.



CAUTION: If a large number of bottles are loaded into the Incubation Module at the same time and in the same areas, a large heat mass loss within the racks may occur. This heat loss may trigger the acceleration or rate algorithms to erroneously flag positive.

- 13) When all bottles are loaded, ensure that all drawers are completely closed, then press the **Check** button.

If no operator or bottle loading activity has been recorded in a period of two minutes, the instrument will terminate the Load Bottle operation.

Operator activity includes:

- pressing keys on the keyboard
 - scanning barcodes
 - touching the Operator Panel
 - loading or unloading bottles
- 14) If applicable, enter the *patient* and *accession* data associated with the loaded bottles into the bioMérieux data management computer.

Changing the Maximum Test Time — Individual Bottles



- 1) From the Load screen, press the **Change Maximum Test Time** button



() after scanning the bottle barcode.

The Change Maximum Test Time screen overlays and disables the Load screen.

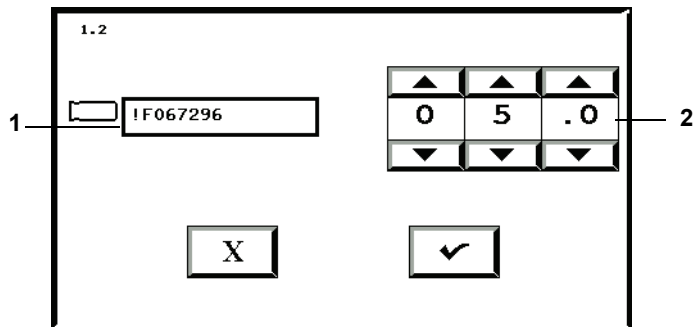


Figure 3-8: Change Maximum Test Time Screen

1 — Bottle ID Field

2 — Max Test Time Scroll Buttons

- 2) Verify the bottle ID matches that of the bottle for which you wish to change the maximum test time.
- 3) Adjust the maximum test time in days using the **Max Test Time** scroll buttons.
- 4) Press the **Check** button to accept the changes, or the **Cancel** button to retain the original setting.

The system returns to the Load Mode screen.

Note: Changing the maximum test time of an individual bottle during loading does not affect any other bottles of the same type.

Note: The maximum test time of an individual bottle can also be changed from the Edit Bottle Detail screen after loading (see [Viewing/Editing Bottle Data in Chapter 5](#)).

Handling Anonymous Bottles

Bottles loaded into the Incubation Module without accessing the Load Bottles function on the Main screen are referred to as Anonymous bottles because they are not associated with a bottle ID.



CAUTION: BacT/ALERT® MB (Mycobacteria Blood) Bottles should never be loaded anonymously. Appropriate testing of anonymous bottles occurs only when BacT/ALERT® MP bottles are loaded into drawers labeled MB and non-MB bottles are loaded into drawers labeled BC.

The maximum test time of anonymous bottles loaded into MB drawers is the same time specified for the BacT/ALERT® MP media type on the Set Maximum Test Time screen.

Anonymous bottles loaded into BC drawers are assigned the standard default algorithm.

Anonymous bottles should be removed and identified as specified in [Unloading Bottles on page 3-16](#), or identified using the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 5](#)).

Unloading Bottles

The BacT/ALERT® 3D signals which type of bottles are ready for unloading by enabling the appropriate Unload button.



CAUTION: In order to preserve test data integrity, handle only one bottle at a time. It is important to complete the procedure for each bottle before proceeding to the next bottle.

Unloading Bottles



- 1) Generate an Unload report (see [Viewing, Printing, and Exporting Test Data on page 3-24](#)).
- 2) From the Main screen (see [Figure 3-1, Main Screen, on page 3-3](#)), press the appropriate **Unload** button.
 - The Unload Mode screen appears (see [Figure 3-9](#)).
 - Green indicators illuminate on drawers containing bottles of the selected unload type.

1.9 **BacT/ALERT 3D SelectLink** 13:13:26 08/18/05

MB	1	1	0	0	0
BC	14	4	1	9	1

1 [Grid Icon] →

2 [Checkmark Icon]

3 [BacT/ALERT SA Scroll Button]

4 [Bottle ID Field: SABB477W]

5 [Patient First Name Field: JANE]

6 [Accession Number Field: 11082005]

7 [Patient Last Name Field: DOE]

8 [Hospital ID Field: 4824538547]

Grid details: 2 37.0C MB, 1, 1 37.0C MB, MB, B.30, MB

Figure 3-9: Main Screen — Unload Mode

- | | |
|-------------------------------|------------------------------|
| 1 — Unload Bottles Icon | 5 — Patient First Name Field |
| 2 — Unload Buttons | 6 — Accession Number Field |
| 3 — Bottle Type Scroll Button | 7 — Patient Last Name Field |
| 4 — Bottle ID Field | 8 — Hospital ID Field |

- 3) Open the indicated drawer. When the indicated drawer is open, cell indicator lights light up next to all bottles in the selected category.
- 4) Remove one of the bottles indicated. Wait for the cell light to blink slowly to acknowledge the removal of the bottle.
- 5) If the bottle was identified when loaded:
 - The bottle ID and bottle type will appear on the Unload Mode screen; however, the accession number, hospital ID, patient first name, and

patient last name will appear in disabled text fields, if information is available.

- a. It is not necessary to re-scan the bottle ID; however, doing so will verify that bottle's identity.
 - b. If fields are blank or need editing, then use the Edit Data Relationships function (see [Editing Data Relationships in Chapter 5](#)).
- 6) Go to [Step 8](#).
- 7) If the bottle was anonymously loaded (the **Bottle ID** field is blank), scan or manually enter the bottle ID.
- a. Identify the bottle by entering the bottle ID, bottle type, accession number, hospital ID, and patient first and last name as performed in the procedure, [Loading Bottles on page 3-11](#).
 - Scanning the bottle ID successfully results in two short beeps.
 - Scanning the accession number successfully results in three short beeps.
 - When identifying anonymous bottles, information entered in the **Bottle ID**, **Bottle Type**, **Accession Number**, **Hospital ID**, **Patient First Name**, and **Patient Last Name** fields, and with the **Bottle Type** scroll button, is associated with the unloaded bottle once the next bottle is unloaded or the **Check** button is pressed.
 - b. If the bottle is to be reloaded, immediately return the bottle to the cell with the slowly blinking cell indicator light before unloading another bottle.

WARNING



Bottles with a critical determination in progress will be temporarily removed from the bottle count table on the Main screen.

- c. Do not reload the bottle if its status is negative, positive, or if it is necessary to use the bottle label to enter information into the bioMérieux data management computer.



CAUTION: Reloading bottles anonymously that were previously loaded will result in duplicate bottle records.

- 8) Repeat [Step 3](#) through [Step 5](#) for the remaining bottles to be unloaded. Limit the bottle unload time to no more than two minutes in one area. Close the drawer to allow temperature to equilibrate before unloading from the area again. See [Appendix C, Best Practices](#) for unloading bottles and preventing false positives.



CAUTION: If a large number of bottles are unloaded from the Incubation Module at the same time and in the same areas, a large heat mass loss within the racks may occur. This heat loss may trigger the acceleration or rate algorithms to erroneously flag positive.

- 9) When finished unloading bottles, ensure that all drawers are completely closed.
- 10) Press the **Check** button on the Unload Mode screen.
- 11) Verify that the bottles listed on the Unload report were unloaded.
- 12) Where applicable, enter *patient* and *accession* data associated with the unloaded bottles into the bioMérieux data management computer.
- 13) Reload any previously Anonymous negative-to-date bottles.

If no operator or bottle loading activity has been recorded in a period of two minutes, the instrument will terminate the Unload Bottle operation. Operator activity includes:

- pressing keys on the keyboard
- scanning barcodes
- touching the Operator Panel
- loading or unloading bottles

Handling Unconfirmed Positive Bottles (False Positives)

If a smear of a positive bottle reveals no microorganisms, the bottle should be subcultured and reloaded into the instrument via the Load Bottles function (see [Loading Bottles on page 3-11](#)).

Note: *If a bottle is reloaded into the instrument, its status will revert to negative-to-date once a reading has been taken (maximum time – 10 minutes).*

If growth appears on the subculture, edit the bottle's status to Positive on the Edit Test Result screen which is accessed from the Edit Bottle Detail screen (see [Edit Test Result Button in Chapter 5](#)), and unload the now positive bottle.

Note: Results that have been manually changed to negative or positive via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 5](#)), will be marked on the report with a stick figure (☿).

Note: If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (☿) will not appear.


Accessing the Setup Screen Function Buttons

To view and print data, print graphs, and perform all of the editing, configuration, and maintenance functions of the system, you must first access the Setup screen function buttons.

Accessing the Setup Screen



- 1) From the Main screen (see [Figure 3-1, Main Screen, on page 3-3](#)), press

the **Next Screen** button ().

The Setup screen appears.

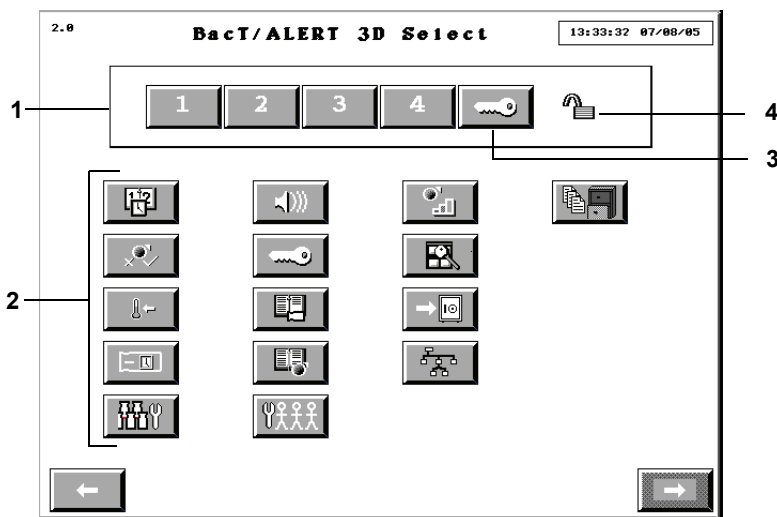


Figure 3-10: Setup Screen

- 1 — Password Entry Keypad
- 2 — Function Buttons
- 3 — Key Symbol Button
- 4 — Padlock Icon

- 2) Enter a valid password using the **1 – 4** buttons. Until a valid password is entered and accepted, the **Padlock** icon will appear in the closed position.

Note: Acceptable passwords consist of any combination of the numbers 1 to 4 and have a maximum length of eight characters.

Note: Instruments are shipped with a password of 1234. For information on changing the default password, see [Changing the System Password in Chapter 7](#).

- 3) Press the **Key Symbol** button to accept the password.
- 4) After a valid password is accepted, the **Padlock** icon changes to the full open position and the function buttons become enabled.

Note: To correct an error made while entering a password, press the **Key Symbol** button and re-enter the password.

Note: If more than 8 characters are entered, the **Password Entry** buttons will become inactive and turn gray. Press the **Key Symbol** button and re-enter the password.

Inactivity Timeout for all Setup Screens

While you are in the Setup screen (see [Figure 3-10, Setup Screen](#)) or one of its sub-menus, an inactivity timeout will occur (within a period of time configured by your bioMérieux Service Representative) if you do not perform one of the following actions:

- press a screen or keyboard button
- scan a barcode
- load or unload a bottle

If an inactivity timeout occurs, the instrument display reverts from the currently displayed screen to the Main screen. Any pending function is cancelled as if the **Cancel** button on each successive screen was pressed.

Note: If a timeout occurs, it is possible that partially entered information will be lost.

Note: The inactivity timeout feature is disabled while a red operator error is displayed on the screen.


Setup Screen Function Buttons



Set Date/Time button (see [Setting and Formatting the System Date and Time in Chapter 8](#))



Enable/Disable Module, Drawer, Rack, or Cell button (see [Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#))

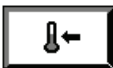

Calibrate Module Temperature button (see [Adjusting an Incubation or Combination Module's Temperature in Chapter 8](#))



Set Maximum Test Time button (see [Setting the Maximum Test Time in Chapter 7](#))



Set Audible Alarm Options button (see [Setting the Audible Alarms in Chapter 7](#))



Change Password button (see [Changing the System Password in Chapter 7](#))



Select Bottle to Edit/Graph button (see [Selecting Bottles Using the Select Bottle to Edit/Graph Button in Chapter 5](#))



Edit Cell Contents button (see [Selecting Bottles Using the Edit Cell Contents Button in Chapter 5](#))



Calibrate Cell button (see [Calibrating an Instrument Cell in Chapter 8](#))



View Incubation Module Information button (see [Viewing Incubation Module Information in Chapter 8](#))



Backup Management button (see [Initiating Manual Backup in Chapter 7](#))



Edit Data Relationships button (see [Initiating the Edit Data Relationships Function in Chapter 5](#))



Report button — Select and SelectLink only (see [Viewing, Printing, and Exporting Test Data on page 3-24](#))



Configure Users button — Button only appears in 21 CFR Part 11 mode (see [Configuring Users — 21 CFR Part 11 Mode in Chapter 10](#))



Bottle Processing/Customization button (For use only with instruction from bioMérieux)

Viewing and Printing

Introduction

Viewing Bottle Data

The following bottle information can be viewed by accessing the Edit Bottle Detail screen as described in Editing Test Data ([see Viewing/Editing Bottle Data in Chapter 5](#)):

- Bottle ID
- Accession number
- Hospital ID (where applicable)
- Patient first and last name (where applicable)
- Cell ID
- Maximum test time
- Bottle type
- Date/time loaded
- Date/time unloaded
- Date/time of last bottle reading
- Test time
- Test result
- Algorithm type
- How determined/positivity index

Viewing/Printing Reports

With a BacT/ALERT® 3D Signature configuration, reports are viewed and printed at the data management computer.

With the BacT/ALERT® 3D Select or SelectLink configurations, reports are viewed and printed by using the **Report** button as described in the topic, [Viewing, Printing, and Exporting Test Data on page 3-24](#).

Viewing/Printing Graphs

Bottle Graphs can be viewed and printed on the BacT/ALERT® 3D Select and SelectLink configurations as described in the topic, [Viewing and Printing Bottle Graphs on page 3-31](#). For the Signature configuration, graphics can be viewed as described in the topic, [Viewing and Printing Bottle Graphs on page 3-31](#), or at the data management computer. Since the BacT/ALERT® 3D Signature configuration is not connected to a printer, you must use the data management computer to print a graph.

Note: *The print feature can be made unavailable regardless of software configuration. When the feature is unavailable, the **Print** buttons do not*

display. To make the print feature unavailable, contact your local bioMérieux Representative.

Using the Print Screen Function

You can print the current screen on the instrument by pressing **Ctrl + P** on the keyboard.

Note: Select and SelectLink configurations only.

Viewing, Printing, and Exporting Test Data

Note: If you have a BacT/ALERT® 3D Signature configuration, you must use the data management computer to view and print reports.



- 1) From the Setup screen (see [Figure 3-10, Setup Screen, on page 3-20](#)),

press the **Report** button ().

Note: The **Report** button is not available with the BacT/ALERT® 3D Signature configuration.

The Report Selection screen appears.

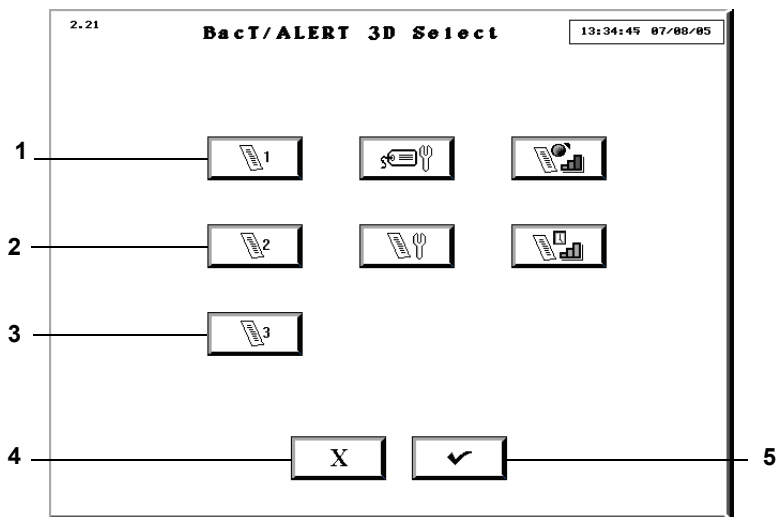


Figure 3-11: Report Selection Screen

- | | |
|-----------------------------|----------------------------|
| 1 — Display Report 1 Button | 4 — Previous Screen Button |
| 2 — Display Report 2 Button | 5 — Check Button |
| 3 — Display Report 3 Button | |

- 2) Press the **Display Report 1, 2, or 3** button that is configured for the information desired.

The Report screen will display. There are three default report configurations:

- **Display Report 1** button — Generates the Load Report screen with **1st Load Time** as the primary sort, and **Accession Number** as the secondary sort. Report has section breaks based on **1st Load Time**.
- **Display Report 2** button — Generates the Status Report screen with **Accession Number** as the primary sort, and **Bottle Type** as the secondary sort. Report has section breaks based on **Accession Number**.
- **Display Report 3** button — Generates the Unload Report screen with **Loaded** as the primary sort, and **Test Result** as the secondary sort. Report has section breaks based on **Loaded** and **Test Result**.

See [Configuring Report Contents in Chapter 7](#) for examples of the Load, Status, and Unload Report Configuration screens.

Note: *Data from the last 1,920 bottles are displayed each time a Report screen is accessed. The report screens come configured with default settings, but the report configurations can be changed to display different data and to sort the data differently (see [Configuring Report Contents in Chapter 7](#)).*

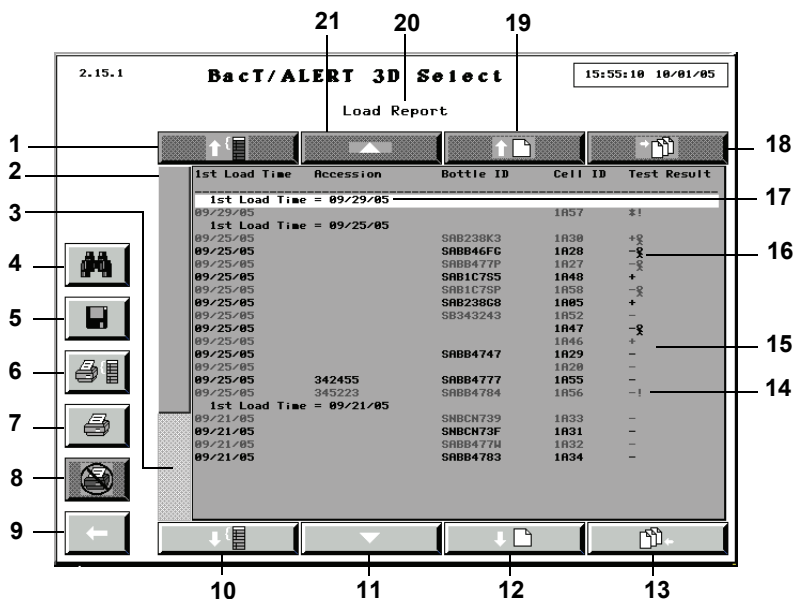


Figure 3-12: Sample Report Screen

- | | |
|--------------------------------------|------------------------------|
| 1 — Section Scroll Up Button | 11 — Line Scroll Down Button |
| 2 — Relative Record Scroll Indicator | 12 — Page Scroll Down Button |
| 3 — Relative Record Scroll Bar | 13 — End Scroll Button |
| 4 — Find Text Button | 14 — Gap Detection Indicator |
| 5 — Save Button | 15 — Report Data Lines |
| 6 — Print Current Section Button | 16 — Stick Figure |
| 7 — Print Report Button | 17 — Current Section Line |
| 8 — Cancel Print Button | 18 — Home Scroll Button |
| 9 — Previous Screen Button | 19 — Page Scroll Up Button |
| 10 — Section Scroll Down Button | 20 — Report Title |
| | 21 — Line Scroll Up Button |

Note: Results that have been manually changed to negative or positive (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 5](#)) will be marked on the report with a stick figure (⌘).

Note: If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (⌘) will not appear.

Note: *Bottles with an Instrument Fault Code 80 will be marked on the report with a **Gap Detection** indicator (!) next to the result. If a negative bottle has an Instrument Fault Code 80 while it is being unloaded, the **Gap Detection** indicator will remain on the report (see [Instrument Fault Codes in Chapter 9](#)).*

Note: *The Current Group Line will always display the group that is associated with the first displayed data record. It is always highlighted for easy reference. If there are no section breaks in the displayed report, this line will become the first line of data records displayed and reference highlighting will not occur.*

3) To scroll up or down a group, press the appropriate **Group** scroll button.

Note: *The **Group** scroll buttons are disabled if there are no section breaks in the displayed report or if there are no available sections in the indicated direction.*

4) To scroll up or down a data line, press the appropriate **Line** scroll button or press the appropriate ↑ or ↓ key on the keyboard.

Note: *The **Line** scroll buttons are disabled if there are no data lines in the indicated direction.*

5) To scroll up or down a data page, press the appropriate **Page** scroll button or press the appropriate **Page Up** or **Page Down** key on the keyboard.

Note: *The **Page** scroll buttons are disabled if there are no data lines in the indicated direction.*

6) To scroll up to the oldest data line (the first data record in the report), press the **Home** scroll button or press the **Home** key on the keyboard.

Note: *The **Home** scroll button is disabled if there are no report data lines in the indicated direction.*

7) To scroll down to the newest data line (the last data record in the report), press the **End** scroll button or press the **End** key on the keyboard.

Note: *The **End** scroll button is disabled if there are no report data lines in the indicated direction.*

8) To move to a relative record position, press the **Relative Record** scroll bar above/below the Relative Record Indicator.

Note: *The Relative Record Indicator is sized proportionally to the amount of records in the report.*

Note: *The **Relative Record** scroll bar is disabled if the Relative Record Indicator is the same size as the scroll bar and all the data records in the report are displayed.*

9) To print the report, press the appropriate **Print** button:

- Pressing the **Print Report** button prints all records in the database (with a maximum of 1,920 records).
- Pressing the **Print Current Group** button prints the Current Group Line and all data records associated with the group.

Note: *The **Print** buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.*

10) While the report is printing:

- All **Print** and **Save** buttons are disabled.
- The operator may, however, view and scroll through the displayed report, search for text in the displayed report, or exit the Report screen and perform other operations.
- The **Cancel Print** button becomes available.

11) Press the **Cancel Print** button to stop sending data to the printer and to empty the queue of any data waiting to be sent to the printer.

Note: *Once the cancellation has been completed, the **Print** and **Save** buttons will become enabled, and the **Cancel Print** button will be disabled.*

12) To specify a text string and initiate a search of the report data for the specified text, press the **Find Text** button. The Find Text screen will appear.

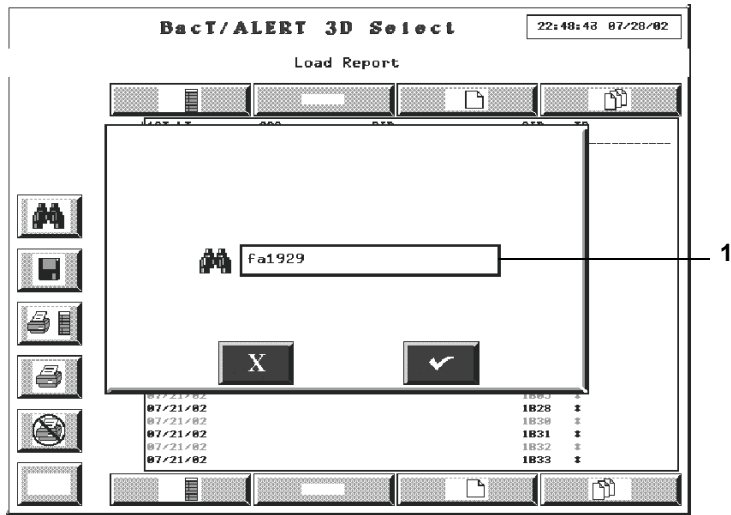


Figure 3-13: Find Text Screen

1 — Find Text Field

Use the keyboard to enter the text you wish to seek, and press the **Check** button to return to the Report screen. Pressing the **Cancel** button will cancel the search request and return to the Report screen.

Note: The search will be performed upon returning to the report screen starting from the top of the report. The record that contains the first occurrence of the text will be scrolled to the first line of data displayed and the text itself will be highlighted for reference.

If no occurrence of the specified text is found, the data displayed will not change and no reference highlighting will occur.

13) To find the next instance of the specified text, press the **F3** key on the keyboard.

Note: If no new instance of the text is found the data displayed, and the highlighted text will remain the same.

14) To save the displayed report to a text file, press the **Save** button.

The Save To File screen appears.

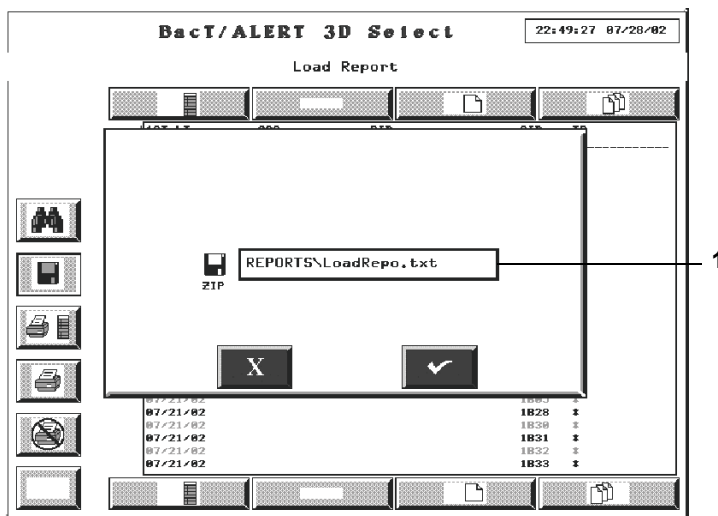


Figure 3-14: Save to File Screen

1 — File Name Field

Note: The default file name and path will appear automatically in the **File Name** field. Use the keyboard to change the name of the file.

Note: All reports are saved to the pathname, **D:\REPORTS**, either using the default file name or a specified file name. All reports are automatically given an extension of **.TXT**.

15) Place the backup media in the backup drive.

Note: Backup media can either be a Zip[®] disk or a USB flash drive.

16) Press the **Check** button to initiate the save and return to the Report screen. Pressing the **Cancel** button will cancel the save request and return to the Report screen.

Note: The **Print** buttons will be disabled while the save is taking place. Once the save has completed, the **Print** buttons will be enabled.

WARNING



Do not insert any object, other than a Zip[®] disk, into the Zip[®] drive under any circumstances.




CAUTION: Do not forcibly remove the Zip[®] disk from the instrument. Forcibly removing the Zip[®] disk may cause damage to the Zip[®] disk or Zip[®] drive and may cause the system to lock up.

- 17) Press the **Previous Screen** button to return to the screen from which the Report screen was accessed.

Viewing and Printing Bottle Graphs

Note: *If you have a BacT/ALERT[®] 3D Signature configuration, you must use the data management computer to print a bottle graph since the BacT/ALERT[®] 3D Signature configuration is not connected to a printer.*



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons on page 3-20](#)).
- 2) Press the **Select Bottle to Edit/Graph** button ().

The Select Bottle to Edit/Graph screen overlays and disables the Setup screen.

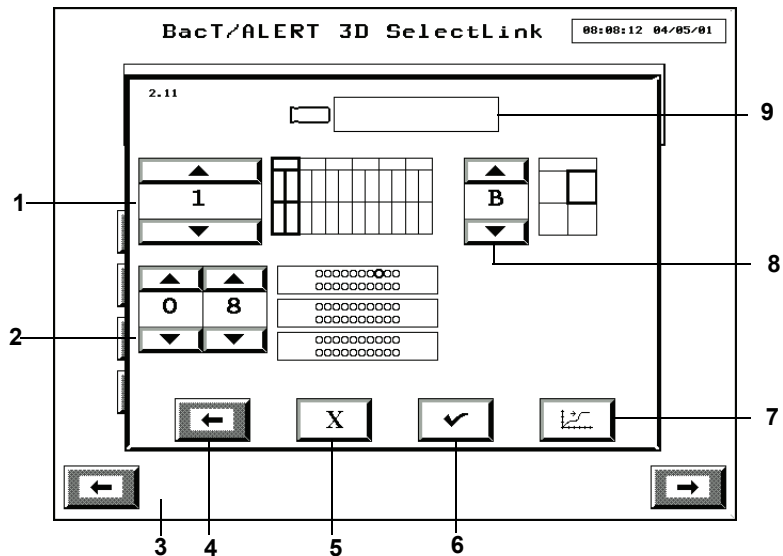


Figure 3-15: Select Bottle to Edit/Graph Screen

- | | |
|-------------------------------------|----------------------------------|
| 1 — Incubation Module Scroll Button | 6 — Check Button |
| 2 — Cell Scroll Buttons | 7 — Graph Bottle Readings Button |
| 3 — Setup Screen (Disabled) | 8 — Drawer Scroll Button |
| 4 — Previous Screen Button | 9 — Bottle ID Field |
| 5 — Cancel Button | |

- 3) If you know the bottle ID of the bottle whose readings you wish to graph, then enter the *bottle ID* in the **Bottle ID** field ([see Text/Data Entry on page 3-9](#)) and go to [Step 5](#). If you know the cell location but not bottle ID, then continue to [Step 4](#).

Note: Only the last 1,920 bottles loaded can be retrieved. Any other entry in the **Bottle ID** field is invalid and will cause an Operator Error 940 ([see Operator Error Codes in Chapter 9](#)).

- 4) Adjust the **Incubation Module** (1–6), **Drawer** (A–D), and **Cell** (1–60) scroll buttons to select the cell location of the bottle whose readings you wish to graph. The cell location defaults to Module 1, Drawer A, Cell 1 (1A01).

Note: Only cell locations with currently loaded bottles can be used to view bottle graphs. To view graphs of recently unloaded bottles (the most recent 1,920 bottles loaded), you must use the **Bottle ID** field.

- 5) Press the **Graph Bottle Readings** button ().

The Graph Bottle Readings screen appears (see [Figure 3-16](#)).

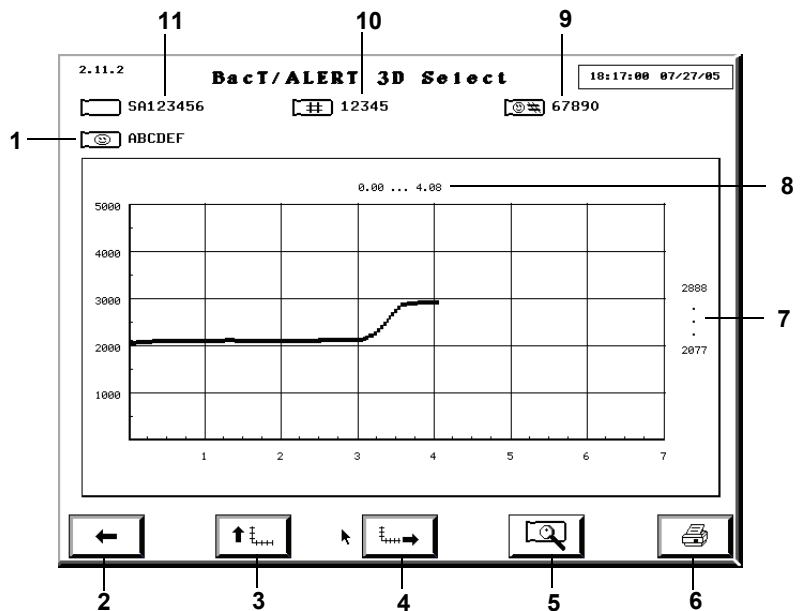


Figure 3-16: Graph Bottle Readings Screen

- | | |
|----------------------------|--------------------------------------------|
| 1 — Name Field* | 7 — Bottle Readings Range |
| 2 — Previous Screen Button | 8 — Days Tested Range |
| 3 — Adjust Y Scale Button | 9 — Hospital ID Field* |
| 4 — Adjust X Scale Button | 10 — Accession Number Field |
| 5 — Bottle Readings Button | 11 — Bottle ID Field |
| 6 — Print Graph Button | * Available in Select and SelectLink only. |

You can also access the Graph Bottle Readings screen via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 5](#)). The screen ID number is the same regardless of how it is accessed.

- The Y axis range defaults to 0 – 5000, and the X axis range defaults to 0 – maximum test time of the bottle in days.

- If the bottle was determined positive, then a dot and the time to detection (days) are displayed at the point where it was determined positive.
- 6) To adjust the Y or X axis, click the appropriate **Adjust Scale** button. When one of these buttons is clicked, the axis re-scales so that the maximum endpoint of the scale is just higher than the maximum value of the range. Adjusting the scale may mean increasing or decreasing the range of the scale.

When one of the **Adjust Scale** buttons is clicked, the arrow on the button changes directions. To return to the original scale, click the **Adjust Scale** button(s) a second time.

- 7) Press the **Print Graph** button, if available, to print the graph as it appears on the screen.

Note: The **Print Graph** button is only available if a printer is configured for the system. The **Print Graph** button is disabled while a print is in progress.

Note: To configure a printer for the system, contact your local bioMérieux Representative.

- 8) When done, press the **Previous Screen** button to return to the Select Bottle to Edit/Graph screen.

Display Bottle Readings




The Bottle Readings screen displays the exact values of the bottle readings for an individual bottle, along with the date and time each reading was taken.

Note: The polynomial is applied to the bottle readings. The readings are not the raw readings.

If there are bottle readings available for you to view, the **Bottle Readings** button on the Graph Bottle Readings screen turns blue (see [Figure 3-16](#)). If there are no readings available, the button is gray.

- 1) From the Graph Bottle Readings screen (see [Figure 3-16](#)), press the

Bottle Readings button ().

The Bottle Readings screen appears.

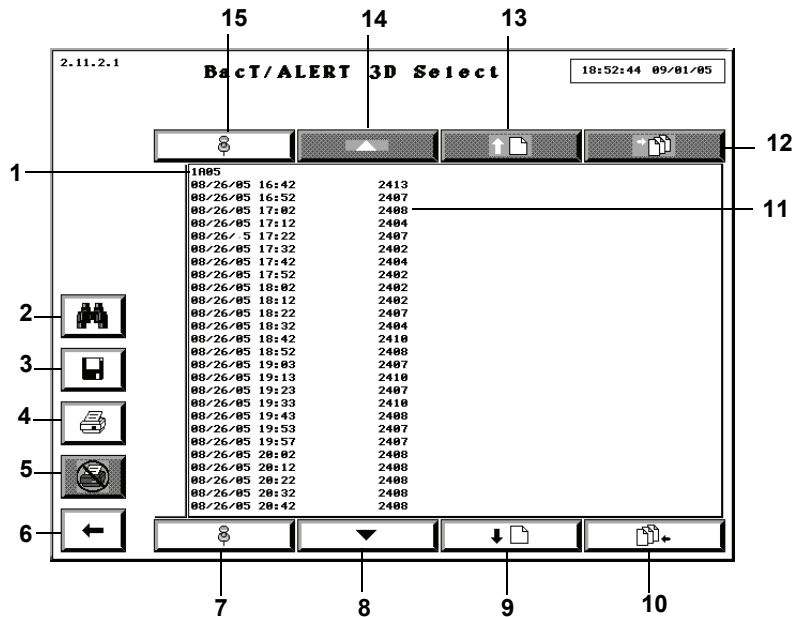


Figure 3-17: Bottle Readings Screen

- | | |
|----------------------------------|-----------------------------------------|
| 1 — Bottle Cell Location | 9 — Page Scroll Down Button |
| 2 — Find Text Button | 10 — End Scroll Button |
| 3 — Save Button | 11 — Bottle Reading (Date/Time/Reading) |
| 4 — Print Button | 12 — Home Scroll Button |
| 5 — Cancel Print Button | 13 — Page Scroll Up Button |
| 6 — Previous Screen Button | 14 — Line Scroll Up Button |
| 7 — Anchor Display Bottom Button | 15 — Anchor Display Top Button |
| 8 — Line Scroll Down Button | |

- 2) To scroll up or down a data line, press the appropriate **Line** scroll button or press the appropriate \uparrow or \downarrow key on the keyboard.

Note: The **Line** scroll buttons are disabled if there are no data lines in the indicated direction.

- 3) To scroll up or down a data page, press the appropriate **Page** scroll button or press the appropriate **Page** key on the keyboard.

Note: The **Page** scroll buttons are disabled if there are no data lines in the indicated direction.

- 4) To scroll up to the oldest data line (the first data record in the report), press the **Home** scroll button or press the **Home** key on the keyboard.

Note: *The **Home** scroll button is disabled if there are no report data lines in the indicated direction.*

- 5) To scroll down to the newest data line (the last data record in the report), press the **End** scroll button or press the **End** key on the keyboard.

Note: *The **End** scroll button is disabled if there are no report data lines in the indicated direction.*

- 6) To print the bottle readings, press the **Print** button.

Note: *The **Print** buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.*

- 7) While the report is printing:
 - All **Print** and **Save** buttons are disabled.
 - The operator may, however, view and scroll through the displayed list, search for text in the displayed list, or exit the Bottle Readings screen and perform other operations.
 - The **Cancel Print** button becomes available.
- 8) Press the **Cancel Print** button to stop sending data to the printer and to empty the queue of any data waiting to be sent to the printer.

Note: *Once the cancellation has been completed the **Print** and **Save** buttons will become enabled, and the **Cancel Print** button will be disabled.*

- 9) To specify a text string and initiate a search of the report data for the specified text, press the **Find Text** button.

The Find Text screen appears.

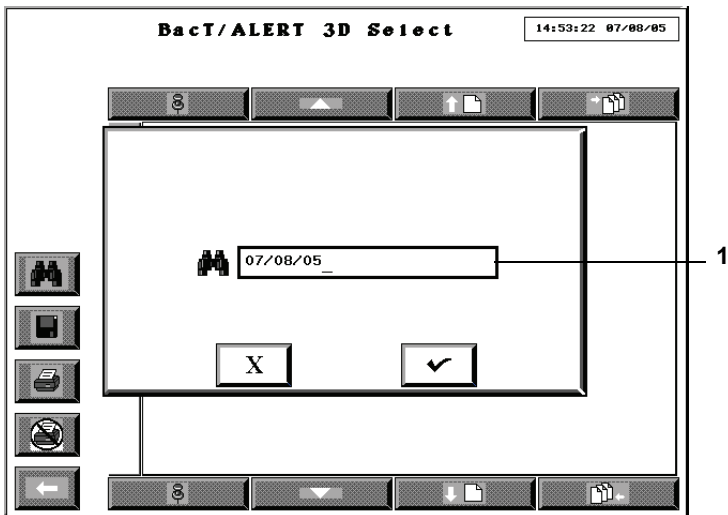


Figure 3-18: Find Text Screen

1 — Find Text Field

Use the keyboard to enter the text you wish to seek, and press the **Check** button to return to the Bottle Readings screen. Pressing the **Cancel** button will cancel the search request and return to the Bottle Readings screen.

Note: The search will be performed upon returning to the report screen starting from the top of the report. The record that contains the first occurrence of the text will be scrolled to the first line of data displayed and the text itself will be highlighted for reference.

If no occurrence of the specified text is found, the data displayed will not change and no reference highlighting will occur.

10) To find the next instance of the specified text, press the **F3** key on the keyboard.

Note: If no new instance of the text is found, the data displayed, and the highlighted text will remain the same.

11) To save the bottle readings to a text file, press the **Save** button.

The Save To File screen appears.

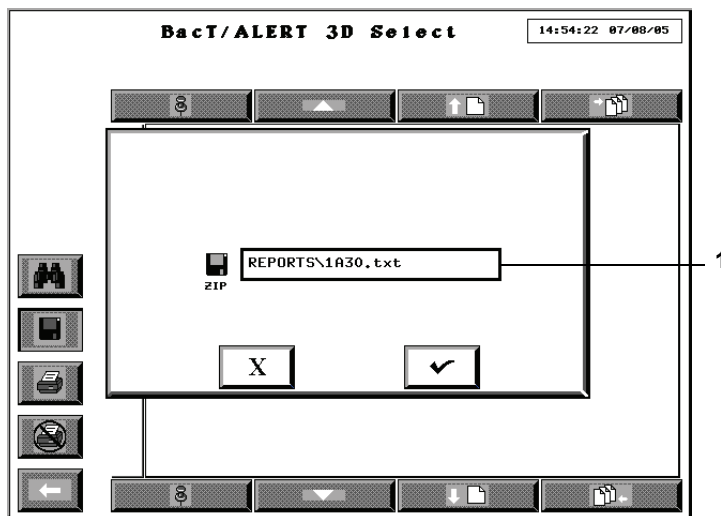


Figure 3-19: Save to File Screen

1 — File Name Field

Note: The default file name and path will appear automatically in the **File Name** field. Use the keyboard to change the name of the file.

Note: All reports are saved to the pathname, **D:\REPORTS**, either using the default file name or a specified file name. All reports are automatically given an extension of **.TXT**.

12) Place the backup media in the backup drive.

Note: Backup media can either be a Zip® disk or a USB flash drive.

13) Press the **Check** button to initiate the save and return to the Bottle Readings screen. Pressing the **Cancel** button will cancel the save request and return to the Bottle Readings screen.

Note: The **Print** buttons will be disabled while the save is taking place. Once the save has completed, the **Print** buttons will be enabled.

WARNING



Do not insert any object, other than a Zip® disk, into the Zip® drive under any circumstances.



CAUTION: Do not forcibly remove the Zip[®] disk from the instrument. Forcibly removing the Zip[®] disk may cause damage to the Zip[®] disk or Zip[®] drive and may cause the system to lock up.

- 14) Press the **Previous Screen** button to return to the screen from which the Bottle Readings screen was accessed.
- 15) When done, press the **Previous Screen** button to return to the Graph Bottle Readings screen.

Sending/Requesting LIS Information

Results can be manually sent to, and demographics can be manually requested from, an LIS when the BacT/ALERT[®] 3D SelectLink configuration is used.

Note: Data is transmitted according to the BacT/LINK[®] protocol. For more information, contact your local bioMérieux Representative.

Note: The system can be configured to perform these functions automatically. To adjust these settings, contact your local bioMérieux Representative.

Note: The arrows on the **Manual Send Test Results** and **Manual Request Test Orders** buttons also serve as an indicator of automatic data transfer.

Sending Results to the LIS



- 1) From the Main screen (see [Figure 3-1, Main Screen, on page 3-3](#)), press

the **Manual Send Test Results** button ().

The arrow on the button face will blink during the transfer.

Note: Only accessions with one or more bottles that have had a status change to positive, or that have reached maximum test time with a negative status and have been unloaded, since the last transfer will be sent.

Requesting Information from the LIS



- 1) From the Main screen (see [Figure 3-1, Main Screen, on page 3-3](#)), press

the **Manual Request Test Orders** button ().

The arrow on the button face will blink during the transfer.

About This Chapter

This chapter introduces you to the Main screen and provides you with procedures on how to perform daily basic functions for INDUSTRY use.

Chapter Contents

- Introduction • 4-2
- Monitoring the System • 4-3
 - Main Screen Introduction • 4-3*
 - Viewing Faults • 4-6*
 - Viewing the Cell Status Screen • 4-7*
- Text/Data Entry • 4-9
 - Common Text Fields and Field Limits • 4-9*
 - Using the Barcode Scanner to Enter Data • 4-10*
 - Manually Entering Text into a Data Entry Field (Keyboard) • 4-10*
- Loading Bottles • 4-11
 - Loading Bottles • 4-11*
 - Changing the Maximum Test Time — Individual Bottles • 4-14*
 - Handling Anonymous Bottles • 4-15*
- Unloading Bottles • 4-16
 - Unloading Bottles • 4-16*
 - Handling Unconfirmed Positive Bottles (False Positives) • 4-19*
- Accessing the Setup Screen Function Buttons • 4-20
 - Accessing the Setup Screen • 4-20*
 - Setup Screen Function Buttons • 4-21*
- Viewing and Printing • 4-23
 - Introduction • 4-23*
 - Viewing, Printing, and Exporting Test Data • 4-24*
 - Viewing and Printing Bottle Graphs • 4-31*
 - Display Bottle Readings • 4-34*
- Sending/Requesting LIS Information • 4-39
 - Sending Results to the LIS • 4-39*
 - Requesting Information from the LIS • 4-39*

Introduction

The basic functions are those tasks that may be performed during daily workflow. They include:

- Monitoring system
- Entering data (where applicable)
- Loading and unloading bottles
- Viewing and printing data (printing with the BacT/ALERT® 3D Signature is performed at the data management computer.)
- LIS interaction with the BacT/ALERT® 3D SelectLink
- Accessing the Setup screen

Monitoring the System

Main Screen Introduction

The BacT/ALERT® 3D system can be monitored from the Main screen (see [Figure 4-1](#)).

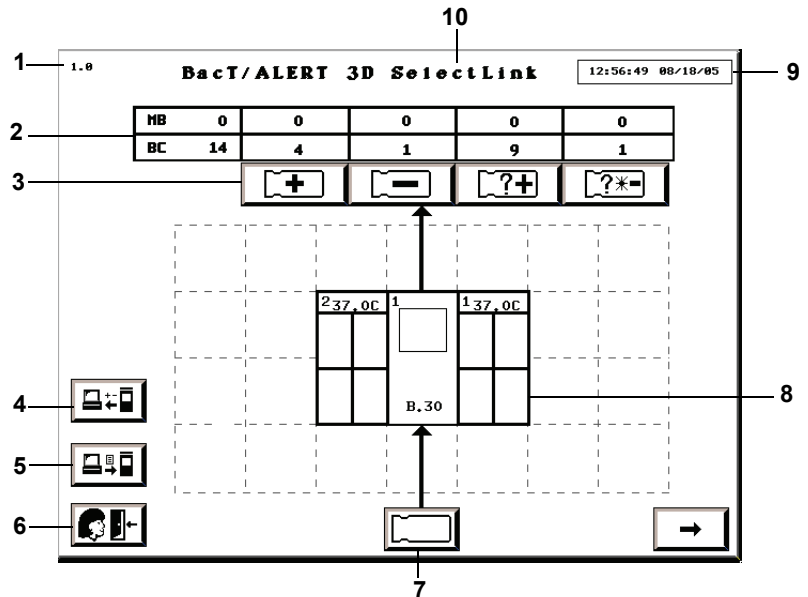


Figure 4-1: Main Screen

- 1 — Screen ID Number
- 2 — Bottle Count Table
- 3 — Unload Buttons
- 4 — Manual Send Test Results Button (SelectLink only)
- 5 — Manual Request Test Orders Button (SelectLink only)
- 6 — Logout Button (21 CFR Part 11 Mode only)
- 7 — Load Bottles Button
- 8 — Instrument Icon
- 9 — Current Date/Time
- 10 — Software Configuration



CAUTION: If the current date/time displayed at the top of the screen does not advance, contact bioMérieux Customer Service immediately.

Background Color

The default background color is determined by the software configuration (see [Software Configuration Options in Chapter 2](#)). The following conditions will override default background colors:

- A yellow screen indicates that the instrument has detected a positive bottle.
- A red screen indicates that an instrument fault has occurred. Touching the screen or pressing any key on the keyboard returns a red screen to yellow or the configuration default color depending on whether positive bottles are present. The fault code will remain on the screen until the error is cleared.

Instrument Icon

The following information is indicated on the **Instrument** icon:

- System ID numbers are assigned to both the Controller/Combination Module and Incubation Modules.
- Combination Module has 2 System ID numbers.
- The programmed optimal temperature (°C) is displayed for each Incubation Module.
- Software version number for the BacT/ALERT® 3D Controller/Combination Module.
- Disabled or uninstalled components appear diagonally striped in gray on the screen.
- An entire rack will appear diagonally striped if only one of its cells is disabled.

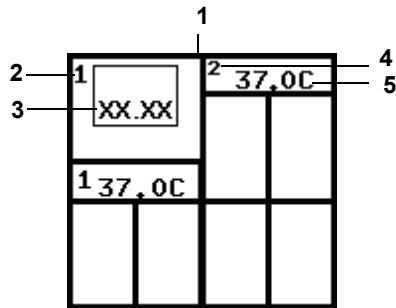


Figure 4-2: Combination Module with an Additional Incubation Module

- 1 — Combination Module Instrument Icon
- 2 — Combination Module ID
- 3 — Software Version
- 4 — Incubation Module ID
- 5 — Optimal Temp (°C)

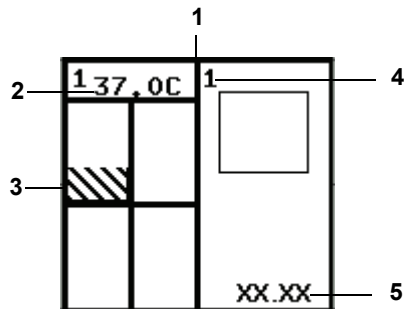


Figure 4-3: Controller Module with One Incubation Module

- 1 — Controller Module Instrument Icon
- 2 — Optimal Temp (°C)
- 3 — Disabled Rack or Cell
- 4 — Controller ID
- 5 — Software Version

Bottle Count Table

Located just above the **Instrument** icon are the **Unload** buttons and a bottle count table indicating the number of bottles of each type currently loaded in the instrument.

		3	5	7	9
1 —	MB	0	0	0	0
2 —	BC	14	4	1	9
		4	6	8	10
		+	-	?+	?*-

Figure 4-4: Bottle Count Table/Unload Buttons

- 1 — Total number of mycobacteria (MB) bottles loaded in the system (reserved for other applications).
- 2 — Total number of bottles loaded in the system. BC refers to bottle count.
- 3 — Total number of identified bottles with a positive test status.
- 4 — Unload Positive Identified Bottles Button
- 5 — Total number of bottles (identified and anonymous) with a negative test status.
- 6 — Unload Negative Bottles Button
- 7 — Total number of anonymous bottles with a positive test status.
- 8 — Unload Positive Anonymous Bottles Button
- 9 — Total number of anonymous bottles with a negative-to-date or negative test status.
- 10 — Unload Anonymous Negative or Negative-to-Date Bottles Button

Viewing Faults



Instrument faults are reported using a numeric code within a red diamond shape. Fault codes are displayed on the **Instrument** icon where the fault condition exists.

Note: Only high priority codes are displayed on the **Instrument** icon.

- 1) If the code appears on the Controller Module or the top half of the Combination Module, then move to [Step 4](#). If the fault code appears on an Incubation Module or bottom half of the Combination Module, then continue to [Step 2](#).
- 2) Touch the Incubation or Combination Module on the **Instrument** icon that contains the fault code.
- 3) The View Cell Status screen appears (see [Viewing the Cell Status Screen on page 4-7](#) and [Figure 4-5](#)).
 - Drawer faults appear at the top of the screen.

- Rack faults appear at the left end of the rack display.
 - Cell faults appear in the cell display.
- 4) For a complete list and description of Instrument Fault Codes, see the topic, [Instrument Fault Codes in Chapter 9](#).

Viewing the Cell Status Screen



The View Cell Status screen is continuously updated with changes that occur while the screen is active, such as the loading/unloading of bottles, new test results, and the error status of the drawer, as indicated by fault codes appearing or disappearing. To view the cell status screen:

- 1) Display the View Cell Status screen by touching the appropriate Incubation Module on the Instrument icon (see [Figure 4-1, Main Screen, on page 4-3](#)).

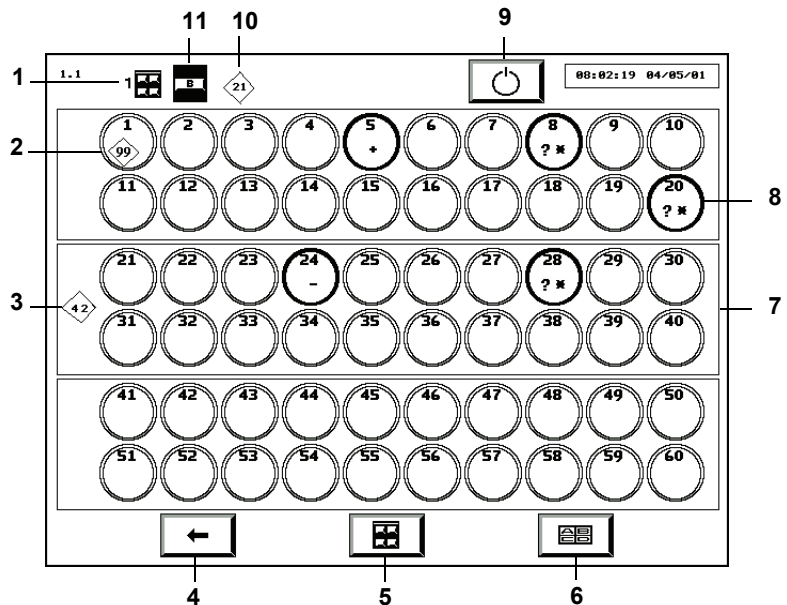


Figure 4-5: View Cell Status Screen

- | | |
|----------------------------------------|--------------------------------------|
| 1 — Incubation Module Indicator | 6 — Drawer Selection Button |
| 2 — Cell Fault Code | 7 — Rack |
| 3 — Rack Fault Code | 8 — Cell |
| 4 — Previous Screen Button | 9 — Restart Incubation Module Button |
| 5 — Incubation Module Selection Button | 10 — Drawer Fault Code |
| | 11 — Drawer Indicator |

- 2) If necessary, use the Incubation Module Selection and Drawer Selection buttons to display the desired drawer.

Note: *If a drawer is not installed, it is not displayed.*

- 3) Press the **Previous Screen** button to return to the Main screen.

Understanding the View Cell Status Screen Display

For each cell, the cell identification number appears at the top of the circle.

A hollow circle indicates an empty cell, and a solid circle indicates a loaded cell. The color of the solid circle indicates the bottle status or a cell that is pending quality control check.

- Black — negative-to-date bottle
- Green — negative bottle
- Yellow — positive bottle
- White — cell is pending quality control check

Loaded cells also contain symbols to indicate the bottle's status:

- | | |
|-----|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| + | Positive |
| — | Negative |
| * | Negative-to-date |
| ~ + | Critical determination in progress. (Represents a bottle that is presently undergoing a critical determination as to whether it will turn positive or remain negative or negative-to-date.) |

Note: *Bottles with a critical determination in progress status will be temporarily removed from the bottle count table on the Main screen.*

Cells loaded with an anonymous bottle contain a ?. If a loaded cell contains no ?, then the bottle is identified.

Disabled racks and cells are indicated by gray diagonal stripes within their borders. Disabled drawers are indicated by gray diagonal stripes within the drawer indicator (see [Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#)).

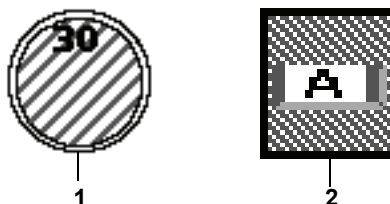


Figure 4-6: Disabled Cell and Drawer






1 — Disabled Cell

2 — Disabled Drawer

Note: If a drawer is disabled, its racks and cells are disabled and turn gray.

Text/Data Entry

Common Text Fields and Field Limits

 <input type="text"/>	Bottle ID field
 <input type="text"/>	User Defined 1 field — may contain up to 20 characters
 <input type="text"/>	User Defined 2 field — may contain up to 31 characters
 <input type="text"/>	Sample ID # field — may contain up to 16 characters
 <input type="text"/>	User Defined 3 field — may contain up to 22 characters

Note: The field length and initial character type (alpha, numeric, or other) can be configured for the **Sample ID** field. The initial character type can be configured for the **Bottle ID** field. To configure these fields, contact your local bioMérieux Representative.

Note: The **User Defined 1**, **User Defined 2**, and **User Defined 3** fields do not display with the BacT/ALERT® 3D Signature configuration.

Note: All fields except the **Bottle ID** field may be hidden or disabled regardless of software configuration. To hide or disable a field, contact your local bioMérieux Representative.

Using the Barcode Scanner to Enter Data



To scan a bottle or sample barcode:

- 1) Before scanning the barcode, touch the desired field to place focus on that field. The field should turn white indicating focus.
- 2) Rotate the bottle so the bottle ID or sample ID barcode is on top.
- 3) Place the bottle over the barcode strip located in the Barcode Reader Aperture below the Control Panel of the Controller Module (see [Figure 2-2, Instrument Front View](#)) or at the base of the barcode reader stand with the Combination Module (see [Figure 2-6, Front View Combination Module](#)).
- 4) There will be two short beeps when the bottle ID is successfully scanned into the **Bottle ID** field.
- 5) If the **Sample ID** field is enabled, there will be three short beeps when the **Sample ID** is successfully scanned into the sample ID field.

If an Operator Error occurs, a series of beeps will alert the operator to view the Operator Panel.

If the barcode is not read:

- 1) Verify the appropriate field has focus.
- 2) Move the bottle away from the barcode strip and initiate another scan.

Note: *The **User Defined 3** field is a keyboard-entry field only.*

Manually Entering Text into a Data Entry Field (Keyboard)



Field text, where applicable, can be entered using the keyboard (see [Figure 2-4, Accessing the Keyboard and Reference Card](#)). If a barcode label cannot be scanned successfully, the bottle ID or sample ID can also be entered using the keyboard.

See Appendix A for International character entry instructions.

Note: *Before entering text into a field, touch the desired field to place focus on that field. The field should turn white indicating focus.*

- 1) Using the keyboard, enter the desired text.
- 2) Press the **Tab** key to move focus to the next field.

Note: *If desired, change the cursor location using the keyboard keys. Keys that provide cursor positioning and editing functions are as follows:*

LEFT ARROW	Moves cursor left one position
RIGHT ARROW	Moves cursor right one position
HOME	Moves cursor to the start of the text field
END	Moves cursor to one position past end of text
DELETE	Deletes the character at the current cursor position
BACKSPACE	Deletes the character at the current cursor position and moves the cursor left one position

Note: *Entering text does not over-write existing text located to the right of the insertion point.*

Note: *Entered text defaults to all-uppercase. To change the default, contact your local bioMérieux Representative.*


Loading Bottles



CAUTION: In order to preserve test data integrity, handle only one bottle at a time. Completely load a bottle according to this procedure before proceeding to the next bottle.

Loading Bottles



- 1) From the Main screen (see [Figure 4-1, Main Screen, on page 4-3](#)), press the **Load Bottles** button ().

The Load Mode screen appears.

MB	0	0	0	0	0
BC	14	4	1	9	1

+ - ?+ ?-

2 37.0C	1	1 37.0C
	B.30	

ILB02207	3245	73
932221	PRODUCT X	

Figure 4-7: Main Screen — Load Mode

- | | |
|-------------------------------------|---------------------|
| 1 — Load Bottles Icon | 5 — User Defined 1* |
| 2 — Change Maximum Test Time Button | 6 — Sample ID Field |
| 3 — Bottle Type Scroll Button | 7 — User Defined 2* |
| 4 — Bottle ID Field | 8 — User Defined 3* |

*Available with Select and SelectLink only.

The number of available cells appears at the bottom of each drawer on the **Instrument** icon.

Green indicators on Incubation or Combination Module drawers illuminate drawers that contain available cells.



CAUTION: Inspect each bottle and sensor before loading:

If the sensor is yellow, treat the bottle as a positive culture. If the bottle is cracked, do not load the bottle.

- 2) Verify the **Bottle ID** field appears white, then scan or manually enter the bottle ID (see [Text/Data Entry on page 4-9](#)).

If the field is left blank when a bottle is loaded, then the bottle is considered anonymously loaded (see [Handling Anonymous Bottles on page 4-15](#)).

- 3) Verify the correct bottle type is displayed on the **Bottle Type** scroll button.

If the **Bottle ID** field contains data from a generic label, the bottle type can be manually entered using the **Bottle Type** scroll button before inserting the bottle to ensure proper testing of the bottle. The instrument will also beep continuously to alert the operator that the bottle type needs to be manually entered. The audible alert can be disabled by calling bioMérieux for assistance. See [Figure 4-7, Main Screen — Load Mode, on page 4-12](#).

- 4) If the **Sample ID** field is enabled and blank, then continue to [Step 5](#). If the **Sample ID** field is disabled, then go to [Step 7](#).
- 5) Verify the **Sample ID** field appears white, then scan or manually enter the *sample ID*.
- 6) If the fields are displayed and enabled, then manually enter the following in the order listed: *User Defined 3*, *User Defined 1*, and *User Defined 2*.
 - Entries to the **User Defined 1** and **User Defined 2** fields cannot be made without a *User Defined 3* entry.

Note: *The **User Defined 3** field is a keyboard-entry only field.*

- 7) The default maximum test time is displayed above the Change Maximum Test Time button. The scanned bottle's maximum test time can be adjusted if desired. See [Changing the Maximum Test Time — Individual Bottles on page 4-14](#).
- 8) If all drawers are closed, slowly open a drawer with an illuminated indicator. Available cells will have an illuminated cell indicator light.
- 9) Insert the bottle, sensor first, into a cell with an illuminated cell indicator light.

WARNING



An erroneous test result (ex. false negative or false positive) could occur if a bottle is not fully seated into a cell. When inserting a bottle, ensure the bottle is fully seated into the cell. See [Appendix C, Best Practices](#) for additional information on preventing false positives

- 10) The cell indicator light blinks slowly to acknowledge the bottle is loaded.

- 11) Verify that all text fields clear before proceeding.
- 12) Repeat [Step 2](#) through [Step 11](#) for each remaining bottle. Limit the bottle load time to two minutes in one area to control entry of room temperature bottles into racks. Close the drawer to allow temperature to equilibrate before loading in that area again. Load bottles into different drawers (ex. If you have four incubators, use the drawers in all four incubators). See [Appendix C, Best Practices](#) for loading bottles.



CAUTION: If a large number of bottles are loaded into the Incubation Module at the same time and in the same areas, a large heat mass loss within the racks may occur. This heat loss may trigger the acceleration or rate algorithms to erroneously flag positive.

- 13) When all bottles are loaded, ensure that all drawers are completely closed, then press the **Check** button.

If no operator or bottle loading activity has been recorded in a period of two minutes, the instrument will terminate the Load Bottle operation.


Operator activity includes:

- pressing keys on the keyboard
 - scanning barcodes
 - touching the Operator Panel
 - loading or unloading bottles
- 14) If applicable, enter *User Defined 3* and *sample* data associated with the loaded bottles into the bioMérieux data management computer.

Changing the Maximum Test Time — Individual Bottles



- 1) From the Load screen, press the **Change Maximum Test Time** button

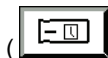

() after scanning the bottle barcode.

The Change Maximum Test Time screen overlays and disables the Load screen.

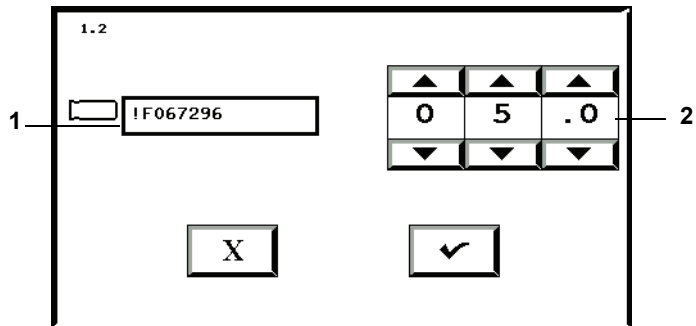


Figure 4-8: Change Maximum Test Time Screen

1 — Bottle ID Field

2 — Max Test Time Scroll Buttons

- 2) Verify the bottle ID matches that of the bottle for which you wish to change the maximum test time.
- 3) Adjust the maximum test time in days using the **Max Test Time** scroll buttons.
- 4) Press the **Check** button to accept the changes, or the **Cancel** button to retain the original setting.

The system returns to the Load Mode screen.

Note: Changing the maximum test time of an individual bottle during loading does not affect any other bottles of the same type.

Note: The maximum test time of an individual bottle can also be changed from the Edit Bottle Detail screen after loading (see [Viewing/Editing Bottle Data in Chapter 6](#)).

Handling Anonymous Bottles

Bottles loaded into the Incubation Module without accessing the Load Bottles function on the Main screen are referred to as Anonymous bottles because they are not associated with a bottle ID.

- Anonymous bottles should be removed and identified as specified in [Unloading Bottles on page 4-16](#) or identified using the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 6](#)).
- Anonymous bottles are assigned the standard default algorithm.

Unloading Bottles

The BacT/ALERT® 3D signals which type of bottles are ready for unloading by enabling the appropriate Unload button.



CAUTION: In order to preserve test data integrity, handle only one bottle at a time. It is important to complete the procedure for each bottle before proceeding to the next bottle.

Unloading Bottles



- 1) Generate an Unload report (see [Viewing, Printing, and Exporting Test Data on page 4-24](#)).
- 2) From the Main screen (see [Figure 4-1, Main Screen, on page 4-3](#)), press the appropriate **Unload** button.
 - The Unload Mode screen appears (see [Figure 4-9](#)).
 - Green indicators illuminate on drawers containing bottles of the selected unload type.

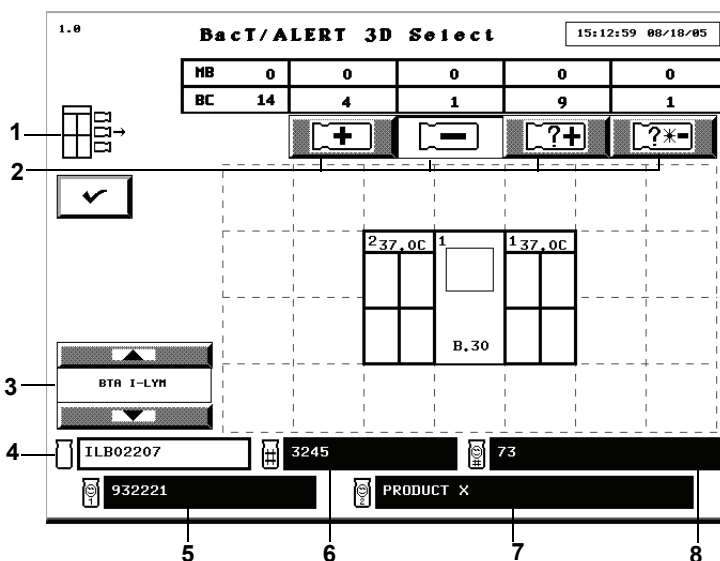


Figure 4-9: Main Screen — Unload Mode

- | | |
|-------------------------------|---------------------|
| 1 — Unload Bottles Icon | 5 — User Defined 1 |
| 2 — Unload Buttons | 6 — Sample ID Field |
| 3 — Bottle Type Scroll Button | 7 — User Defined 2 |
| 4 — Bottle ID Field | 8 — User Defined 3 |

The **User Defined 3**, **User Defined 1**, and **User Defined 2** fields are unavailable with the BacT/ALERT® 3D Signature configuration.

The fields may manually be made unavailable or disabled regardless of software configuration. To make the field(s) unavailable, contact your local bioMérieux Representative.

- 3) Open the indicated drawer. When the indicated drawer is open, cell indicator lights light up next to all bottles in the selected category.
- 4) Remove one of the bottles indicated. Wait for the cell light to blink slowly to acknowledge the removal of the bottle.
- 5) If the bottle was identified when loaded:
 - The bottle ID and bottle type will appear on the Unload Mode screen; however, the sample ID, User Defined 3, User Defined 1, and User Defined 2 will appear in disabled text fields, if information is available.
 - a. It is not necessary to re-scan the bottle ID; however, doing so will verify that bottle's identity.

- b. If fields are blank or need editing, then use the Edit Data Relationships function (see [Editing Data Relationships in Chapter 6](#)).
- 6) Go to [Step 8](#).
- 7) If the bottle was anonymously loaded (the **Bottle ID** field is blank), scan or manually enter the bottle ID.
 - a. Identify the bottle by entering the bottle ID, bottle type, sample ID, User Defined 3, User Defined 1, and User Defined 2.
 - The User Defined 1 and User Defined 2 entries cannot be made without a User Defined 3 entry.

Scanning the bottle ID successfully results in two short beeps.

Scanning the sample ID successfully results in three short beeps.

When identifying anonymous bottles, information entered in the **Bottle ID**, **Bottle Type**, **Sample ID**, **User Defined 3**, **User Defined 1**, and **User Defined 2** fields, and with the **Bottle Type** scroll button, is associated with the unloaded bottle once the next bottle is unloaded or the **Check** button is pressed.

- b. If the bottle is to be reloaded, immediately return the bottle to the cell with the slowly blinking cell indicator light before unloading another bottle.

WARNING



Bottles with a critical determination in progress will be temporarily removed from the bottle count table on the Main screen.

- c. Do not reload the bottle if its status is negative, positive, or if it is necessary to use the bottle label to enter information into the bioMérieux data management computer.



CAUTION: Reloading bottles anonymously that were previously loaded will result in duplicate bottle records.

- 8) Repeat [Step 3](#) through [Step 5](#) for the remaining bottles to be unloaded. Limit the bottle unload time to no more than 2 minutes in one area. Close the drawer to allow temperature to equilibrate before unloading from the area again. See [Appendix C, Best Practices](#) for unloading bottles and preventing false positives.



CAUTION: If a large number of bottles are unloaded from the Incubation Module at the same time and in the same areas, a large heat mass loss within the racks may occur. This heat loss may trigger the acceleration or rate algorithms to erroneously flag positive.

- 9) When finished unloading bottles, ensure that all drawers are completely closed.
- 10) Press the **Check** button on the Unload Mode screen.
- 11) Verify that the bottles listed on the Unload report were unloaded.
- 12) Where applicable, enter *User Defined 3* and *sample* data associated with the unloaded bottles into the bioMérieux data management computer.
- 13) Reload any previously Anonymous negative-to-date bottles.

If no operator or bottle loading activity has been recorded in a period of two minutes, the instrument will terminate the Unload Bottle operation. Operator activity includes:

- pressing keys on the keyboard
- scanning barcodes
- touching the Operator Panel
- loading or unloading bottles

Handling Unconfirmed Positive Bottles (False Positives)

If a smear of a positive bottle reveals no microorganisms, the bottle should be subcultured and reloaded into the instrument via the Load Bottles function (see [Loading Bottles on page 4-11](#)).

Note: *If a bottle is reloaded into the instrument, its status will revert to negative-to-date once a reading has been taken (maximum time – 10 minutes).*

If growth appears on the subculture, edit the bottle's status to Positive on the Edit Test Result screen which is accessed from the Edit Bottle Detail screen (see [Edit Test Result Button in Chapter 6](#)), and unload the now positive bottle.

Note: *Results that have been manually changed to negative or positive via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 6](#)), will be marked on the report with a stick figure (☒).*

Note: *If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (☒) will not appear.*


Accessing the Setup Screen Function Buttons

To view and print data, print graphs, and perform all of the editing, configuration, and maintenance functions of the system, you must first access the Setup screen function buttons.

Accessing the Setup Screen



- 1) From the Main screen (see [Figure 4-1, Main Screen, on page 4-3](#)), press

the **Next Screen** button ().

The Setup screen appears.

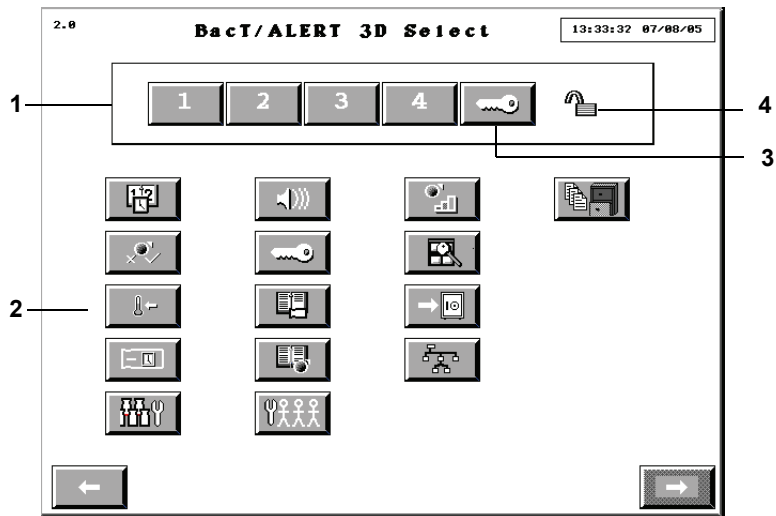


Figure 4-10: Setup Screen

- 1 — Password Entry Keypad
- 2 — Function Buttons
- 3 — Key Symbol Button
- 4 — Padlock Icon

- 2) Enter a valid password using the 1 – 4 buttons. Until a valid password is entered and accepted, the **Padlock** icon will appear in the closed position.

Note: Acceptable passwords consist of any combination of the numbers 1 to 4 and have a maximum length of eight characters.

Note: Instruments are shipped with a password of 1234. For information on changing the default password, [see Changing the System Password in Chapter 7](#).

- 3) Press the **Key Symbol** button to accept the password.
- 4) After a valid password is accepted, the **Padlock** icon changes to the full open position and the function buttons become enabled.

Note: To correct an error made while entering a password, press the **Key Symbol** button and re-enter the password.

Note: If more than 8 characters are entered, the **Password Entry** buttons will become inactive and turn gray. Press the **Key Symbol** button and re-enter the password.

Inactivity Timeout for all Setup Screens

While you are in the Setup screen (see [Figure 4-10, Setup Screen](#)) or one of its sub-menus, an inactivity timeout will occur (within a period of time configured by your bioMérieux Service Representative) if you do not perform one of the following actions:

- press a screen or keyboard button
- scan a barcode
- load or unload a bottle

If an inactivity timeout occurs, the instrument display reverts from the currently displayed screen to the Main screen. Any pending function is cancelled as if the **Cancel** button on each successive screen was pressed.


Note: If a timeout occurs, it is possible that partially entered information will be lost.

Note: The inactivity timeout feature is disabled while a red operator error is displayed on the screen.


Setup Screen Function Buttons



Set Date/Time button (see [Setting and Formatting the System Date and Time in Chapter 8](#))

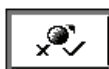

Enable/Disable Module, Drawer, Rack, or Cell button (see [Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#))

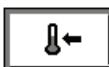

Calibrate Module Temperature button (see [Adjusting an Incubation or Combination Module's Temperature in Chapter 8](#))



Set Maximum Test Time button (see [Setting the Maximum Test Time in Chapter 7](#))



Set Audible Alarm Options button (see [Setting the Audible Alarms in Chapter 7](#))



Change Password button (see [Changing the System Password in Chapter 7](#))



Select Bottle to Edit/Graph button (see [Selecting Bottles Using the Select Bottle to Edit/Graph Button in Chapter 6](#))



Edit Cell Contents button (see [Selecting Bottles Using the Edit Cell Contents Button in Chapter 6](#))



Calibrate Cell button (see [Calibrating an Instrument Cell in Chapter 8](#))



View Incubation Module Information button (see [Viewing Incubation Module Information in Chapter 8](#))



Backup Management button (see [Initiating Manual Backup in Chapter 7](#))



Edit Data Relationships button (see [Initiating the Edit Data Relationships Function in Chapter 6](#))



Report button — Select and SelectLink only (see [Viewing, Printing, and Exporting Test Data on page 4-24](#))



Configure Users button — Button only appears in 21 CFR Part 11 mode (see [Configuring Users — 21 CFR Part 11 Mode in Chapter 10](#))



Bottle Processing/Customization button (For use only with instruction from bioMérieux)

Viewing and Printing

Introduction

Viewing Bottle Data

The following bottle information can be viewed by accessing the Edit Bottle Detail screen as described in Editing Test Data ([see Viewing/Editing Bottle Data in Chapter 6](#)):

- Bottle ID
- Sample ID
- User Defined 3 (where applicable)
- User Defined 1 (where applicable)
- User Defined 2 (where applicable)
- Cell ID
- Maximum test time
- Bottle type
- Date/time loaded
- Date/time unloaded
- Date/time of last bottle reading
- Test time
- Test result
- Algorithm type
- How determined/positivity index

Viewing/Printing Reports

With a BacT/ALERT® 3D Signature configuration, reports are viewed and printed at the data management computer.

With the BacT/ALERT® 3D Select or SelectLink configurations, reports are viewed and printed by using the **Report** button as described in the topic, [Viewing, Printing, and Exporting Test Data on page 4-24](#).

Viewing/Printing Graphs

Bottle Graphs can be viewed and printed on the BacT/ALERT® 3D Select and SelectLink configurations as described in the topic, [Viewing and Printing Bottle Graphs on page 4-31](#). For the Signature configuration, graphics can be viewed as described in the topic, [Viewing and Printing Bottle Graphs on page 4-31](#), or at the data management computer. Since the BacT/ALERT® 3D Signature configuration is not connected to a printer, you must use the data management computer to print a graph.

Note: *The print feature can be made unavailable regardless of software configuration. When the feature is unavailable, the **Print** buttons do not*

display. To make the print feature unavailable, contact your local bioMérieux Representative.

Using the Print Screen Function

You can print the current screen on the instrument by pressing **Ctrl + P** on the keyboard.

Note: Select and SelectLink configurations only.

Viewing, Printing, and Exporting Test Data

Note: If you have a BacT/ALERT® 3D Signature configuration, you must use the data management computer to view and print reports.



- 1) From the Setup screen (see [Figure 4-10, Setup Screen, on page 4-20](#)),

press the **Report** button ().

Note: The **Report** button is not available with the BacT/ALERT® 3D Signature configuration.

The Report Selection screen appears.

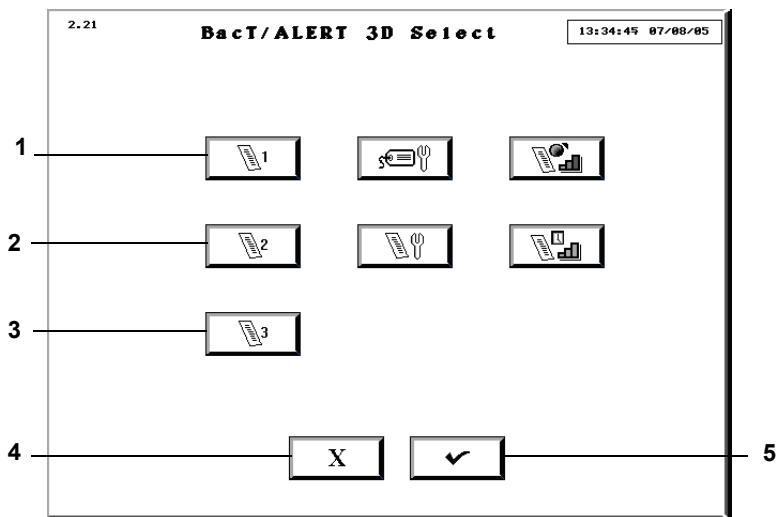


Figure 4-11: Report Selection Screen

- | | |
|-----------------------------|----------------------------|
| 1 — Display Report 1 Button | 4 — Previous Screen Button |
| 2 — Display Report 2 Button | 5 — Check Button |
| 3 — Display Report 3 Button | |

- 2) Press the **Display Report 1, 2, or 3** button that is configured for the information desired.

The Report screen will display. There are three default report configurations:

- **Display Report 1** button — Generates the Load Report screen with **1st Load Time** as the primary sort, and **Sample ID** as the secondary sort. Report has section breaks based on 1st Load Time.
- **Display Report 2** button — Generates the Status Report screen with **Sample ID** as the primary sort, and **Bottle Type** as the secondary sort. Report has section breaks based on **Sample ID**.
- **Display Report 3** button — Generates the Unload Report screen with **Loaded** as the primary sort, and **Test Result** as the secondary sort. Report has section breaks based on **Loaded** and **Test Result**.

See [Configuring Report Contents in Chapter 7](#) for examples of the Load, Status and Unload Report Configuration screens.

Note: *Data from the last 1,920 bottles are displayed each time a Report screen is accessed. The report screens come configured with default settings, but the report configurations can be changed to display different data and to sort the data differently (see [Configuring Report Contents in Chapter 7](#)).*

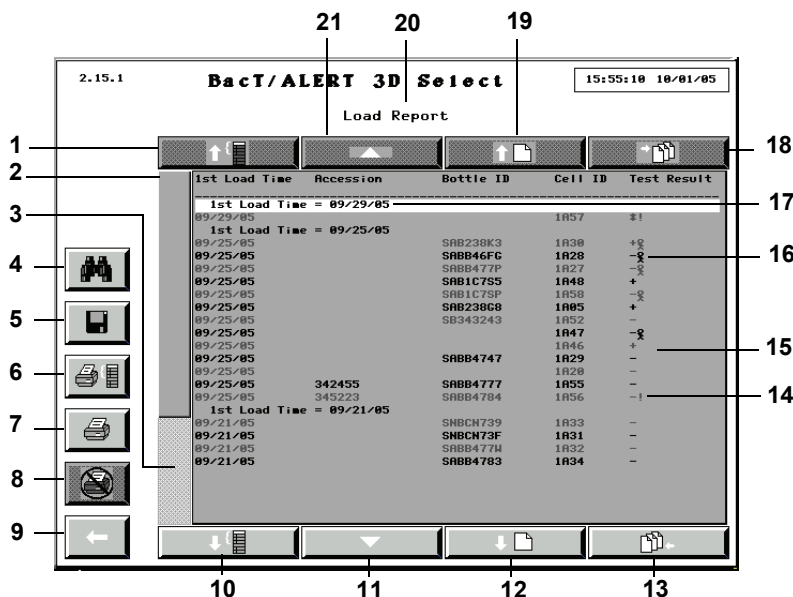


Figure 4-12: Sample Report Screen

- | | |
|--------------------------------------|------------------------------|
| 1 — Section Scroll Up Button | 11 — Line Scroll Down Button |
| 2 — Relative Record Scroll Indicator | 12 — Page Scroll Down Button |
| 3 — Relative Record Scroll Bar | 13 — End Scroll Button |
| 4 — Find Text Button | 14 — Gap Detection Indicator |
| 5 — Save Button | 15 — Report Data Lines |
| 6 — Print Current Section Button | 16 — Stick Figure |
| 7 — Print Report Button | 17 — Current Section Line |
| 8 — Cancel Print Button | 18 — Home Scroll Button |
| 9 — Previous Screen Button | 19 — Page Scroll Up Button |
| 10 — Section Scroll Down Button | 20 — Report Title |
| | 21 — Line Scroll Up Button |

Note: Results that have been manually changed to negative or positive (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 6](#)) will be marked on the report with a stick figure (⌘).

Note: If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (⌘) will not appear.

Note: *Bottles with an Instrument Fault Code 80 will be marked on the report with a **Gap Detection** indicator (!) next to the result. If a negative bottle has an Instrument Fault Code 80 while it is being unloaded, the **Gap Detection** indicator will remain on the report (see [Instrument Fault Codes in Chapter 9](#)).*

Note: *The Current Group Line will always display the group that is associated with the first displayed data record. It is always highlighted for easy reference. If there are no section breaks in the displayed report, this line will become the first line of data records displayed and reference highlighting will not occur.*

3) To scroll up or down a group, press the appropriate **Group** scroll button.

Note: *The **Group** scroll buttons are disabled if there are no section breaks in the displayed report or if there are no available sections in the indicated direction.*

4) To scroll up or down a data line, press the appropriate **Line** scroll button or press the appropriate ↑ or ↓ key on the keyboard.

Note: *The **Line** scroll buttons are disabled if there are no data lines in the indicated direction.*

5) To scroll up or down a data page, press the appropriate **Page** scroll button or press the appropriate **Page Up** or **Page Down** key on the keyboard.

Note: *The **Page** scroll buttons are disabled if there are no data lines in the indicated direction.*

6) To scroll up to the oldest data line (the first data record in the report), press the **Home** scroll button or press the **Home** key on the keyboard.

Note: *The **Home** scroll button is disabled if there are no report data lines in the indicated direction.*

7) To scroll down to the newest data line (the last data record in the report), press the **End** scroll button or press the **End** key on the keyboard.

Note: *The **End** scroll button is disabled if there are no report data lines in the indicated direction.*

8) To move to a relative record position, press the **Relative Record** scroll bar above/below the Relative Record Indicator.

Note: *The Relative Record Indicator is sized proportionally to the amount of records in the report.*

Note: *The **Relative Record** scroll bar is disabled if the Relative Record Indicator is the same size as the scroll bar and all the data records in the report are displayed.*

9) To print the report, press the appropriate **Print** button:

- Pressing the **Print Report** button prints all records in the database (with a maximum of 1,920 records).
- Pressing the **Print Current Group** button prints the Current Group Line and all data records associated with the group.

Note: *The **Print** buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.*

10) While the report is printing:

- All **Print** and **Save** buttons are disabled.
- The operator may, however, view and scroll through the displayed report, search for text in the displayed report, or exit the Report screen and perform other operations.
- The **Cancel Print** button becomes available.

11) Press the **Cancel Print** button to stop sending data to the printer and to empty the queue of any data waiting to be sent to the printer.

Note: *Once the cancellation has been completed, the **Print** and **Save** buttons will become enabled, and the **Cancel Print** button will be disabled.*

12) To specify a text string and initiate a search of the report data for the specified text, press the **Find Text** button. The Find Text screen will appear.

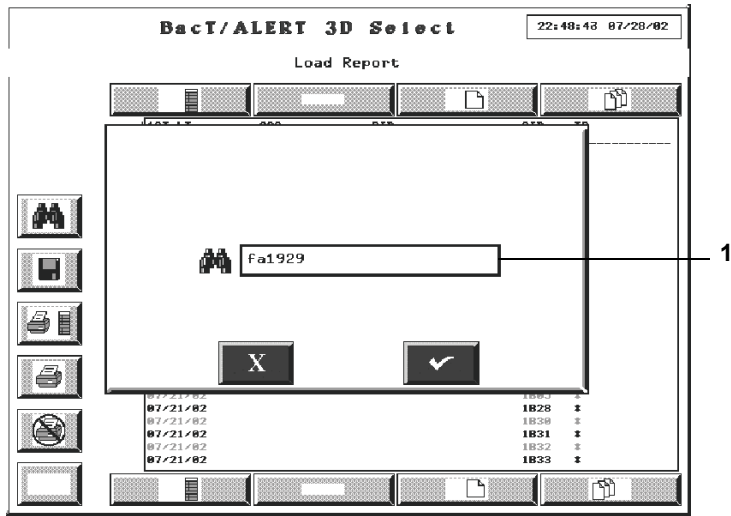


Figure 4-13: Find Text Screen

1 — Find Text Field

Use the keyboard to enter the text you wish to seek, and press the **Check** button to return to the Report screen. Pressing the **Cancel** button will cancel the search request and return to the Report screen.

Note: The search will be performed upon returning to the report screen starting from the top of the report. The record that contains the first occurrence of the text will be scrolled to the first line of data displayed and the text itself will be highlighted for reference.

If no occurrence of the specified text is found, the data displayed will not change and no reference highlighting will occur.

13) To find the next instance of the specified text, press the **F3** key on the keyboard.

Note: If no new instance of the text is found the data displayed, and the highlighted text will remain the same.

14) To save the displayed report to a text file, press the **Save** button.

The Save To File screen appears.

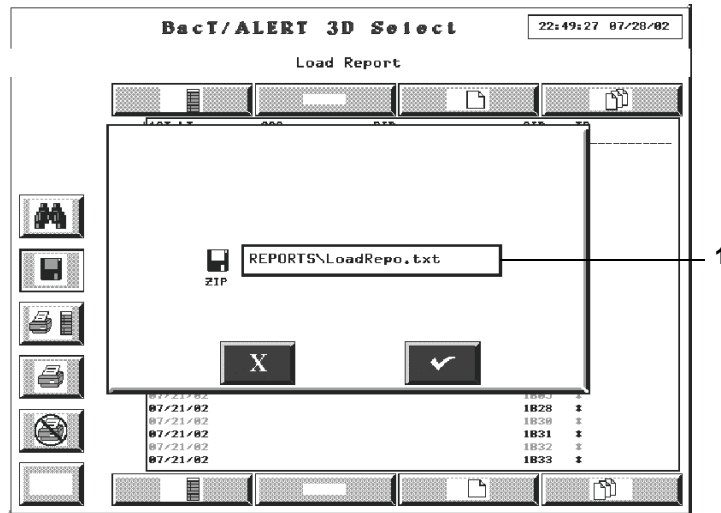


Figure 4-14: Save to File Screen

1 — File Name Field

Note: The default file name and path will appear automatically in the **File Name** field. Use the keyboard to change the name of the file.

Note: All reports are saved to the pathname, **D:\REPORTS**, either using the default file name or a specified file name. All reports are automatically given an extension of **.TXT**.

15) Place the backup media in the backup drive.

Note: Backup media can either be a Zip[®] disk or a USB flash drive.

16) Press the **Check** button to initiate the save and return to the Report screen. Pressing the **Cancel** button will cancel the save request and return to the Report screen.

Note: The **Print** buttons will be disabled while the save is taking place. Once the save has completed, the **Print** buttons will be enabled.

WARNING



Do not insert any object, other than a Zip[®] disk, into the Zip[®] drive under any circumstances.



CAUTION: Do not forcibly remove the Zip[®] disk from the instrument. Forcibly removing the Zip[®] disk may cause damage to the Zip[®] disk or Zip[®] drive and may cause the system to lock up.

- 17) Press the **Previous Screen** button to return to the screen from which the Report screen was accessed.

Viewing and Printing Bottle Graphs

Note: *If you have a BacT/ALERT[®] 3D Signature configuration, you must use the data management computer to print a bottle graph since the BacT/ALERT[®] 3D Signature configuration is not connected to a printer.*



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons on page 4-20](#)).

- 2) Press the **Select Bottle to Edit/Graph** button ().

The Select Bottle to Edit/Graph screen overlays and disables the Setup screen.

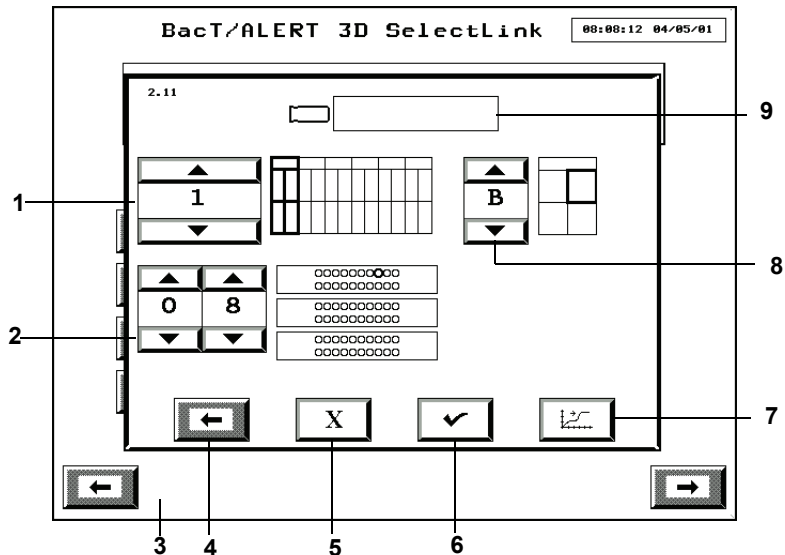


Figure 4-15: Select Bottle to Edit/Graph Screen

- | | |
|-------------------------------------|----------------------------------|
| 1 — Incubation Module Scroll Button | 6 — Check Button |
| 2 — Cell Scroll Buttons | 7 — Graph Bottle Readings Button |
| 3 — Setup Screen (Disabled) | 8 — Drawer Scroll Button |
| 4 — Previous Screen Button | 9 — Bottle ID Field |
| 5 — Cancel Button | |

- 3) If you know the bottle ID of the bottle whose readings you wish to graph, then enter the *bottle ID* in the **Bottle ID** field (see [Text/Data Entry](#) on [page 4-9](#)) and go to [Step 5](#). If you know the cell location but not bottle ID, then continue to [Step 4](#).

Note: Only the last 1,920 bottles loaded can be retrieved. Any other entry in the **Bottle ID** field is invalid and will cause an Operator Error 940 (see [Operator Error Codes](#) in [Chapter 9](#)).

- 4) Adjust the **Incubation Module** (1–6), **Drawer** (A–D), and **Cell** (1–60) scroll buttons to select the cell location of the bottle whose readings you wish to graph. The cell location defaults to Module 1, Drawer A, Cell 1 (1A01).

Note: Only cell locations with currently loaded bottles can be used to view bottle graphs. To view graphs of recently unloaded bottles (the most recent 1,920 bottles loaded), you must use the **Bottle ID** field.

- 5) Press the **Graph Bottle Readings** button ().

The Graph Bottle Readings screen appears (see [Figure 4-16](#)).

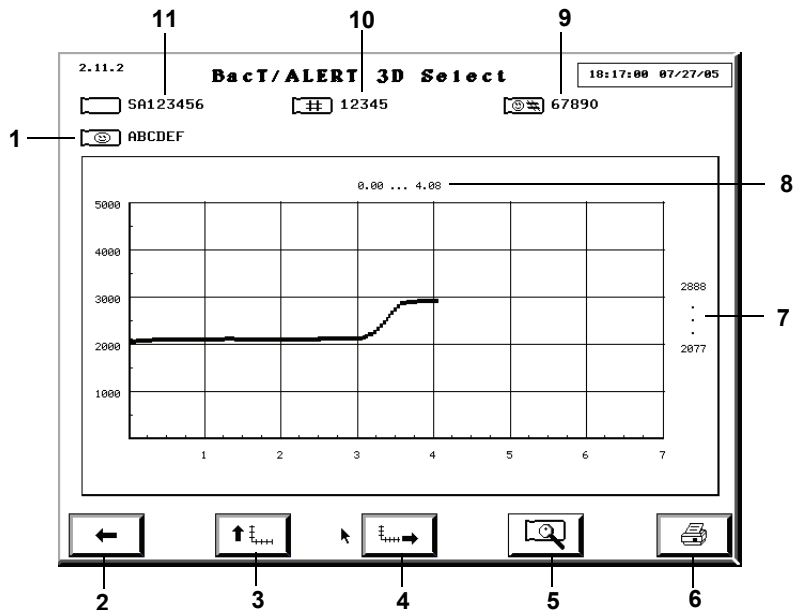


Figure 4-16: Graph Bottle Readings Screen

- | | |
|------------------------------------|--------------------------------------------|
| 1 — User Defined 1/User Defined 2* | 7 — Bottle Readings Range |
| 2 — Previous Screen Button | 8 — Days Tested Range |
| 3 — Adjust Y Scale Button | 9 — User Defined 3* |
| 4 — Adjust X Scale Button | 10 — Sample ID Field |
| 5 — Bottle Readings Button | 11 — Bottle ID Field |
| 6 — Print Graph Button | * Available in Select and SelectLink only. |

You can also access the Graph Bottle Readings screen via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen in Chapter 6](#)). The screen ID number is the same regardless of how it is accessed.

- The Y axis range defaults to 0 – 5000, and the X axis range defaults to 0 – maximum test time of the bottle in days.
- If the bottle was determined positive, then a dot and the time to detection (days) are displayed at the point where it was determined positive.

- 6) To adjust the Y or X axis, click the appropriate **Adjust Scale** button. When one of these buttons is clicked, the axis re-scales so that the maximum endpoint of the scale is just higher than the maximum value of the range. Adjusting the scale may mean increasing or decreasing the range of the scale.

When one of the **Adjust Scale** buttons is clicked, the arrow on the button changes directions. To return to the original scale, click the **Adjust Scale** button(s) a second time.

- 7) Press the **Print Graph** button, if available, to print the graph as it appears on the screen.

Note: The **Print Graph** button is only available if a printer is configured for the system. The **Print Graph** button is disabled while a print is in progress.

Note: To configure a printer for the system, contact your local bioMérieux Representative.

- 8) When done, press the **Previous Screen** button to return to the Select Bottle to Edit/Graph screen.

Display Bottle Readings




The Bottle Readings screen displays the exact values of the bottle readings for an individual bottle, along with the date and time each reading was taken.

Note: The polynomial is applied to the bottle readings. The readings are not the raw readings.

If there are bottle readings available for you to view, the **Bottle Readings** button on the Graph Bottle Readings screen turns blue (see [Figure 4-16](#)). If there are no readings available, the button is gray.

- 1) From the Graph Bottle Readings screen (see [Figure 4-16](#)), press the

Bottle Readings button ().

The Bottle Readings screen appears.

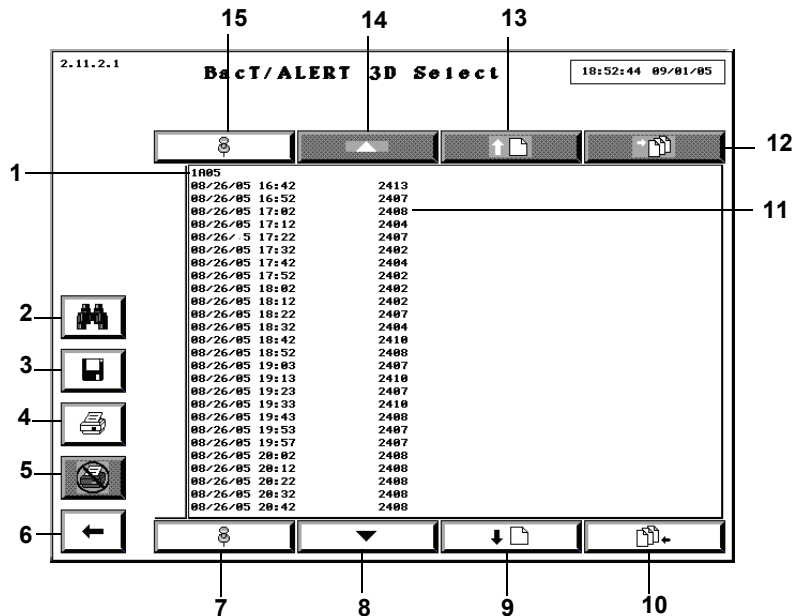


Figure 4-17: Bottle Readings Screen

- | | |
|----------------------------------|-----------------------------------------|
| 1 — Bottle Cell Location | 9 — Page Scroll Down Button |
| 2 — Find Text Button | 10 — End Scroll Button |
| 3 — Save Button | 11 — Bottle Reading (Date/Time/Reading) |
| 4 — Print Button | 12 — Home Scroll Button |
| 5 — Cancel Print Button | 13 — Page Scroll Up Button |
| 6 — Previous Screen Button | 14 — Line Scroll Up Button |
| 7 — Anchor Display Bottom Button | 15 — Anchor Display Top Button |
| 8 — Line Scroll Down Button | |

- 2) To scroll up or down a data line, press the appropriate **Line** scroll button or press the appropriate **↑** or **↓** key on the keyboard.

Note: The **Line** scroll buttons are disabled if there are no data lines in the indicated direction.

- 3) To scroll up or down a data page, press the appropriate **Page** scroll button or press the appropriate **Page** key on the keyboard.

Note: The **Page** scroll buttons are disabled if there are no data lines in the indicated direction.

- 4) To scroll up to the oldest data line (the first data record in the report), press the **Home** scroll button or press the **Home** key on the keyboard.

Note: *The **Home** scroll button is disabled if there are no report data lines in the indicated direction.*

- 5) To scroll down to the newest data line (the last data record in the report), press the **End** scroll button or press the **End** key on the keyboard.

Note: *The **End** scroll button is disabled if there are no report data lines in the indicated direction.*

- 6) To print the bottle readings, press the **Print** button.

Note: *The **Print** buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.*

- 7) While the report is printing:
 - All **Print** and **Save** buttons are disabled.
 - The operator may, however, view and scroll through the displayed list, search for text in the displayed list, or exit the Bottle Readings screen and perform other operations.
 - The **Cancel Print** button becomes available.
- 8) Press the **Cancel Print** button to stop sending data to the printer and to empty the queue of any data waiting to be sent to the printer.

Note: *Once the cancellation has been completed the **Print** and **Save** buttons will become enabled, and the **Cancel Print** button will be disabled.*

- 9) To specify a text string and initiate a search of the report data for the specified text, press the **Find Text** button.

The Find Text screen appears.

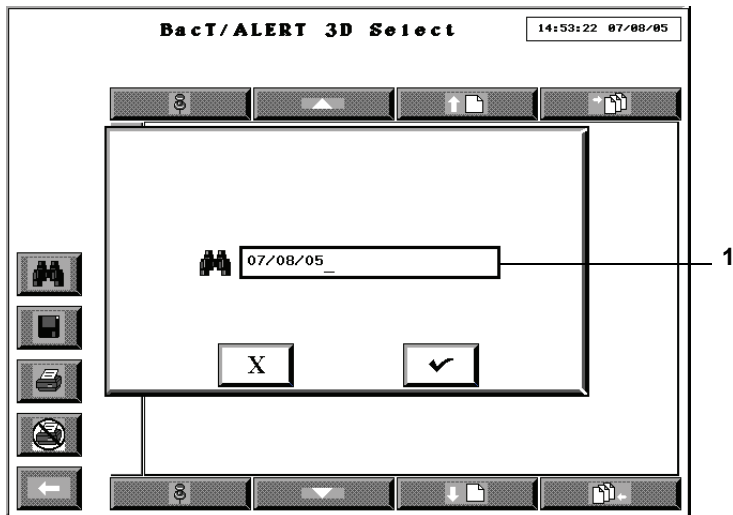


Figure 4-18: Find Text Screen

1 — Find Text Field

Use the keyboard to enter the text you wish to seek, and press the **Check** button to return to the Bottle Readings screen. Pressing the **Cancel** button will cancel the search request and return to the Bottle Readings screen.

Note: The search will be performed upon returning to the report screen starting from the top of the report. The record that contains the first occurrence of the text will be scrolled to the first line of data displayed and the text itself will be highlighted for reference.

If no occurrence of the specified text is found, the data displayed will not change and no reference highlighting will occur.

- 10) To find the next instance of the specified text, press the **F3** key on the keyboard.

Note: If no new instance of the text is found, the data displayed, and the highlighted text will remain the same.

- 11) To save the bottle readings to a text file, press the **Save** button.

The Save To File screen appears.

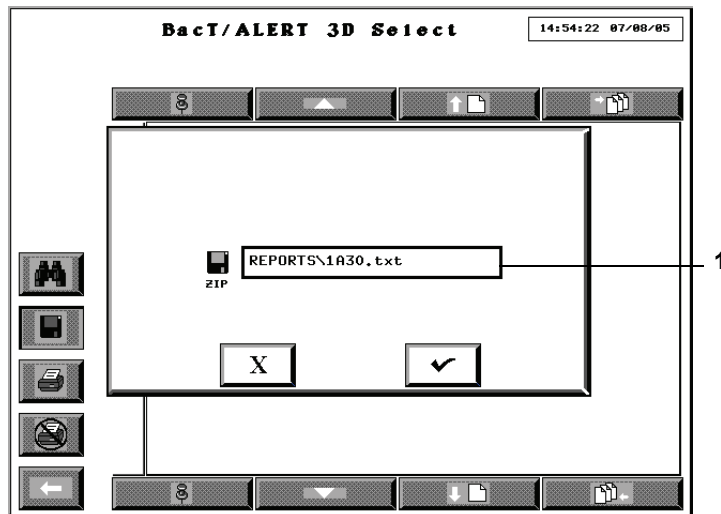


Figure 4-19: Save to File Screen

1 — File Name Field

Note: The default file name and path will appear automatically in the **File Name** field. Use the keyboard to change the name of the file.

Note: All reports are saved to the pathname, **D:\REPORTS**, either using the default file name or a specified file name. All reports are automatically given an extension of **.TXT**.

12) Place the backup media in the backup drive.

Note: Backup media can either be a Zip[®] disk or a USB flash drive.

13) Press the **Check** button to initiate the save and return to the Bottle Readings screen. Pressing the **Cancel** button will cancel the save request and return to the Bottle Readings screen.

Note: The **Print** buttons will be disabled while the save is taking place. Once the save has completed, the **Print** buttons will be enabled.

WARNING



Do not insert any object, other than a Zip[®] disk, into the Zip[®] drive under any circumstances.



CAUTION: Do not forcibly remove the Zip® disk from the instrument. Forcibly removing the Zip® disk may cause damage to the Zip® disk or Zip® drive and may cause the system to lock up.

- 14) Press the **Previous Screen** button to return to the screen from which the Bottle Readings screen was accessed.
- 15) When done, press the **Previous Screen** button to return to the Graph Bottle Readings screen.

Sending/Requesting LIS Information

Results can be manually sent to, and demographics can be manually requested from, an LIS when the BacT/ALERT® 3D SelectLink configuration is used.

Note: Data is transmitted according to the BacT/LINK® protocol. For more information, contact your local bioMérieux Representative.

Note: The system can be configured to perform these functions automatically. To adjust these settings, contact your local bioMérieux Representative.

Note: The arrows on the **Manual Send Test Results** and **Manual Request Test Orders** buttons also serve as an indicator of automatic data transfer.

Sending Results to the LIS



- 1) From the Main screen (see [Figure 4-1, Main Screen, on page 4-3](#)), press

the **Manual Send Test Results** button ().

The arrow on the button face will blink during the transfer.

Note: Only accessions with one or more bottles that have had a status change to positive, or that have reached maximum test time with a negative status and have been unloaded, since the last transfer will be sent.

Requesting Information from the LIS



- 1) From the Main screen (see [Figure 4-1, Main Screen, on page 4-3](#)), press

the **Manual Request Test Orders** button ().

The arrow on the button face will blink during the transfer.

About This Chapter

This chapter provides you with procedures for viewing and editing bottle data for Clinical Use.

Chapter Contents

Viewing/Editing Bottle Data • 5-2

Introduction • 5-2

Selecting Bottles Using the Edit Cell Contents Button • 5-3

Selecting Bottles Using the Select Bottle to Edit/Graph Button • 5-4

Editing Bottle Details Using the Edit Bottle Detail Screen • 5-5

Editing Data Relationships • 5-12

Introduction • 5-12

Initiating the Edit Data Relationships Function • 5-12

Editing Bottle ID to Accession Number Relationships • 5-14

Editing Accession Number to Hospital ID Relationships • 5-16

Editing Hospital ID to Patient Name Relationships • 5-17

Viewing/Editing Bottle Data

Introduction



By accessing the Edit Bottle Detail screen of both currently loaded and recently unloaded bottles, you can perform the following bottle editing functions:


- Identifying an anonymous bottle
- Editing a bottle ID
- Editing the maximum test time of a currently loaded bottle
- Editing a bottle type
- Manually changing a bottle's test result
- Unloading a bottle from a faulty rack

Note: *The bottle record must be one of the 1,920 bottle records stored in the database.*

- 1) Access the Setup screen and enter a valid password ([see Accessing the Setup Screen Function Buttons in Chapter 3](#)).
- 2) Access the Edit Bottle Detail screen for a specific bottle. There are two methods you can use to access the Edit Bottle Detail screen:

- Select loaded bottles using the **Edit Cell Contents** button (). See [Selecting Bottles Using the Edit Cell Contents Button on page 5-3](#).

or

- Select loaded and recently unloaded bottles using the **Select Bottle to Edit/Graph** button (). See [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 5-4](#).

- 3) Edit the bottle details as necessary ([see Editing Bottle Details Using the Edit Bottle Detail Screen on page 5-5](#)).
- 4) Press the **Check** button to save the changes and exit the screen or press the **Cancel** button to restore the information originally displayed and exit the screen.

Selecting Bottles Using the Edit Cell Contents Button



Note: This function can only be used for currently loaded bottles. To view information of recently unloaded bottles, select the bottle from the Select Bottle to Edit/Graph screen (see [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 5-4](#)).

- 1) From the Setup screen (see [Figure 3-10, Setup Screen](#)), press the **Edit Cell Contents** button.

The Edit Cell Contents screen appears.

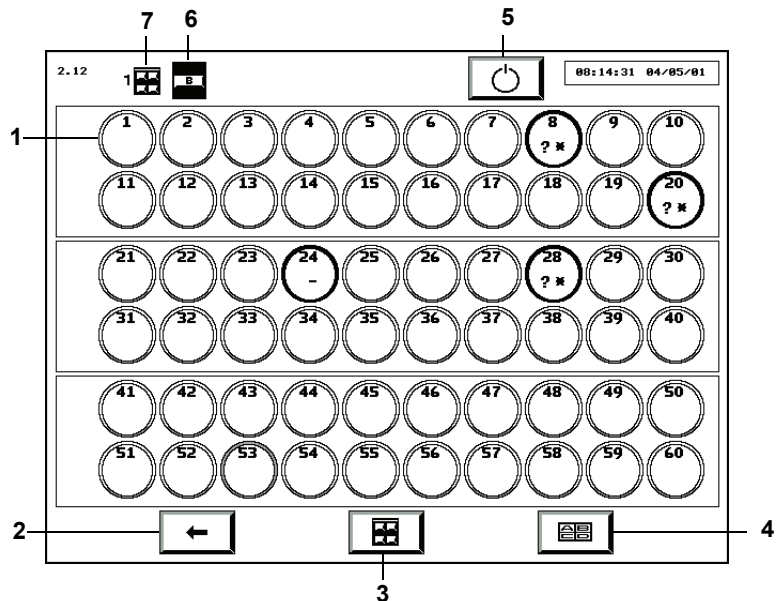


Figure 5-1: Edit Cell Contents Screen

- 1 — Cell
- 2 — Previous Screen Button
- 3 — Incubation Module Selection Button
- 4 — Drawer Selection Button
- 5 — Restart Incubation Module Button
- 6 — Drawer Indicator
- 7 — Incubation Module Indicator

Note: The Edit Cell Contents screen displays the same information as the View Cell Status screen (see [Understanding the View Cell Status Screen Display in Chapter 3](#)) and is continuously updated.

- 2) Select the desired drawer and Incubation Module using the **Incubation Module Selection** and **Drawer Selection** buttons.
- 3) Touch the appropriate **Cell** icon.

The Edit Bottle Detail screen appears (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 5-5](#)).

Selecting Bottles Using the Select Bottle to Edit/Graph Button



- 1) From the Setup screen (see [Figure 3-10, Setup Screen](#)), press the **Select Bottle to Edit/Graph** button.

The Select Bottle to Edit/Graph screen overlays and disables the Setup screen.

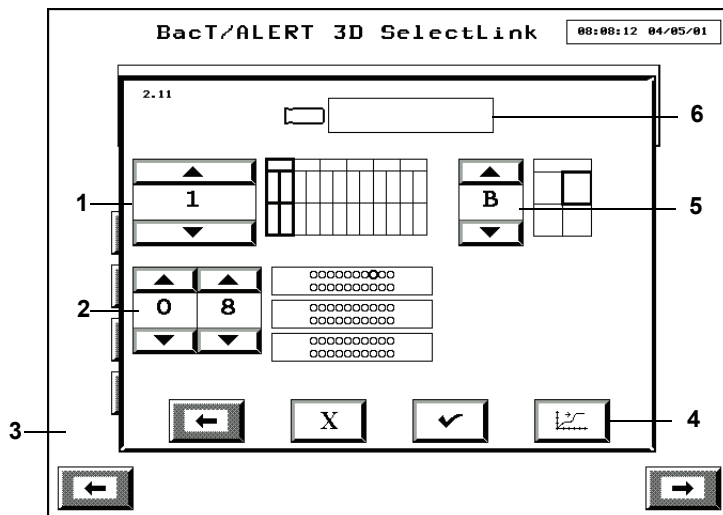


Figure 5-2: Select Bottle to Edit/Graph Screen

- 1 — Incubation Module Scroll Button
 - 2 — Cell Scroll Buttons
 - 3 — Setup Screen (Disabled)
 - 4 — Graph Bottle Readings Button
 - 5 — Drawer Scroll Button
 - 6 — Bottle ID Field
- 2) If the bottle ID is known, enter the *bottle ID* in the **Bottle ID** field (see [Text/Data Entry in Chapter 3](#)) and proceed to [Step 4](#).

Note: Only the last 1,920 bottles loaded can be retrieved. Any other entry in the **Bottle ID** field is invalid and will cause an Operator Error 940 (see [Operator Error Codes in Chapter 9](#)).

- 3) If the cell location (but not bottle ID) is known, adjust the **Incubation Module** (1–6), **Drawer** (A–D), and **Cell** (1–60) scroll buttons to select the appropriate bottle location.

Note: Only cell locations with currently loaded bottles can be used to view/edit bottle data. To view data of recently unloaded bottles, you must use the **Bottle ID** field.

- 4) Press the **Check** button.

The Edit Bottle Detail screen appears (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 5-5](#)).

Editing Bottle Details Using the Edit Bottle Detail Screen

Note: The bottle record can be edited before and after the bottle has been unloaded from the system.

If a bottle has not already been selected, you can select a bottle to edit using either the **Edit Cell Contents** or the **Select Bottle to Edit/Graph** button as described in [Selecting Bottles Using the Edit Cell Contents Button on page 5-3](#), and [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 5-4](#).

The screenshot shows the 'BacT/ALERT 3D Select' interface. At the top, it displays '2.12.1' and '18:42:09 09/13/05'. The main area contains several input fields and buttons:


- Top Left:** A text field containing 'SFBPSPXZ'.
- Top Right:** A field containing '34245'.
- Middle Left:** A field containing '31.3309'.
- Middle Right:** A field containing 'JOHN'.
- Bottom Left:** A field containing 'SMITH'.
- Bottom Center:** A numeric keypad with buttons for '0', '7', and '.0', along with directional arrows.
- Bottom Right:** A section with a date/time stamp '08/26/05 15:14:18 (18.14)' and a list of test results:
 - 09/13/05 18:40:06 (0.00)
 - 09/13/05 18:36:26 (0.00)
 - 08/26/05 19:58:00 (0.19)
- Bottom Bar:** Contains buttons for 'X', a checkmark, a graph icon, and a status bar showing '12/2', '8/0', and '1'.

Figure 5-3: Edit Bottle Detail Screen

The screen ID is determined by the method in which the Edit Bottle Detail screen was accessed. The screen behaves in the same way regardless of how it is accessed.

- The screen ID is 2.11.1 if you access it from the Select Bottle to Edit/Graph screen.
- The screen ID is 2.12.1 if you access it from the Edit Cell Contents screen.

Edit Bottle ID Field

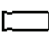


A blank **Bottle ID** field indicates the bottle is currently anonymous. To assign a bottle ID to an anonymous bottle or edit the bottle ID of an identified bottle:

- 1) Touch the **Bottle ID** field (the field turns white).
- 2) Manually enter the bottle ID using the keyboard or scan the barcode ([see Text/Data Entry in Chapter 3](#)).
- 3) Press the **Check** button.

View Accession Number Field



Disabled field that displays the accession number associated with the displayed bottle record.

View Hospital ID Field



Disabled field that displays the hospital ID associated with the displayed bottle record.

View Patient First Name Field



Disabled field that displays the patient first name associated with the displayed bottle record.

View Patient Last Name Field



Disabled field that displays the patient last name associated with the displayed bottle record.

Note: The **View Accession Number**, **View Hospital ID**, **View Patient First Name**, and **View Patient Last Name** fields are “view-only” fields.

Note: The **Hospital ID**, **Patient First Name**, and **Patient Last Name** fields are not available with the BacT/ALERT® 3D Signature configuration.

Note: The fields may be hidden regardless of BacT/ALERT® 3D configuration. To make the field(s) unavailable, contact your local bioMérieux Representative.

Edit Load Status Sidebar Switch

It is possible to change the load status of an identified bottle in a faulty rack from loaded to unloaded. However, it is not possible to change a bottle's load status from unloaded to loaded.

Sliderbar interpretation:

- 0 = unloaded
- 1 = loaded

Note: The sliderbar is disabled if the associated rack is functioning properly.

Edit Maximum Test Time Scroll Buttons

Displays the incubation period for the selected bottle after which the bottle's status will change from negative-to-date to negative.

Note: The maximum test time of a bottle can be edited in days and tenths of a day at any time.


Edit Bottle Type Scroll Button

Displays the media type of the selected bottle. Use the scroll buttons to select the appropriate bottle type.

Note: Anonymous bottles loaded into a **BC** type drawer are assigned to the UNKNOWN bottle type.

Note: The **Bottle ID**, **Load Status**, **Maximum Test Time**, and **Bottle Type** fields can be edited concurrently or one at a time. Press the **Check** button to save changes.

View Cell Location Icon

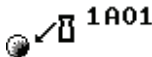


Displays the location of the selected bottle. The location is either the current location or the last cell location, depending on whether the bottle is still loaded.

Cell ID consists of the following three components in order:


- Incubation Module #
- Drawer letter
- Cell #

View First Loaded Time Icon

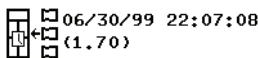


Displays the date and time the selected bottle was first loaded into the system. The number of days since the bottle was first loaded appears in parenthesis below the date and time.

View Last Unloaded Time Icon

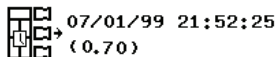


Displays the date and time the selected bottle was last unloaded from the system. The number of days since the bottle was unloaded appears in parenthesis below the date and time. No time will be displayed if the bottle has never been unloaded.

View Time of Last Bottle Reading Icon

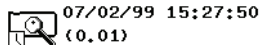

Displays the date and time of the last bottle reading that was successfully processed by the detection algorithm. The number of days since the last bottle reading was processed appears in parenthesis below the date and time of the last bottle reading.

View Test Time Icon

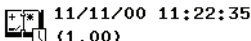

Displays the date and time the last test result changed. The number of days that have passed between load and last determination appears in parenthesis below the date and time. The date and time will not be displayed if the bottle's status has not changed from negative-to-date.

View Test Result Icon

Displays the test status of the selected bottle. Test status is displayed inside the bottle icon:

- + Positive (yellow bottle)
- Negative (green bottle)
- * Negative-to-date (gray bottle)
- ~ + Critical determination in progress (gray bottle)

Edit Test Result Button

You can use the **Edit Test Result** button to change the status of the displayed bottle:

- 1) Press the **Edit Test Result** button.

The Edit Test Result screen overlays and disables the Edit Bottle Detail screen.

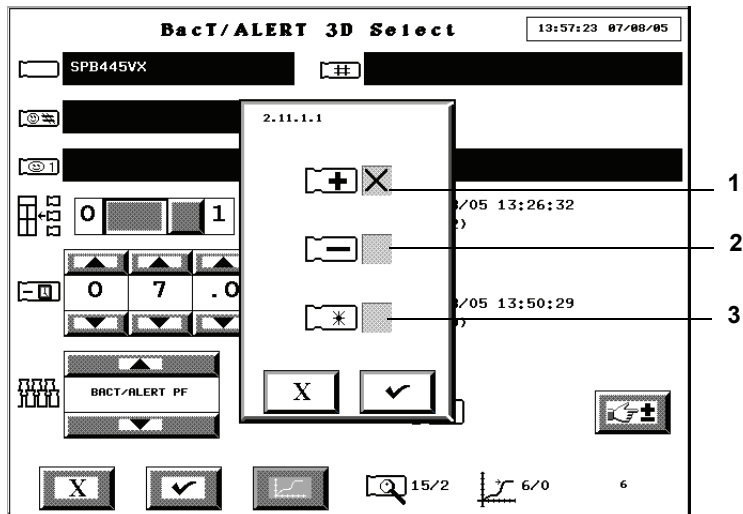


Figure 5-4: Edit Test Result Screen

- 1 — Positive Status Checkbox
- 2 — Negative Status Checkbox
- 3 — Negative-To-Date Status Checkbox

- 2) Touch the checkbox that corresponds to the desired status to insert an **X**.
- 3) Press the **Check** button to accept the change and return to the Edit Bottle Detail screen, or press the **Cancel** button to keep the current status and return to the Edit Bottle Detail screen.

Note: Results that have been manually changed to negative or positive via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 5-5](#)) will be marked on the report with a stick figure (☿).

Note: If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (☿) will not appear.

Graph Bottle Readings Button



Displays the bottle graph on the Graph Bottle Readings screen (see Viewing and Printing Bottle Graphs in Chapter 3).

View Algorithm/Polynomial Icon



The number above the slash represents the type of bottle specific algorithm used for the data analysis of the selected bottle.

Table 5-1: Bottle Specific Algorithms

Algorithm No.	Bottle Type
11	SA Plastic, SN Plastic
12	FA Plastic
13	MP Plastic
15	PF Plastic
17	MB Glass

The number below the slash represents the type of polynomial. There are two types of polynomials:

- Polynomial 1 — LES Glass
- Polynomial 2 — LES Plastic

View How Determined/Positivity Index Icon 

The number above the slash displays a numerical code representing how the selected bottle's status was determined.

Table 5-2: Status Determination Codes

Status Determination Code	How Status Determined
1	Rate
2	Acceleration
3	Initial value
4	Maximum test time exceeded
5	Delta
6	Test in progress
7	Manual (changed at Database Management System)
8	Manual (changed at BacT/ALERT® 3D)
9	Algorithm change (see Fault Code 74)
10	Manual (changed by LIS)
11	Wrong controller module (see Fault Code 79)
20–23	Corrupted bottle record
200–204	Detection of invalid data
205	Missing readings

The number below the slash is the Positivity Index. An index less than 1 indicates a negative or negative-to-date bottle. An index greater than or equal to 1 indicates a positive bottle.

Note: *The number displayed on the lower right of the Edit Bottle Detail screen (not associated with an icon) is a bottle sequence number, which may be utilized by your bioMérieux Representative.*

Editing Data Relationships

Introduction

Bottle information (Bottle ID, Accession Number, Hospital ID, Patient First Name, Patient Last Name) is typically associated to a bottle when a bottle is loaded. If necessary, however, these values can later be changed by editing the data relationships. This is initiated by pressing the **Edit Data Relationships** button on the Setup screen.

The following sections will cover:

- Initiating the Edit Data Relationships function ([see Initiating the Edit Data Relationships Function on page 5-12](#)).
- Editing Bottle ID to Accession Number relationships ([see Editing Bottle ID to Accession Number Relationships on page 5-14](#)).
- Editing Accession Number to Hospital ID relationships ([see Editing Accession Number to Hospital ID Relationships on page 5-16](#)).
- Editing Hospital ID to Patient Name relationships ([see Editing Hospital ID to Patient Name Relationships on page 5-17](#)).

Initiating the Edit Data Relationships Function



- 1) Access the Setup screen and enter a valid password ([see Accessing the Setup Screen Function Buttons in Chapter 3](#)).

- 2) Press the **Edit Data Relationships** button ().

The Edit Data Relationships screen appears.

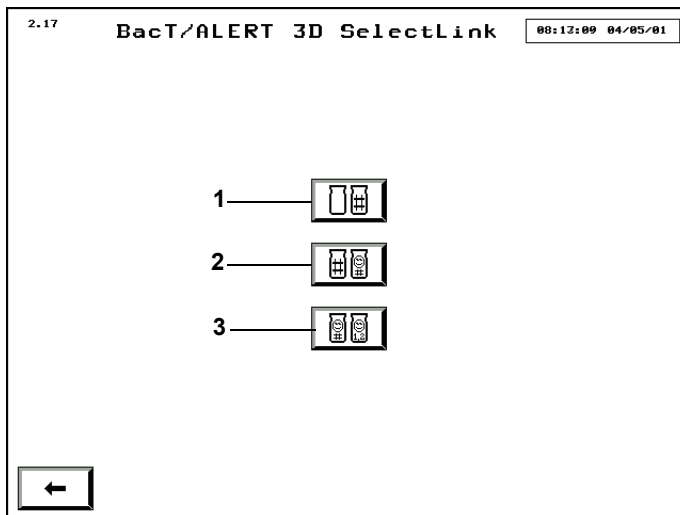


Figure 5-5: Edit Data Relationships Screen

1 — Edit Bottle ID to Accession Number Relationships Button

2 — Edit Accession Number to Hospital ID Relationships Button

3 — Edit Hospital ID to Patient Name Relationships Button

Note: The Edit Data Relationships screen is not available with the BacT/ALERT® 3D Signature configuration. Pressing the **Edit Data Relationships** button will display the Edit Bottle ID to Accession Number screen (see [Editing Bottle ID to Accession Number Relationships on page 5-14](#)).

Note: The availability of the buttons also depends on the availability of the fields. Example: If the **Patient First Name** and **Patient Last Name** fields are not displayed on the system, then the **Edit Hospital ID to Name Relationships button** is not available.

- 3) To edit a Bottle ID to accession number relationship, press the corresponding button and see [Editing Bottle ID to Accession Number Relationships on page 5-14](#).
- 4) To edit an Accession Number to Hospital ID relationship, press the corresponding button and see [Editing Accession Number to Hospital ID Relationships on page 5-16](#).
- 5) To edit a Hospital ID to patient name relationship, press the corresponding button and see [Editing Hospital ID to Patient Name Relationships on page 5-17](#).

Editing Bottle ID to Accession Number Relationships

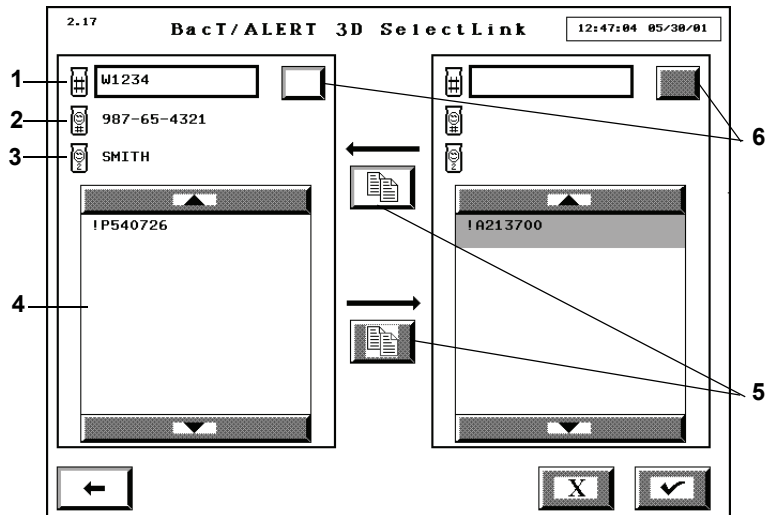


Figure 5-6: Edit Bottle ID to Accession Number Relationships Screen

- 1 — Accession Number Field
- 2 — Hospital ID Field
- 3 — Patient Last Name Field
- 4 — Bottle ID Scroll Box
- 5 — Data Transfer Buttons
- 6 — Accession Number Selection Buttons

Note: The **Accession Number Selection** button will only appear if the system is set to allow reuse of accession numbers. By default, the BacT/ALERT® 3D is set to regard accession numbers as always unique. If you desire to re-use accession numbers, contact your local bioMérieux Representative.

Note: When the Edit Bottle ID to Accession Number Relationships screen is first accessed, the **Accession Number** field is blank and all bottle IDs not associated with an Accession Number are displayed in both Bottle ID scroll boxes.

Attaching Bottle IDs without an Accession Number to an Accession Number



- 1) Access the Edit Bottle ID to Accession Number Relationships screen (see [Initiating the Edit Data Relationships Function on page 5-12](#)).
- 2) Enter the *accession number* in the left **Accession Number** field that you wish to associate to the bottle(s).

If the entered accession number is associated with more than one hospital ID, use the **Accession Number Selection** button to scroll through each instance as indicated in the **Hospital ID** and **Patient Last Name** fields.

- 3) Touch the bottle ID(s) in the right **Bottle ID** scroll box to be associated with the accession number. The selected bottle ID(s) highlight.
- 4) Press the top **Data Transfer** button to move the Bottle ID(s) to the left **Bottle ID** scroll box.
- 5) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 6) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Note: *If you accessed the Edit Bottle ID to Accession Number Relationships screen directly from the Setup screen, the **Previous Screen** button will return to the Setup screen.*

Moving a Bottle ID Association from one Accession Number to Another



- 1) Access the Edit Bottle ID to Accession Number Relationships screen (see [Initiating the Edit Data Relationships Function on page 5-12](#)).
- 2) Enter the *accession number* in the left **Accession Number** field to which the bottle(s) is currently associated.

If the entered accession number is associated with more than one Hospital ID, the **Accession Number Selection** button can be used to scroll through each instance as indicated in the **Hospital ID** and **Patient Last Name** fields.

- 3) Enter the *accession number* in the right **Accession Number** field that you wish to associate to the bottle(s).

If the entered accession number is associated with more than one hospital ID, the **Accession Number Selection** button can be used to scroll through each instance as indicated in the **Hospital ID** and **Patient Last Name** fields.

- 4) Touch the bottle ID(s) in the left **Bottle ID** scroll box that are to be transferred. The selected bottle ID(s) highlight.
- 5) Press the bottom **Data Transfer** button to move the bottle ID(s) to the right **Bottle ID** scroll box.
- 6) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.

- 7) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Note: If you accessed the Edit Bottle ID to Accession Number Relationships screen directly from the Setup screen, the **Previous Screen** button will return to the Setup screen.

Editing Accession Number to Hospital ID Relationships

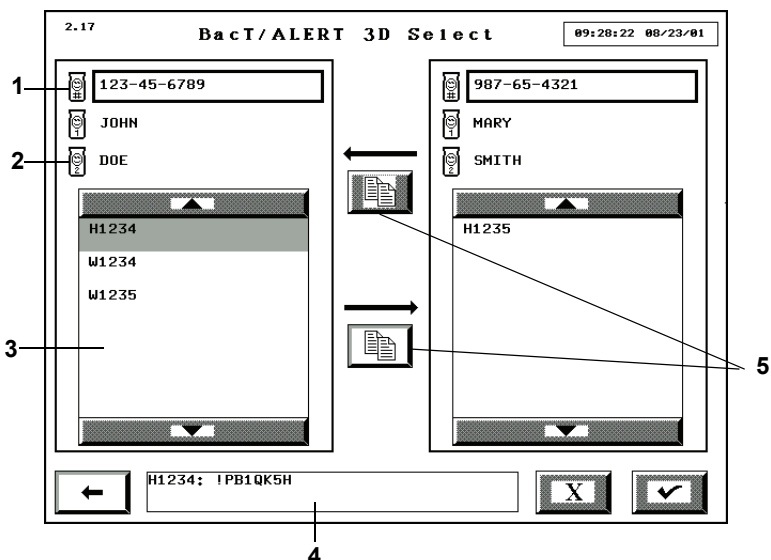


Figure 5-7: Edit Accession Number to Hospital ID Relationships Screen

- 1 — Hospital ID Field
- 2 — Patient Last Name Field
- 3 — Accession Number Scroll Box
- 4 — Bottle ID List Box
- 5 — Data Transfer Buttons

Note: When the Edit Accession Number to Hospital ID Relationships screen is first accessed, the **Hospital ID** field is blank and all accession number(s) not associated with a hospital ID are displayed in both **Accession Number** scroll boxes.

Attaching Accession Numbers without a Hospital ID to a Hospital ID



- 1) Access the Edit Accession Number to Hospital ID Relationships screen (see [Initiating the Edit Data Relationships Function](#) on page 5-12).

- 2) Enter the *hospital ID* in the left **Hospital ID** field that you wish to associate to the accession number(s).
- 3) Touch the accession number(s) in the right **Accession Number** scroll box that are to be associated to the hospital ID.

The selected accession number(s) highlight. The bottle IDs associated with the accession number that was touched last appears in the **Bottle ID** list box.
- 4) Press the top **Data Transfer** button to move the accession number(s) to the left **Accession Number** scroll box.
- 5) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all original relationships.
- 6) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Moving an Accession Number Association from one Hospital ID to Another



- 1) Access the Edit Accession Number to Hospital ID Relationships screen ([see Initiating the Edit Data Relationships Function on page 5-12](#)).
- 2) Enter the *hospital ID* in the left **Hospital ID** field that the accession number(s) is currently associated.
- 3) Enter the *hospital ID* in the right **Hospital ID** field with which you wish to associate to the accession number(s).
- 4) Touch the accession number(s) in the left **Accession Number** scroll box that are to be transferred.

The selected accession number(s) highlight. The bottle IDs associated with the accession number that was touched last appears in the **Bottle ID** list box.
- 5) Press the bottom **Data Transfer** button to move the accession number(s) to the right **Accession Number** scroll box.
- 6) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 7) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Editing Hospital ID to Patient Name Relationships



- 1) Access the Edit Hospital ID to Patient Name Relationships screen ([see Initiating the Edit Data Relationships Function on page 5-12](#)).

2.17 BacT/ALERT 3D SelectLink 08:23:12 04/05/01

1 — Hospital ID Field

2 — Patient First Name Field

3 — Patient Last Name Field

4 — Previous Screen Button

Figure 5-8: Edit Hospital ID to Patient Name Relationships Screen

- 1 — Hospital ID Field
- 2 — Patient First Name Field
- 3 — Patient Last Name Field
- 4 — Previous Screen Button

- 2) Enter the *hospital ID* in the **Hospital ID** field for which you want to add/edit the patient name association.
- 3) Enter the correct *patient name* into the **Patient First Name** and **Patient Last Name** fields.
- 4) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 5) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Note: The **Patient First Name** and **Patient Last Name** fields will remain unavailable until a known hospital ID is entered.

About This Chapter

This chapter provides you with procedures for viewing and editing bottle data for INDUSTRY Use.

Chapter Contents

Viewing/Editing Bottle Data • 6-2

Introduction • 6-2

Selecting Bottles Using the Edit Cell Contents Button • 6-3

Selecting Bottles Using the Select Bottle to Edit/Graph Button • 6-4

Editing Bottle Details Using the Edit Bottle Detail Screen • 6-5

Editing Data Relationships • 6-12

Introduction • 6-12

Initiating the Edit Data Relationships Function • 6-12

Editing Bottle ID to Sample ID Relationships • 6-14

Editing Sample ID to User Defined 3 Relationships • 6-16

Editing User Defined 3 to User Defined 1/User Defined 2 Relationships • 6-17

Viewing/Editing Bottle Data

Introduction



By accessing the Edit Bottle Detail screen of both currently loaded and recently unloaded bottles, you can perform the following bottle editing functions:


- Identifying an anonymous bottle
- Editing a bottle ID
- Editing the maximum test time of a currently loaded bottle
- Editing a bottle type
- Manually changing a bottle's test result
- Unloading a bottle from a faulty rack

Note: *The bottle record must be one of the 1,920 bottle records stored in the database.*

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 4](#)).
- 2) Access the Edit Bottle Detail screen for a specific bottle. There are two methods you can use to access the Edit Bottle Detail screen:

- Select loaded bottles using the **Edit Cell Contents** button (). See [Selecting Bottles Using the Edit Cell Contents Button on page 6-3](#).

or

- Select loaded and recently unloaded bottles using the **Select Bottle to Edit/Graph** button (). See [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 6-4](#).

- 3) Edit the bottle details as necessary (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 6-5](#)).
- 4) Press the **Check** button to save the changes and exit the screen or press the **Cancel** button to restore the information originally displayed and exit the screen.

Selecting Bottles Using the Edit Cell Contents Button



Note: This function can only be used for currently loaded bottles. To view information of recently unloaded bottles, select the bottle from the Select Bottle to Edit/Graph screen (see [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 6-4](#)).

- 1) From the Setup screen (See [Figure 4-10, Setup Screen](#)), press the **Edit Cell Contents** button.

The Edit Cell Contents screen appears.

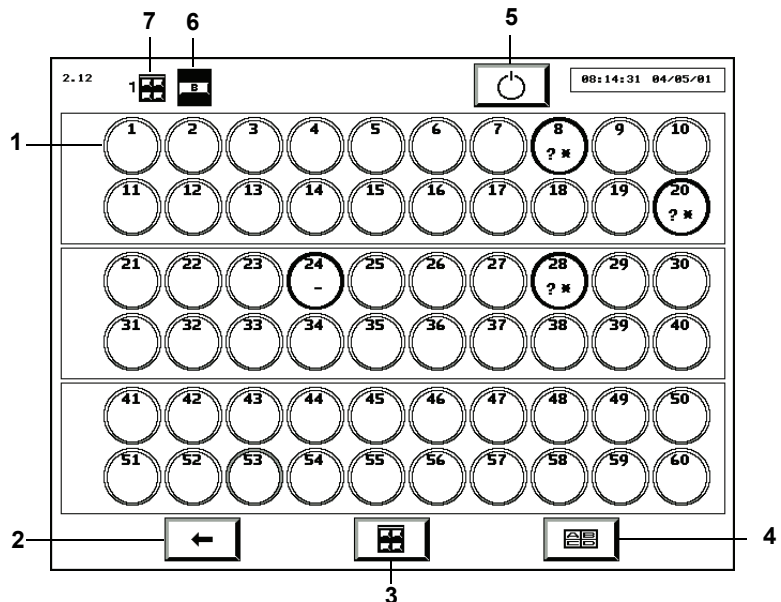


Figure 6-1: Edit Cell Contents Screen

- 1 — Cell
- 2 — Previous Screen Button
- 3 — Incubation Module Selection Button
- 4 — Drawer Selection Button
- 5 — Restart Incubation Module Button
- 6 — Drawer Indicator
- 7 — Incubation Module Indicator

Note: The Edit Cell Contents screen displays the same information as the View Cell Status screen (see [Understanding the View Cell Status Screen Display in Chapter 4](#)) and is continuously updated.

- 2) Select the desired drawer and Incubation Module using the **Incubation Module Selection** and **Drawer Selection** buttons.
- 3) Touch the appropriate **Cell** icon.

The Edit Bottle Detail screen appears (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 6-5](#)).

Selecting Bottles Using the Select Bottle to Edit/Graph Button



- 1) From the Setup screen (see [Figure 4-10, Setup Screen](#)), press the **Select Bottle to Edit/Graph** button.

The Select Bottle to Edit/Graph screen overlays and disables the Setup screen.

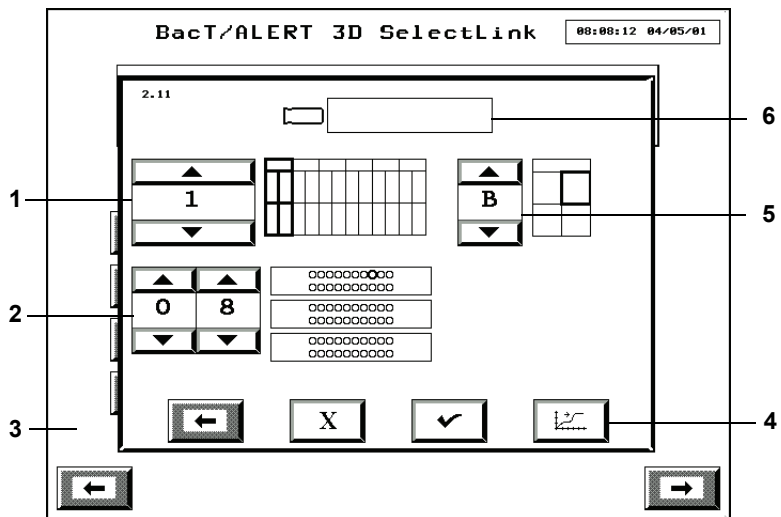


Figure 6-2: Select Bottle to Edit/Graph Screen

- 1 — Incubation Module Scroll Button
 - 2 — Cell Scroll Buttons
 - 3 — Setup Screen (Disabled)
 - 4 — Graph Bottle Readings Button
 - 5 — Drawer Scroll Button
 - 6 — Bottle ID Field
- 2) If the bottle ID is known, enter the *bottle ID* in the **Bottle ID** field (see [Text/Data Entry in Chapter 4](#)) and proceed to [Step 4](#).

Note: Only the last 1,920 bottles loaded can be retrieved. Any other entry in the **Bottle ID** field is invalid and will cause an Operator Error 940 (see [Operator Error Codes in Chapter 9](#)).

- 3) If the cell location (but not bottle ID) is known, adjust the **Incubation Module** (1–6), **Drawer** (A–D), and **Cell** (1–60) scroll buttons to select the appropriate bottle location.

Note: Only cell locations with currently loaded bottles can be used to view/edit bottle data. To view data of recently unloaded bottles, you must use the **Bottle ID** field.

- 4) Press the **Check** button.

The Edit Bottle Detail screen appears (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 6-5](#)).

Editing Bottle Details Using the Edit Bottle Detail Screen

Note: The bottle record can be edited before and after the bottle has been unloaded from the system.

If a bottle has not already been selected, you can select a bottle to edit using either the **Edit Cell Contents** or the **Select Bottle to Edit/Graph** button as described in [see Selecting Bottles Using the Edit Cell Contents Button on page 6-3](#), and [Selecting Bottles Using the Select Bottle to Edit/Graph Button on page 6-4](#).

The screenshot shows the 'BacT/ALERT 3D Select' interface. At the top, it displays '2.12.1' and '18:43:13 09/13/05'. The main area contains several input fields and buttons:


- Bottle ID:** SFBPPSY4
- Cell Location:** 345223
- Module/Drawer/Cell:** 555220, ABCDE, FGHIJ
- Test Results:** 08/26/05 15:55:20 (18.11), 09/13/05 18:41:01 (0.00), 09/13/05 18:36:26 (0.00), 08/26/05 17:43:32 (0.07)
- Buttons:** X, Check, Graph, 12/2, 8/0, 5

Figure 6-3: Edit Bottle Detail Screen

The screen ID is determined by the method in which the Edit Bottle Detail screen was accessed. The screen behaves in the same way regardless of how it is accessed.

- The screen ID will be 2.11.1 if you access it from the Select Bottle to Edit/Graph screen.
- The screen ID will be 2.12.1 if you access it from the Edit Cell Contents screen.

Edit Bottle ID Field

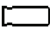



A blank **Bottle ID** field indicates the bottle is currently anonymous. To assign a bottle ID to an anonymous bottle or edit the bottle ID of an identified bottle:

- 1) Touch the **Bottle ID** field (the field turns white).
- 2) Manually enter the bottle ID using the keyboard or scan the barcode (see [Text/Data Entry in Chapter 4](#)).
- 3) Press the **Check** button.

View Sample ID Field



Disabled field that displays the Sample ID associated with the displayed bottle record.

View User Defined 1



Disabled field that displays the User Defined 1 associated with the displayed bottle record.

View User Defined 2



Disabled field that displays the User Defined 2 associated with the displayed bottle record.

View User Defined 3



Disabled field that displays the User Defined 3 associated with the displayed bottle record.

Note: The **View Sample ID**, **View User Defined 1**, **View User Defined 2**, and **View User Defined 3** fields are “view-only” fields.

Note: The **User Defined 1**, **User Defined 2**, and **User Defined 3** fields are not available with the BacT/ALERT® 3D Signature configuration.

Note: The fields may be hidden regardless of BacT/ALERT® 3D configuration. To make the field(s) unavailable, contact your local bioMérieux Representative.

Edit Load Status Sidebar Switch

It is possible to change the load status of an identified bottle in a faulty rack from loaded to unloaded. However, it is not possible to change a bottle's load status from unloaded to loaded.

Sliderbar interpretation:

- 0 = unloaded
- 1 = loaded

Note: The sliderbar is disabled if the associated rack is functioning properly.

Edit Maximum Test Time Scroll Buttons

Displays the incubation period for the selected bottle after which the bottle's status will change from negative-to-date to negative.

Note: The maximum test time of a bottle can be edited in days and tenths of a day at any time.


Edit Bottle Type Scroll Button

Displays the media type of the selected bottle. Use the scroll buttons to select the appropriate bottle type.

Note: Anonymous bottles are assigned to the UNKNOWN bottle type.

Note: The **Bottle ID**, **Load Status**, **Maximum Test Time**, and **Bottle Type** fields can be edited concurrently or one at a time. Press the **Check** button to save changes.

View Cell Location Icon

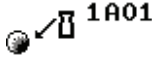


Displays the location of the selected bottle. The location is either the current location or the last cell location, depending on whether the bottle is still loaded.

Cell ID consists of the following three components in order:


- Incubation Module #
- Drawer letter
- Cell #

View First Loaded Time Icon

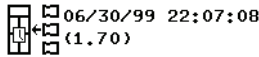


Displays the date and time the selected bottle was first loaded into the system. The number of days since the bottle was first loaded appears in parenthesis below the date and time.

View Last Unloaded Time Icon

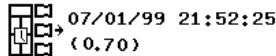


Displays the date and time the selected bottle was last unloaded from the system. The number of days since the bottle was unloaded appears in parenthesis below the date and time. No time will be displayed if the bottle has never been unloaded.

View Time of Last Bottle Reading Icon

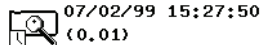

Displays the date and time of the last bottle reading that was successfully processed by the detection algorithm. The number of days since the last bottle reading was processed appears in parenthesis below the date and time of the last bottle reading.

View Test Time Icon

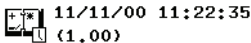

Displays the date and time the last test result changed. The number of days that have passed between load and last determination appears in parenthesis below the date and time. The date and time will not be displayed if the bottle's status has not changed from negative-to-date.

View Test Result Icon

Displays the test status of the selected bottle. Test status displayed inside the bottle icon:

- + Positive (yellow bottle)
- Negative (green bottle)
- * Negative-to-date (gray bottle)
- ~ + Critical determination in progress (gray bottle)

Edit Test Result Button

You can use the **Edit Test Result** button to change the status of the displayed bottle:

- 1) Press the **Edit Test Result** button.

The Edit Test Result screen overlays and disables the Edit Bottle Detail screen.

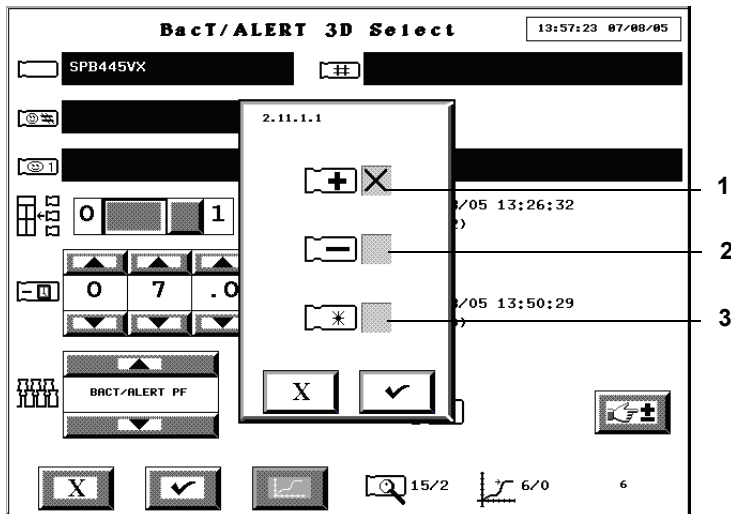


Figure 6-4: Edit Test Result Screen

- 1 — Positive Status Checkbox
- 2 — Negative Status Checkbox
- 3 — Negative-To-Date Status Checkbox

- 2) Touch the checkbox that corresponds to the desired status to insert an **X**.
- 3) Press the **Check** button to accept the change and return to the Edit Bottle Detail screen, or press the **Cancel** button to keep the current status and return to the Edit Bottle Detail screen.

Note: Results that have been manually changed to negative or positive via the Edit Bottle Detail screen (see [Editing Bottle Details Using the Edit Bottle Detail Screen on page 6-5](#)) will be marked on the report with a stick figure (☿).

Note: If a bottle is positive (set manually, or positive by any other reason) and then is manually changed to negative-to-date, the stick figure (☿) will not appear.

Graph Bottle Readings Button



Displays the bottle graph on the Graph Bottle Readings screen (see [Viewing and Printing Bottle Graphs in Chapter 4](#)).

View Algorithm/Polynomial Icon



The number above the slash represents the type of bottle specific algorithm used for the data analysis of the selected bottle.

Table 6-1: Bottle Specific Algorithms

Algorithm No.	Bottle Type
11	iAST, iNST, iLYM, iFN, BPA, BPN
12	iFA
15	iPF

The number below the slash represents the type of polynomial. Polynomial 2 (LES Plastic) is used for the INDUSTRY bottle.

View How Determined/Positivity Index Icon 

The number above the slash displays a numerical code representing how the selected bottle's status was determined.

Table 6-2: Status Determination Codes

Status Determination Code	How Status Determined
1	Rate
2	Acceleration
3	Initial value
4	Maximum test time exceeded
5	Delta
6	Test in progress
7	Manual (changed at Database Management System)
8	Manual (changed at BacT/ALERT® 3D)
9	Algorithm change (see Fault Code 74)
10	Manual (changed by LIS)
11	Wrong controller module (see Fault Code 79)
20–23	Corrupted bottle record
200–204	Detection of invalid data
205	Missing readings

The number below the slash is the Positivity Index. An index less than 1 indicates a negative or negative-to-date bottle. An index greater than or equal to 1 indicates a positive bottle.

Note: The number displayed on the lower right of the Edit Bottle Detail screen (not associated with an icon) is a bottle sequence number, which may be utilized by your bioMérieux Representative.

Editing Data Relationships

Introduction

Bottle information (Bottle ID, Sample ID, User Defined 3, User Defined 2, User Defined 1) is typically associated to a bottle when a bottle is loaded. If necessary, however, these values can later be changed by editing the data relationships. This is initiated by pressing the **Edit Data Relationships** button on the Setup screen.

The following sections will cover:

- Initiating the Edit Data Relationships function ([see Initiating the Edit Data Relationships Function on page 6-12](#)).
- Editing Bottle ID to Sample relationships ([see Editing Bottle ID to Sample ID Relationships on page 6-14](#)).
- Editing Sample ID to User Defined 3 relationships ([see Editing Sample ID to User Defined 3 Relationships on page 6-16](#)).
- Editing User Defined 3 to User Defined 1/User Defined 2 relationships ([see Editing User Defined 3 to User Defined 1/User Defined 2 Relationships on page 6-17](#)).

Initiating the Edit Data Relationships Function



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 4](#)).

- 2) Press the **Edit Data Relationships** button ().

The Edit Data Relationships screen appears.

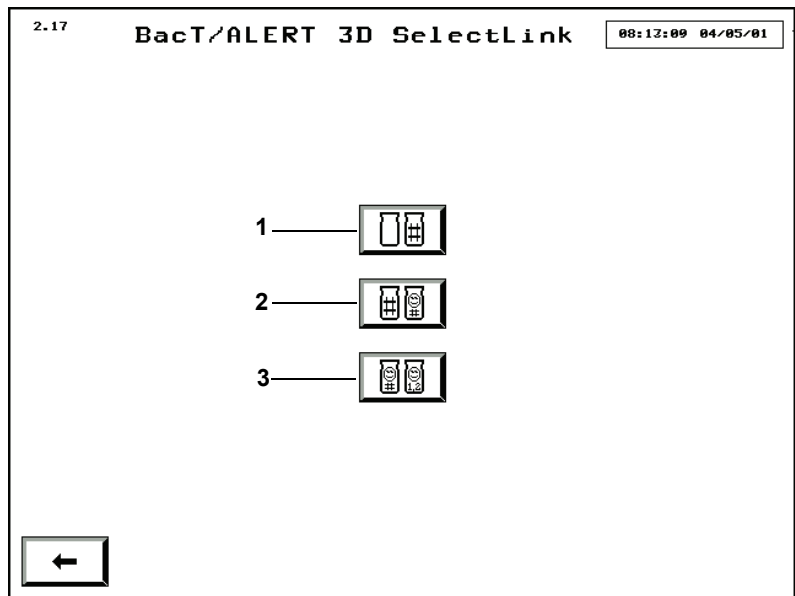


Figure 6-5: Edit Data Relationships Screen

- 1 — Edit Bottle ID to Sample ID Relationships Button
- 2 — Edit Sample ID to User Defined 3 Relationships Button
- 3 — Edit User Defined 3 to User Defined 1/User Defined 2 Relationships Button

Note: The Edit Data Relationships screen is not available with the BacT/ALERT® 3D Signature configuration. Pressing the **Edit Data Relationships** button will display the Edit Bottle ID to Sample ID screen ([see Editing Bottle ID to Sample ID Relationships on page 6-14](#)).

Note: The availability of the buttons also depends on the availability of the fields. Example: If the **User Defined 1** and **User Defined 2** fields are not displayed on the system, then the **Edit User Defined 3 to User Defined 1/User Defined 2** fields are not available.

- 3) To edit a Bottle ID to Sample ID relationship, press the corresponding button and see [Editing Bottle ID to Sample ID Relationships on page 6-14](#).
- 4) To edit a Sample ID to User Defined 3 relationship, press the corresponding button and see [Editing Sample ID to User Defined 3 Relationships on page 6-16](#).

- 5) To edit a User Defined 3 to User Defined 1/User Defined 2 relationship, press the corresponding button and see [Editing User Defined 3 to User Defined 1/User Defined 2 Relationships on page 6-17](#).

Editing Bottle ID to Sample ID Relationships

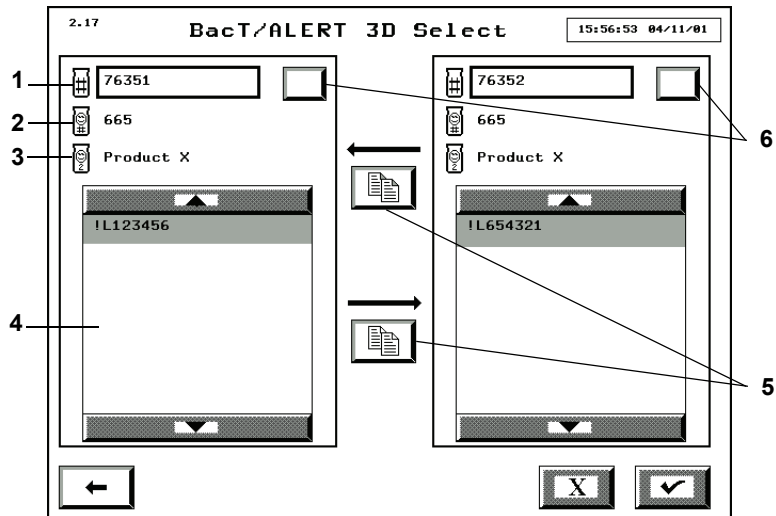


Figure 6-6: Edit Bottle ID to Sample ID Relationships Screen

- 1 — Sample ID Field
- 2 — User Defined 3
- 3 — User Defined 2
- 4 — Bottle ID Scroll Box
- 5 — Data Transfer Buttons
- 6 — Sample ID Selection Buttons

Note: The **Sample ID Selection** button will only appear if the system is set to allow reuse of Sample IDs. By default, the BacT/ALERT® 3D is set to regard Sample IDs as always unique. If you desire to re-use sample IDs, contact your local bioMérieux Representative.

Note: When the Edit Bottle ID to Sample ID Relationships screen is first accessed, the **Sample ID** field is blank and all bottle IDs not associated with a Sample ID are displayed in both Bottle ID scroll boxes.

Attaching Bottle IDs without a Sample ID to a Sample ID



- 1) Access the Edit Bottle ID to Sample ID Relationships screen (see [Initiating the Edit Data Relationships Function on page 6-12](#)).

- 2) Enter the *sample ID* in the left **Sample ID** field that you wish to associate to the bottle(s).

If the entered Sample ID is associated with more than one User Defined 3, use the **Sample ID Selection** button to scroll through each instance as indicated in the **User Defined 3** and **User Defined 2** fields.

- 3) Touch the bottle ID(s) in the right **Bottle ID** scroll box to be associated with the Sample ID. The selected bottle ID(s) highlight.
- 4) Press the top **Data Transfer** button to move the Bottle ID(s) to the left **Bottle ID** scroll box.
- 5) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 6) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Note: *If you accessed the Edit Bottle ID to Sample ID Relationships screen directly from the Setup screen, the **Previous Screen** button will return to the Setup screen.*

Moving a Bottle ID Association from one Sample ID to Another



- 1) Access the Edit Bottle ID to Sample ID Relationships screen ([see Initiating the Edit Data Relationships Function on page 6-12](#)).
- 2) Enter the *sample ID* in the left **Sample ID** field to which the bottle(s) is currently associated.

If the entered sample ID is associated with more than one User Defined 3, the **Sample ID Selection** button can be used to scroll through each instance as indicated in the **User Defined 3** and **User Defined 2** fields.
- 3) Enter the *sample ID* in the right **Sample ID** field that you wish to associate to the bottle(s).

If the entered sample ID is associated with more than one User Defined 3, the **Sample ID Selection** button can be used to scroll through each instance as indicated in the **User Defined 3** and **User Defined 2** fields.
- 4) Touch the bottle ID(s) in the left **Bottle ID** scroll box that are to be transferred. The selected bottle ID(s) highlight.
- 5) Press the bottom **Data Transfer** button to move the bottle ID(s) to the right **Bottle ID** scroll box.
- 6) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 7) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Note: If you accessed the *Edit Bottle ID to Sample ID Relationships* screen directly from the *Setup* screen, the **Previous Screen** button will return to the *Setup* screen.

Editing Sample ID to User Defined 3 Relationships

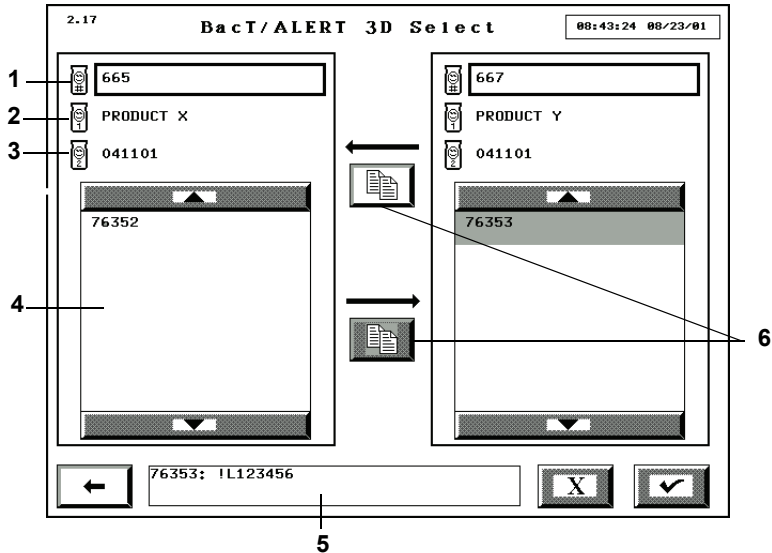


Figure 6-7: Edit Sample ID to User Defined 3 Relationships Screen

- 1 — User Defined 3 Field
- 2 — User Defined 1 Field
- 3 — User Defined 2 Field
- 4 — Sample ID Scroll Box
- 5 — Bottle ID List Box
- 6 — Data Transfer Buttons

Note: When the *Edit Sample ID to User Defined 3 Relationships* screen is first accessed, the **User Defined 3** is blank and all Sample ID(s) not associated with a User Defined 3 are displayed in both **Sample ID** scroll boxes.

Attaching Sample IDs without a User Defined 3 to a User Defined 3



- 1) Access the *Edit Sample ID to User Defined 3 Relationships* screen ([see Initiating the Edit Data Relationships Function on page 6-12](#)).
- 2) Enter the **User Defined 3** in the left **User Defined 3** that you wish to associate to the Sample ID(s).
- 3) Touch the sample ID(s) in the right **Sample ID** scroll box that are to be associated to the User Defined 3.

The selected sample ID(s) highlight. The bottle IDs associated with the Sample ID that was touched last appears in the **Bottle ID** list box.

- 4) Press the top **Data Transfer** button to move the sample ID(s) to the left **Sample ID** scroll box.
- 5) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all original relationships.
- 6) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Moving a Sample ID Association from One User Defined 3 to Another



- 1) Access the Edit Sample ID to User Defined 3 Relationships screen ([see Initiating the Edit Data Relationships Function on page 6-12](#)).
- 2) Enter the *User Defined 3* in the left **User Defined 3** that the sample ID(s) is currently associated.
- 3) Enter the *User Defined 3* in the right **User Defined 3** with which you wish to associate to the sample ID(s).
- 4) Touch the sample ID(s) in the left **Sample ID** scroll box that are to be transferred.

The selected sample ID(s) highlight. The bottle IDs associated with the sample ID that was touched last appears in the **Bottle ID** list box.

- 5) Press the bottom **Data Transfer** button to move the sample ID(s) to the right **Sample ID** scroll box.
- 6) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 7) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

Editing User Defined 3 to User Defined 1/User Defined 2 Relationships



- 1) Access the Edit User Defined 3 to User Defined 1/User Defined 2 Relationships screen ([see Initiating the Edit Data Relationships Function on page 6-12](#)).

The screenshot shows a window titled "BacT/ALERT 3D Select" with a status bar at the top displaying "2.17" and a timestamp "15:59:16 04/11/01". The main area contains three input fields arranged horizontally. The top field is labeled "1" and contains the value "665". The bottom-left field is labeled "2" and contains the value "041101". The bottom-right field is labeled "4" and contains the value "Product Y". At the bottom left, there is a button labeled "3" with a left-pointing arrow. At the bottom right, there are two buttons: one labeled "X" and one with a checkmark.

Figure 6-8: Edit User Defined 3 to User Defined 1/User Defined 2 Relationships Screen

- 1 — User Defined 3
- 2 — User Defined 1
- 3 — Previous Screen Button
- 4 — User Defined 2

- 2) Enter the *User Defined 3* in the **User Defined 3** for which you want to add/edit the User Defined 1/User Defined 2 association.
- 3) Enter the correct *User Defined 1*/*User Defined 2* into the **User Defined 1** and **User Defined 2** fields.
- 4) Press the **Check** button to accept the changes, or press the **Cancel** button to restore all relationships originally displayed.
- 5) When done, press the **Previous Screen** button to return to the Edit Data Relationships screen.

About This Chapter

This chapter provides you with procedures on how to configure the software.

Note: *Information in this chapter that is intended for clinical use only or industry use only is marked Clinical Use or INDUSTRY Use.*

Chapter Contents

Setting the Maximum Test Time • 7-2

Setting the Audible Alarms • 7-3

Priority of Alarms • 7-5

Terminating an Instrument Alarm • 7-5

Changing the System Password • 7-5

Initiating Manual Backup • 7-7

Configuring Report Screens • 7-9

Introduction • 7-9

Entering Report Labels • 7-10

Configuring Report Contents • 7-13

Viewing and Printing Calibration Data • 7-21

Viewing and Printing Calibration History • 7-25

Setting the Maximum Test Time



Perform the following procedure to set a universal maximum test time (or time that a negative-to-date bottle will be tested before being called final negative) for each media type. The result is that each time a bottle is loaded, its maximum test time will default to the maximum test time configured for its media type.

Note: The maximum test time can be changed for an individual bottle while loading, if desired, or can be changed after a bottle is loaded in the Edit Bottle Detail screen. For Clinical Use, see [Changing the Maximum Test Time — Individual Bottles in Chapter 3](#). For INDUSTRY Use, see [Changing the Maximum Test Time — Individual Bottles in Chapter 4](#).

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) Press the **Set Maximum Test Time** button ().

The Set Maximum Test Time screen overlays and disables the Setup screen.

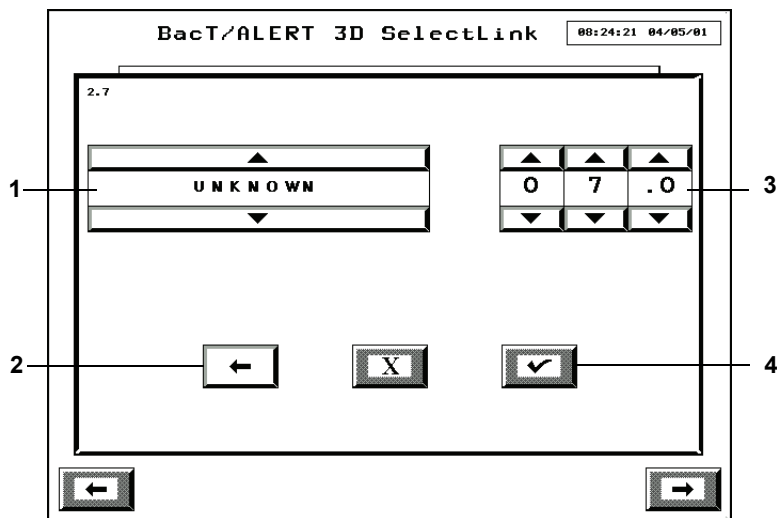


Figure 7-1: Set Maximum Test Time Screen

- | | |
|------------------------------|--------------------------------------|
| 1 — Media Type Scroll Button | 3 — Incubation Period Scroll Buttons |
| 2 — Previous Screen Button | 4 — Check Button |

- 3) Select the appropriate bottle type from the **Media Type** scroll button.
- 4) Set the incubation period in days and tenths of a day using the **Incubation Period** scroll buttons.

Note: *The minimum test time that can be specified is 0.1 day.*

- 5) Press the **Check** button to save the maximum test time settings, or press the **Cancel** button to return the system to the previous maximum test time settings.

Note: *Changing the maximum test time has no affect on bottles currently loaded. The new maximum test time will be attributed to bottles loaded after the setting is changed.*

- 6) Press the **Previous Screen** button to return to the Setup screen.

Note: *Generic or anonymous bottles in a BC drawer will be assigned the unknown maximum test time.*

Note: *Clinical Use: Generic or anonymous bottles in the MB drawer will use the maximum test time assigned to the BacT/ALERT® MP bottle.*

Setting the Audible Alarms



You can enable/disable the following audible alarms:

- **Positive Bottle Alarm** – A repeated sequence of brief beeps followed by a pause. This sequence of beeps are repeated every 60 seconds until the bottle is removed.
- **Instrument Fault Alarm** – An ongoing, continuous sequence of brief beeps. Once the condition is acknowledged (see [Terminating an Instrument Alarm on page 7-5](#)), the alarm will sound again in 30 minutes if the error condition still exists.

Note: *The default setting for the fault reminder period is 30 minutes. If you require a different time period, contact your local bioMérieux Representative.*

- **Operator Error Alarm** – Two brief beeps to alert the operator to look at the Operator Panel where an error code or picture is displayed. For resolution of a specific error code, see [Operator Error Codes in Chapter 9](#).

Note: Operator error alarms do not repeat.

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use.).

- 2) Press the **Set Audible Alarm Options** button ().

The Set Audible Alarm Options screen overlays and disables the Setup screen.

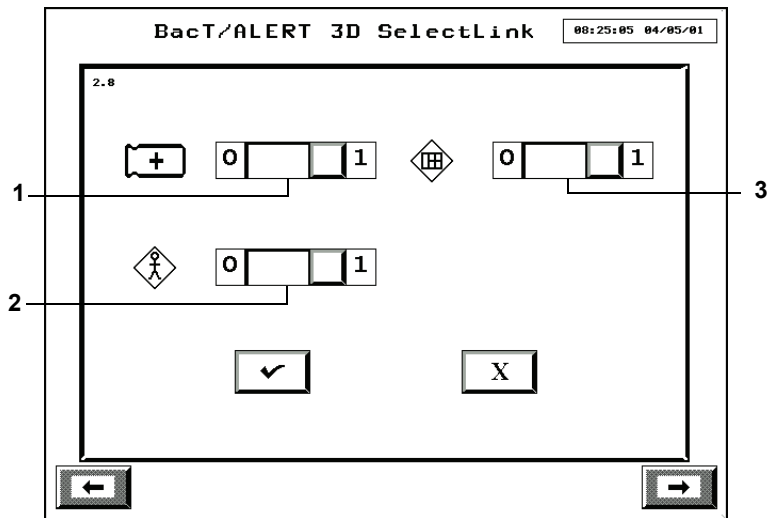


Figure 7-2: Setting the Audible Alarm Options Screen

- 1 — Positive Bottle Alarm Slidebar Switch
- 2 — Operator Error Alarm Slidebar Switch
- 3 — Instrument Fault Alarm Slidebar Switch

- 3) Enable and/or disable alarms by adjusting the slidebar switches to the desired settings:
 - 0 — disabled
 - 1 — enabled
- 4) Press the **Check** button to save any changes, or press the **Cancel** button to return the system to the original settings.

The system returns to the Setup screen.

Note: In addition to the Instrument Audible Alarms, the bar code reader is also programmed to provide an audible alarm in three instances. These are:

- Bar code is scanned with entry to Bottle ID field
- Accession bar code is scanned with entry to Accession Number field
- Any Operator errors related to scanning, loading or unloading

The bar code reader alarms are not disabled by setting the Operator Error Alarm slide bar to the zero setting.

Priority of Alarms

The audible alarms which will sound correspond to the most critical condition according to the priorities listed (highest to lowest) below:

- 1) Instrument fault
- 2) Positive bottles
- 3) Operator errors

Terminating an Instrument Alarm

To terminate an alarm, perform one of the following:

- Acknowledge the condition causing the alarm by touching any area of any screen of the Operator Panel or by pressing any key on the keyboard.
- Disable the alarm.

Note: To remove or adjust your error alarm delay cycle, contact your local bioMérieux Representative.

Changing the System Password



Perform the following procedure to change the system password that is used to enable the Setup screen function buttons.

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) Press the **Change Password** button ().

The Change Password screen overlays and disables the Setup screen.

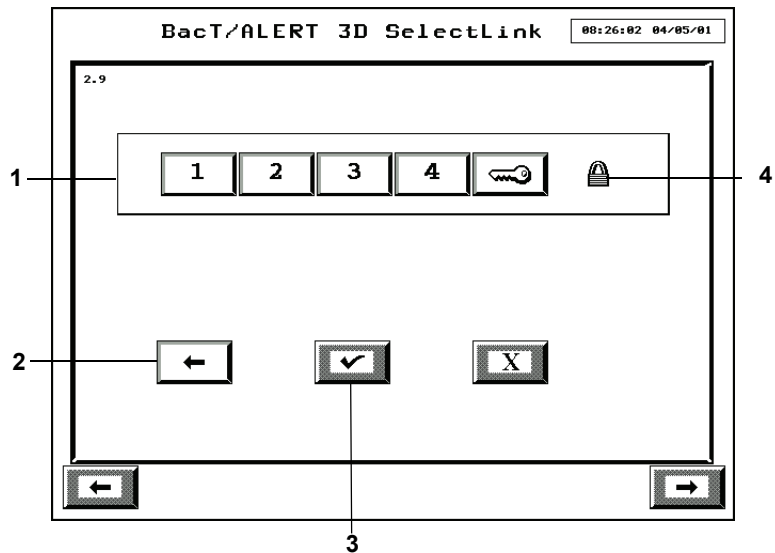


Figure 7-3: Change Password Screen

- 1 — Password Entry Keypad
- 2 — Previous Screen Button
- 3 — Check Button
- 4 — Padlock Icon

3) Enter a valid *password* using the **Password Entry Keypad** buttons.

Note: Acceptable passwords consist of any combination of the numbers 1 to 4 and have a maximum length of eight characters.

4) The **Padlock** icon changes to a full open position after the **Key Symbol** button is pressed. If the **Padlock** icon does not open, then repeat [Step 3](#).

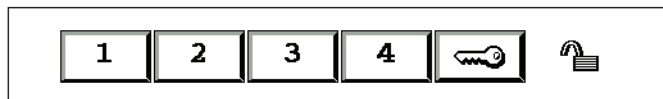


Figure 7-4: Padlock Icon (Full Open position)

- 5) Enter the new *password* using the **Password Entry Keypad** buttons.
- 6) Press the **Key Symbol** button.

The **Padlock** icon changes to a half open position.

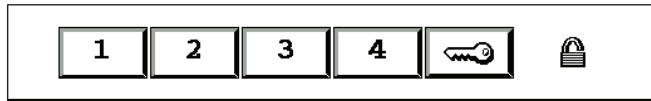


Figure 7-5: Padlock Icon (Half Open Position)

- 7) Re-enter the new *password* to verify.
- 8) Press the **Key Symbol** button. If the same password is entered both times, then the **Padlock** icon closes. If the new password was not verified correctly, the **Padlock** icon will open completely. The user must then start over with [Step 5](#) to change the password.

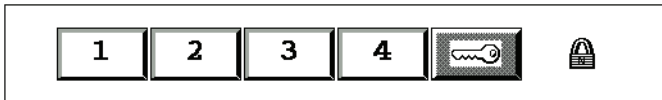


Figure 7-6: Padlock Icon (Closed Position)

- 9) Press the **Check** button to accept the new password, or press the **Cancel** button to retain the old password.
- 10) Press the **Previous Screen** button to return to the Setup screen.

Note: Record the new password in a secure location. This password will be needed to access all function buttons on the Setup screen.

Initiating Manual Backup



Perform the following procedure to initiate an immediate backup of the system to a Zip[®] disk or USB flash drive. The backup procedure, though lengthy, does not prohibit you from simultaneously performing any other functions on the system.

Note: If you have a 250 MB Zip[®] Drive, use only 250 MB Zip[®] disks for backups.

Note: If you have a 750 MB Zip[®] Drive, you may use either 250 or 750 MB Zip[®] disks for backups.

IMPORTANT: *If you have a USB Port, use only USB Flash Drives provided by bioMérieux for backups.*

- 1) Access the Setup screen and enter a valid password see ([Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) Press the **Backup Management** button ().

The Backup Management screen overlays and disables the Setup screen.

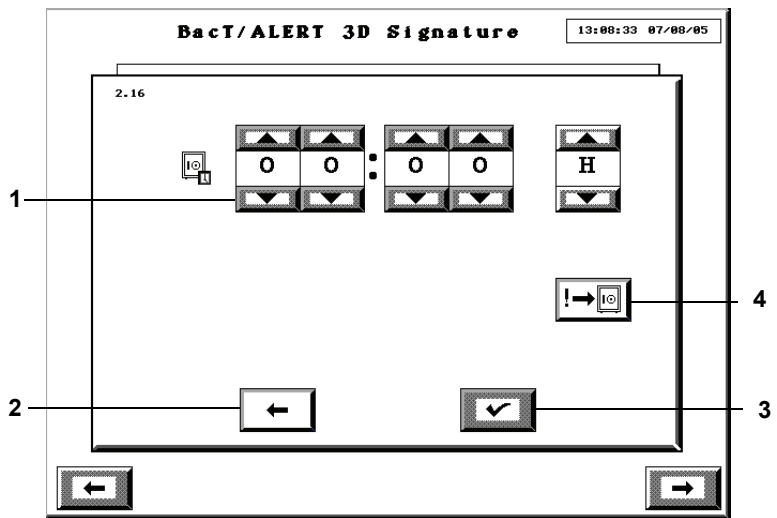


Figure 7-7: Backup Management Screen

- 1 — Automatic Backup Time Scroll Buttons
- 2 — Previous Screen Button
- 3 — Check Button
- 4 — Manual Backup Button

Note: *The **Automatic Backup Time** scroll buttons are disabled by default. If you wish to set an automatic backup time, contact your local bioMérieux Representative.*

- 3) To initiate an immediate backup, press the **Manual Backup** button.

The system returns to the Setup screen.

Note: *If a manual backup is initiated without a Zip® disk inserted in the drive or USB flash drive inserted in the USB port, then an Instrument Status Code 810 will display (see [Instrument Status Codes in Chapter 9](#)).*

Note: While the system is being backed up, the **Backup In Progress** icon will appear in the upper left corner of the Backup Management and Main screens. The icon will disappear every 5 seconds for half a second.

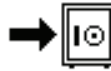


Figure 7-8: Backup In Progress Icon

Note: If you attempt to perform a function (for example, load or unload a bottle) while a manual backup is taking place, you may encounter a brief screen grayout. This is a safety feature that prevents database corruption by allowing the software to close and exit files prior to giving you access.

WARNING



Do not insert any object, other than a Zip[®] disk, into the Zip[®] drive under any circumstances.



CAUTION: Do not forcibly remove the Zip[®] disk from the instrument. Forcibly removing the Zip[®] disk may cause damage to the Zip[®] disk or Zip[®] drive and may cause the system to lock up.

Configuring Report Screens

Introduction

Report screens are accessed by pressing the appropriate **Display Report Screen** button from the Report Selection screen (see [Viewing and Printing in Chapter 3](#) for Clinical Use, or [Chapter 4](#) for INDUSTRY Use). The following procedures show you how to configure these report screens.

Note: The report function is available with the BacT/ALERT[®] 3D Select and SelectLink configurations.

Entering Report Labels



The following procedure describes how you can select labels to use as column headings for all reportable fields on system reports.

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) From the Setup screen, press the **Report** button ().

The Report Selection screen appears.

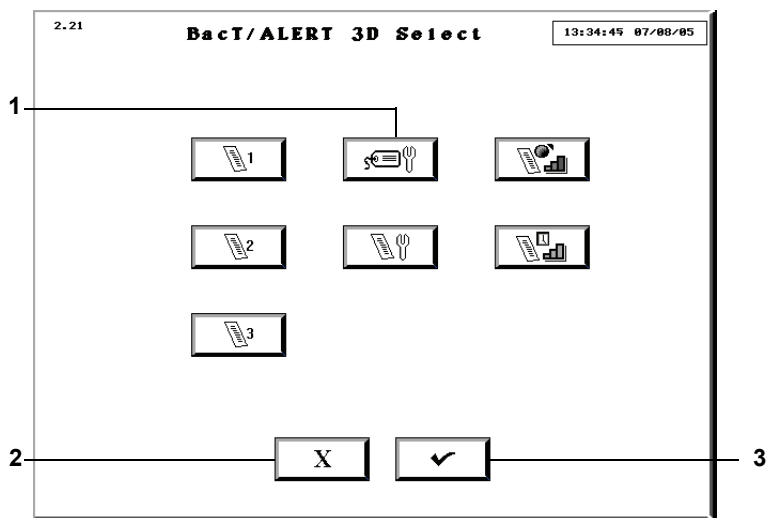


Figure 7-9: Report Selection Screen

- 1 — Report Label Entry Button 3 — Check Button
2 — Cancel Button

- 3) Press the **Report Label Entry** button ().

The Report Label Entry screen appears.

2.14 **BacT/ALERT 3D Select** 10:29:45 08/25/05

Max Test Time	Time Tested
Bottle Type	Test Result
Loaded	Bottle ID
Cell ID	Accession Num
1st Load Time	Hospital ID
Last Unld Time	First Name
Last Rdnng Time	Last Name

1 2

Figure 7-10: Report Label Entry Screen

1 — Cancel Button

2 — Check Button

Note: The above default report labels are assigned at installation and can be modified or changed.

- 4) Access the keyboard (see [Accessing the Controller Module Keyboard in Chapter 2](#)).
- 5) Touch the field where you will enter the new label.
The field turns white.
- 6) Enter the new *label*. Labels may contain up to 15 characters; however, labels will be truncated on the reports if the column widths do not allow room for the entire label (see [Configuring Report Contents on page 7-13](#)).
- 7) After you have made all of the desired label changes, verify that all labels are unique and there are no empty label fields.
- 8) Press the **Check** button to accept the label changes, or press the **Cancel** button to retain the previous labels.

The system returns to the Setup screen.

Note: Operator Error 960 displays if the **Check** button is pressed with duplicate Report Label entries present (see [Operator Error Codes in Chapter 9](#)).

Note: Operator Error 961 displays if the **Check** button is pressed with blank **Report Label** fields present.



CAUTION: The user is solely responsible for the choice of customized report label text and for validating that the intended label text appears in all associates reports. bioMérieux shall not be liable for any consequences resulting from misinterpretation of customized report labels.

Table 7-1: Report Field Descriptions








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






Field	Description
 Max Test Time	Displays the maximum number of days a bottle will be tested to the tenth of a day (ex. XX.X).
 Bottle Type	Displays the bottle type with no abbreviation.
 Loaded	Displays 1 for bottle loaded or 0 for bottle not loaded.
 Cell ID	Displays cell location. Example: 1A01.
 1st Load Time	Displays the date the bottle was loaded in one of the following formats, depending on Set Date/Time screen setting: MM/DD/YY or DD/MM/YY.
 Last Unld Time	Displays the date the bottle was unloaded in one of the following formats, depending on Set Date/Time screen setting: MM/DD/YY or DD/MM/YY.
 Last Rdnng Time	Displays the Date and Time of the last successful bottle reading in one of the following formats, depending on Set Date/Time screen setting: MM/DD/YY HH:MM or DD/MM/YY HH:MM. The time is always displayed in 24 hour format regardless of the Set Date/Time screen setting.
 Time Tested	Displays the time in days since the bottle has been loaded for negative-to-date bottles, or the time it took to make the result determination for positive and negative bottles. The time is displayed in hundredths of days (ex. XX.XX).
 Test Result	Displays the bottle test result as + for positive, - for negative, or * for negative to date.

Table 7-1: Report Field Descriptions (Continued)


Field	Description
 Bottle ID	Displays the bottle ID.
 Accession Num	Clinical Use: Displays the accession number or other kind of data entered by the user. INDUSTRY Use: Referred to in this manual as Sample ID .
 Hospital ID	Clinical Use: Displays the hospital ID or other kind of data entered by the user. INDUSTRY Use: Referred to in this manual as User Defined 3 .
 First Name	Clinical Use: Displays the patient's first name or other kind of data entered by the user. INDUSTRY Use: Referred to in this manual as User Defined 1 .
 Last Name	Clinical Use: Displays the patient's last name or other kind of data entered by the user. INDUSTRY Use: Referred to in this manual as User Defined 2 .

Configuring Report Contents



Perform the following procedure to configure the Report screens to show the desired information (see [Figure 3-12, Sample Report Screen](#) for Clinical Use, or [Figure 4-12, Sample Report Screen](#) for INDUSTRY Use).

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) From the Setup screen, press the **Report** button ().

The Report Selection screen appears.

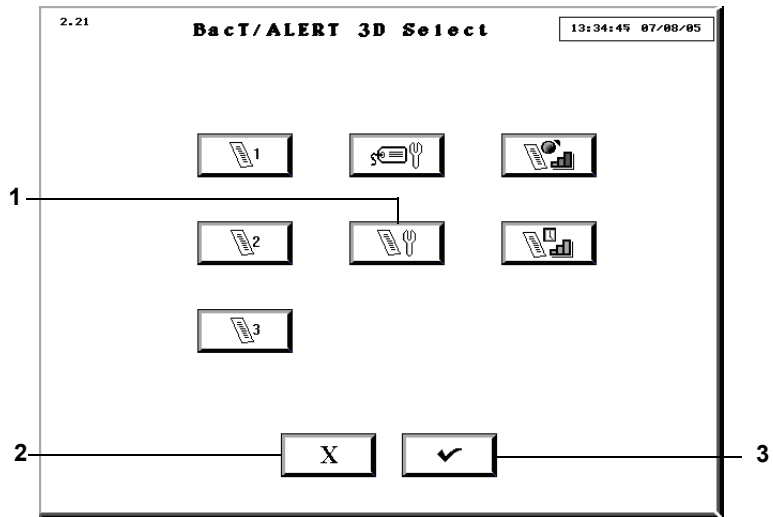


Figure 7-11: Report Selection Screen

1 — Report Configuration Button

2 — Cancel Button

3 — Check Button

- 3) Press the **Report Configuration** button ().

The Report Configuration screen displays.

Clinical Use: See [Figure 7-12](#), [Figure 7-13](#), and [Figure 7-14](#) for examples of the Load, Status, and Unload Report Configuration screens, respectively.

INDUSTRY Use: See [Figure 7-15](#), [Figure 7-16](#), and [Figure 7-17](#) for examples of the Load, Status, and Unload Report Configuration screens, respectively.

- 4) Press the **Edit Report** button that corresponds to the report you wish to reconfigure.
- 5) Adjust the **Column Selection** scroll buttons so that the desired fields are reported in the desired order. The scroll buttons correspond to the report columns from left to right.

Note: Each **Column Selection** scroll button contains all reportable fields plus a blank selection so that the column can be left blank.

- 6) Adjust the column widths by touching the **Column Width** selection box under the column to be altered and enter the new width. Make certain the column widths do not exceed the total available.

Note: *The **70** to the left of the selection boxes denotes the total available width, and the number to the right of the equals sign denotes the available width left (ex. the total minus the sum of the **Column Width** selection boxes).*

- 7) Adjust the **Primary/Secondary Sort** scroll buttons, if desired, to sort the report.

Note: *Each **Primary/Secondary Sort** scroll button both contain all reportable fields plus a blank selection if a sort is not to be performed.*

- 8) Select the order of any sorts with the **Sort Order Selection** button. If the arrow is pointed upward, there will be an ascending sort and if the arrow is pointed downward, there will be a descending sort.

- 9) Touch the **Section Break** checkbox(es) to set a section break.

Note: *You can select a sort for a field that is not chosen as one of the report columns. If a section break is also chosen for that sort, then the field's data will display in the section break line. This, in effect, allows you to report two extra data fields.*

- 10) Press the **Preview Report** button to view the report configuration (see [Figure 3-12, Sample Report Screen](#) for Clinical Use, or [Figure 4-12, Sample Report Screen](#) for INDUSTRY Use).
- 11) Press the **Previous Screen** button to return to the Report Configuration screen.
- 12) Make any adjustments to the report configuration desired.
- 13) Press the **Check** button to accept the changes to the report configuration, or press the **Cancel** button to retain the original configuration.
- 14) Press the **Previous Screen** button to return to the Setup screen.

Load Report Configuration Screen (Clinical Use)

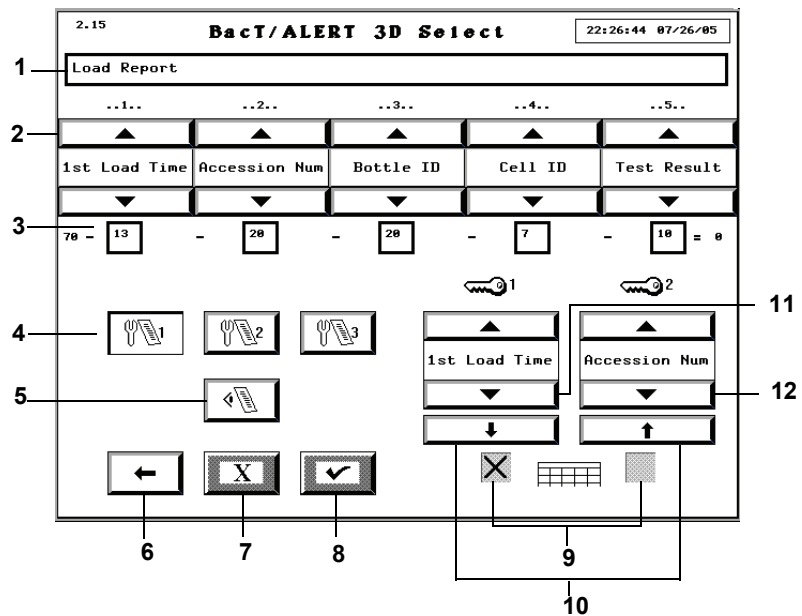


Figure 7-12: Load Report Configuration Screen (Clinical Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Status Report Configuration Screen (Clinical Use)

The screenshot shows the 'BacT/ALERT 3D Select' configuration screen. At the top, it displays '2.15' and a timestamp '22:27:36 07/26/05'. The main title is 'Status Report'. Below this, there are five columns labeled '1..', '2..', '3..', '4..', and '5..'. Each column has a scroll button (up and down arrows) and a dropdown menu. The columns are labeled 'Hospital ID', 'Last Name', 'Bottle ID', '1st Load Time', and 'Test Result'. Below the columns, there are five selection boxes for column widths, labeled '70', '12', '20', '15', and '13', followed by an equals sign and a zero. To the left of the main area, there are three edit report buttons (labeled 1, 2, 3) and a preview report button (labeled 4). Below these are three buttons: a left arrow (labeled 6), a cancel button with an 'X' (labeled 7), and a check button with a checkmark (labeled 8). To the right, there are two scroll buttons for 'Accession Num' (labeled 11) and 'Bottle Type' (labeled 12). Below these are two sort order buttons (labeled 10) and two checkboxes for section breaks (labeled 9). The screen is divided into two main sections by a horizontal line, with the top section labeled '1' and the bottom section labeled '2'.

Figure 7-13: Status Report Configuration Screen (Clinical Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Unload Report Configuration Screen (Clinical Use)

2.15 **Bact/ALERT 3D Select** 22:28:22 07/26/05

1 Unload Report

2 ..1.. ..2.. ..3.. ..4.. ..5..

3 Accession Num Last Name Hospital ID Bottle ID Cell ID

70 - 13 - 20 - 7 - 10 = 0

4 1 2 3

5

6 7 8

9 10

11 12

Figure 7-14: Unload Report Configuration Screen (Clinical Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Load Report Configuration Screen (INDUSTRY Use)

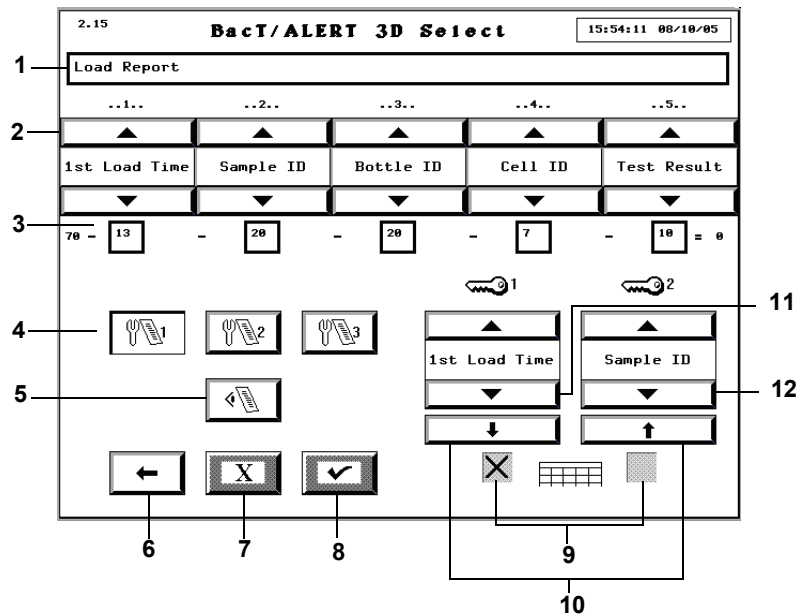


Figure 7-15: Load Report Configuration Screen (INDUSTRY Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Status Report Configuration Screen (INDUSTRY Use)

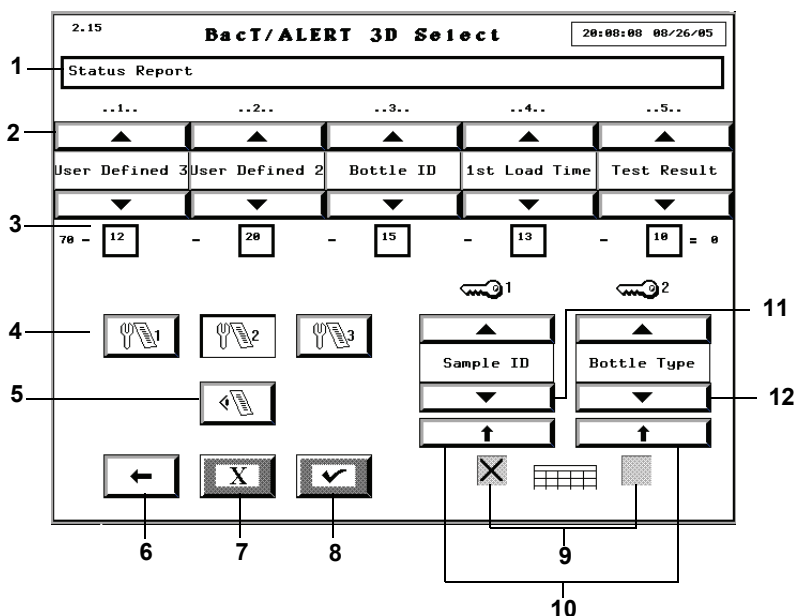


Figure 7-16: Status Report Configuration Screen (INDUSTRY Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Unload Report Configuration Screen (INDUSTRY Use)

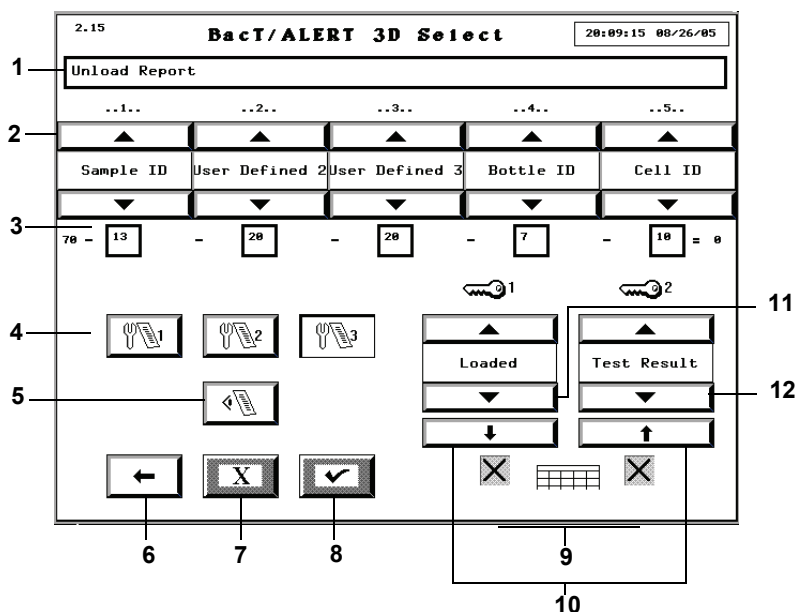


Figure 7-17: Unload Report Configuration Screen (INDUSTRY Use)

- 1 — Report Title
- 2 — Column Selection Scroll Buttons
- 3 — Column Width Selection Boxes
- 4 — Edit Report Buttons
- 5 — Preview Report Button
- 6 — Previous Screen Button
- 7 — Cancel Button
- 8 — Check Button
- 9 — Section Break Checkboxes
- 10 — Sort Order Buttons
- 11 — Primary Sort Scroll Button
- 12 — Secondary Sort Scroll Button

Viewing and Printing Calibration Data



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) From the Setup screen, press the **Report** button ().

The Report Selection screen appears.

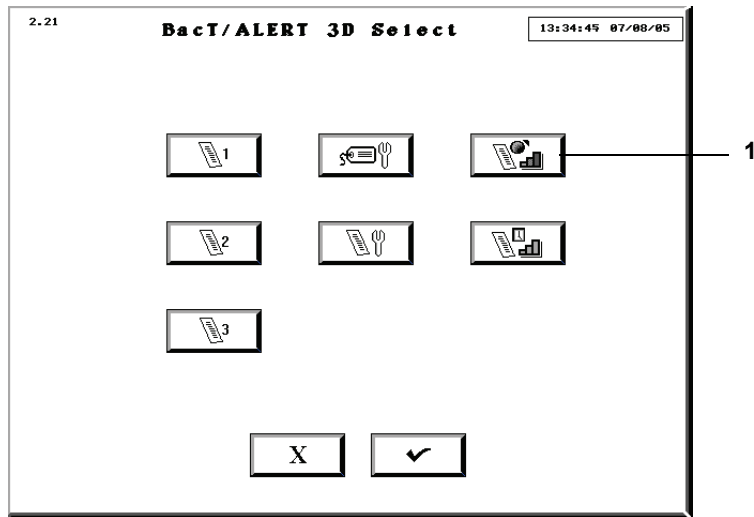


Figure 7-18: Report Selection Screen

1 — Calibration Report Button

- 3) Press the **Calibration Report** button ().

The Cell Calibration Report screen appears.

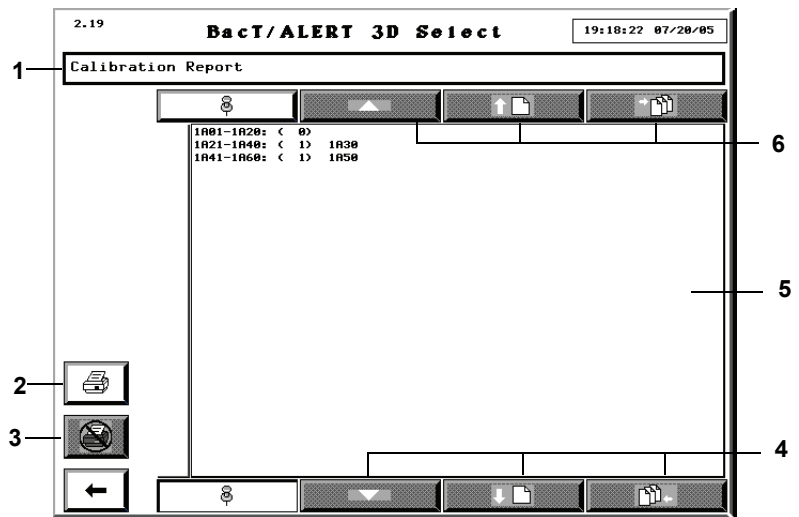


Figure 7-19: Cell Calibration Report

- 1 — Report Header
- 2 — Print Button
- 3 — Cancel Print Button
- 4 — Scroll Down Buttons
- 5 — View Window
- 6 — Scroll Up Buttons


- 4) Enter a *report title* in the **Report Header** field (a maximum of 64 alphanumeric characters).
- 5) To scroll up a line, page, or home, press the appropriate **Scroll Up** buttons.
- 6) To scroll down a line, page, or end, press the appropriate **Scroll Down** buttons.

Note: The **Scroll Up/Down** buttons are disabled if there are no available lines/pages in the indicated direction.

- 7) To print the report, press the **Print** button.

Note: The **Print** buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.

- 8) When the report begins printing:

- The **Print** button is disabled while a report is printing. You may, however, view and scroll through the displayed report.
- The **Exit** button is disabled while a report is printing.
- The **Cancel Print** button () is enabled. Press the **Cancel Print** button to stop the printing as soon as the print queue empties.

- 9) Press the **Previous Screen** button to return to the Report Selection screen.

The report content is displayed in the View window and contains the following features:

- The ID of each installed rack.
- The number of cells out of calibration for the rack just identified.
- The ID of the cells that are out of calibration for the rack just identified.

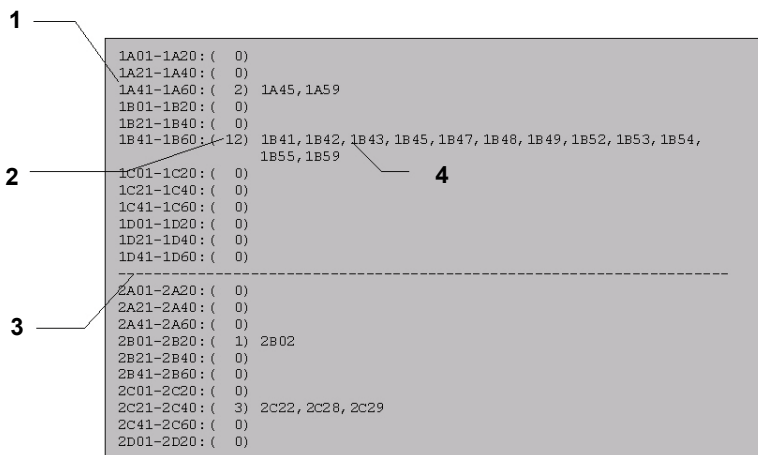


Figure 7-20: Calibration Report

- 1 — Rack ID**
- 2 — Number of Cells out of Calibration for this Rack**
- 3 — Incubation Module Separator Line**
- 4 — IDs of Cells out of Calibration**


Note: Always print a calibration report prior to calibrating any cells. Then print a report after calibrating the cells. In this manner, you will have a record showing corrective action was taken.

Viewing and Printing Calibration History

The Calibration History screen shows you when cells went in and out of calibration.



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) From the Setup screen, press the **Report** button ().

The Report Selection screen appears.

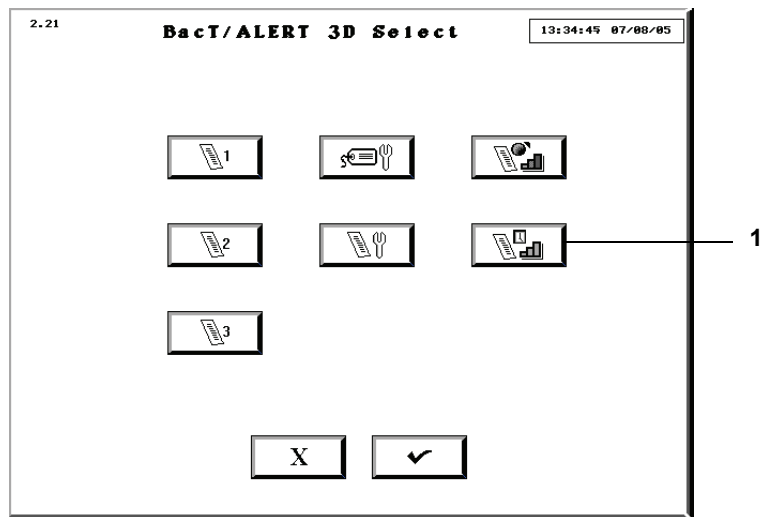


Figure 7-21: Report Selection Screen

1 — Calibration History Button

- 3) Press the **Calibration History** button ().

The Calibration History screen appears.

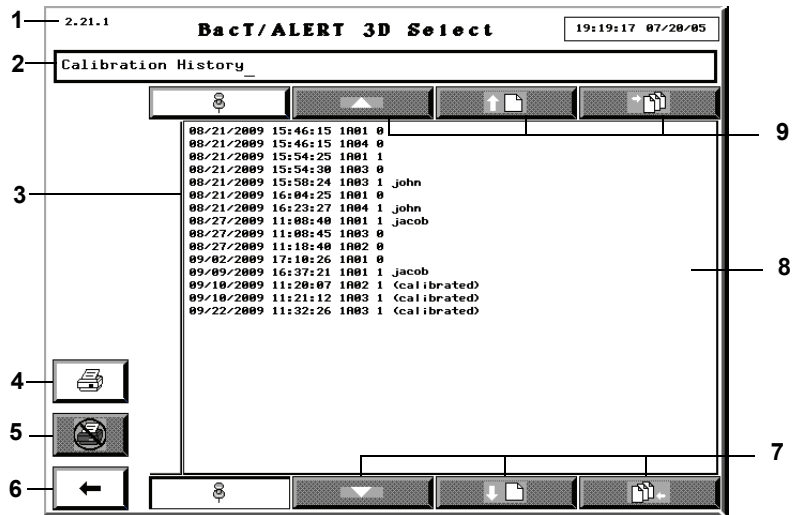


Figure 7-22: Calibration History Screen

- 1 — Screen ID
- 2 — Report Header
- 3 — Date, Time, Cell Location, and Cell Status
- 4 — Print Button
- 5 — Cancel Print Button
- 6 — Previous Screen Button
- 7 — Scroll Down Buttons
- 8 — View Window
- 9 — Scroll Up Buttons

The report content is displayed in the View window and contains the following information:

- The date and time event occurred (in or out of calibration)
- Cell Location
- Cell Status: **0** = when cell falls out of calibration/**1** = when cell goes back in calibration

If the cell is manually calibrated, **(calibrated)** is displayed to the right of the **1**. Or, if in 21 CFR Part 11 mode, the *user name* is displayed.

Note: *There is nothing displayed to the right of the **1** or **0** if the cell falls out of calibration, or drifts back into calibration.*

Note: *The last 10,000 calibration events are saved in the calibration history. When the maximum number of events is reached, the oldest events are overwritten.*

- 4) Enter a *report title* in the **Report Header** field (a maximum of 65 alphanumeric characters).


This title appears at the top of the report when printed.

- 5) To scroll up a line, page, or home, press the appropriate **Scroll Up** buttons.
- 6) To scroll down a line, page, or end, press the appropriate **Scroll Down** buttons.

Note: *The **Scroll Up/Down** buttons are disabled if there are no available lines/pages in the indicated direction.*

- 7) To print the report, press the **Print** button.

Note: *The print buttons are only available if a printer is configured for the system. To configure a printer for the system, contact your local bioMérieux Representative.*

- 8) When the report begins printing:
 - The **Print** button is disabled while a report is printing. You may, however, view and scroll through the displayed report.
 - The **Exit** button is disabled while a report is printing.
 - The **Cancel Print** button () is enabled. Press the **Cancel Print** button to stop the printing as soon as the print queue empties.
- 9) Press the **Previous Screen** button to return to the Report Selection screen.

About This Chapter

This chapter provides you with procedures on how to perform hardware and software maintenance on the BacT/ALERT® 3D instrument.

Note: *Information in this chapter that is intended for clinical use only or industry use only is marked Clinical Use or INDUSTRY Use.*

Chapter Contents

Hardware Maintenance • 8-2

Preventative Maintenance • 8-2

Safety Precautions and Procedures • 8-2

Using the Keyboard in Place of the Operator Panel • 8-8

UPS On/Off Button Location • 8-9

Controller/Combination Module Reboot/Shutdown (BacT/ALERT® 3D Select and SelectLink) • 8-10

Controller or Combination Module Startup • 8-13

Full System Shutdown (BacT/ALERT® 3D Signature) • 8-13

Full System Startup (BacT/ALERT® 3D Signature) • 8-14

Set Up an External Monitor • 8-14

Restore Internal Monitor Settings • 8-14

Software Maintenance • 8-15

Restarting an Incubation Module or the Combination Module Drawers • 8-15

Setting and Formatting the System Date and Time • 8-17

Enabling and Disabling Modules, Drawers, Racks, and Cells • 8-18

Adjusting an Incubation or Combination Module's Temperature • 8-21

Calibrating an Instrument Cell • 8-25

Viewing Incubation Module Information • 8-29

Hardware Maintenance

Preventative Maintenance

Routine BacT/ALERT® 3D preventive maintenance is included in bioMérieux's regular service program. There is no additional requirement for customer performed preventive maintenance. Contact your local bioMérieux Representative to schedule periodic service.

Safety Precautions and Procedures

The design of the BacT/ALERT® 3D provides several features in the interest of operator and laboratory safety.

- All glass bioMérieux BacT/ALERT® media bottles are manufactured with a polyethylene safety sleeve to limit the effects of damage to a bottle.
- All bioMérieux BacT/ALERT® media bottles are non-venting (except *i*AST and *i*LYM — Industry Use) to minimize the risk of needle sticks.
- Each bottle cell within the BacT/ALERT® 3D Incubation and Combination Modules is sealed to help contain and minimize effects of liquid leakage.
- A drip tray is incorporated at the bottom of each drawer, beneath the opening end of the racks, to minimize the effect of any spillage of liquid.
- The circulating fans within the Incubation and Combination Modules turn off whenever any drawer is open, minimizing airflow and the potential for aerosols.
- To reduce the chance for binding a heavily overlabeled bottle, Incubation and Combination Module cells are gently tapered and fabricated from a pliable polymer.
- Electrical voltages within each Incubation or Combination Module drawer are low (approximately 5 volts or less) to minimize potential electrical hazards.
- Should the optimal temperature in an Incubation Module be set to a point outside the range 35–37°C, a yellow Operator alert will appear on the Calibrate Module Temperature screen of the Controller or Combination Module.

- Any mention of bleach refers to the standard 5.25% sodium hypochlorite. A 10% bleach solution would indicate a 1:10 dilution of the standard 5.25% sodium hypochlorite.

WARNING



Clinical Use: Pathogenic microorganisms including Hepatitis B virus and Human Immunodeficiency Virus (HIV) may be present in specimens. *Universal Precautions* and Local Laboratory guidelines should be followed in handling all items contaminated with blood or body fluids. If an inoculated bottle is found to be leaking or is accidentally broken during collection or transport, use the established procedures in your facility for dealing with biohazardous material. As a minimum, *Universal Precautions* should be employed. Bottles should be discarded in an appropriate manner.

General Precautions

The following precautions should be observed during maintenance and repair, even in situations where a spill is neither observed nor suspected. Any parts removed or tools used should be cleaned using a 10% bleach solution before removal from the laboratory. Anything that cannot be disinfected should be sealed in a plastic bag, labeled as biohazardous and handled accordingly. In addition, the laboratory's safety precautions should always be observed.

Spill Cleanup



- Where appropriate, follow your laboratory's spill clean up procedures.

WARNING



Clinical Use: For spills that might involve *M. tuberculosis*, proper protective equipment should be worn, including suitable respirator, gloves, eye protection, and a lab coat. In some cases, coveralls or shoe covers should be worn to avoid contaminating street clothing.

- 1) Clinical Use: Gently cover the spill area with a paper towel. Apply a 10% bleach solution or other EPA registered tuberculocidal disinfectant. Apply a 10% bleach solution.

or

INDUSTRY Use: Gently cover the spill area with a paper towel. Apply a 10% bleach solution.

- 2) Using the bleach solution, wet down all surfaces with which the spill may have come in contact.
- 3) Allow all surfaces adequate contact time with the bleach solution (15–30 minutes) before cleanup.
- 4) All materials used in the clean up should be treated as biohazardous waste.

Disinfection Procedure for Spills Onto the Instrument (Clinical Use)



Any blood or test specimen spilled on an instrument should be removed immediately using the following procedure.

WARNING



Protective gloves should always be worn when handling blood, blood components, or blood-contaminated material.

WARNING



Handle specimens and inoculated culture bottles as though capable of transmitting infectious agents. All inoculated bottles, specimen collection needles, and blood drawing devices should be autoclaved before they are discarded.

- 1) Clean the spill from the instrument following your institution's recommended procedure for decontamination or the procedure described in the latest revision of Clinical and Laboratory Standards Institute (CLSI) guideline "Protection of Laboratory Workers from Instrument Biohazards and Infectious Disease Transmitted by Blood, Body fluids, and Tissue" CLSI Document M29-A.
- 2) After decontamination, wipe with damp (water only) towel and thoroughly dry.

Disinfection Procedure for Spills Onto the Instrument (INDUSTRY Use)



Any test sample spilled on an instrument should be removed immediately using the following procedure.

- 1) Clean the spill from the instrument following your laboratory's recommended procedure for decontamination.
- 2) After decontamination, wipe with damp (water only) towel and thoroughly dry.

Disinfection Procedure for Spills Within the Instrument

Note: This procedure applies to both Clinical Use and INDUSTRY Use.



Any test specimen/sample spilled in an instrument should be removed immediately and the affected areas decontaminated using the following procedure.

WARNING



Clinical Use: If a spill is detected that might involve *M. tuberculosis*, only persons wearing protective clothing and suitable respiratory protection should remain in the room.

- 1) Visually inspect the extent of the leakage or spill. Determine if one or more racks, or the drawer itself is contaminated.
- 2) Remove the leaking bottle if possible.

Note: If a bottle becomes lodged in the cell, contact bioMérieux Customer Service. Do not try to dislodge bottle by pulling on the rack.

- 3) Unload positive and negative bottles in affected rack(s) ([see Unloading Bottles in Chapter 3](#), for Clinical Use, or [see Unloading Bottles in Chapter 4](#) for INDUSTRY Use).
- 4) Relocate remaining bottles in affected rack(s) using the following procedure.
 - a. From the Main screen, press **Ctrl + F10** to enter a special Bottles Relocation mode that allows bottles to be relocated.

The cell indicator lights will illuminate in the following pattern:
 - Cells with bottles are lit continuously.
 - Cells available for reloading are unlit.
 - b. Pull bottles out one at a time and reload one at a time into any available cell.

Note: When relocating bottles using **Ctrl + F10**, you must unload one bottle from a cell and reload it into the destination cell **BEFORE** unloading another bottle. This procedure must be done one bottle at a time.

- c. If the **Bottle ID** field for a loaded bottle is blank, be sure to identify the anonymous bottle by scanning the barcode as it is unloaded (see [Handling Anonymous Bottles in Chapter 3](#) for Clinical Use, or see [Handling Anonymous Bottles in Chapter 4](#) for INDUSTRY Use).
- d. When all bottles are unloaded, press the **Check** button on the Main screen.



CAUTION: When unloading multiple bottles using **Ctrl + F10**, the bottles must be reloaded before 1 hour has passed using the normal **Load Bottles** icon to avoid a subculture event.

Note: *Clinical Use: Bottles affected by the spill should not be reloaded until they have been decontaminated. Decontaminate the bottles following your institution's recommended procedure for decontamination or the procedure described in the latest revision of CLSI guideline "Protection of Laboratory Workers from Instrument Biohazards and Infectious Disease Transmitted by Blood, Body Fluids, and Tissue" CLSI Document M29-A. Press the **Load Bottles** button to reload the decontaminated bottles.*

Note: *INDUSTRY Use: Bottles affected by the spill should not be reloaded until they have been cleaned. Clean the bottles following your laboratory's recommended procedure. Press the **Load Bottles** button to reload the cleaned bottles.*

- 5) Disable the affected cells, rack(s), or drawer (see [Enabling and Disabling Modules, Drawers, Racks, and Cells on page 8-18](#)).
- 6) If an affected cell contains a large amount of liquid, carefully aspirate it using a pipette or similar device, and dispose of it in an appropriate biohazardous waste container.

- 7) If the spill is confined to one or a few cells of one drawer rack, the affected cells may be cleaned and disinfected with a 10% bleach solution, using the following procedure:



CAUTION: Do not expose the cell or rack to the bleach solution for an extended period. 10% bleach is the ONLY disinfectant that has been validated for use with the cells. DO NOT use a different disinfectant or a bleach solution stronger than 10% or damage to cell components may occur.

- a. Insert absorbent material, such as gauze, into the cell to remove any remaining fluid. Carefully remove and discard the gauze in a biohazardous waste container.
 - b. Wipe out the interior of the cell with gauze soaked with 10% bleach solution and discard in a biohazardous waste container.
 - c. Insert several layers of gauze saturated with a 10% bleach solution into the cell and let sit for 15 to 30 minutes to decontaminate. The gauze should be soaked to the point of saturation but not so wet that liquid drips out of the cell.
 - d. Remove and discard the gauze in a biohazardous waste container.
 - e. Wipe the interior of the cell with gauze soaked in distilled water to rinse.
 - f. Allow the cell to air dry.
 - g. Calibrate the cell. If the cell passes calibration, enable the cell ([see Enabling and Disabling Modules, Drawers, Racks, and Cells on page 8-18](#)).
- 8) Clinical Use: If the cell fails calibration or there is visible blood stain residue still present in the cell, particularly on the bottom, clean as follows:
- or*
- INDUSTRY Use: If the cell fails calibration or there is visible residue still present in the cell, particularly on the bottom, clean as follows:
- a. Insert absorbent material, such as gauze, into the cell to remove any remaining fluid. Carefully remove and discard the gauze in a biohazardous waste container.
 - b. Rinse with 10% bleach solution, then distilled water and let air dry.
 - c. Calibrate the cell. If the cell still cannot be calibrated, disable the cell and insert an orange cell plug to indicate the cell should not be used.

- 9) For more extensive spills, it may be necessary to remove one or more racks or an entire drawer. To gain access to the bottom drip tray and internal drawer surfaces, remove all three racks, or remove a drawer.

Note: *If removing a rack or drawer is necessary, contact your local bioMérieux Representative.*

Note: *Clinical Use: Decontaminate bottles, cells, racks, and drawers following your institution's recommended procedure for decontamination or the procedure described in the latest revision of CLSI guideline "Protection of Laboratory Workers from Instrument Biohazards and Infectious Disease Transmitted by Blood, Body Fluids, and Tissue" CLSI Document M29-A.*

Note: *INDUSTRY Use: Clean bottles, cells, racks, and drawers following your laboratory's recommended procedure for cleaning.*

Using the Keyboard in Place of the Operator Panel

Normally, the Operator Panel is used to make selections and enter data. The keyboard provides an alternate method as follows (see [Accessing the Controller Module Keyboard in Chapter 2](#)).

- Items on a screen may be selected using the **Tab** key. The selected item is identified by a contrasting color outlining the item.
- Pressing the **Enter** key initiates the function for the selected item.
- For menus that require entry of a numeric password, you can use the numeric keys in place of touching the screen. Use the **1** to **4** keys to enter the password value. Use the **5** key in place of pressing the **Key Symbol** button.
- Scroll buttons are selected using the **Tab** key. Once you select the desired scroll button, use the **>** or **<** keys to select the **Up** or **Down** scroll arrow.

Function keys can be used to invoke the following functions.

Alt + F5	Clears Instrument Fault Code 80 faults for all cells at the same time. This function only works while the Setup screen is displayed and after a password is entered.
-----------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------

- | | |
|-------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Ctrl + F1 | <p>Activates a special unload mode. This identifies bottles that are loaded in cells that are non-operational. The green drawer indicator is lit in drawers containing non-operational cells. When the drawer is opened, the cell indicator lights will light for those bottles in non-operational cells. Cells with rapidly flashing cell indicator lights are disabled or unusable.</p> <p>In this mode, the Controller or Combination Module automatically activates the Unload Bottles function. Bottles can be removed from the indicated cells one at a time and moved to usable cells whose cell indicator light will be off. This function only works while the Main screen is displayed.</p> |
| Ctrl + F2 | <p>Activates a special unload mode. This identifies cells out of calibration. All such cells are identified whether they are loaded or not. The green drawer indicator is lit on drawers containing out-of-calibration cells. When the drawer is opened, the cell indicator lights will light for cells out of calibration. This function only works while the Main screen is displayed.</p> |
| Ctrl + F3 | <p>Activates a special unload mode. This identifies bottles which have an Instrument Fault Code 80. When the drawer is opened, the cell indicator lights will light for those bottles with the 80 instrument fault. In this mode, the instrument automatically activates the Unload Bottles function. This function only works while the Main screen is displayed.</p> |
| Ctrl + F10 | <p>Activates a special bottle relocation mode. This identifies all currently loaded bottles regardless of their test status. The green drawer indicator is lit on drawers with loaded bottles. When the drawer is opened, the cell indicator lights will light for the loaded cells. Cells available for reloading are unlit. This function only works while the Main screen is displayed.</p> |

UPS On/Off Button Location

Controller Module



There are two Controller Module UPS configurations: Front and Rear Access. Use the following steps to identify the relevant configuration.

- 1) Remove the upper Pop-off Access Panel from the front of the Controller Module.
- 2) If configured for Front Access, the UPS **On/Off** button will be visible inside the instrument (see [Figure 8-1](#)). Immediate access to the UPS **On/Off** button is available; otherwise, continue with [Step 3](#).

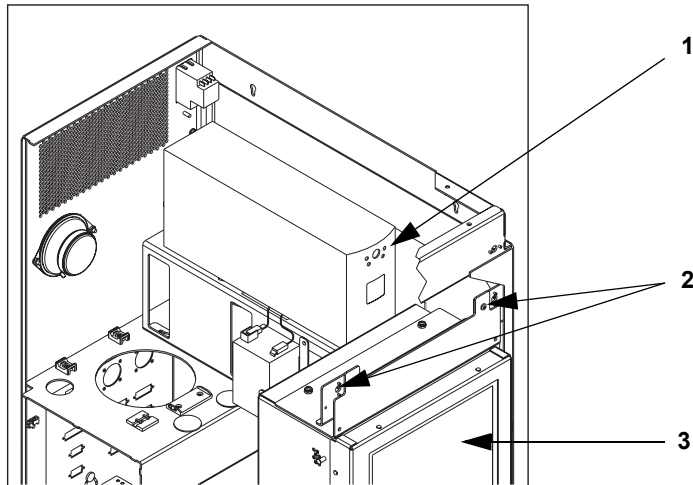


Figure 8-1: Front Access UPS Orientation

1 — UPS On/Off Button

2 — Lower Attachments for Pop-Off Access Panel

3 — Operator Panel

- 3) If the UPS **On/Off** button is not visible, the instrument is configured for Rear Access.
- 4) Carefully move the Controller Module to gain access to the rear of the instrument. Locate and remove the UPS On/Off Access Plug (see [Figure 2-5, Back View Controller Module](#)). The UPS **On/Off** button is approximately ½ inch into the instrument through the Access opening.

Combination Module

The UPS for the Combination Module is external. The UPS **On/Off** button is located at the front of the UPS ([Figure 2-6, Front View Combination Module](#)).

Controller/Combination Module Reboot/Shutdown (BacT/ALERT® 3D Select and SelectLink)



The following situations require the Controller or Combination Module to be turned off or rebooted.

- Anticipated power outage.
- Moving a system to a new location.
- Fixing an unresponsive Operator Panel or keyboard.

Note: Before rebooting the Controller or Combination Module, always perform a system backup (see [Initiating Manual Backup in Chapter 7](#)). Do not proceed with the reboot until the **Backup in Progress** icon no longer displays.

Note: Contact your local bioMérieux Customer Service Representative prior to rebooting to clear an error condition.

Perform in sequence as many of Methods 1 — 4 as are required to power down or reboot the system.



CAUTION: Contact your local bioMérieux Customer Service prior to using Method 4.

Shutdown Method 1

- 1) Access the Controller or Combination Module keyboard (see [Accessing the Controller Module Keyboard in Chapter 2](#)).
- 2) Press each key in the order listed: **Esc Y E S**.
- 3) Wait for the software to exit to a black screen with a C:\ > prompt.

To reboot the program, press each keyboard key in the order listed keeping each key depressed and then simultaneously releasing all three keys: **Ctrl + Alt + Delete**.

or

Alternately, to turn Off the power for the Controller or Combination Module and the UPS, see Method 4.

Shutdown Method 2

- 1) Press the **Right Arrow** button to advance to the Setup screen.

Note: If the Operator Panel is unresponsive, proceed to Method 4.

- 2) Enter the Software Exit Password: **24313124**.
- 3) The password entered is accepted by pressing the **Key Symbol** button at the right end of the row of **Password Entry** buttons.
- 4) After the Software Exit Password is correctly entered, the **Padlock** icon changes to the full open position. The **Exit Software** button will become visible and will be the only active button on the Setup screen.

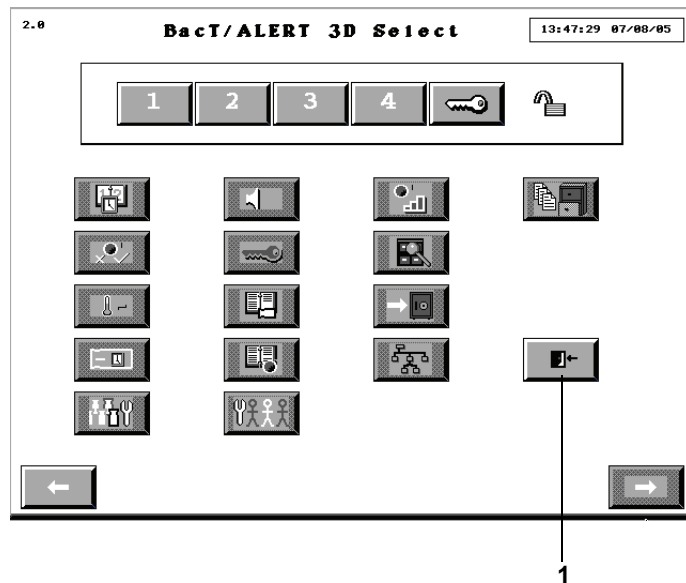


Figure 8-2: Exit Software Button

1 — Exit Software Button

- 5) Press the **Exit Software** button.
- 6) Wait for the C:\> prompt to appear on a black screen.

To reboot the BacT/ALERT® 3D, press each keyboard key in the order listed, keeping each key depressed. Then simultaneously release all three keys: **Ctrl + Alt + Delete**.

or

Alternately, to turn Off the power for the Controller or Combination Module and the UPS, see Method 4.

Shutdown Method 3

- 1) Access the Controller or Combination Module keyboard (see [Accessing the Controller Module Keyboard in Chapter 2](#)).
- 2) Press each keyboard key in the order listed, keeping each key depressed and then simultaneously releasing all four keys: **Ctrl + Alt + Shift + 2**.

Note: If keyboard is unresponsive, proceed to Method 4.

- 3) Wait for the C:\> prompt to appear on a black screen.

To reboot the program, press each keyboard key in the order listed keeping each key depressed and then simultaneously releasing all three keys: **Ctrl + Alt + Delete**.

or

Alternately, to turn Off the power for the Controller or Combination Module and the UPS, see Method 4.

Shutdown Method 4



CAUTION: Contact your local bioMérieux Customer Service Representative prior to using Method 4.

- 1) Turn Off the Controller or Combination Module Power Switch. This switch is located in the upper left corner facing the back of the module.
- 2) Press the UPS **On/Off** button once to turn Off the power to the UPS ([see UPS On/Off Button Location on page 8-9](#)).

Controller or Combination Module Startup



- 1) Turn On the Controller or Combination Module Power Switch. This switch is located in the upper left corner facing the back of the module.
- 2) Press the UPS **On/Off** button once to turn On the power to the UPS ([see UPS On/Off Button Location on page 8-9](#)).

Full System Shutdown (BacT/ALERT® 3D Signature)

If you have a BacT/ALERT® 3D Signature system and there is an anticipated power outage, or if the entire system (including the bioMérieux data management computer) must be moved, a full system shutdown is required. Use the following steps to perform a full system shutdown.



- 1) If applicable, exit bioMérieux data management computer application.
- 2) If applicable, turn Off the power to the bioMérieux data management system computer and other hardware.
- 3) Exit software on the Controller or Combination Module ([see Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\) on page 8-10](#)).
- 4) Turn Off the power of the Controller or Combination Module ([see Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\) on page 8-10](#)).

- 5) Turn Off power to the UPS (see [UPS On/Off Button Location on page 8-9](#)).
- 6) Turn Off the power of the Incubation Module(s).

Full System Startup (BacT/ALERT® 3D Signature)



Use the following steps to restart the BacT/ALERT® 3D Signature system after performing the Full System Shutdown.

- 1) If applicable, turn On the power to all of the bioMérieux data management system hardware.
- 2) Turn On the power to the UPS (see [UPS On/Off Button Location on page 8-9](#)).
- 3) Turn On the power for the Controller or Combination Module.
- 4) Turn On the power for the Incubation Module(s).

Set Up an External Monitor



Use the following steps to set up an external monitor for use with the BacT/ALERT® 3D.

Note: Only newer versions of the CPU will support an external monitor.

- 1) Connect the external monitor to the **MONITOR** port on the rear panel of the instrument.
- 2) Set the display switch assembly to the **EXTERNAL** position.
- 3) Reboot the system. For Select and SelectLink configurations, see [Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\) in Chapter 8](#). For Signature configuration, see [Full System Shutdown \(BacT/ALERT® 3D Signature\)](#), and [Full System Startup \(BacT/ALERT® 3D Signature\) in Chapter 8](#).

IMPORTANT: *Changes to the display switch assembly settings do not become effective until after the system has been rebooted.*

- 4) Verify that the external monitor is working correctly.

Restore Internal Monitor Settings



Use the following steps to restore the internal monitor settings for the BacT/ALERT® 3D.

- 1) Disconnect the external monitor from the **MONITOR** port on the rear panel of the instrument.
- 2) Set the display switch assembly to the **INTERNAL** position.

- 3) Reboot the system. For Select and SelectLink configurations, [see Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\)](#) in Chapter 8. For Signature configuration, [see Full System Shutdown \(BacT/ALERT® 3D Signature\)](#), and [Full System Startup \(BacT/ALERT® 3D Signature\)](#) in Chapter 8.

IMPORTANT: *Changes to the display switch assembly settings do not become effective until after the system has been rebooted.*

- 4) Verify that the internal monitor is working correctly.

Software Maintenance

The following procedures describe maintenance procedures that are performed using the software. These procedures are available regardless of software configuration, but should not need to be performed on a daily basis. The functions described are:

- Restarting an Incubation Module or the Combination Module drawers ([see Restarting an Incubation Module or the Combination Module Drawers on page 8-15](#)).
- Setting the date/time and selecting date/time formats ([see Setting and Formatting the System Date and Time on page 8-17](#)).
- Enabling and disabling modules, drawers, racks, and cells ([see Enabling and Disabling Modules, Drawers, Racks, and Cells on page 8-18](#)).
- Checking the temperature, setting the optimal temperature, and calibrating the temperature of the Incubation or Combination Modules ([see Adjusting an Incubation or Combination Module's Temperature on page 8-21](#)).
- Calibrating and viewing readings of cells ([see Calibrating an Instrument Cell on page 8-25](#)).
- Viewing Incubation Module information ([see Viewing Incubation Module Information on page 8-29](#)).

Restarting an Incubation Module or the Combination Module Drawers



Perform the following to restart an Incubation Module or the drawers of a Combination Module without using the power switch in the case of a communication problem.

- 1) Touch the **Instrument** icon on the Main screen ([see Figure 3-1, Main Screen](#) for Clinical Use, or [Figure 4-1, Main Screen](#) for INDUSTRY Use).

The View Cell Status screen appears.

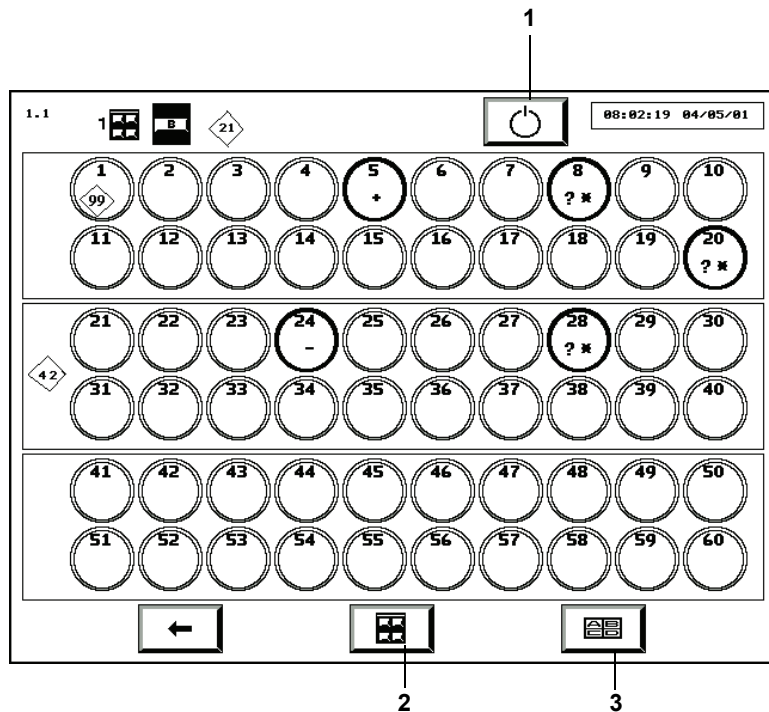


Figure 8-3: View Cell Status Screen

1 — Restart Incubation Module Button

2 — Incubation Module Selection Button

3 — Drawer Selection Button

- 2) Press the **Incubation Module Selection** button, if necessary, to display a drawer from the affected Incubation or Combination Module.
- 3) Press the **Restart Incubation Module** button.

Note: It is normal for Instrument Status Code 710 to display briefly.


Setting and Formatting the System Date and Time

Note: The date and time are set at installation, but the system does not adjust automatically for daylight savings time.



CAUTION: DO NOT change the date without first contacting your local bioMérieux Customer Service Representative.



- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).
- 2) From the Setup screen, press the **Set Date/Time** button ().

The Set Date/Time screen overlays and disables the Setup screen.

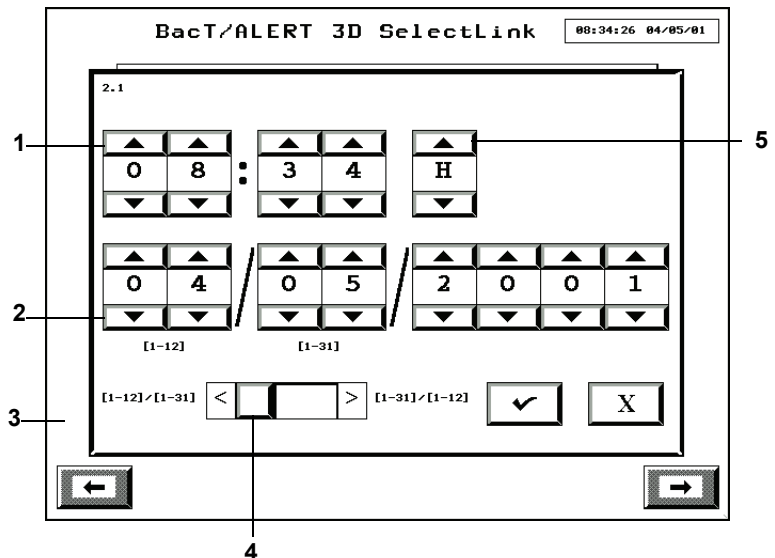


Figure 8-4: Set Date/Time Screen

- 1 — Set Time Scroll Buttons
- 2 — Set Date Scroll Buttons
- 3 — Setup Screen (Disabled)
- 4 — Date Format Sidebar Switch
- 5 — Time Format Scroll Button

- 3) Adjust the time using the **Set Time** scroll buttons. The scroll buttons to the left of the colon determine the hour, and the scroll buttons to the right of the colon determine the minutes.
- 4) Adjust the time format using the **Time Format** scroll button. When the scroll button is set to **AM** or **PM**, a 12-hour format is employed. If **H** is selected, the 24-hour format will apply.
- 5) Adjust the date using the **Set Date** scroll buttons. The two scroll bars used to enter the month have [1–12] displayed below them, and the two scroll bars used to enter the day have [1–31] displayed below them. The order of these will depend on the date format selection.
- 6) Adjust the date format using the **Date Format** sidebar switch. The left option is MM/DD/YY and the right option is DD/MM/YY.

Note: *The date format selected applies to the current date/time display at the top of each screen, to all fields that display a date and/or time, and to reports where available.*

- 7) Press the **Check** button to accept the changes, or press the **Cancel** button to retain the original settings.
- 8) The system returns to the Setup screen.

Note: *With BacT/ALERT® 3D Signature systems, changes to date and time in the bioMérieux data management computer will be reflected in the BacT/ALERT® 3D. Changes made at the BacT/ALERT® 3D do not affect the data management computer.*

Enabling and Disabling Modules, Drawers, Racks, and Cells



- 1) Before disabling an Incubation Module, drawer, rack, or cell, make certain there are no bottles in the part to be disabled ([see Relocating Bottles on page 8-21](#)).
- 2) Access the Setup screen and enter a valid password ([see Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).
- 3) Press the **Enable/Disable Module, Drawer, Rack, or Cell** button



The Enable/Disable Module, Drawer, Rack, or Cell screen overlays and disables the Setup screen.

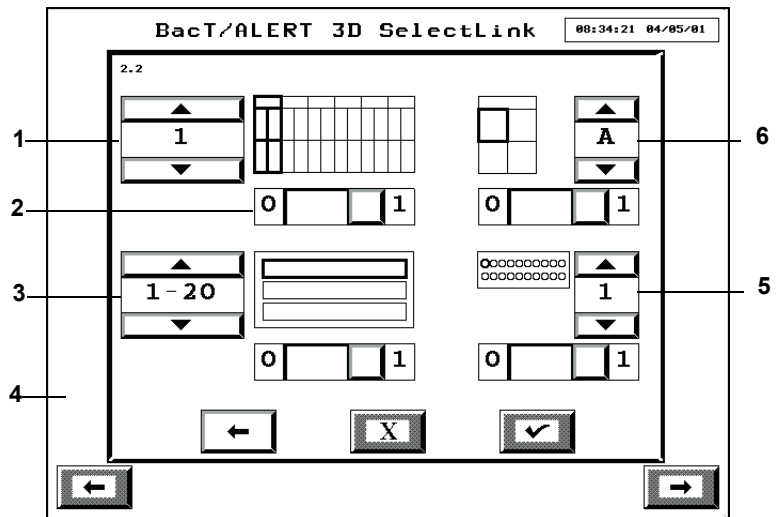


Figure 8-5: Enable/Disable Module, Drawer, Rack, or Cell Screen

- 1 — Incubation Module Scroll Button
- 2 — Enable/Disable Slider Switch
- 3 — Rack Scroll Button
- 4 — Setup Screen (Disabled)
- 5 — Cell Scroll Button
- 6 — Drawer Scroll Button

- 4) To enable/disable an entire Incubation module or Combination Module drawers:
 - a. Select the module desired (1–6) with the **Incubation Module** scroll button.
 - b. Set the Enable/Disable slider switch below the **Incubation Module** icon to enable (**1**) or disable (**0**).
 - c. Go to [Step 8](#).

Note: *Disabling a module also disables the drawers, racks, and cells contained in that module; however, disabling the module will not affect the slider settings for those drawers, racks, and cells.*

- 5) To enable/disable an entire drawer:
 - a. Select the Incubation or Combination Module (1–6) that holds the drawer with the **Incubation Module** scroll button.

- b. Select the Drawer desired (A–D) with the **Drawer** scroll button.
- c. Set the Enable/Disable sidebar switch below the **Drawer** icon to enable (1) or disable (0).
- d. Go to [Step 8](#).

Note: *If a drawer is disabled, then its racks and cells are disabled and turn gray; however, disabling a drawer does not change the current setting of the slidebars for the corresponding racks and cells.*

- 6) To enable/disable an entire rack:
 - a. Select the Incubation or Combination Module (1–6) that holds the rack with the **Incubation Module** scroll button.
 - b. Select the drawer (A–D) that holds the rack with the **Drawer** scroll button.
 - c. Select the rack desired (1–20, 21–40, or 41–60) with the **Rack** scroll button.
 - d. Set the **Enable/Disable** sidebar switch below the **Rack** icon to enable (1) or disable (0).
 - e. Go to [Step 8](#).

Note: *If a rack is disabled, then the cells in that rack are disabled and turn gray; however, disabling a rack does not change the current setting of each of the slidebars for those cells.*

- 7) To enable/disable a cell:
 - a. Select the Incubation or Combination Module (1–6) that holds the cell with the **Incubation Module** scroll button.
 - b. Select the Drawer (A–D) that holds the cell with the **Drawer** scroll button.
 - c. Select the Rack (1–20, 21–40, or 41–60) that holds the cell with the **Rack** scroll button.
 - d. Select the Cell desired (1–20) with the **Cell** scroll button.
 - e. Set the **Enable/Disable** sidebar switch below the **Cell** icon to enable (1) or disable (0).
- 8) Press the **Check** button to accept change(s), or the **Cancel** button to return previous settings.
- 9) Press the **Previous Screen** button to return to the Setup screen.

Relocating Bottles



In certain situations (ex. before disabling a rack or drawer), it may be necessary to move bottle(s) into another available cell, rack, or drawer.



CAUTION: When unloading multiple bottles using Ctrl + F10, the bottles must be reloaded before 1 hour has passed using the normal Load Bottles icon to avoid a subculture event.

- 1) From the Main screen, press **Ctrl + F10** to enter a special Bottles Relocation mode. The cell indicator lights will illuminate in the following pattern:
 - Cells with bottles loaded are lit continuously.
 - Cells available for reloading remain unlit.
- 2) Pull bottles out one at a time and reload one at a time into any available cell.

Note: When relocating bottles using **Ctrl + F10**, you must unload one bottle from a cell and reload it into the destination cell **BEFORE** unloading another bottle. This procedure must be done one bottle at a time.

- 3) If the Bottle ID field for a loaded bottle is blank, be sure to identify the anonymous bottle by scanning the barcode as they are unloaded (see [Handling Anonymous Bottles in Chapter 3](#)).
- 4) When all bottles are relocated, press the **Check** button on the Main screen (see [Figure 3-1, Main Screen](#)).

Adjusting an Incubation or Combination Module's Temperature

The following procedures describe:

- How to check an Incubation or Combination Module's temperature.
- How to set an Incubation or Combination Module's optimal temperature setting.
- How to calibrate an Incubation or Combination Module's temperature.

Checking an Incubation or Combination Module's Temperature



Each Incubation and Combination Module includes either an NIST-traceable reference thermometer, or a digital thermometer, in one of two different configurations.

The NIST-traceable reference thermometer (see [Figure 8-6](#)) is located in the back of drawer B, if module is right-handed, or in the back of drawer A for left-handed modules. This thermometer is the reference used to calibrate the module temperature.

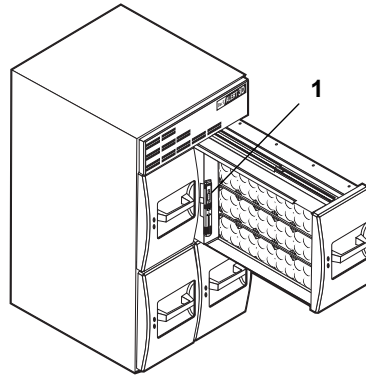


Figure 8-6: Reference Thermometer (Drawer B — Right-hand Configuration)

1 — Reference Thermometer

The digital thermometer (see [Figure 8-7](#)) is located on the inner front panel of drawer B, if module is right-handed, or on the inner front panel of drawer A, for left-handed modules. This thermometer is the reference used to calibrate the module temperature.

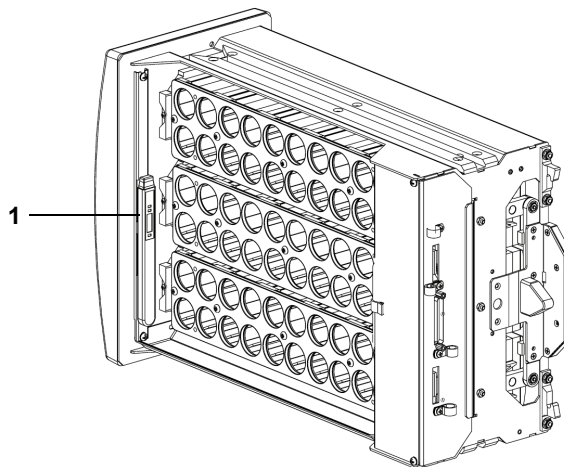
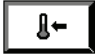


Figure 8-7: Digital Thermometer (Drawer A — Left-hand Configuration)

1 — Digital Thermometer

Note: The thermometer should be periodically validated according to your current protocol.

Before opening the drawer to take a reading with this thermometer, verify that the module temperature is stable by doing the following.

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).
- 2) Press the **Calibrate Module Temperature** button ().

The Calibrate Module Temperature screen overlays and disables the Setup screen.

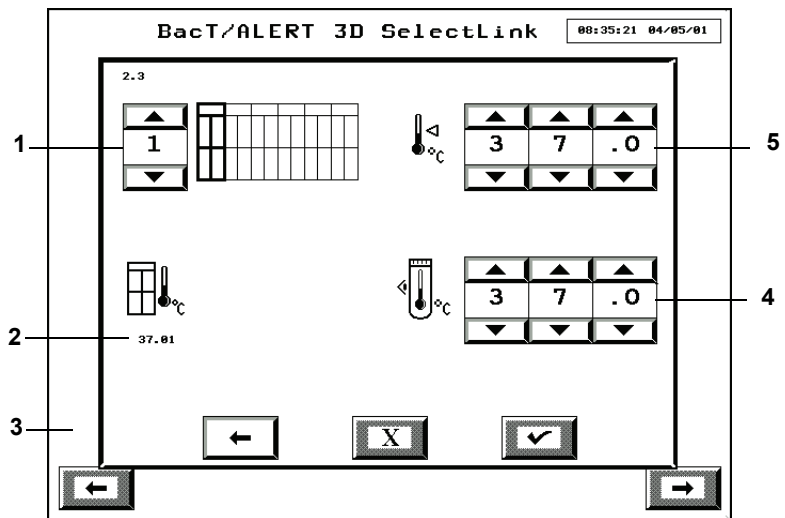


Figure 8-8: Calibrate Module Temperature Screen

- 1 — Incubation Module Scroll Button
- 2 — Internal Air Temp Indicator
- 3 — Setup Screen (Disabled)
- 4 — Actual Temp Scroll Buttons
- 5 — Optimal Temp Scroll Buttons

- 3) Select the desired Incubation or Combination Module (1–6) with the **Incubation Module Scroll** button.

- 4) Observe the icon to the left of the **Actual Temp** scroll buttons.



Indicates the temperature is not stable. Do not read the Reference Thermometer at this time.



14:51



Indicates the temperature is nearly stable. The time underneath the hourglass is the number of minutes: seconds remaining until stable temperature is reached.



Indicates the temperature is stable. The Reference Thermometer may be read.

- 5) Once the temperature is stable, open the drawer containing the Reference Thermometer and read the temperature.
- 6) Close the drawer immediately after taking the reading.

Setting the Optimal Temperature for an Incubation or Combination Module




- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) Press the **Calibrate Module Temperature** button ().

The Calibrate Module Temperature screen overlays and disables the Setup screen (see [Figure 8-8, Calibrate Module Temperature Screen](#)).

- 3) Select the desired Incubation or Combination Module (1–6) with the **Incubation Module** scroll button.
- 4) Enter the desired optimal temperature in degrees Celsius (°C) on the **Optimal Temp** scroll buttons.

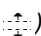Note: *Clinical Use:* The valid range of values for the optimal temperature of an Incubation or Combination Module is 25.0°C to 45.0°C. However, the usual test range encountered in clinical laboratories is 35°C to 37°C. If a temperature is entered that is outside the range of 35°C to 37°C, a bright yellow icon () is displayed below the scroll button. **The lowest temperature the instrument can maintain is 7°C above ambient.**

Note: *INDUSTRY Use:* The valid range of values for the optimal temperature of an Incubation or Combination Module is 25.0°C to 45.0°C. **The lowest temperature the instrument can maintain is 7°C above ambient.**

- 5) Press the **Check** button to initiate the temperature adjustment, or press the **Cancel** button to retain the previous setting.
- 6) Press the **Previous Screen** button to return to the Setup screen.
- 7) After the temperature becomes stable (see [Checking an Incubation or Combination Module's Temperature on page 8-21](#)), calibrate the Incubation module.

Calibrating an Incubation or Combination Module's Temperature



Note: *Temperature calibration is required: Anytime the optimal (setpoint) temperature is changed; or anytime the periodic check of the Reference Thermometer reveals that the temperature is more than 0.5°C above or below the module's setpoint.*

- 1) Check the Incubation or Combination Module's temperature (see [Checking an Incubation or Combination Module's Temperature on page 8-21](#)).
- 2) Enter the temperature from the thermometer in degrees Celsius (°C) on the **Actual Temp** scroll buttons (see [Figure 8-8, Calibrate Module Temperature Screen](#)).
- 3) Press the **Check** button to initiate the calibration, or press the **Cancel** button to retain the previous calibration.
- 4) Press the **Previous Screen** button to return to the Setup screen.
- 5) Check the temperature when stable to verify the calibration was successful.

Calibrating an Instrument Cell

The following procedures describe how to:

- locate a cell which failed calibration
- view a cell's readings
- calibrate a cell

Note: It is not necessary to perform routine cell calibration. If a cell fails its automatic internal diagnostic check, Instrument Fault Code 60 appears in the **Instrument** icon on the Main screen.



CAUTION: A cell which has failed the automatic internal diagnostic check no longer records bottle readings and will not be indicated as available when loading bottles. The cell must be calibrated or disabled.



CAUTION: Do not invoke the Calibrate Cell screen unless the cell to be calibrated is empty.

Locating a Cell Which Failed Calibration



- 1) View the Main screen to locate the Instrument Fault Code 60 in the **Instrument** icon.
- 2) Locate the cell with the fault ([see Viewing Faults in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).


Note: If the cell cannot be calibrated immediately, disable the cell ([see Enabling and Disabling Modules, Drawers, Racks, and Cells on page 8-18](#)).



CAUTION: To prevent anonymous loading into the disabled cell, place an orange cell plug into the cell.

Viewing a Cell's Readings and/or Calibrating a Cell



- 1) If calibrating, print out the Cell Calibration Report ([see Viewing and Printing Calibration Data in Chapter 7](#)).
- 2) If the cell to be calibrated contains a bottle, then relocate it ([see Relocating Bottles on page 8-21](#)).
- 3) Access the Setup screen and enter a valid password ([see Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).
- 4) From the Setup screen, press the **Calibrate Cell** button (.

The Calibrate Cell screen overlays and disables the Setup screen.


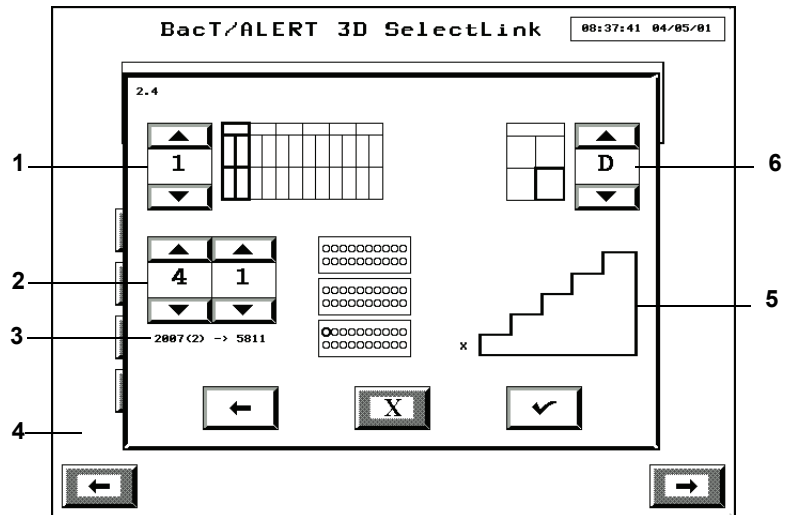


Figure 8-9: Calibrate Cell Screen

- 1 — Incubation Module Scroll Button
- 2 — Cell Scroll Buttons
- 3 — Cell Reading Indicator
- 4 — Setup Screen (Disabled)
- 5 — Calibration Staircase Icon
- 6 — Drawer Scroll Button

- 5) Select a particular cell to be calibrated:
- Choose the appropriate Incubation or Combination Module (1–6) with the **Incubation Module scroll** button.
 - Choose the appropriate drawer (A–D) with the **Drawer** scroll button.
 - Choose the appropriate cell (1–60) with the **Cell** scroll buttons.

The first screen displays an **X** at the base of the **Staircase** icon. This prompts the user to verify the cell is empty and the drawer is closed.

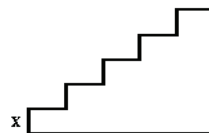

Note: The **Module**, **Drawer**, and the **Cell** scroll buttons will be grayed-out and inactive until calibration of the cell is complete.

To view a cell's reading:


- 6) The cell's readings can now be read below the **Cell** scroll buttons. The first set of numbers represents the uncalibrated reading and the number to the right of the arrow represents the calibrated readings.

Note: *If an Incubation module is not configured properly or is not responding, asterisks will replace the digits on the screen.*

If you would like to calibrate a cell, proceed to next step.


- 7) Press the **Check** button.

The cell light of the cell selected for calibration is now illuminated and **#1** appears above the first step of the **Calibration Staircase** icon.

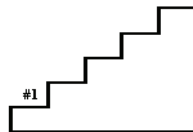


- 8) Insert Standard Number One into the selected cell without touching the ends of the calibration standard as calibration could be affected. (A single ring around the end of the reflectance standard identifies Standard #1).
- 9) Press the **Check** button.

A **#2** appears above the second step of the **Calibration Staircase** icon.

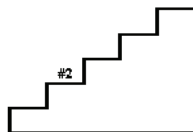


- 10) Insert Standard Number Two into the selected cell. (Two rings around the end of the reflectance standard identifies Standard #2).
- 11) Press the **Check** button.

A **#3** appears above the third step of the **Calibration Staircase** icon.

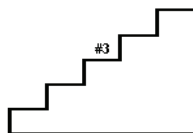


- 12) Insert Standard Number Three into the selected cell. (Three rings around the end of the reflectance standard identifies Standard #3).
- 13) Press the **Check** button.

A **#4** appears above the fourth step of the Calibration Staircase icon.

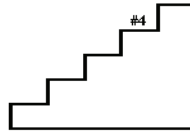

- 14) Insert Standard Number Four into the selected cell. (Four rings around the end of the reflectance standard identifies Standard #4).
- 15) Press the **Check** button.
- 16) If a check mark appears at the top of the **Calibration Staircase** icon, then the calibration of the cell was successful. Remove Standard Number Four and press the **Check** button to save the new calibration values.

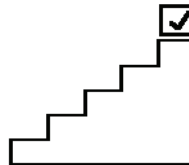

- 17) If Instrument Fault Code 60 appears at the top of the **Calibration Staircase** icon, then the calibration of the cell is unsuccessful. Press the **Cancel** button and follow [Step 4](#) through [Step 14](#) to recalibrate the cell.

Note: *The cell is not automatically enabled after calibration is completed successfully. If the cell was previously disabled, it must be enabled (see [Enabling and Disabling Modules, Drawers, Racks, and Cells on page 8-18](#)).*

- 18) Press the **Previous Screen** button to return to the Setup screen.

Note: *The calibration process can be canceled and any new calibration values discarded by pressing the **Cancel** button at any step during the calibration process.*

- 19) Print out the Cell Calibration Report (see [Viewing and Printing Calibration Data in Chapter 7](#)).

Viewing Incubation Module Information



This procedure describes how to locate the following information:

- Serial Numbers
- Hardware Revisions
- Software Revisions

You may be requested to retrieve this information during the course of troubleshooting by bioMérieux Instrument Services Personnel.

- 1) Access the Setup screen and enter a valid password to access the function buttons (see [Accessing the Setup Screen Function Buttons in Chapter 3](#) for Clinical Use, or in [Chapter 4](#) for INDUSTRY Use).

- 2) Press the **View Incubation Module Information** button ().

The View Incubation Module Information screen replaces the Setup screen.

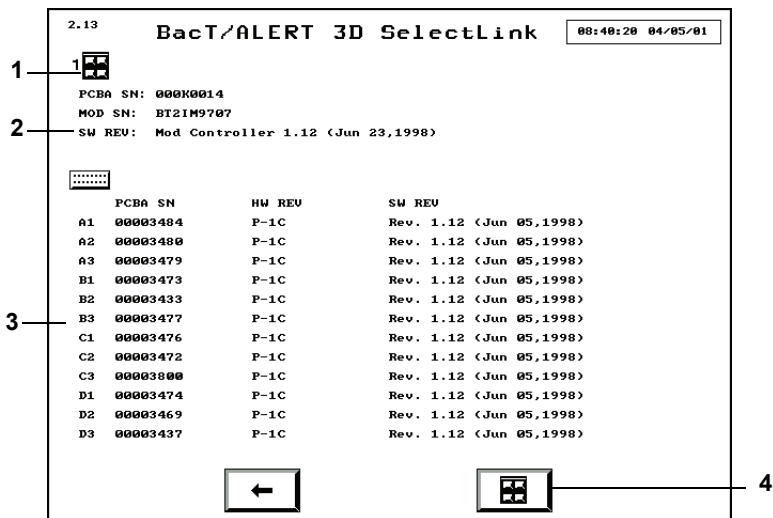


Figure 8-10: View Incubation Module Information Screen

- | | |
|-----------------------------------|----------------------------------------|
| 1 — Incubation Module Indicator | 3 — Rack Information |
| 2 — Incubation Module Information | 4 — Incubation Module Selection Button |

- 3) Select the appropriate Incubation module by pressing the **Incubation Module Selection** button.
- 4) Press the **Previous Screen** button to return to the Setup screen.

About This Chapter

This chapter provides you with a description of the different types of instrument fault, instrument status and operator error codes, as well as bottle and user output device problems, that may be encountered when using the instrument. Cause(s) and solutions(s) for each type of fault/error/problem are also listed.

Note: *Information in this chapter that is intended for clinical use only or industry use only is marked Clinical Use or INDUSTRY Use.*

Note: *In this chapter, “accession number” refers to Clinical Use, and “sample ID” refers to INDUSTRY Use.*

Chapter Contents

Introduction • 9-2

Fault Codes • 9-3

Instrument Fault Codes • 9-3

Instrument Status Codes • 9-24

Operator Error Codes • 9-27

Bottle Problems • 9-39

User Output Device Problems • 9-39

Introduction



CAUTION: Failure to follow the procedures in this User Manual or failure to attend to fault conditions reported by the instrument within 1 hour may lead to invalid test results and the need to subculture bottles.

The following codes are discussed in this chapter.

1	39	56	81	921
2	41	57	99	923
3	42	60	710	930
4	43	62	800	931
5	44	71	810	932
10	45	72	820	940
11	46	73	901	941
12	47	74	902	942
14	51	75	909	943
19	52	77	910	944
20	53	78	911	945
21	54	79	912	960
22	55	80	913	961

Note: *If the problem cannot be corrected using the solutions listed or a code is displayed that does not appear in this list, contact your local bioMérieux Representative.*

Fault Codes

Instrument Fault Codes

Instrument Fault Codes appear in the Instrument icon. A description of each Instrument Fault Code is provided. The following is an example of an Instrument icon with Instrument Fault Codes 4, 10, and 20. Fault codes are displayed on the particular components where the errors are located.

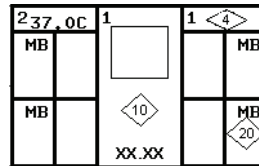


Figure 9-1: Instrument Icon with Fault Codes

Note: Locate specific bottles with fault by pressing the Incubation Module on the **Instrument** icon on the Main screen to display the View Cell Status screen.



CAUTION: Fault codes referring to loss of incubator temperature may adversely affect bottle testing and should be addressed immediately.

#1 POWER FAULT IN INCUBATION MODULE

Cause: Power cable is missing or loose.

Solution: Secure both ends of power cable.

Cause: Incubation Module Power Switch is turned Off.

Solution: Turn on Power Switch (located in upper left corner facing the back of the Incubation Module).

#2 COMMUNICATION LOST WITH INCUBATION MODULE

Cause: Communications cable between the Controller or Combination Module and the Incubation Module is missing or loose.

Solution: Secure both ends of the communications cable. Restart the Incubation Module ([see Restarting an Incubation Module or the Combination Module Drawers in Chapter 8](#)).

Cause: Possible hardware problem with the Controller Module Board.

Solution: Call bioMérieux. DO NOT load bottles in the affected Incubation Module(s) until the fault is corrected.

#3 INCUBATION MODULE TEMPERATURE IS TOO HIGH

Background Information: The optimal temperature (setpoint) and actual Incubation or Combination Module temperature are not equilibrated. Two temperature sensors, one in the supply side of the air plenum, and one in the cold air return, are used to monitor the Incubation or Combination Module temperature. This error will occur if there is a $\pm 2^{\circ}\text{C}$ change from the setpoint measured at the sensors for approximately 1 minute (5 or more readings are taken every 10 seconds). Five or more readings out of range are required to report this error. Common causes for this error are: ambient room temperature is too high, temperature calibration was not performed correctly, or defective hardware (for example, module controller, temperature sensors, or relay).

Cause: The Incubation or Combination Module optimal temperature (setpoint) has been changed and has not yet equilibrated at the new temperature.

Solution: Wait for the module temperature to equilibrate.

Cause: Operating temperature is not within specification for the Incubation or Combination Module.

Solution: Perform the following procedure:

1. Ensure the operating temperature is within the specification (see [Combination Module Environmental Requirements](#) or [Incubation Module Environmental Requirements in Chapter 2](#)).
2. Wait for the module temperature to equilibrate.

Cause: Bottle testing may be adversely affected due to one of the following reasons:

- Hardware fault with the Controller or Combination Module Board.
- Hardware fault with the Solid State Relays that energize the heater.
- Faulty temperature sensors.

Solution: Contact bioMérieux immediately.

#4 INCUBATION MODULE TEMPERATURE IS TOO LOW

Background Information: The optimal temperature (setpoint) and actual Incubation or Combination Module temperature are not equilibrated. Two temperature sensors, one in the supply side of the air plenum, and one in the cold air return, are used to monitor the Incubation or Combination Module temperature. This error will occur if there is a $\pm 2^{\circ}\text{C}$ change from the setpoint measured at the sensors for approximately 1 minute (5 or more readings are taken every 10 seconds). Five or more readings out of range are required to report this error. Common causes for this error are drawer left open, setpoint changed and the module has not yet equilibrated at the new temperature, drawer switch misaligned, defective hardware (for example, module controller, temperature sensors, relay, cutoff switch tripped, or heater element).

Cause: The Incubation or Combination Module optimal temperature (setpoint) has been changed and the module has not yet equilibrated at the new temperature.

Solution: Wait until module temperature equilibrates.

Cause: One or more drawers have been left open or ajar for an extended period.

Solution: Make sure all doors are tightly closed and allow the module temperature 1 hour to equilibrate.

Cause: Bottle testing may be adversely affected due to one of the following reasons:

- Hardware fault with the Controller or Combination Module Board.
- Hardware fault with the Solid State Relays that energize the heater.
- Faulty temperature sensors.

Solution: Call bioMérieux immediately.

#5 INCORRECT INCUBATION MODULE CONNECTION

Cause: Incubation Module communication cable may be connected to wrong port.

Solution: Call bioMérieux.

Cause: Incubation Module ID switch may be incorrectly set.

Solution: Call bioMérieux. DO NOT load bottles into the affected Incubation Module(s) until the fault is corrected.

#10 CONTROLLER MODULE POWER FAULT

Cause: The Controller Module is not receiving AC power.

Solution:

1. Ensure AC power source is adequate or connect to an alternate power source.
2. The Controller Module runs on UPS power for 1 minute. If AC power is not restored during this time, the Instrument and UPS turn Off. When AC power is restored the Instrument will automatically reboot.
3. If power is not restored within an hour, it may be necessary to subculture bottles.



CAUTION – Do not load or unload bottles until power is restored.

#11 INSUFFICIENT CHARGE IN UPS

Cause: The UPS battery is low.

Solution: Call bioMérieux.

#12 LIS COMMUNICATION FAULT

Cause: The LIS is not responding to the Controller Module.

Solution: Check the LIS. The fault automatically goes away when communication with the LIS resumes.

Cause: The message from the Controller or Combination Module is not reaching the LIS.

Solution: Check the cable connections between the Controller or Combination Module and the LIS.

#14 SYSTEM AUTOMATICALLY REBOOTED

Cause: Unresponsive state detected in the software.

Solution: Call bioMérieux immediately. System performance may be compromised.

#19 SOFTWARE EXCEPTION

Cause: A software exception has occurred.

Solution: Call bioMérieux immediately. System performance may be compromised.

#20 **DRAWER OPEN FOR TOO LONG**

Cause: One or more drawers have been left open or ajar for an extended period.

Solution: Make sure all doors are tightly closed.

Cause: Magnetic switches in the back of the drawer have failed or become misaligned.

Solution: Call bioMérieux.

Cause: Fault in the drawer interface cable or printed board.

Solution: Call bioMérieux.

Cause: Fault in the Controller or Combination Module printed circuit board.

Solution: Call bioMérieux.

#21 **DRAWER AGITATION FAILURE**

Cause: Mechanical failure in the agitation linkage.

Solution: Call bioMérieux.

Cause: Agitation motor either disconnected or inoperative.

Solution: Call bioMérieux.

Cause: Fault in rack #1 (for left facing drawer) or rack #3 (for right facing drawer).

Solution: Call bioMérieux.

#22 UNEXPECTED DRAWER AGITATION

Cause: Instrument not configured properly.

Solution: Relocate bottles in affected drawer using the following procedure:

1. From the Main screen, press **Ctrl + F10**. Cells with bottles are lit continuously.
2. Pull bottles out one at a time.
3. If the bottle is anonymous (the **Bottle ID** field is blank), then enter the bottle ID ([see Text/Data Entry in Chapter 3](#)). When all bottles are unloaded, press the **Check** button on the Main screen.
4. The removed bottles should be subcultured.
5. Disable the drawer ([see Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#)).
6. Reload bottles via the Load Bottle function on the Main screen.
7. Call bioMérieux.

#39 RACK TEMPERATURE FAULT

Background Information: The temperature in a specific rack has changed beyond the setpoint (high or low). Two temperature sensors, one at each end of the bottle rack, are polled for the input signals. This fault code was added as an early warning system requiring the user to observe the system for temperature-related issues. This error will occur if there is a $\pm 4^{\circ}\text{C}$ change from the setpoint measured at the bottle racks for more than 60 minutes (six readings are taken every 10 minutes). Six or more readings out of range are required to report this error. Typically, this error is seen after bottle loading or unloading events. The change in thermal mass takes the BacT/ALERT[®] 3D system several minutes to readjust. If the temperature is high, bottle relocation is required and the defective hardware is suspected (for example: temperature sensor). If the temperature is low, ensure that the drawers are closed and monitor the temperature for the next 30 to 40 minutes. If the fault code does not clear, bottle relocation is required and defective hardware is suspected (for example, module controller, temperature sensors, relay, cutoff switch tripped, or heater element).

Cause: One or more drawers have been left ajar for an extended period or a system component may have failed.

Solution: Perform the following procedure:

1. Check the incubation module temperature with an independent thermometer. If the temperature is correct, go to [Step 4](#).
 - If the temperature is high, the heater temperature sensor may have failed. If temperature of incubator is high, relocate bottles per [Step 3](#).
 - If the temperature is not high, go to [Step 2](#).
2. Make sure all drawers are tightly closed.
 - If an open drawer is found, then close the drawer and wait 30 minutes for the fault to disappear.
 - If no open drawers are found, a system component may have failed. If temperature of incubator is low, relocate bottles as per [Step 3](#).
3. If temperature of incubator is incorrect, relocate all bottles in affected incubator to a different incubator, using the Bottle Relocation Procedure below.
4. If incubator temperature is correct, recycle power to the incubator.

If error will not clear after recycling power, relocate all bottles from the rack(s) display Fault Code #39 to different racks that are not displaying errors. Use the Bottle Relocation procedure below.

BOTTLE RELOCATION PROCEDURE



CAUTION – After Unloading multiple bottles using Ctrl + F10, the bottles must be reloaded before 1 hour has passed using the normal Load Bottles icon to avoid a subculture event.

- a) Remove all positive bottles (anonymous and identified) via the Unload Positive Bottle function on the Main screen.
- b) From the Main screen, press **Ctrl + F10**. Cells with bottles are lit continuously.
- c) Pull bottles out one at a time and immediately reload one at a time in an incubator without the 39 fault.

Note - When relocating bottles using **Ctrl + F10**, you must unload one bottle from a cell and reload it into the destination cell **BEFORE** unloading another bottle. This procedure must be done one bottle at a time.

- d) If the bottle is anonymous (the **Bottle ID** field is blank), then enter the bottle ID ([see Text/Data Entry in Chapter 3](#)).
 - e) When all bottles have been relocated, press the **Check** button on the Main screen.
 - f) Call bioMérieux.
-

#41 - RACK(S) X NOT RESPONDING**47**

Cause: Incubation Module is disconnected. The number of the Incubation Module is displayed in the **Instrument** icon on the Main screen.

Solution:

1. Reconnect each end of the communication cable.
2. Ensure that the proper communications port of the Controller or Combination Module (1–6) is connected to the corresponding Incubation Module.

Cause: Faulty drawer interface cable or printed circuit board.

Solution: Relocate bottles manually (see [Figure 5-3, Edit Bottle Detail Screen](#) for Clinical Use, or [Figure 6-3, Edit Bottle Detail Screen](#) for INDUSTRY Use). The Edit Load Status Slidebar switch will be bolded and available to manually unload all affected bottles. Identify any anonymous bottles prior to manually unloading. Reload bottles into functional cells with the Load Bottles button on the Main Screen. Call bioMérieux.

Cause: Faulty rack controller or rack interface cable(s).

Solution: Relocate bottles manually (see [Figure 5-3, Edit Bottle Detail Screen](#) for Clinical Use, or [Figure 6-3, Edit Bottle Detail Screen](#) for INDUSTRY Use). The Edit Load Status Slidebar switch will be bolded and available to manually unload all affected bottles. Identify any anonymous bottles prior to manually unloading. Reload bottles into functional cells with the Load Bottles button on the Main Screen. Call bioMérieux.

#51 - RACK HARDWARE FAILURE**57**

Cause: Faulty rack printed circuit board.

Solution: From the Main screen, press **Ctrl + F10** to enter a special Bottle Relocation mode that allows bottles to be relocated, then call bioMérieux. The quicker the bottles are moved, the less likely they'll need to be subcultured.



CAUTION – When Unloading multiple bottles using Ctrl + F10, the bottles must be reloaded before 1 hour has passed using the normal Load Bottles icon to avoid a subculture event.

Note - When relocating bottles using **Ctrl + F10**, you must unload one bottle from a cell and reload it into the destination cell **BEFORE** unloading another bottle. This procedure must be done one bottle at a time.

#60 CELL FAILED QC

Cause: A cell failed the automatic internal diagnostic check.

Solution: Perform the following procedure:

1. Identify the cell that failed the QC using the View Cell Status screen.
2. Inspect the cell for debris and remove or clean, if necessary.
3. Calibrate ([see Calibrating an Instrument Cell in Chapter 8](#)).
4. If calibration cannot be performed at this time, or if a cell fails to calibrate, disable cell ([see Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#)), and insert an orange cell plug in the disabled cell.

Call bioMérieux if cell calibration is unsuccessful.

#62 ERRONEOUS CELL LOAD STATUS REPORTED

Cause: A cell is out of calibration.

Solution: Use the View Cell Status screen (see Chapter 3, Viewing Faults) to locate affected cells. Relocate bottle in affected cells using the following procedure:

1. From the Main screen, press **Ctrl + F10** to enter a special Bottle Relocation mode that allows bottles to be relocated. The cell indicator lights will illuminate in the following pattern:



CAUTION – After Unloading multiple bottles using Ctrl + F10, the bottles must be reloaded before 1 hour has passed using the normal Load Bottles icon to avoid a subculture event.

- Cells with bottles are lit continuously.
 - Cells available for reloading are unlit.
2. Pull bottles out one at a time and reload one at a time into any available cell.

Note - When relocating bottles using **Ctrl + F10**, you must unload one bottle from a cell and reload it into the destination cell **BEFORE** unloading another bottle. This procedure must be done one bottle at a time.

3. If the bottle is anonymous (the **Bottle ID** field is blank), then enter the *bottle ID* (see [Text/Data Entry in Chapter 3](#) for Clinical Use, or [Text/Data Entry in Chapter 4](#) for INDUSTRY Use). When all bottles are unloaded, press the **Check** button on the Main screen.
4. Attempt to calibrate the cell (see [Calibrating an Instrument Cell in Chapter 8](#)).
5. If the calibration fails, then disable the cell (see [Enabling and Disabling Modules, Drawers, Racks, and Cells in Chapter 8](#)).
 - Insert an orange cell plug into the disabled cell.
 - Call bioMérieux.

Note - Call bioMérieux if error occurs with numerous cells (it could be a faulty rack controller.).

#71 INVALID TIME STAMP

Cause: Hardware failure involving the real time clock.



CAUTION – Bottle readings are not processed while the fault is present.

Solution: Call bioMérieux.

#72 INVALID CALCULATION DATA

Cause: Hardware failure involving the non-volatile random access memory.



CAUTION – Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Determine how long bottle readings have not been processed by going to the Edit Bottle Detail screen and noting the time of the last bottle reading (see [Viewing/Editing Bottle Data in Chapter 5](#) for Clinical Use, or [Viewing/Editing Bottle Data in Chapter 6](#) for INDUSTRY Use). If the time passed is greater than 1 hour, bottles must be subcultured.

1. Unload the affected bottle via the Unload Positive Bottle function on the Main screen.
 2. Subculture the bottle if necessary.
 3. Reload the bottle via the Load Bottle function on the Main screen. The original first loaded date and time is maintained.
 4. If the fault code persists, call bioMérieux.
-

#73 INVALID METHOD DATA

Cause: Important information used for bottle analysis has been corrupted and therefore bottle readings are not being processed. Bottles with fault are flagged positive 1 hour after the fault code first appears. Bottle readings are not processed while the fault is present.

Solution: Each bottle type uses the same method data. Eventually, all bottles of the same type as the one with the fault will also present the same fault code.

1. Reboot the system. For Select and SelectLink configurations, see [Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\)](#) in Chapter 8. For Signature configuration, see [Full System Shutdown \(BacT/ALERT® 3D Signature\)](#) and [Full System Startup \(BacT/ALERT® 3D Signature\)](#) in Chapter 8.
2. Call bioMérieux.

#74 ALGORITHM CHANGE (CLINICAL USE)

Cause: An incorrect media type has been previously assigned to a bottle, or an anonymous bottle has been placed in an inappropriate drawer:

- An MB bottle has been placed anonymously into a non-MB drawer.
- A non-MB bottle has been placed anonymously into an MB drawer.
- A BacT/ALERT® MB bottle has been loaded anonymously in any drawer type.

Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Perform the following procedure:

1. Press the **Unload Positive Bottle** button and unload the affected bottle.
 2. Subculture the bottle.
 3. Reload the bottle via the Load Bottle function on the Main screen. The original first loaded date and time are maintained.
 4. If the fault code persists, call bioMérieux.
-

#74 ALGORITHM CHANGE (INDUSTRY USE)

Cause: An incorrect media type has been previously assigned to a bottle. Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Perform the following procedure:

1. Press the **Unload Positive Bottle** button and unload the affected bottle.
2. Subculture the bottle.
3. Reload the bottle via the Load Bottle function on the Main screen. The original first loaded date and time are maintained.
4. If the fault code persists, call bioMérieux.

#75 BOTTLE DATA CORRUPTED

Cause: The data integrity check has found corrupted bottle data. Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Perform the following procedure:

1. Press the **Unload Positive Bottle** button and unload the affected bottle.
 2. Subculture the bottle.
 3. Reload the bottle via the Load Bottle function on the Main screen to begin testing of bottle.
 - The original first loaded date and time is replaced with the date and time the bottle is reloaded.
 - The maximum test time is set to the default associated with the bottle type.
 4. If desired, the bottle's maximum test time may be shortened so that the bottle will complete testing when expected.
 5. If the fault code persists, call bioMérieux.
-

#77 READING POLYNOMIAL CHANGE

Cause: An incorrect bottle type has been previously assigned to a bottle. Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Perform the following procedure:

1. Press the **Unload Positive Bottle** button and unload the affected bottle.
2. Subculture the bottle.
3. Reload the bottle via the Load Bottle function on the Main screen to restart testing. The original first loaded date and time are maintained.
4. If the fault code persists, call bioMérieux.

#78 INCOMPLETE TEST DATA

Cause: This fault occurs when an identified bottle is reloaded anonymously and later is identified and reloaded in the instrument. The original bottle data cannot be merged with the data collected when the bottle was anonymous and therefore the test data is incomplete. Bottles with fault are flagged positive and bottle readings are not processed while the fault is present.

Solution: Perform the following procedure:

1. Unload bottle with fault via the Unload Positive Bottle function on the Main screen.
 2. Subculture the bottle.
 3. Reload the bottle via the Load Bottle function on the Main screen to restart testing. The original first loaded date and time are maintained.
 4. If the fault code persists, call bioMérieux.
-

#79 BOTTLE RELOADED VIA WRONG CONTROLLER (BACT/ALERT® 3D SIGNATURE SYSTEMS)

Cause: This fault occurs when a bottle is initially loaded in an Incubation Module attached to a particular Controller Module and then is reloaded into another Incubation Module attached to a different Controller Module. Bottles with fault are flagged positive.

Solution: Perform the following procedure:

1. Unload bottle with the fault via the Unload Positive Bottle function on the Main screen.
 2. Subculture the bottle.
 3. Apply a generic bottle label to the bottle.
 4. Press the **Load Bottle** button on the Main screen.
 5. Scan the generic label.
 6. Select the correct bottle type.
 7. If desired, enter an Accession Number/Sample ID and adjust the bottle's maximum test time so the bottle will complete testing when expected.
 8. Load the bottle. The date and time first loaded will reflect this load.
 9. If the fault code persists, call bioMérieux.
-

#80 READING GAP DETECTED

Cause: This fault occurs when there is a gap in bottle readings greater than 1 hour. This situation may occur when the power to the Instrument or Incubation Module is lost for more than 1 hour. It may also occur if a negative-to-date bottle is unloaded, then reloaded more than 1 hour later.

There are two procedures that can be followed when this fault occurs. It is recommended that you print an Unload report before clearing this fault to ensure that you have a record of all bottles with an 80 fault.

Note - On the Load, Status and Unload reports, bottles with Instrument Fault Code 80 will be marked on the report with a **Gap Detection** indicator (!) next to the result. If a negative bottle has an Instrument Fault Code 80 while it is unloaded, the **Gap Detection** indicator will remain on the report (see [Viewing, Printing, and Exporting Test Data in Chapter 3](#) for Clinical Use, or [Viewing, Printing, and Exporting Test Data in Chapter 4](#) for INDUSTRY Use).

Solution 1: This procedure instructs you to unload and subculture each bottle to clear the 80 fault:

1. From the Main screen, press **Ctrl + F3** to enter a special unload mode.
2. Subculture the bottle.
3. Reload the bottle via the Load Bottle function on the Main screen to restart testing.

Solution 2: This procedure allows you to clear 80 faults for each affected bottle without unloading.



WARNING – These bottles must have a terminal subculture performed if their final status is negative.

1. From the Setup screen, enter the system password (see [Accessing the Setup Screen in Chapter 3](#) for Clinical Use, or [Accessing the Setup Screen in Chapter 4](#) for INDUSTRY Use).
 2. Press **Alt + F5** to clear all faults at the same time.
-

#81 BOTTLE LOADED IN CELL DURING QC

Cause: This fault occurs when a bottle is loaded in a cell that has not completed the QC cycle. In this case, the fault is displayed with no other fault on the View Cell Status screen. The QC cycle for a cell is indicated by a white circle on the cell status screen (see [Figure 3-5, View Cell Status Screen for Clinical Use](#), or [Figure 4-5, View Cell Status Screen for INDUSTRY Use](#)). Bottle readings are taken while the fault is present.

Solution: Perform the following procedure:

1. Remove the bottle from the cell that was just loaded.
2. Reload the bottle into an available cell (one which has its cell indicator light lit).

Cause: This fault occurs when a bottle is loaded in a cell that is out of calibration (indicated by Fault Code 60). In this case, Fault Codes 60 and 81 are both displayed on the View Cell Status screen, alternating between the two faults (see [Figure 3-5, View Cell Status Screen](#) for Clinical Use, or [Figure 4-5, View Cell Status Screen](#) for INDUSTRY Use). Bottle readings are not taken while Fault Codes 60 and 81 are present at the same time.

Solution 1: Perform the following procedure if this fault is recognized within one hour of loading a bottle.

Note - To check the time the bottle was loaded, access the Edit Bottle Detail screen (see [Figure 5-3, Edit Bottle Detail Screen](#) for Clinical use, or [Figure 6-3, Edit Bottle Detail Screen](#) for INDUSTRY Use).

1. Remove the bottle from the cell that was just loaded.
2. Reload the bottle into an available cell (one which has its cell indicator light lit).

Solution 2: Perform the following procedure if this fault is recognized more than one hour after loading a bottle, or the time the bottle was loaded cannot be determined on the Edit Bottle Detail screen.

1. Access the View Cell Status screen (see [Figure 3-5, View Cell Status Screen](#) for Clinical Use, or [Figure 4-5, View Cell Status Screen](#) for INDUSTRY Use) to view the cell error.
2. If the cell displays Fault Code 81 and does not alternate between Fault Codes 81 and 60, readings have been taken on this bottle. Unload the bottle from the cell and reload the bottle into a functional cell using the Load Bottle function (see [Figure 3-7, Main Screen — Load Mode](#) for Clinical use, or [Figure 4-7, Main Screen — Load Mode](#) for INDUSTRY Use).
3. If the cell displays an error code that alternates between Fault Codes 81 and 60, then readings have not been taken on this bottle. Unload the bottle and perform a subculture. Reload the bottle into a functional cell using the Load Bottle function (see [Figure 3-7, Main Screen — Load Mode](#) for Clinical use, or [Figure 4-7, Main Screen — Load Mode](#) for INDUSTRY Use).



WARNING – The bottle must be subcultured.

#99 BOTTLE LOADED IN A NON-OPERATIONAL CELL

Cause: A bottle is currently loaded in either:

- A cell that has been disabled, or the rack in which the cell is located is disabled.
- An enabled cell that is not taking bottle readings because of one of the following faults: 51, 52, 54, 60, or 62.

Bottle readings are not processed in faulty cells.

Solution: Use the Edit Bottle Detail screen to determine when the last bottle reading was successfully processed ([see Viewing/Editing Bottle Data in Chapter 5](#) for Clinical Use, or [Viewing/Editing Bottle Data in Chapter 6](#) for INDUSTRY Use). If less than 1 hour has passed, it is only necessary to relocate the bottles using the following procedure:

1. From the Main screen, press **Ctrl + F1**. This mode identifies cells containing bottles with 99, 51, 52, 54, 60, or 62 faults with a lit cell indicator light.
2. Remove bottles from these cells one at a time, move them to cells whose cell indicator lights are unlit. Bottles can be moved to a different drawer (of the same configuration) as necessary.

Note - If no bottle(s) are present in the indicated cell, call *bioMérieux Customer Support*.

If more than an hour has passed, unload and subculture the bottle in the faulty cells using the following procedure:

1. From the Main screen, press **Ctrl + F10**.
2. Remove a bottle to subculture and scan the bottle ID. Repeat this step for the remaining bottles needing subcultures.
3. Press the **Check** button when done.
4. After subculturing bottles, reload them via the Load Bottle function on the Main screen.

Affected bottles that do not need to be subcultured should be relocated. Relocate bottles.

Note - If no bottle(s) are present in the indicated cell, call *bioMérieux*.

Instrument Status Codes

Instrument Status Codes appear in a separate window on the Main screen and are identified by a 98 in the top left corner. A description of each Instrument Status Code is provided. The following is an example of an Instrument Status Code 710.

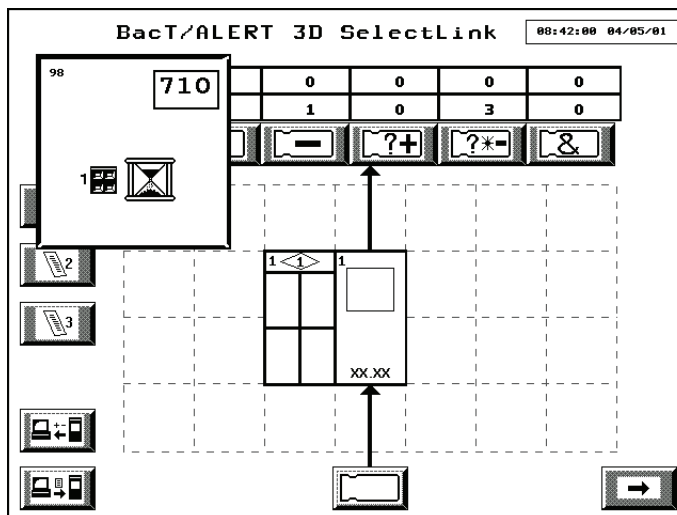


Figure 9-2: Instrument Status Code 710

#710 INCUBATION MODULE TEMPORARILY UNAVAILABLE WHILE POWER UP INITIALIZATION TAKES PLACE

Cause: Turning On the power of an Instrument or pressing the **Restart Incubation Module** button invokes the Power Up screen to appear on the operator panel. It is NOT possible to perform any action when the Power Up screen is present.

Solution: No operator action is necessary. The Power Up screen will disappear when the Incubation Module is ready. Do NOT load bottles while the Power Up screen is present.

#800 PRINTER FAULT

Cause: The Controller or Combination Module was unable to print to the printer.

Solution: Check the following:

- The printer is powered on and connected to the Instrument properly.
 - The printer has paper.
 - The paper is not jammed in the printer.
-

#810 BACKUP FAILED

Cause: The Controller or Combination Module was unable to write backup data to the backup media, because the backup media was either missing, defective, or removed while the backup was still in progress.

Solution 1: Perform the following procedure if the backup media is a Zip® drive:

1. Verify that there is a Zip® disk in the Zip® drive and attempt a manual backup (see [Initiating Manual Backup in Chapter 7](#)).
2. If this backup fails, insert a new Zip® disk and attempt a manual backup.
3. If this backup fails, call bioMérieux.

Solution 2: Perform the following procedure if the backup media is a USB flash drive:

1. Verify that there is a USB flash drive in the USB port and attempt a manual backup (see [Initiating Manual Backup in Chapter 7](#)).
2. If this backup fails, remove the USB flash drive and verify that it is an approved bioMérieux USB flash drive. If it is an approved bioMérieux USB flash drive, re-insert and attempt a manual backup.

or

If the USB flash drive is not an approved bioMérieux USB flash drive, then insert a USB flash drive approved by bioMérieux and attempt a manual backup.

3. If this backup fails, call bioMérieux.
-

#820 **BACKUP MEDIA MISSING OR BACKUP DRIVE FAULT**

Cause: Backup media is not fully inserted into the drive.

Solution: Remove backup media and re-insert into the drive.

Cause: Backup media is corrupted.

Solution: Remove the backup media and replace with a different backup media.

Cause: Backup drive failure.

Solution: Call bioMérieux.

Cause: Backup media is not an approved bioMérieux device.

Solution: Insert an approved bioMérieux device.

Operator Error Codes

Operator Error Codes appear in a separate window on the Main screen and are identified by a 99 in the top left corner. A description of each Operator Error Code is provided. The following is an example of an Operator Error Code 911.

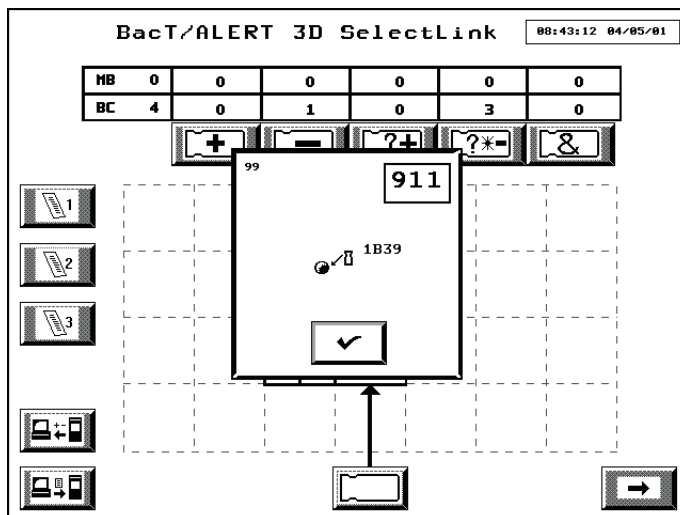


Figure 9-3: Operator Error Code 911

#901 CORRUPTED BOTTLE RECORD HAS BEEN SELECTED

Cause: A corrupted bottle record has been selected for the Edit Bottle Detail screen.

Solution: This error will automatically disappear.

1. Note the cell ID for the corrupted bottle record on the Edit Bottle Detail screen.
2. Go to the View Cell Status screen and check the cell for Instrument Fault Code 75.
3. If Instrument Fault 75 is present, then follow the procedure provided to correct the problem ([see Instrument Fault Code #75 on page 9-17](#)).
4. If error persists, call bioMérieux.

#902 REQUIRED DATA MISSING

Cause: Required data was not entered while loading bottles.

Solution:

1. Remove the bottle from the cell that was just loaded.
2. Complete required fields (see [Text/Data Entry in Chapter 3](#) for Clinical Use, or [Text/Data Entry in Chapter 4](#) for INDUSTRY Use).
3. Reload bottle.

#909 INVALID CELL LOAD — CELL QC IN PROCESS

Cause: This fault occurs when a bottle is loaded in a cell that has not completed the QC cycle. The QC cycle for a cell is indicated by a white circle on the cell status screen (see [Figure 3-5, View Cell Status Screen](#) for Clinical Use, or see [Figure 4-5, View Cell Status Screen](#) for INDUSTRY Use).

Note - *The maximum time a cell may be disabled while undergoing the QC cycle is 12 minutes.*

Solution: Perform the following procedure:

1. Remove the bottle from the cell that was just loaded.
The error will disappear.
 2. Scan the Bottle ID again and Accession Number (Clinical) or Sample ID (INDUSTRY) (if needed).
 3. Reload the bottle into an available cell (one which has its cell indicator light lit).
-

#910 INVALID CELL LOAD

Cause: Indicates that a bottle was loaded in a cell belonging to a wrong drawer type, or in a cell that is disabled or non-functional. This error also occurs if a bottle is loaded while an Unload function is active.

Solution: Perform the following procedure:

1. Remove the bottle from the cell that was just loaded. The cell ID will be displayed in the error window on the Controller or Combination Module.
2. If in the Load function, re-scan the bottle ID and load the bottle in a cell which has its cell indicator light lit. Removing the indicated bottle also causes the Operator Error screen to vanish.

DO NOT press the **Check** button on the Operator Error screen except to intentionally ignore the error and leave the bottle loaded.

Loading another bottle will clear the error and leave the original bottle loaded in the non-functional cell.

#911 INVALID CELL UNLOAD

Cause: Indicates an incorrect bottle was unloaded from a non-indicated cell or that a bottle was unloaded while not in an Unload function.

Solution: Replace the bottle into the cell from which it was just unloaded. The cell ID will be displayed in the error window on the Controller or Combination Module. The cell will also have a rapidly flashing cell indicator light. Reloading the indicated bottle also causes the Operator Error screen to vanish.

Press the **Check** button ONLY if the unload was intentional.

#912 INVALID CELL RELOAD

Cause: When reloading a bottle, the bottle was placed in a disabled or non-functional cell.

Solution: Perform the following procedure:

1. Remove the bottle from the non-functional cell whose cell indicator light will be flashing rapidly. The cell ID will also be displayed in the error window on the Controller or Combination Module.
2. Reload the bottle into a functional cell. Moving the indicated bottle to another cell causes the Operator Error screen to vanish.

DO NOT press the **Check** button on the Operator Error screen except to intentionally ignore the error and leave the bottle loaded in the non-functional cell.

Ignoring the error leaves the original bottle in a non-functional cell.

#913 BOTTLE ID JUST ENTERED MATCHES THAT OF A CURRENTLY LOADED BOTTLE

Note - *The bottle ID just entered is displayed at the top of the Operator Error screen. The cell location of a bottle loaded with the same ID is displayed below the bottle ID.*

Cause: The bottle ID is incorrect for the bottle being loaded or unloaded.

Solution: Rescan the bottle and press the **Check** button after it turns red.

Cause: If the bottle ID is correct for the bottle being loaded or unloaded, the bottle ID for the bottle in the cell location indicated was incorrect.

Solution: Perform the following procedure:

1. Record the cell location displayed in the Error Box.
 2. If the error occurred while loading, then do not load the bottle at this time. Set the bottle aside and go to [Step 4](#).
 3. If the error occurred while unloading an anonymous bottle, return the bottle to its original cell location.
 4. Pull the bottle from the cell recorded in [Step 1](#) and record its bottle ID. (Ignore the Operator Error 911 that appears. DO NOT press the **Check** button.)
 5. Return the above bottle to the cell from which it came (the LED will be flashing).
 6. Go to the Edit Cell Contents screen and select the cell recorded in [Step 1](#) (see [Selecting Bottles Using the Edit Cell Contents Button in Chapter 5](#) for Clinical Use, or [Selecting Bottles Using the Edit Cell Contents Button in Chapter 6](#) for INDUSTRY Use). The Edit Bottle Detail screen appears.
 7. Enter the correct *bottle ID* (the one recorded in [Step 4](#)) in the **Bottle ID** field.
 8. Press the **Check** button.
 9. If the error occurred while identifying an anonymous bottle, identify the anonymous bottle.
- or
- If the error occurred while loading a bottle (see [Step 2](#)), re-load the bottle from [Step 2](#).
10. If the error persists, call bioMérieux.

#921 BARCODE SCAN NOT ALLOWED IN THIS MODE

Cause: Indicates a barcode label was scanned but current screen does not accept barcodes. Frequently occurs if appropriate function is not initiated, such as load or unload bottles.

Solution: No action is required to correct this error. It automatically resolves after 2 seconds.

#923 INVALID BARCODE ENTRY

Cause: The scanned barcode cannot be accepted because it is the wrong length, has the wrong starting character, or has an invalid character.

- Unacceptable Bottle ID characters are ' , " , \ , * .
- Unacceptable Accession Number/Sample ID characters are ' , " , \ , * , @ .

Solution: Acknowledge error and re-scan bottle. If error persists after re-scanning the barcode, attempt to enter the *Accession Number/Sample ID* or *bottle ID* via the keyboard.

Ensure that barcode is entered into the correct field.

If error persists, use a Generic Bottle ID barcode.

#930 INCORRECT OR UNEXPECTED BARCODE LABEL
SCANNED WHILE UNLOADING A BOTTLE

Cause: A previously identified bottle is unloaded, but the wrong label was scanned while unloading the bottle. The instrument determines that it is not the same barcode currently associated with the bottle and reports this error.

Note - *Two bottle IDs are displayed in a box on the Operator Error screen. At the top is the bottle ID of the bottle just unloaded and scanned. The second bottle ID displayed was assigned to the bottle when it was originally loaded or identified.*

Solution: If an error was made during the scanning of the bottle, rescan the bottle and press the **Check** button after it turns red.

Cause: The correct barcode label is scanned after unloading the bottle; apparently, the incorrect label was scanned when the bottle was originally loaded.

Solution: Perform the following procedure:

1. If the unloaded bottle was scanned correctly, DO NOT TOUCH the screen.
2. Record the bottle ID and the cell ID, and return the bottle to the cell from which it came.
3. Go to the Select Bottle to Edit screen and use the scroll buttons to enter the cell ID recorded.
4. Press the **Check** button to display this bottle's information on the Edit Bottle Detail screen. Edit the bottle ID to match the bottle ID recorded and press the **Check** button.
5. Return to the Unload function to unload the original bottle.
6. If the error persists, call bioMérieux.

#931 DETECTION ALGORITHM HAS BEEN CHANGED

Cause: An anonymous or identified bottle is unloaded and its barcode is scanned. The instrument determines that the media type associated with the barcode does not match the algorithm used in analyzing the initial data.

Solution: Perform the following procedure:

1. If the bottle has multiple labels, it may be that the wrong label was scanned. Try re-scanning the bottle ID and if correctly entered, press the **Check** button.
2. If no changes are made and the **Check** button is pressed, the bottle will be flagged positive and Instrument Fault Code 74 or 77 will appear in the cell where the bottle was loaded.
3. Unload the bottle with the fault via the Positive Bottle Unload function.
4. Subculture the bottle and reload the bottle to restart testing. See [Instrument Fault Code #74 on page 9-16](#) for Clinical Use (or INDUSTRY Use) or [see Instrument Fault Code #77 on page 9-18](#).

Note - Unload another bottle or press the **Cancel** button to ignore the error.



WARNING – The bottle must be subcultured.

Cause: An anonymous bottle is unloaded with a generic barcode label. The operator uses the **Bottle Type** scroll button to specify the media type. The error is reported while selecting the bottle type.

Solution: Perform the following procedure:

1. Note that the Operator Error screen also has a **Bottle Type** scroll button. If an incorrect media type was entered, correct it now and press the **Check** button.
2. See instructions above if no changes are made and the **Check** button is pressed.

Note - *Unload another bottle or press the **Cancel** button to ignore the error.*



WARNING – The bottle must be subcultured.

#932 BOTTLE ID SCANNED MATCHES THAT OF A PREVIOUSLY UNLOADED BOTTLE

Cause: An anonymous bottle is identified with a previously unloaded bottle's ID.

Solution: If the bottle ID was entered incorrectly, enter the correct *bottle ID* and press the **Check** button.

If the bottle ID was entered correctly, perform the following procedure:

1. Re-enter the *bottle ID* and press the **Check** button.

Note - *The bottle readings for the previously unloaded bottle will be lost and the Bottle ID, Accession Number/Sample ID, First Loaded Time, Maximum Test Time, and Bottle Type will be assigned to the newly identified bottle.*

2. Reload the bottle, if desired. Instrument Fault Code 78 appears and the bottle is flagged positive ([see Instrument Fault Code #78 on page 9-18](#)).



CAUTION – Performing Step 1 and Step 2 result in duplicate bottle ID's that will be seen in Editing Bottle ID to Accession Number relationships and on the reports in the BacT/ALERT® 3D Select and SelectLink configurations.

3. If the bottle is not reloaded (and another bottle is unloaded or the **Cancel** button is pressed), the bottle's anonymous record will be lost.

If the bottle is returned to its cell without identifying the bottle, the bottle will remain anonymous and Operator Error Code 932 will disappear.

#940 BOTTLE DETAIL NOT FOUND

Cause: Occurs only in the Select Bottle to Edit/Graph screen when:

- An empty cell is selected.
- The bottle ID entered is not recognized as belonging to a currently loaded or recently unloaded bottle.

Solution: This Operator Error Code will terminate spontaneously after being briefly displayed. When the Select Bottle to Edit/Graph screen is redisplayed, correct the previous entry or press the **Cancel** button.

Cause: This error may occur while in the Edit Cell Contents screen when a loaded cell is selected, but no bottle record was found.

Solution: Call bioMérieux.

#941 BOTTLE ID ENTERED MATCHES THAT OF A BOTTLE THAT IS CURRENTLY LOADED IN A DIFFERENT CELL

Cause: When editing a bottle ID on the Edit Bottle Detail screen, the bottle ID entered is the same as the bottle ID of a bottle loaded in a different cell. Press the **Cancel** button to clear the error.

Solution: If the bottle ID was entered incorrectly, enter the correct *bottle ID* now and press the **Check** button.

If the bottle ID is correctly entered, perform the following procedure.

1. Press the **Cancel** button on the Edit Bottle Detail screen.
2. Enter the same *bottle ID* (the one used on the previous screen) and press the **Check** button to view information on this bottle. Note the cell location on the Edit Bottle Detail screen and remove the bottle currently in that cell to record the actual bottle ID. Ignore the Operator Error 911 that appears on the screen when the bottle is unloaded.



CAUTION – DO NOT press the Check button.

3. Return the bottle to the same cell from which it was removed.
 4. Correct the bottle ID and press the **Check** button. It is now possible to edit or identify the original bottle ID.
-

#942 BOTTLE TYPE CHANGE REQUIRES DIFFERENT ALGORITHM

Cause: In the Edit Bottle Detail screen, a bottle's media type has been changed to a media type that requires different data analysis.

Solution: If an error was made entering the new media, perform the following procedure:

1. Press the **Cancel** button and return to the Edit Bottle Detail screen to correct the entry.
2. Press the **Check** button on the Edit Bottle Detail screen to save the changes.

If an error was not made, perform the following procedure:

1. Press the **Check** button to override the error. Note the cell ID of the bottle.
2. Press the **Check** button on the Edit Bottle Detail screen to save changes.
3. Go to the View Cell Status screen and check the cell for Instrument Fault Code 74.
4. If the fault is present, follow the procedure provided to correct the problem. See [Instrument Fault Code #74 on page 9-16](#) for Clinical Use (or INDUSTRY Use).

#943 IDENTIFIED BOTTLE CHANGED TO ANONYMOUS

Cause: The **Bottle ID** field has been cleared for an identified bottle in the Edit Bottle Detail screen.

Solution: Press **Cancel** to return to the Edit Bottle Detail Screen. Enter the correct *bottle ID* and press the **Check** button.

#944 MANUAL UNLOAD OF ANONYMOUS BOTTLE

Cause: An attempt was made to manually unload an anonymous bottle from the Edit Bottle Detail screen.

Solution: Press the **Cancel** button to return to the Edit Bottle Detail screen. Enter a *bottle ID* or change the **Load Status** sidebar switch back to the load position (1).

#945 BOTTLE ID ENTERED MATCHES THAT OF A PREVIOUSLY UNLOADED BOTTLE

Cause: In the Edit Bottle Detail screen, a bottle has been given the bottle ID of a previously unloaded bottle.

Solution: If the bottle ID was entered incorrectly, perform the following procedure:

1. Press the **Cancel** button on the error screen.
2. Enter the correct *bottle ID* on the Edit Bottle Detail screen and press the **Check** button.

If the Bottle ID was entered correctly, perform the following procedure:

1. Press the **Check** button on the error screen.

Note - *The bottle readings for the previously unloaded bottle will be lost and the Bottle ID, Accession Number/Sample ID, First Loaded Time, Maximum Test Time, and Bottle Type will be assigned to the newly identified bottle.*

2. Press the **Check** button on the Edit Bottle Detail screen. Instrument Fault Code 78 appears and the bottle is flagged positive ([see Instrument Fault Code #78 on page 9-18](#)).

#960 DUPLICATE ENTRIES

Cause: Two or more label entry fields on the Label Entry screen contained the same entry when the **Check** button was pressed.

Solution: Change one of the duplicate labels to a unique entry and press the **Check** button.

#961 INVALID BLANK FIELD

Cause: One of the label entry fields on the Label Entry screen was blank when the **Check** button was pressed.

Solution: Enter a *label* in the blank field and press the **Check** button.

Bottle Problems

ANONYMOUS BOTTLE INDICATED IN EMPTY CELL

Cause: The cell flag is stuck.

Solution: Verify that the flag is covering the bottom of the cell. If not, try gently pressing on the flag spring to dislodge it

Cause: Flag is soiled.

Solution: Call bioMérieux.

Cell may need to be recalibrated. Calibrate cell ([see Calibrating an Instrument Cell in Chapter 8](#)).

BOTTLE LODGED IN CELL

Cause: Too many labels applied to a bottle.

Solution: Call bioMérieux.



CAUTION – Do not twist the bottle.

Cause: The bottle label was wet when the bottle was loaded and became stuck to the side of the cell.

Solution: Call bioMérieux.

User Output Device Problems

DRAWER INDICATOR(S) NEVER COME ON

Cause: Various.

Solution: Call bioMérieux.

CELL INDICATOR(S) NEVER COME ON

Cause: Various.

Solution: Call bioMérieux.

CONTROLLER OR COMBINATION MODULE ALARM NOT SOUNDING

Cause: Alarm disabled.

Solution: Verify the alarm settings are correct ([see Setting the Audible Alarms in Chapter 7](#)). If this does not correct the problem, call bioMérieux.

BARCODE SCANNER DOESN'T BEEP

Cause: Various.

Solution: Call bioMérieux.

BARCODE CANNOT BE SCANNED

Cause: Barcode scanning beam does not appear when a bottle is placed over the bottle template label in the Barcode Reader aperture.

Solution: Ensure the barcode scanner is fully depressed in the cradle so the trigger is fully engaged.

Cause: The barcode gun needs cleaning.

Solution: Wipe the face of the gun with a soft cloth and water.

Cause: The connectors are improperly plugged in.

Solution: The keyboard beneath the barcode reader aperture may be used to enter barcodes.

Cause: Scanner's Operational Parameters are no longer set.

Solution: Scan the applicable Reset Barcode (see [Figure 9-4](#) or [Figure 9-5](#)) to reset the Operational Parameters.



Figure 9-4: QuickScan® 6000 Reset Barcode



Figure 9-5: Honeywell Reset Barcode

KEYBOARD NOT RESPONDING

Cause: Various.

Solution: Reboot the system. For Select and SelectLink configurations, see [Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\)](#) in Chapter 8. For Signature configuration, see [Full System Shutdown \(BacT/ALERT® 3D Signature\)](#) and [Full System Startup \(BacT/ALERT® 3D Signature\)](#) in Chapter 8.

OPERATOR PANEL BLANK

Cause: Controller or Combination Module not receiving power.

Solution: Check the power switch and the power cord connection.

OPERATOR PANEL UNRESPONSIVE

Cause: Various.

Solution: Reboot the system. For Select and SelectLink configurations, see [Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\)](#) in Chapter 8. For Signature configuration, see [Full System Shutdown \(BacT/ALERT® 3D Signature\)](#), and [Full System Startup \(BacT/ALERT® 3D Signature\)](#) in Chapter 8. Call bioMérieux if the problem is not resolved.

Note - *The keyboard may be used instead of the Operator Panel to operate the instrument and to activate the load and unload functions (see [Using the Keyboard in Place of the Operator Panel](#) in Chapter 8).*

OPERATOR PANEL — NO DISPLAY IMAGE


Cause: The monitor switch on the back of the Controller/Combination Module is incorrectly set to **EXTERNAL**.

Solution: Check the monitor switch setting. If the monitor switch is set to **EXTERNAL**, perform the following procedure:

1. Set the monitor switch to **INTERNAL**.
 2. Reboot the system. For Select and SelectLink configurations, see [Controller/Combination Module Reboot/Shutdown \(BacT/ALERT® 3D Select and SelectLink\)](#) in Chapter 8. For Signature configuration, see [Full System Shutdown \(BacT/ALERT® 3D Signature\)](#), and [Full System Startup \(BacT/ALERT® 3D Signature\)](#) in Chapter 8. Call bioMérieux if the problem is not resolved.
-

RED INFORMATIONAL WARNING SCREEN APPEARS

Cause: A red Informational Warning screen indicates that the BIOS clock setting is incorrect.

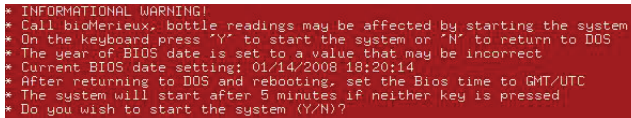

* INFORMATIONAL WARNING!
* Call bioMérieux; bottle readings may be affected by starting the system
* On the keyboard, press 'Y' to start the system or 'N' to return to DOS
* The year of BIOS date is set to a value that may be incorrect
* Current BIOS date setting: 01/14/2008 18:20:14
* After returning to DOS and rebooting, set the Bios time to GMT/UTC
* The system will start after 5 minutes if neither key is pressed
* Do you wish to start the system (Y/N)?

Figure 9-6: Red Informational Warning Screen

Solution 1: If the software was exited and restarted, but the system was not powered off, perform the following procedure:

1. Press **Y** on the keyboard to start the system.
2. Call your local bioMérieux representative immediately to report the incorrect BIOS setting.

Solution 2: If the system was powered off and the Red Information Warning screen (see [Figure 9-6](#)) appears when powered on, perform the following procedure:

1. Call your local bioMérieux representative immediately.
2. If a representative is unavailable, press **Y** on the keyboard to start the system.

All bottles will display Fault Code 72 and/or Fault Code 80 and need to be subcultured.



WARNING – These bottles must be subcultured.

About This Chapter

This chapter provides you with instructions on how to log in and out of the instrument while in 21 CFR Part 11 mode, as well as instructions for configuring users (ex. adding or deleting a user, or clearing a user password). This chapter also describes the events recorded in the audit trail and how to access the audit trail.

Chapter Contents

Log In/Out of System — 21 CFR Part 11 Mode • 10-2

Logging In for the First Time • 10-2

Logging In • 10-5

Inactivity Timeout • 10-7

Login Errors • 10-7

Change Password • 10-8

Change Password Errors • 10-9

Logging Out • 10-12

Configuring Users — 21 CFR Part 11 Mode • 10-12

Adding a User • 10-13

Deleting a User • 10-15

Clearing a User Password • 10-16

Audit Trail • 10-18

Accessing the Audit Trail • 10-20

Log In/Out of System — 21 CFR Part 11 Mode

When installed, the BacT/ALERT® 3D instrument can be configured to operate in 21 CFR Part 11 mode. If 21 CFR Part 11 mode is enabled, you need to enter a user name and password to access all functions available.

While logged out of the instrument in 21 CFR Part 11 mode, the following Main screen is displayed (see [Figure 10-1](#)).

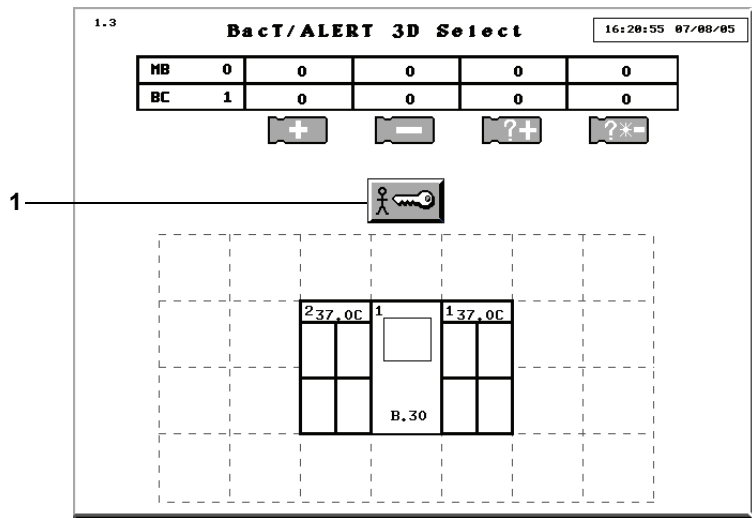


Figure 10-1: Main Screen While Logged Out — 21 CFR Part 11 Mode

1 — Login button

When logged out, you will have limited access to the instrument's functions. The following activities can be performed while logged out of the instrument:

- view error status
- view bottle count status
- hear and clear audible alarm
- view screen color changes when a bottle turns positive

Logging In for the First Time



When you log in to the instrument for the first time, the default password is *blank* (see [Adding a User on page 10-13](#)).

To log in to the instrument for the first time:

- 1) From the Main screen while logged out (see [Figure 10-1](#)), press the

Login button ().

The User Login screen appears.

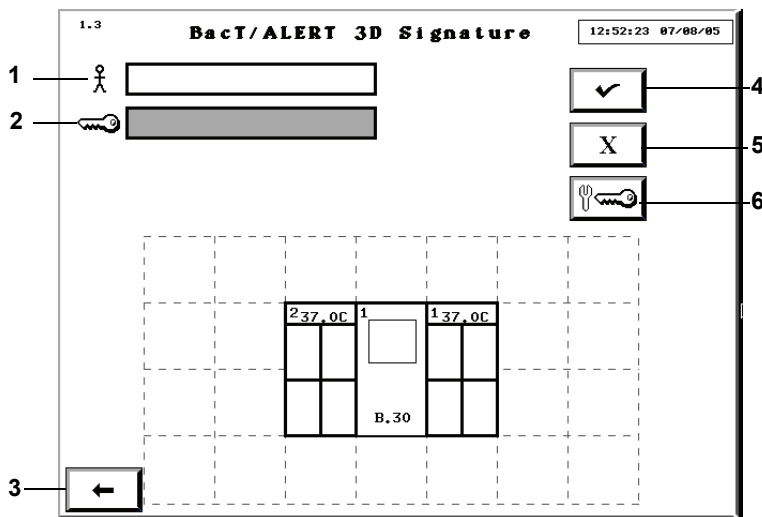


Figure 10-2: User Login Screen

- 1 — User Name Field
- 2 — Password Field
- 3 — Previous Screen Button
- 4 — Check Button
- 5 — Cancel Button
- 6 — Change Password Button

- 2) Touch the **User Name** field (field turns white) and enter your *user name*.
- 3) From the User Login screen (see [Figure 10-2, User Login Screen](#)), press

the **Change Password** button ().

The **New Password** and **Confirm Password** fields appear on the **User Login** Screen (see [Figure 10-3](#)).

Figure 10-3: User Login Screen with Change Password Fields

- 1 — New Password Field
- 2 — Confirm Password Field
- 3 — Previous Screen Button
- 4 — Check Button
- 5 — Change Password Button

- 4) Enter your new *password* in the **New Password** field (password length must be 6 to 24 characters).
- 5) Re-enter your new *password* in the **Confirm Password** field.
- 6) Press either the **Check** button or the **Enter** key to change your password.

The Main screen appears.

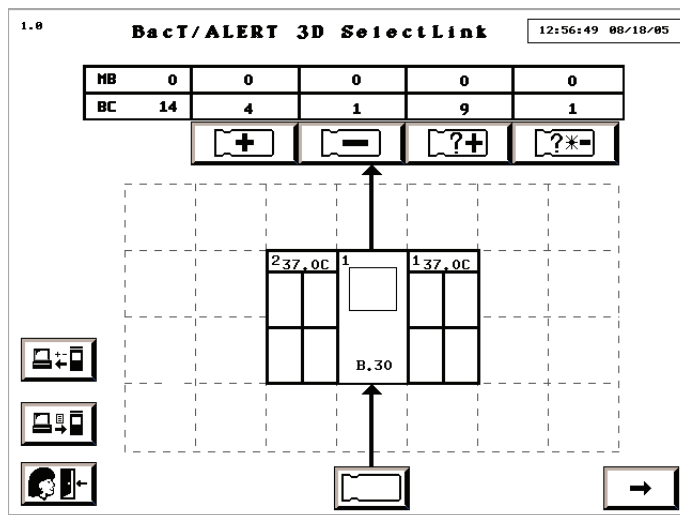


Figure 10-4: Main Screen While Logged In — 21 CFR Part 11 Mode

Note: The **Previous Screen** button returns to the User Login screen (see [Figure 10-5, User Login Screen, on page 10-6](#)) without logging in.

Logging In



When the instrument is configured for 21 CFR Part 11 mode, you will need to log in to the instrument to have full access of all functions.

- 1) From the Main screen while logged out (see [Figure 10-1](#)), press the

Login button ().

The User Login screen appears.

Figure 10-5: User Login Screen

- 1 — User Name Field
- 2 — Password Field
- 3 — Previous Screen Button
- 4 — Check Button
- 5 — Cancel Button
- 6 — Change Password Button

- 2) Click in the **User Name** field (field turns white) and enter your *user name*.
- 3) Press the **Tab** key on the keyboard to advance to the **Password** field (or click in the **Password** field).

The **Password** field turns white.

- 4) Enter your *password* in the **Password** field (password length must be 6 to 24 characters).
- 5) Press the **Check** button or the **Enter** key to log in to the instrument.

The Main screen appears (see [Figure 10-4, Main Screen While Logged In — 21 CFR Part 11 Mode, on page 10-5](#)).

Note: See [Configuring Users — 21 CFR Part 11 Mode on page 10-12](#) for detailed information on how to add and delete users, and clear user passwords.

Inactivity Timeout

While you are logged in, an inactivity timeout will occur (within a period of time configured by your bioMérieux Service Representative) if you do not perform one of the following actions:

- press a screen or keyboard button
- scan a barcode
- load or unload a bottle

If an inactivity timeout occurs, the instrument display reverts back to the Login screen (see [Figure 10-1, Main Screen While Logged Out — 21 CFR Part 11 Mode, on page 10-2](#)). Any pending function is cancelled as if the **Cancel** button on each successive screen was pressed.

Note: *If a timeout occurs, it is possible that partially entered information will be lost.*

Note: *The inactivity timeout feature is disabled while a red operator error is displayed.*

Login Errors

An alert displays on the User Login screen if you enter an unknown user name or an incorrect password and press the **Enter**, **Check**, or **Change Password** button (see [Figure 10-6](#)).

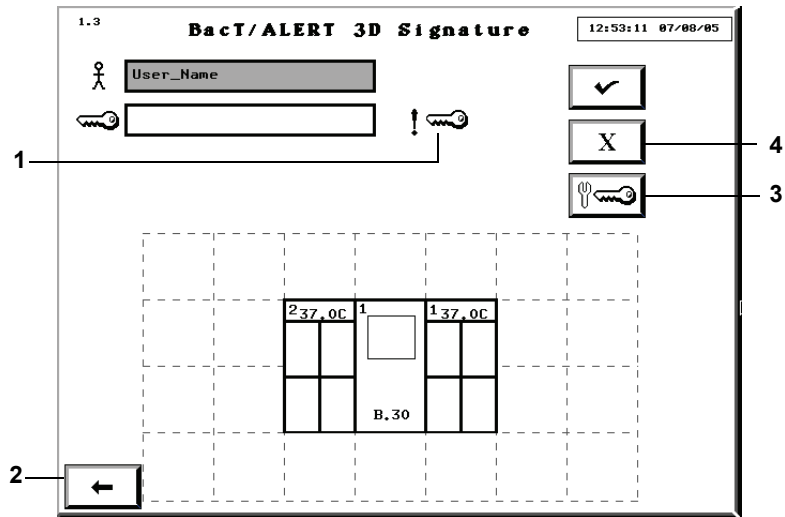


Figure 10-6: User Login Screen with Wrong Password Alert

- 1 — Wrong Password Alert
- 2 — Previous Screen Button
- 3 — Change Password Button
- 4 — Cancel Button

Note: A Wrong User Alert () appears to the right of the **User Name** field.

To clear the alert:

- 1) Press the **Cancel** button to clear the alert and all of the fields.
- 2) Re-enter your *user name* or *password* (see [Logging In on page 10-5](#)).


or

Press the **Previous Screen** button to return to the User Login screen without logging in (see [Figure 10-5, User Login Screen, on page 10-6](#)).

Change Password



Once you have entered a valid user name and password, you can change your password.

- 1) From the User Login screen (see [Figure 10-5, User Login Screen](#)), press the **Change Password** button ().

The **New Password** and **Confirm Password** fields appear on the **User Login Screen** (see [Figure 10-7](#)).

Figure 10-7: User Login Screen with Change Password Fields

- 1 — New Password Field
- 2 — Confirm Password Field
- 3 — Previous Screen Button
- 4 — Check Button
- 5 — Change Password Button

- 2) Enter your new *password* in the **New Password** field.
- 3) Re-enter your new *password* in the **Confirm Password** field.
- 4) Press either the **Check** button or the **Enter** key to change your password.

Note: The **Previous Screen** button returns to the User Login screen (see [Figure 10-5, User Login Screen, on page 10-6](#)) without logging in.

Change Password Errors



If you enter an incorrect new password, a change password error appears on the User Login screen (see [Figure 10-8](#)). If you enter a password with less than 6 characters, a wrong password alert appears on the User Login screen (see [Figure 10-9](#)).

Note: If more than 24 characters are entered, the additional characters are ignored and no error is displayed.

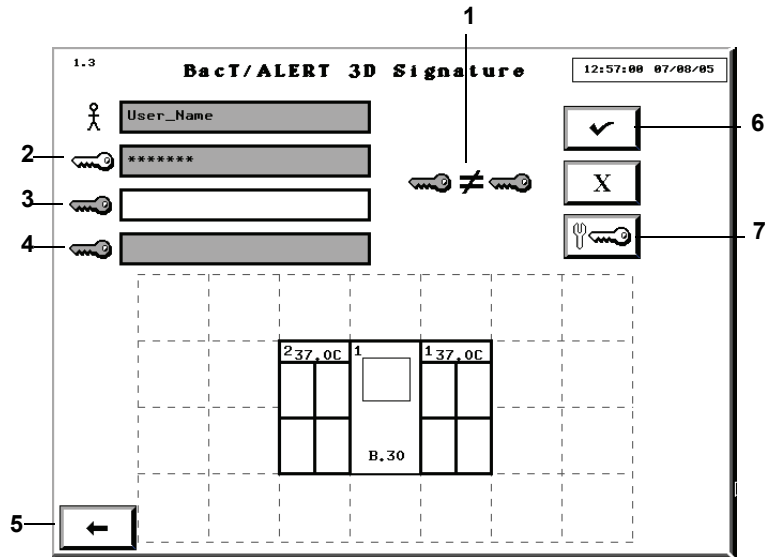


Figure 10-8: User Login Screen with Change Password Error

- 1 — Change Password Error
- 2 — Password Field
- 3 — New Password Field
- 4 — Confirm Password Field
- 5 — Previous Screen Button
- 6 — Check Button
- 7 — Change Password Button

Figure 10-9: User Login Screen with Wrong Password Alert

- 1 — Wrong Password Alert
- 2 — Password Field
- 3 — New Password Field
- 4 — Confirm Password Field
- 5 — Previous Screen Button
- 6 — Check Button
- 7 — Change Password Button

To clear the password error:

- 1) Press the **Password** field.
The **New Password** and **Confirm Password** fields will clear.
- 2) Enter your new *password* in the **New Password** field.
- 3) Re-enter your new *password* in the **Confirm Password** field.
- 4) Press the **Check** button or the **Enter** key to change your password.


Note: The **Previous Screen** button returns to the User Login screen (see [Figure 10-5, User Login Screen, on page 10-6](#)) without logging in.

Logging Out



- 1) From the Main screen (see [Figure 10-4, Main Screen While Logged In — 21 CFR Part 11 Mode, on page 10-5](#)), press the **Logout** button



() to log out of the instrument.

The Main screen (while logged out) appears (see [Figure 10-1, Main Screen While Logged Out — 21 CFR Part 11 Mode, on page 10-2](#)).

Note: *If you do not log out, the system will automatically time out when it reaches the inactivity timeout period (see [Inactivity Timeout on page 10-7](#)).*

Configuring Users — 21 CFR Part 11 Mode



The User Configuration screen allows you to manage user accounts for up to 100 users. You can add users, delete users, and clear user passwords.

- 1) Access the Setup Screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 4](#)).
- 2) From the Setup screen (see [Figure 4-10, Setup Screen](#)), press the

Configure Users button ().

The User Configuration screen appears.

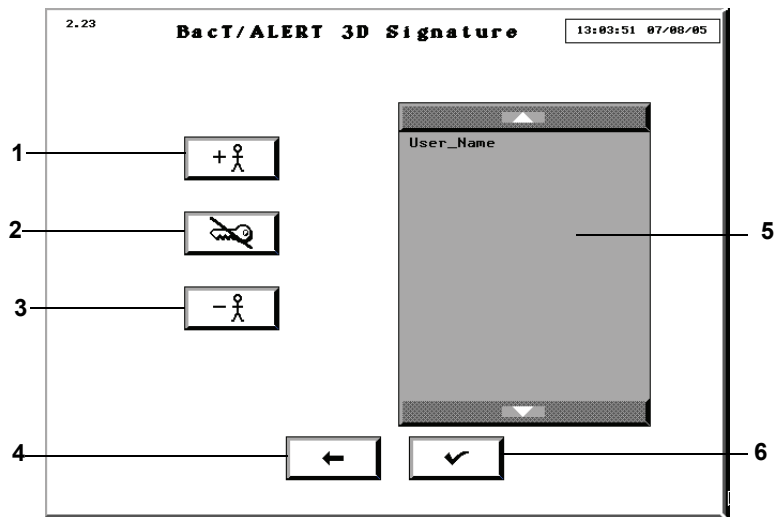


Figure 10-10: User Configuration Screen


- 1 — Add User Button
- 2 — Clear Password Button
- 3 — Delete User Button
- 4 — Previous Screen Button
- 5 — User List
- 6 — Check Button

Note: The **Configure Users** button is only active when the instrument is configured for 21 CFR Part 11 mode.

Adding a User

To add a user to the User List:

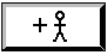

- 1) From the User Configuration screen (see [Figure 10-10, User Configuration Screen](#)), press the **Add User** button ().

The Add User screen appears.

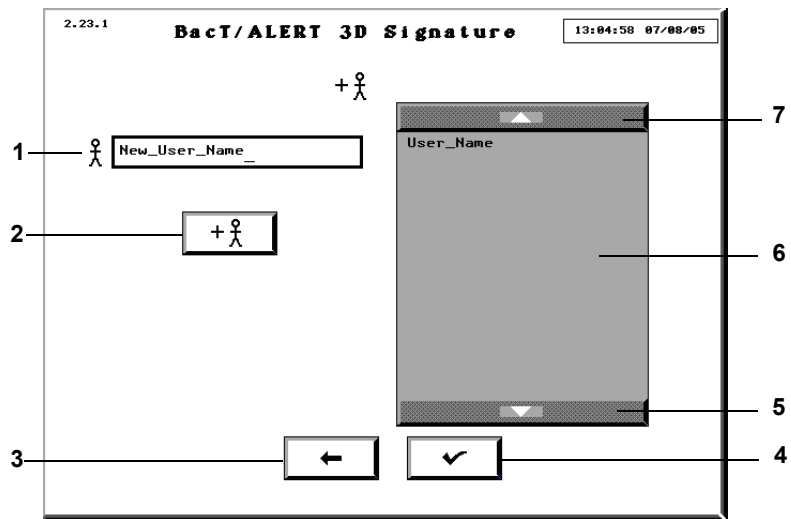


Figure 10-11: Add User Screen

- 1 — User Name Field
- 2 — Add User Button
- 3 — Previous Screen Button
- 4 — Check Button
- 5 — Scroll Down Button
- 6 — User List
- 7 — Scroll Up Button

Note: Multiple users can be added without leaving this screen.

- 2) Click in the **User Name** field (field turns white) and enter the *user name*.
The minimum user name length is 1 character and the maximum is 24 characters.

Note: Do not use the asterisk (*) when adding a user name.

- 3) Press the **Add User** button or the **Enter** key.
The new user is added to the User List.
- 4) Press the **Check** or **Previous Screen** button to return to the User Configuration screen.


Note: When you log in to the instrument for the first time, the default password is blank. See [Logging In for the First Time on page 10-2](#), to set your password.

Deleting a User

To delete a user from the User List:



- 1) From the User Configuration screen (see [Figure 10-10, User Configuration Screen, on page 10-13](#)), press the **Delete User** button

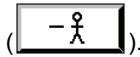

The Delete User screen appears.

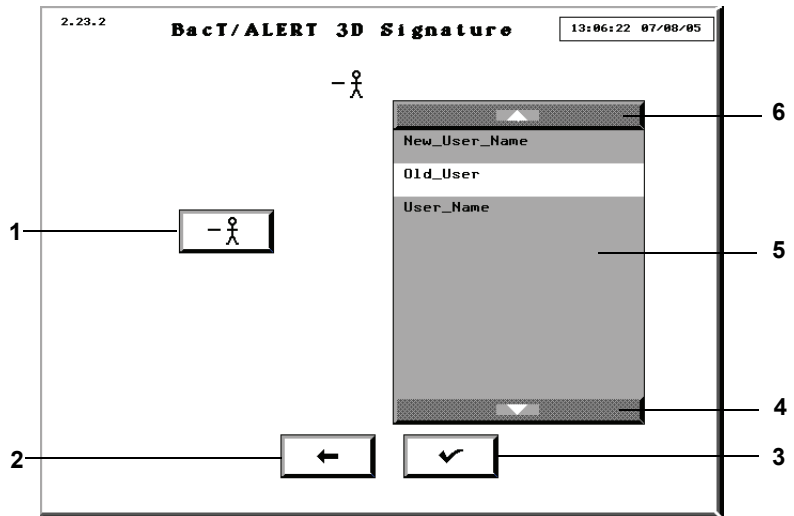


Figure 10-12: Delete User Screen

- 1 — Delete User Button
- 2 — Previous Screen Button
- 3 — Check Button
- 4 — Scroll Down Button
- 5 — User List
- 6 — Scroll Up Button

- 2) Press the **Scroll Up** or **Scroll Down** button to locate the user name to be deleted.
- 3) Click (or press) the user name to select.
- 4) Press the **Delete User** button to delete the selected user from the User List.
- 5) Press the **Check** or **Previous Screen** button to return to the User Configuration screen.

Deleting More Than One User

To delete more than one user at a time:



- 1) From the Delete User screen (see [Figure 10-12](#)), press the user names to select.

Note: If you select the wrong user name, press on the user name again to deselect.

- 2) Press the **Delete User** button to delete the selected users from the User List.
- 3) Press the **Check** or **Previous Screen** button to return to the User Configuration screen.

Clearing a User Password

In the event you may need to clear a user's password (ex. a user forgets their password), perform the following procedure.



- 1) From the User Configuration screen (see [Figure 10-10, User Configuration Screen, on page 10-13](#)), press the **Clear Password** button



The Clear Password screen appears.

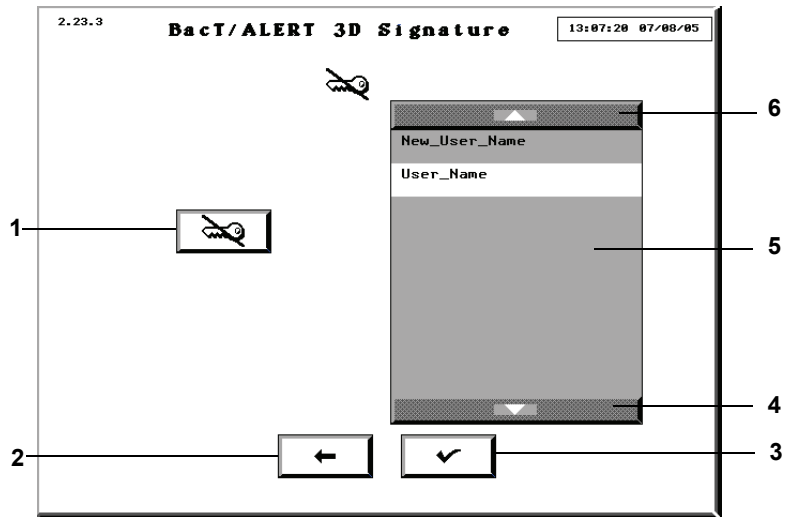


Figure 10-13: Clear Password Screen

- 1 — Clear Password Button
- 2 — Previous Screen Button
- 3 — Check Button
- 4 — Scroll Down Button
- 5 — User List
- 6 — Scroll Up Button

- 2) Press the **Scroll Up** or **Scroll Down** button to locate the user name.
- 3) Click (or press) the user name to select.
- 4) Press the **Clear Password** button to clear the password for the selected user.
- 5) Press the **Check** or **Previous Screen** button to return to the User Configuration screen.

Note: When you clear a user password, the password is reset to the blank setting. The user will then have to recreate a password (see [Logging In for the First Time on page 10-2](#)).

Clearing More Than One User Password

To clear more than one user password at a time:



- 1) From the Clear Password screen (see [Figure 10-13](#)), press the user name to select.

Note: If you select the wrong user name, press on the user name again to deselect.

- 2) Press the **Clear Password** button to delete the passwords for the selected users from the User List.
- 3) Press the **Check** or **Previous Screen** button to return to the User Configuration screen.

Audit Trail

One of the features of 21 CFR Part 11 mode is an audit trail that records activities and other related information (ex. bottle events). Each audit trail entry consists of an event, the time and date the event occurred. See [Table 10-1](#) for a list of events recorded in the audit trail.

Table 10-1: Audit Trail Events

Event	Event Parameters
User Log in	Time, Date, User ID
User Log out	Time, Date, User ID
Anonymous Bottle First Loaded	Time, Date, User ID, Cell Location, Max Test Time, Sequence Number
Identified Bottle First Loaded	Time, Date, User ID, Cell Location, Bottle Type, Max Test Time, Bottle ID, Accession Number, Sequence Number
Bottle Identified	Time, Date, User ID, Cell Location, Bottle Type, Max Test Time, Bottle ID, Accession Number, Sequence Number
Bottle Moved	Time, Date, User ID, Original Cell, Destination Cell, Sequence Number
Bottle Unloaded	Time, Date, User ID, Cell Location, Sequence Number

Table 10-1: Audit Trail Events (Continued)

Event	Event Parameters
Bottle Record Changed	<p>Time, Date, User ID or LIS, Cell Location, Action (Bottle Type Changed, Loaded Status Changed, Max Test Time Per Bottle Changed, Test Result Manually Changed, Bottle ID Changed, Accession Number Changed), Sequence Number, Bottle Type, Loaded Status, Max Test Time, Test Result, How Determined, Bottle ID, Accession Number</p> <p>Note - The Bottle Type, Loaded Status, Max Test Time, Test Result, How Determined and Accession Number are recorded in the Audit Trail when they change.</p>
Temperature Calibration	Time, Date, User ID, Module ID, New Temperature Setpoint
Cell Calibration	Time, Date, User ID, Cell Location
Display Time Changed	Time, Date, User ID, Amount of Change relative to UTC (Universal Coordinated Time)
Firmware Updated	<p>Time, Date, Device, Firmware Revision</p> <p>Note - User ID is not recorded.</p>
Module, Drawer, Rack, or Cell Enabled or Disabled	<p>Time, Date, User ID, Location, Action (enable or disable), Device</p> <p>Note - This event is recorded only from the Enable/Disable Module, Drawer, Rack, or Cell screen (screen 2.2).</p>
Maximum Test Time for Bottle Type Changed	Time, Date, User ID, Bottle Type, Max Test Time
Positive Alarm Setting Changed	Time, Date, User ID, Action (enable or disable)
Instrument Alarm Setting Changed	Time, Date, User ID, Action (enable or disable)
Operation Error Alarm Setting Changed	Time, Date, User ID, Action (enable or disable)
System Password Changed	Time, Date, User ID
21 CFR Part 11 Mode Changed	<p>Time, Date, User ID, Action (enable or disable)</p> <p>Note - User ID is recorded only for disable.</p>

Table 10-1: Audit Trail Events (Continued)

Event	Event Parameters
User Account Changed	Time, Date, User ID, Action (user added, user deleted, password changed or cleared)
Patient Name Changed	Time, Date, User ID, Hospital ID, First Name, Last Name
Hospital ID Changed	Time Date, User ID, Accession Number, Hospital ID
Cell Gap Detection Faults Cleared	Time, Date, User ID, Number of bottles cleared

Note: Some recorded events may occur while you are logged out and will not have an associated User ID (ex., upgrading firmware). In these cases, the user is listed as “Unknown”.

Note: The audit trail is available on all BacT/ALERT® 3D software configurations, regardless of 21 CFR Part 11 status.

Accessing the Audit Trail

The audit trail is available in English only and is not editable. The audit trail is available as one of the Software Test screens.

To access the audit trail:

- 1) Access the Setup screen and enter a valid password (see [Accessing the Setup Screen Function Buttons in Chapter 4](#)).
- 2) From the Setup screen, press **Ctrl + D** on the keyboard.
The Software Test screen appears.
- 3) Press the **7** key on the keyboard.
The Audit trail appears in the Software Test screen window.

About This Chapter

This chapter provides you with a complete description of the of the BacT/ALERT® 3D instrument when it is configured for Mycobacterial (MB) functions.

Chapter Contents

System Description • 11-2

Intended Use • 11-2

Overview • 11-2

BacT/ALERT® 3D Instrument • 11-3

System Startup • 11-6

General Information • 11-6

Theory of Operation • 11-7

Safety Features • 11-8

Device Comparison for Recovery of Mycobacteria • 11-10

System Description

Intended Use

The Mycobacterial (MB) configuration of the BacT/ALERT® 3D instrument is designed to grow and detect mycobacteria from patient samples.

Overview

The BacT/ALERT® 3D system uses no radioactive reagents, is non-invasive, and offers walk-away automation for mycobacteria detection. No handling of the bottle inoculated with specimen is required after it is placed in the BacT/ALERT® 3D instrument. Positives are signaled immediately upon detection. After a specified time, which can be determined by the operator, negatives are signaled if no growth has been detected.

The BacT/ALERT® 3D instrument consists of a Controller Module that directs the activity of one or more Incubation Modules. Each Incubation Module may contain one or more MB Drawers for mycobacterial detection. Each drawer can be converted by a service engineer to perform either an MB or non-MB function. One important difference between the drawer configurations is that MB Drawers do not agitate the bottles during incubation. MB Drawers are clearly identified by a red label on the drawer handle. It is important to open and close the door gently to minimize avoidable agitation when adding or removing bottles.

The disposable Mycobacteria bottles contain bioMérieux's patented colorimetric sensor which is continuously monitored for positive sample detection. Unloading or manipulating the bottles when not indicated by the system may interfere with critical bottle readings. In addition, the bottles contain broth and atmosphere which promote the recovery of a wide variety of *Mycobacterium* species without venting. MB/BacT® Antimicrobial Supplement and Reconstitution Fluid inhibit the growth of contaminating organisms and enhance the recovery of mycobacteria.

The data generated by the BacT/ALERT® 3D are managed by extensive data management software which provides a high level of flexibility. Mycobacterial data can easily be stored, queried, sorted and reported. Predefined reports and macros are available, or they may be customized by the user.

In the following pages, the elements of the MB configuration of the BacT/ALERT® 3D will be described with special emphasis on how they differ from the non-MB configuration described in the rest of this manual. While the

function and use of MB configuration are similar to those of non-MB configuration, there are important differences.

WARNING



Laboratory procedures involving mycobacteria require special equipment and techniques to minimize biohazards. Specimen preparation and positive sample handling must be done in a biological safety cabinet.

BacT/ALERT® 3D Instrument

The instrument consists of a Controller Module and one or more Incubation Modules. The instrument incubates and continuously monitors the status of each mycobacterial culture bottle. The Incubation Modules consist of four drawers each with a 60-bottle capacity. MB and non-MB drawers can coexist in the same Incubation Module.

MB Drawers may be provided directly by bioMérieux or may be converted from a non-MB configuration in the field.

Each MB Drawer handle has a red MB label also displaying the Incubation Module number and letter corresponding to that drawer. The MB Drawer faces on the **Instrument** icon of the Main screen are also labeled (see [Figure 11-1](#)).

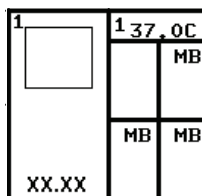


Figure 11-1: Instrument Icon for MB-Configured System

Circulation Fan

The circulation fan, normally running to distribute air within the BacT/ALERT® 3D instrument, is turned off whenever a drawer in the Incubation Module is opened.

Rocker Clamp

The BacT/ALERT® 3D drawer configured for MB has the agitation mechanism disengaged so the three racks within each drawer remain immobile.

Note: *The BacT/ALERT® 3D software must also be configured to activate the MB status of each drawer (see [MB Drawer Configuration on page 11-6](#)).*

Installation Procedures and Special Requirements

Initial installation and setup of the BacT/ALERT® 3D Microbial Detection System, or of any added Incubation Module(s), are to be performed only by a bioMérieux Service Representative.

If it is necessary to move or store the BacT/ALERT® 3D System or one of its modules, contact bioMérieux Service for assistance.

The BacT/ALERT® 3D has been designed to minimize risks associated with Mycobacterial testing. However, to further reduce the risks of accidental exposure to infectious agents, additional precautions should be taken. It is strongly recommended that the instrument be placed in a laboratory used for the routine culture of *M. tuberculosis*. For activities involving the propagation and manipulation of *M. tuberculosis* or *Mycobacterium* species grown in culture, BioSafety Level 3 Practice, Containment Equipment, and Facilities are required as recommended by CDC and NIH guidelines.

At a minimum, the instrument should be placed in a contained environment with controlled access which has a tuberculosis exposure control plan.

The locations should have surfaces which can be easily decontaminated using an appropriate topical disinfectant. The instrument must not be placed in an open corridor or hallway that is accessible to the general public or the patient population.

See Drawer Safety Closure Kit in Reagents & Accessories.

Barcodes

MP Process Bottle barcodes are encoded with the bottle type. MP Process Bottle barcodes have an SM (plastic) prefix. When an MP Process Bottle barcode is entered, the BacT/ALERT® 3D system will direct the user to load the bottle into an MB Drawer. All MB Drawers are clearly marked with a red label on the drawer handle.

MB Blood bottle barcodes are encoded with the bottle type. MB Blood bottle barcodes have an !B (glass) prefix. When an MB Blood Bottle barcode is

entered, the BacT/ALERT® 3D system will direct the user to load the MB Blood Bottle into any drawer containing cells available for loading.

WARNING



Do not use Generic barcodes to load BacT/ALERT® MB Bottles.

Load Bottles

Cells are assigned based on the type of bottle. MP Process Bottles are directed exclusively to MB Drawers.

Note: *Anonymously loaded MP Process Bottles must be placed in an MB drawer for proper incubation and detection of positives.*

Note: *MB Blood bottles must never be loaded anonymously.*

Unload Positives and Negatives

If an instrument contains both MB and non-MB bottles, both bottle types will be identified for unloading when any of the unload buttons is pressed. The user then has an option of unloading all indicated bottles, all indicated bottles of a certain type (ex. MB or non-MB), or a portion of the indicated bottles of any type.

Set Maximum Test Time

The system default maximum test time is 42 days for Mycobacteria bottles. Test times may be changed manually through the Set Maximum Test Time screen ([see Setting the Maximum Test Time in Chapter 7](#)).

Positive Detection Algorithm

A special detection algorithm is used to detect growth of *Mycobacterium* species. This algorithm is designed to detect growth at the earliest possible time.

Bottle Status

The Main Screen on the monitor displays the status of positive or negative Mycobacteria bottles separately from the status of other bottle types.



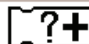
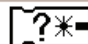
MB	1	1	0	0	0
BC	14	4	1	9	1
					

Figure 11-2: Bottle Count Table and Unload Buttons

Located just above the **Instrument** icon on the Main screen are the **Unload** buttons. Above each **Unload** button is a rectangular box containing the number of bottles currently matching the unload criteria for that button.

The Bottle Count Table contains two rows. The top row displays the bottle count for Mycobacteria culture bottles and is labeled **MB**. The bottom row displays the bottle counts for non-mycobacterial culture bottles and is labeled **BC**.

The leftmost box in each bottle count row contains the culture type label (ex. BC, MB) and is not associated with an **Unload** button. The number displayed within this box represents the total number of bottles (of any status) currently loaded into the system.

The second leftmost box shows the total number of identified bottles with a positive test status. It appears above the **Positive Bottle** icon.

The box third from the left shows the total number of identified and anonymous bottles with a negative test status. It appears above the **Negative Bottle** icon.

The box second from the right shows the total number of anonymous bottles with a positive test, and is associated with the corresponding icon.

The total number of anonymous bottles with a negative-to-date or negative test status appears rightmost, and is associated with the corresponding icon.

System Startup

General Information

MB Drawer Configuration

The BacT/ALERT® 3D drawer agitation mechanism can be disengaged at the factory or by a service engineer on site, after which the three racks within the drawer remain immobile. The BacT/ALERT® 3D Controller Module is also configured by bioMérieux personnel to activate the MB status of each drawer.

Limitations of the Test

Refer to the package insert enclosed with the MP Process Bottles for additional information.

Service and Maintenance

Preventive maintenance and troubleshooting are performed as described in [Chapter 8, System Maintenance](#).

Additional procedures pertinent to Mycobacteria are referenced in [Safety Features on page 11-8](#).

Theory of Operation

The BacT/ALERT® 3D Mycobacterial Detection System utilizes a colorimetric sensor and reflectance detector to determine the level of carbon dioxide within the bottle. If microorganisms grow in the bottle, carbon dioxide is produced which will change the color of the sensor on the bottom of the bottle. The instrument monitors this color while incubating the sample and determines if growth has occurred.

The MP Process Bottles contain a media which will, in combination with the MB/BacT® Reconstitution Fluid, promote the growth of mycobacteria. If a non-sterile sample is tested, the Reconstitution Fluid should be combined with the MB/BacT® Antibiotic Supplement, which is then added to the Process Bottle in accordance with the instructions in the package insert. The addition of these antibiotics inhibits the growth of contaminating, non-mycobacterial microorganisms while permitting the growth, detection, and recovery of mycobacteria.

Principle of Detection

Mycobacteria behave like most other aerobic bacteria with respect to carbohydrate metabolism, energy production, and the biosynthesis of low weight metabolites. They are able to assimilate a large range of carbohydrates, lipids, and proteins. How mycobacteria assimilate and organize these substrates is shown diagrammatically in [Figure 11-3](#).

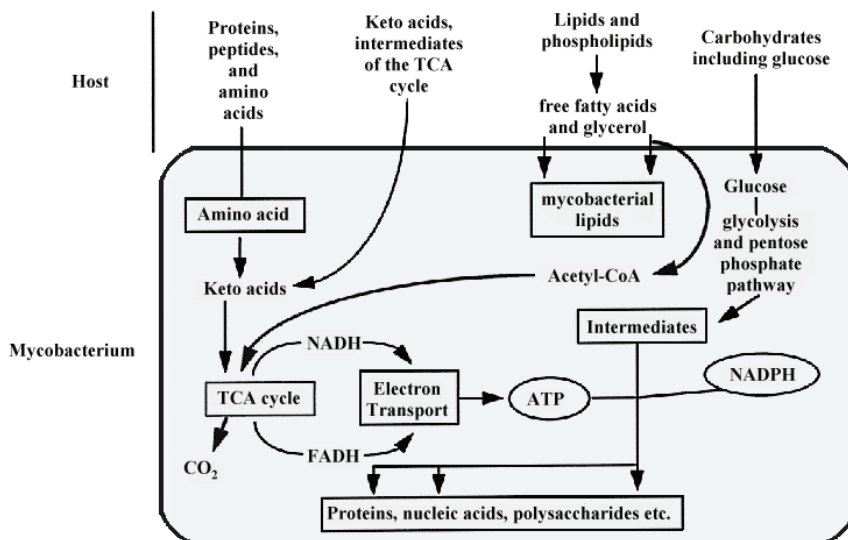


Figure 11-3: Metabolic Pathway for Mycobacterial CO₂

Glycerol was selected as the primary metabolic source in the MP Process Bottle because of its unique metabolic pathway and its ability to maximize the amount of CO₂ generated by mycobacteria. Once ingested, glycerol is converted to Acetyl-CoA and oxidized through the Krebs or Tricarboxylic Acid Cycle (TCA). CO₂ and reduced electron carriers are the major metabolic byproducts of this oxidation.

A solid-state sensor at the base of each MP Process Bottle detects CO₂ as an indicator of microbial growth. As the concentration of CO₂ increases, the sensor undergoes a color change from green to yellow.

Safety Features

The design of the BacT/ALERT® 3D provides several safety measures to help mitigate hazards associated with mycobacteria and other organisms in the interest of operator and laboratory safety. They are summarized as follows.

- All glass bioMérieux BacT/ALERT® media bottles are manufactured with a polymeric safety sleeve to help limit the effects of damage to a bottle. Plastic bottles are made from a break-resistant material.
- Each bottle cell within the BacT/ALERT® 3D Incubation Module is sealed to help contain and minimize effects of liquid spillage.

- A drip tray is incorporated at the bottom of each drawer beneath the opening end of the racks to minimize the effect of any spillage or liquid.
- The circulating fans within the Incubation Module turn off whenever any Incubator Drawer is open to minimize airflow and the potential for aerosols.
- To reduce the chance of binding a heavily overlabeled bottle in a cell, Incubation Module cells are gently tapered and fabricated from a compliant polymer.

WARNING



Pathogenic Microorganisms including Hepatitis B virus and Human Immunodeficiency Virus (HIV) may be present in specimens. *Universal Precautions* and Local Laboratory guidelines should be followed in handling all items contaminated with blood or body fluids. If an inoculated tube is found to be leaking or is accidentally broken during collection or transport, use the established procedures in your facility for dealing with Mycobacterial spills. As a minimum, *Universal Precautions* should be employed. Tubes should be discarded in an appropriate manner.

General Precautions

The following precautions should be observed during Maintenance and Repair, even in situations where a spill is neither observed nor suspected. At a minimum, disposable gloves, eye protection, and a laboratory coat should be worn. Any parts removed or tools used should be decontaminated using a 10% bleach solution or other EPA registered tuberculocidal disinfectant before removal from the laboratory. Anything which cannot be disinfected should be sealed in a plastic bag, labeled as biohazardous, and handled accordingly. In addition, the institution's safety precautions should always be observed. Refer to *Biosafety in Microbiological and Biomedical Laboratories*. HHS Publication No. CDC 93-8395. US Dept of Health and Human Services, Public Health Service, Centers for Disease Control and Prevention, National Institutes of Health, ed 3, 1993, pp 93-96.

Spill Cleanup

For specific spill cleanup procedures, see [Safety Precautions and Procedures in Chapter 8](#). (See also [Spill Cleanup in Chapter 8](#).)

Disinfection Procedure for Spills Within/Onto the Instrument

If a spill is detected, only persons wearing protective clothing including gloves, and a HEPA filter respirator should remain in the room. Any test

specimen spilled in an instrument should be removed immediately using the disinfection procedures in [Chapter 8, System Maintenance](#).

Remove Drawer

If necessary, remove the Drawer according to the instructions packaged with the Drawer Safety Closure Kit (Part Number 13228). The Drawer Safety Closure Kit is stored beneath the top cover of the Controller Module.

Contact bioMérieux Instrument Service for replacement parts.

Device Comparison for Recovery of Mycobacteria

The following table shows device comparison of recovery of mycobacteria based on concordance and discordance between the BacT/ALERT® 3D and the MB/BacT systems.

Table 11-1: Device Comparison for Recovery of Mycobacteria

Test Results		Number Specimens	% of Total
MB/BacT	BacT3D		
Positive	Positive	22	-
Negative	Negative	268	-
Total Concordance		290	93.2%
Positive	Negative	8	-
Negative	Positive	13	-
Total Discordance		21	6.8%
Total Specimens Tested		311	100.0%

About This Chapter

This chapter explains how to enter international characters on the BacT/ALERT® 3D instrument.

Chapter Contents

International Character Entry • A-2

Entering International Characters • A-4

International Character Entry

You can enter International characters on the BacT/ALERT® 3D. Since the keyboard does not contain the International characters, enter a base character then use the **Page Up** or **Page Down** keys to toggle to the variant character desired.

Each block in the table below lists a base character on the left and the variants that can be accessed from that base character on the right.

Table A-1: International Character Table

Uppercase		Lowercase	
Base Char	Variant	Base Char	Variant
A	À	a	à
	Á		á
	Â		â
	Ã		ã
	Ä		ä
	Å		å
	Æ		æ
			ä
E	È	e	è
	É		é
	Ê		ê
	Ë		ë
I	Ì	i	ì
	Í		í
	Î		î
	Ï		ï

Table A-1: International Character Table (Continued)

Uppercase		Lowercase	
Base Char	Variant	Base Char	Variant
O	Ô	o	ò
	Ó		ó
	Õ		ô
	Ö		ö
	Ø		ø
	Ù		ù
	Ú		ú
	Û		û
	Ü		ü
	Ý		ý
	Ñ		ñ
	Ç		ç
	ß		-
	μ		-
	Ɔ		-
	Ɔ		-

International characters can be used in any of the following text fields on the system:

Clinical Use

- **Accession Number** field
- **Hospital ID** field
- **Patient First Name** field
- **Patient Last Name** field
- **Report Label** fields
- **Report Title** field

INDUSTRY Use

- **Sample ID** field
- **User Defined 3**
- **User Defined 1**

- **User Defined 2**
- **Report Label** fields
- **Report Title** field

Entering International Characters



- 1) Enter a *base character* from the table. See [Manually Entering Text into a Data Entry Field \(Keyboard\) in Chapter 3](#).
- 2) Use the **Page Up** or **Page Down** key to scroll through the characters in the block from the table.

Example: To enter an **Å**, perform one of the following:

- Press **Shift + A** then press **Page Up** six times.

or

- Press **Shift + E**, then press **Page Down** twice.

Note: *The entire International Character table can be scrolled through after entering one base character. After scrolling through an entire block from the table, the system scrolls to the base character in the next block. For example, if **n** is entered and **Page Up** is pressed twice, then **c** will display.*

About This Chapter

This chapter provides you with a description of the BacT/ALERT® 3D culture bottle along with a quality control procedure.

Note: Information in this chapter that is intended for clinical use only or industry use only is marked *Clinical Use* or *INDUSTRY Use*.

Chapter Contents

BacT/ALERT® Culture Bottles • B-2

Introduction • B-2

Quality Control of Growth Performance • B-4

BacT/ALERT® Culture Bottles

Introduction

Disposable culture bottles to which samples are added for testing. Each bottle contains a sensor that detects carbon dioxide (CO₂) as an indicator of microbial growth.

BacT/ALERT® culture bottles are ready for use. An expiration date is printed on each bottle label. Do not use the media beyond the last day of the month indicated. If the bottles are refrigerated, precipitates may form that will disappear when the bottles are warmed to room temperature. Store protected from light at room temperature (15–30°C). Bottles must be at room temperature before use.

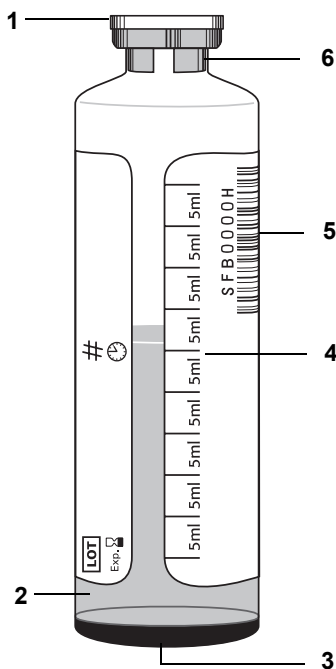


Figure B-1: Typical BacT/ALERT® Culture Bottle

- | | |
|--------------------|-------------------------|
| 1 — Flip Cap | 4 — Volume Designations |
| 2 — Culture Medium | 5 — Barcode |
| 3 — Sensor | 6 — Stopper/Seal |

Flip Cap

Plastic flip cap that ensures septum seal cleanliness, but not sterility.

Stopper/Seal

Butyl stopper secured with a color-coded aluminum seal. The stopper is not sterile and should be disinfected prior to sample inoculation.

Bottle

See [Figure B-1](#).

Volume Designations (Clinical Use)

Markings which indicate approximate volume in 5 mL increments for adult bottles and 4 mL increments for pediatric bottles. These demarcations assist in approximating the volume of sample added to a bottle.

Volume Designations (INDUSTRY Use)

Markings which indicate approximate volume in 5 mL increments. These demarcations assist in approximating the volume of sample added to a bottle.

Barcode

The BacT/ALERT® 3D uses a barcode system to load bottle information. Each BacT/ALERT® bottle has a barcode on its label. For convenience, a portion of this barcode label may be peeled off and affixed to other records (ex. requisition slips). When any other hospital or laboratory label is applied to the bottle, ensure that at least one complete lengthwise portion of the BacT/ALERT® barcode is left visible for proper scanning and that the bottle can be easily removed from its cell.

Entering the barcode commands the BacT/ALERT® 3D system to apply the proper algorithm to the bottle readings and to direct the bottle to the proper drawer type:

- (Clinical Use) BacT/ALERT® MP Bottles will be directed to MB drawers only.
- (Clinical Use) BacT/ALERT® MB Bottles will be directed to any drawer type containing available cells.
- Non-MB bottles will be directed to non-MB drawers only.

Occasionally, a BacT/ALERT® barcode label may become unreadable to the barcode scanner (ex. something spilled on the label, the label is covered or torn). If this happens, generic barcode labels are available (contact your local bioMérieux Representative). These labels begin with a G or a number and do not contain bottle type information (ex. aerobic, anaerobic, etc.). Attach the

generic BacT/ALERT® barcode label lengthwise on the bottle, not horizontally around the bottle. Do not place the barcode label over the sensor on the bottom of the bottle.



CAUTION: (Clinical Use) BacT/ALERT® MB Bottles must never be loaded anonymously or with a Generic barcode.

Sensor

Colorimetric carbon dioxide sensor attached internally to the bottom of each bottle.

Limitations of the Test

Refer to the package insert enclosed with the BacT/ALERT® culture bottles for additional information.

Clinical Use

Many variables involved in blood culture testing cannot be practically controlled to provide total confidence that results obtained are due solely to proper or improper performance of any culture medium or detection system.

Note: *A Gram-stained smear from a negative bottle may sometimes contain a small number of non-viable organisms that were derived from culture medium components, staining reagents, immersion oil, or glass slides; therefore, false-positive results are indicated.*

INDUSTRY Use

Many variables involved in sample testing cannot be practically controlled to provide total confidence that results obtained are due solely to proper or improper performance of any culture medium or detection system.

Note: *A smear from a negative bottle may sometimes contain a small number of non-viable organisms that were derived from culture medium components, staining reagents, immersion oil, or glass slides; therefore, false positive results are indicated.*

Quality Control of Growth Performance

If desired, individual laboratories can perform routine quality control testing of different lots of BacT/ALERT® blood culture bottles when used with the BacT/ALERT® 3D System.

Blood Culture Bottles (SA, SN, FA, FN, and PF)

Each case of BacT/ALERT® blood culture bottles (including BacT/ALERT® SA, SN, FA, FN, and PF) is packaged with a Certificate of Conformance that should be filed for inspection purposes. If additional QC is desired, bottles should be seeded at approximately 400 CFU/bottle using SPS anticoagulated fresh human blood with fresh growth of one or more of the organisms listed on the Certificate of Conformance for each bottle type. The general procedure is described below:

- 1) Add 1–2 mL blood (Clinical Use) to the bottle(s).
- 2) Using growth from solid media which is 18–24 hours old, prepare a suspension in Tryptic Soy Broth (TSB) at 85–90% transmittance at 660 nm or a McFarland 0.5 density (10^8).
- 3) Perform serial dilutions of this suspension as follows:
 - a. Dilute 1:100: Pipette 0.1 mL of suspension from [Step 2](#) into 9.9 mL TSB.
 - b. Dilute 1:100: Pipette 0.1 mL of suspension from [Step 3a](#) into 9.9 mL TSB.
 - c. Dilute 1:10: Pipette 1.0 mL of dilution from [Step 3b](#) into 9.0 mL TSB (density should then be approximately 10^3).
- 4) Inoculate 0.4 mL of the final dilution from [Step 3c](#) into the desired adult bottle. If using BacT/ALERT® PF, inoculate 0.2 mL of the final dilution from [Step 3c](#).
- 5) As soon as possible, load the bottle(s). All aerobic organisms should be positive within 48 hours and all anaerobic organisms within 72 hours.

Mycobacteria Culture Bottles (MB and MP)

Each case of BacT/ALERT® Mycobacteria culture bottles (including MP and MB) is packaged with a Certificate of Conformance that should be filed for inspection purposes. If additional QC is desired, refer to the instructions in the package insert.

Culture Bottles (iAST, iNST, and iLYM)

At the time of release, each case of BacT/ALERT® culture bottles (including iAST, iNST, and iLYM) is tested with a group of microorganisms that verify the performance of the media. A Certificate of Conformance is issued by bioMérieux that should be filed for inspection purposes. If additional QC is desired, refer to the instructions in the package insert.

About This Chapter

This chapter provides you with best practices for preventing false positives.

Note: *Information in this chapter that is intended for clinical use only or industry use only is marked Clinical Use or INDUSTRY Use.*

Chapter Contents

Best Practices for Preventing False Positives • C-2

Preventing False Positives — User • C-2

Preventing False Positives — Instrument • C-5

Best Practices for Preventing False Positives

Investigation into the causes of false positive rates for BacT/ALERT® 3D systems have resulted in a list of Best Practices that, if performed, will reduce the occurrence of false positives. In this appendix, false positives are divided into two separate categories: User and Instrument. Detailed descriptions of conditions that may cause false positives are provided, along with Best Practices and/or recommendations to prevent these conditions.

Preventing False Positives — User

This section describes conditions that may cause false positives that can be prevented by the User.

High Amount of White Blood Cells (Clinical Use)

More than 10^5 leukocytes per mL of blood can be caused by hematological disease, such as leukemia, or can be a reaction of the reticulo-endothelial system to infections. These white blood cells can produce high amounts of CO₂. Bottles can be flagged positive by acceleration or rate algorithms.

Best Practice #1 — When no organisms are seen on the Gram stain, it is appropriate to use the Load Bottles function to reload the bottle (see [Loading Bottles in Chapter 3](#) for Clinical Use, or [Loading Bottles in Chapter 4](#) for Industry Use). The status will default to Negative-to-Date and continue incubation to detect growing organisms for the duration of the Maximum Test Time. See [Handling Unconfirmed Positive Bottles \(False Positives\) in Chapter 3](#) (Clinical Use) or [Chapter 4](#) (INDUSTRY Use).

Blood (or Sample) Volume too High

If the bottles are filled with more than 10 mL of blood, or turbid body fluid, they can be flagged positive because of the CO₂ produced by the blood cells. An overfilled bottle may contain a blood cell content high enough to trigger acceleration or rate algorithms.

Best Practice #2 — Do not fill the bottles with more than the volume recommended in the package insert. See [Table C-1](#) for recommended volumes for each bottle type.

Table C-1: Bottle Volumes

Bottle Type	Recommended Volume
SA, SN, FA, FN	10 mL of sample
PF	Up to 4 mL of sample

Table C-1: Bottle Volumes (Continued)

Bottle Type	Recommended Volume
BPA, BPN	4 mL of Leucocyte Reduced Apheresis Platelets
MP	0.5 mL of processed sample pellet
MB	3 to 5 mL of blood
iAST, iNST (INDUSTRY Use)	User validated and sample dependent
iLYM (INDUSTRY Use)	Not more than 20 mL sample



CAUTION: Do not fill the bottles with more than the recommended volume of sample.

Large Bottle Loading and Unloading Events

If large numbers of bottles are loaded into the BacT/ALERT® 3D, or unloaded from the BacT/ALERT® 3D, at the same time and in the same areas, there may be a very large heat loss within the racks. This may trigger the acceleration or rate algorithms to erroneously flag positive.

Best Practice #3 — Limit the Loading time to 2 minutes in one area to control the entry of room temperature bottles into the racks. Close the drawer to allow the temperature to equilibrate before loading in that area again. Keep the unload time to no more than 15 minutes. Close the drawer to allow the temperature to equilibrate before unloading from that area again. You can load bottles into different drawers. If you have four incubation modules, use the drawers in all four incubation modules.

Temperature Changes in the Environment

The detection process of the BacT/ALERT® 3D is sensitive to temperature changes. False positives can occur because of temperature changes. These temperature changes can be caused by the following:

- 1) Drawer of the BacT/ALERT® 3D open for more than 10 minutes for extended workflow.
- 2) Failure to close the drawer properly. Look for yellow drawer indicator lights when leaving the room.
- 3) Locating the instrument under an air conditioning or heating vent.
- 4) Heating or cooling differences in the lab. For example, the lab technicians on a different shift turn up or down the heat or air

conditioning, there are open windows in the laboratory, or there are drafts that are directed at the instrument.

- 5) Loading too many bottles into a drawer in which bottles are already up to temperature and incubating.



CAUTION: Bottles should never be refrigerated for transport because some organisms are temperature sensitive.

Best Practice #4 — Ensure that temperature changes in the lab are minimized, ensure that the drawers are properly closed when work is done, and spread large bottle loads (unloads) across multiple drawers.

IMPORTANT: *Bottle reflectance and output of electronics varies with temperature. Limiting temperature fluctuations reduces the false positive rate.*

Anonymous Bottle Loading of BacT/ALERT® FA and PF Bottles

When FA or PF bottles are loaded anonymously, the BacT/ALERT® 3D will assign these bottles the default initial value threshold for the SA bottle of 3290. This default initial value threshold assignment is not changed, or re-calculated, and because of the threshold differences between these bottle types, false positives may occur.

Best Practice #5 — Avoid anonymous loading of the FA and PF bottles in a BacT/ALERT® 3D instrument.

Bottle Not Completely Loaded into the Cell

False positives may occur when the bottle is not completely loaded into the cell and is later pushed all the way into the cell. [Figure C-1](#) an example of how a cell graph appears when this occurs.

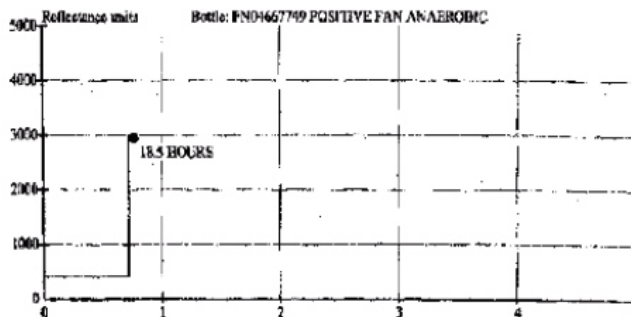


Figure C-1: Example Cell Graph

Best Practice #6 — Always press the bottle all the way into the cell when loading bottles.

Preventing False Positives — Instrument

This section describes causes of false positives that are due to the instrument.

Cell Noise

Cells may exhibit electronic “noise” that is visible in the graph. There are three main types instrument noise:

- 1) The first type of noise is when cell graphs show irregular fluctuating signals and sharp peaks and valleys, or large spikes in the graph. Contact your bioMérieux representative to help determine if the power supply is unstable, or if there are other electronics issues may be isolated to specific locations and corrected. Determine if a first loaded location is prevalent. Several bottles that were first loaded into a particular cell, rack, or instrument may demonstrate cell noise causing the algorithm(s) to be triggered. This is an infrequent, but typical, kind of electro-mechanical failure mode for the BacT/ALERT® 3D.

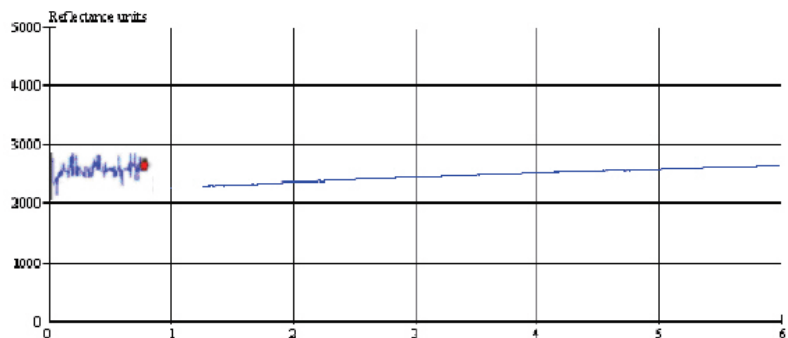


Figure C-2: Cell Graph Example — Electronics

- 2) A second type of noise reverts to a normal graph when the bottle is reloaded into another location (see [Figure C-3](#)). Contact your bioMérieux representative if this occurs.

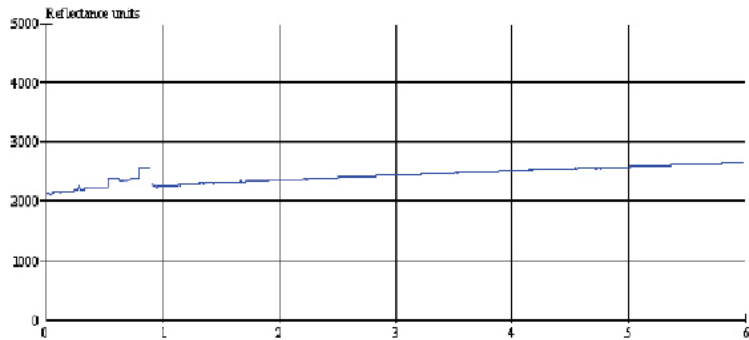


Figure C-3: Cell Graph Example — Reverts to Normal

- 3) A third type of cell noise involves a characteristic signal pattern in BacT/ALERT® 3D instruments that repeats (sometimes simultaneously in multiple cells) and seems to “cycle” on a daily basis at particular times (see [Figure C-4](#)). It may correlate to particular events such as temperature, lighting, or facility power events. These cyclic “blips” trigger a false positive event. Verify that there is a dedicated circuit for the BacT/ALERT® 3D (for example, verify that there is not a centrifuge or a refrigerator on the same circuit). Ensure there are no voltage drops at the power outlet. Also follow **Best Practice #4** to minimize temperature fluctuations.

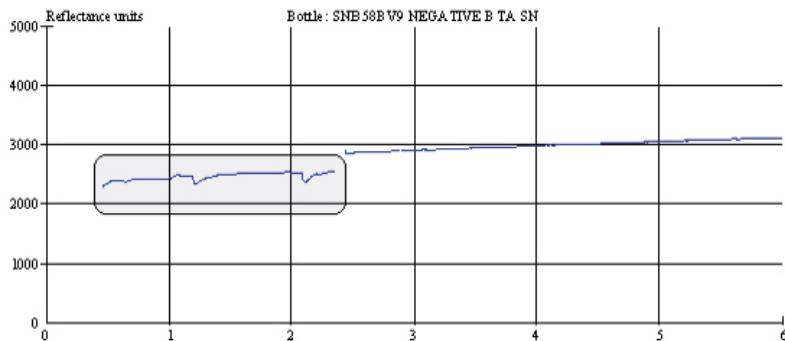


Figure C-4: Cell Graph Example — Cyclic

High Initial Value Readings

Other false positives are due to high initial value readings in a cell that are consistent in that cell, but normal when the positive bottle is unloaded and reloaded into another cell (see [Figure C-5](#)). If this occurs, the cells should be calibrated. If calibrating the cells does not correct the problem, disable and plug the cell and contact your bioMérieux representative.

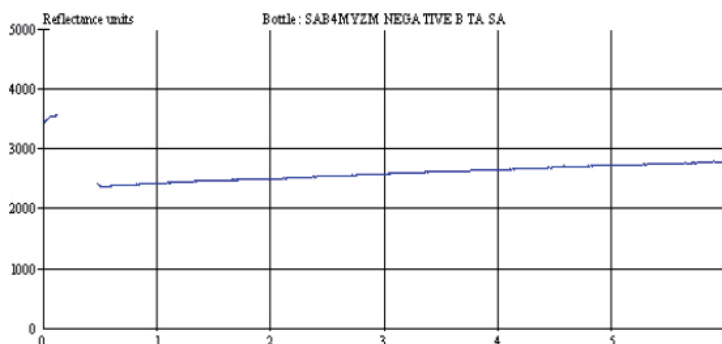


Figure C-5: Cell Graph Example — High Initial Value Reading

False Positive MP Process Bottles Flagged by Delta Algorithm (Clinical Use)

These false positives are specific to BacT/ALERT® MP Process bottles and are triggered by the very sensitive “Delta” portion of the algorithm necessary to detect mycobacteria that produce very small amounts of CO₂. These false positives may increase in number when calibration has drifted, or when cells have been calibrated at a temperature lower or higher than what is recommended, or if mycobacteria specimen processing is not consistent.

Perform cell calibration to unload MP Process bottles sequentially, and to calibrate all cells as they are unloaded on a daily basis, until the process is complete for all cells. Calibrate no more than three cells at a time to maintain temperature and load MP Process bottles **ONLY** in Mycobacterial incubation modules, if possible, to avoid temperature losses created by blood culture loading events in drawers of a mixed blood/MB incubation module. Follow **Best Practice #4** to minimize temperature fluctuations.

IMPORTANT: *Bottle reflectance and output of electronics varies with temperature. Limiting temperature fluctuations reduces the false positive rate.*

A standard method of processing mycobacteria specimens also helps to prevent false positives. The most critical steps for mycobacteria specimen processing are:

- 1) Maintain a consistent sample size of 10 mL by adding saline to small samples and splitting larger samples. This avoids too harsh or too little decontamination when reagents are pre-measured in 10 mL aliquots.
- 2) Decant liquid completely after centrifugation.
- 3) Re-suspend the pellet in sterile 0.067M phosphate buffer, pH 6.8.
- 4) Establish a processing method that consistently results in a sample with a neutral pH.
- 5) Use no more than the recommended 0.5 mL Mycobacterial Antibiotic Supplement (MAS) per bottle.

These steps will help to minimize the BacT/ALERT® MP culture bottle false positive rate while helping to maximize mycobacterium recovery rates. For additional information, see the best practices for mycobacteria specimen processing outlined in the *Customer Training Manual: Mycobacterial Testing with BacT/ALERT® Systems and Media*, P/N 60-00467.

GLOSSARY

Accession	(Clinical Use) A sample collected from a patient during a single event.
Accession ID	(Clinical Use) Synonym for accession number.
Accession Number	(Clinical Use) A number that uniquely identifies the sample or patient specimen.
Anonymous Bottle	A bottle that has been placed in an Incubation Module without going through the formal bottle loading process. Bottles loaded anonymously must eventually be identified and assigned a bottle ID.
Audible Alarm	Specified by the user to flag positive bottle(s), instrument failure, or an operator error.
Audit Trail	Record of activities and other related information (ex. bottle events).
BacT/ALERT®	bioMérieux's automated instrumentation to screen test specimens (Clinical Use)/samples (INDUSTRY Use) for the presence of microbes.
BacT/LINK®	BacT/LINK® is software for interfacing a BacT/ALERT® system on an LIS.
Barcode Reader Aperture	Houses a removable Barcode Reader. Used to scan accession numbers (Clinical Use)/Sample IDs (INDUSTRY Use) or bottle barcode labels and to identify bottles when loading or unloading.
Bottle Graph	A line graph that plots the microbial activity in a culture over time.
Bottle ID	Usually a barcode that uniquely identifies a single culture bottle.
Bottle Type	BacT/ALERT® bottle with a specific type of culture medium (ex. BacT/ALERT® SA - Clinical Use, or BacT/ALERT® iAST - INDUSTRY Use).
Cell	Holds and monitors each culture bottle. Each cell is numbered from 1 through 60.
Cell Flag	Secures bottles in cells. Aids in cell diagnostics and with bottle loading and unloading determinations.

Cell Indicator Light	Illuminates to show the location of positive, negative or anonymous bottles as well as to show where new bottles should be placed/removed. A cell indicator light is adjacent to each cell.
Check Box	Textless YES/NO prompt. A check in the box equals YES while no check equals NO .
CLSI	Clinical and Laboratory Standards Institute
Combination Module	Unit which contains the User Interface capability for the BacT/ALERT® 3D bottle handling system and has two incubation drawers built into the same module. The Combination Module also controls up to three Incubation Modules.
Controller Module	Unit which contains the User Interface capability for the BacT/ALERT® 3D bottle handling system and controls up to six Incubation Modules.
Controller Module Communications Port	Connects the Controller Module to an Incubation Module.
CPU 1	Connector fitting used to connect the data management system computer to the Controller Module or to connect two Controller Modules together.
CPU 2	Connector fitting used to connect two Controller Modules together.
Dimmed	BacT/ALERT® 3D disables (“dims” or “grays”) user interface items (ex. menu items, buttons, text fields) that are unavailable due to security restrictions or due to the current context of the software. Disabled user interface items are not selectable.
Disabled	Describes a user interface item (ex. button, text entry field) that is not currently selectable by the user. Disabled user interface items are usually displayed in gray.
Drawer	In the Incubation Module, each drawer contains three racks (60 cells in total) for a drawer capacity of 60 culture bottles. The four drawers of each Incubation Module are labeled A, B, C, or D. In the Controller Module, a drawer located below the barcode reader aperture in the front of the unit affords access to the Controller Keyboard and Quick Reference Cards.
Drawer Indicator Light — Yellow	Illuminates whenever that Incubation Module drawer is open and turns off when the drawer is closed. Flashes if the drawer remains open too long.

Drawer Indicator Light — Green	Illuminates if that Incubation Module drawer or its cells are involved in the operation the user has selected. Flashes if the drawer remains open too long.
Enabled	The opposite of disabled.
Field	Refers to an area on the screen where variable text is displayed.
Generic Bottle ID	A Bottle ID that does not indicate one of the pre-defined bottle types.
Grayed	See <i>dimmed</i> . Refers to user interface items that are not selectable.
Hospital ID	(Clinical Use) Text string that uniquely identifies a patient.
Icons	System functions or buttons appearing on-screen in pictorial form.
Incubation Module	Unit containing the four incubation drawers to hold and incubate BacT/ALERT® bottles.
Indicator Light or Lamp	An identifier for a cell or drawer that lights up to indicate location of a particular bottle or bottle-type. On drawers, indicator lamps are adjacent to the drawer handle. Within drawers, they are located on the racks adjacent to the indicated cell.
Incubation Module Communications Ports (6)	Each port connects an Incubation Module to the Controller Module.
Keyboard	<p>Provides a means of input for:</p> <p>(Clinical Use) Bottle IDs, Accession Numbers, Hospital IDs, and Names; or</p> <p>(INDUSTRY Use) Bottle IDs, Sample IDs, User Defined 1, User Defined 2 and User Defined 3 fields.</p> <p>Also serves as a backup input device should the Touchscreen or Barcode Reader fail.</p>
Laboratory Information System	Name for a customer maintained computer (usually a mainframe) that is used to collect information from laboratory instruments. Abbreviated LIS.
LIS	Acronym for Laboratory Information System.
LIS Port (Diagnostic)	Connector fitting on the Controller Module and Combination Module to connect to a Laboratory Information System.
Module	A component or subset of a system.

Monitor	A device upon which video images are displayed. In the BacT/ALERT® 3D system, there is a monitor located on the Controller Module and another as part of the bioMérieux data management computer system.
Negative	Describes a culture bottle that has no evidence of microbial activity.
Negative-to-date	Describes a culture bottle that is under test and negative so far (pending).
NIST	National Institute of Standards and Technology.
Operator Panel	The Operator Touchscreen allowing user interface with all instrument-related functions. Selections and other input can be made by simply touching the Operator Panel.
Positive	Describes a culture bottle that has evidence of microbial activity.
Power Connector	A fitting to attach the Alternating Current (AC) power cord.
Printer Port	Printer interface port (parallel) on the Controller Module or Combination Module for producing hard copy reports and graphs.
Prompt	A text entry field that appears on a screen. The primary purpose for a prompt is for data entry. Read-only prompts are not selectable and are dimmed. User entries into a prompt are checked for validity according to the value range internally assigned for that prompt.
Quick Reference Card	Outlines instructions for system operation and describes specific error codes. Located in the drawer beneath the barcode reader aperture.
Rack	Contains 20 cells, each of which can hold and monitor an individual culture bottle. Racks gently rock to agitate bottles.
Reference Thermometer	See Thermometer.
Scroll Button	A Controller Module touchscreen button that allows selection of numerical or symbolic values by pressing arrows above or below a window which shows the selected character. While an arrow is being pressed, the center portion shows one character from among the available list of choices.
Select	To choose a screen option with a pointing device.
Select Configuration	A BacT/ALERT® 3D configuration where the Controller Module or Combination Module is connected to neither an bioMérieux data management computer nor an LIS.

SelectLink Configuration	A BacT/ALERT® 3D configuration where the Controller Module or Combination Module is connected directly to an LIS.
Signature Configuration	A BacT/ALERT® 3D configuration where the Controller Module is connected directly to an bioMérieux data management computer.
Slidebar Switch	A rectangular Controller Module touchscreen device which moves between ON and OFF (symbolized by 1 and 0 , respectively) when the corresponding end of the rectangle is touched.
Thermometer	An independent, NIST-traceable temperature monitoring device normally located in Drawer A or B of an Incubation Module.
Toggle Button	A Controller Module screen button that becomes active (appears to be pressed) after the corresponding screen area is touched, and continues to be active until it is touched again.
Unconfirmed Positive	Describes a bottle the system determined to be positive but no confirmation is available by Gram stain or subculture. Also referred to as a false positive.
UPS	Uninterruptible Power Supply (battery backup).
USB Flash Drive	Flash memory storage device integrated with a Universal Serial Bus (USB).
User Interface	The methods by which a computer communicates with the user.

INDEX

Numerics

- 21 CFR Part 11 Mode 10-1
 - accessing the audit trail* 10-20
 - adding a user* 10-13
 - audit trail* 10-18
 - change password* 10-8
 - change password errors* 10-9
 - clearing a user password* 10-16
 - clearing more than one user password* 10-17
 - configuring users* 10-12
 - deleting a user* 10-15
 - deleting more than one user* 10-16
 - inactivity timeout* 10-7
 - log in/out of system* 10-2
 - logging in* 10-5
 - logging in for the first time* 10-2
 - logging out* 10-12
 - login errors* 10-7
 - user list* 10-13
 - wrong password alert* 10-8
 - wrong user alert* 10-8

A

- accession
 - attaching to a hospital ID*
 - Clinical Use 5-16
 - moving from one hospital ID to another*
 - Clinical Use 5-17
- add user screen (21 CFR Part 11 mode) 10-14
- additional supplies 1-3
- alarm
 - priority* 7-5
 - setting* 7-3
 - terminating an instrument* 7-5
- alerts (21 CFR Part 11 mode)
 - wrong password* 10-8
 - wrong user* 10-8
- anchor display/scroll buttons 2-29
- anonymous bottle indicated in empty cell 9-39
- attaching accession numbers to a hospital ID
 - Clinical Use* 5-16
- attaching bottle IDs to a sample ID

INDUSTRY Use 6-14

- attaching bottle IDs to an accession number
 - Clinical Use* 5-14
- attaching sample IDs to a User Defined 3
 - INDUSTRY Use* 6-16
- audit trail
 - accessing* 10-20
 - events* 10-18

B

- background color, monitor
 - Clinical Use* 3-4
 - INDUSTRY Use* 4-4
- backup drive
 - USB port* 2-9, 2-18
 - zip* 2-9, 2-18
- backups
 - initiating/scheduling* 7-7
- barcode cannot be scanned 9-40
- barcode reader
 - description* 2-9
 - doesn't beep* 9-40
 - location* 2-9
- barcode scanner
 - entering data*
 - Clinical Use 3-10
 - INDUSTRY Use 4-10
- basic functions
 - Clinical Use* 3-1
 - INDUSTRY Use* 4-1
- best practices for bottle handling C-1, C-2
- bottle
 - attaching ID to a sample ID*
 - INDUSTRY Use 6-14
 - attaching ID to an accession*
 - Clinical Use 5-14
 - broken* See *spills*.
 - counts*
 - Clinical Use 3-5
 - INDUSTRY Use 4-5
 - description* B-2
 - diagram* B-2
 - display readings*

- Clinical Use 3-34
- INDUSTRY Use 4-34
- loading procedure*
 - Clinical Use 3-11
 - INDUSTRY Use 4-11
- lodged in cell* 9-39
- moving from one accession number to another*
 - Clinical Use 5-15
- moving from one sample number to another*
 - INDUSTRY Use 6-15
- quality control* B-1
- relocation procedure* 8-21
- selecting*
 - Clinical Use 5-3, 5-4
 - INDUSTRY Use 6-3, 6-4
- unconfirmed positive handling*
 - Clinical Use
 - INDUSTRY Use
- unloading procedure*
 - Clinical Use 3-16
 - INDUSTRY Use 4-16
- viewing/printing graphs*
 - Clinical Use 3-31
 - INDUSTRY Use 4-31
- bottle handling
 - best practices* C-1
- bottle quality control B-1
- bottle readings screen, diagram
 - Clinical Use 3-35
 - INDUSTRY Use 4-34
- bottle type
 - entry tips*
 - Clinical Use 3-13
 - INDUSTRY Use 4-13
 - scroll button*
 - Clinical Use 3-17
 - INDUSTRY Use 4-17
- button
 - accession number selection*
 - Clinical Use 5-14
 - add user (21 CFR Part 11 mode)* 10-13
 - adjust X scale*
 - Clinical Use 3-33
 - INDUSTRY Use 4-33
 - adjust Y scale*
 - Clinical Use 3-33
 - INDUSTRY Use 4-33
 - anchor display top or bottom* 2-29
 - anchor display/scroll* 2-29
 - backup management* 7-8
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - bottle processing/customization*
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - bottle readings*
 - Clinical Use 3-33
 - INDUSTRY Use 4-33
 - bottle type scroll*
 - Clinical Use 3-17
 - INDUSTRY Use 4-17
 - calibrate cell* 8-26
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - calibrate module temperature* 8-23, 8-24
 - Clinical Use 3-22
 - INDUSTRY Use 4-21
 - calibration history* 7-25
 - calibration report* 7-22
 - cancel* 2-28
 - cell scroll*
 - Clinical Use 3-32
 - INDUSTRY Use 4-32
 - change password* 7-5
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - change password (21 CFR Part 11 Mode)* 10-4
 - change password (21 CFR Part 11 mode)* 10-3, 10-8
 - check* 2-28
 - clear password (21 CFR Part 11 mode)* 10-13
 - column selection scroll* 7-16, 7-17, 7-18, 7-19, 7-20, 7-21
 - configure users (21 CFR Part 11 mode)* 10-12
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - data transfer*
 - Clinical Use 5-14
 - INDUSTRY Use 6-14
 - delete user (21 CFR Part 11 mode)* 10-13, 10-15
 - description* 2-28
 - display report 1 (load report screen)*
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
 - display report 2 (status report screen)*
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
 - display report 3 (unload report screen)*

- Clinical Use 3-24
- INDUSTRY Use 4-24
- drawer scroll*
 - Clinical Use 3-32
 - INDUSTRY Use 4-32
- edit*
 - accession number to hospital ID relationships
 - Clinical Use 5-13
 - bottle ID to accession number relationships
 - Clinical Use 5-13
 - bottle ID to sample ID relationships
 - INDUSTRY Use 6-13
 - bottle type scroll
 - Clinical Use 5-7
 - INDUSTRY Use 6-7
 - cell contents
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - data relationships
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
 - hospital ID to patient name relationships
 - Clinical Use 5-13
 - maximum test time scroll
 - Clinical Use 5-7
 - INDUSTRY Use 6-7
 - sample ID to user defined 3 relationships
 - INDUSTRY Use 6-13
 - test result
 - Clinical Use 5-9
 - INDUSTRY Use 6-9
 - user defined 3 to user defined 1/2 relationships
 - INDUSTRY Use 6-13
- edit data relationships*
 - Clinical Use 5-12
 - INDUSTRY Use 6-12
- enable/disable hardware*
 - Clinical Use 3-22
 - INDUSTRY Use 4-21
- enable/disable module, drawer, rack, or cell* 8-18
- enabled vs disabled* 2-28
- end scroll* 2-29
- exit software* 8-12
- graph bottle readings*
 - Clinical Use 5-10
 - INDUSTRY Use 6-10
- home scroll* 2-29
- incubation module scroll*
 - Clinical Use 3-32
 - INDUSTRY Use 4-32
- incubation period scroll* 7-2
- line scroll up or down* 2-29
- login (21 CFR Part 11 Mode)* 10-2
- manual backup* 7-8
- manual request test orders*
 - Clinical Use 3-39
 - INDUSTRY Use 4-39
- manual send test results*
 - Clinical Use 3-39
 - INDUSTRY Use 4-39
- next screen* 2-28
- page scroll up or down* 2-29
- preview report* 7-16, 7-19
- previous screen* 2-28
- primary sort scroll* 7-16, 7-17, 7-19, 7-20
- print graph*
 - Clinical Use 3-33
 - INDUSTRY Use 4-33
- report* 7-21, 7-25
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
- report configuration* 7-14
- report label entry* 7-10
- restart incubation module* 8-16
- sample ID selection*
 - INDUSTRY Use 6-14
- scroll* 2-29
- secondary sort scroll* 7-16, 7-17, 7-18, 7-19, 7-20, 7-21
- select bottle to edit/graph*
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
- set audible alarm options*
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
- set date/time*
 - Clinical Use 3-21
 - INDUSTRY Use 4-21
- set maximum test time*
 - Clinical Use 3-22
 - INDUSTRY Use 4-22
- unload buttons*
 - Clinical Use 3-3, 3-5
 - INDUSTRY Use 4-3, 4-5
- view incubation module information*
 - Clinical Use 3-22
 - INDUSTRY Use 4-22

C

- calibrate cell screen, diagram 8-27
- calibrate module temperature screen, diagram 8-23
- calibration
 - incubation or combination module temperature* 8-25
 - instrument cell* 8-25
 - locating cells which need* 8-26
- calibration data
 - printing* 7-21
 - viewing* 7-21
- calibration history
 - printing* 7-25
 - viewing* 7-25
- cancel button 2-28
- carbon dioxide detection. *See* principle of detection
- cell
 - calibration procedure* 8-25
 - description*
 - combination module 2-18
 - incubation module 2-22
 - enable/disable* 8-18
 - indicator light* 2-18, 2-23
 - viewing readings* 8-26
- cell calibration report screen, diagram 7-23
- cell flag 2-18, 2-23
- cell reading indicator 8-27
- changing maximum test time. *See* editing maximum test time.
- chapter organization 1-6
- check button 2-28
- clear password screen (21 CFR Part 11 mode) 10-16
- codes
 - (1-99) 9-3
 - (710-820) 9-24
 - (901-961) 9-27
 - status determination*
 - Clinical Use 5-11
 - INDUSTRY Use 6-11
- combination module
 - adjusting temperature* 8-21
 - backup drive* 2-16
 - diagram*
 - front view 2-16, 2-17
 - display switch assembly* 2-17, 2-20
 - drawer* 2-18
 - drawer restart procedure* 8-15
 - environmental requirements* 2-21
 - setting optimal temperature of* 8-24
 - specifications* 2-20
 - USB port* 2-18
 - zip drive* 2-18
- configuring report contents 7-13
 - load report configuration screen*
 - Clinical Use 7-16
 - INDUSTRY Use 7-19
 - section break checkboxes* 7-16, 7-17, 7-19, 7-20
 - status report configuration screen*
 - Clinical Use 7-17
 - INDUSTRY Use 7-20
 - unload report configuration screen*
 - Clinical Use 7-18
 - INDUSTRY Use 7-21
- configuring report screens 7-9
- Configuring Users (21 CFR Part 11 Mode)
 - user list* 10-13
- configuring Users (21 CFR Part 11 Mode) 10-12, 10-13
 - add user screen* 10-14
 - adding a user* 10-13
 - clear password screen* 10-16
 - clearing a user password* 10-16
 - clearing more than one user password* 10-17
 - delete user screen* 10-15
 - deleting a user* 10-15
 - deleting more than one user* 10-16
- controller module
 - accessing the keyboard* 2-11
 - backup drive* 2-9
 - diagram*
 - back view 2-12
 - front view 2-9
 - environmental requirements* 2-15
 - shutdown/startup procedure* 8-10
 - specifications* 2-15
 - USB port* 2-10
 - Zip® drive* 2-10
- conventions
 - typographic* 1-7

usage 1-7

D

data entry field
 size limits
 Clinical Use 3-9
 INDUSTRY Use 4-9

delete user screen (21 CFR Part 11 mode) 10-15

disinfection
 spill cleanup 8-3
 spills onto the instrument
 Clinical Use 8-4
 INDUSTRY Use 8-5
 spills within the instrument 8-5

display bottle readings
 Clinical Use 3-34
 INDUSTRY Use 4-34

display switch assembly
 combination module 2-17

drawer
 combination module 2-18
 controller module 2-10
 enable/disable 8-18
 incubation module 2-22
 indicator light 2-19, 2-23
 keyboard 2-10

drawers, mycobacteria (Clinical Use) 2-8

E

edit
 load status
 Clinical Use 5-7
 INDUSTRY Use 6-7

edit bottle detail screen
 fields
 Clinical Use 5-5
 INDUSTRY Use 6-5

edit bottle detail screen, diagram
 Clinical Use 5-5
 INDUSTRY Use 6-5

edit cell contents screen, diagram
 Clinical Use 5-3
 INDUSTRY Use 6-3

edit data relationships screen, diagram

 Clinical Use 5-13
 INDUSTRY Use 6-13

edit test result
 procedure
 Clinical Use 5-9
 INDUSTRY Use 6-9

edit test result screen, diagram
 Clinical Use 5-9
 INDUSTRY Use 6-9

editing
 accession number to hospital ID relationship
 procedure (Clinical Use) 5-16
 screen (Clinical Use) 5-16
 bottle ID to accession number relationship
 procedure (Clinical Use) 5-14
 screen (Clinical Use) 5-14
 bottle ID to sample ID relationship
 procedure (INDUSTRY Use) 6-14
 screen (INDUSTRY Use) 6-14
 hospital ID to patient name relationship
 procedure (Clinical Use) 5-17
 screen (Clinical Use) 5-17
 maximum test time
 Clinical Use 3-15
 INDUSTRY Use 4-14
 report content 7-13
 report labels 7-10
 sample ID to User Defined 3 relationship
 procedure (INDUSTRY Use) 6-16
 screen (INDUSTRY Use) 6-16
 User Defined 3 to User Defined 1/2 relationship
 procedure (INDUSTRY Use) 6-17
 screen (INDUSTRY Use) 6-17

electrical and electronic recycling 2-7

electrical warnings 2-4

enable/disable hardware 8-18

enable/disable module, drawer, rack, or cell screen, diagram 8-19

entering report labels 7-10

entering text into a data entry field
 Clinical Use 3-10
 INDUSTRY Use 4-10

environmental requirements
 combination module 2-21
 controller module 2-15

- incubation module* 2-25
- error codes, operator (901-961) 9-27
- errors
 - bottle problems* 9-39
 - user output device problems* 9-39
- errors, nonsoftware 9-39
- exporting
 - test data*
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
- external monitor
 - setup* 8-14

F

- false positives
 - preventing* C-2
- false positives. *See* unconfirmed positives.
- fault codes (1-99) 9-3
- faults, viewing
 - Clinical Use* 3-6
 - INDUSTRY Use* 4-6
- field, text entry 2-30
- finding topics 1-6
- full system shutdown (Signature only) 8-13
- full system startup (Signature only) 8-14
- function buttons
 - setup screen* 3-21, 4-21

G

- graph bottle readings screen, diagram
 - Clinical Use* 3-33
 - INDUSTRY Use* 4-33
- graphs
 - Clinical Use* 3-31
 - INDUSTRY Use* 4-31
- growth performance
 - quality control* B-4

H

- hardware
 - maintenance* 8-2
 - overview* 2-8
- hazards and precautions

- electrical and electronic recycling* 2-7
- Health Insurance and Portability and Accountability Act (HIPAA) requirements 2-3

I

- icon 2-28
 - backup in progress* 7-9
 - calibration staircase* 8-27
 - instrument*
 - Clinical Use 3-4
 - INDUSTRY Use 4-4
 - instrument with fault codes* 9-3
 - load bottles*
 - Clinical Use 3-12
 - INDUSTRY Use 4-12
 - padlock* 7-6
 - unload bottles*
 - Clinical Use 3-17
 - INDUSTRY Use 4-17
 - view algorithm/polynomial*
 - Clinical Use 5-10
 - INDUSTRY Use 6-10
 - view cell location*
 - Clinical Use 5-8
 - INDUSTRY Use 6-8
 - view first loaded time*
 - Clinical Use 5-8
 - INDUSTRY Use 6-8
 - view how determined/positivity index*
 - Clinical Use 5-11
 - INDUSTRY Use 6-11
 - view last unloaded time*
 - Clinical Use 5-8
 - INDUSTRY Use 6-8
 - view test result*
 - Clinical Use 5-9
 - INDUSTRY Use 6-9
 - view test time*
 - Clinical Use 5-8
 - INDUSTRY Use 6-8
 - view time of last bottle reading*
 - Clinical Use 5-8
 - INDUSTRY Use 6-8
- inactivity timeout (21 CFR Part 11 mode) 10-7
- incubation module
 - adjusting temperature* 8-21
 - diagram*
 - back view 2-23

- front view 2-22
- environmental requirements 2-25
- restart procedure 8-15
- setting optimal temperature of 8-24
- specifications 2-25
- viewing information for 8-29
- indicator light
 - cell 2-18, 2-23
 - drawer 2-19, 2-23
- Installation
 - instrument
 - INDUSTRY Use 2-26
- installation instructions
 - INDUSTRY Use 2-26
- instrument fault codes 9-3
- instrument icon
 - Clinical Use 3-4
 - INDUSTRY Use 4-4
- instrument status codes 9-24
- internal monitor
 - restore settings 8-14
- international character
 - entering A-4
 - entry A-2
 - table A-2
- Introduction
 - chapter organization 1-6
 - finding topics 1-6

K

- keyboard
 - accessing the controller module 2-11
 - description (combination module) 2-18
 - drawer (controller module) 2-10
 - entering text into a data entry field
 - Clinical Use 3-10
 - INDUSTRY Use 4-10
 - using in place of the operator panel 8-8

L

- light-emitting diode (LED) 2-4
- LIS, sending and receiving information
 - Clinical Use 3-39
 - INDUSTRY Use 4-39
- load report configuration screen, diagram

- Clinical Use 7-16
- INDUSTRY Use 7-19
- loading bottles
 - Clinical Use 3-11
 - INDUSTRY Use 4-11
- locating a cell which failed calibration 8-26
- log In/Out of System (21 CFR Part 11 Mode)
 - change password 10-8
 - change password button 10-3, 10-4
 - change password errors 10-9
 - confirm password field 10-4
 - inactivity timeout 10-7
 - logging in 10-5
 - logging in for the first time 10-2
 - logging out 10-12
 - login button 10-2
 - login errors 10-7
 - main screen while logged in 10-5
 - main screen while logged out 10-2
 - new password field 10-4
 - user login screen 10-3
- log In/out of system (21 CFR Part 11 mode) 10-2

M

- main screen
 - diagram
 - Clinical Use 3-3
 - INDUSTRY Use 4-3
 - diagram (load mode)
 - Clinical Use 3-12
 - INDUSTRY Use 4-12
 - diagram (unload mode)
 - Clinical Use 3-17
 - INDUSTRY Use 4-16
- maintenance
 - hardware 8-2
 - preventative 8-2
 - software 8-15
 - system 8-1
- Manual
 - standard symbols B-4, C-2, C-4
- manual
 - standard symbols B-4, C-2, C-4
- maximum test time
 - changing for an individual bottle
 - Clinical Use 3-15

- INDUSTRY Use 4-14
- setting defaults* 7-2
- mode, 21 CFR Part 11 10-1
- module
 - enable/disable* 8-18
- monitor
 - background color*
 - Clinical Use 3-4
 - INDUSTRY Use 4-4
- moving a bottle ID association from one accession number to another
 - Clinical Use* 5-15
- moving a bottle ID association from one sample number to another
 - INDUSTRY Use* 6-15
- moving a sample ID to User Defined 3
 - INDUSTRY Use* 6-17
- moving an accession number association from one hospital ID to another
 - Clinical Use* 5-17
- moving bottles. *See* relocating bottles.
- mycobacteria drawers (Clinical Use) 2-8
- mycobacterial testing
 - disinfection procedures*
 - Clinical Use 8-4
 - INDUSTRY Use 8-5
 - installation procedures and special requirements* 11-4
 - safety features* 11-8
 - spill cleanup* 8-3
 - system description* 11-2
 - system startup* 11-6
 - theory of operation* 11-7

N

- next screen button 2-28
- nonsoftware errors 9-39

O

- operator error codes 9-27
- operator panel
 - description (combination module)* 2-17
 - description (controller module)* 2-9

- screens* 2-27
- optimal temperature setting 8-24

P

- password
 - changing* 7-5
 - entry*
 - Clinical Use 3-20
 - INDUSTRY Use 4-20
- power down procedure . *See* Controller Module, reboot/shutdown procedure
- power up procedure. *See* Controller Module, startup procedure
- preventative maintenance 8-2
 - safety precautions and procedures* 8-2
- preventing false positives C-2
- previous screen button 2-28
- principle of detection 2-4
- printing
 - calibration data* 7-21
 - calibration history* 7-25
 - test data*
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
 - using the print screen function*
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
- printing bottle graphs
 - Clinical Use* 3-31
 - INDUSTRY Use* 4-31

Q

- quality control of growth performance B-4
 - blood culture bottles* B-5
 - culture bottles* B-5
 - mycobacteria culture bottles* B-5
- quick reference card 2-10

R

- rack
 - enable/disable* 8-18
- rebooting computer. *See* Controller Module, reboot/shutdown procedure.
- rebooting computer. *See* Controller Module,

startup procedure.

recycling
 electrical and electronic 2-7

reference card. See quick reference card.

relocating bottles 8-21

report configuration screen, diagram
 load report
 Clinical Use 7-16
 INDUSTRY Use 7-19
 status report
 Clinical Use 7-17
 INDUSTRY Use 7-20
 unload report
 Clinical Use 7-18
 INDUSTRY Use 7-21

report field descriptions 7-12

report label entry screen, diagram 7-11

report selection screen, diagram 7-10, 7-22, 7-25

reports
 configuring content 7-13
 entering labels 7-10
 reportable fields 7-12

requesting bottle demographics from LIS
 Clinical Use 3-39
 INDUSTRY Use 4-39

requirements
 21 CFR Part 11 2-3
 HIPAA 2-3

S

safety precautions and procedures 8-2

sample ID
 attaching to a User Defined 3
 INDUSTRY Use 6-16
 moving from one User Defined 3 to another
 INDUSTRY Use 6-17

samples
 hazards and precautions 1-8, B-4, C-2, C-4

screen
 add user (21 CFR Part 11 mode) 10-14
 backup management 7-8
 bottle readings
 Clinical Use 3-35
 INDUSTRY Use 4-34

calibrate cell 8-27

calibrate module temperature 8-23

cell calibration report 7-23

change maximum test time (individual bottle)
 Clinical Use 3-15
 INDUSTRY Use 4-15

change password 7-6

clear password (21 CFR Part 11 mode) 10-16

colors determined by configuration 2-3

delete user (21 CFR Part 11 mode) 10-15

edit accession number to hospital ID relationships (Clinical Use) 5-16

edit bottle detail
 Clinical Use 5-5
 INDUSTRY Use 6-5

edit bottle ID to accession number relationships (Clinical Use) 5-14

edit bottle ID to accession number relationships (INDUSTRY Use) 6-14

edit cell contents
 Clinical Use 5-3
 INDUSTRY Use 6-3

edit data relationships
 Clinical Use 5-13
 INDUSTRY Use 6-13

edit hospital ID to patient name relationships (Clinical Use) 5-17

edit sample ID to User Defined 3 relationships (INDUSTRY Use) 6-16

edit test result
 Clinical Use 5-9
 INDUSTRY Use 6-9

edit User Defined 3 to User Defined 1/2 relationships (INDUSTRY Use) 6-17

enable/disable module, drawer, rack, or cell 8-19

graph bottle readings
 Clinical Use 3-33
 INDUSTRY Use 4-33

load report configuration
 Clinical Use 7-16
 INDUSTRY Use 7-19

main
 Clinical Use 3-3
 INDUSTRY Use 4-3

main (load mode)
 Clinical Use 3-12
 INDUSTRY Use 4-12

main (unload mode)
 Clinical Use 3-17
 INDUSTRY Use 4-16

- main screen while logged out (21 CFR Part 11 Mode)* 10-2
- mains screen while logged in (21 CFR Part 11 Mode)* 10-5
- report label entry* 7-11
- report selection* 7-10, 7-14, 7-22, 7-25
- select bottle to edit/graph*
 - Clinical Use 3-32
 - INDUSTRY Use 4-31
- set audible alarm options* 7-4
- set maximum test time* 7-2
- setup*
 - Clinical Use 3-20
 - INDUSTRY Use 4-20
- status report configuration*
 - Clinical Use 7-17
 - INDUSTRY Use 7-20
- unload report configuration*
 - Clinical Use 7-18
 - INDUSTRY Use 7-21
- user configuration (21 CFR Part 11 mode)* 10-13
- user login (21 CFR Part 11 mode)* 10-3
- view cell status*
 - Clinical Use 3-7
 - INDUSTRY Use 4-7
- view incubation module information* 8-30
- screen ID numbers 2-27
- scroll button 2-29
- select bottle to edit/graph screen, diagram
 - Clinical Use 3-32
 - INDUSTRY Use 4-31
- selecting a bottle
 - via edit cell contents screen*
 - Clinical Use 5-3
 - INDUSTRY Use 6-3
 - via select bottle to edit/graph screen*
 - Clinical Use 5-4
 - INDUSTRY Use 6-4
- sending test results to LIS
 - Clinical Use 3-39
 - INDUSTRY Use 4-39
- set maximum test time screen, diagram 7-2
- setting optimal temperatures 8-24
- setting system date/time 8-17
- setup instructions
 - INDUSTRY Use 2-26
- setup screen
 - accessing*
 - Clinical Use 3-20
 - INDUSTRY Use 4-20
 - function buttons* 3-21, 4-21
- setup screen, diagram
 - Clinical Use 3-20
 - INDUSTRY Use 4-20
- shutdown/startup procedure. See Controller Module, shutdown/startup procedure.
- shutting down the system (Signature only) 8-13
- slidebar switch 2-29
 - edit load status*
 - Clinical Use 5-7
 - INDUSTRY Use 6-7
- software
 - configuration* 7-1
 - configuration options* 2-2
 - background color
 - Clinical Use 3-4
 - INDUSTRY Use 4-4
 - maintenance* 8-15
- software overview 2-27
- specifications
 - combination module* 2-20
 - controller module* 2-15
 - incubation module* 2-25
- spills
 - cleanup* 8-3
 - onto instrument*
 - Clinical Use 8-4
 - INDUSTRY Use 8-5
 - within instrument* 8-5
- starting up the system (Signature only) 8-14
- status codes, instrument (710-820) 9-24
- status determination codes
 - Clinical Use 5-11
 - INDUSTRY Use 6-11
- status report configuration, diagram
 - Clinical Use 7-17
 - INDUSTRY Use 7-20
- system
 - maintenance* 8-1
 - monitoring*
 - Clinical Use 3-3
 - INDUSTRY Use 4-3
 - setting date/time* 8-17

- setup instructions*
 - INDUSTRY Use 2-26
- troubleshooting* 9-1
- system backup
 - USB flash drive 7-7
 - zip drive 7-7
- system configuration
 - Select 2-2
 - SelectLink 2-3
 - Signature 2-3
- system shutdown. See full system shutdown.
- system startup. See full system startup.

T

- temperature
 - adjusting incubation or combination module 8-21
 - setting optimal 8-24
- test data
 - viewing, printing and exporting
 - Clinical Use 3-24
 - INDUSTRY Use 4-24
- text entry field
 - description 2-30
 - size limits
 - Clinical Use 3-9
 - INDUSTRY Use 4-9
- theory of operation 2-4
- thermometer
 - location 2-19, 2-22
- tips
 - backup 7-8
 - entering bottle type
 - Clinical Use 3-13
 - INDUSTRY Use 4-13
- typographic conventions 1-7
 - name and titles 1-7
 - press 1-7
 - procedural steps 1-7
 - references 1-7
 - select 1-8
 - user input
 - literal use 1-8
 - placeholder 1-8
 - warnings and cautions 1-8

U

- unconfirmed positives
 - Clinical Use 3-19
 - INDUSTRY Use 4-19
- unload report configuration, diagram
 - Clinical Use 7-18
 - INDUSTRY Use 7-21
- unloading bottles
 - Clinical Use 3-16
 - INDUSTRY Use 4-16
- unresponsive operator panel 9-41
- UPS
 - orientation diagram 8-9
- usage conventions 1-7
- USB flash drive
 - system backup 7-8
- USB port
 - combination module 2-16
 - controller module 2-10
- user configuration screen 10-13
- user list (21 CFR Part 11 mode) 10-13
- user output device problems 9-39

V

- view cell status screen
 - diagram
 - Clinical Use 3-7
 - INDUSTRY Use 4-7
- view incubation module information screen, diagram 8-30
- viewing
 - bottle counts
 - Clinical Use 3-5
 - INDUSTRY Use 4-5
 - bottle data
 - Clinical Use 3-23
 - INDUSTRY Use 4-23
 - bottle graphs
 - Clinical Use 3-31
 - INDUSTRY Use 4-31
 - calibration data 7-21
 - calibration history 7-25
 - faults
 - Clinical Use 3-6
 - INDUSTRY Use 4-6

incubation module information 8-29

test data

Clinical Use 3-24

INDUSTRY Use 4-24

Z

Zip® drive

combination module 2-16

controller module 2-10