

A Dräger and Siemens Company

Operator's Instruction Manual

Part Number: 4117966-003 Rev: -Date: 30 May 2004 © 2004 Draeger Medical, Inc.

Integrated Patient Monitor Option for the Narkomed 6400 Anesthesia Machine

WARNING: For a full understanding of the performance characteristics of this equipment, the user should carefully read this manual before operating.

Section 1. Introduction

Dperator's Responsibility for Patient Safety	1-2
imitation of Liability	1-2
Restriction	1-3
Copyright	1-3
rademark Notices	1-3
Disclaimer	1-3
Recommendations	1-4
Symbol Definition	1-4
fow This Manual Is Organized	1-4
Conventions Used in This Manual	1-5
Typefaces	1-5
Warnings and Cautions.	1-5
General Safety Information	1-6
Seneral Warnings and Cautions	1-6

Section 2. General Description

Integrated Patient Monitor (IPM) Option Overview	2-2
IPM Interface Panel	2-2
IPM Control Keys	2-4
Monitoring Information.	2-5
Display Colors	2-5
Screen Configuration	2-5
Configuring the Screen Display.	2-7
Trends	-10
Data Log	-14
Printing IPM Data	-18
Templates	-19
Alarms	-20
Replacing the IPM Module	-20

Section 3. ECG Monitoring/ST Segment Analysis

Setting Up ST Segment Analysis Parameters	3-17
ST Segment Analysis Setup Tab	3-17
Summary of Alarms	3-18
Problem Resolution	3-19
Care and Cleaning	3-20

Section 4. IBP Monitoring

verview	2
P Display	2
reparing for IBP Monitoring	7
eroing the Transducer	7
etting Up IBP Parameters	8
IBP Setup Tab	8
IBP Alarms Tab	0
IBP Site-Scale Tab	2
IBP Wedge Tab	5
IBP Cardiac CalcsTab	8
ummary of IBP Alarms	9
roblem Resolution	0

Section 5. Cardiac Output Monitoring

Overview
Cardiac Output Display
Preparing for Cardiac Output Monitoring5-3
Suggested Cardiac Output Procedure5-5
Setting Up Cardiac Output Parameters
Cardiac Output CO Trials Tab
Cardiac Output Alarms Tab
Cardiac Output Site-Scale Tab
Cardiac Output Cardiac Calcs Tab
Summary of Cardiac Output Alarms5-19
Problem Resolution
Care and Cleaning

Section 6. Temperature Monitoring

Overview
Temperature Display
Preparing the Patient for Temperature Monitoring
Setting Up Temperature Parameters
Temperature Setup Tab
Temperature Alarms Tab
Temperature Site-Scale Tab6-8
Summary of Temperature Alarms
Problem Resolution
Care and Cleaning

Section 7. NIBP Monitoring

Overview	.7-2
NIBP Display	.7-2
Patient Preparation for NIBP Monitoring	.7-4
Selecting the Blood Pressure Cuff	.7-4
Placing the Cuff	.7-5
Setting Up NIBP Parameters	.7-6
NIBP Setup Tab	.7-6
NIBP Alarms Tab	.7-8
NIBP Volume Tab	7-10
Summary of NIBP Alarms	7-11
Problem Resolution	7-13
Care and Cleaning	7-13

Section 8. SpO2 Monitoring

Dverview
SpO2 Display
Preparing for SpO2 Monitoring
Applying the Pulse Oximeter Sensor
Setting Up SpO2 Parameters
SpO2 Setup Tab
SpO2 Alarms Tab
Volume Tab
Summary of SpO2 Alarms
Problem Resolution

Section 9. Specifications

eneral
nvironmental
Storage
Operating
ectrical
CG Monitoring
vasive Blood Pressure Monitoring
ardiac Output Monitoring
emperature Monitoring
oninvasive Blood Pressure Monitoring
Ilse Oximetry Monitoring

Appendix 1: Spare and Replacement Parts

Manuals	A-1-2
ECG Monitoring Accessories.	A-1-2
Invasive Blood Pressure Monitoring Accessories	A-1-2
Cardiac Output Monitoring Accessories	A-1-2
Temperature Monitoring Accessories	A-1-3
Noninvasive Blood Pressure Monitoring Accessories	A-1-3
SpO2 Monitoring Accessories	A-1-3
Miscellaneous	A-1-3

Appendix 2: Tables

Factory Default Settings	A-2-2
Catheters/Settings Supported by the Narkomed 6400	A-2-7

Introduction

This section introduces you to the Integrated Patient Monitor Option Operator's Manual and provides important safety information.

Operator's Responsibility for Patient Safety	1-2
Limitation of Liability	1-2
Restriction	1-3
Copyright	1-3
Trademark Notices	1-3
Disclaimer	1-3
Recommendations	1-4
Symbol Definition	1-4
How This Manual Is Organized	1-4
Conventions Used in This Manual	1-5
General Safety Information	1-6
General Warnings and Cautions	1-6

Operator's Responsibility for Patient Safety

Draeger Medical anesthesia products are designed to provide the greatest degree of patient safety that is practically and technologically feasible. The design of the equipment, the accompanying literature, and the labeling on the equipment take into consideration that the purchase and use of the equipment are restricted to trained professionals, and that certain inherent characteristics of the equipment are known to the trained operator. Instructions, warnings, and caution statements are limited, therefore, to the specifics of the Draeger Medical design. This publication excludes references to hazards that are obvious to a medical professional, to the consequences of product misuse, and to potentially adverse effects in patients with abnormal conditions. Product modification or misuse can be dangerous. Draeger Medical disclaims all liability for the consequences of product alterations or modifications, as well as for the consequences that might result from the combination of Draeger Medical products with products supplied by other manufacturers if such a combination is not endorsed by Draeger Medical.

The operator of the anesthesia system must recognize that the means of monitoring and discovering hazardous conditions are specific to the composition of the system and the various components of the system. It is the operator, and not the various manufacturers or suppliers of components, who has control over the final composition and arrangement of the anesthesia system used in the clinical practice. Therefore, the responsibility for choosing the appropriate safety monitoring devices rests with the operator and user of the equipment.

Patient safety may be achieved through a variety of different means depending on the institutional procedures, the preference of the operator, and the application of the system. These means range from electronic surveillance of equipment performance and patient condition to simple, direct contact between operator and patient (direct observation of clinical signs). The responsibility for the selection of the best level of patient monitoring belongs solely to the equipment operator. To this extent, the manufacturer, Draeger Medical, disclaims responsibility for the adequacy of the monitoring package selected for use with the anesthesia system. However, Draeger Medical is available for consultation to discuss monitoring options for different applications.

Limitation of Liability

Draeger Medical's liability, whether arising from or related to the manufacture and sale of the products, their installation, demonstration, sales representation, use, performance, or otherwise, including any liability based upon Draeger Medical's product warranty, is subject to and limited to the exclusive terms of Draeger Medical's limited warranty, whether based upon breach of warranty or any other cause of action

1

whatsoever, regardless of any fault attributable to Draeger Medical and regardless of the form of action (including, without limitation, breach of warranty, negligence, strict liability, or otherwise).

Draeger Medical shall in no event be liable for any special, incidental, or consequential damages (including loss of profits) whether or not foreseeable and even if Draeger Medical has been advised of the possibility of such loss or damage. Draeger Medical disclaims any liability arising from a combination of its product with products from another manufacturer if the combination has not been endorsed by Draeger Medical. Buyer understands that the remedies noted in Draeger Medical's limited warranty are its sole and exclusive remedies.

Furthermore, buyer acknowledges that the consideration for the products, equipment, and parts sold reflects the allocation of risk and the limitations of liability referenced herein.

Restriction

Federal law restricts this device to sale by, or on the order of, a physician.

Copyright

Copyright 2004 by Draeger Medical, Inc. All rights reserved. No part of this publication may be reproduced, transmitted, transcribed, or stored in a retrieval system in any form or by any means, electronic or mechanical, including photocopying and recording, without written permission of Draeger Medical, Inc.

Trademark Notices

Datagrip, DrägerService, Narkomed, Narkomed GS, ORM, Quality Service For Life, Respitone, Vigilance Audit, Vitalert, and Vitalink are registered trademarks of Draeger Medical, Inc. All other products or name brands are trademarks of their respective owners.

Disclaimer

The content of this manual is furnished for informational use only and is subject to change without notice. Draeger Medical, Inc. assumes no responsibility or liability for any errors or inaccuracies that may appear in this manual.

Recommendations

In the interest of patient safety, Draeger Medical strongly advocates the use of an oxygen analyzer, pressure monitor, a volume monitor, and an end-tidal CO_2 monitor in the breathing circuit at all times.

Because of the sophisticated nature of Draeger Medical anesthesia equipment and its critical importance in the clinical setting, it is highly recommended that only appropriately trained and experienced professionals be permitted to service and maintain this equipment. Please contact DrägerService at (800) 543-5047 for service of this equipment.

Draeger Medical also recommends that its anesthesia equipment be serviced at three-month intervals. Periodic Manufacturer's Service Agreements are available for equipment manufactured by Draeger Medical. For further information concerning these agreements, contact DrägerService at (800) 543-5047.

Symbol Definition

The following symbols appear on the Integrated Patient Monitor:



CAUTION: Refer to accompanying documents before operating equipment.



Defibrillator-proof type BF equipment. Type BF equipment is suitable for intentional external and internal application to the patient, excluding direct cardiac application. Type BF equipment is type B equipment with an F-type isolated (floating) part. The paddles indicate the equipment is defibrillator-proof.



Defibrillator-proof type CF equipment. Type CF equipment is specifically designed for applications where a conductive connection directly to the heart is established. The paddles indicate the equipment is defibrillator-proof.

How This Manual Is Organized

All users of the Integrated Patient Monitor must read this manual completely before using the equipment. This manual is intended to be used in conjuction with the *Narkomed 6400 Anesthesia System Operator's Instruction Manual*.

To make this document more convenient for future reference, it is divided into several independent sections.

Section 2 - "General Description" provides a summary of Integrated Patient Monitor features and functions.

Introduction

1

Section 3 - "ECG Monitoring/Segment Analysis" contains detailed configuration and operational information specific to ECG monitoring and ST segment analysis.

Section 4 - "IBP Monitoring" contains detailed configuration and operational information specific to IBP monitoring.

Sections 5 - "Cardiac Output Monitoring" contains detailed configuration and operational information specific to cardiac output monitoring.

Section 6 - "Temperature Monitoring" contains detailed configuration and operational information specific to Temperature Monitoring.

Section 7 - "NIBP Monitoring" contains detailed configuration and operational information specific to NIBP Monitoring.

Section 8 - " SpO_2 Monitoring" contains detailed configuration and operational information specific to SpO_2 Monitoring.

Section 9 - "Specifications" lists the specifications for all system components.

Appendix 1 contains a list of spare and replacement parts.

Appendix 2 contains template tables which list the settings that can be stored in templates.

Conventions Used in This Manual

This manual has several conventions to help organize the information presented. Please read about these conventions carefully to understand their significance in the manual.

Typefaces Different typefaces are used throughout the manual to differentiate between narrative information and machine messages and labels.

Warnings and
CautionsAll parts of this manual contain warning and caution statements about the
Integrated Patient Monitor.

- *Warning* statements give important information that, if ignored, could lead directly to personal injury.
- *Caution* statements give important information that, if ignored, could lead directly to equipment damage and indirectly to personal injury.

General Safety Information

The Integrated Patient Monitor is protected against the effect of cardiac defibrillator discharge to ensure proper recovery, as required by test standards. (The screen may blank during a defibrillator discharge but recovers within seconds as required by test standards.)

The Integrated Patient Monitor is not likely to cause abnormal operation of other patient-connected equipment such as a cardiac pacemaker or other electrical stimulators.

Periodically, and whenever the integrity of the Integrated Patient Monitor is in doubt, connect a patient simulator and test all functions.

General Warnings and Cautions

The following list of warnings and cautions apply to general operation and maintenance of the Integrated Patient Monitor. Warnings and cautions about installing and operating specific parts appear with those topics.

- **WARNING:** Any person involved with the setup, operation, or maintenance of the Integrated Patient Monitor must be thoroughly familiar with this instruction manual. However, instructions in this manual in no way supersede established medical procedures for patient care.
- **WARNING:** The Integrated Patient Monitor is designed to be operated under the constant surveillance of a qualified operator.
- **WARNING:** Connecting to auxiliary output to remain within risk current limits, connect only to devices tested and certified for use in patient environment.
- **WARNING:** The Integrated Patient Monitor must not be used in the presence of flammable anesthetics.
- **WARNING:** Repair of the Integrated Patient Monitor must only be performed by an authorized representative of DrägerService[®].
- **WARNING:** A test for leakage current must be performed by qualified biomedical engineering personnel before use if the Integrated Patient Monitor is interfaced with other equipment.
- **WARNING:** To avoid electrical shock hazard:
 - Due to the risk of electric shock, do not remove any component cover. Refer any servicing to an authorized representative of DrägerService.
 - Use only hospital-grade grounded electrical outlets and power cord.

Part Number: 4117966-003 Rev: -

1

- Make sure the external equipment is hospital-grade grounded before connecting the equipment.
- Disconnect the power supply of the Narkomed 6400 from the electrical outlet before cleaning. Let it dry completely before reconnecting it to the electrical outlet.

WARNING: To ensure patient safety:

- This device should be used by, or on the order of, a physician.
- Constant attention by a qualified professional is needed whenever a patient is under anesthesia or connected to a ventilator. Some equipment malfunctions may pass unnoticed in spite of the monitor alarm.
- Always make sure that alarm limits are set and alarms are active when monitoring a patient. Do not rely exclusively on the audible alarm system for patient monitoring. Adjusting the alarm volume to a low level during patient monitoring can jeopardize the patient.
- Keep pacemaker patients under close surveillance. Rate meters may continue to count pacemaker rate during incidents of cardiac arrest or some arrhythmias. Do not rely solely on rate meter alarms.
- Use only protected lead wires and patient cables with the Integrated Patient Monitor. Using unprotected lead wires and patient cables creates the potential for making an electrical connection to a high voltage power source that can cause serious injury or death to the patient.
- Make sure the return electrode of the electrosurgery unit is properly connected to the patient to avoid possible burns at ECG electrode or other probe sites. Avoid making any conductive connection to the patient that is likely to affect the patient's safety.
- Use only patient cables and accessories approved by Draeger Medical. Other cables and accessories may not provide defibrillation protection, protection against high frequency burns, or can damage the system and interfere with measurement.
- Do not come into contact with the Integrated Patient Monitor or patient cables while a patient is being defibrillated. Serious injury or death could result.
- Proper placement of defibrillator paddles in relation to the electrodes is required to minimize harm to the patient.

- Route all cables away from the patient's throat to avoid possible strangulation.
- Exercise extreme care when applying medical electrical equipment. Many parts of the man/machine circuit are conductive, such as the patient, connectors, electrodes, and transducers. Ensure that these conductive parts, when connected to the isolated patient input of the device, do not come into contact with other grounded, conductive parts. Such contact would bridge the patient's isolation and cancel the protection provided by the isolated input.
- All invasive procedures involve patient risk. Use aseptic technique. Follow catheter manufacturer's instructions.
- Do not perform cardiac output measurement during electrosurgery. Incorrect readings can result.
- If the accuracy of any value display is in doubt, first determine the patient's vital signs by alternate means before verifying that the Integrated Patient Monitor is working correctly.
- If the display loses patient data, it is possible that active monitoring is not being performed. Close patient observation or alternate monitoring devices should be used until monitor function is restored.
- **WARNING:** If fluids are accidentally spilled on the Integrated Patient Monitor, take the Narkomed 6400 temporarily out of service by putting the machine in monitor standby mode and unplugging the main power cord from the AC outlet. Refer to "Powering Down the Narkomed 6400 for Cleaning" in the "General Care and Maintenance" section of the Narkomed 6400 Operator's Instruction Manual.
- **CAUTION:** When moving equipment connected to the Integrated Patient Monitor, disconnect the connection to the Integrated Patient Monitor and move the equipment separately.
- **CAUTION:** Although designed to minimize the effects of ambient radiofrequency interference, the Integrated Patient Monitor may be adversely affected by the operation of electrosurgical equipment or shortwave or microwave diathermy equipment in the vicinity. To prevent ESU overload, use the leads and cables specified in "Spare and Replacement Parts" in the Appendix in the back of this manual.

Introduction

1

- **CAUTION:** Patient signal inputs labeled with the CF and BF symbols with paddles are protected against damage resulting from defibrillation and electrocautery voltages.
- **CAUTION:** Proper placement of defibrillator paddles in relation to the electrodes is required to minimize harm to the patient.
- **CAUTION:** Disposable devices are intended for single use only. They should not be reused as performance could degrade or contamination could occur.

Part Number: 4117966-003 Rev: - Integrated Patient Monitor Option Operator's Manual

This section describes Integrated Patient Monitor Option features and general functionality.

Integrated Patient Monitor (IPM) Option Overview 2-2
IPM Interface Panel
Monitoring Information 2-5
Display Colors 2-5
Screen Configuration 2-5
Configuring the Screen Display 2-7
Trends 2-10
Data Log 2-14
Printing IPM Data 2-18
Templates 2-19
Alarms 2-20
Replacing the IPM Module 2-20

Integrated Patient Monitor (IPM) Option Overview

The IPM is a measurement module built into the side of the Narkomed 6400. It sends real-time patient measurement data to the Narkomed 6400 which integrates all the monitoring information and alarms for display on its main screen.

The IPM Option provides the following patient monitoring functions:

- seven-lead ECG with ST segment analysis
- four invasive blood pressure channels
- thermodilution cardiac output
- two temperature channels
- one noninvasive blood pressure (NIBP) channel
- one pulse oximetry (SpO_2) channel

The IPM module can be replaced while the Narkomed 6400 system is operating. The system will be fully functional within 60 seconds after the module is installed without any further intervention by the clinician. For replacement instructions, see "Replacing the IPM Module" later in this section of the manual.

IPM Interface Panel

The IPM interface panel contains connectors, control keys, and indicator lights for the various monitoring functions. All control key functions except **[ZERO ALL]** can also be accessed via the main touch screen.

The interface panel is shown in Figure 2-1.

General Description

Figure 2-1. IPM Interface Panel



IPM Control	The functions of the control keys on the IPM interface panel are as follows:
Keys	

ZERO	
ALL	

The **[ZERO ALL]** key zeroes all invasive pressure channels simultaneously.



The **[VOL]** key increases or decreases the volume of the pulse tone.



The **[ALARM SUSPEND]** key stops the system from displaying and sounding patient alarms. It provides the same function as the **[ALARM SUSPEND]** control button on the Narkomed 6400 screen.



The [START/STOP] key is for NIBP operation:

- If NIBP operation is suspended, pressing this key starts an NIBP measurement and will continue to take measurements at a preset time interval (Auto Mode)
- If NIBP is started, pressing this key stops measurement, and further NIBP operation is suspended.

STAT

Pressing the **[STAT]** key begins the process of taking successive NIBP samples continuously for five minutes. After five minutes, samples are taken in the time interval set in the NIBP notebook.

General Description

2

Monitoring Information

Patient information obtained from the IPM is displayed in the same numerical and graphical format as the other Narkomed 6400 information.

This manual provides information specific to monitoring functions provided by the IPM. For information on all other Narkomed 6400 monitoring functions as well as complete information on screen displays and the use of the Narkomed 6400 touch screen, see the *Narkomed 6400 Anesthesia System Operator's Instruction Manual*.

DisplayColors in the displays are not configurable by the clinician. Consistent
colors have been programmed to appear on labels, numeric values,
waveforms, and trends for a given parameter. The colors for the
parameters monitored by the IPM are shown in Table 2-1.

Parameter	Color
ECG	green
ST Segment	white
IBP channel 1	red
IBP channel 2	blue
IBP channel 3	yellow
IBP channel 4	white
Cardiac Output	red
Temperature	white
NIBP	white
SpO2	blue

Table 2-1. Display Colors for IPM Parameters

Screen Configuration

When the Narkomed 6400 is configured with an IPM, the screen configuration, which includes the number of waveforms and location of parameter boxes, is determined by the availability of data for each parameter (i.e., measurement cable connected or not connected). The screen display is reconfigured automatically, without input from the clinician.

The five-waveform screen is the standard Narkomed 6400 screen display without any monitoring information from the IPM. On machines with an installed IPM, the number of waveforms and vertical parameter boxes will vary from five to eight, depending on available parameters. In addition, up to seven horizontal parameter boxes can be displayed along the bottom of the waveform area. The location of parameter boxes can vary, and if data is

unavailable, the contents of some boxes may be blank or the boxes may be removed from the screen altogether.

Note: Note the following with six, seven, and eight-waveform screen configurations:

- the size of the numerics in the parameter boxes *decreases* when the alarm limits are displayed and *increases* when alarm limits are *not* displayed.
- the label for the breathing pressure parameter box is Paw (instead of Pressure in the five-waveform screen)

See Figure 2-2 and Figure 2-3 for examples of two possible screen configurations:

Figure 2-2. Screen Display with IPM Information (least possible number of waveforms and parameter boxes)



General Description

2



Figure 2-3. Screen Display with IPM Information (greatest possible number of waveforms and parameter boxes)

Configuring
the Screen
DisplayThe clinician can choose to display only respiratory information (no IPM
data) or to allow the display of IPM data configured automatically by the
Narkomed 6400.

The screen display as well as the trace speed are set in the traces page of the system setup notebook.

To display the traces page:

- 1. Touch the **[Setup Notebook]** button in the main screen taskbar to display the system setup notebook.
- 2. Touch the **[Traces]** tab to view the traces page.

Figure 2-4. System Setup Traces Page



Cardio- vascular Waveforms	This option allows the clinician to select the trace speed of the cardiovascular waveforms. Touching [Cardiovascular Waveforms] or its associated selection field toggles the trace speed between 25 mm/sec and 50 mm/sec.
	Trace speed for all other waveforms is selected using the [Respiratory Waveforms] button.
Selecting the Screen	The screen configuration options are shown in the bottom half of the traces page under the heading Screen Config (see Figure 2-4).
Configuration	To select the screen configuration:
	• Touch [Respiratory Information] to display the standard Narkomed 6400 screen with no IPM-related information.
	• Touch [Automatic] to display all available parameters. The screen is updated to include available IPM-related information as well as all standard respiratory information.
	Note: Screen configuration changes are not allowed while a cardiac output or an NIBP measurement is in progress.

2

Following the screen configuration change, most user settings will remain unchanged. Alarms active prior to the screen configuration change will remain active, with the exception that all IPM-related alarms will be suppressed when the screen configuration is changed to the Respiratory Information Only display.

Locking Position of Agent and Volume Parameter Boxes

The positions of parameter boxes on the screen are normally determined automatically by the Narkomed 6400. However, controls are provided for the clinician to lock the position of the agent and volume parameter boxes in a horizontal orientation.

Note: Locking the position of the agent or volume parameter box will not be allowed if doing so would result in a screen configuration of fewer than five waveforms.

To lock the agent and/or volume parameter box in a horizontal position:

- 1. Touch the agent or volume parameter box anywhere except the alarm bell to display the parameter notebook.
- 2. Touch the **[Setup]** tab to display the setup page. See Figure 2-5 and Figure 2-6.
- 3. Touch **[Numbox Location]** or its associated selection field until the preferred setting is displayed.
 - Horizontal locks the parameter box in a horizontal position
 - Auto allows the position of the parameter box to be determined by the Narkomed 6400

General Description



Figure 2-5. Setup Page in Agent Parameter Notebook Figure 2-6. Setup Page in Volume Parameter Notebook

Trends

The trend display contains data for those parameters that are displayed in the current screen configuration and for which data is currently available. Up to six pages of trend data can be displayed, including a custom trend page configured by the clinician. Table 2-2 shows the parameters displayed on each trend page (if available), along with corresponding colors and scale limits.

Note: If some trends do not fit on a particular trend page, they will moved to the next page.

To display the trends:

- 1. Touch the [Trend] button in the main screen taskbar.
- 2. Touch the displayed trend window to bring up the trend dialog box.
- 3. Touch the **[Next]** or **[Previous]** button in the dialog box to view other trend pages.

For complete information on trends, see the Narkomed 6400 Anesthesia System Operator's Instruction Manual.

General Description

2

Table 2-2. Trend Display with IPM Parameters

Parameter	Color	Scale min/max displayed in trend
Page 1		
Heart Rate	Green	0/200 bpm
ST (if available)	White	-12/12 mm*
IBP Channel 1 (if available)	Red	0/200 mm Hg
IBP Channel 2 (if available)	Blue	0/200 mm Hg
IBP Channel 3 (if available)	Yellow	0/200 mm Hg
IBP Channel 4 (if available & parameter box is vertical)	White	0/200 mm Hg
SPO ₂ (if available)	Blue	85/100%
Breathing Pressure	Yellow	-10/100 cm H ₂ O
Minute Volume (if parameter box is vertical)	Blue	0.0/30.0 L
Primary Agent (if parameter box is vertical)	Agent-specific	0/9.0%,except Desflurane 0/20.0%
CO ₂	White	0.0/10.0%, 0.0/10.0 kPa, 0/76 mm Hg
Page 2		
MAC	White	0/3.5
Respiration Rate	White	0/20 or 0/70 BPM (autoscale)
Fresh Gas Flow - Air	Yellow	0/10 L/min
Fresh Gas Flow - O2	Green	0/10 L/min
Fresh Gas Flow - N2O	Blue	0/10 L/min
Fresh Gas Flow - Total	White	0/20 L/min

Parameter	Color	Scale min/max displayed in trend
Minute Volume (if parameter box is horizontal & there are more than six trends on this page)	Blue	0.0/30.0 L
Primary Agent (if parameter box is horizontal & there are more than seven trends on this page)	Agent-specific	0/9.0%,except Desflurane 0/20.0%
Page 3		
Minute Volume (if not displayed on previous page & parameter box is horizontal)	Blue	0.0/30.0 L
Primary Agent (if not displayed on previous page & parameter box is horizontal)	Agent-specific	0/9.0%,except Desflurane 0/20.0%
Inspired N2O	Peach	0/100%
O ₂	Green	0/100%
Lung Compliance	White	0/150 ml/cm H ₂ O
SPO2 Pulse (if available)	White	40/235 bpm
NIBP	White	0/300 mm Hg
IBP Channel 4 (if available & parameter box is horizontal)	White	0/200 mm Hg
Temperature	White	20.0/45.0°C or 68.0/113.0°F
Blood Temperature (if available)	Red	20.0/45.0°C or 68.0/113.0°F
Cardiac Output (if available)	White	0/15 L/min
Page 4 (if required)		
SPO2 Pulse (if available)	White	40/235 bpm
NIBP	White	0/300 mm Hg

General Description

2	

Parameter	Color	Scale min/max displayed in trend
IBP Channel 4 (if available & parameter box is horizontal)	White	0/200 mm Hg
Temperature	White	20.0/45.0°C or 68.0/113.0°F
Blood Temperature (if available)	Red	20.0/45.0°C or 68.0/113.0°F
Cardiac Output (if available)	White	0/15 L/min
Page 5 (if required)		
ST I (if available)	White	-12/12 mm*
ST II (if available)	White	-12/12 mm*
ST III (if available)	White	-12/12 mm*
ST V (if available)	White	-12/12 mm*
ST aVR (if available)	White	-12/12 mm*
ST aVL (if available)	White	-12/12 mm*
ST aVF (if available)	White	-12/12 mm*
Page 6 - Custom Trend		

*autoscales over ±2.0, ±4.0, ±12.0 mm

Data LogThe data log contains data for those parameters that are displayed in the
current screen configuration and for which data is currently available.
Data is recorded in intervals selected by the user. Table 2-3 on page 2-15
shows the order of the entries displayed in the log (if available). For
complete information on the data log, see the Narkomed 6400 Anesthesia
System Operator's Instruction Manual.

Data Log Control

To customize the data log display, touch any part of the data log currently displayed on the screen. The Data Log Control dialog box appears:



Figure 2-7. Data Log Control Dialog Box

The **[NIBP]** and **[Cardiac Output]** control buttons are available only when the Narkomed 6400 is configured with an IPM. Their functions are described below. The functions of the remaining control buttons are described in the *Narkomed 6400 Anesthesia System Operator's Instruction Manual*.

This control button determines when new NIBP data is entered into the data log.

Touch **[NIBP]** or its associated selection field until the preferred setting is displayed:

- On new NIBP data appears in the data log within 30 seconds of the completion of a successful NIBP measurement. NIBP data is displayed at the 1-minute data log interval, regardless of the current data log interval setting. If multiple readings are taken within one minute, additional data log rows will be added, all showing the same time stamp.
- Off new NIBP data appears in the data log at the next data log interval.

NIBP

2

Cardiac This control button determines when new cardiac output data is entered into the data log.

Touch **[Cardiac Output]** or its associated selection field until the preferred setting is displayed:

- On new cardiac output data appears in the data log within 30 seconds of the completion of a successful cardiac output measurement. Cardiac output data is displayed at the 1-minute data log interval, regardless of the current data log interval setting. If multiple readings are taken within one minute, additional data log rows will be added, all showing the same time stamp.
- Off new cardiac output data appears in the data log at the next data log interval.

Table 2-3.	Data I	_og	Entries	

Data Log with IPM Parameters		
Data Log Entry	Monitoring Measure	
Time	time readings were taken	
NIBP S/D	NIBP systolic/diastolic pressure (mm Hg)	
NIBP Mean	NIBP mean pressure (mm Hg)	
HR	Heart rate (bpm)	
SPO2	oxygen saturation (%)	
SPO2 Pulse	SpO ₂ pulse (bpm)	
ET CO2	end tidal carbon dioxide concentration readings in the currently selected units	
INSP CO2	inspiratory carbon dioxide concentration readings in the currently selected units	
N2O	inspiratory nitrous oxide (percent) readings	
Resp Rate	respiratory rate (BPM) (from flow sensor)	
P1 S/D	Systolic/diastolic pressure from channel P1 (mm Hg)	
P1 Mean	Mean pressure from channel P1 (mm Hg) (site shown)	
P2 S/D	Systolic/diastolic pressure from channel P2 (mm Hg)	
P2 Mean	Mean pressure from channel P2 (mm Hg) (site shown)	
P3 S/D	Systolic/diastolic pressure from channel P3 (mm Hg)	
P3 Mean	Mean pressure from channel P3 (mm Hg) (site shown)	

Data Log with IPM Parameters				
Data Log Entry	Monitoring Measure			
P4 S/D	Systolic/diastolic pressure from channel P4 (mm Hg)			
P4 Mean	Mean pressure from channel P4 (mm Hg) (site shown)			
Agt (1) I/E	primary agent inspiratory and expiratory concentration (percent) readings			
Agt (2) I/E	secondary agent inspiratory and expiratory concentration (percent) readings			
MAC	MAC value based on detected agent type			
Tid Vol	tidal volume (liters)			
Min Vol	minute volume (liters)			
Peak Pres	peak pressure (cmH ₂ O)			
Plat Pres	plateau pressure (cmH ₂ O)			
Mean Pres	mean pressure (cmH ₂ O)			
PEEP	PEEP (cmH ₂ O)			
02	oxygen concentration (percent) readings			
ST CH1	ST value from Channel 1			
ST CH2	ST value from Channel 2			
ST MAX	Maximum ST value			
Temp 1	T1 temperature (°C or °F)			
Temp 2	T2 temperature (°C or °F)			
СО	cardiac output (L/min)			
Blood Temp	blood temperature (°C)			
PAWP*	PA wedge pressure (mmHg)			
Vent Mode	ventilator mode setting			
Vt Set	ventilator tidal volume setting (mL) (logged in Volume mode only)			
Rate Set	ventilator breathing rate setting (bpm) (logged in Volume and Pressure modes only)			
SIMV Rate	ventilator breathing rate setting (bpm) (logged in SIMV mode only)			

General Description

2

Data Log with IPM Parameters				
Data Log Entry	Monitoring Measure			
I:E Ratio**	inspiratory/expiratory ratio (logged in Volume, Pressure, and SIMV modes only)			
Pmax	maximum ventilator pressure setting (cmH $_2$ O) (logged in Volume mode only)			
PSet	peak ventilator airway pressure setting (cmH $_2$ O) (logged in Pressure mode only)			
PEEP Set	ventilator PEEP setting (cmH_2O) (logged in Volume, Pressure, and SIMV modes only)			
Insp Pause	ventilator inspiratory pause setting (%) (logged in Volume and SIMV modes only)			
Insp Flow	ventilator inspiratory flow rate setting (L/min) (logged in Pressure mode only)			
FG Flow Air	fresh gas flow for air (L/min)			
FG Flow N2O	fresh gas flow for nitrous oxide (L/min)			
FG Flow O2	fresh gas flow for oxygen (L/min)			
FG Flow Total	total fresh gas flow for all gases (L/min)			

*PA wedge pressure is displayed in the data log when the wedge pressure measurement is complete and confirmed by the clinician. The time stamp will be the same as the time displayed in the PA parameter box. PA wedge pressure data will be displayed at the 1-minute interval, regardless of the current data log setting. If multiple measurements are taken in one minute, additional data log rows will be added, all showing the same time stamp.

**the I:E Ratio setting displayed in the data log may differ from the ventilator setting by ±0.2.

2-17

Printing IPMWhen the Narkomed 6400 is configured with an IPM, the clinician can
selectively print the following IPM-related data:

- only NIBP and SpO₂ information
- saved Cardiac Calculations
- 1. Press [Print] on the main screen taskbar. The options page appears.

Strip Chart Recorder-Print Options Waveforms Quick Waveform Print Continuous Waveform Print	Strip Chart Print Strip Chart Waves
Data Log NIBP / SPO2 Data Log Vital Signs and Calculations Print Vital Signs Print Cardiac Calculations	External Printer Config External Datalog Print
Stop Print Ready to Print	

Figure 2-8. Strip Chart Recorder Print Options Page in Print Notebook

2. Touch **[NIBP/SPO2 Data Log]** or **[Print Cardiac Calculations]** to initiate the printing of the desired data. While the print is taking place, a "Print in Progress" message appears at the bottom of the options page. All other control buttons on the options page are disabled until the print is completed, with the exception of the **[Stop Print]** button.

The most recent data is located at the bottom of the printout. If monitoring information for a particular parameter is invalid or not available, dashes appear for that parameter on the printed data log. See Figure 2-9 and Figure 2-10.

For complete information on printing data using the Strip Chart Recorder, see the Narkomed 6400 Anesthesia System Operator's Instruction Manual.

General Description

2

TIENT:		PATIENT:
ITE: 11/13/01 ME: 11:42:29		DATE: 10/06/03
.me SYS DIA MBP HA I:MmmHg BPM	SA %	
Ø2 55	98	CARDIAC CAL
04 55	98	DATE: 10/06/03
5 121 73 91 55	98	HT 168.5(cm)
16 55	98	USE: PAWP
18 55	98	CO * 6.1 HR MOP * 97 CVP
9121 73 91 55	98	PAM * 19 PAWP
0 55	98	BSA 1.79
2 55	98	CI 3.4 SV SVR 1061 SVRI
3 120 73 91 55	98	PVR 105 PVRI
4 55	98	
6 55	98	
.7 120 73 91 55	98	
.8 55	98	
20 55	98	
1 121 73 91 55	98	
× 55	98	

Figure 2-9. Example of an NIBP/SPO2 Data Log Print

Figure 2-10. Example of a Cardiac Calculations Print

TemplatesThe clinician may choose a set of preconfigured system settings from a
library of previously configured templates. See Appendix 2 for a list of
settings that can be stored in a template as well as the default factory
values for those settings. Template operations are fully described in the
Narkomed 6400 Anesthesia System Operator's Instruction Manual.

2

Alarms The following are general alarms that can occur with the IPM. Alarms specific to particular monitoring functions appear in the corresponding chapters later in this manual.

Table 2-1. System Advisories

Message	Condition	Suggested Action
IPM COMM LOST	IPM module communications failure	Contact an authorized representative of DrägerService.
IPM FAN FAIL	Fan failure within IPM module	Contact an authorized representative of DrägerService.
SRVC IPM MON	Internal system fault	Contact an authorized representative of DrägerService.

Replacing the IPM Module

The IPM module can be replaced while the Narkomed 6400 system is operating. This allows the anesthesia machine to remain functional in the event of an IPM module failure during a case.

Draeger Medical recommends the following precautions:

- spare IPM modules should be controlled carefully by the facility.
- before replacing the IPM module, the clinician should verify that the replacement module appears to be in good condition. If there is any doubt about the replacement module's quality, the module should not be used without first contacting an authorized representative of DrägerService.
- the clinician should perform a complete Narkomed 6400 system self-test as soon as is practical. Full system self-diagnostics will run at that time.

Integrated Patient Monitor Option Operator's Manual Part Number: 4117966-003 Rev: -
General Description

2

To replace the IPM module:

- 1. Remove the patient cables from the front of the IPM module being replaced.
- 2. Unscrew the thumbscrew on the front panel of the IPM module being replaced.
- 3. Pull the module forward and out of the machine just enough to gain access to the rear of the module.
- 4. Remove the two thumbscrews securing the electrical cable connector to the rear of the module.

Note: Do not let this cable slip into the machine.

- 5. Pull the module completely out of the machine.
- 6. Insert the replacement IPM module into the slot, allowing access to the rear of the module. Connect the electrical cable to the rear of the replacement IPM module using the two thumbscrews. Tighten the thumbscrews and make sure that the cable is on straight and fully seated.
- 7. Slide the IPM module fully into the machine until the front panel is flush with the side of the machine. Tighten the thumbscrew on the front panel of the module.
- 8. Reconnect the patient cables to the front panel of the module.
- 9. Verify that the IPM module is functional within 60 seconds of installation.
- 10. When practical, perform a complete system self-test in order to allow full diagnostics to be run on the IPM module.

This section contains configuration and operational information specific to ECG monitoring and ST Segment analysis.

Overview	3-2
ECG Display	3-2
ST Segment Analysis Display	3-3
Electrode Placement	3-5
ECG Signal Quality	3-5
Surgical Considerations (Adults)	3-6
5-Leadwire Electrode Placement	3-6
Alternate 3-Leadwire Electrode Configuration	3-7
Electrode Placement for Patients With Pacemakers	3-7
ESU ECG Filters	3-7
Setting Up ECG Parameters	3-9
ECG Setup Tab	3-9
ECG Alarms Tab	3-11
ECG Site-Scale Tab	3-13
ECG Volume Tab	3-14
ECG Freeze Tab	3-15
Setting Up ST Segment Analysis Parameters	3-17
ST Segment Analysis Setup Tab	3-17
Summary of Alarms	3-18
Problem Resolution	3-19
Care and Cleaning	3-20

Overview

The Narkomed 6400 with the Integrated Patient Monitor option can monitor ECG and perform ST segment analysis.

It provides full ECG monitoring of up to seven leads with simultaneous display of numeric and waveform data from up to two leads. The system acquires signal data from a 5-leadwire or a 3-leadwire cable set.

Table 3-1. 3 and 5-Leadwire Cable Monitoring

Cable	Number of Leads Monitored	Number of ECG Waveforms Displayed
3-leadwire set	3 leads	1 waveform
5-leadwire set 7 leads		2 waveforms

The heart rate is detected and displayed. A QRS pulse tone is available for audio monitoring. A pace pulse option can be set to detect pacemaker pulses, remove them from the heart rate calculation, and display them in the waveform.

The system is compatible with defibrillators and intra-aortic balloon pumps. The system displays the point where the defibrillator synchronized pulse occurs in the ECG waveform.

The system can identify and display lead fault errors. When the heart rate alarms are turned on, the system also alerts the operator to heart rate violations.

ST segment analysis information is performed on all leads and displayed in the ST parameter box. The ST numeric values show the ST deviation for the lead selected for channel 1, for the lead selected for channel 2, and for the lead with the maximum ST value.

ECG Display

The ECG parameter box is labeled HR. The current heart rate is displayed numerically and also visually with a flashing heart icon that "beats" with each pulse. The numeric display range is 0 to 300 beats per minute (bpm) and the value is updated every two seconds.

Up to two simultaneous ECG waveforms are displayed as the two top-most waveforms (two waveforms when either a 5-leadwire set or no leadwire set is connected, and one waveform when a 3-leadwire set is connected). The color of the waveforms is the same as the color of the ECG numeric data.

Lead labels are shown on the left side of the waveform. The lead can be changed by touching the lead label until the desired lead is displayed. The lead can also be changed in the Setup page of the ECG notebook.

Integrated Patient Monitor Option Operator's Manual

ECG Monitoring/ST Segment Analysis

3

The scale factor for the ECG waveform is also shown on the left side of the waveform. The scale factor can only be changed in the setup page of the ECG notebook (it is not touch sensitive).

Figure 3-1. ECG Parameter Box and Waveform



ECG Trend The heart rate trend is represented as a single line. The low scale value is 0 bpm and the high scale value is 200 bpm.

Trends are displayed by touching the **[Trend]** button in the main screen taskbar. For complete information on trends, see the Narkomed 6400 Operator's Instruction Manual.

Figure 3-2. ECG (Heart Rate) Trend



ST Segment Analysis Display

The ST segment parameter box is labeled ST. Numeric ST values for the lead selected for channel 1, for the lead selected for channel 2, and for the lead with the maximum ST value are displayed in millimeters and are updated every two seconds.

NOTE: The ST parameter box is displayed only when a 5-leadwire set or no leadwire set is connected to the IPM.

A waveform is not displayed for the ST segment, nor are any alarms available.

NOTE: ST segment values may at times be unavailable for patients with pacemakers.

Figure 3-3. ST Segment Analysis Parameter Box



ST Segment The ST Segment trend trends ST values for the two currently selected ECG leads. Leads)

Trends are displayed by touching the **[Trend]** button in the main screen taskbar. For complete information on trends, see the Narkomed 6400 Operator's Instruction Manual.

Figure 3-4. ST Segment Trend (Current Leads)

-7.7	12.0 STII
-4.4 ==	= stv
	-12.0

ST Segment
Trend (All
Leads)

This option allows you to view trends for the following ECG leads: I, II, III, V, aVR, aVL, and aVF.

- 1. Touch the **[Trend]** button in the main screen taskbar.
- 2. Touch the displayed trend window to bring up the trend dialog box.
- 3. Touch the **[Next]** or **[Previous]** button in the dialog box to view other trend pages. The trends for all leads will be displayed.

For complete information on trends, see the Narkomed 6400 Operator's Instruction Manual and Section 2 of this manual.

3

ECG Monitoring/ST Segment Analysis

3

NABKOMED 6400 User/Site 1g 12, 2002 - 11:47:39 TREND HR Smooth Scroll 60 0.0 ST 0.0 Jump Scroll 0.0 ST II 0.0 ◀ 18 -2.0 79 1 Trend Page 0.0 PIs 60 вт ш CVP2 Previous Next (11)Jump to Custom 0.0 еты PA3 Trend Page 24 (14) -2.0 2.0 Page 5 of 6 5 -0.0 SpO2 96 Exit -2.0 2.0 76 🔔 Pulse Paw 0.0 ST av 25 PEAK MEAN -2.0 2.0 CO2, 15 2 12 0.0 🔔 RR 10:58:56 FEM4 Temp 811 STOP SKIN 37.6 Air-Only Mode Limits Utilities Alarm Suspe Main Scree To Primary Respi-Vent Info Low Flow O2 Cal Template Noteboo Autos

Figure 3-5. ST Segment Trend (All Leads)

Electrode Placement

ECG Signal Secure the electrode and leadwire with a leadwire stress loop near the electrode, and tape the stress loop to the patient. This prevents leadwire rotation, leadwire tugging at the electrode, and ECG artifact.

Electrodes should be replaced at least every 48 hours to maintain quality signals. Over the course of 48 hours, the electrode gel will start to dry out and the adhesive will age. The gel or adhesive may also irritate sensitive patient skin.

- **CAUTION:** Line isolation monitor transients may resemble actual cardiac waveforms and, if picked up, may inhibit heart rate alarms. Ensure that leadwires and patient connections are held in place in order to help guard against loosening of connections.
- **CAUTION:** To keep recovery time after defibrillation to a minimum, avoid simultaneous use of electrodes made of dissimilar materials. The use of squeeze bulb electrodes is not recommended.

Surgical Consider- ations	Proper skin preparation is important in order to minimize electrosurgical (ESU) interference. Thoroughly prepare the skin next to the grounding pad as well.
(Adults)	Place the right leg electrode near the ESU grounding pad.
5-Leadwire Electrode Placement	The illustration below shows suggested electrode placement when using 5 leadwires.

Figure 3-6. 5-Lead Electrode Placement



- 1. Place the right arm (RA) and left arm (LA) electrodes just below the right and left clavicle near the shoulder.
- 2. Place the right leg (RL) and left leg (LL) electrodes on a non-muscular site in the lower right and left abdomen, just below the rib cage.
- 3. Place the chest (V) electrode in any of the six chest positions:
 - V1 Fourth intercostal space at the right border of the sternum.
 - V2 Fourth intercostal space at the left border of the sternum.
 - V3 Midway between locations V2 and V4.
 - V4 At the mid-clavicular line in the fifth intercostal space.
 - V5 At the anterior axillary line on the same horizontal level as V4.
 - V6 At the mid-axillary line on the same horizontal level as V4 and V5.

3

Iternate -Leadwire Iectrode Configura- on	When a 5-leadwire configuration is not desirable, for instance with neonatal monitoring or burn patients, a 3-leadwire cable (RA, LA, and LL) can be used. The clinician can select lead I, II, or III in the ECG notebook.
lectrode Configura- Ion	can be used. The clinician can select lead I, II, or III in the ECG notebook.

The illustration below shows suggested electrode placement for patients with pacemakers.

Figure 3-7. Electrode Placement for Patients with Pacemakers



The procedure is the same as for 5-leadwire placement, except the RA electrode is placed in the right fifth intercostal space. The LL electrode is placed in the left fifth intercostal space. Make sure each lead has at least 1/2 mV of signal after all the electrodes are in place.

ESU ECG Filters

Electrode

Placement

for Patients

Pacemakers

With

When using the Integrated Patient Monitor in the presence of an electrosurgical unit, the use of the Multi-Link ESU ECG patient cable is recommended. The cable has a built-in ESU filter which helps to reduce ESU noise in the ECG signal.

Preparing for ECG Monitoring

- 1. Prepare the patient's skin before placing the electrodes. Proper skin preparation is vital to obtaining a good electrical signal. Use the following guidelines for skin preparation:
 - shave the hair from the skin, if necessary
 - clean the skin thoroughly with soap and water
 - dry the skin completely
- 2. Place the electrodes on the patient.
- 3. Attach the patient cable to the ECG connector on the Integrated Patient Monitor module.
- 4. Attach the electrode leadwires to the patient cable.

Standard leadwire labels and colors are identified in the following table.

Leadwire	Leadwire Color (AHA)	Leadwire Label (AHA)	Leadwire Color (IEC)	Leadwire Label (IEC)
RA (right arm)	white	RA	red	L
LA (left arm)	black	LA	yellow	R
RL (right leg)	green	RL	black	F
LL (left leg)	red	LL	green	N
V1 (precordial)	brown	V1	white	C1
V2 (precordial)	yellow	V2	yellow	C2
V3 (precordial)	green	V3	green	C3
V4 (precordial)	blue	V4	brown	C4
V5 (precordial)	orange	V5	black	C5
V6 (precordial)	purple	V6	purple	C6

Table 3-2. Standard Leadwire Labels and Colors

5. Set the appropriate ECG monitoring settings in the ECG notebook.

3

Setting Up ECG Parameters

Touch the ECG parameter box (labeled HR) anywhere except the alarm bell to display the ECG notebook.

ECG Setup Touch the **[Setup]** tab in the ECG notebook to display the Setup page: **Tab**

Figure 3-8. ECG Setup Page



Pace Pulse This option controls pacemaker pulse detection. Follow these procedures to successfully monitor pacemaker patients:

- Use the recommended ECG placement.
- Check each lead in the top trace and locate the lead with the best defined pacemaker spikes. Use this lead for the primary ECG.

The system usually effectively monitors a pacemaker patient when the Normal setting is used. However, try the Alternate setting if problems occur.

- **WARNING:** False low heart rate or false asystole indicators can result due to electrical overshoots in certain pacemakers.
- **WARNING:** Do not monitor pacemaker patients without the pace program activated.

	WARNING: Do not use pacemaker spike and shape for diagnostic interpretation.
	WARNING: Monitor pacemaker patients closely and do not rely entirely on the rate meter alarms. Rate meters can continue to count the pacemaker rate during cardiac arrest and some arrhythmias. For pacemaker pulse rejection capability, see Section 9 - "Specifications".
	Touch [Pace Pulse] or its associated selection field until the desired setting is displayed:
	Off - turns off pacemaker pulse detection
	• Normal - pacemaker pulse detection that disregards events occurring within a few milliseconds after a pacer pulse
	• Alternate - pacemaker pulse detection that monitors events occurring within a few milliseconds after a pacer pulse
	WARNING: When the Normal or Alternate settings are used, a pacemaker pulse could be falsely counted as a QRS during asystole.
	When Normal or Alternate are chosen, a "P" appears in the ECG parameter box and an artificial spike is placed on the ECG waveform (upper waveform only).
	A pacer spike of greater amplitude is added to the analog output from the Integrated Patient Monitor when pace detection is on and detection occurs.
QRS Tone	Touch [QRS Tone] or its associated selection field to toggle the state of the QRS tone ON or OFF.
	NOTE: When the QRS tone is set to OFF, the SpO_2 tone is automatically set to ON; when the QRS tone is set to ON, the SpO_2 tone is automatically set to OFF.
Relearn	Relearn is a command button that causes the system to learn the patient's new ECG pattern. The relearn function is used when a significant change in the patient's ECG pattern occurs that could affect the accuracy of subsequent measurements. A change in the patient's ECG pattern can result in losing ST measurements, inaccurate heart rate, or both.
	Touch [Relearn] . The heart rate numeric value and the heart icon are removed from the ECG parameter box and the message Relearn in Progress is displayed until the system learns the new ECG pattern.

3

ECG Alarms
TabTouch the [Alarms] tab in the ECG notebook to display the Alarms page:
NOTE: The heart rate alarm limits are linked to the SpO2 pulse rate

alarm limits. Any changes to the high, low, autoset narrow, or autoset wide settings on the heart rate alarm page will be automatically reflected on the SpO₂ alarm page.

Figure 3-9. ECG (Heart Rate) Alarms Page



Alarm Control Use this option to turn the ECG alarm notification system on or off.

Touch **[Alarm Control]** or its associated selection field until the desired option is displayed:

- On turns the ECG alarms notification system on
- Off turns the ECG alarm notification system off
- Stby suspends the alarm notification system for ECG monitoring. The alarms are turned on automatically as soon as Integrated Patient Monitor communications are established.
- **NOTE:** ECG alarms are turned off automatically while a SRVC IPM MON advisory alarm is active.

Heart Rate Limit-Set Hi	This option sets the high alarm limits for ECG. The range of available settings is 31 to 300 bpm. The lowest possible setting must be one bpm higher than the low alarm setting. The default setting is 150 bpm.		
	1. Touch [Set Hi] or its associated selection field to activate the slider bar.		
	2. Move the slider control until the desired setting is displayed next to the [Set Hi] button.		
	• Use the fine adjustment to change the setting by 1 bpm.		
	• Use the coarse adjustment to change the setting by 10 bpm.		
	NOTE: If the new ECG high alarm limit setting exceeds the SpO_2 alarm limit range, the SpO_2 alarm limit will be set to the highest setting allowed in its range (235 bpm).		
Heart Rate Limit-Set Lo	This option sets the low alarm limits for ECG. The range of available settings is 30 to 299 bpm. The highest possible setting must be one bpm lower than the high alarm setting. The default setting is 50 bpm.		
	1. Touch [Set Lo] to activate the slider bar.		
	2. Move the slider control until the desired setting is displayed next to the [Set Lo] button.		
	• Use the fine adjustment to change the setting by 1 bpm.		
	• Use the coarse adjustment to change the setting by 10 bpm.		
	NOTE: If the new ECG low alarm limit settings exceeds the SpO_2 alarm limit range, the SpO_2 alarm limit will be set to the lowest setting allowed in its range (40 bpm).		
Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the ECG parameter box ON or OFF.		
AUTOSET Narrow	This option sets heart rate alarm limits to +/- 10 of the current reading within the 30 to 300 beats/minute range. When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.		
	Touch [AUTOSET Narrow] . The setting is automatic. The new range values appear next to the [Set Hi] and [Set Lo] buttons.		

3-12

3

AUTOSET This option sets heart rate alarm limits to +/- 20 of the current reading Wide within the 30 to 300 beats/minute range. When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.

Touch **[AUTOSET Wide]**. The setting is automatic. The new range values appear next to the **[Set Hi]** and **[Set Lo]** buttons.

ECG Site-
Scale TabTouch the [Site-Scale] tab in the ECG notebook to display the Site-Scale
page:

Figure 3-10. ECG Site-Scale Page



- ECG Channel The **[ECG Channel]** button is used to select the screen location currently being set up. Channel 1 is the top waveform location and Channel 2 is the second waveform location from the top.
 - **NOTE:** The Channel 2 waveform area is displayed on the screen only when a 5-leadwire set or no leadwire set is connected to the IPM.

Touch **[ECG Channel]** or its associated selection field until the desired channel is displayed.

Lead This option specifies the lead displayed for each ECG channel. The choices include I, II, III, V, aVR, aVL, and aVF.
Touch [Lead] or its associated selection field to display the list of options, then touch the correct lead identifier.
NOTE: It is not possible to display duplicate ECG leads simultaneously.
Scale This option sets the scale for the selected lead. The choices include 0.25x, 0.5x, 1.0x, 2.0x, and 4.0x.
Touch [Scale] or its associated selection field to display the list of options, then touch the desired scale value.
ECG Volume Touch the [Volume] tab in the ECG notebook to display the Volume page:

Tab

Figure 3-11. ECG (QRS) Volume Page



QRS Volume Use this option to adjust the volume for the QRS complex tone.

Move the slider control to adjust the volume. The relative position of the control indicates the setting. The volume is turned off when the slider control is at the bottom of the slider. The setting is the highest when the slider control is at the top.

erator's Manual Part Number: 4117966-003 Rev: -

3

ECG Freeze Touch the **[Freeze]** tab in the ECG notebook to display the Freeze page. **Tab**



Figure 3-12. ECG Freeze Page (most recent waveform and stored waveform shown)

The Freeze page allows the clinician to display a fixed portion of the ECG waveform for closer analysis. The page contains control buttons for capturing, viewing, and storing waveform segments which are displayed in three separate viewing areas or "frames":

- the freeze frame at the top of the page shows the most recent 10-second segment of the ECG Channel 1 waveform.
- the expanded frame in the middle of the page shows a bigger, more detailed view of a smaller part of the waveform (the part enclosed in a box on the freeze frame). At least one second of the waveform appears in the expanded frame.
- the stored frame at the bottom of the page shows the last expanded frame that was stored by the clinician.

Freeze Touch the **[Freeze]** button to display the most recent 10 seconds of the ECG channel 1 waveform in the freeze frame.

An indicator box is drawn in the freeze frame at most recent part of the waveform. That part is displayed in detail in the expanded frame.

The time, heart rate, ST value, and ECG channel 1 lead label recorded at the time the Freeze button was pressed are displayed above the expanded frame. The expanded frame waveform is scaled automatically based on the amplitude, and two reference lines are drawn: one at zero millivolts and another, calculated reference line from the following list: ± 0.1 mV, ± 0.25 mV, ± 0.5 mV, ± 1.0 mV.

Use the left and right arrow keys to move the waveform in the expanded frame back or forward in time. The box in the freeze frame moves at the same time to indicate which portion of the 10-second waveform is currently displayed. The clinician can also touch any portion of the freeze frame to move the box and display that portion in the expanded frame.

Touching the **[Freeze]** button again removes the current waveform and displays a new 10-second segment of the ECG waveform.

Store Frame Touch the **[Store Frame]** button to copy the waveform currently in the expanded frame into the store frame. The reference lines and the time, heart rate, ST value, and ECG lead label information above the frame are copied as well. The expanded frame is cleared, except for the zero reference line.

A new portion of the waveform can be displayed in the expanded frame by touching the freeze frame.

Part Number: 4117966-003 Rev: -

3

Setting Up ST Segment Analysis Parameters

Touch the ST parameter box anywhere except the alarm bell to display the ST segment notebook.

ST Segment
AnalysisTouch the Setup tab in the ST notebook to display the ST segment Setup
page:Setup TabTouch the Setup tab in the ST notebook to display the ST segment Setup

Figure 3-13. ST Segment Setup Page

	ST Segment Setup	Setup
	ST Lead I 0.6	Alarms
	ST Lead II 0.9 Max	Site - Scale
>	ST Lead V 0.1 ST Lead aVR -0.8	Print
	ST Lead aVL 0.1 ST Lead aVF 0.6	Volume
		Calc.

The ST segment Setup page displays real time ST values for all seven ECG leads. These values are updated every two seconds.

NOTE: Alarms are not available for ST segment data.

Summary of Alarms

This section contains all warning, caution, and advisory alarms associated with ECG monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 3-3. ECG Warnings

Message	Condition	Suggested Action
ASYSTOLE	No systolic heartbeat is detected for 6 seconds and the heart rate is less than 30 beats/minute	If the patient's condition is not the cause, check the ECG leads placement of electrodes, or check the skin preparation. Also check the ECG settings, waveform scale, or set the system to relearn the ECG signal. At least a .5 mV amplitude is needed for QRS detection (at 1.0 size).
HEART RATE LOW	The heart rate is below the alarm limit setting	Check the patient. If needed, change the alarm limit setting.

NOTE: When an ASYSTOLE condition persists continuously for approximately three (3) minutes, the ASYSTOLE and HEART RATE LOW alarms will be automatically cleared.

Table 3-4. ECG Cautions

Message	Condition	Suggested Action
HEART RATE HIGH	The heart rate exceeds the alarm limit setting	Check the patient. If warranted, change the alarm limit setting.

ECG Monitoring/ST Segment Analysis

Table	3-5.	ECG	Advis	ories
-------	------	-----	-------	-------

Message	Condition	Suggested Action
ECG LEADS ERR	The system is not getting adequate signals from the ECG leads	Check RL lead. Check all leads for proper connections at the electrode and the Integrated Patient Monitor module.
ECG (xx) DISC or ECG LEAD V ERR	The identified lead is not sending adequate signals to the system	Check lead and electrode placement.
ECG SIG ERR	ECG artifact detected	Switch to view another lead waveform. Check the electrode placement and lead connections at the electrode and the Integrated Patient Monitor module.

Problem Resolution

Inaccurate heart rate:

- Check the leads. Adjust them if necessary.
- Make sure the patient's skin was properly prepared.
- Check the electrode placement. Reposition them correctly on the patient. Replace the electrodes, if needed.
- Check the waveform scale. Select higher scales to increase QRS detection, e.g., 4x scale provides maximum sensitivity. This may be helpful for low amplitude QRS waveforms. Use with caution since baseline artifact may be detected as QRS.
- Relearn the ECG.

Heart rate double counting, inaccurate alarms for low heart rate or asystole, or pacemaker spikes not recognized for pacemaker patients:

- Reassign the lead in the top position.
- Try another electrode placement.

Care and Cleaning

Clean cables, leads, and electrodes after each use using mild detergent and a soft, damp cloth.

Check the cables, leads, and electrodes for any signs of damage.

Hang the cables and leads or lay them flat to prevent damage.

To avoid damaging the components and accessories, **do not:**

- Use acetone, alcohol, ammonia, chloroform, or other strong solvents, because they can eventually damage the vinyl casing
- Immerse the cables in water
- Autoclave leads, cables, or electrodes
- Use heavy abrasives such as steel wool for cleaning

4

IBP Monitoring

This section contains configuration and operational information specific to Invasive Blood Pressure (IBP) Monitoring.

Overview 4-2
IBP Display 4-2
Preparing for IBP Monitoring 4-7
Zeroing the Transducer 4-7
Setting Up IBP Parameters 4-8
IBP Setup Tab 4-8
IBP Alarms Tab 4-10
IBP Site-Scale Tab 4-12
IBP Wedge Tab 4-15
IBP Cardiac CalcsTab 4-18
Summary of IBP Alarms 4-19
Problem Resolution 4-20

Overview

4

The Narkomed 6400 with the Integrated Patient Monitor option can monitor up to four invasive blood pressures. The Integrated Patient Monitor processes information provided by strain-gauge transducers to monitor and display pressure values. Pulsatile and nonpulsatile blood pressures can be monitored. The capability to perform PA wedge measurement is provided as well.

For pulsatile sites, the numeric systolic, diastolic, and mean pressures are displayed. Pulse rates are also displayed for arterial and femoral sites. The numeric mean pressure is displayed for nonpulsatile sites.

IBP Display

The Narkomed 6400 can display up to four waveforms and corresponding numerical data from transducers in channels P1, P2, P3, and P4.

The waveforms for each channel can be displayed separately, as shown in Figure 4-1, or they can be combined into one display of overlapping waveforms, as shown in Figure 4-2.

Figure 4-1. IBP Waveforms and Parameter Boxes (ART1, CVP2, PA3, & FEM4) in Separate IBP Mode



IBP Monitoring

4



Figure 4-2. IBP Waveforms and Parameter Boxes (ART1, CVP2, PA3, & FEM4) in Overlapping IBP Mode

The IBP parameter boxes are identified by labels that correspond to the site and channel being monitored. The current IBP value is shown next to its label. Systolic, diastolic, and/or mean values can be displayed, depending on the pressure site being monitored. The pulse rate is displayed next to its label (Pls). When a transducer zero is required, the message ZERO BP appears in the center of the parameter box.

The numeric display range for the systolic, diastolic, and mean values for all sites is -98 to 350 mmHg. The numeric display range for the pulse rate is 0 to 333 bpm. All values are updated every two seconds.

The following table lists sites and corresponding labels and displayed values:

Table 4-1. Pressure Sites and Labels

Label	Site	Displayed Values
ART	arterial (pulsatile site)	systolic, diastolic, mean, pulse rate
PA	pulmonary artery (pulsatile site)	systolic, diastolic, mean, wedge
FEM	femoral (pulsatile site)	systolic, diastolic, mean, pulse rate
CVP	central venous pressure (nonpulsatile site)	mean
RA	right atrial (nonpulsatile site)	mean
LA	left atrial (nonpulsatile site)	mean
ICP	intracranial pressure (nonpulsatile site)	mean, cpp
SP	special (nonpulsatile site)	mean

NOTE: Pulse rate is not available for the PA site.

NOTE: CPP is present only if an ART or FEM site is active along with the ICP site.

The color of the waveform is the same as the color of the corresponding IBP numeric data. The waveforms can be scaled either by a selection made in the Site-Scale page of the corresponding IBP setup notebook or by touching the scale indicator in the waveform area. The available scaling options are 20, 30, 40, 60, 100, 180, 200, and 300. In overlapping waveform display mode, there are two scales available (left and right). A label for each IBP site and channel number is displayed below its assigned scale indicator. The scale assignment can be changed in the Site-Scale page of the corresponding IBP setup notebook or simply by touching the IBP label in the waveform area. Scale assignments specific to IBP site can be stored in a template. The factory default values for each IBP site are given in Appendix 2.

4

IBP TrendTrends are displayed by touching the [Trend] button in the main screen
taskbar. For complete information on trends, see the Narkomed 6400
Operator's Instruction Manual.

Each IBP trend is labeled with IBP site and channel. The IBP site can be changed in the Site-Scale page of the IBP notebook; the channel setting cannot be changed. The low scale value for all IBP trends is 0 mmHg and the high scale value is 200 mmHg.

The number of parameters trended in an IBP trend varies by site. The ART, FEM, and PA sites trend three parameters: Systolic (high point of the trend), Diastolic (low point of the trend), and Mean (a black line through the trend). See Figure 4-3.

NOTE: The PA trend also displays PA wedge measurement values, indicated by the symbol "▼" in a contrasting color. The wedge numeric value is displayed when the "▼" symbol intersects the vertical reference line. See Figure 4-4.

Figure 4-3. Example of IBP Trend (ART, FEM, and PA Sites)



NOTE: For IBP data to be recorded in the trend window, the waveform scale for a particular waveform must be large enough to accommodate the displayed pressure values.

Figure 4-4. Example of IBP Trend (PA Sites, wedge values shown)



4

The CVP, RA, LA, and SP sites trend a single parameter–Mean–represented by a single line. See Figure 4-5.

Figure 4-5. Example of IBP Trend (CVP, RA, LA, and SP Sites)



The ICP site trends two parameters–Mean and CPP–each represented by a single line. See Figure 4-6.

Figure 4-6. Example of IBP Trend (ICP Site)

		200
14	<u> </u>	
105		ICPZ
		0

4

Preparing for IBP Monitoring

- 1. Apply the IBP catheter to the patient.
- 2. Attach the patient cable to the appropriate IBP connector labeled P1 through P4 on the Integrated Patient Monitor module.
- 3. Ensure that the pressure transducer is at the same level as the heart.
- 4. Zero the pressure transducer (see "Zeroing the Transducer" for more information).
- 5. Configure all IBP monitoring variables (see "Setting Up IBP Parameters" for more information).

Zeroing the Transducer

All transducers can be zeroed simultaneously by pressing the Zero All key on the Integrated Patient Monitor interface panel or the **[Zero All Transducers]** button in the IBP setup notebook. The Zero Transducer button in the IBP setup notebook zeroes only the channel displayed on the page.

NOTE: When a transducer zero is required, the message ZERO BP appears in the IBP parameter box.

Follow this procedure:

- 1. Close the transducer stopcock.
- 2. Open the venting stopcock to atmosphere.
- 3. Display the Setup page of the IBP notebook and press **[Zero Transducer]** or **[Zero All Transducers]**, or press Zero All on the front of the Integrated Patient Monitor module. When the system is zeroed, zeroes appear in the IBP parameter boxes. If a channel cannot be zeroed because it is currently measuring a non-static pressure, the message **PRESS** appears in the corresponding parameter box for a few seconds.
- 4. When zero reference is established, close the venting stopcock and open stopcock to the patient. If the zeroes do not appear, check the stopcock and try this procedure again. Contact an authorized representative of DrägerService if the procedure fails again.

Setting Up IBP Parameters

Touch the IBP parameter box anywhere except the alarm bell to display the IBP notebook for the selected pressure channel (P1 through P4).

The title at the top of each IBP notebook page indicates both the channel (P1, P2, P3, or P4) and the currently selected site for the corresponding parameter box.

IBP Setup Touch the Setup tab to display the IBP setup options.

Tab

4

Figure 4-7. IBP Setup Page for ART



Display Use this option to select the IBP waveform display mode.

Touch **[Display]** until the desired setting is displayed:

- Separate each IBP waveform is displayed separately adjacent to its corresponding parameter box
- Overlapping all IBP waveforms are combined into one overlapping waveform display adjacent to the IBP parameter boxes

IBP Monitoring

Filter Frequency	Use this option to set the correct low pass filter cutoff frequency.
	Touch [Filter Frequency] until the desired setting is displayed:
	• 12 - sets the filter frequency at 12 Hz
	• 40 - sets the filter frequency at 40 Hz
Transducer Calibrate	Use this option to calibrate the transducer. The range for the calibration value is 0.90 to 1.10.
	NOTE: Error is determined when a pressure calibration is performed with another instrument like a manometer.
	1. Touch [Transducer Calibrate] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Transducer Calibrate] button.
	• Use the fine adjustment to change the setting by 0.01.
	• Use the coarse adjustment to change the setting by 0.05.
Zero Transducer	This option initiates a zeroing procedure on the individual pressure transducer. For more information, see "Zeroing the Transducer" earlier in this chapter.
Zero All Transducers	This option initiates a zeroing procedure on all connected IBP transducers simultaneously. For more information, see "Zeroing the Transducer" earlier in this chapter.

IBP Alarms Touch the **[Alarms]** tab in the IBP notebook to display the alarms page. **Tab**

Figure 4-8. IBP Alarms Page for ART



Alarm Control Use this option to set or suspend the IBP alarm notification system.

Touch **[Alarm Control]** or its associated selection field until the desired setting is displayed:

- On turns the alarm notification system on for that IBP site only
- Off turns the alarm notification system off for that IBP site only
- Stby (standby) suspends the alarm notification system for that IBP site until valid data for that site is received, then the alarm notification system for that site only is turned on
- **NOTE:** IBP alarms are turned off automatically while a SRVC IPM MON advisory alarm is active.

Alarm Limits- Pressure	Use this option to select a pressure parameter for setting alarm limits. The alarm limits are established for systolic, diastolic, and mean pressures for the sites monitored. For IBP sites that do not support systolic and diastolic values, the Pressure field is permanently set to Mean.
	Touch [Pressure] until the desired parameter is displayed:
	• Sys - for setting the alarm limits for systolic pressures for that site
	• Mean - for setting alarm limits for mean pressures for that site
	• Dias - for setting the alarm limits for diastolic pressures for that site
	After the pressure parameter is selected, alarm limits can be set. The settings for each pressure parameter are independent and do not affect settings for other pressure parameters or any other site.
Alarm Limits- Set Hi	This option sets the high alarm limits for IBP. The range of available settings is -97 to 350 mmHg. The lowest possible setting for the high alarm limit must be 1 mmHg higher than the current low alarm setting. The default setting is 300 mmHg.
	1. Touch [Set Hi] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Hi] button.
	• Use the fine adjustment to change the setting by 1 mmHg
	• Use the coarse adjustment to change the setting by 10 mmHg
Alarm Limits- Set Lo	This option sets the low alarm limits for IBP. The range of available settings is -98 to 349 mmHg. The highest possible setting for the low alarm limit must be one mmHg lower than the high alarm setting. The default setting is -30 mmHg.
	1. Touch [Set Lo] or its associated selection field to activate the slider bar.
	2. Move the slider control until the preferred setting is displayed next to the [Set Lo] button.
	• Use the fine adjustment to change the setting by 1 mmHg
	• Use the coarse adjustment to change the setting by 10 mmHg
Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the IBP parameter box ON or OFF for that site only.
	NOTE: If the site label is ART or FEM, displaying the alarm limits will remove the displayed pulse value from the numeric box.

AUTOSET Narrow	This option sets pressure alarm limits +/-25 from the current pressure reading within the -98 to 350 mmHg range. When the autoset limit falls below the minimum allowable range, the lowest setting within the acceptable range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Narrow] . The change is automatic. The new range values appear next to the [Set Hi] and [Set Lo] buttons.
AUTOSET Wide	This option sets pressure alarm limits +/-50 from the current pressure reading within the -98 to 350 mmHg range. When the autoset limit falls below the minimum allowable range, the lowest setting within the acceptable range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Wide] . The change is automatic. The new range values appear next to the [Set Hi] and [Set Lo] buttons.
IBP Site- Scale Tab	Touch the Site-Scale tab in the IBP notebook to display the Site-Scale page. Note that if the overlapping waveform display is selected, the Site-Scale page contains additional control buttons. See Figure 4-9 and Figure 4-10.

Figure 4-9. IBP Site-Scale Page for ART (Separate Waveform Display Mode)



=	P1 ART SI	TE and SCALE	Setup
	Site Selection		
	Horiz Cursor	OFF	Alarms
	Left Scale High	180	Site -
	Right Scale High	60	Scale
	Scale Location		
	ART1	Left	Wedge
C	CVP2	Right	Volume
Γ	PA3	Right	
	FEM4	Left	Cardiac Calcs

Figure 4-10. IBP Site-Scale Page for ART (Overlapping Waveform Display Mode)

Site Selection Use this option to select the site label for the pressure channel used. If a new site is chosen, all labels and data values in the parameter box, setup notebook, data log, trend, and waveform (if any) are updated to reflect the new site.

Touch **[Site Selection]** or its associated selection field to open the list box, then touch the desired setting:

- ART arterial pulsatile pressure
- PA pulmonary artery pulsatile pressure
- FEM femoral pulsatile pressure
- CVP central venous pressure nonpulsatile pressure
- RA right atrial nonpulsatile pressure
- LA left atrial nonpulsatile pressure
- ICP intracranial pressure nonpulsatile pressure
- SP special-use this option for a nonpulsatile pressure site that is not included in the list

IBP Monitoring

Set Scale High (Separate Mode only)	Use this option to set the waveform scale. The choices include 20, 30, 40, 60, 100, 180, 200, and 300.
	Touch [Set Scale High] or its associated selection field to display the list of options, then touch the desired scale value.
	This option is available only when there is a waveform associated with the currently selected IBP site or when there is a Strip Chart Recorder installed in the Narkomed 6400 system.
Horiz Cursor	Use this option to display a reference horizontal cursor on top of the IBP waveform.
	Touch [Horiz Cursor] or its associated selection field to toggle the display of the cursor ON or OFF. When the cursor is on, the value associated with its position will be displayed above it. In overlapping waveform display mode, the left and right scale positions will be displayed above and below the cursor, respectively.
	Touch and drag the cursor to any point on the waveform to determine the value of the waveform at that point.
	This option is available only when there is a waveform associated with the currently selected IBP site.
Left Scale High	Use this option to set the scale for the left waveform. The choices include 20, 30, 40, 60, 100, 180, 200, and 300 mmHg.
Mode only)	Touch [Left Scale High] or its associated selection field to display the list of options, then touch the desired scale value.
	This option is available only when there is a waveform associated with the currently selected IBP site or when there is a Strip Chart Recorder installed in the Narkomed 6400 system.
Right Scale High	Use this option to set the scale for the right waveform. The choices include 20, 30, 40, 60, 100, 180, 200, and 300 mmHg.
(Overlapping Mode only)	Touch [Right Scale High] or its associated selection field to display the list of options, then touch the desired scale value.
	This option is available only when there is a waveform associated with the currently selected IBP site or when there is a Strip Chart Recorder installed in the Narkomed 6400 system.
Scale Location	The control buttons for this option indicate the current IBP site and channel for all available IBP channels. They are used to specify a left or right scale location for each channel.
	Touch the appropriate control button or its associated selection field to toggle the scale location between left and right.
IBP Monitoring

4

IBP Wedge Tab	Touch the Wedge tab in the IBP notebook to display the Wedge page.
	NOTE: This tab is available only with PA sites. The tab will be disabled

OTE: This tab is available only with PA sites. The tab will be disabled if the IBP site is changed to a non-PA site, if more than one PA site is selected, or if the IBP transducer requires a zero calibration.

WARNING: Follow the PA catheter manufacturer's instructions when performing a wedge procedure.



Figure 4-11. IBP Wedge Page - Ready for Figure 4-12. IBP Wedge Page -Measurement Measurement Complete

The top half of the Wedge page is used for the display of wedge status messages (before and during measurements) and for the display of the wedge review window (displayed when the measurement is completed). The bottom half of the Wedge page contains wedge control buttons and a wedge status message area.

NOTE: When the message "Wedge Ready-Inflate Balloon" or "Wedge in Progress" is displayed on the Wedge page, all visible waveforms (except ECG) will be drawn at a trace speed of 6.25 mm/sec, regardless of the current trace speed setting. When the measurement is completed, the trace speeds will resume at their previous settings.

Initiating	а	Wedge	Measurement
------------	---	-------	-------------

1.	Ensure that the "Wedge Ready - Inflate Balloon" message is displayed
	on the screen (see Figure 4-11.)

2. Inflate the balloon. The system will monitor inflation and begin processing a wedge value. The message "Wedge In Progress" will be displayed on the screen.

Most of the wedge control buttons are disabled and the IBP alarms are disabled for the duration of the measurement. Wedge processing messages are displayed in the top half of the Wedge page and in the Wedge Status Window (see Table 4-2.)

3. Deflate the balloon when you feel wedging is adequate (3-4 respiratory cycles, but no longer than 20 seconds).

When a valid measurement is completed, a two-tone sound sequence is annunciated and the message "Wedge Complete" is displayed in the Wedge Status Window. All the wedge control buttons and IBP alarms are enabled and ECG, P_{AIRWAY} , and PA waveforms are displayed in the wedge review window. The numeric wedge value and the time the measurement was completed are also displayed in the PA parameter box. See Figure 4-12.)

Wedge processing is aborted if the Wedge page is closed, if **[New Wedge]** is pressed, or if additional IBP channels labeled as PA are activated at any time during the process.

New Wedge The **[New Wedge]** button is used to clear the current measurement (if not yet accepted) in order to allow the clinician to repeat the measurement.

Touch **[New Wedge]**. Begin a new measurement as described in "Initiating a Wedge Measurement" above.

WedgeThe wedge review window, which is displayed when the measurement is
completed, shows a maximum of 20 seconds of waveform data for ECG,
PAIRWAY, and PA. High and low scale indicators are displayed on the PA
waveform which can be automatically rescaled by touching the waveform
screen area. A dashed cursor line appears on the PA waveform which
corresponds to the numeric wedge value displayed at the top of the
waveform. The displayed time indicates the time of the completion of the
measurement.

NOTE: If an error occurs during wedge processing (e.g., no pulse is detected, etc.), no initial wedge value or cursor line is displayed. The Adjust Cursor arrow keys are enabled to allow the clinician to adjust the cursor line and specify the wedge value.

Adjust Cursor Touch the Adjust Cursor arrow keys to move the *dashed* cursor line up or down. As the cursor line is moved, the numeric value is changed to reflect the new cursor position. A *dotted* reference line remains on the waveform to indicate the position of the initial, measured value.

IBP Monitoring

4

Accept Wedge	Touch the [Accept Wedge] button to accept the value displayed in the wedge review window. The value and time of measurement are then
	displayed in the PA parameter box. The Adjust Cursor keys are disabled until the next wedge measurement.
	NOTE: The wedge value is also considered accepted if the Wedge page is closed at any time after the wedge measurement is completed, whether or not the [Accept Wedge] button was pressed.
Cardiac Output	Touching the [Cardiac Output] button displays the Cardiac Output setup page.
	This button is disabled if a wedge measurement is in progress or if the Cardiac Output parameter box is not displayed.
Wedge Status Window	Table 4-2 contains the messages that can be displayed in the wedge status window. Messages are also displayed in place of the wedge review window until the measurement is completed.

Table 4-2. Wedge Status Messages

Message	System Status
Wedge Ready - Inflate Balloon	The system is ready for balloon inflation.
Wedge Waiting for Pulse	The system is detecting a pulse.
Wedge In Progress	The wedge calculation is in progress.
Wedge Complete	The wedge calculation is finished.
No Pulse Detected	The system did not detect a pulse.

IBP Monitoring

IBP CardiacTouch the Cardiac Calcs tab in the IBP notebook to display the Cardiac**CalcsTab**Calculations page.

NOTE: This tab is available only with PA sites.

Figure 4-13. IBP Cardiac Calculations Page (PA Sites Only)



4

Summary of IBP Alarms

This section contains all Warning, Caution, and Advisory alarms associated with IBP monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 4-3. IBP Cautions

Message	Condition	Suggested Action
(Site) (x) DIAS HI	The diastolic pressure reading for a specific site exceeds the high alarm limit setting	If the alarm setting is too low, access the Alarms page of the IBP notebook and reset the high limit for diastolic pressure
(Site) (x) DIAS LO	The diastolic pressure reading for a specific site is below the low alarm limit setting	If the alarm setting is too high, access the Alarms page of the IBP notebook and reset the low limit setting for diastolic pressure
(Site) (x) MEAN HI	The calculated mean pressure for a specific site exceeds the high alarm limit setting	If the alarm setting is too low, access the Alarms page of the IBP notebook and reset the high limit for mean pressure
(Site) (x) MEAN LO	The calculated mean pressure for a specific site is below the low alarm limit setting	If the alarm setting is too high, access the Alarms page of the IBP notebook and reset the low limit setting for mean pressure
(Site) (x) SYSTOLIC HI	The systolic pressure reading for a specific site exceeds the high alarm limit setting	If the alarm setting is too low, access the Alarms page of the IBP notebook and reset the high limit for systolic pressure
(Site) (x) SYSTOLIC LO	The systolic pressure reading for a specific site is below the low alarm setting	If the alarm setting is too high, access the Alarms page of the IBP notebook and reset the low limit setting for systolic pressure

IBP Monitoring

Table 4-4. IBP Advisories

Message	Condition	Suggested Action
(Site) (x) SENSOR ERR	A sensor has failed at a specific site due to a detached or failing sensor or cable	Check the patient to determine if the sensor is inappropriately placed or dislodged. Check the cable connection from the sensor to the Integrated Patient Monitor module. Contact an authorized representative of DrägerService if the cause cannot be determined.
IBP ZERO ERR	The IBP transducer could not be set to zero	Follow the Zeroing the Transducer procedure again. If it does not work, contact an authorized representative of DrägerService.

NOTE: The (Site) (x) SENSOR ERR alarms are not displayed at initial power-up if present.

Problem Resolution

Displayed pressure values are different than expected:

- Check the tubing for bubbles or kinks
- Rezero the transducer
- Review transducer manufacturer's recommendations

Unable to detect wedge or "No Pulse Detected" message displayed:

- Ensure that the PA catheter balloon is completely deflated
- Touch the [New Wedge] button to start wedge processing again
- Perform a wedge measurement as specified in "Initiating a Wedge Measurement" on page 4-16 of this manual

IBP Monitoring

4

Monitor displays "Wedge In Progress" when balloon has not been inflated:

- Ensure that the PA catheter balloon is completely deflated
- Touch the **[New Wedge]** button to start wedge processing again
- Perform a wedge measurement as specified in "Initiating a Wedge Measurement" on page 4-16 of this manual

Wedge value displayed is different than expected:

- Ensure that the PA catheter balloon is completely deflated
- Repeat wedge measurement as specified in "Initiating a Wedge Measurement" on page 4-16 of this manual, and allow at least 3 respiratory cycles of data
- Verify end-expiration using the P_{AIRWAY} waveform on the display and observing patient's breathing pattern
- Adjust PA wedge cursor to end-expiratory wedge value

This section contains configuration and operational information specific to Cardiac Output (CO) Monitoring.

Overview 5-2
Cardiac Output Display 5-2
Preparing for Cardiac Output Monitoring 5-3
Suggested Cardiac Output Procedure 5-5
Setting Up Cardiac Output Parameters 5-6
Cardiac Output CO Trials Tab 5-6
Cardiac Output Alarms Tab 5-10
Cardiac Output Site-Scale Tab 5-12
Cardiac Output Cardiac Calcs Tab 5-15
Summary of Cardiac Output Alarms 5-19
Problem Resolution 5-20
Care and Cleaning 5-21

Overview

The Narkomed 6400 with the Integrated Patient Monitor option measures and displays thermodilution cardiac output. The clinician can compare multiple trials and waveform displays. The system calculates the average for up to four results.

Edwards and other probes can be used to obtain cardiac output results. These probes are recognized when the clinician enters the calibration constant data, catheter type, and other pertinent information in the cardiac output notebook.

Cardiac Output Display

The cardiac output and blood temperature are displayed in the cardiac output parameter box which is labeled CO.

The numeric display range is 0.2 to 15 liters/minute for cardiac output and 30° C to 42° C for blood temperature. The values are updated every two seconds.

Figure 5-1. Cardiac Output (CO) Parameter Box



The blood temperature trend is represented as a single line. The low scale value is 20°C and the high scale value is 45°C.

Trends are displayed by touching the **[Trend]** button in the main screen taskbar. For complete information on trends, see the Narkomed 6400 Operator's Instruction Manual.

Figure 5-2. Blood Temperature Trend

	45.0
37.0	BT

Integrated Patient Monitor Option Operator's Manual Part N

Blood

Trend

Temperature

Cardiac Output Monitoring

Preparing for Cardiac Output Monitoring

- 1. Determine whether bath probe or in-line sensor will be used.
- 2. Make sure the cardiac output settings in the CO notebook are correct. See "Setting up Cardiac Output Parameters."
- 3. Insert the catheter into the patient.
- 4. Attach the cardiac output cable to the cardiac output connector on the Integrated Patient Monitor module.
- 5. Ensure that the sensor (bath or in-line) is cabled properly as shown in the following illustrations.
- 6. Open the CO notebook to the CO Trials page.
- 7. Begin the injection when the **Inject When Ready** message appears at the bottom of the CO Trials page. Refer to "Cardiac Output CO Trials Tab" for more information.
- 8. Begin the next injection when the **Inject When Ready** message appears again after the **CO Complete** message is displayed.









Cardiac Output Monitoring

5

Suggested Cardiac Output Procedure

- 1. Set up the appropriate cables for an in-line sensor or bath probe sensor, depending on which procedure will be used.
- 2. Touch the cardiac output parameter box. Open the notebook to the CO Trials page. An **Inject When Ready** message should be displayed.
 - **NOTE:** If Other is selected as the catheter type, a computation constant must be entered manually for the **Inject When Ready** message to display (see "Cardiac Output Site-Scale Tab" later in this section).
- 3. Make other changes to the parameter settings as needed. Settings for these parameters are located on the Site-Scale page. Typical adjustments include:
 - type of injectate temperature probe
 - injectate volume
 - catheter size
 - computation constant
- 4. Perform the injection. The **Computing** message is displayed and the waveform appears on-screen.
- 5. The message **CO Complete** appears when the calculation is complete. The cardiac output value and the average are displayed.
- 6. Perform another injection when the **Inject When Ready** message is displayed.
- 7. Perform as many injections as needed. Up to four measurements can be taken. When the fifth injection is complete, the first result taken is deleted.
- 8. When any injection is complete, touch **[Reject]** to delete the last displayed trial results. The average is recalculated automatically.

5

Setting Up Cardiac Output Parameters

Touch the cardiac output parameter box (labeled CO) anywhere except the alarm bell to display the cardiac output notebook.

Cardiac
Output CO
Trials TabTouch the [CO Trials] tab to display the CO Trials page.NOTE: This notebook page must be displayed for a cardiac output
measurement to be successfully performed.

Figure 5-5. Cardiac Output CO Trials Page



WaveformThe CO Trials page contains up to four waveform graphs, with the most
recent measurement at the bottom of the page and the oldest at the top.
After four readings are taken, the next cardiac output will replace the
oldest reading. In addition to the waveform, each graph contains the
following data:

- the cardiac output rate and time the reading was taken
- the computational constant (CC) that was used for the reading
- the injectate temperature (IT)
- the blood temperature (BT)

The **READY** message appears in waveform channels that are ready to accept new waveform data.

Integrated Patient Monitor Option Operator's Manual Part Num

Cardiac Output Monitoring

5

CO Average	An average (Avg :) of the most recent measurements is calculated automatically and displayed below the waveforms. If more than four measurements are taken, only the four displayed waveforms are included in the calculation.
Reject	Touch the [Reject] button to remove the most recent cardiac output graph and recalculate the average. The reading is deleted from the cardiac notebook page, the data log, the trend, and the vital signs report. The next cardiac output reading will be placed in that space.
	The [Reject] button is disabled during a cardiac output computation and when there are no cardiac output graphs present.
Message Area	Messages are displayed at the bottom of the CO Trials page. These messages indicate system status. The messages and their meanings are listed in the following table.

Table 5-1. CO Messages

Message	System Status
INJECT WHEN READY	The system is ready to perform a cardiac output reading. Begin the injection.
COMPUTING	The system is processing the results and calculating the outcome.
CO COMPLETED	The cardiac output waveform is complete.
PLEASE WAIT	The system is not ready for the injection process.
UNSTABLE BT	The system cannot perform a measurement due to unstable blood temperature readings.
UNABLE TO COMPUTE WITH CC=0	The clinician has chosen "Other" for the catheter type on the Site-Scale page but has not set a computation constant.

USE: The **[USE:]** button is used to select the pressure source for future cardiac calculations. The currently selected pressure source is shown as part of the button label.

The **[USE:]** button is disabled while a cardiac output measurement is in progress.

1. Touch the **[USE:]** button to display the Change USE Selection dialog box:



Change USE Selection		
USE: PAWP for Cardiac Calculations		
PAWP PAD	LA	
Confirming this selection will erase all C.O. trials.		
ΟΚ	CANCEL	

- 2. Touch the button for the desired pressure source: PAWP, PAD, or LA.
- 3. Touch the **[OK]** button to confirm the selection. All future cardiac calculations will be done using the newly selected source, all existing cardiac trials are deleted, and the dialog box is removed from the screen.
 - Touch the **[Cancel]** button to close the dialog box without making any changes to the source selection or cardiac output trials.

5

Trial Calcs Use this option to display a dialog box that contains the measured cardiac output value and other vital signs for each cardiac output trial, along with an average of the displayed trials. If the patient's height and weight have been entered in the Cardiac Calculations page (see "Cardiac Output Cardiac Calcs Tab" on page 5-15), various calculated parameters are displayed in the bottom half of the window as well.

The time of the last successful cardiac output trial is displayed in the title bar. Trial data that was rejected is removed from the dialog box and is excluded from the average calculations. Only the currently selected pressure source is displayed (PAWP, PAD, or LA).

Touch the [Trial Calc] button to display the Trial Calcs dialog box.

To remove the dialog box from the display, touch the **[Trial Calc]** button again or touch the **[Close]** button inside the dialog box.





Cardiac Touch the Alarms tab in the CO notebook to display the Alarms page: Output Alarms Tab

Figure 5-8. Cardiac Output Alarms Page



Alarm Control Use this option to turn the blood temperature alarm notification system on or off. Alarms are available for blood temperature, not for cardiac output.

Touch **[Alarm Control]** or its associated selection field until the desired setting is displayed:

- On turns the cardiac output alarm notification system on
- Off turns the cardiac output alarm notification system off
- Stby (standby) suspends the cardiac output alarm notification system until valid blood temperature data is received, then the cardiac output notification system is turned on automatically
- **NOTE:** Cardiac output alarms are turned off automatically while a SRVC IPM MON advisory alarm is active.

Cardiac Output Monitoring

-
-

Blood Temp Limits-Set Hi	This option sets the high alarm limits for blood temperature. The range of available settings is 30.1 to 42.0 °C. The lowest possible setting for the high alarm limit must be one degree higher than the current low alarm setting. The default setting is 40.0 °C.
	1. Touch [Set Hi] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Hi] button.
	• Use the fine adjustment to change the setting by 0.1 $^{\circ}$ C.
	• Use the coarse adjustment to change the setting by 1.0 $^{\circ}$ C.
Blood Temp Limits-Set Lo	This option sets the low alarm limits for blood temperature. The range of available settings is 30.0 to 41.9 °C. The lowest possible setting for the low alarm limit must be one degree lower than the current high alarm setting. The default setting is 30.0 °C.
	1. Touch [Set Low] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Lo] button.
	• Use the fine adjustment to change the setting by 0.1 °C.
	• Use the coarse adjustment to change the setting by 1.0 $^{\circ}$ C.
Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the CO parameter box ON or OFF.
AUTOSET Narrow	This option sets the alarm limits to $\pm -2^{\circ}$ C of the current blood temperature reading within the 30° C to 42° C range. When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Narrow] . The change is automatic. The new settings appear next to the [Set Hi] and [Set Lo] buttons.
AUTOSET Wide	This option sets the alarm limits to +/- 4° C of the current blood temperature reading within the 30° C to 42° C range. When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Wide] . The change is automatic. The new settings appear next to the [Set Hi] and [Set Lo] buttons.

5

Cardiac
Output Site-
Scale TabTouch the Site-Scale tab in the CO notebook to display the Site-Scale page.If any of the values on the Site-Scale page are changed, any existing
cardiac output graphs on the CO Trials page are removed. The next
reading with the new settings will be placed in the first graph on the CO
Trials page.

Figure 5-9. Cardiac Output Site-Scale Page



- Units The system is currently set to calculate and display in Celsius. The option to change the setting is disabled.
- Comp.Use this option to enter the computation constant into the system when a
probe other than an Edwards, Abbot, Arrow, or Ohmeda probe is used.
This option is available only when "Other" is selected for **Catheter Type**.
The range for the computation constant is 0.000-0.999.
 - **NOTE:** The cardiac output waveforms displayed on the CO Trials page and any associated calculations are deleted when the computation constant is changed.
 - 1. Touch **[Comp. Constant]** or its associated selection field to activate the slider bar.
 - 2. Move the slider control until the desired setting appears next to the **[Comp. Constant]** button.

5

- Use the fine adjustment to change the setting by 0.001.
- Use the coarse adjustment to change the setting by 0.010.
- **NOTE:** The catheter type must be set to Other before manually entering the computation constant or the Comp. Constant button will be unavailable. When the computation constant is entered, the catheter size, injectate volume, and injectate probe settings do not need to be changed.
- Catheter Type Use this option to specify the type of catheter used for the injection. When Edwards, Abbott, Arrow, or Ohmeda is selected, the system calculates the computation constant automatically. When Other is selected, the computation constant must be entered manually.
 - **NOTE:** The cardiac output waveforms displayed on the CO Trials page and any associated calculations are deleted when the catheter type is changed.

Touch **[Catheter Type]** or its associated selection field until the correct setting is displayed. The options are Edwards, Abbott, Arrow, or Ohmeda, or Other.

Changing the catheter type causes the remaining CO parameters to be automatically changed to settings commonly used with that type of catheter (see Table 5-2.) The clinician can subsequently adjust those settings manually to values that are permitted for a given catheter type. For a complete list of allowable settings, see "Catheters/Settings Supported by the Narkomed 6400" on page A-2-7.

Catheter Type	Injectate Probe	Catheter Size	Injectate Volume	Cumputation Constant
Edwards	Inline	7.5	10 cc	-
Abbott	Inline	7(85 cm)	5 cc	-
Arrow	Inline	7	3 cc	-
Ohmeda	Inline	7	10 cc	-
Other	Inline	7	3 cc	0.0000

 Table 5-2. Automatic Values Assigned Based on Catheter Type

Injectate Probe	Use this option to specify the type of probe being used.
	NOTE: The cardiac output waveforms displayed on the CO Trials page and any associated calculations are deleted when the injectate probe is changed.
	Touch [Injectate Probe] or its associated selection field until the correct probe is displayed - Inline or Bath.
	NOTE: Injectate probe selection must be compatible with the other CO parameter settings as specified in "Catheters/Settings Supported by the Narkomed 6400" on page A-2-7. Incompatible selections will be unavailable.
Catheter Size	Use this option to specify the catheter size when an Edwards catheter is used. The catheter size is used to calculate the computation constant.
	NOTE: The cardiac output waveforms displayed on the CO Trials page and any associated calculations are deleted when the catheter size is changed.
	Touch [Catheter Size] or its associated selection field until the desired size is displayed. The options, which vary depending on catheter type and other selections, include 4, 5, 6, 7, 7.5, and 8.
	NOTE: Catheter size selection must be compatible with the other CO parameter settings as specified in "Catheters/Settings Supported by the Narkomed 6400" on page A-2-7. Incompatible selections will be unavailable.
Injectate Volume	Use this option to specify the volume of the injectate in cubic centimeters. The volume setting is used to calculate the computation constant.
	NOTE: The cardiac output waveforms displayed on the CO Trials page and any associated calculations are deleted when the injectate volume is changed.
	Touch [Injectate Volume] or its associated selection field until the correct setting is displayed. The options, which vary depending on catheter type and other selections, include 3, 5, and 10.
	NOTE: Injectate Volume selection must be compatible with the other CO parameter settings as specified in "Catheters/Settings Supported by the Narkomed 6400" on page A-2-7. Incompatible selections will be unavailable.

5

Cardiac	Touch the Cardiac Calcs tab in the CO notebook to display the Cardiac
Output	Calculations page (Figure 5-10.)
Cardiac Calcs Tab	The Cardiac Calculations page can also be displayed from the Utilities Notebook (Figure 5-11.) Follow this procedure:

1. Touch [To Secondary Keys] on the main screen taskbar.

Several control buttons change to choose additional tasks.

- 2. Touch **[Utilities Notebook]** on the new taskbar. The utilities notebook appears.
- 3. Touch the [Cardiac Calcs] tab to access the Cardiac Calculations page.

Figure 5-10. CO Cardiac Calculations Page in CO Notebook

Figure 5-11. CO Cardiac Calculations Page in Utilities Notebook



The Cardiac Calculations page displays both current and saved cardiac average calculations for the patient parameters listed in Table 5-1. The current data appears in the left-most column on a white background, and the saved data appears in the three remaining columns. The time of the last CO measurement for each set of calculations is shown at the top of each column. The parameters in the top half of the page are measured parameters (with the exception of the Height and Weight which are

entered by the clinician), while the parameters in the bottom half of the page are calculated using the measured parameters. All data fields are initially blank until a valid value is either measured or supplied by the clinician.

Cardiac calculations can be saved, deleted, and changed on this page as explained in the following paragraphs. A slider bar is provided to access data currently off the screen.

Parameter Label	Definition
CO	Cardiac Output (measured average)
HR	Heart Rate (measured average)
MAP	Mean Arterial Pressure (measured average)
CVP	Central Venous Pressure (measured average)
PAM	Pulmonary Artery Mean Pressure (measured average)
PAWP	Pulmonary Artery Wedge Pressure (last measurement)
PAD	Pulmonary Artery Diastolic Pressure (measured average)
LA	Left Atrial Pressure (measured average)
HT	Patient height (entered by clinician)
WT	Patient Weight (entered by clinician)
BSA	Body Surface Area (calculated)
CI	Cardiac Index (calculated)
SV	Stroke Volume (calculated)
SVR	Systemic Vascular Resistance (calculated)
SVRI	Systemic Vascular Resistance Index (calculated)
PVR	Pulmonary Vascular Resistance (calculated)
PVRI	Pulmonary Vascular Resistance Index (calculated)
LVSWI	Left Ventricular Stroke Work Index (calculated)
RVSWI	Right Ventricular Stroke Work Index (calculated)

Table 5-1. Cardiac Calculations Page Parameter Labels and Definitions

Cardiac Output Monitoring

5

Save	Touch the [Save] button to save the current calculations (shown in the left- most column on the Cardiac Calculations page) into temporary storage. All trial graphs on the cardiac output CO Trials page are deleted, and all values are removed from the Trial Calcs dialog box and from the current values column in the Cardiac Calculations page (with the exception of Height and Weight).
	Up to 30 sets of calculations can be stored. The storage is cleared at power- up or [New Case] selection.
	The [Save] button is disabled during a cardiac output measurement and when there are no current values available (i.e., when all current values columns on the Cardiac Calculations page are blank).
Delete	This option is used to delete saved calculations from the Cardiac Calculations page. The option is disabled when there is no saved data available.
	1. Touch the entry or entries to be deleted on the Cardiac Calculations page. Use the scroll bar to view data currently off the screen. Note that only data in the <i>saved</i> columns can be deleted.
	2. Touch the [Delete] button. The following confirmation dialog box is displayed:

Figure 5-12. Delete Saved Calculations Dialog

Delete Saved Calculations	:		
The highlighted calculations will be erased.			
OK	Cancel		

- 3. Touch the **[OK]** button to confirm the deletion. The selected calculations are deleted from the Cardiac Calculations page. The adjacent data shifts to the left to fill in the deleted column(s).
 - Touch the **[Cancel]** button to close the dialog box without deleting any saved data.

Changing The clinician can enter or change current data for any of the measured parameters on the Cardiac Calculations page (shown in the top half of the page). Changes are not allowed when a cardiac output measurement is in progress.

To enter/change data on the Cardiac Calculations page:

1. Touch the desired data value in the current values (first) column on the Cardiac Calculations page. The Change Value dialog box is displayed:





- 2. Use the left and right arrow keys to adjust the values up or down. Default values and ranges are provided in Table 5-2.
- 3. Touch the **[OK]** button to confirm the change. Generally, any new data entry or any change to an existing entry will delete all trial graphs on the Cardiac Output CO Trials page, all values in the Trial Calcs dialog box, and all current values on the Cardiac Calculations page. The exception is the *first time* that patient Height or Weight is entered by the clinician: parameter values are recalculated with the supplied height and weight data, but existing trials or calculations are not deleted.
 - Touch the **[Cancel]** button to close the dialog box without changing any saved data.

5

Select Units Use this option to select English and Metric units for Height and Weight.

- 1. Touch anywhere in the current values column on the Cardiac Calculations page. The Change Value dialog box is displayed (see Figure 5-13).
- 2. Touch the **[Select Units]** button to toggle between English and Metric units.
- 3. Touch the **[OK]** button to confirm the change.
 - Touch the **[Cancel]** button to close the dialog box without changing the units.

Parameter	Initial Default	Upper Limit	Lower Limit	Adjustment Increment
CO	6.0	20.0	0.2	0.1
HR	100	250	20	1
MAP	90	200	1	1
CVP	10	100	-5	1
PAM	20	100	-5	1
PAWP	10	100	-5	1
PAD	10	100	-5	1
LA	10	100	-5	1
Weight (default units:metric)	68 kg/150 lb	453.5 kg/ 1000 lb	0.5 kg/0.5 lb	0.5 kg/0.5 lb
Height (default units:metric)	167.5 cm/ 66 in	300 cm/ 118.0 in	1.0 cm/0.5 in	0.5 cm/0.5 in

Table 5-2. Cardiac Calculations Parameter Defaults and Ranges

Summary of Cardiac Output Alarms

This section contains all Warning, Caution, and Advisory alarms associated with cardiac output monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 5-3. Cardiac Output Cautions

Message	Condition	Suggested Action
T BLOOD HIGH	Blood temperature exceeds the alarm limits	Check the patient. If needed, change the high alarm limit setting.
T BLOOD LOW	Blood temperature is below the alarm limit	Check the patient. If needed, change the low alarm limit setting.

Table 5-4. Cardiac Output Advisories

Message	Condition	Suggested Action
BT SENSOR ERR	The blood temperature sensor is not working	Make sure the sensor is properly placed and connections are secure. If this is not the problem, contact an authorized representative of DrägerService.
INJ TEMP HI	Injectate temperature is too high	Ensure that the injectate temperature is within the 0° to 30° C range.
IT SENSOR ERR	Injectate temperature sensor failure	Ensure that the injectate temperature is within the 0° to 30° C range.
		Check or replace sensor.

Problem Resolution

Inaccurate cardiac output readings:

- When room temperature solution is used, be sure the bag is not in a warmer area of the room or exposed to other solutions or equipment.
- Do not allow body temperature to affect the solution. Always hold the syringe by the plunger, not by the barrel.
- Allow at least 1 minute between injections to allow the baseline to stabilize.
- To reduce respiratory noise, inject at the patient's end expiration.
- Make sure the solution temperature is at least 10° lower than the patient's temperature.

Cardiac Output Monitoring

5

Care and Cleaning

Remove any adhesive used to attach the cable to the patient. Use a mild detergent in warm water with a soft cloth to wipe the cable clean. Check each cable to corrosion or other signs of damage or deterioration. Make sure the cable is dry before storing. Store the cable flat or hang the cable to prevent damage during storage.

To avoid damaging the components and accessories, **do not**:

- use strong solvents such as acetone, alcohol, ammonia, chloroform on the cable.
- autoclave the cable.

Temperature Monitoring

This section contains configuration and operational information specific to Temperature Monitoring.

Overview	
Temperature Display 6-2	•
Preparing the Patient for Temperature Monitoring 6-3	;
Setting Up Temperature Parameters 6-4	Ļ
Temperature Setup Tab 6-4	Ļ
Temperature Alarms Tab 6-5	,
Temperature Site-Scale Tab 6-8	;
Summary of Temperature Alarms 6-9)
Problem Resolution 6-10)
Care and Cleaning 6-10)

Temperature Monitoring

Overview

6

The Narkomed 6400 with the Integrated Patient Monitor option monitors T1 and T2 temperature and calculates the difference between the two. This is in addition to the blood temperature readings available using the cardiac output channel.

Temperature data is acquired by thermistor temperature probes (YSI Series 400 or Series 700). The system automatically senses the type of probe used and processes the data accordingly.

T1 and T2 temperatures are automatically included in the data log in 1-minute intervals.

Temperature Display

The information displayed in the Temperature parameter box depends on the settings made in the Temperature notebook and the System Setup notebook. The temperature parameter box can display the temperatures monitored, the difference between T1 and T2, the site labels, and the alarm limits. Temperature can be displayed in either degrees Centigrade or degrees Fahrenheit.

The numeric display range for temperature is 0 to $45.1^{\circ}C$ (32.0 to $113.0^{\circ}F$). The numeric display range for temperature difference is -5 to $5^{\circ}C$ (-9.0 to $9.0^{\circ}F$). The values are updated every two seconds.

Figure 6-1. Temperature (Temp) Parameter Box



Temperature The two temperature parameters that are trended-T1 and T2-are each represented as a single line. The low scale value is $20^{\circ}C$ ($68^{\circ}F$) and the high scale value is $45^{\circ}C$ ($113^{\circ}F$).

Trends are displayed by touching the **[Trend]** button in the main screen taskbar. For complete information on trends, see the Narkomed 6400 Operator's Instruction Manual.

Temperature Monitoring

6

Figure 6-2. Temperature Trend

97.9	 45.0
J7.2 95.5	Temp
25.5	20.0

Preparing the Patient for Temperature Monitoring

- 1. Place the temperature probe(s) on the patient.
- 2. Attach the patient cable to the Temperature connector(s) on the Integrated Patient Monitor module.
- 3. Set the Temperature monitoring variables in the Temperature notebook.

Setting Up Temperature Parameters

Touch the Temperature parameter box anywhere except the alarm bell to display the Temperature notebook.

TemperatureTouch the Setup tab in the Temperature notebook to display the setup
options:

Figure 6-3. Temperature Setup Page



T1 - T2 The setting for this option determines whether the difference between T1 and T2 temperature measurements are displayed in the parameter box:

Touch **[T1 - T2]** or its associated selection field to toggle the display ON or OFF.

The difference will be displayed only if both T1 and T2 values are present and the difference is within the valid range of -5 to 5 $^{\circ}$ C (-9 to 9 $^{\circ}$ F). The default setting is OFF.

Select Units This setting determines whether the temperature readings are displayed in degrees Centigrade or Fahrenheit.

Touch **[Select Units]** or its associated selection field to toggle the temperature units between °C and °F.

Temperature Monitoring

6

TemperatureTouch the Alarms tab in the Temperature notebook to display the alarmsAlarms Tabpage:

Figure 6-4. Temperature Alarms Page

Temperature Alarms	Setup
Alarm Control ON Alarm Limits	Alarms
Channel T1 Set Hi 23.9	Site - Scale
Set Lo	Print
Disp Limits Off	Volume
AUTOSET Narrow Wide	Calc.

Alarm Control Use this option to turn the temperature alarm notification system on or off.

Touch **[Alarm Control]** or its associated selection field until the desired setting is displayed:

- On turns the temperature alarm notification system on
- Off turns the temperature alarm notification system off
- Stby (standby) suspends the temperature alarm notification system until valid temperature data is received, then the notification system is turned on automatically

Alarm Limits-
ChannelUse this option to specify the temperature channel for making high and
low limit alarm adjustments.

Touch **[Alarm Control]** or its associated selection field until the desired channel (T1 or T2) or channel difference (T1 - T2) is displayed.

Temperature Monitoring

After the channel is selected, high and low alarm limits can be set. The settings for each channel are independent and do not affect settings for other channels.

Alarm Limits-
Set HiThis option sets the high alarm limits for temperature. See Table 6-1 for
available settings.

- 1. Touch [Set Hi] or its associated selection field to activate the slider bar.
- 2. Move the slider control until the desired setting is displayed next to the **[Set Hi]** button.
 - Use the fine adjustment to change the setting by 0.1 °C or °F.
 - Use the coarse adjustment to change the setting by 1.0 °C or °F.

Table 6-1. Temperature High Alarm Limits

Alarm Limit	Default	Valid Settings (must be at least 0.1° greater than low alarm limit)
High T1 and T2	42.0 °C 107.6 °F	0.1 to 45.0 °C 32.1 to 113.0 °F
High T1 - T2	5.0 °C 9.0 °F	-4.9 to 5.0 °C -8.9 to 9.0 °F

Alarm Limits-
Set LoThis option sets the low alarm limits for temperature. See Table 6-2 for
available settings.

- 1. Touch **[Set Low]** or its associated selection field to activate the slider bar.
- 2. Move the slider control until the desired setting is displayed next to the **[Set Lo]** button.
 - Use the fine adjustment to change the setting by 0.1 °C or °F.
 - Use the coarse adjustment to change the setting by 1.0 °C or °F.

Table 6-2. Temperature Low Alarm Limits

Alarm Limit	Default	Valid Settings (must be at least 0.1° lower than high alarm limit)
Low T1 and T2	30.0 °C 86.0 °F	0 to 44.9 °C 32.0 to 112.9 °F
Low T1 - T2	-5.0 °C -9.0 °F	-5.0 to 4.9 °C -9.0 to 8.9°F
Temperature Monitoring

Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the temperature parameter box ON or OFF.
AUTOSET Narrow	This option sets the alarm limits for both temperature channels to $\pm 2^{\circ}$ C ($\pm 3.6^{\circ}$ F) of the current temperature reading within the 0 to 45° C (32 to 113° F) range ($\pm 1^{\circ}$ C or $\pm 1.8^{\circ}$ F for the temperature difference). When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Narrow] . The change is automatic. The new settings appear next to the [Set Hi] and [Set Lo] buttons.
AUTOSET Wide	This option sets the alarm limits for both temperature channels to $\pm 4^{\circ}$ C ($\pm 7.2^{\circ}$ F) of the current temperature reading within the 0 to 45° C (32 to 113° F) range ($\pm 2^{\circ}$ C or $\pm 3.6^{\circ}$ F for the temperature difference). When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Wide] . The change is automatic. The new settings appear next to the [Set Hi] and [Set Lo] buttons.

6

TemperatureTouch the Site-Scale tab in the Temperature notebook to display the Site-
Scale page:Site-ScaleScale page:TabScale page:

Figure 6-5. Temperature Site-Scale Page



T1 Site,T2 Site Each site button establishes the patient site used for each temperature probe used.

The site chosen appears next to the temperature reading in the Temperature parameter box.

Touch **[T1 Site]** or **[T2 Site]** or the associated selection field to display the list of options, then touch the desired site. The sites listed include:

Temperature Monitoring

6

Name	site
NONE	none
ESO	esophageal temperature
NAS	nasopharyngeal temperature
ТҮМР	tympanic temperature
RECT	rectal temperature
BLAD	bladder temperature
AXIL	axillary temperature
SKIN	skin temperature
AIRW	airway temperature
ROOM	room temperature
МҮО	myocardial temperature
UKNW	unknown - use this option for temperature probe placements that are not listed among the available sites

Table 6-3. Temperature Sites

Summary of Temperature Alarms

This section contains all warning, caution, and advisory alarms associated with temperature monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 6-4. Temperature Cautions

Message	Condition	Suggested Action
T1/T2 DIFF	The difference between T1 and T2 high or low alarm limit is exceeded	If the alarm setting is incorrect, reset the T1-T2 limit.
TEMP (number) HIGH	The temperature reading exceeds the high alarm limit	If the alarm setting is incorrect, reset the high alarm limit.
TEMP (number) LOW	The temperature reading is below the low alarm limit setting	If the alarm setting is incorrect, reset the low alarm limit.

Table 6-5. Temperature Advisories

Message	Condition	Suggested Action
TEMP 1 DISC	The T1 connection is interrupted	Check the cable connection to the IPM.
TEMP 2 DISC	The T2 connection is interrupted	Check the cable connection to the IPM.
T1 SENSOR ERR	The T1 connection is interrupted	Check the cable connection to the temperature probe.
T2 SENSOR ERR	The T2 connection is interrupted	Check the cable connection to the temperature probe.
SRVC CV MON	Internal system fault	Contact an authorized representative of DrägerService.

NOTE: The following alarms are not displayed at initial power-up if present: TEMP 1 DISC, TEMP 2 DISC, T1 SENS ERR, T2 SENS ERR.

Problem Resolution

Inaccurate temperature readings:

Check probe placement and adhesion

Care and Cleaning

Use only detergents that do not contain alcohol for disinfection. Use a lintfree cloth to rub the probe down to the direction of the connector.

To avoid damaging the components and accessories, **do not:**

- heat the probe above 100° C
- sterilize the probe in steam
- use alcohol or other strong solutions
- reuse or resterilize disposable probes

This section contains configuration and operational information specific to Noninvasive Blood Pressure (NIBP) Monitoring.

Overview
Patient Preparation for NIBP Monitoring 7-4
Selecting the Blood Pressure Cuff 7-4
Placing the Cuff 7-5
Setting Up NIBP Parameters 7-6
NIBP Setup Tab 7-6
NIBP Alarms Tab 7-8
NIBP Volume Tab 7-10
Summary of NIBP Alarms 7-11
Problem Resolution 7-13
Care and Cleaning

Overview

7

The Narkomed 6400 with the Integrated Patient Monitor option monitors noninvasive systolic, diastolic, and mean blood pressure.

The current blood pressure measurements are displayed in the NIBP parameter box. Pulse data obtained during an NIBP measurement can also be displayed. In addition, systolic, diastolic, and mean readings are displayed numerically in the Data Log.

The oscillometric method is used to measure pressure. The Integrated Patient Monitor has a pressure transducer and microprocessor to translate cuff pressure oscillations into blood pressure readings. The Integrated Patient Monitor module accepts input from a full range of blood pressure cuffs-from neonatal to large adult sizes.

Some conditions can interfere with accurate blood pressure measurement if a regular arterial pressure pulse cannot be detected, including:

- patient movement (excessive movement, shivering, or convulsing)
- cardiac arrhythmias
- rapid blood pressure changes
- severe shock
- heart rate extremes of less than 30 bpm or more than 300 bpm

WARNING: Noninvasive blood pressure measurement is not recommended for patients in shock or patients with arrhythmias, extremely high or low blood pressure, or an extremely high or low pulse rate.

NIBP Display

NIBP readings are displayed in the NIBP parameter box:

Figure 7-1. NIBP Parameter Box



- systolic blood pressure (top position in large numerals)
- diastolic blood pressure (bottom position in large numerals)
- mean blood pressure (middle position in smaller numerals surrounded by parentheses)
- time of last successful NIBP measurement in 24-hour format (at top of parameter box near the systolic value)
- pulse rate (PLS) below the diastolic blood pressure value (if enabled)
- cuff pressure status on a bar graph (during measurement only)

- countdown timer until next measurement (shown in numerics and illustrated by a pie chart)
- cuff inflation mode (ADULT, PED, or NEO) in lower left corner of parameter box

The numeric display range for all NIBP values is 0 to 300 mmHg. All values are updated every time an NIBP reading is performed. The reading is removed when it is 15 minutes old or when a new cuff inflation mode is selected.

NOTE: The systolic, diastolic, and mean values are removed from the NIBP parameter box if NIBP measurement error alarm messages are posted in the alarm window.

NIBP monitoring is performed in an automatic cycle or in stat mode. The automatic cycle inflates the cuff, takes readings, then deflates the cuff before the next inflation in the time period (interval) set in the Setup notebook. Stat mode provides continuous blood pressure readings for 5 minutes.

Three buttons for making these settings are located in the NIBP parameter box:

START - starts, resumes, or continues blood pressure measurement in the automatic cycle set in the NIBP notebook

- STOP deflates the cuff and suspends blood pressure measurement
- **STAT** starts continuous blood pressure measurements for 5 minutes

These functions are also performed by pressing the START/STOP and STAT control keys located on the front of the Integrated Patient Monitor module.

- **NOTE:** Under no condition will the Narkomed 6400 with a Integrated Patient Monitor take blood pressure measurements continuously for over 5 minutes. If STAT is pressed during a pressure measurement cycle, the time the cuff was already inflated is taken into account for the 5-minute count.
- **NOTE:** The following conditions may cause the NIBP parameter box to display only the mean blood pressure value, and not display the associated systolic and diastolic values:
 - very low systolic and diastolic amplitude fluctuations (e.g. patients in shock)
 - very small difference between the mean blood pressure and the systolic pressure or the mean blood pressure and the diastolic pressure
 - loss of system identity (e.g. loose connections or worn parts)

If this occurs, be sure to perform a visual inspection to ensure system integrity.

Patient Preparation for NIBP Monitoring

- 1. Determine the patient's limb circumference
- 2. Select the correct cuff size for the patient. See "Selecting the Blood Pressure Cuff" that follows for more information about cuff selection.
- 3. Make sure the cuff is completely deflated.
- 4. Apply the cuff to the patient in the correct position described in "Placing the Blood Pressure Cuff" that follows.
- 5. Connect the line to the NIBP connection on the Integrated Patient Monitor module.
- 6. Arrange the tubing to the right or left of the arm to avoid kinking if the elbow is bent.
- 7. Make sure the system settings in the NIBP parameter notebook are correct. Instructions for making setting selections follow.
- 8. When the settings are verified, touch START or STAT to begin readings.
- **WARNING:** Periodically check the patient's limb extremities for signs of circulation impairment, including color changes, temperature changes, and insensitivity. Check the limb more frequently when continuous readings or short automatic intervals such as 1 or 2 minutes are used. Short intervals are not recommended for extended periods. Stop pressure measurements immediately if circulation impairment is evident.

Selecting the Blood Pressure Cuff

Be sure to choose the correct cuff size, patient age setting, and to place the cuff correctly. The cuffs provided in the Accessory Kit are color-coded and the sizes are indicated clearly on the cuff.

Cuff Size	Limb Circumference	Color
Neonatal	6 to 11 cm	gray
Infant	10-19 cm	tan
Pediatric	18-26 cm	green
Adult	25-35 cm	blue
Large Adult	33-47 cm	red
Thigh	46-66 cm	brown

Use the following table as a general reference for cuff size selection:

Integrated Patient Monitor Option Operator's Manual

7

Use the patient's age range (Neonatal, Pediatric, or Adult) to determine the correct pressure setting. Patient Age is an option in the NIBP notebook Setup page. See "Setting NIBP Parameters - NIBP Setup Tab" for more information.

Placing the Cuff

When applying the cuff, place the center of the cuff inflation bag over the artery (for the brachial artery, place the bag on the inside of arm, above the elbow). Make sure the cuff fits securely on the limb and that the INDEX line falls between the two RANGE lines. If the INDEX line does not fall between the RANGE lines, select a smaller or larger cuff. The cuff can be used on either a left or right extremity, but the left is preferred.

CAUTION: Do not place the cuff on a limb being used for infusion. Tissue damage around the catheter can result when the infusion is slowed or blocked during cuff inflation.

Position the cuff at the same level as the patient's heart for an accurate measurement. Placing the cuff above the heart causes the reading to be low. Placing the cuff below the heart causes the reading to be high. Use these general rules when the cuff cannot be placed at heart level:

- For every inch above the heart, add 1.8 mmHg to the reading.
- For every inch below the heart, subtract 1.8 mmHg from the reading.

Figure 7-2. Positioning the Cuff



7

Setting Up NIBP Parameters

Touch the NIBP parameter box (labeled NIBP) in the area to the right of the STOP button to display the NIBP notebook:

NIBP SetupTouch the Setup tab in the NIBP notebook to display the setup options:Tab

Figure 7-3. NIBP Setup Page



Auto Interval The interval is the period of time from when the cuff inflates to the next cuff inflation. This interval consists of inflation, measurement, deflation, and rest period before the next interval begins.

[Auto Interval] controls the interval for blood pressure readings. The interval can be set from 2 to 30 minutes in increments of 1/2 minute. The default interval is 5 minutes.

- **NOTE:** At interval settings of 5, 10, 15, and 30 minutes, the NIBP inflation timer is synchronized with the Narkomed 6400 system clock, such that the next measurement will complete at an interval matching the system clock. Once synchronized, subsequent NIBP measurements will occur at the selected interval setting.
- **NOTE:** Short interval settings are not recommended when blood pressure measurement must be taken for an extended period.

NIBP Monitoring

	1. Touch [Aut e slider bar.	o Interval] or its associated selection field to activate the	
	2. Move the slinterval set	lider control until the desired setting is displayed. The string is displayed below the [Auto Interval] button.	
	• Use incr	the fine adjustment to change the setting by 1/2 minute ements.	
	• Use incr	the coarse adjustment to change the setting by 5 minute ements.	
	NOTE: If the the time a	interval is changed during a case without pressing STOP, ning cycle does not start over. The new interval includes the lready elapsed under the previous setting.	
	The pie chart a parameter box	and associated countdown numerics located in the NIBP show the new interval.	
Patient Age	This option cor on the patient?	ntrols the initial and maximum cuff pressure setting based s age.	
	Touch Patient Age or its associated selection field until the correct setting is displayed. The options include:		
	Neonatal	for patients ranging in age from newborn to age 2. At this setting, the first inflation pressure is 100 mmHg. Subsequent inflation pressure is 20 mmHg over the last systolic pressure up to 150 mmHg.	
	Pediatric	for patients ranging in age from 3 to 13 years. At this setting, the first inflation pressure is 130 mmHg, or the last Adult inflation pressure. Subsequent inflation pressure is 30 mmHg over the last systolic pressure up to 250 mmHg.	
	Adult	for patients over 13 years of age. At this setting, the first inflation pressure is 160 mmHg. Subsequent inflation pressure is 30 mmHg over the last systolic pressure up to 300 mmHg.	
	The cuff will no turns the pump selected cuff is	ot exceed maximum pressure. The system automatically p off and the cuff is deflated if maximum pressure for the exceeded.	
Display Pulse	Touch [Display Pulse] or its associated selection field to toggle the display of pulse data in the NIBP parameter box ON or OFF.		

NIBP Alarms Touch the Alarms tab in the NIBP notebook to display the Alarms page: **Tab**

Figure 7-4. NIBP Alarms Page



Alarm Control Use this option to turn the NIBP alarm notification system on or off.

Touch **[Alarm Control]** or its associated selection field until the desired setting is displayed:

- On turns the NIBP alarm notification system on
- Off turns the NIBP alarm notification system off
- Stby (standby) suspends the NIBP alarm notification system. The alarms are turned on automatically as soon as Integrated Patient Monitor communications are established
- **NOTE:** NIBP alarms are turned off automatically while a SERVICE NIBP advisory alarm is active.

Alarm Limits-Use this option to select a pressure parameter for setting high and low
alarm limits.

Touch **[Pressure]** or its associated selection field until the desired parameter is displayed:

7

7
•

	• Sys - for setting the alarm limits for systolic pressure
	 Mean - for setting alarm limits for mean pressure
	 Disc. for setting the clower limits for disctalic pressure
	• Dias - for setting the alarm limits for diastolic pressure
	After the pressure parameter is selected, alarm limits can be set. The settings for each pressure parameter are independent and do not affect settings for other pressure parameters.
Alarm Limits- Set Hi	This option sets the high alarm limits for NIBP. The range of available settings is 1 to 300 mmHg. The lowest possible setting for the high alarm limit must be 1 mmHg higher than the current low alarm setting. The default setting is 300 mmHg.
	1. Touch [Set Hi] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Hi] button.
	• Use the fine adjustment to change the setting by 1 mmHg
	• Use the coarse adjustment to change the setting by 10 mmHg
Alarm Limits- Set Lo	This option sets the low alarm limits for NIBP. The range of available settings is 0 to 299 mmHg. The highest possible setting for the low alarm limit must be one mmHg lower than the high alarm setting. The default setting is 0 mmHg.
	1. Touch [Set Lo] or its associated selection field to activate the slider bar.
	2. Move the slider control until the preferred setting is displayed next to the [Set Lo] button.
	• Use the fine adjustment to change the setting by 1 mmHg
	• Use the coarse adjustment to change the setting by 10 mmHg
Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the NIBP parameter box ON or OFF.
AUTOSET Narrow	This option sets pressure alarm limits for all pressure parameters +/- 25 from the current pressure values within the 0 to 300 mmHg range. When the autoset limit falls below the minimum allowable range, the lowest setting within the acceptable range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Narrow] . The change is automatic. The new range values appear next to the [Set Hi] and [Set Lo] buttons.

AUTOSET This option sets pressure alarm limits for all pressure parameters +/- 50 Wide from the current pressure values within the 0 to 300 mmHg range. When the autoset limit falls below the minimum allowable range, the lowest setting within the acceptable range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.

Touch **[AUTOSET Wide]**. The change is automatic. The new range values appear next to the **[Set Hi]** and **[Set Lo**] buttons.

NIBP Volume Touch the Volume tab in the NIBP notebook to view the Volume page. **Tab**

Figure 7-5. NIBP Volume Page



The system sounds a single tone when the NIBP measurement is complete. Use this option to adjust the volume for the tone.

Move the slider control to adjust the volume. A test tone is provided for each volume adjustment. When the slider is at the highest position, the tone will be at maximum volume. When the slider is at the lowest position, no tone will be annunciated.

7

Summary of NIBP Alarms

This section contains all warning, caution, and advisory alarms associated with NIBP monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 7-1. NIBP Cautions

Message	Condition	Suggested Action
NIBP DIAS HI	The diastolic blood pressure exceeds the high range setting	Check the patient. If in doubt about the measurement, perform an independent blood pressure measurement.
NIBP DIAS LO	The diastolic blood pressure is below the low range setting	Check the patient. If in doubt about the measurements, perform an independent blood pressure measurement.
NIBP MEAN HI	The mean blood pressure is above the high range setting	Check the patient. If in doubt about the measurement, perform an independent blood pressure measurement.
NIBP MEAN LO	The mean blood pressure is below the low alarm setting	If the alarm setting is incorrect, change the mean alarm setting.
NIBP SYSTOLIC HI	The systolic blood pressure reading exceeded the high range setting	Check the patient. If in doubt about the measurement, perform an independent blood pressure measurement.
NIBP SYSTOLIC LO	The systolic blood pressure reading is below the low range setting	Check the patient. If in doubt about the measurement, perform an independent blood pressure measurement.

Table 7-2. NIBP Advisories

Message	Condition	Suggested Action
BP CUFF DISC	The system did not detect a cuff inflation	Check the cuff and its connections and try another measurement. If the cuff is inflated, contact an authorized representative of DrägerService.
BP CUFF ERROR	The cuff will not deflate	Remove the cuff and contact an authorized representative of DrägerService.
CHECK NIBP CUFF	Inflation >5 min., excessive noise, >3 min. for measurement, or exceeds maximum pressure	If the cuff is deflated, try another measurement. Check placement. Check for arrhythmia condition. If the cuff is inflated, remove the cuff and contact an authorized representative of DrägerService.
HIGH NIBP	Excessive pulse amplitude.	Possible excessive patient movement. Check patient and retry measurement.
NIBP LEAK	System pressure leak	Check connections, hoses, cuff.
NIBP STAT	The STAT button was touched and the system is taking continuous blood pressure readings for a period not to exceed 5 minutes	To turn off the STAT mode, touch the STOP or START buttons in the NIBP parameter box.
WEAK NIBP	The system cannot detect a measurable pulse	Perform an independent blood pressure measurement to determine if the system is at fault (see "Problem Resolution"). If a patient condition exists that affects blood pressure measurement, do not use the NIBP method.
SERVICE NIBP	Internal system fault.	Contact an authorized representative of DrägerService.

NIBP Monitoring

7

Problem Resolution

Inaccurate NIBP measurement:

- Make sure the correct cuff size is used. An inaccurate low reading occurs when the cuff is too large. An inaccurate high reading occurs when the cuff is too small.
- Check for residual air left in the cuff from a previous measurement.
- The cuff is applied too tightly or too loosely.
- The cuff and heart are not at the same level.
- The patient is moving.
- There is a leak in the cuff or tubing.
- The system needs calibration.

Care and Cleaning

Avoid allowing any saline solution from an intravenous drip onto the Integrated Patient Monitor module connectors.

Do not allow liquid of any kind to enter the Integrated Patient Monitor module connections when the cables are not plugged in.

Wipe up any spills immediately with a soft, clean cloth.

Do not use strong cleaning solvents.

Avoid kinking or compressing the cuff air tubes.

Do not allow any liquid or foreign objects to enter the cuff air tubes.

Refer to the manufacturer's instructions for the cleaning and sterilization requirements for the blood pressure cuffs. Instructions are provided in the Accessory Kit.

8

SpO₂ Monitoring

This section contains configuration and operational information specific to SpO_2 Monitoring.

Overview 8-2
SpO2 Display 8-2
Preparing for SpO2 Monitoring 8-3
Applying the Pulse Oximeter Sensor 8-3
Setting Up SpO2 Parameters 8-5
SpO2 Setup Tab 8-5
SpO2 Alarms Tab 8-7
Volume Tab 8-9
Summary of SpO2 Alarms 8-10
Problem Resolution 8-11

SpO2 Monitoring

Overview

The Narkomed 6400 with the Integrated Patient Monitor option monitors arterial oxygen saturation and pulse rate.

The SpO₂ monitoring is noninvasive. All information is gathered through a spectrophotometric transmission sensor applied to the patient's finger. Optional sensors can be applied at different sites. The system is compatible with Nellcor SpO₂ sensors and detects a pulse rate in the range of 40 to 235 beats per minute.

The spectrophotometric transmission sensor has two low-power LED light sources and a photodetector. Two wavelengths of transmitted light illuminate the tissue under the probe. The lights are absorbed differently by the oxyhemoglobin and deoxyhemoglobin. The light transmissions are read by the photoreceptor and converted to electrical pulses that represent absorbance. From these electrical pulses, the monitor derives the pulse rate and the percentage of available hemoglobin saturated with oxygen.

The arterial hemoglobin oxygen saturation is calculated as a percentage of total functional hemoglobin (hemoglobin available to transport oxygen). Significant levels of dysfunctional hemoglobin, such as carboxyhemoglobin or methemoglobin, can cause an inaccurate measurement.

WARNING: SpO₂ monitoring can interfere with MRI operation.

 SpO_2 readings and pulse rates are recorded in the Data Log in 1-minute intervals. SpO_2 readings are trended graphically in the trend area.

SpO₂ Display

The SpO₂ parameter box displays the SpO₂ result and pulse rate. The SpO₂ waveform is displayed next to the SpO₂ parameter box. The color of the waveform is the same as the color of the SpO₂ numeric data.

Figure 8-1. SpO₂ Parameter Box and Waveform



The numeric display range is 0 to 100% for SpO₂ and 40 to 235 bpm for pulse. All values are updated every two seconds.

SpO₂ Trend The SpO₂ trend is represented as a single line. The low scale value is 85% and the high scale value is 100%.

Trends are displayed by touching the **[Trend]** button in the main screen taskbar. For complete information on trends, see the *Narkomed 6400 Operator's Instruction Manual*.

SpO2 Monitoring

8

Figure 8-2. SpO₂ Trend

	100
99	SpO2
	85

Preparing for SpO₂ Monitoring

	1. Apply the SpO_2 sensor on the patient (see "Applying the Pulse Oximeter Sensor").
	2. Attach the patient cable to the ${\rm SpO}_2$ connector on the Integrated Patient Monitor module.
	3. Make sure all SpO_2 settings in the SpO_2 notebook are correct. Instructions for making setting selections follow.
Applying the Pulse Oximeter Sensor	For accurate readings, it is important to choose the correct sensor type and placement.
Sensor Types	A reusable finger sensor is included in the Accessory Kit in a nonsterile package. Other sensors that have limited reuse capability are available (see Spare and Replacement Parts) and can be applied to different sites. For more information on optional sensors, consult the detailed instructions provided with each sensor.
	The supplied finger sensor is recommended for relatively immobile patients who weigh more than 40 kg. It is not recommended for active patients or for prolonged cases (for example, nonanesthesia applications, such as monitoring of long-term respiratory support).
Sensor Site	The preferred site for the sensor is the index finger. If another digit must be used, avoid connecting the sensor to large or very small digits. (If the patient is large or obese, use a small finger.)
	Do not place the sensor on a thumb or on any toe. Do not place the sensor distal to an inflated pneumatic tourniquet or on an extremity with an arterial catheter in place. Avoid placing the sensor on any extremity with a blood pressure cuff or intravascular venous line. If the sensor must be placed distal to the blood pressure cuff used with the internal noninvasive blood pressure monitor, turn off the pulse oximetry alarms during blood pressure cuff inflation with the NIBP/SpO ₂ Interlock (see NIBP Interlock later in this section).

8

Sensor Apply the sensor with the cable running down the backside of the hand. This ensures that the light source is on the nail side of the finger, and the detector is on the underside. This arrangement allows the sensor light source to shine down through the top side of the finger.

Figure 8-3. Application of the Finger Sensor



The fingertip should lightly touch the stop at the end of the soft pad on the sensor. Make sure that the sensor's side walls are not pinching the finger or causing a pressure point anywhere on the finger.

If the patient has long fingernails, the fingernail tip must extend over the stop to ensure that the fingertip lightly touches the stop.

Do not tape the sensor shut.

For best results, the patient's hand should be at the level of the patient's heart.

- **NOTE:** Check the sensor placement frequently. The sensor must be reapplied to a different appropriate digit at least once every 4 hours, or more often if clinical conditions indicate that the site should be changed.
- **NOTE:** Although the sensor is designed to minimize interference from ambient light, bright light sources (for example, surgical lamps and direct sunlight) can interfere with the accuracy of the measurement. If bright ambient light proves to be a problem, cover the sensor site with an opaque material, such as a towel or blanket.

Part Number: 4117966-003 Rev: -

SpO2 Monitoring

8

Setting Up SpO₂ Parameters

Touch the SpO_2 parameter box anywhere except the alarm bell to display the SpO_2 notebook.

Figure 8-4. SpO₂ Setup Page



Touch **[NIBP Interlock]** or its associated selection field until the desired setting is displayed:

- On turns on the NIBP interlock feature
- Off turns off the NIBP interlock feature
- **NOTE:** When the interlock function is turned off, the following alarm information applies:

SpO2 Monitoring

- if an SpO_2 alarm occurs at the time an NIBP measurement begins, it will continue (even if the alarm condition no longer exists) until the NIBP measurement is complete
- likewise, if an $\rm SpO_2$ alarm occurs while an NIBP measurement is in progress, the alarm will not be posted until the NIBP measurement is complete
- SpO_2 Tone This option determines whether the SpO_2 tone is turned on. This is a tone that sounds each time an SpO_2 pulse is detected. It is a variable-pitch tone that changes as the patient's saturation level changes.

Touch **[SpO2 Tone]** or its associated selection field until the desired setting is displayed:

- On activates the tone–using this option automatically turns the QRS tone off.
- Off deactivates the tone

It is possible to turn the pulse tone off by selecting on for this option and setting the SpO_2 pulse tone volume to the minimum setting.

SpO2 Monitoring

SpO2 AlarmsTouch the Alarms tab in the SpO2 notebook to display the alarms options:TabNOTE: The SpO2 pulse rate alarm limits are linked to the ECG heart rate

alarm limits. Any changes to the pulse rate high, low, autoset narrow, or autoset wide settings on the SpO_2 alarm page will be automatically reflected on the ECG heart rate alarm page.

Figure 8-5. SpO₂ Alarms Page



Touch **[Alarm Control]** or its associated selection field until the desired setting is displayed:

- On turns the SpO₂ alarm notification system on
- Off turns the SpO₂ alarm notification system off
- Stby (standby) suspends the SpO₂ alarm notification system until valid SpO₂ data is received, then the system is automatically turned on

NOTE: SpO₂ alarms are turned off automatically while a SRVC IPM MON advisory alarm is active.

SpO2 Monitoring

Alarm Limits- Channel	Use this option to specify the channel for making high and low limit alarm adjustments.
	Touch [Channel] or its associated selection field until the desired channel-SpO2 or Pulse-is displayed.
	After the channel is selected, high and low alarm limits can be set for that channel. The settings for each channel are independent and do not affect settings for the other channel.
Alarm Limits- Set Hi	This option sets the high alarm limits for SpO_2 or pulse. The range of available settings is 1 to 100% for SpO_2 and 41 to 235 bpm for pulse. The lowest possible setting must be one higher than the low alarm setting. The default setting is 100% for SpO_2 and 150 bpm for pulse.
	1. Touch [Set Hi] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Hi] button.
	• Use the fine adjustment to change the setting by 1% or bpm.
	• Use the coarse adjustment to change the setting by 10% or bpm.
Alarm Limits- Set Lo	This option sets the low alarm limits for SpO_2 or pulse. The range of available settings is 0 to 99% for SpO_2 and 40 to 234 bpm for pulse. The highest possible setting must be one lower than the high alarm setting. The default setting is 90% for SpO_2 and 50 bpm for pulse.
	1. Touch [Set Lo] or its associated selection field to activate the slider bar.
	2. Move the slider control until the desired setting is displayed next to the [Set Lo] button.
	• Use the fine adjustment to change the setting by 1% or bpm.
	• Use the coarse adjustment to change the setting by 10% or bpm.
Disp Limits	Touch [Disp Limits] or its associated selection field to toggle the display of alarm limits in the SpO_2 parameter box ON or OFF.
AUTOSET Narrow	This option sets alarm limits to +/- 3% of the current SpO_2 reading within the 0 to 100% range and +/- 10 bpm of the current pulse reading within the 40 to 235 bpm range. When the autoset limit falls below the minimum setting, the lowest possible setting within the range is automatically selected. When the autoset limit exceeds the maximum setting, the highest possible setting within the range is automatically selected.
	Touch [AUTOSET Narrow] . The setting is automatic. The new range values appear next to the [Set Hi] and [Set Lo] buttons.

SpO2 Monitoring

8

AUTOSETThis option sets alarm limits to +/- 5% of the current SpO_2 reading within
the 0 to 100% range and +/- 20 bpm of the current pulse reading within the
40 to 235 bpm range. When the autoset limit falls below the minimum
setting, the lowest possible setting within the range is automatically
selected. When the autoset limit exceeds the maximum setting, the
highest possible setting within the range is automatically selected.

Touch **[AUTOSET Wide]**. The setting is automatic. The new range values appear next to the **[Set Hi]** and **[Set Lo]** buttons.

Volume Tab Touch the Volume tab in the SpO_2 notebook to display the Volume page:

Figure 8-6. SpO₂ (Pulse Tone) Volume Page



Setting the Pulse Tone Volume Use this option to adjust the volume of the tone that is sounded for each SpO_2 pulse beat.

Move the slider control to adjust the volume. The volume is turned off when the slider control is at the bottom of the slider. The volume is set to maximum when the slider control is at the top.

Summary of SpO₂ Alarms

This section contains all warning, caution, and advisory alarms associated with ${\rm SpO}_2$ monitoring.

NOTE: This summary assumes that the patient's condition is checked before attempting to change any system settings when patient condition alarms occur.

Table 8-1. SpO₂ Warnings

Message	Condition	Suggested Action
NO OXI PULSE	No pulse detected	Check the patient. Check the sensor placement.
SPO2 LOW	The reading is lower than the alarm limit setting	If the alarm setting is incorrect, reset the low alarm limit.
OXI PULSE LOW	The pulse rate is lower than the alarm limit setting	If the alarm setting is incorrect, reset the low alarm setting.

Table 8-2. SpO₂ Cautions

Message	Condition	Suggested Action
SPO2 HIGH	The reading exceeds the alarm limit setting	If the alarm setting is incorrect, reset the high alarm limit.
OXI PULSE HIGH	The pulse rate is higher than the alarm limit setting	If the alarm setting is incorrect, reset the high alarm setting.

Table 8-3. SpO₂ Advisories

Message	Condition	Suggested Action
SPO2 SENS ERR	Sensor failure	Ensure good sensor connections or replace sensor. If failure continues, contact an authorized representative of DrägerService.

SpO2 Monitoring

8

Problem Resolution

Inaccurate SpO₂ results:

- Too much ambient light on the sensor. Cover the sensor with opaque material.
- Long or acrylic fingernails and nail polish can affect readings.
- Patients with Sickle Cell Anemia, anemia, or jaundice may have inaccurate readings.

No pulse tone:

- SpO_2 tone is on, but the SpO_2 probe is not connected. Turn SpO_2 tone to Off.
- SpO₂ pulse tone volume is at minimum setting. Increase volume using slider control on the Volume page of the SpO₂ setup notebook.
- Long or acrylic fingernails and nail polish can affect readings.
- Patients with Sickle Cell Anemia, anemia, or jaundice may have inaccurate readings.
- Smokers have a higher CO blood level affecting the carboxyhemoglobin count.
- Some antibiotics can cause increased levels of methemoglobin, which produces a result of reduced hemoglobin, resulting in a low SpO_2 reading.

9

Specifications

This section describes the specifications for the Integrated Patient Monitor Option.

9-2
9-2
9-2
9-2
9-3
9-3
9-3
9-4
9-4

Specifications

General

9

$Dimensions (W \ x \ H \ x \ D) \dots $	$.4 \ge 7 \ge 14$ inches
Weight (approximate)	8 lbs

Environmental

Storage	Temperature10–50°C
	Humidity
Operating	Temperature $\ldots \ldots 15-35^{\circ}\mathrm{C}$
	Humidity
	NOTE: Operating specifications are for an Integrated Patient Monitor mounted within a Narkomed 6400. The conditions refer to conditions external to the Narkomed 6400.

Electrical

Equipment class	IEC 601-2-27 Class 1, Type C	CF
-----------------	------------------------------	----

ECG Monitoring

Acquisition of up to 5 electrode	ECG I, II, III, aVR, AVL, aVF, V
Lead fail	.identifies failed electrode and posts alarm
Waveform display aspect ratio .	$\ldots \ 0.40 \ sec/mV \ (amplitude = 10 \ mm/mV,$
	length = 25 mm/sec

Heart rate:

Heart rate averaging
Display update interval
Response time to heart rate change <6 seconds (per AAMI EC13)
Time to alarm for tachycardia < 10 seconds
Limit alarm delay <13 seconds after limit alarm condition exceeded
Heart rate detection
Measurement accuracy $\pm 5\%$
QRS detection
and 40 msec to 120 msec Q to S duration
Bandwidth
Common mode rejection 90 dB minimum at 60 Hz
Tall T-wave rejection

Specifications

9

Pacemaker detection/rejection:

Input voltage range±2 mV to ±700 mV
Input pulse width 0.1 ms to 2 ms
Rise time $\ldots \ldots \ldots$
$Over/under \ shoot. \ldots 2 \ mV \ (max)$
Baseline drift
pacemaker pulse applied
Defibrillator sync output pulse
and 10 msec ±20% pulse width
Protection against electric shock type CF (defibrillator-proof)

Invasive Blood Pressure Monitoring

$Input \ measurement \ range \ (after \ autozero) \ldots \ldots \ldots$	98 to 350 mmHg
Zero balance range	±150 mmHg
Zero balance accuracy	±1 mmHg
Measurement accuracy greate	er of ±1 mmHg or 2%
Protection against electric shock type Cl	F (defibrillator-proof)

Cardiac Output Monitoring

Output range	$\dots \dots 0.2$ to 15.0 liters/min.
Blood temperature range	$\ldots \ldots \ldots 30^\circ C$ to $42^\circ C$
Injectate temperature range	$\dots \dots -0.3^{\circ}C$ to $30^{\circ}C$
Waveform frequency response	0 to 10 Hz
Protection against electric shock	. type CF (defibrillator-proof)

Temperature Monitoring

Input temperature $\dots \dots 0^{\circ}C$ to $45.1^{\circ}C$
Probe compatibility YSI 400 or YSI 700
Maximum time to accurate measurement \leq 45 sec.
Accuracy $\dots \pm 0.1^{\circ}C$ for YSI 400
±0.3°C for YSI 700
Resolution
Protection against electric shock type CF (defibrillator-proof)

Specifications

Noninvasive Blood Pressure Monitoring

	Blood pressure measurement range	$\dots \dots 15$ to 275 mmHg
	Blood pressure resolution	1 mmHg
	Protection against electric shock ty	vpe BF (defibrillator-proof)
Cuff Inflation	Maximum inflation pressure	150 mmHg Neonatal
	5	300 mmHg Adult/Pediatric
	Typical sample duration	
	Stat mode duration	

Pulse Oximetry Monitoring

SpO_2 saturation range $\ldots \ldots \ldots \ldots \ldots \ldots \ldots \ldots \ldots \ldots 0$ to 100%
Pulse rate range
Accuracy- $\mathrm{SpO}_2 \ldots \ldots \ldots \ldots \pm 1.5\%$ for 90 to 100% saturation
±2.1% for 80 to 89.9% saturation
$\pm 2.4\%$ overall for 60 to 100% saturation
Accuracy - pulse rate $\dots \dots \pm 1.7\%$ at a constant pulse rate
Alarm limit ranges
40 to 235 bpm for pulse rate
Displayed frequency 1.5 Hz to 10.5 Hz
Protection against electric shock type BF (defibrillator-proof)



Spare and Replacement Parts

This section lists the spare and replacement parts for the Integrated Patient Monitor along with their part numbers.

Description

Part Number

Manuals

ntegrated Patient Monitor Option Operator's Instruction Manual
his manual)
larkomed 6400 Binder and Instruction Manual Assembly
ncludes Integrated Patient Monitor Option Operator's Manual)

ECG Monitoring Accessories

5-lead ECG cable with ESU filter for OR use (AHA colors)4113273
$Leadwire \; set, AHA, individual \; 5\text{-lead } w/grabbers \; (mixed \; length) \; . \; .4113274$
5-lead ECG cable with ESU filter for OR use (IEC colors)4113397
$Leadwire \; set, IEC, individual \; 5\text{-lead } w/grabbers \; (mixed \; length). \; . \; .4113455$
Leadwire set, AHA, snaps
Leadwire set, IEC, snaps
Leadwire set, AHA, individual 3-lead w/grabbers, 51 in
Leadwire set, IEC, individual 3-lead w/grabbers, 51 in
3-lead ECG cable, neonatal (AHA colors)
3-lead ECG cable, neonatal (IEC colors)
Leadwire set, AHA, neonatal 3-lead
Leadwire set, IEC, neonatal 3-lead

Invasive Blood Pressure Monitoring Accessories

BP adapter cable for Edwards transducer	4113275
BP adapter cable for Ohmeda/Spectramed transducer $\ldots \ldots$	4113276
BP adapter cable for Utah transducer	4113277

Cardiac Output Monitoring Accessories

Cardiac output cable, 12 feet	.4113261
Bath probe sensor, 6 feet	.4113262
In-line CritiKit Ohmeda/Spectramed temp. probe, 3 feet	.4113263
In-line CO-set Edwards temp. probe, 3 feet	.4113264
In-line Abbott Thermoset temp. probe, 3 feet	.4113265

A-1
Spare and Replacement Parts

Temperature Monitoring Accessories

Temperature cable for disposable probes (400 series). $\ldots \ldots \ldots$	4112937
Disposable rectal/esophageal temp. probe, 12 fr (400 series)	4112938
Box of 100 disposable probes (400 series)	2938-001
Temperature probes YSI series 700 (2)	4107800

Noninvasive Blood Pressure Monitoring Accessories

NIBP tubing, 12 feet, adult4113454
NIBP tubing, 8 feet, neonatal
Newborn reusable cuff, gray, 6-11 cm
Infant reusable cuff, tan, 10-19 cm
Child reusable cuff, green, 18-26 cm 4113395-003
Adult reusable cuff, blue, 25-35 cm
Large adult reusable cuff, red, 33-47
Thigh reusable cuff, brown, 46-66 cm
Newborn reusable cuff, gray, 6-11 cm, non-latex
Infant reusable cuff, tan, 10-19 cm, non-latex
Child reusable cuff, green, 18-26 cm, non-latex
Adult reusable cuff, blue, 25-35 cm, non-latex
Large adult reusable cuff, red, 33-47, non-latex 4113395-011
Thigh reusable cuff, brown, 46-66 cm, non-latex

SpO₂ Monitoring Accessories

SpO ₂ interface cable, 10 feet	.4113453
Finger Sensor	.4113823
Durasensor	4108983
Oxisensor D-25, adult	4108984
Oxisensor D-20, child	4108985
Oxisensor D-15, adult nasal	4108986
Oxisensor N-25, neonate	4108987
Oxisensor I-20, infant	4108988

Miscellaneous

Cable, defib sync, unterminated	4113460
Integrated Patient Monitor Module41	13465-005

A-1



Tables

This section provides details of system information in tabular format.

Factory Default Settings	A-2-2
Catheters/Settings Supported by the Narkomed 6400	A-2-7

-2 Tables

Factory Default Settings

The following tables contain the values stored in the Factory template for various parameters and machine settings (exceptions are noted). They also specify the parameters and settings that can be stored within a Site or usercreated template (with the noted exceptions). The specific values for those parameters and settings are determined by the clinician. The values that are invoked automatically at power-up are stored in the Site template.

Parameter	Alarm Limit (High)	Alarm Limit (Low)	Site or Units	Waveform Scale	Limits Display	Units Display	Other
Oxygen	100	30			Off	Off	
CO ₂ Expiratory Inspiratory Cal Delay Period Sample Flow	50 4	8	mmHg	53 mmHg	Off	Off	5 minutes Normal
Breathing Pressure Peak PEEP Apnea Threshold Plateau Display Mean Display	50 6				Off	Off	8 cmH ₂ O Off On
Minute Volume		1.0			Off	Off	
Agent Halothane Isoflurane Enflurane Desflurane Sevoflurane N ₂ O	3.0 3.0 3.0 9.0 6.0	0.0 0.0 0.0 0.0 0.0		10% 50%	Off	Off	Note: the agent scale changes to 20% for Desflurane
Heart Rate (ECG) Channel 1 Channel 2 Pace Pulse QRS Tone QRS Volume	150	50	Lead II Lead V	1x 1x	Off	Off	Off Off 50%
IBP Channel 1 Systolic Mean Diastolic Cursor	300 300 300	-30 -30 -30	ART	180 mmHg separate mode 	Off	Off	Off

Tables A-2

Parameter	Alarm Limit (High)	Alarm Limit (Low)	Site or Units	Waveform Scale	Limits Display	Units Display	Other
IBP Channel 2 Mean Cursor	300	-30	CVP	30 mmHg separate mode 	Off	Off	Off
IBP Channel 3 Systolic Mean Diastolic Cursor	300 300 300	-30 -30 -30	PA	40 mmHg separate mode 	Off	Off	Off
IBP Channel 4 Systolic Mean Diastolic Cursor	300 300 300	-30 -30 -30	FEM	180 mmHg separate mode 	Off	Off	Off
SPO ₂ Saturation Pulse SPO2 Tone Tone Volume NIBP Interlock	100 150	90 50			Off	Off	On 40% On
Cardiac Output Blood Temp. Catheter Size Injectate Volume Catheter Type Injectate Probe	40.0	30.0			Off	Off	7.5 10 cc Edwards Inline
	Note: The se	etting for Com	putation Cons	stant cannot be	stored withi	n a template.	
Temperature T1 T2 T1 - T2 T1 - T2 Display	42 42 5	30 30 -5	°C ESO SKIN		Off	Off	Off

A-2 <u>Tables</u>

Parameter	Alarm Limit (High)	Alarm Limit (Low)	Site or Units	Waveform Scale	Limits Display	Units Display	Other
NIBP					Off	Off	
Systolic	300	0					
Mean	300	0					
Diastolic	300	0					
Interval							5 minutes
CompleteToneVol							20%
Patient Age			Adult				

System Settings	Value	Notes
Trace Speed CV Respiratory Waveform	25 mm/sec 12.5 mm/sec	
Alarm Volume	Minimum	
Countdown Timer Interval Enabled AutoCycle Audio Notify	5 minutes Off Off Off	The Countdown Timer settings cannot be stored within a template. The settings listed here will be restored with each machine power-up or template invocation.
Elapsed Timer Enabled	Off	The Elapsed Timer settings cannot be stored within a template. The settings listed here will be restored with each machine power-up or template invocation.
Pressure Gauge Threshold Enable Audio	20 cmH ₂ O Off	The Pressure Gauge settings cannot be stored within a template. The settings listed here will be restored with each machine power-up or template invocation.
Data Log - Setup Interval NIBP Cardiac Output Warning Caution Position Font Time Location	5 minutes On Off Off Horizontal Small Left	
Data Log - Custom Displayed Columns	All	

Tables

A-2

System Settings	Value	Notes
Numeric Box Position Lock Volume Agent	Off Off	
Waveform Print <i>IPM Installed (3-wire ECG)</i> Waveform #1 Waveform #2 <i>IPM Installed (5-wire ECG)</i> Waveform #1 Waveform #2 <i>IPM Not Installed</i> Waveform #1 Waveform #2	ECG II None ECG II ECG V Pressure None	The Waveform Print settings cannot be stored within a template. The settings listed here will be restored with each machine power-up or by exiting Monitor Standby via [New Case] .
Vent Confirm Tone Volume Level	On 50%	
MAC Display	Off	
NIBP Pulse Display	Off	
Ventilation Sound Selection Volume	None 50%	
N ₂ O Waveform Display	Off	
Screen Configuration Mode IPM Installed IPM Not Installed	Automatic Respiratory Only	
IBP Waveform Mode	Single Trace	
Fresh Gas Flow Flow Threshold Time Delay	4 L/min Off	
CO2 Alarm Mode	Limited	
Configurable Trend Window #1 (from top) Window #2 Window #3 Window #4	CPL CO2 VOL PAW	
Tidal Volume Alarm	Enabled	The Tidal Volume alarm setting cannot be stored within a template. The setting listed here will be restored with each machine power-up.

A-2 Tables

System Settings	Value	Notes
Quick Print Time	30 seconds	The setting for Quick Print Time cannot be stored within a template. The last setting will persist until changed by clinician.
Cardiac Calculations USE Selection Units	PAWP Metric	The setting for units cannot be stored within a template. The last setting will persist until changed by clinician.
External Printer Select Printer	None	The setting for printer selection cannot be stored within a template. The last setting will persist until changed by clinician.

Tables A-2

Catheters/Settings Supported by the Narkomed 6400

Edwards Catheters

Catheter Size (Solar)	Injectate Volume (In-Line)			Inje	ctate Volu (Bath)	ume
	3	5	10	3	5	10
5	✓	1	1	1	✓	1
6	✓	1	1	1	✓	1
7	✓	1	1	1	✓	1
7.5	✓	1	1	1	✓	1
8	~	1	1	1	~	1

Abbott

Catheters

Catheter Size (Solar)	Injectate Volume (In-Line)			Injectate Volume (Bath)			
	3	5	10	3	5	10	
5.5 (75 cm)	1	1		1	1		
7 (85 cm)		1	1	1	1	1	
7 (110 cm)		1	1	1	1	1	
8 (110 cm)		1	1	1	1	1	

Arrow Catheters

Catheter Size (Solar)	Injectate Volume (In-Line)			Injectate Volume (Bath)			
	3	5	10	3	5	10	
4				1			
5	1	1		1	1		
6	1	1	1	1	1	1	
7	1	1	1	1	1	1	
7.5	1	1	1	1	1	1	

A-2 Tables

Ohmeda Catheters

Catheter Size (Solar)	Injectate Volume (In-Line)			Injectate Volume (Bath)			
	3	5	10	3	5	10	
5	1	1	1	1	1	1	
7	1	1	1	1	1	1	
7.5	1	1	1	1	1	1	

Other Catheters

Catheter Size (Solar)	Injectate Volume (In-Line)			Injectate Volume (Bath)		
	3	5	10	3	5	10
4	1	1	1	1	\	1
5	✓	1	1	1	✓	1
6	✓	1	1	1	✓	1
7	✓	1	1	1	✓	1
7.5	✓	1	1	1	✓	1
8	~	1	1	1	1	1



A Dräger and Siemens Company

Draeger Medical, Inc. 3135 Quarry Road Telford, PA 18969 Tel: (215) 721-5404 (800) 462-7566 Fax: (215) 721-9561 Web: www.draegermedical.com Printed in the U.S.A.