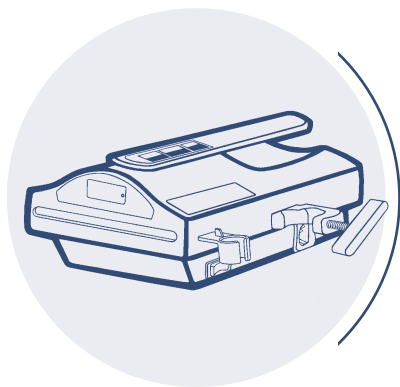
Ranger *
Blood/Fluid Warming System

Ranger

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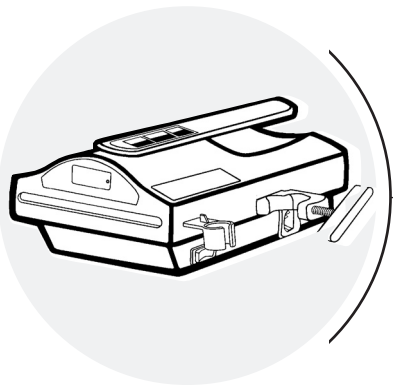
OPERATOR'S Manual

Table of Contents

Technical Service and Order Placement	1
Introduction	2
Indications for Use	2
Warnings and Precautions	2
Product Description	3
Ranger Warming Unit	3
Ranger Disposable Set	3
Distal Fluid Temperatures	4
Product Safety Features	4
Instructions for Use	5
Preparation and Setup of the Ranger Warming Unit	5
Removing the Disposable Set From the Ranger Warming Unit	6
Transferring the Disposable Set From One Ranger Warming Unit to Another	6
Troubleshooting	7
Maintenance and Storage	9
Specifications	40
Definition of Symbols	11

Technical Service and Order Placement

Customer service and technical support

USA

TEL: +1-952-947-1200

FAX: +1-952-947-1400

GERMANY

TEL: +49-4154-9934-0

FAX: +49-4154-9934-20

Order placement

USA

TEL: +1-952-947-1200

FAX: +1-952-947-1400

In-warranty repair and exchange

Replacement parts to correct a problem are delivered at no charge. To return a device to Arizant Healthcare Inc. for service, first obtain a Return Authorization (RA) number from a customer service representative. Please use the (RA) number on all correspondence when returning a device for service. A shipping carton will be delivered to you at no charge, if needed. We will service and ship your device within five (5) working days of our receipt. Call your local supplier or sales representative to inquire about loaner devices while your device is being serviced.

When you call for technical support

Remember, we will need to know the serial number of your unit when you call us. The serial number label is located on the bottom of the warming unit.

Introduction

The Ranger blood/fluid warming system includes a warming unit and a disposable IV blood/fluid administration set. The Ranger warming system is designed to warm blood, blood products, and liquids and deliver these at flow rates from KVO to 500 ml/min. At these flow rates, the device maintains fluid output temperatures ranging from 33°C to 41°C (Note: This is for room temperature fluids only). It takes less than 2 minutes to warm up to the 41°C set point temperature.

Disposable blood/fluid sets are available for standard and high flow applications, and for irrigation applications. Disposable sets are sterile, latex-free, single-use-only items and are designed to be used with the reusable warming unit.

The Ranger warming unit is designed to be mounted to an IV pole. A handle located on the top of the unit makes transport easy. When mounted to the IV pole, the unit fits easily above the Bair Hugger warming unit.

Indications for use

The Ranger warming system is intended to warm blood, blood products, and liquids.

Warnings and precautions

DANGER

Explosion hazard. Do not use in the presence of flammable anesthetics.

WARNING

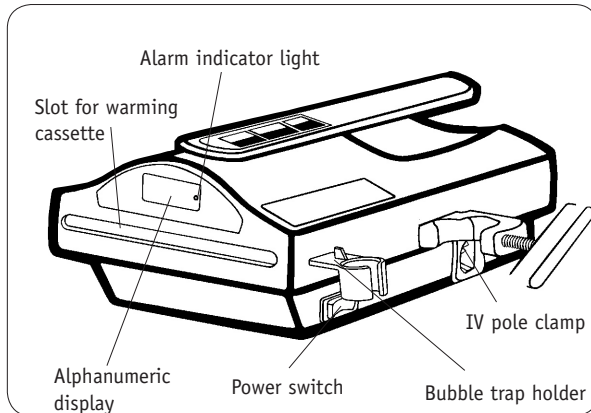
- Electrical shock hazard. Do not open the warming unit case.
- Do not substitute other devices for the Ranger warming unit or Ranger disposable sets. Thermal or electrical injury or device damage may occur.
- Never infuse fluids if air bubbles are present in the fluid line, as air embolism may result.
- Do not continue use of the unit if the over-temperature alarm continues to sound and the temperature does not return to the set point temperature. Immediately stop fluid flow and discard the disposable set. Have the warming unit tested by a biomedical technician or call Arizant Healthcare Customer Service.

CAUTION

Do not immerse the blood/fluid warming unit. Wipe with a clean, slightly damp cloth.

Product Description


The Ranger blood/fluid warming system consists of a warming unit and a sterile disposable IV blood/fluid delivery set.

The RANGER warming unit

The warming unit is a compact, lightweight, liquid-resistant device with a clamp located on the side for attachment to an IV pole. A carrying handle on the top of the unit makes it easy to transport. Located on the front panel you will find:

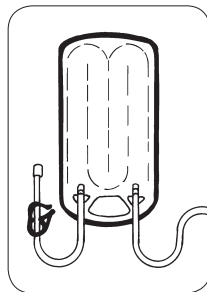- Alphanumeric display that indicates the heater temperature during normal operation. In an over temperature condition, the display alternately flashes a temperature of 43°C or higher and the word "HI." In an under-temperature condition, the display alternately flashes a temperature of 33°C or lower and word "LO."
- Alarm indicator light that comes on when either an over- or under-temperature condition occurs.

Warming Cassette

RANGER disposable sets

Blood/fluid administration sets include a warming cassette (heat exchanger), tubing and connectors compatible with hospital standard IV blood/fluid administration sets, bubble trap, and injection port. The High Flow set also includes 2 blood/fluid bag spikes and a 150 micron filter located in a central drip chamber.

	Flow Rate	Priming Volume	Patient Line Length
Standard Flow Set	150 ml/mn	39-44 ml	76 cm 152 cm (76 cm patient line plus 76 cm extension)
High Flow Set	100-150 ml/min	150 ml	152 cm (76 cm patient line plus 76 cm extension)

Disposable irrigation sets deliver 30 L/hr flow rates and 142 ml priming volume. They include spike sets and a 183 cm patient line.

Refer to the instructions provided with each disposable set for information on use.

Distal fluid temperatures

Nominal measurements only. Temperatures taken at 18°C room temperature.

FLOW RATE (L/HR)	USING 10°C INLET FLUID	USING 20°C INLET FLUID
0.5	32°C	32°C
1.0	36°C	36°C
2.0	39°C	38°C
3.0	39°C	40°C
4.5	39°C	40°C
6.0	39°C	39°C
9.0	38°C	38°C
15.0	-	36°C
20.0	-	35°C
30.0	-	33°C

Product safety features

The following chart describes the safety alarm features of the Ranger warming unit.

ALARM TYPE	WHAT TO LOOK FOR	DESCRIPTION/CAUSE	ACTION
Over-temperature alarm - 43°C	Alarm indicator light illuminates and alarm sounds, alphanumeric display alternately flashes a temperature of 43°C or higher and the word "HI."	Heater temperature rose to 43°C because of transient conditions.*	Observe alphanumeric display. If temperature does not drop to 41°C (may take a few minutes), discontinue use of unit. Call Arizant Healthcare Customer Service.
Under-temperature alarm - 33°C	Alarm indicator light illuminates and alarm sounds, alphanumeric display alternately flashes a temperature of 33°C or lower and the word "LO."	Heater temperature has fallen to 33°C.	Alarms stop when temperature rises above 33°C. Continue use of unit. If the temperature does not rise above 33°C, unplug the unit and call Arizant Healthcare Customer Service.
Independent backup safety alarm - 46°C	- Alarm indicator light illuminates and alarm sounds, alphanumeric display alternately flashes a temperature of 43°C or higher and the word "HI." - Alphanumeric display is dark, alarm sounds (backup safety alarm still works even if display is dark).	Heater temperature rose to 46°C. Safety backup system is activated at 46°C and unit shuts off power to the heating plates.	TURN OFF THE UNIT AND UNPLUG IT. Do not use the unit. Discard the disposable set. Call Arizant Healthcare Customer Service.

*Transient conditions may cause an over-temperature alarm condition. These conditions include:

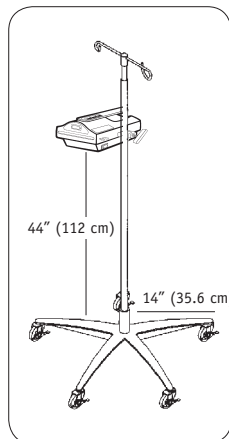
- There was an extreme change in flow rates (e.g., from 500 mL/min to stop flow).
- The unit was turned on and reached the set point temperature of 41°C before the warming cassette was inserted in the device.
- Fluids were prewarmed to above 42°C before being infused.

Instructions for Use

Preparation and setup of the Ranger warming unit


DANGER

Explosion hazard. Do not use in the presence of flammable anesthetics.

1. Attach the Ranger warming unit to the IV pole. Tighten the pole clamp securely.

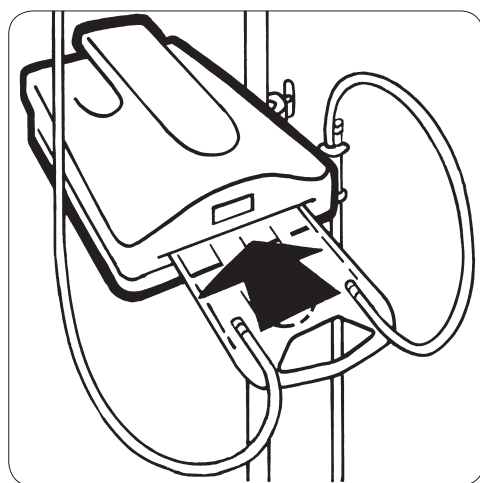CAUTION: To prevent tipping, clamp the Ranger warming unit to an IV pole with a minimum 14" (35.6 cm) radius wheelbase and at a height no higher than 44" (112 cm). Failure to do so may result in damage to the product or catheter site trauma.

2. Slide the fluid warming cassette into the slot in the warming unit. The cassette can only fit into the device one way.
3. Prime the disposable set. For more information about priming the set, refer to instructions provided with the disposable sets.
4. Place the bubble trap in the holder.
5. Plug the power cord into an appropriate outlet. Turn the unit ON. In a few seconds the alphanumeric display will illuminate. It takes less than 2 minutes to warm up to the 41°C set point temperature.
6. Begin infusion. When infusion is completed, remove the disposable set and discard according to institutional protocol.

Removing the disposable set from the Ranger warming unit

1. Close the inlet clamp proximal to the fluid warming cassette and open all clamps distal to the cassette.
2. Disconnect the disposable set from the fluid source, if applicable.
3. Allow fluid to flow to the patient (this may take 2-3 seconds). Close a distal clamp.
4. Remove the fluid warming cassette from the warming unit and discard according to institutional protocol.
5. Reconnect the patient IV line to the fluid source to continue the infusion without warming.

Transferring the disposable set from one Ranger warming unit to another

1. Follow steps 1-3 above, then remove the disposable set from the first warming unit.
2. During transport, keep the clamps closed and do not infuse fluids while the warming cassette is outside of the warming unit.
3. Slide the warming cassette into the second warming unit.
4. Ensure that air is removed from the tubing.
5. Open the clamps and continue the infusion.

Troubleshooting

Condition	Cause	Solution
Nothing illuminates on the warming unit panel.	•Unit is not turned on, plugged in, or power cord is not plugged into an appropriate outlet. •Unit failure.	•Turn unit on. Make sure the power cord is plugged into the power entry module of the warming unit. Make sure the warming unit is plugged into a properly grounded outlet. •Call Arizant Healthcare Customer Service.
Condition	Cause	Solution
Alarm indicator illuminates and alarm sounds, alphanumeric display alternately flashes a temperature of 43°C or higher and the word "HI."	Temporary over-temperature condition because: •An extreme change in flow rates occurred (e.g., from 500 ml/min to stop flow). •Unit was turned on and reached set point temperature before warming cassette was inserted. •Fluids were prewarmed to above 42°C before being run through the warming unit.	•Open flow to reduce temperature. Alarms will stop when the display reads 41°C. The unit is ready to use. •Alarms will stop when the display reads 41°C. The unit is ready to use. •Turn off unit. Discontinue infusion of fluids. Do not warm fluids before infusing them through the Ranger unit.
Condition	Cause	Solution
Alarm sounds, alphanumeric display and alarm indicator light go dark.	Primary controller failure. Unit will no longer operate.	Power to heating plates will shut off if temperature rises to 46°C. Turn unit off and unplug it. Discontinue use of unit. Discard disposable set. Alarm will continue to sound if you do not unplug unit. Call Arizant Healthcare Customer Service.

Condition

- Unit alarms soon after plugging it in (unit does not have to be turned on for this condition to occur).
- Heater temperature rises to 46°C and unit shuts down soon after plugging it in (unit does not have to be turned on for this condition to occur).

Cause

Test screw on bottom of unit is loose or missing.

Solution

Make sure test screw is completely tightened. If it is missing, turn unit off and unplug it. Call Arizant Healthcare Customer Service.

Condition

Alarm sounds but unit has been turned off.

Cause

Independent backup safety system has been activated.

Solution

Unplug unit. Call Arizant Healthcare Customer Service.

Condition

Cannot remove warming cassette from unit.

Cause

- Warming cassette is too full, fluids are still being infused, or clamp is open proximal to the warming cassette.

Solution

- Make sure fluid is drained from warming cassette before sliding out the cassette, that fluids are no longer infusing, and that clamp is closed proximal to the warming cassette.

- Warming unit is below patient level, creating excessive back pressure.

- Raise unit above patient level.

Condition

Alarm indicator light illuminates and alarm sounds, alphanumeric display alternately flashes a temperature of 33°C or below and the word "LO."

Cause

Under temperature condition caused by very high flow using very cold fluid, or defective heater/relay.

Solution

Alarm should stop when temperature rises above 33°C. If alarm continues, unplug unit and discontinue use. Call Arizant Healthcare Customer Service.

Condition

Alphanumeric display reads "Er 4" or "Open."

Cause

Open wire on temperature sensor.

Solution

Do not use unit. Call Arizant Healthcare Customer Service.

Condition

Alphanumeric display reads "Er 5" or "Open."

Cause

Electrical interference.

Solution

Remove the unit. Refer to biomedical technician or call Arizant Healthcare Customer Service.

Maintenance and Storage

CAUTION

- Do not immerse the warming unit in cleaning or sterilizing solutions. The unit is not liquid proof.
- Do not clean the warming unit with solvents. Damage to the case, label, and internal components may result.

Cleaning

1. Disconnect the Ranger warming unit from the power source.
2. Wipe the outside of the unit with warm, soapy water, nonabrasive cleaning solutions, dilute bleach, or cold sterilants. Do not use abrasive materials.
3. Wipe with a dry, soft cloth.

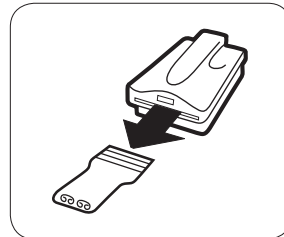
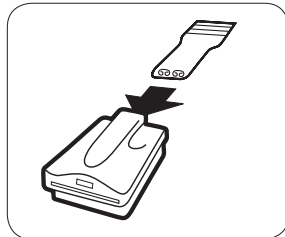
TO CLEAN THE INSIDE OF THE UNIT:

The Ranger cleaning tool is intended to clean both heating plates of the warming unit.

CAUTION

- Do not allow spills to dry inside the unit, as this may make it more difficult to clean the unit.
- The cleaning tool provides only superficial cleaning—it does not disinfect or sterilize the interior of the unit.

1. Unplug the warming unit.
2. Unfold the cleaning tool. Wet the foam pads with a nonabrasive solution, such as Alconox[®] brand detergent.

3. Insert the tool from the back of the unit and pull the tool all the way out from the front.
4. Rinse the tool with water and repeat 3 times.
5. Wipe off the unit to remove excess fluid.

Note: If the cleaning tool cannot be inserted through the slot of the warming unit because of excess dried-on fluids, send the unit to a biomedical technician.

Do not use abrasive materials or solutions to clean the heater plates.

Storage

Store all components in a cool, dry place when not in use.

Servicing

There are no user-serviceable parts in the Ranger warming unit. All service must be performed by Arizant Healthcare Inc. or an authorized service technician.

Call Arizant Healthcare Customer Service at +1-952-947-1200 for service information.

[®]Alconox is a registered trademark of Alconox, Inc.

Specifications

Physical characteristics

WARMING UNIT

4.5 in. (11 cm) high x 7.5 in. (19 cm) wide x 10 in. (25 cm) long;
wt.: 7 lb. 7 oz. (3.4 kg)

CLASSIFICATION

Classified under EN 60601-1 Guidelines (and other national versions of the Guidelines) as Class I, Type B, Ordinary Equipment. Not suitable for use in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide. Classified by Underwriters Laboratories Inc. with respect to electric shock, fire, and mechanical hazards only, in accordance with UL 2601-1 and in accordance with Canadian/CSA C22.2 No. 601.1.

Electrical characteristics

INPUT VOLTAGE

100-120 or 220-240 VAC

OPERATING FREQUENCY

100-120 VAC, 50/60 Hz
220-240 VAC, 50 Hz

MAXIMUM HEATING POWER

900 W

FUSE

10 Amp

Temperature characteristics

SET POINT TEMPERATURE

41°C

OVER-TEMPERATURE ALARM

43°C

UNDER-TEMPERATURE ALARM

33°C

OVER-TEMPERATURE CUTOFF

46°C

Leakage current

Meets leakage current requirements in accordance with UL 2601-1 and EN 60601-1.

Environmental conditions

OPERATING TEMPERATURE RANGE

15° to 40°C (59° to 104°F)

STORAGE TEMPERATURE RANGE

-20° to 45°C (-4° to 113°F)

OPERATING HUMIDITY


10 to 85% RH, noncondensing

ATMOSPHERIC PRESSURE RANGE

50 kPa to 106 kPa

Definition of Symbols

	ON
	OFF
	Fuse
	Attention - Read accompanying documents
	Non Explosion-Proof
	Type B Applied Part
	Voltage, Alternating Current (AC)

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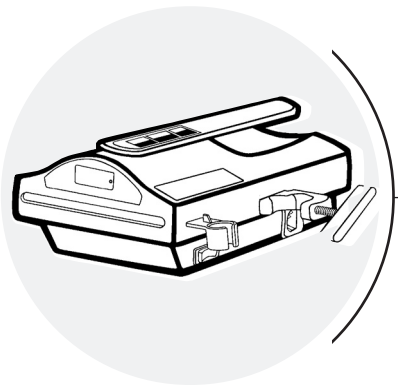Authorized Representative in the European Community
(as defined in Article 14 of the Medical Device Directive: 93/42/EEC):
Actamed Limited, Calder Island Way,
Wakefield WF2 7AW, United Kingdom
Tel (44) 1924 200550 • Fax (44) 1924 200518

*Ranger, Arizant, Arizant Healthcare, bright ideas that work, and the Bair Hugger and Arizant logos are trademarks of Arizant Healthcare Inc., registered or pending in the U.S. Patent & Trademark Office and in other countries.

©2003-2004 Arizant Healthcare Inc. All rights reserved.
241436A 08/04
Patents Pending.