

PARTS LIST MANUAL

Architectural Products Accessories From Hill-Rom



Architectural Products Accessories Parts List Manual

Revisions

Revision Letter	Pages Affected	Date
Original Issue		November, 1987
A	All	January, 1997

man044ra

COPYRIGHT© 1997 HILL-ROM® COMPANY, INC.

All rights reserved. No part of this text shall be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information or retrieval system without written permission from HILL-ROM COMPANY, INC. (Hill-Rom).

Second Edition

First Printing 1987

Printed in the USA

Hill-Rom® is a registered trademark of Hill-Rom Company, Inc.

Horizon® is a registered trademark of Hill-Rom Company, Inc.

Neonatal Horizon® is a registered trademark of Hill-Rom Company, Inc.

Specialite_{TM} is a trademark of Hill-Rom Company, Inc.

Stabilet_{TM} is a trademark of Hill-Rom Company, Inc.

The information contained in this manual is subject to change without notice. Hill-Rom makes no commitment to update or keep current, the information contained in this manual.

The only product warranty intended by Hill-Rom is the express, written warranty accompanying the bill of sale to the original purchaser. Hill-Rom makes no other warranty, express or implied, and in particular, makes no warranty of merchantability or fitness for any particular purpose.

Additional copies of this manual can be obtained from Hill-Rom.

Chapter 1

Parts List

Chapter Contents

Warranty	1 - 5
Ordering Service Parts	1 - 7
Exchange Policy	1 - 8
Raceway, 8"—P644E8L, R (Specialite Patient Light)	1 - 9
Raceway, 4"—P644E4L, R (Specialite Patient Light)	1 - 10
Connection—P644E0L, R (Specialite Patient Light)	1 - 11
Conversion Kit, P991-00 Monitor Mount—P651C991 (Power Column—P950E)	1 - 12
Hanger—Flatwall—P671E01, 02, 03 and Flatwall Hanger, Seismic—P671S01, 02, 03 (Flatwall Headwall System—P670E)	1 - 14
Storage Basket—P672E03, 04 (Flatwall Headwall System—P670E)	1 - 16
Adjustable Shelf—P673E21, 30 (Flatwall Headwall System—P670E)	1 - 18
Flatwall Bumper—P674E01, 02, 03 and Flatwall Bumper/Bottle Rack—P674E04, 05, 06 (Flatwall Headwall System—P670E)	1 - 20
Pivoting Storage Basket—Shallow—P951S03 and Pivoting Waste Basket, Deep—P951W03 (Power Column—P950E)	1 - 22
Basket, Large—P952L01 and Small Storage Basket—P952S01 (Power Column—P950E)	1 - 24
Bottle Guard Bumper—P953-00 (Power Column—P950E)	1 - 26
Floor Mounted Bed Locator—P958-00 (Power Column—P950E)	1 - 27
Ceiling Ring—P959E00 (Power Column—P950E)	1 - 28
Adjustable Shelf—P961E02 (Power Column—P950E)	1 - 30
Incandescent/Fluorescent Arm Lamp—P963C02 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 32
Mount, Arm Light—P963M00 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 33

STAT Clock/Timer Pod Mounted—P964-00 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 34
Mount Assembly—Exam Plus Light—P965M01 (Power Column—P950E) . . .	1 - 36
STAT Clock/Timer—Conventional Wall Mount—P967-01	1 - 37
Universal Holder, Multi Point—P970A01 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 38
Remote Transformer—P969G01, 2, 3, 4 (Flatwall Headwall System—P670E) .	1 - 40
Remote Transformer—P969H01, 2, 3, 4 (Flatwall Headwall System—P670E) .	1 - 42
Bird Blender Mount—P973-B1 (Power Column—P950E) and P1020NB1 (Neonatal Horizon Headwall System—1001AC).	1 - 44
Large Storage Basket—P105101 (Console—P1002AC)	1 - 46
Utility Light—Track Mounted—P966A00 (Power Column—P950E)	1 - 47
Sphygmomanometer Bracket, Aneroid—P975-01 (Flatwall Headwall System—P670EAC, Power Column—P950E)	1 - 48
Utility/Bottle Slide Mount—P973-01 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 50
Ophthalmoscope Mount—P973-S1 (Power Column—P950E)	1 - 52
Utility Mount, Short Post—P972-01 and Long Post—P972L01 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 54
IV Hanger—Slide Mounted—P971H00 (Horizon Headwall System—P1000). .	1 - 56
Utility Mount—Post Type—Dual—P972D01 (Neonatal Horizon Headwall System)	1 - 58
Utility Mount—Post Type—P972N00, 01 (Neonatal Horizon Headwall System)	1 - 60
Sphygmomanometer Mount, Mercurial—P974-01 (Flatwall Headwall System—P670E, Power Column—P950E)	1 - 62
Mercury Sphygmomanometer—P977-00 and Aneroid Sphygmomanometer—P978-01 (Flatwall Headwall System—P670E, Power Column—P950E, Horizon Headwall System—P1000).	1 - 64
Wall Mounted Accessory Track—P979-00	1 - 65
Infusion Pump/IV Holder—P984-00 (Horizon Headwall System—P1000, Flatwall Headwall System—P670E, Power Column—P950E)	1 - 66
Swivel Shelf, Stabile Freestanding Infant Warmer—P986S01 (Power Column—P950E)	1 - 68
NurseMate—Horizon Headwall System Mount—P101105 (Horizon Headwall System—P1000)	1 - 77

NurseMate—Horizon Headwall System Mount Three Drawer (P101135) and Wall Mount Three Drawer (P101136)	1 - 74
Mobile Storage Cabinet—P101202 (Neonatal Horizon Headwall System—P1001) 1 - 78	
Mobile Storage Cabinet—P101203 (Neonatal Horizon Headwall System—P1001) 1 - 80	
Utility Shelf—20"—P101402 (Horizon Headwall System—P1000).	1 - 82
Bed Bumper—P1016A2 (Horizon Headwall System—P1000).	1 - 83
Waste Receptacle Assembly—P101701, 02 (Horizon Headwall System—P1000) 1 - 84	
Pod Mounted STAT Clock/Timer—P101811 (Horizon Headwall System—P1000) 1 - 86	
Utility/Bottle Slide Mount—P1020A1 (Horizon Headwall System—P1000) and Bottle Slide—P1020N1 (Neonatal Horizon Headwall System—P1001AC)	1 - 88
Bird Blender Mount—P1020B1 (Horizon Headwall System—P1000).	1 - 89
Ophthalmoscope Mount—P1020S1, 2 (Horizon Headwall System—P1000) and P1020S3 (Horizon Headwall System—P1000, Console—P1002AC)	1 - 90
Chart Holder—P102101 (Horizon Headwall System—P1000).	1 - 92
Resuscitator Bag Holder—P102201 (Horizon Headwall System—P1000) and P1022PH (Flatwall Headwall System—P670, Power Column—P950).	1 - 94
Universal Holder—P102301 (Horizon Headwall System—P1000, Neonatal Horizon Headwall System—P1001).	1 - 96
Sphygmomanometer Mount, Columnar (Mercurial)—P102402, Aneroid—P102502 (Horizon Headwall System—P1000)	1 - 98
Utility Hook—P102601 (Horizon Headwall System—P1000)	1 - 100
Utility Light—P102701 (Horizon Headwall System—P1000)	1 - 101
Utility Basket—P102801 (Horizon Headwall System—P1000)	1 - 102
Small Storage Basket—P1029S2 and Large Storage Basket—P1029L2 (Horizon Headwall System—P1000).	1 - 104
Bag Assembly—Port Cap, Air Manifold—P1032A1, Nitrogen Manifold—N1, Oxygen Manifold—O1, and Vacuum Manifold—V1 (Horizon Headwall System—P1000) 1 - 106	
Adapter, Air—P1032A7, 8, 9, Oxygen—P1032O7, 8, 9, and Vacuum—P1032V7, 8, 9 1-107	
Rough-In Box—P103402 (Horizon Headwall System—P1000).	1 - 108
TV Arm Mount—RCA/Phillips—P103504, Bunting—P103505, and Phillips—P103506 (Horizon Headwall System—P1000)	1 - 110

Cable Organizer—P103601 (Horizon Headwall System—P1000)	1 - 112
---	---------

Warranty

HILL-ROM® COMPANY, INC. LIMITED WARRANTY

Hill-Rom Company, Inc. (Hill-Rom) has a long tradition of providing superior products and service to our customer. Our goal is "Total Customer Satisfaction". In that spirit, Hill-Rom is proud to offer the following warranty.

GENERAL WARRANTY (APPLICABLE UNLESS A SPECIFIC WARRANTY IS LISTED)

Hill-Rom warrants to the original purchaser that its products and replacement parts shall be free from defects in material and workmanship for a period of one (1) year from date of delivery. Hill-Rom's obligation under this warranty is expressly limited to supplying replacement parts and/or service for, or replacing, at its option, any product which is, in the sole discretion of Hill-Rom, found to be defective. In addition to the foregoing one year warranty, Hill-Rom warrants to the original purchaser that the frame and welds on its products will be free from structural defects for the life of the product. Any product upgrade or modification initiated by Hill-Rom does not affect the original product warranty.

SPECIFIC WARRANTIES

MATTRESS WARRANTIES

Hill-Rom warrants to the original purchaser that its mattress product shall be free from defects in material and workmanship for a period of two (2) years from date of delivery. However, electro mechanical mattress components (compressors, valves, printed circuit boards, hoses, and couplers) are covered by the general one (1) year warranty.

EXPENDABLES WARRANTIES

A sixty (60) day limited warranty from date of delivery applies to expendable parts such as cushions, coverlets, software diskettes, locator badge batteries, dome light incandescent bulbs, overhead fluorescent tubes, heating elements, and temperature probes. This warranty is limited to replacement of the parts covered.

TO OBTAIN PARTS AND SERVICE

In the United States, call Hill-Rom Technical Support Department at (800) 445-3720, Monday through Friday. In Canada, call Hill-Rom Technical Support Department at (800) 267-2337, Monday through Friday. Outside the United States and Canada, call your authorized Hill-Rom Distributor. In order to expedite service, we request you furnish the following information: customer identification number, product model number, serial number, and description of problem. A qualified specialist will provide, via telephone (United States and Canada), or FAX (Outside the United States and Canada), troubleshooting assistance for facility personnel and provide necessary parts to make repairs. If troubleshooting determines the need for on-site technical service, a qualified service representative will be dispatched. Replacement of non-technical items will be the responsibility of the customer. If requested by Hill-Rom, products or parts for which a warranty claim is made shall be returned prepaid to Hill-Rom's factory.

OUT OF WARRANTY EXCHANGE POLICY

After the expiration of the original warranty, upon request, Hill-Rom will ship as a replacement, components such as selected motors, printed circuit boards, and mattresses, for like units returned to Hill-Rom by the original purchaser for forty percent (40%) of the then-current new price.

PART AVAILABILITY POLICY

Hill-Rom will offer parts for new products for fifteen (15) years from date of last manufacture; for remanufactured products, for ten (10) years from date of sale; and for communications products, for five (5) years from date of sale. Note: Some parts may only be available in assembled form, not at the component level.

THE FOREGOING WARRANTIES ARE EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES AND IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS OF PURPOSE. HILL-ROM'S OBLIGATION UNDER THESE

WARRANTIES SHALL NOT INCLUDE ANY LIABILITY FOR LOSS OF PROFITS, DIRECT, INDIRECT OR CONSEQUENTIAL DAMAGES OR DELAYS. Some states, provinces, or countries do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusion or limitation may not apply. Any improper or negligent use, any alterations or repairs not in accordance with Hill-Rom's manuals or performed by others in such manner as in Hill-Rom's sole judgment affects the product materially and adversely, shall void these warranties. These warranties do not cover failures due to misuse, abuse, neglect, or lack of routine maintenance. No employee or representative of Hill-Rom is authorized to change these warranties in any way or grant any other warranty unless in writing and signed by a Hill-Rom officer. These warranties provide specific legal rights; but, there may be other available rights, which vary from state to state, province to province, or country to country.

Revised December 10, 1996

NOTES:

Ordering Service Parts

Use the parts lists in this service manual to identify the part numbers you require.

Call your Technical Customer Support Specialist at the Hill-Rom Technical Support Department—phone (800) 445-3720. To help expedite the processing of your parts order, please have your six-digit customer account number, purchase order number, product number, and serial number available for the Technical Customer Support Specialist when you call.

For your convenience, Hill-Rom provides a telefax number to promptly order parts, request part prices and availability, or to follow up on a service order. The telefax number is (812) 934-8472.

We suggest a minimum of \$40.00 when placing orders for service parts. This will help prevent an increase in the cost of processing your service order.

Terms:

- Net 30 days.
- F.O.B. Batesville, Indiana.
- Shipping charges are prepaid and added to the invoice.
- All service orders are shipped UPS ground, unless you specifically request an alternative method.

Address all inquiries to:

Hill-Rom Company
1069 State Route 46 E
Batesville, Indiana 47006-9167
Attention: Technical Support—Parts

Address all return goods to:

Hill-Rom Company
Distribution Center Door D23
County Road 300E
Batesville, Indiana 47006-9167
Attention: Service Stores

NOTE:

To eliminate possible delays or incorrect billings, **do not** return any items without a Return Material Authorization (RMA) number. A Return Material

Authorization packet is included with each order when a return is requested. This packet includes an RMA number, instructions, and a shipping label. If misplaced, obtain an RMA number by phoning the Hill-Rom Technical Support Department at (800) 445-3720.

Exchange Policy

The following are Hill-Rom's policies for in-warranty and out-of-warranty exchanges.

In-Warranty Exchanges

Hill-Rom will request that parts/products be returned for inspection in some cases. When this occurs, you are expected to return parts/products within 30 days. If you fail to return the inoperative parts/products within the 30 day period, Hill-Rom will invoice your facility for the full selling price of the parts/products.

NOTE:

The preceding billing procedure **only** pertains to parts/products that Hill-Rom requests to be returned.

In some cases, the invoice sent with the parts will show the full selling price of the parts. This is for Hill-Rom's internal use only and should not be confused with the price you will actually pay.


Please do not return any parts without an RMA number. Hill-Rom will include a Return Material Authorization packet with the parts/products shipment when parts/products have been requested to be returned. If misplaced, obtain an RMA number by phoning the Hill-Rom Technical Support Department at (800) 445-3720.

Out-of-Warranty Exchanges

You are expected to return the inoperative parts/product to Hill-Rom within 30 days. Hill-Rom will include a Return Material Authorization packet with the parts/products shipment. If misplaced, obtain an RMA number by phoning the Hill-Rom Technical Support Department at (800) 445-3720. If you fail to return the equipment within thirty days, Hill-Rom will invoice your facility for **the difference between the exchange price and the new price of the part.**

Raceway, 8"—P644E8L, R (Specialite Patient Light)


Figure 1-1. Raceway, 8"—P644E8L, R (Specialite Patient Light)

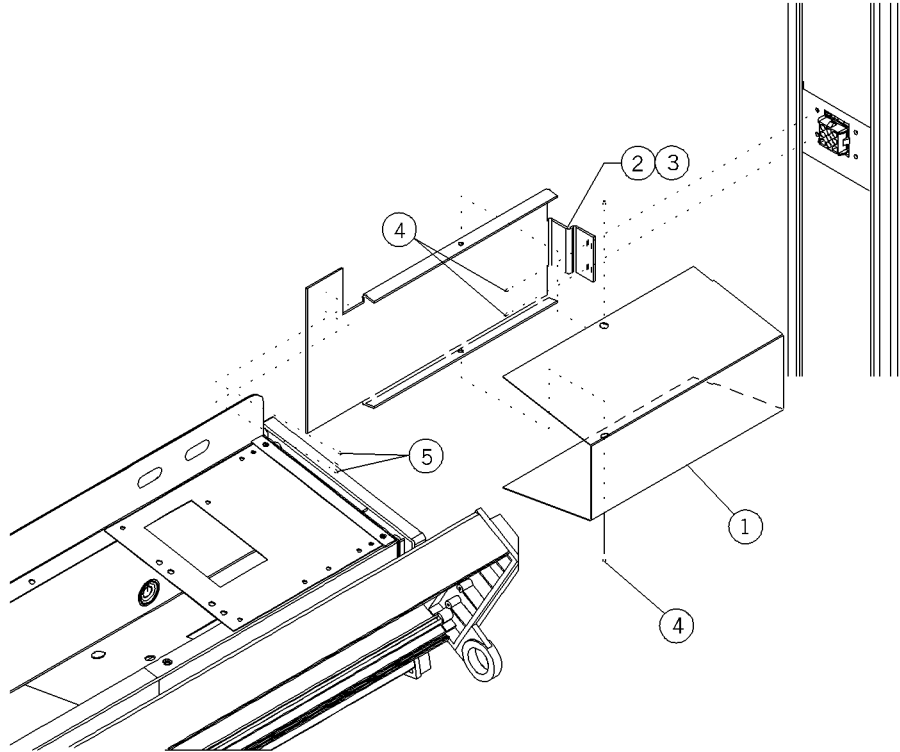

m044a001

Table 1-1. Raceway, 8"—P644E8L, R (Specialite Patient Light)

Item Number	Part Number	Quantity	Description
1	55521	2	Cover—8"—raceway
2	55506-01	1	Backplate—8"—left-hand
3	55506-02	1	Backplate—8"—right-hand
4	52555-03	4	Screw
5	29591	4	Screw

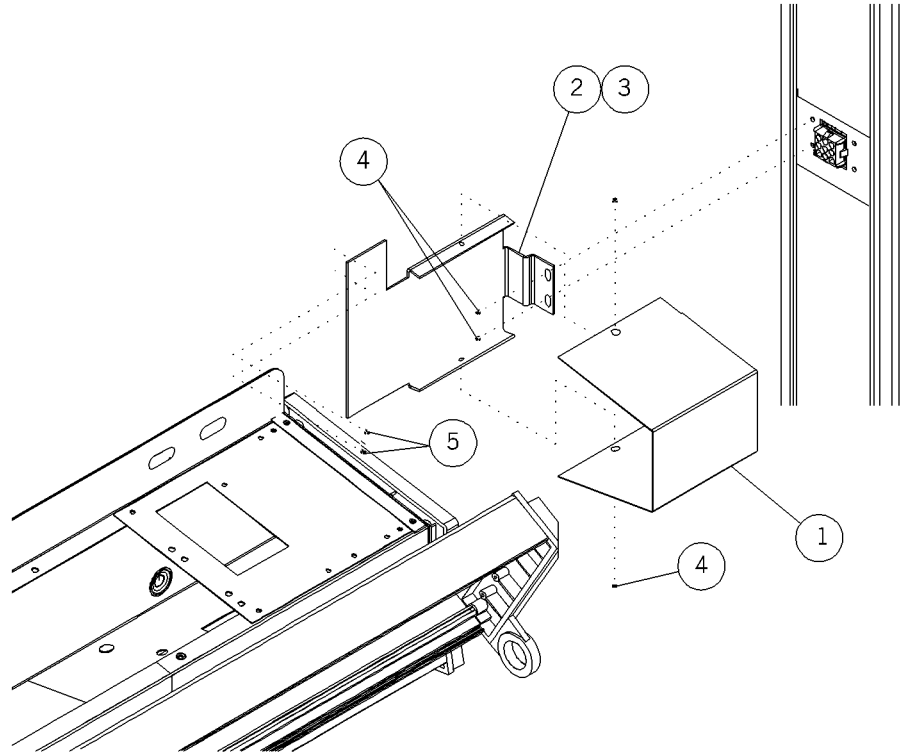Raceway, 4"—P644E4L, R (Specialite Patient Light)**Figure 1-2. Raceway, 4"—P644E4L, R (Specialite Patient Light)**


m044a002

Table 1-2. Raceway, 4"—P644E4L, R (Specialite Patient Light)

Item Number	Part Number	Quantity	Description
1	55520	2	Cover—4" raceway
2	5550701	1	Backplate 4", left-hand
3	5550702	1	Backplate 4", right-hand
4	52555-03	4	Screw
5	29591	4	Screw

Connection—P644E0L, R (Specialite Patient Light)

Figure 1-3. Connection—P644E0L, R (Specialite Patient Light)

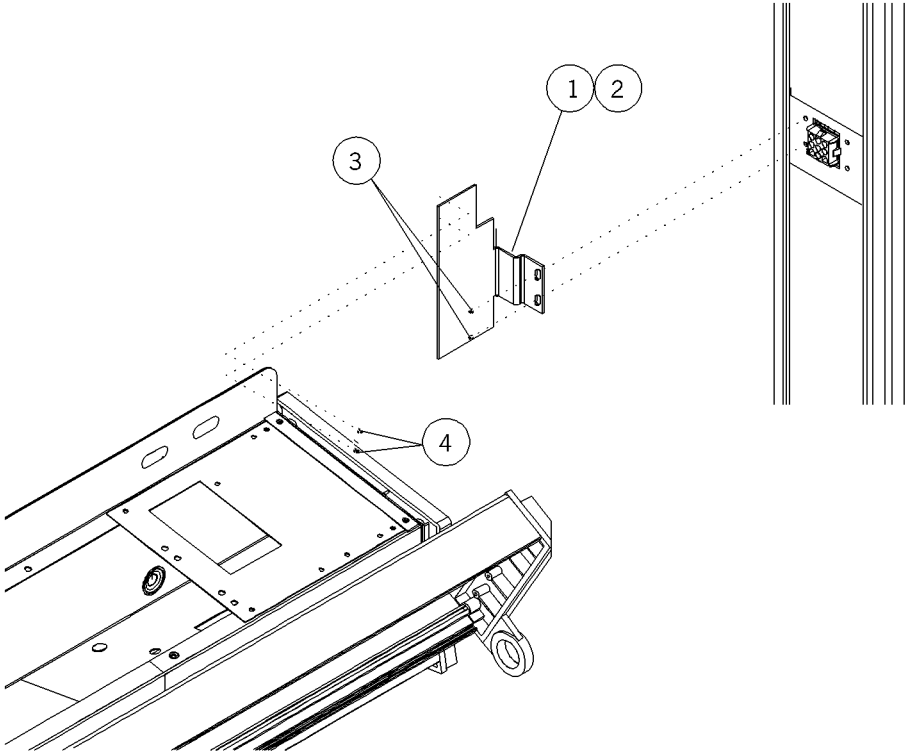


m044a003

Table 1-3. Connection—P644E0L, R (Specialite Patient Light)

Item Number	Part Number	Quantity	Description
1	5549801	1	Splice plate, fw attachment left-hand
2	5549802	1	Splice plate, fw attachment right-hand
3	52555-03	4	Screw
4	29591	4	Screw

Conversion Kit, P991-00 Monitor Mount—P651C991 (Power Column—P950E)

Figure 1-4. Conversion Kit, P991-00 Monitor Mount—P651C991 (Power Column—P950E)

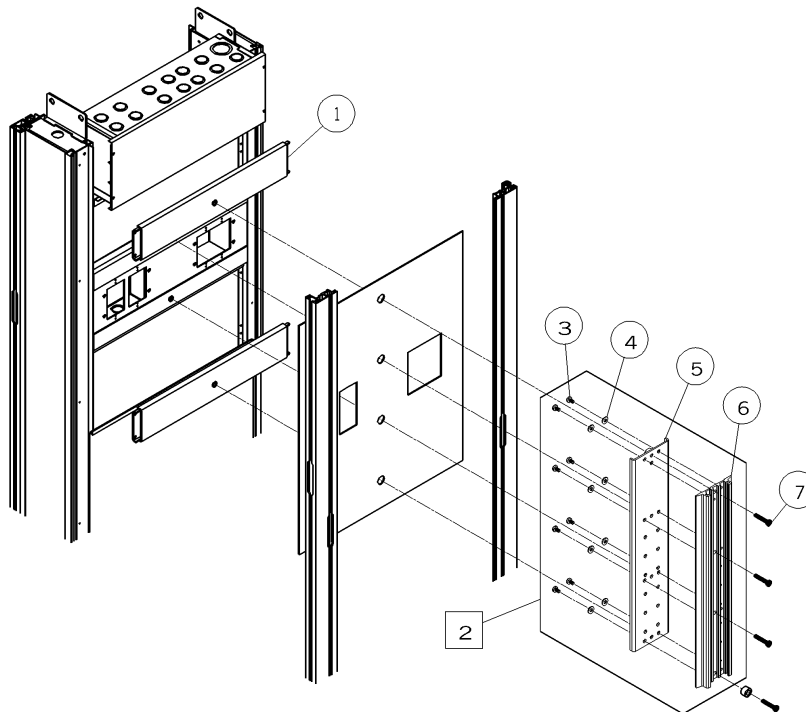

m044a004


Table 1-4. Conversion Kit, P991-00 Monitor Mount—P651C991 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	56750	2	Support assembly
2	56749	1	Channel/backing plate assembly
3	53063	8	Screw, pan head
4	53065-01	8	Washer
5	5664039	1	Backing plate assembly, monitor mounting channel
6	56637	1	Monitor mounting channel
7	5654603	4	Screw, pan head

NOTES:

Hanger—Flatwall—P671E01, 02, 03 and Flatwall Hanger, Seismic—P671S01, 02, 03 (Flatwall Headwall System—P670E)


Figure 1-5. Hanger—Flatwall—P671E01, 02, 03 and Flatwall Hanger, Seismic—P671S01, 02, 03 (Flatwall Headwall System—P670E)

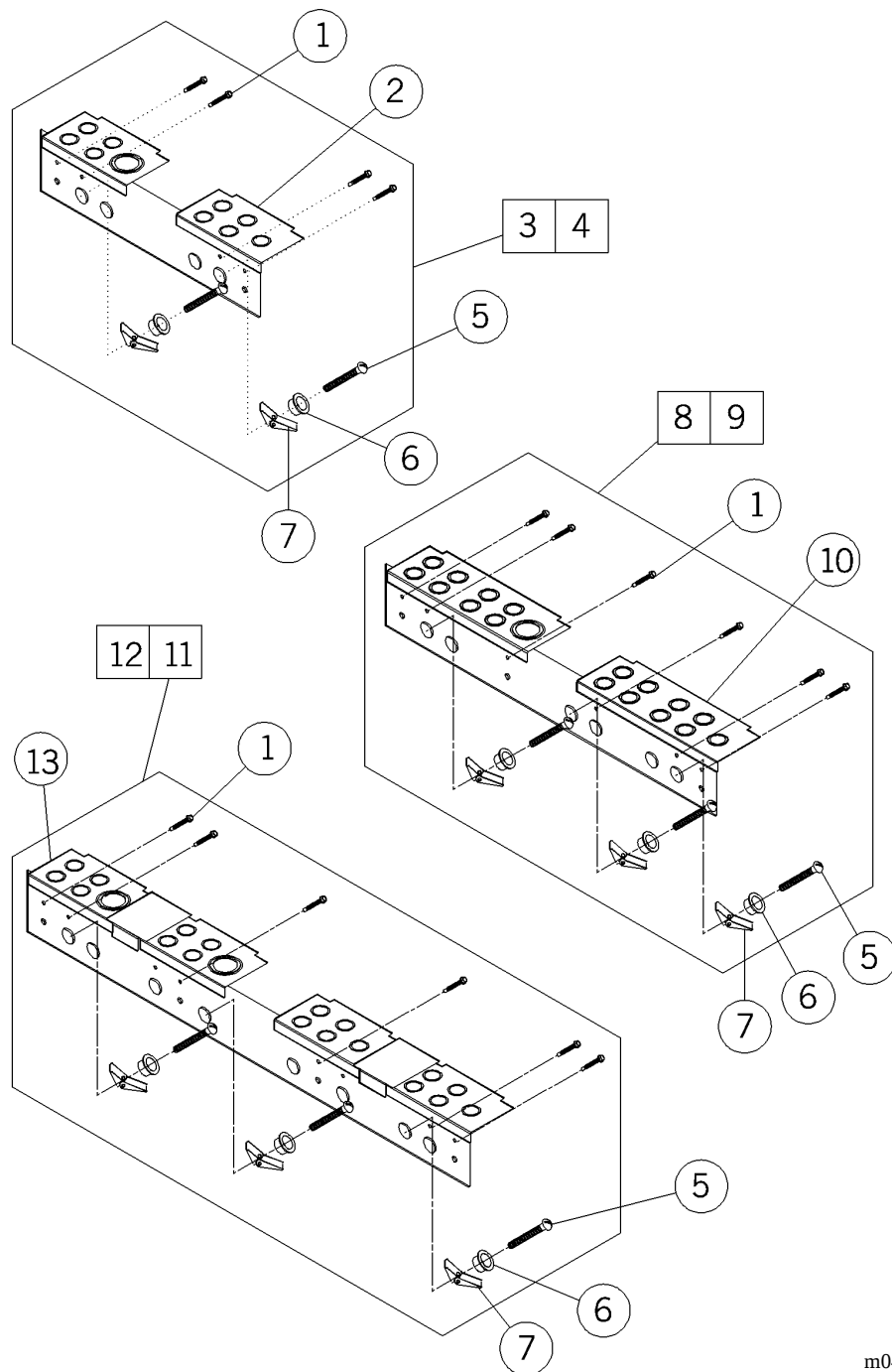

m044a005

Table 1-5. Hanger—Flatwall—P671E01, 02, 03 and Flatwall Hanger, Seismic—P671S01, 02, 03 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	53143-01	4 or 6	Screw, self-drilling hex head
2	5237939	1	Hanger assembly—19"
3	671E01	1	Hanger, flatwall—19"
4	P671S01	1	Flatwall hanger, 19" seismic
5	21045	2 or 3	Toggle bolt
6	29325	2 or 3	Stud guide
7	21046	2 or 3	Toggle wing
8	671E02	1	Hanger—flatwall—28"
9	P671S02	1	Flatwall hanger, 28" seismic
10	5238039	1	Hanger assembly—28"
11	P671E03	1	Hanger—flatwall—37"
12	P671S03	1	Flatwall hanger, 37" seismic
13	5238139	1	Hanger assembly—37"

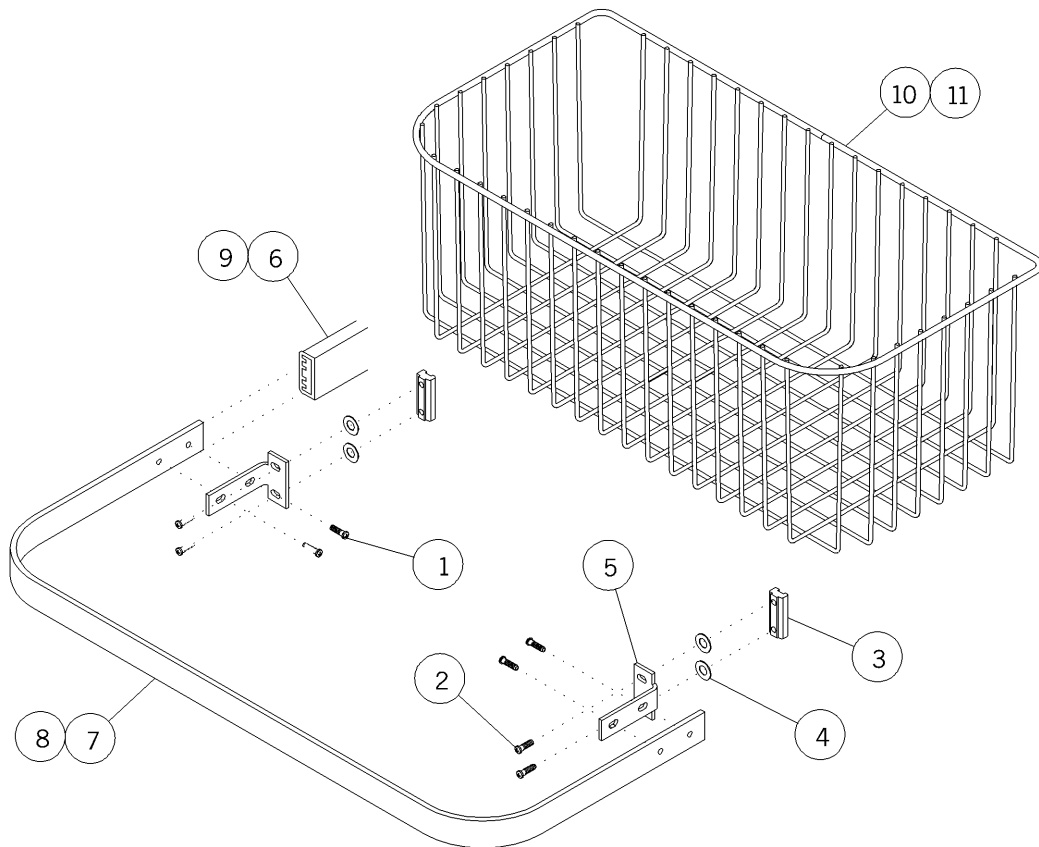
Storage Basket—P672E03, 04 (Flatwall Headwall System—P670E)**Figure 1-6. Storage Basket—P672E03, 04 (Flatwall Headwall System—P670E)**

m044a006

Table 1-6. Storage Basket—P672E03, 04 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	54105	4	Pivot screw
2	15338	4	Screw
3	34026	2	Slide
4	56841	4	Shim washer
5	56830-pl	2	Bottle guard bracket
6	57160	1	Bumper blank
7	522950139*	1	Bottle guard support
8	522950239*	1	Bottle guard support
9	29539	1	Bumper cover
10	55221	1	Basket—large—19"
11	55222	1	Basket—large—28"

* Apply a small amount of liquid soap to this part in assembly process.

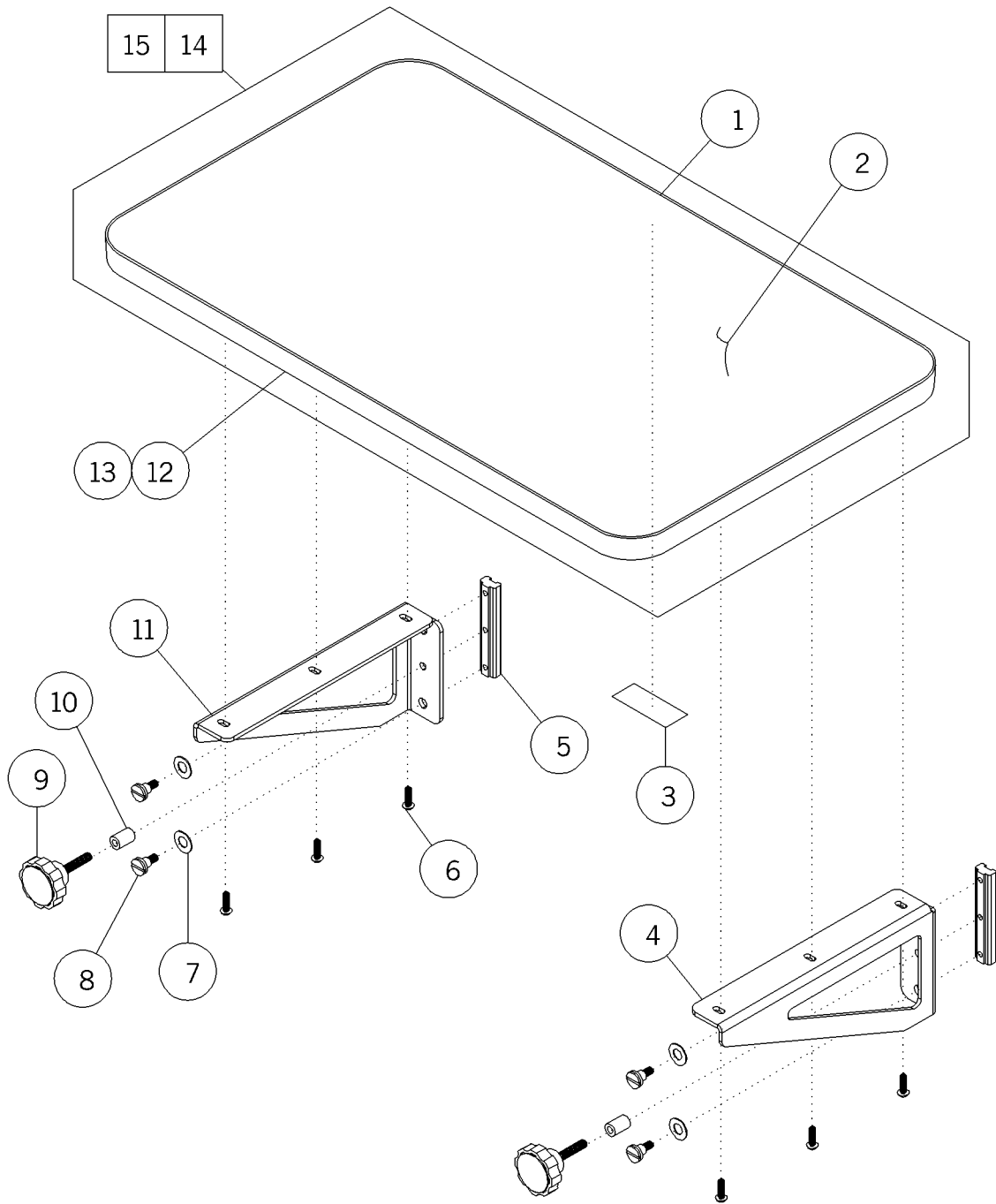Adjustable Shelf—P673E21, 30 (Flatwall Headwall System—P670E)**Figure 1-7. Adjustable Shelf—P673E21, 30 (Flatwall Headwall System—P670E)**

m044a007


Table 1-7. Adjustable Shelf—P673E21, 30 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	27246	2	Power pin
2	32341*	1	Panel blank
3	54607	1	Label, load
4	52482-01	1	Shelf mount—right-hand
5	52658	2	Slide insert
6	50854-03	6	Pan head screw
7	56841	4	Shim washer
8	34002	4	Screw
9	33167	2	Knob
10	29826	2	Spacer
11	52482-02	1	Shelf mount—left-hand
12	54524-02	1	Trim, shelf—light neutral
13	5452401	1	Trim, shelf
14	5317920*	1	Shelf top
15	5317929*	1	Shelf

* Specify high pressure laminate color.

Flatwall Bumper—P674E01, 02, 03 and Flatwall Bumper/Bottle Rack—P674E04, 05, 06 (Flatwall Headwall System—P670E)

Figure 1-8. Flatwall Bumper—P674E01, 02, 03 and Flatwall Bumper/Bottle Rack—P674E04, 05, 06 (Flatwall Headwall System—P670E)

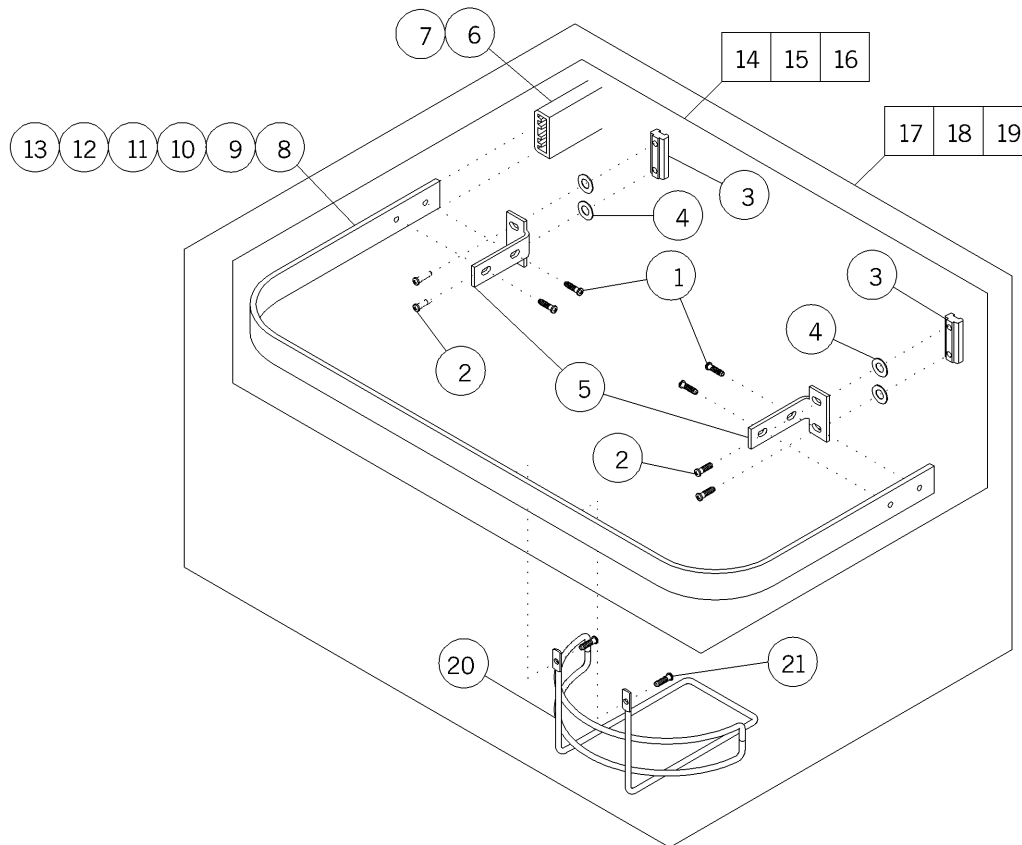

m044a008


Table 1-8. Flatwall Bumper—P674E01, 02, 03 and Flatwall Bumper/Bottle Rack—P674E04, 05, 06 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	54105	4	Pivot screw
2	15338	4	Screw
3	34027	2	Slide assembly
4	56841	4	Shim washer
5	51107	2	Bumper bracket
6	29539	1	Bumper cover
7	57160	1	Bumper blank
8	509940139*	1	Bumper support
9	509940239*	1	Bumper support
10	509940339*	1	Bumper support
11	5113739*	1	Bumper support
12	5113839*	1	Bumper support
13	5113939*	1	Bumper support
14	P674E01	1	Flatwall bumper—19"
15	P674E02	1	Flatwall bumper—28"
16	P674E03	1	Flatwall bumper—37"
17	P674E04	1	Flatwall bumper/bottle rack—19"
18	674E05	1	Flatwall bumper/bottle rack—28"
19	P674E06	1	Flatwall bumper/bottle rack—37"
20	29537	1	Bottle bracket
21	53397-03	2	Screw

* Apply a small amount of liquid soap to this part in assembly process.

Pivoting Storage Basket—Shallow—P951S03 and Pivoting Waste Basket, Deep—P951W03 (Power Column—P950E)


Figure 1-9. Pivoting Storage Basket—Shallow—P951S03 and Pivoting Waste Basket, Deep—P951W03 (Power Column—P950E)

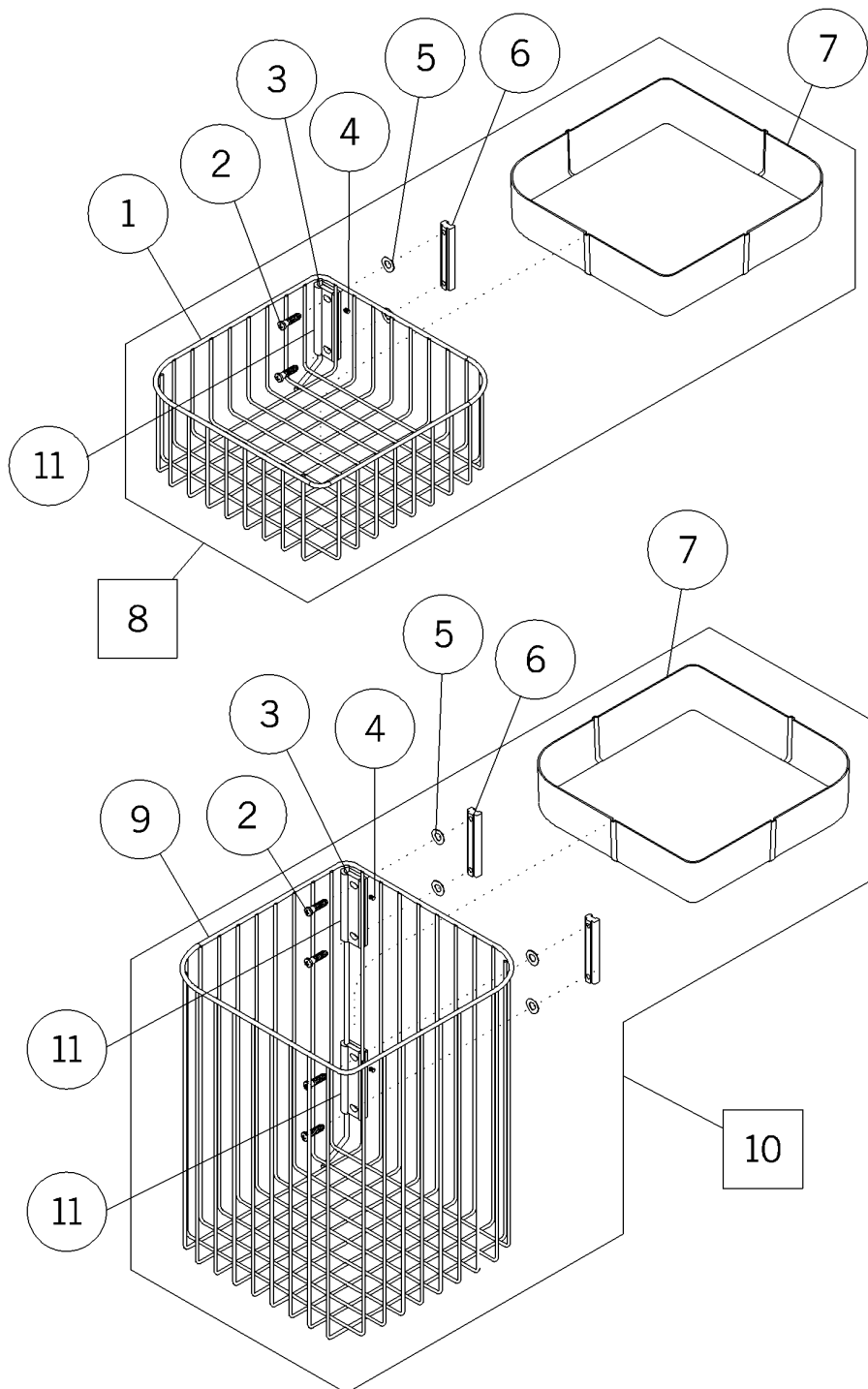

m044a009

Table 1-9. Pivoting Storage Basket—Shallow—P951S03 and Pivoting Waste Basket, Deep—P951W03 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	55202	1	Basket—pivoting—small
2	52396	2 or 4	Screw
3	54852	1 or 2	Hinge hook
4	54850	1 or 2	Screw
5	56841	2 or 4	Shim washer
6	34027	1 or 2	Slide assembly
7	54455	1	Liner
8	951S03	1	Pivot storage basket—shallow
9	55205	1	Deep swing out basket
10	951W03	1	Pivoting waste basket, deep
11	5754339	1 or 2	Locking plate, hinge

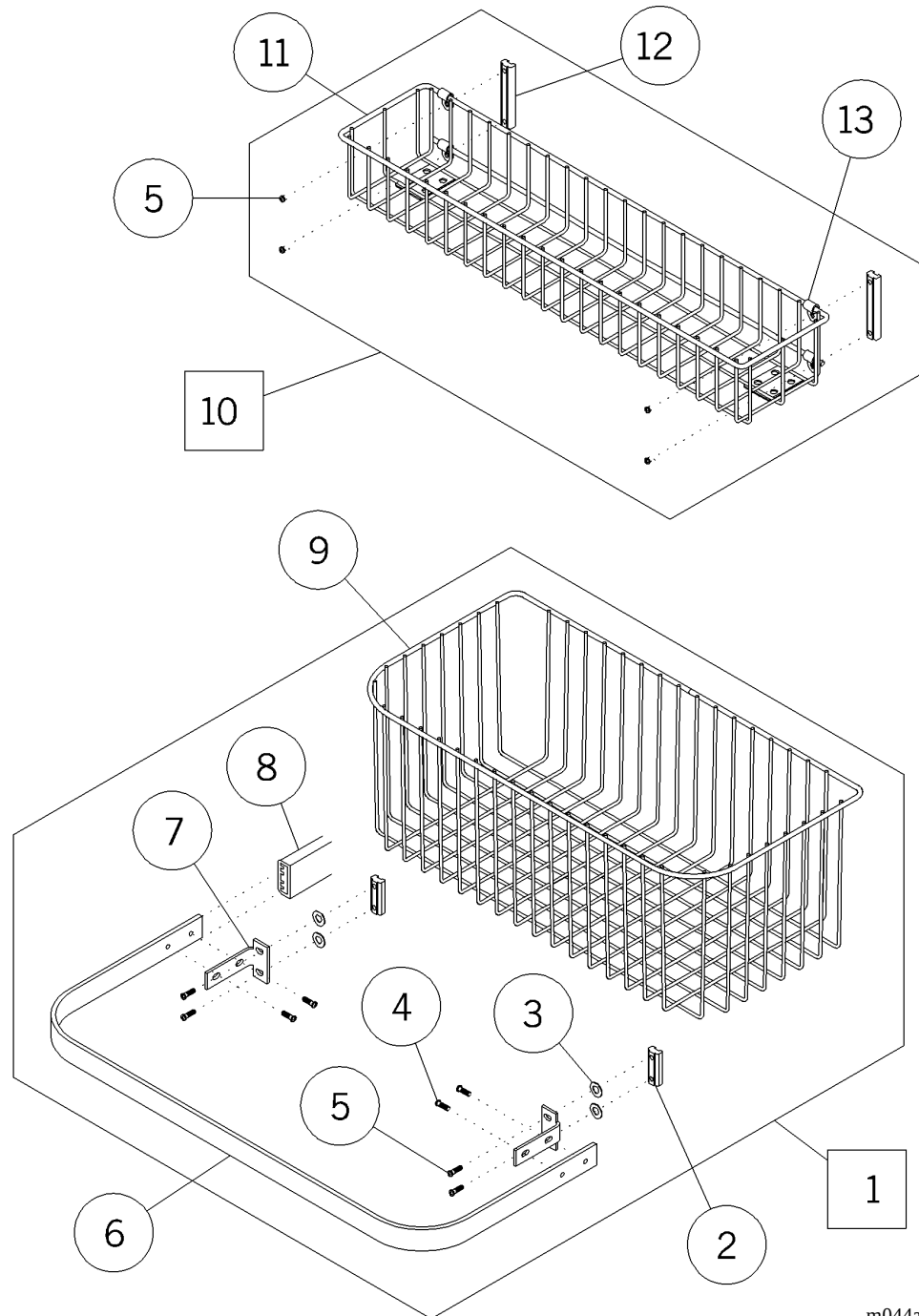
**Basket, Large—P952L01 and Small Storage Basket—P952S01
(Power Column—P950E)****Figure 1-10. Basket, Large—P952L01 and Small Storage Basket—P952S01 (Power
Column—P950E)**

m044a010

Table 1-10. Basket, Large—P952L01 and Small Storage Basket—P952S01 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	952L01	1	Basket—large
2	34026	2	Slide
3	56841	4	Shim washer
4	54105	4	Pivot screw
5	15338	4	Screw
6	31421*	1	Bottle guard support
7	56830-pl	2	Bottle guard bracket
8	57160	1	Bumper blank
9	55223	1	Basket—large
10	P952S01	1	Small storage basket
11	56921	1	Basket, small
12	34027	2	Slide assembly
13	55201	2	Cable clamp

* Apply a small amount of liquid soap to this part in assembly process.

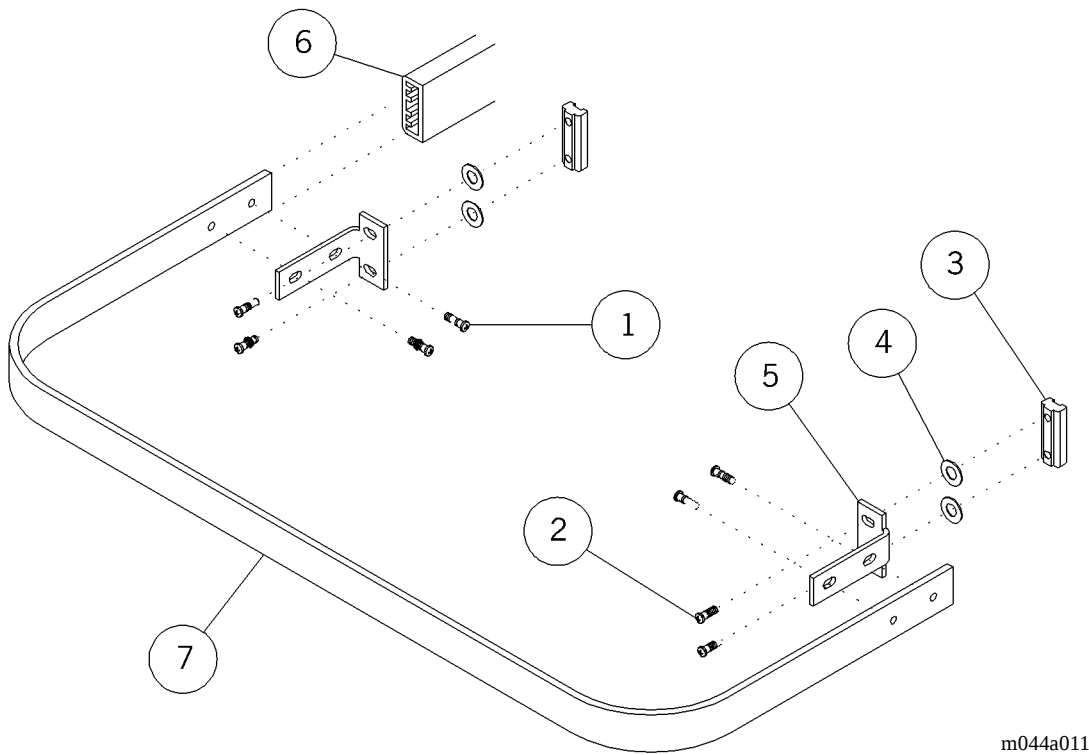Bottle Guard Bumper—P953-00 (Power Column—P950E)**Figure 1-11. Bottle Guard Bumper—P953-00 (Power Column—P950E)**

m044a011


Table 1-11. Bottle Guard Bumper—P953-00 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	54105	4	Pivot screw
2	15338	4	Screw
3	34026	2	Slide
4	56841	4	Shim washer
5	56830-pl	1	Bottle guard bracket
6	57160	1	Bumper blank
7	31421*	1	Bottle guard support

* Apply a small amount of liquid soap to this part in assembly process.

Floor Mounted Bed Locator—P958-00 (Power Column—P950E)


Figure 1-12. Floor Mounted Bed Locator—P958-00 (Power Column—P950E)

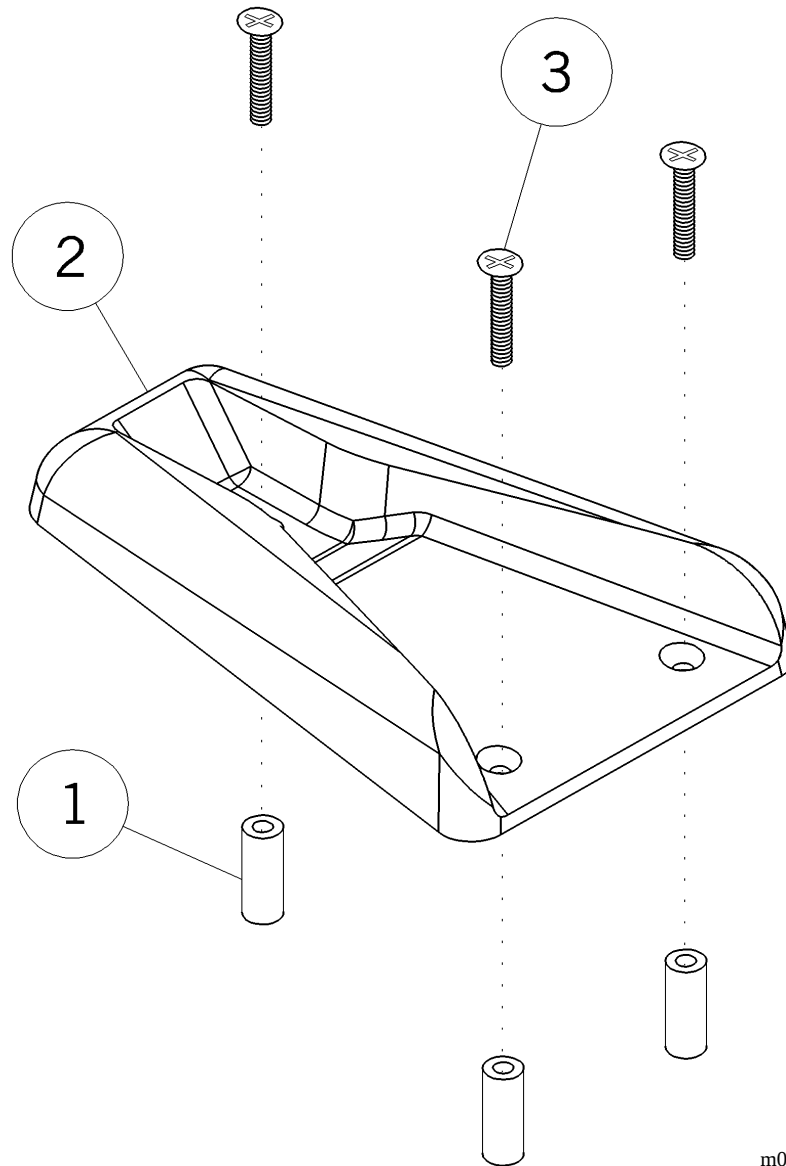

m044a012

Table 1-12. Floor Mounted Bed Locator—P958-00 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	28957	3	Expansion shield
2	28903	1	Floor bed locator
3	50999-01	3	Screw—flat head

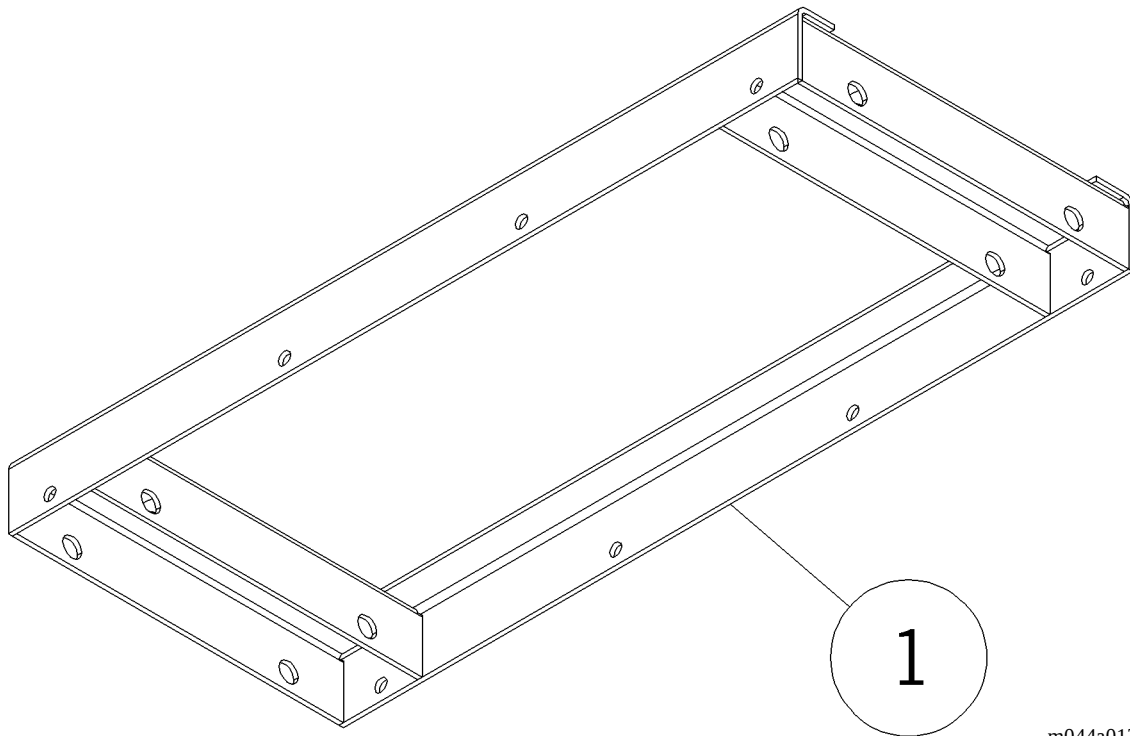
Ceiling Ring—P959E00 (Power Column—P950E)**Figure 1-13. Ceiling Ring—P959E00 (Power Column—P950E)**

m044a013

Table 1-13. Ceiling Ring—P959E00 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	50011	1	Ceiling ring

NOTES:

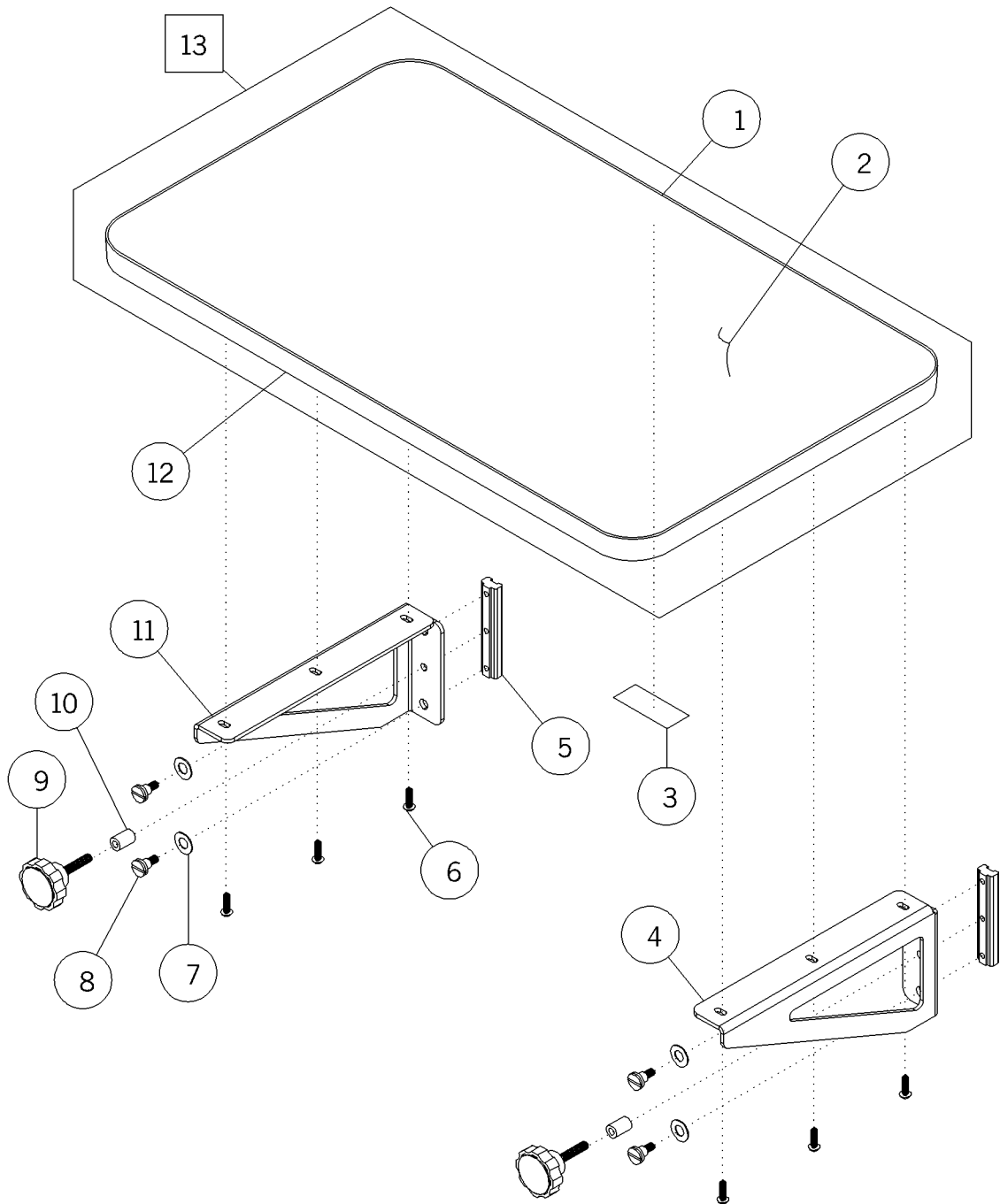
Adjustable Shelf—P961E02 (Power Column—P950E)**Figure 1-14. Adjustable Shelf—P961E02 (Power Column—P950E)**

m044a014

Table 1-14. Adjustable Shelf—P961E02 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	27246	2	Power pin
2	32341*	1	Panel blank
3	54607	1	Label, load
4	52482-01	1	Shelf mount—right-hand
5	52658	2	Slide insert
6	50854-03	6	Pan head screw
7	56841	4	Shim washer
8	34002	4	Screw
9	33167	2	Knob
10	29826	2	Spacer
11	52482-02	1	Shelf mount—left-hand
12	54524-02	1	Trim, shelf—light neutral
13	5317922*	1	Shelf—22"

* Specify high pressure laminate color.

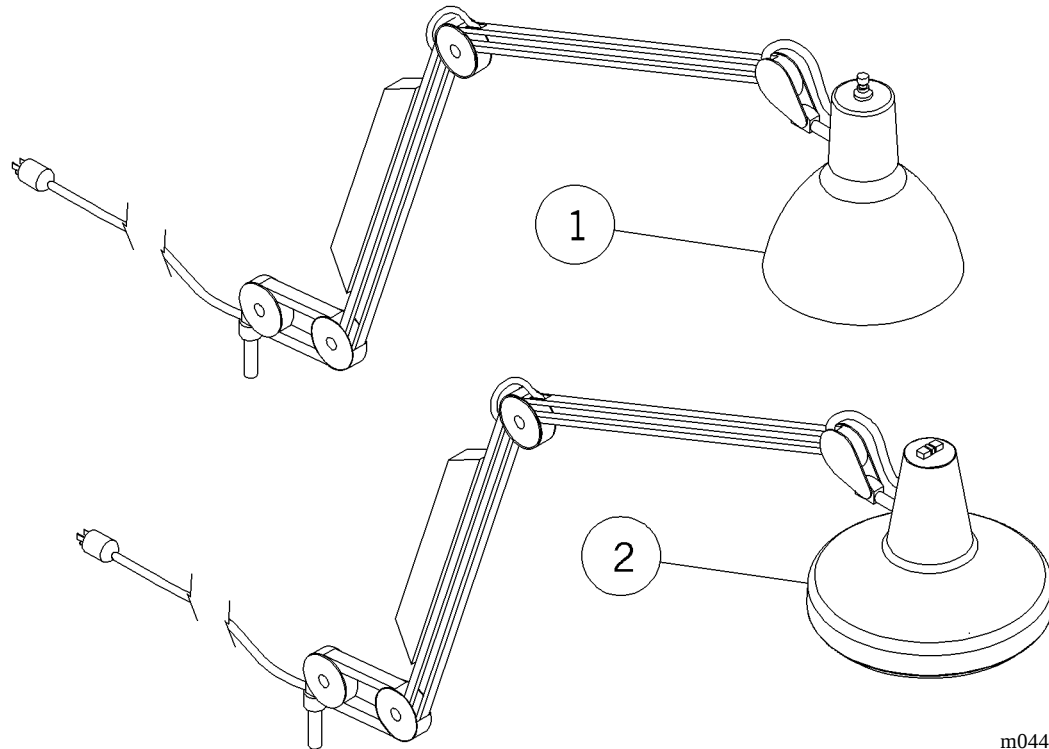Incandescent/Fluorescent Arm Lamp—P963C02 (Flatwall Headwall System—P670E, Power Column—P950E)**Figure 1-15. Incandescent/Fluorescent Arm Lamp—P963C02 (Flatwall Headwall System—P670E, Power Column—P950E)**


m044a015

Table 1-15. Incandescent/Fluorescent Arm Lamp—P963C02 (Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	55924	1	Arm lamp, incandescent, 100 watt maximum
2	55925	1	Arm lamp, incandescent/fluorescent

**Mount, Arm Light—P963M00 (Flatwall Headwall System—P670E,
Power Column—P950E)**

**Figure 1-16. Mount, Arm Light—P963M00 (Flatwall Headwall System—P670E, Power
Column—P950E)**

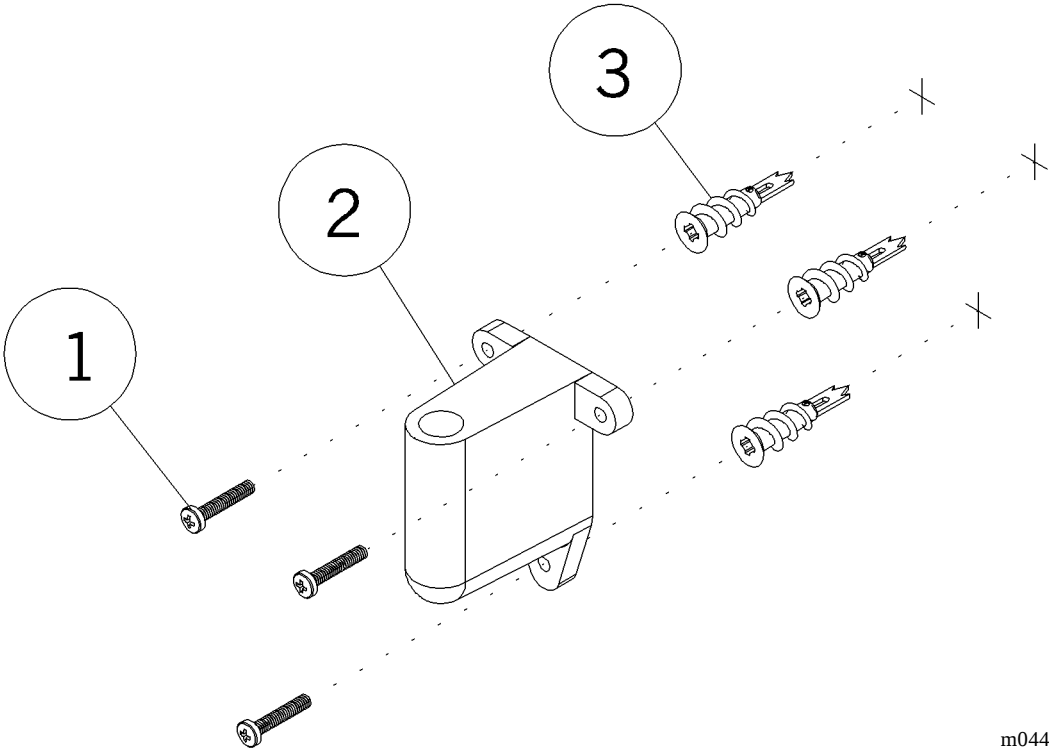


**Table 1-16. Mount, Arm Light—P963M00 (Flatwall Headwall System—
P670E, Power Column—P950E)**

Item Number	Part Number	Quantity	Description
1	52243-07	3	Screw
2	55926	1	Arm lamp bracket mounting
3	57185	3	E-Z anchor

STAT Clock/Timer Pod Mounted—P964-00 (Flatwall Headwall System—P670E, Power Column—P950E)

Figure 1-17. STAT Clock/Timer Pod Mounted—P964-00 (Flatwall Headwall System—P670E, Power Column—P950E)

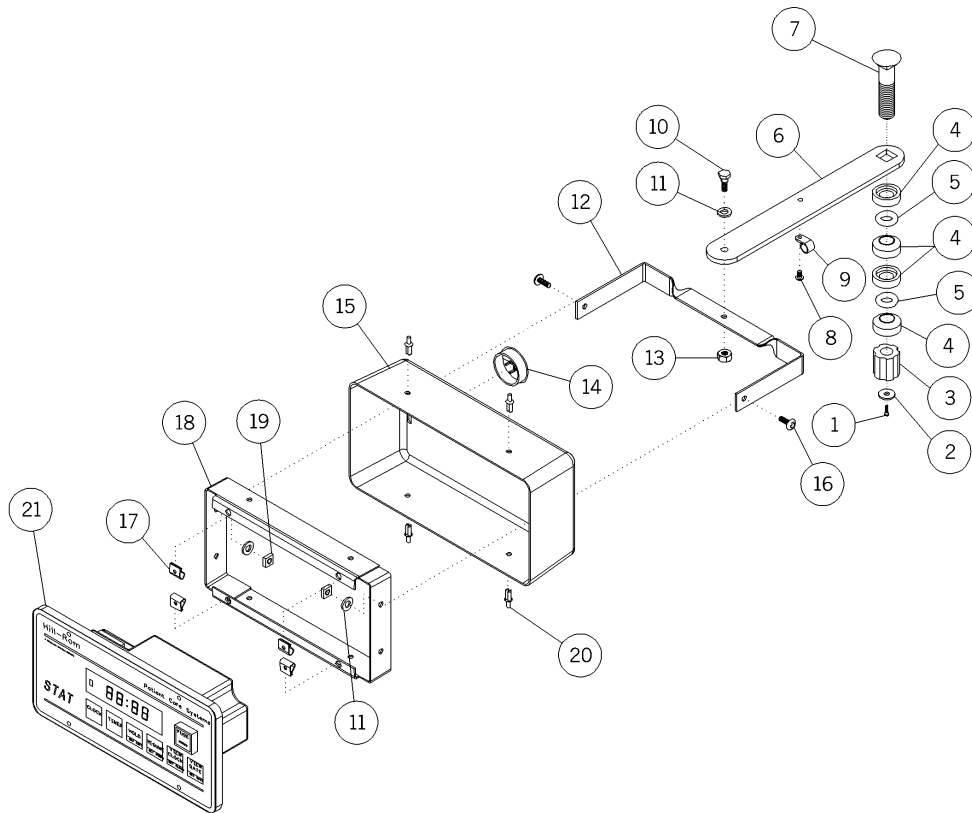


m044a017

Table 1-17. STAT Clock/Timer Pod Mounted—P964-00 (Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53397-03	1	Screw
2	52386	1	Washer
3	34218	1	Compression nut
4	56530	4	Washer, beveled nylon
5	50126	2	O-ring
6	33718	1	Arm
7	33704	1	Bolt
8	52604-02	1	Screw
9	54627	1	Cable clamp
10	56398	1	Shoulder bolt
11	33721	3	Nylon washer
12	33693	1	Bail
13	56400	1	Nut, shakeproof
14	33720	1	Hole plug
15	33701	1	Cover
16	51637	2	Screw
17	31992	4	Speed nut
18	33694	1	Pivot frame
19	33653	2	Nut
20	33719	4	Rivet
21	967-00	1	STAT clock timer

Mount Assembly—Exam Plus Light—P965M01 (Power Column—P950E)

Figure 1-18. Mount Assembly—Exam Plus Light—P965M01 (Power Column—P950E)

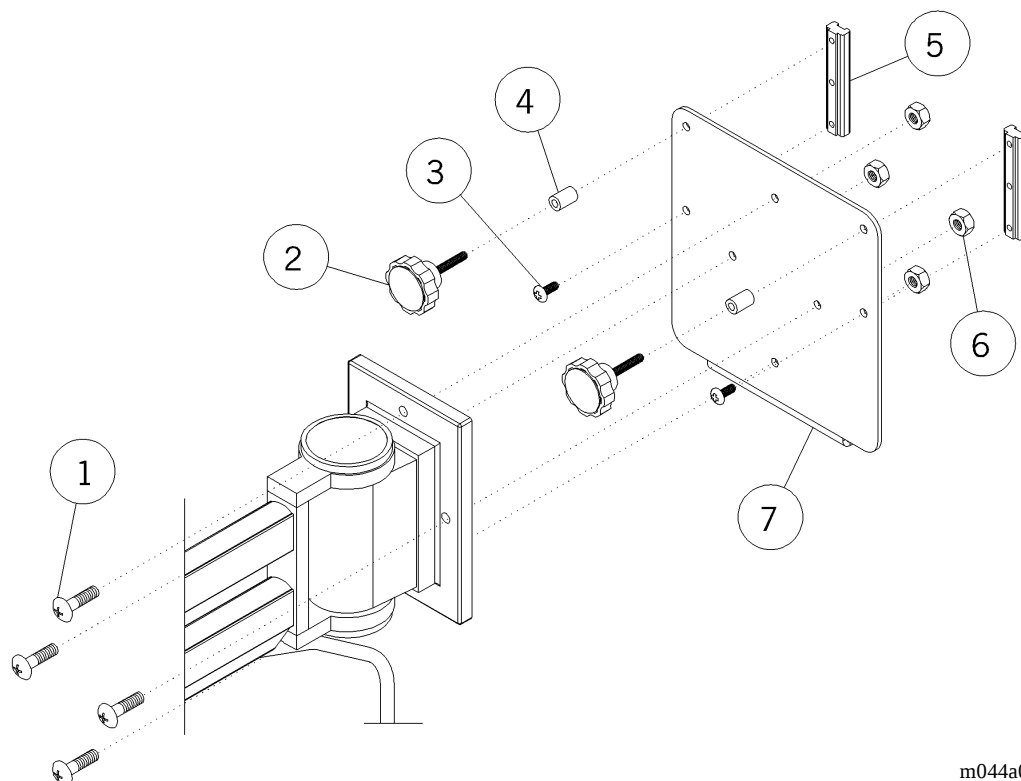


m044a018

Table 1-18. Mount Assembly—Exam Plus Light—P965M01 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53062-01	4	Screw, pan head
2	33167	2	Knob
3	51637	2	Screw
4	29826	2	Spacer
5	52658	2	Slide insert
6	33680	4	Locknut
7	5330039	1	Base, Exam Plus mount

STAT Clock/Timer—Conventional Wall Mount—P967-01

Figure 1-19. STAT Clock/Timer—Conventional Wall Mount—P967-01

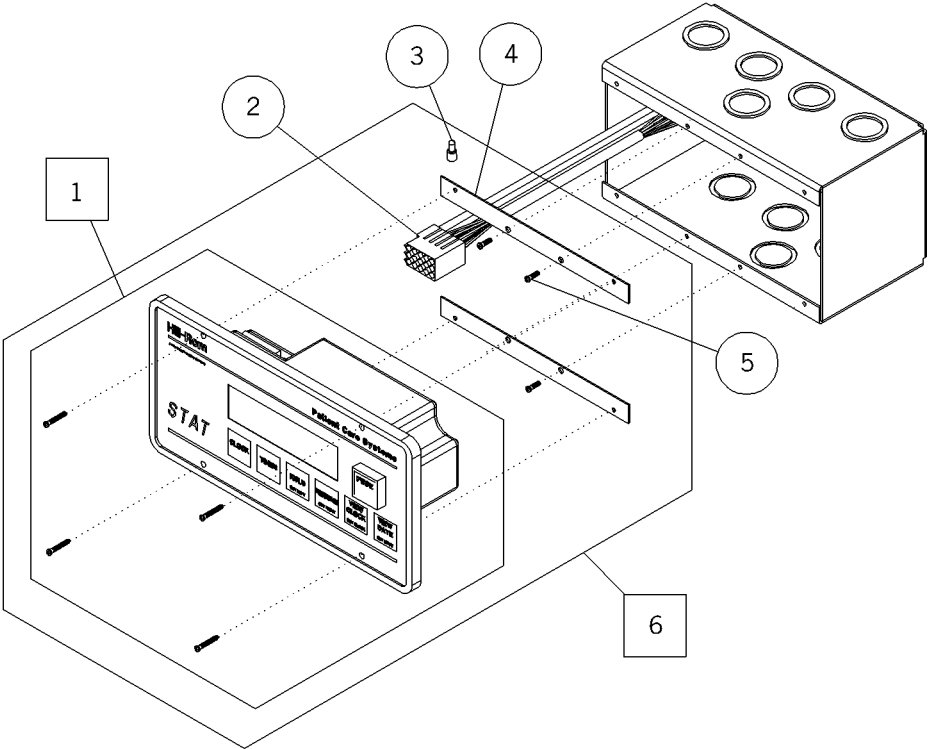


m044a019

Table 1-19. STAT Clock/Timer—Conventional Wall Mount—P967-01

Item Number	Part Number	Quantity	Description
1	967-00	1	STAT clock timer
2	33740	1	Pigtail
3	15408	10	Wire nut
4	33739	2	Mounting strap
5	52604-02	4	Screw
6	967-01	1	STAT clock timer

Universal Holder, Multi Point—P970A01 (Flatwall Headwall System—P670E, Power Column—P950E)

Figure 1-20. Universal Holder, Multi Point—P970A01 (Flatwall Headwall System—P670E, Power Column—P950E)

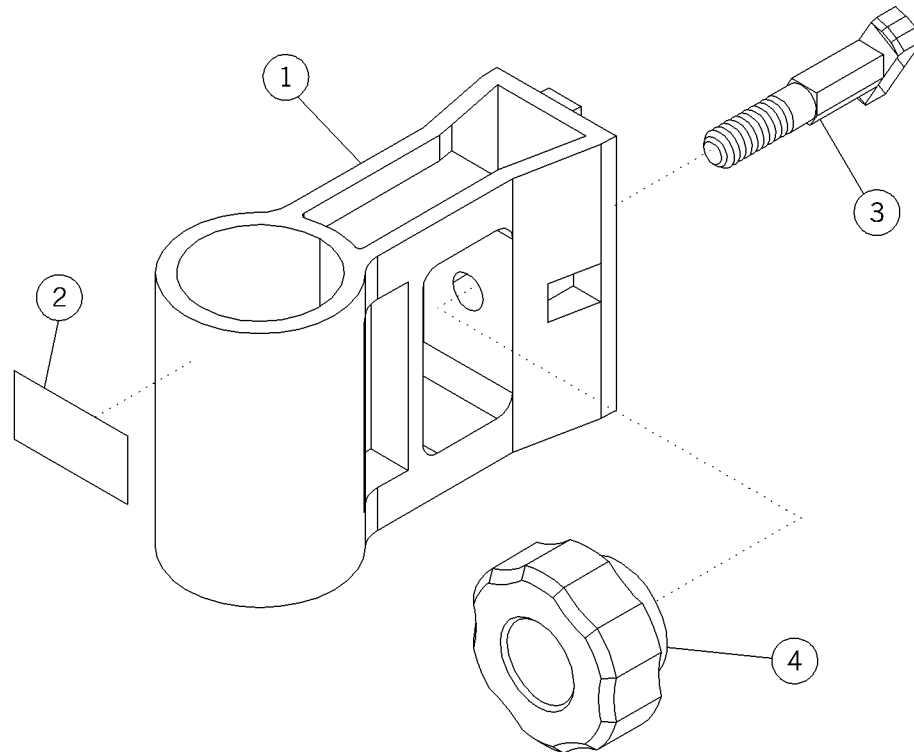

m044a020


Table 1-20. Universal Holder, Multi Point—P970A01 (Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	54526	1	Holder—body
2	53274	1	Label—patent
3	53235	1	Lock—multi-point mount
4	53225	1	Knob

NOTES:

Remote Transformer—P969G01, 2, 3, 4 (Flatwall Headwall System—P670E)

Figure 1-21. Remote Transformer—P969G01, 2, 3, 4 (Flatwall Headwall System—P670E)

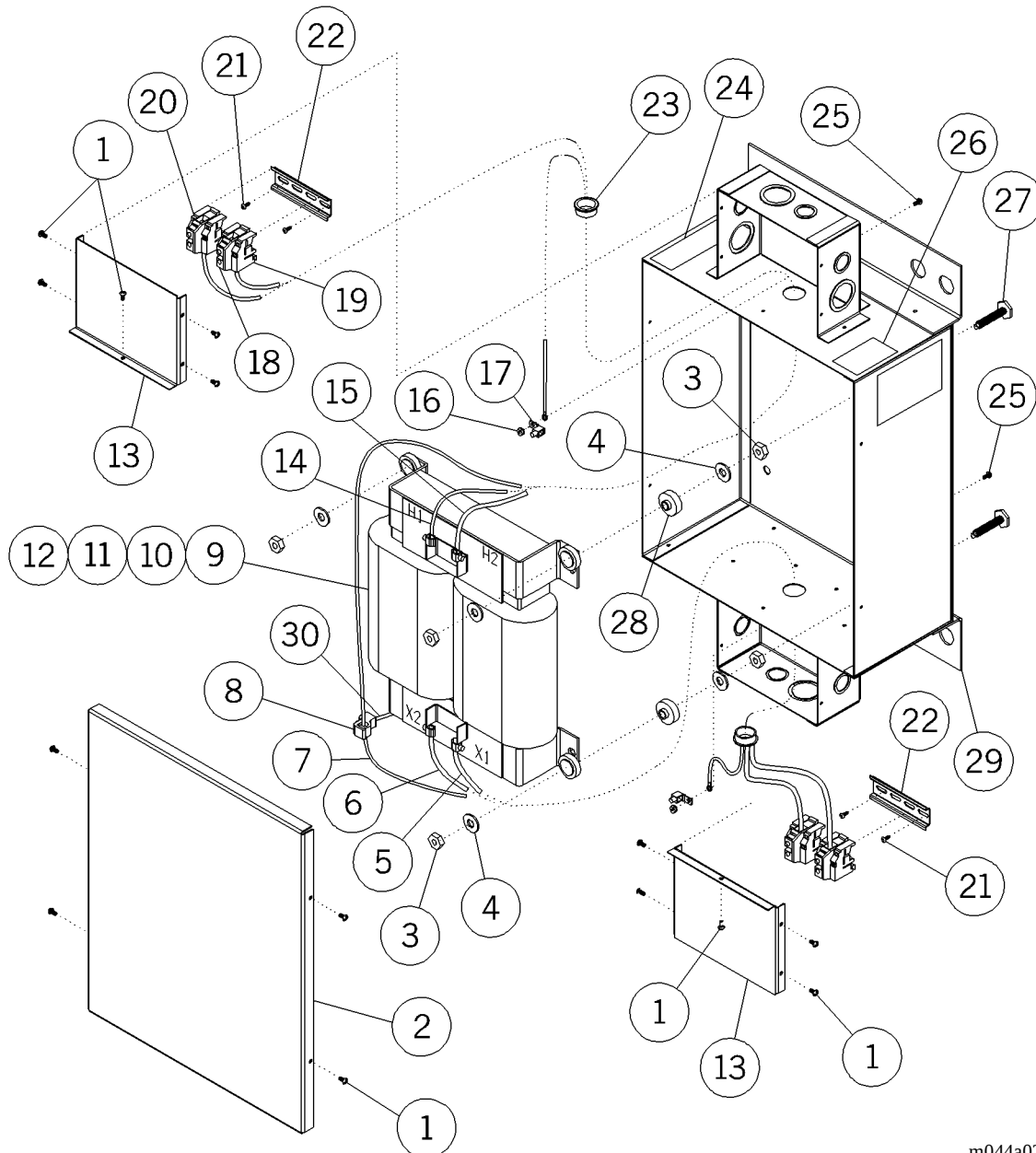


m044a021

Table 1-21. Remote Transformer—P969G01, 2, 3, 4 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	5556501	14	Screw, sems
2	2862539	1	Lid, transformer box
3	3822	8	Nut
4	5306504	8	Washer—plain
5	56344	1	Wire lead—X1
6	56345	1	Wire lead—X2
7	56351	1	Wire lead—green
8	56275	1	T-tap connector
9	29398-1	1	3 kva transformer
10	29398-2	1	Transformer—3 kva—208V/120V
11	29398-3	1	Transformer—3 kva—240V/120V
12	29398-4	1	Transformer—3 kva—277V/120V
13	5646639	1	Cover, remote transformer
14	56346	1	Wire lead—H1
15	56348	1	Wire lead—H2
16	15250	2	Locknut
17	14383	2	Ground lug
18	56548	4	Terminal block, terminal strip
19	56550	4	End section, terminal strip
20	56549	4	End stop, terminal block
21	52945-02	4	Screw, truss head
22	56649	2	Mounting rail, terminal strip
23	31047	2	Snap bushing
24	32252	2	Label, torque
25	52243-05	2	Screw
26	31890	2	Wire type label
27	52872	4	Leveling screw
28	50730	4	Shock pad
29	5653439	1	Transformer box assembly
30	56350	1	Wire lead—green 2"

Remote Transformer—P969H01, 2, 3, 4 (Flatwall Headwall System—P670E)

Figure 1-22. Remote Transformer—P969H01, 2, 3, 4 (Flatwall Headwall System—P670E)

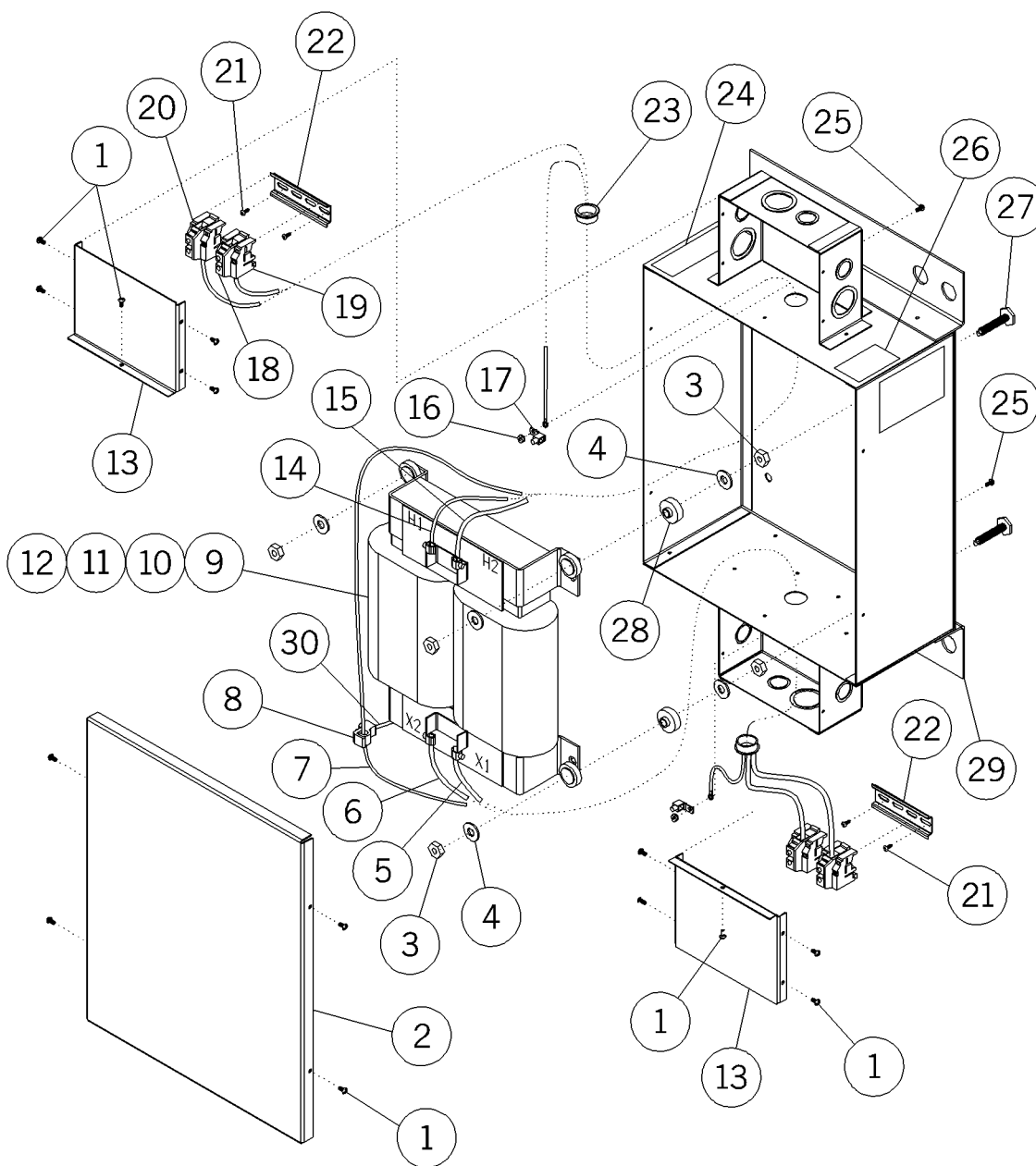


m044a022

Table 1-22. Remote Transformer—P969H01, 2, 3, 4 (Flatwall Headwall System—P670E)

Item Number	Part Number	Quantity	Description
1	5556501	14	Screw, sems
2	2862539	1	Lid, transformer box
3	3822	8	Nut
4	5306504	8	Washer—plain
5	56344	1	Wire lead—X1
6	56345	1	Wire lead—X2
7	56351	1	Wire lead—green
8	56275	1	T-tap connector
9	29397-1	1	5 kva transformer
10	29397-2	1	Transformer
11	29397-3	1	Transformer—5 kva—240V/120V
12	29397-4	1	Transformer—5 kva—277V/120V
13	5646639	1	Cover, remote transformer
14	56346	1	Wire lead—H1
15	56348	1	Wire lead—H2
16	15250	2	Locknut washer base
17	14383	2	Ground lug
18	56548	4	Terminal block, terminal strip
19	56550	4	End section, terminal strip
20	56549	4	End stop, terminal block
21	52945-02	4	Screw, truss head
22	56649	2	Mounting rail, terminal strip
23	31047	2	Snap bushing
24	32252	2	Label, torque
25	52243-05	2	Screw
26	31890	2	Wire type label
27	52872	4	Leveling screw
28	50730	4	Shock pad
29	5653439	1	Transformer box assembly
30	56350	1	Wire lead—green 2"

Bird Blender Mount—P973-B1 (Power Column—P950E) and P1020NB1 (Neonatal Horizon Headwall System—1001AC)


Figure 1-23. Bird Blender Mount—P973-B1 (Power Column—P950E) and P1020NB1 (Neonatal Horizon Headwall System—P1001AC)

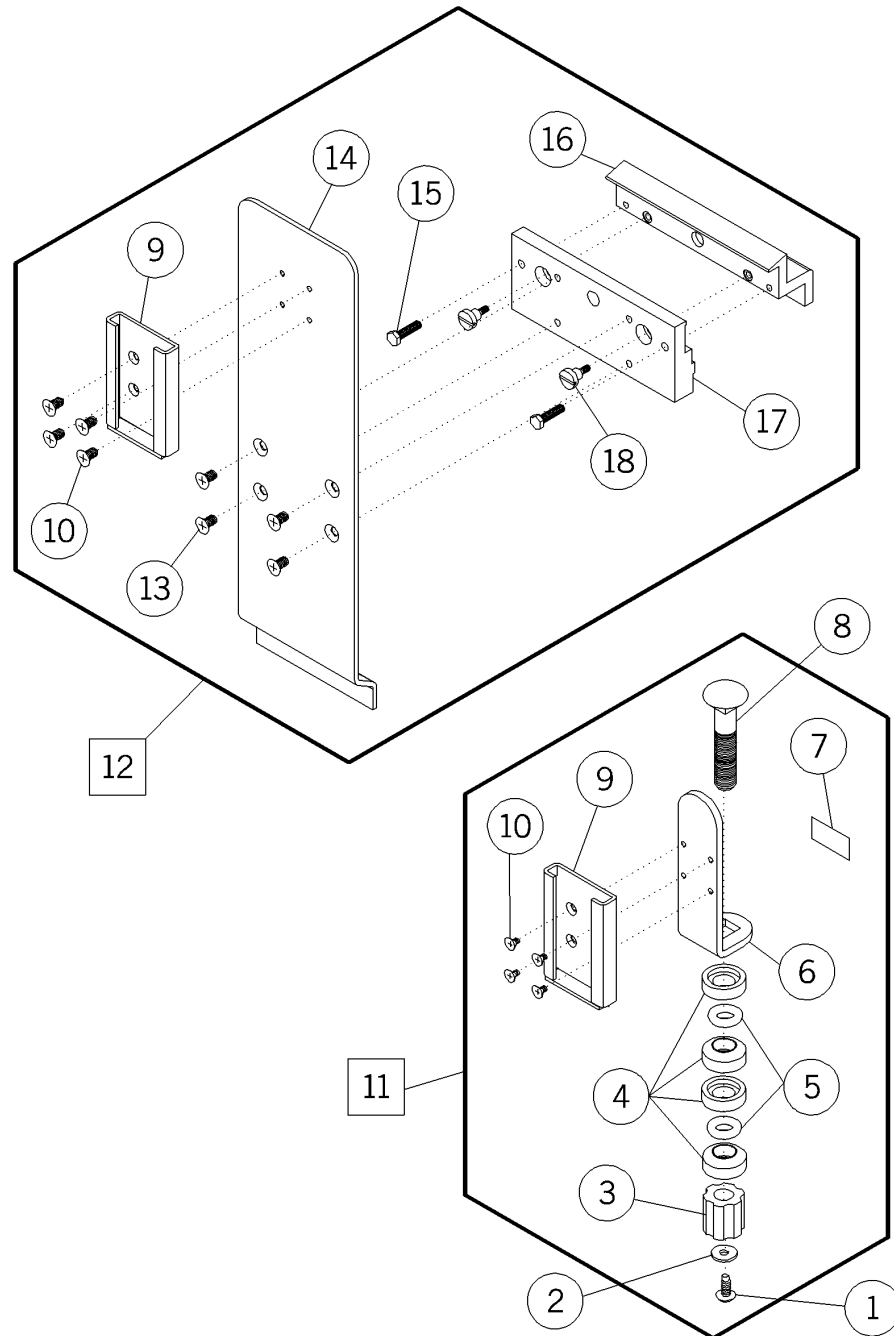

m044a025

Table 1-23. Bird Blender Mount—P973-B1 (Power Column—P950E) and P1020NB1 (Neonatal Horizon Headwall System—P1001AC)


Item Number	Part Number	Quantity	Description
1	53397-03	1	Screw
2	52386	1	Washer
3	34218	1	Compression nut
4	56530	4	Washer, beveled nylon
5	50126	2	O-ring
6	5598939	1	Bracket, slide type
7	51211	1	Label
8	33704	1	Bolt
9	56002	1	Dovetail bracket
10	5487001	4	Short screw, flathead
11	P973-B1	1	Bird blender mount
12	P1020NB1	1	Bird blender mount
13	5254301	4	Screw
14	5600439	1	Bird blender bracket
15	52191	2	Screw
16	51914	1	Trough clamp
17	51786	1	Face clamp
18	34002	2	Screw

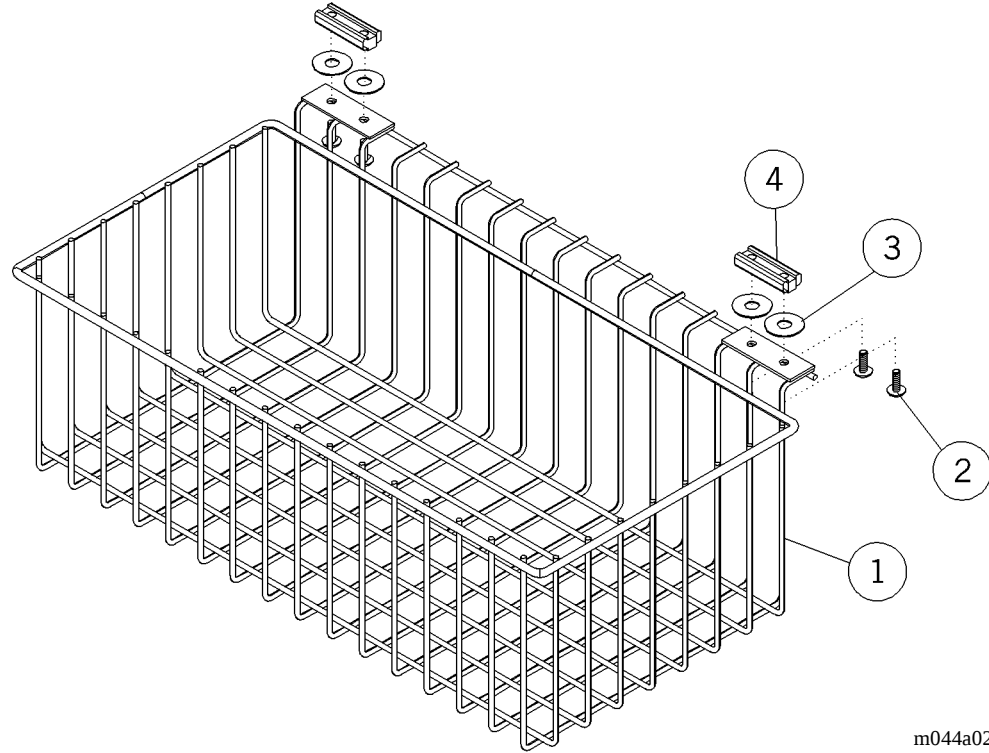Large Storage Basket—P105101 (Console—P1002AC)**Figure 1-24. Large Storage Basket—P105101 (Console—P1002AC)**

m044a026

Table 1-24. Large Storage Basket—P105101 (Console—P1002AC)

Item Number	Part Number	Quantity	Description
1	57831	1	Basket, large storage
2	15338	4	Screw
3	56841	4	Shim washer
4	34026	2	Slide

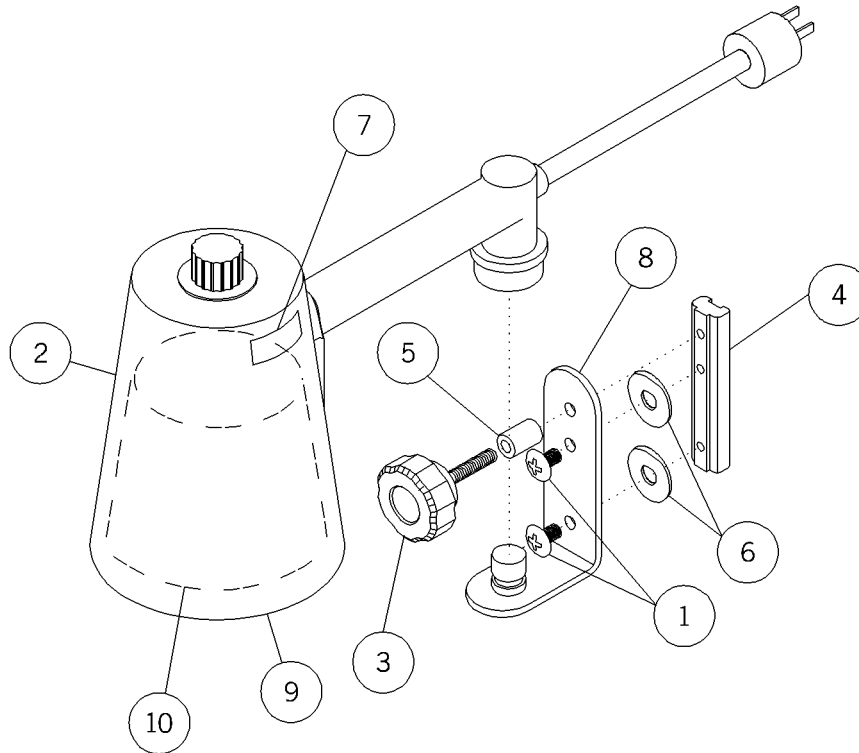Utility Light—Track Mounted—P966A00 (Power Column—P950E)**Figure 1-25. Utility Light—Track Mounted—P966A00 (Power Column—P950E)**


m044a027

Table 1-25. Utility Light—Track Mounted—P966A00 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	51637	2	Screw
2	51571	1	Lamp—966 track mount
3	33167	1	Knob
4	32332	1	Slide, track light
5	29826	1	Spacer
6	56841	2	Shim washer
7	51862	1	Label—identity
8	SA8794	1	Mounting bracket
9	SA8795	1	Guard
10	SA8796	1	Reflector

Sphygmomanometer Bracket, Aneroid—P975-01 (Flatwall Headwall System—P670EAC, Power Column—P950E)


Figure 1-26. Sphygmomanometer Bracket, Aneroid—P975-01 (Flatwall Headwall System—P670EAC, Power Column—P950E)

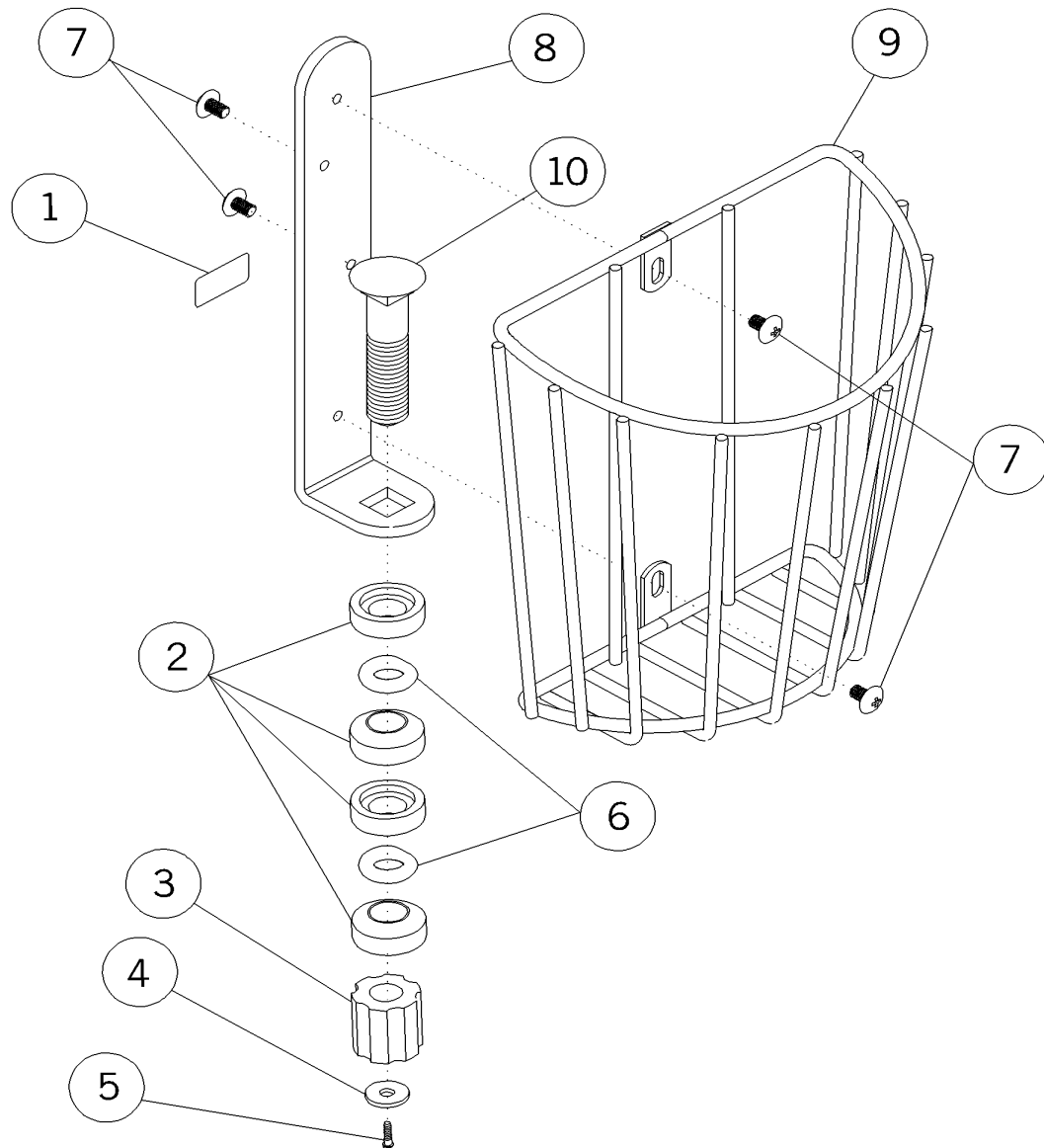

m044a028

Table 1-26. Sphygmomanometer Bracket, Aneroid—P975-01 (Flatwall Headwall System—P670EAC, Power Column—P950E)


Item Number	Part Number	Quantity	Description
1	51211	1	Label
2	56530	4	Washer, beveled nylon
3	34218	1	Compression nut
4	52386	1	Washer
5	53397-03	1	Screw
6	50126	2	O-ring
7	54610-01	4	Screw
8	5700939	1	Sphygmomanometer support bracket
9	56875	1	Cuff basket
10	33704	1	Bolt

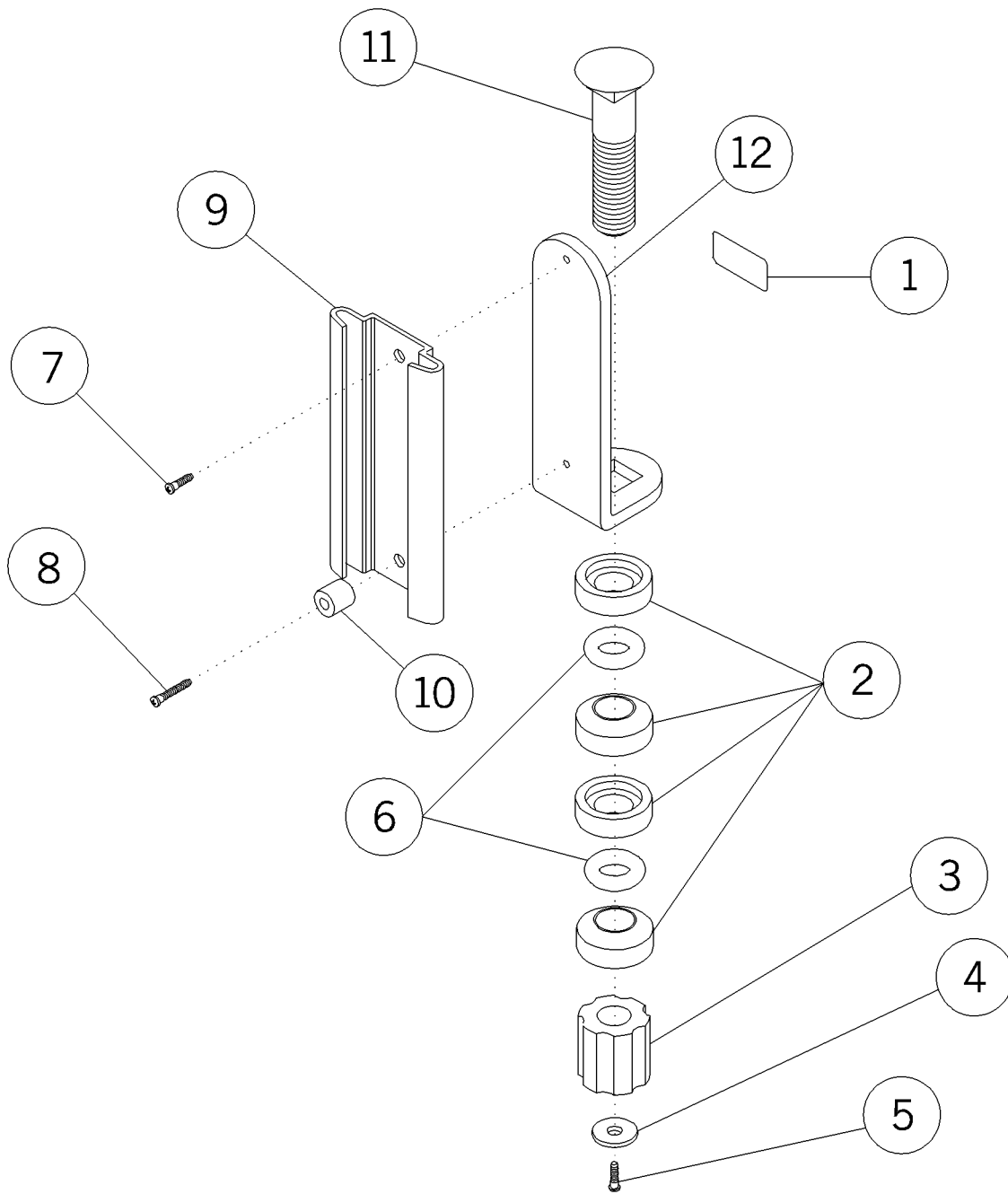Utility/Bottle Slide Mount—P973-01 (Flatwall Headwall System—P670E, Power Column—P950E)**Figure 1-27. Utility/Bottle Slide Mount—P973-01 (Flatwall Headwall System—P670E, Power Column—P950E)**

m044a029

Table 1-27. Utility/Bottle Slide Mount—P973-01 (Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53274	1	Label—patent
2	56530	4	Washer, beveled nylon
3	34218	1	Compression nut
4	52386	1	Washer
5	53397-03	1	Screw
6	50126	2	O-ring
7	50891-03	1	Screw
8	50891-07	1	Screw
9	50561	1	Bracket
10	50560	1	Spacer—nylon
11	33704	1	Bolt
12	33491	1	Bracket—utility mount

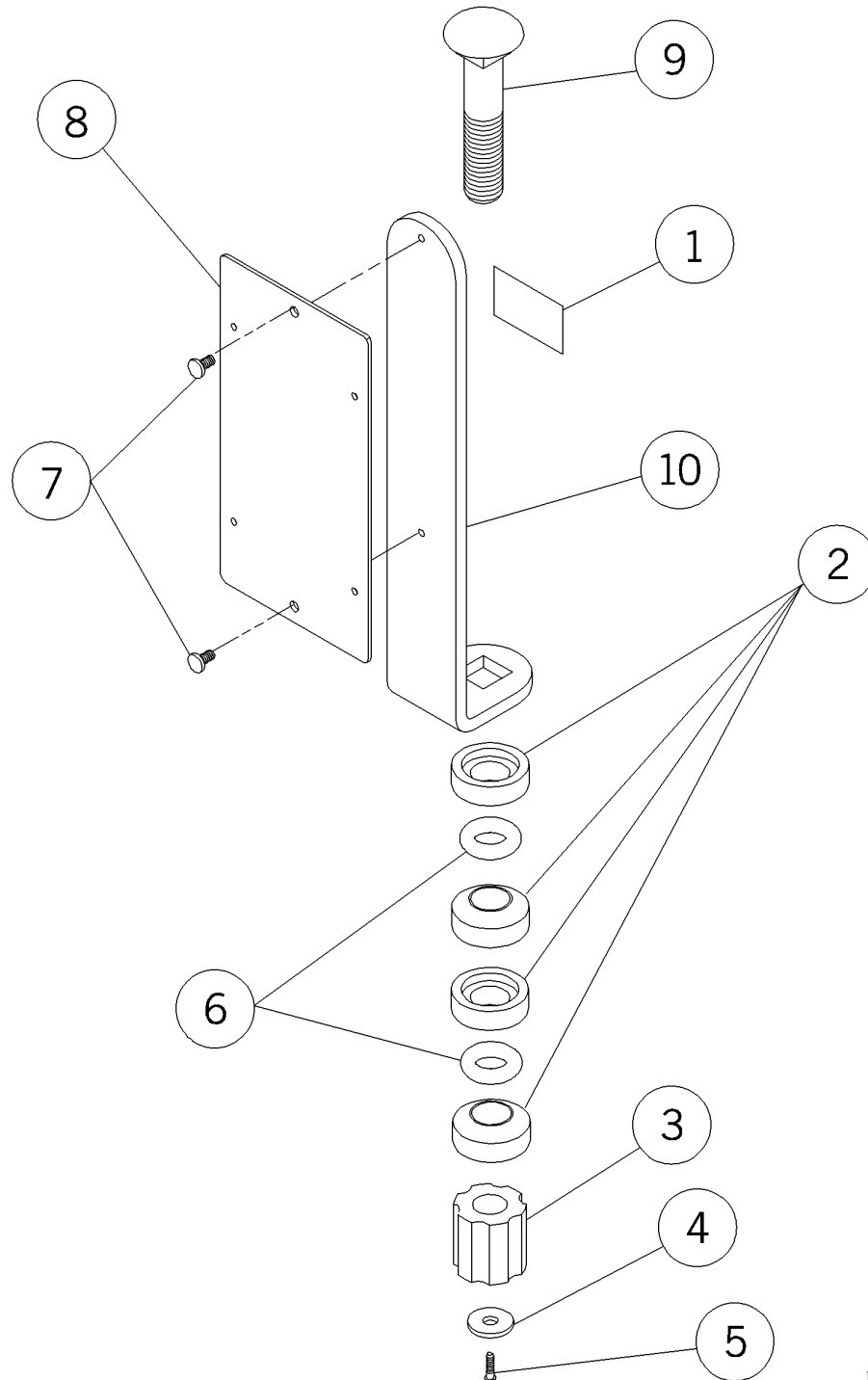Ophthalmoscope Mount—P973-S1 (Power Column—P950E)**Figure 1-28. Ophthalmoscope Mount—P973-S1 (Power Column—P950E)**


m044a030

Table 1-28. Opthalmoscope Mount—P973-S1 (Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53274	1	Label—patent
2	56530	4	Washer, beveled nylon
3	34218	1	Compression nut
4	52386	1	Washer
5	53397-03	1	Screw
6	50126	2	O-ring
7	50891-02	2	Screw
8	5599039	1	Sub plate
9	33704	1	Bolt
10	5599139	1	Mounting bracket

Utility Mount, Short Post—P972-01 and Long Post—P972L01 (Flatwall Headwall System—P670E, Power Column—P950E)

Figure 1-29. Utility Mount, Short Post—P972-01 and Long Post—P972L01 (Flatwall Headwall System—P670E, Power Column—P950E)

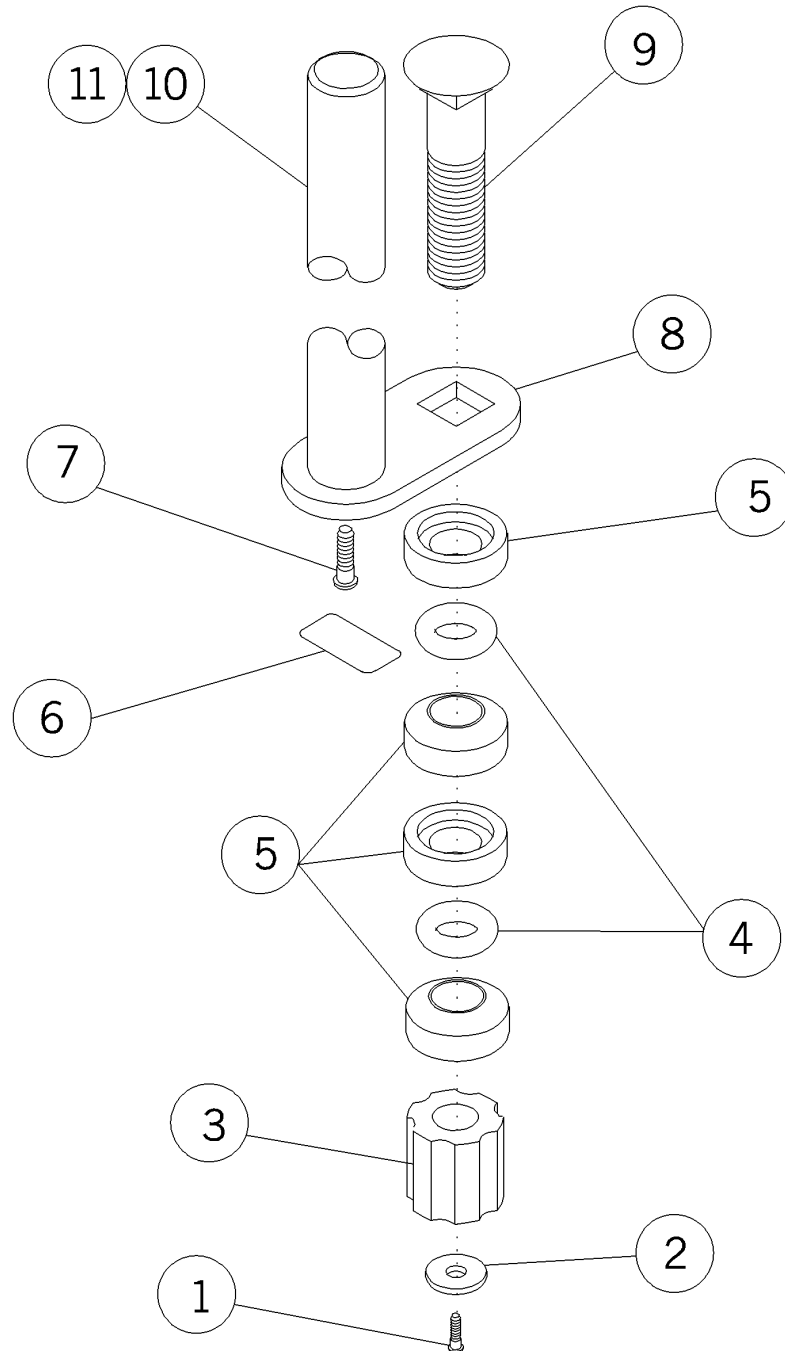

m044a031

**Table 1-29. Utility Mount, Short Post—P972-01 and Long Post—P972L01
(Flatwall Headwall System—P670E, Power Column—P950E)**

Item Number	Part Number	Quantity	Description
1	53397-03	1	Screw
2	52386	1	Washer
3	34218	1	Compression nut
4	50126	2	O-ring
5	56530	4	Washer, beveled nylon
6	53274	1	Label—patent
7	4921	1	Screw, truss head
8	33664	1	Transducer plate
9	33704	1	Bolt
10	33665	1	Transducer holder
11	53270	1	Infusion pump holder

IV Hanger—Slide Mounted—P971H00 (Horizon Headwall System—P1000)

Figure 1-30. IV Hanger—Slide Mounted—P971H00 (Horizon Headwall System—P1000)

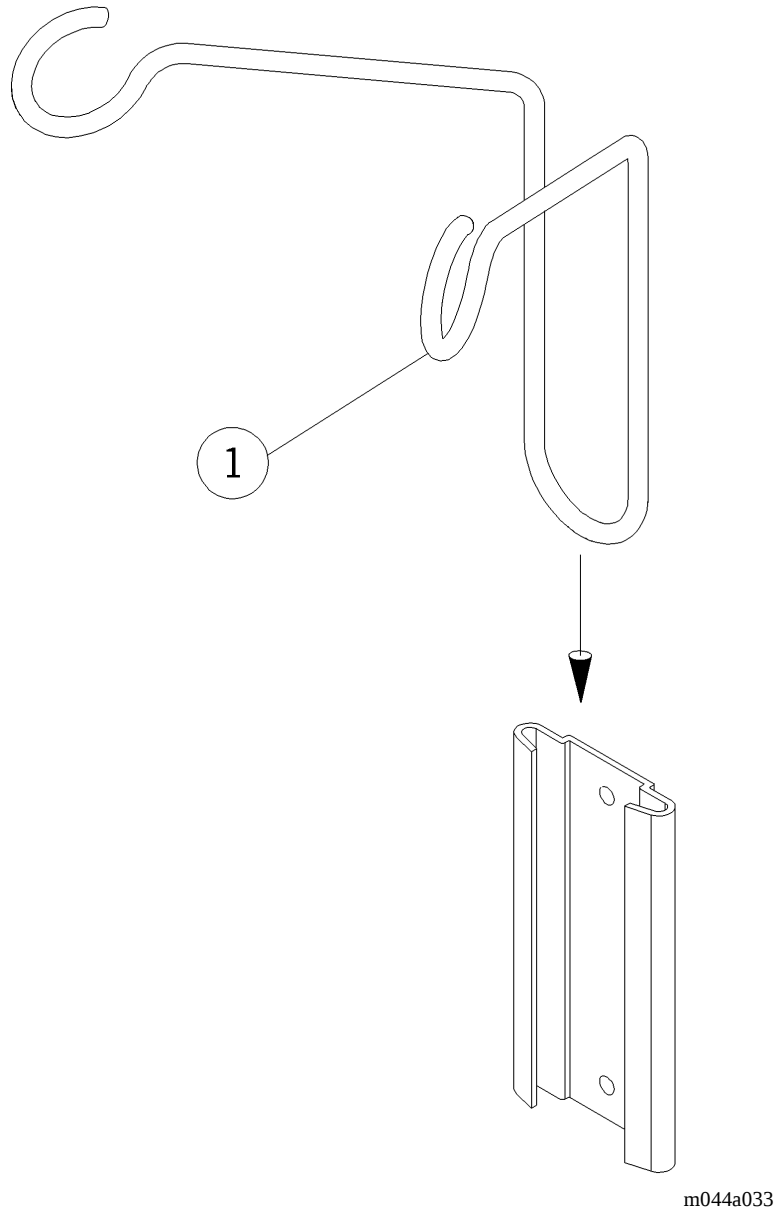



Table 1-30. IV Hanger—Slide Mounted—P971H00 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	53128	1	IV mount

NOTES:

Utility Mount—Post Type—Dual—P972D01 (Neonatal Horizon Headwall System)


Figure 1-31. Utility Mount—Post Type—Dual—P972D01 (Neonatal Horizon Headwall System)

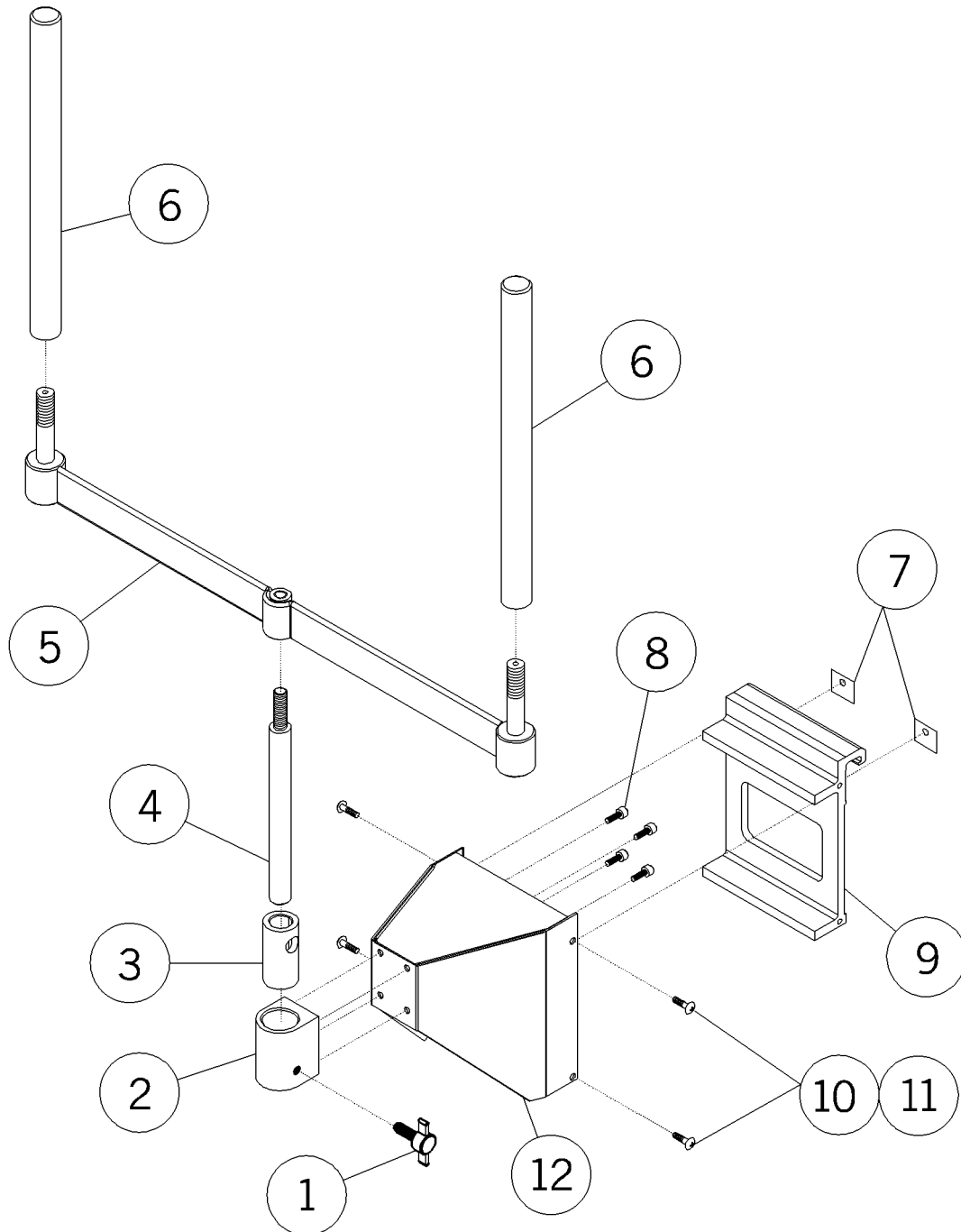

m044a034

Table 1-31. Utility Mount—Post Type—Dual—P972D01 (Neonatal Horizon Headwall System)

Item Number	Part Number	Quantity	Description
1	53913	1	Knob assembly
2	55969	1	IV block
3	55949	1	Adapter
4	53549	1	Pivot post
5	57880pl	1	Post mount assembly
6	53550	2	Tube mount—aluminum
7	52793	2	Skid pad
8	51012	2	Screw
9	53101	1	Mount bracket
10	52605-01	4	Screw
11	54610-01	4	Screw
12	5309839	1	Bracket assembly

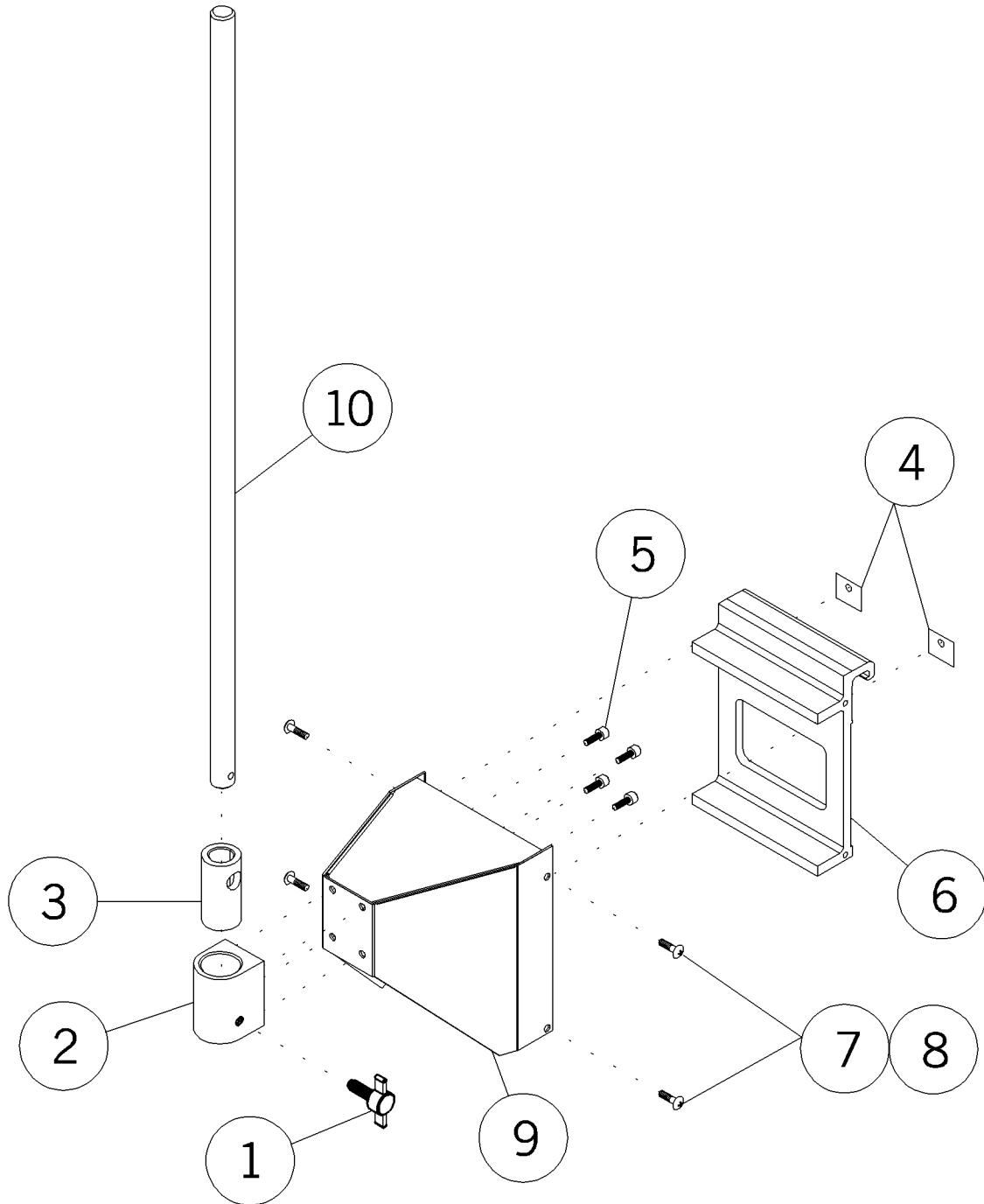Utility Mount—Post Type—P972N00, 01 (Neonatal Horizon Headwall System)**Figure 1-32. Utility Mount—Post Type—P972N00, 01 (Neonatal Horizon Headwall System)**

m044a035

Table 1-32. Utility Mount—Post Type—P972N00, 01 (Neonatal Horizon Headwall System)

Item Number	Part Number	Quantity	Description
1	53913	1	Knob assembly
2	55969	1	IV block
3	55949	1	Adapter
4	52793	2	Skid pad
5	51012	2	Screw
6	53101	1	Mount bracket
7	52605-01	4	Screw
8	54610-01	4	Screw
9	5309839	1	Bracket assembly
10	53102	1	Utility mount—post type

Sphygmomanometer Mount, Mercurial—P974-01 (Flatwall Headwall System—P670E, Power Column—P950E)

Figure 1-33. Sphygmomanometer Mount, Mercurial—P974-01 (Flatwall Headwall System—P670E, Power Column—P950E)


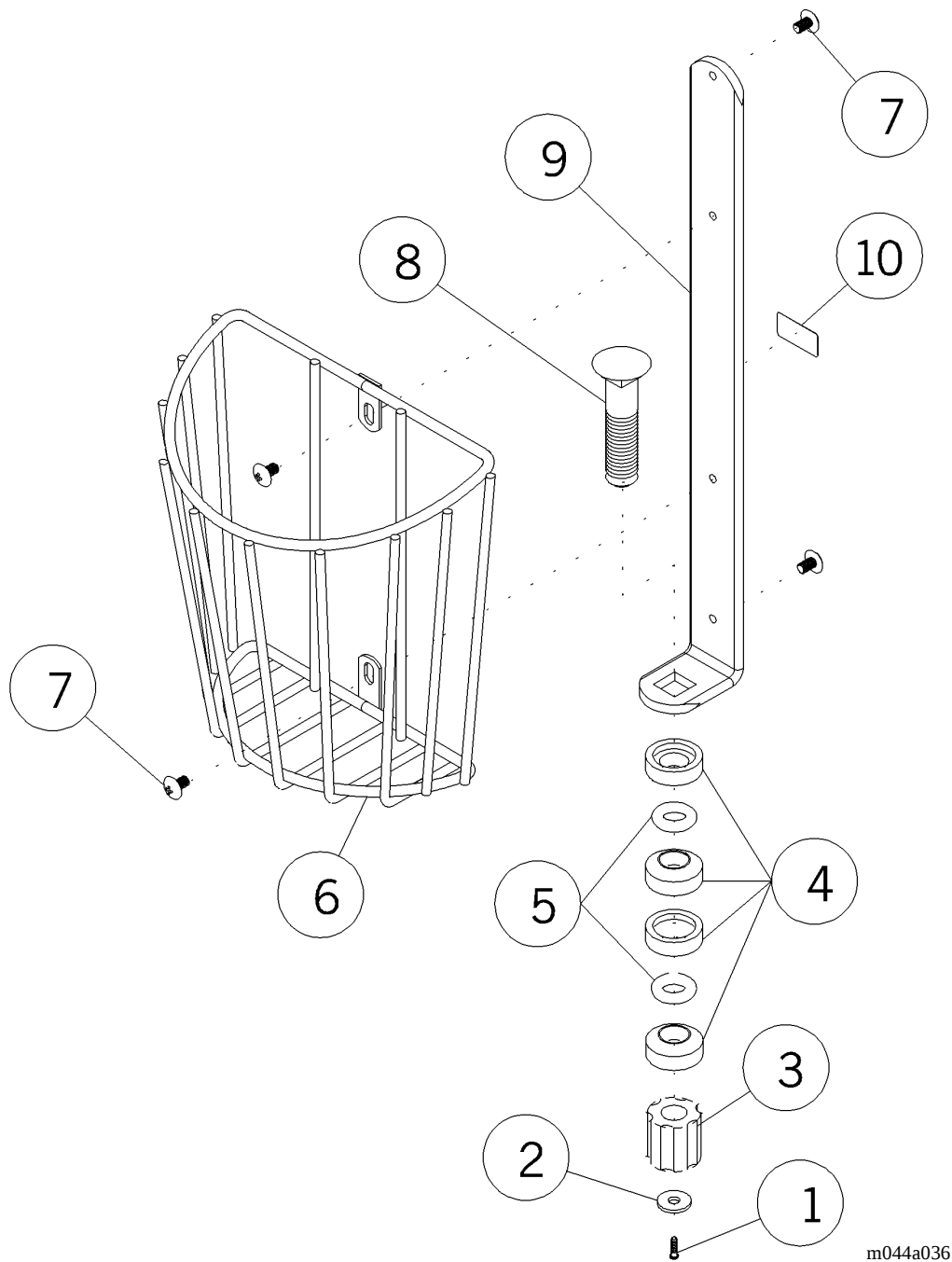


Table 1-33. Sphygmomanometer Mount, Mercurial—P974-01 (Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53397-03	1	Screw
2	52386	1	Washer
3	34218	1	Compression nut
4	56530	4	Washer, beveled nylon
5	50126	2	O-ring
6	56875	1	Cuff basket
7	54610-01	4	Screw
8	33704	1	Bolt
9	57008-39	1	Sphygmomanometer support bracket
10	53274	1	Label—patent

Mercury Sphygmomanometer—P977-00 and Aneroid Sphygmomanometer—P978-01 (Flatwall Headwall System—P670E, Power Column—P950E, Horizon Headwall System—P1000)

Figure 1-34. Mercury Sphygmomanometer—P977-00 and Aneroid Sphygmomanometer—P978-01 (Flatwall Headwall System—P670E, Power Column—P950E, Horizon Headwall System—P1000)

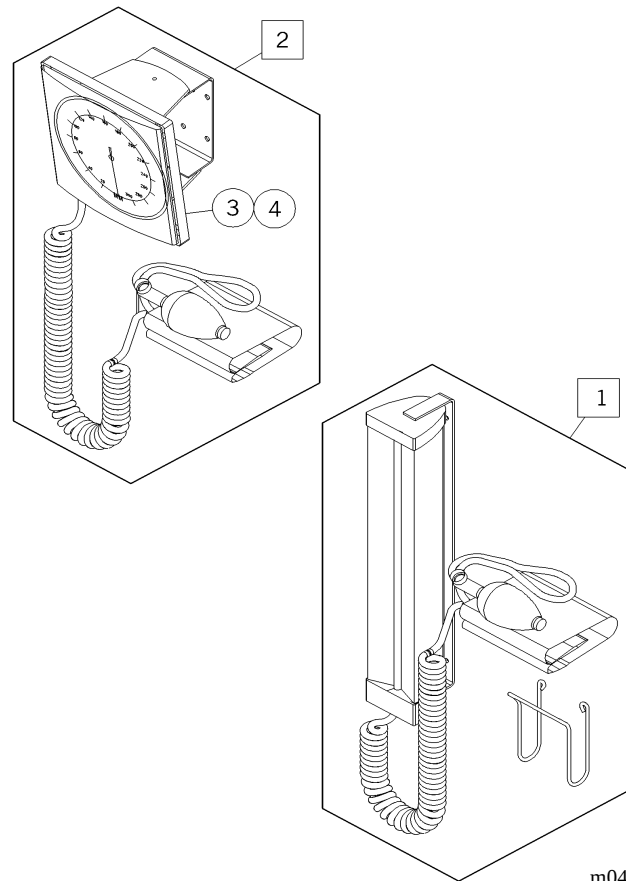


m044a037

Table 1-34. Mercury Sphygmomanometer—P977-00 and Aneroid Sphygmomanometer—P978-01 (Flatwall Headwall System—P670E, Power Column—P950E, Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	977-00	1	Mercury sphygmomanometer
2	P97801	1	Aneroid sphygmomanometer
3	28586	1	Aneroid sphygmomanometer
4	28586-02	1	Aneroid sphygmomanometer—charcoal face

Wall Mounted Accessory Track—P979-00

Figure 1-35. Wall Mounted Accessory Track—P979-00

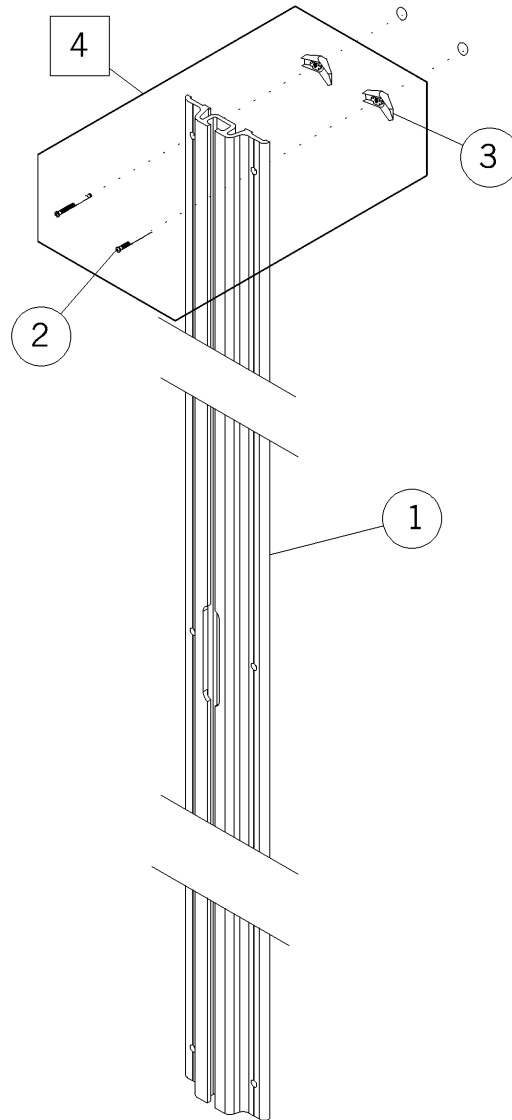


m044a038

Table 1-35. Wall Mounted Accessory Track—P979-00

Item Number	Part Number	Quantity	Description
1	50078	1	Wall extrusion—slotted
2	50167	6	Screw
3	15384	6	Toggle wing
4	33991	1	Bag assembly

Infusion Pump/IV Holder—P984-00 (Horizon Headwall System—P1000, Flatwall Headwall System—P670E, Power Column—P950E)

Figure 1-36. Infusion Pump/IV Holder—P984-00 (Horizon Headwall System—P1000, Flatwall Headwall System—P670E, Power Column—P950E)

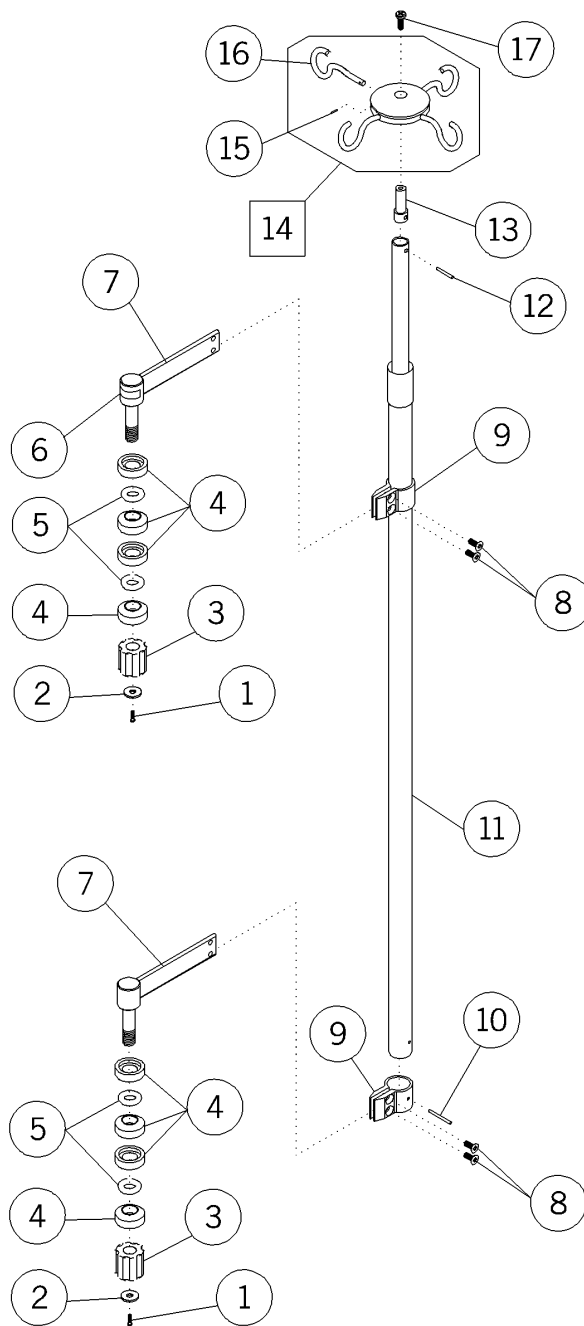


m044a039

Table 1-36. Infusion Pump/IV Holder—P984-00 (Horizon Headwall System—P1000, Flatwall Headwall System—P670E, Power Column—P950E)

Item Number	Part Number	Quantity	Description
1	53397-03	2	Screw
2	52396	2	Washer
3	34218	2	Compression nut
4	56530	8	Washer, beveled nylon
5	50126	4	O-ring
6	53274	1	Label—patent
7	57710pl	2	Support assembly
8	52363	4	Screw
9	52308	2	Clamp, I.V. pole
10	52316	1	Roll pin
11	52306	1	Hanger rod assembly
12	55736	1	Roll pin
13	52300	1	Hub insert
14	52298	1	Hub assembly
15	35018	4	Roll pin
16	35003	4	I.V. hook
17	53063-02	1	Screw

Figure 1-37. Swivel Shelf, Stabilet Freestanding Infant Warmer—P986S01 (Power Column—P950E)

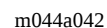


**Table 1-37. Swivel Shelf, Stabilet Freestanding Infant Warmer—P986S01
(Power Column—P950E)**

Item Number	Part Number	Quantity	Description
1	53406	1	Insert cap glide
2	50853-01	4	Hex head bolt
3	5450739	1	Arm assembly, shelf
4	54607	1	Label, load
5	51637	4	Screw
6	34001	2	Slide insert
7	56841	4	Shim washer
8	54026	1	Equipment holder
9	52396	4	Screw
10	33653	4	Nut
11	51695	1	Roll pin
12	54595	1	Pivot
13	54596pl	1	Bar, support
14	53913	1	Knob assembly
15	54597	1	Equipment holder—tapped
16	54497*	1	Top swivel shelf
17	53242	4	Locknut
18	54899	1	Swivel shelf, Stabilet
19	50854-03	4	Pan head screw
20	55579	1	Cover

* Specify high pressure laminate color.

NurseMate—Horizon Headwall System Mount—P101105 (Horizon Headwall System—P1000)

Figure 1-38. NurseMate—Horizon Headwall System Mount—P101105 (Horizon Headwall System—P1000)

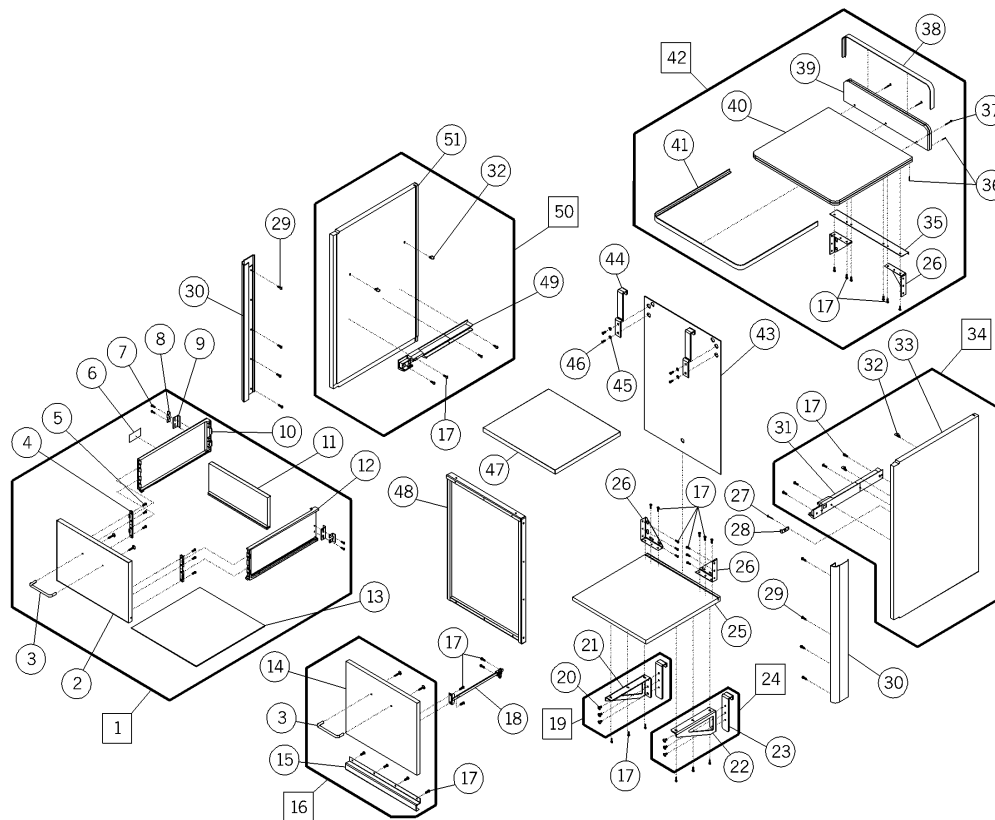

m044a048

**Table 1-38. NurseMate—Horizon Headwall System Mount—P101105
(Horizon Headwall System—P1000)**

Item Number	Part Number	Quantity	Description
1	56411*	1	Drawer assembly
2	56379*	1	Drawer front, top
3	56244	2	Pull with screws
4	53293	2	Mounting clip
5	50415	6	Screw
6	51158	1	Label—drawer removal
7	53397-03	4	Screw
8	51060	2	Latch—anti-withdrawal
9	50161	2	Latch plate
10	53288	1	Side—left-hand modified
11	53291	1	Back
12	53287	1	Side—right-hand modified
13	50412	1	Drawer bottom
14	56380*	1	Drop door
15	51728	1	Hinge
16	56412	1	Drop door assembly
17	50854-03	31	Pan head screw
18	51778	1	Drop door support
19	54832-02	1	Bottom support—left-hand
20	54610-01	6	Screw
21	52482-02	1	Shelf mount—left-hand
22	52482-01	1	Shelf mount—right-hand
23	52563	2	Shelf hanger
24	54832-01	1	Bottom support—right-hand
25	51751*	1	Bottom panel
26	5044939	4	Corner bracket—light neutral
27	15354	1	Pin
28	55692	1	Door stop
29	10595	8	Screw
30	54619	2	Trim

* Specify high pressure laminate color.


Item Number	Part Number	Quantity	Description
31	50385	1	Drawer runner, right-hand
32	15031	4	Pin walnut support
33	51755*	1	End panel—right-hand
34	51764*	1	End panel assembly—right-hand
35	50456	1	Claw
36	27246	4	Power pin
37	52763	3	Screw
38	54794	1	T-moulding
39	53091*	1	Backsplash
40	51752*	1	Top
41	54524-03	1	Trim—shelf—light neutral
42	54827*	1	Top assembly
43	54829	1	Back panel
44	52673	2	Upper support
45	25283	4	Washer
46	4921	4	Screw
47	51750*	1	Shelf
48	5177139	1	Frame
49	50386	1	Drawer runner, left-hand
50	51765*	1	End panel assembly—left-hand
51	51754*	1	End panel
52	5375302	8	Screw, flat head
53	50854-03	6	Pan head screw

* Specify high pressure laminate color.

NOTES:

NurseMate—Horizon Headwall System Mount Three Drawer (P101135) and Wall Mount Three Drawer (P101136)

Figure 1-39. NurseMate—Horizon Headwall System Mount Three Drawer (P101135) and Wall Mount Three Drawer (P101136)

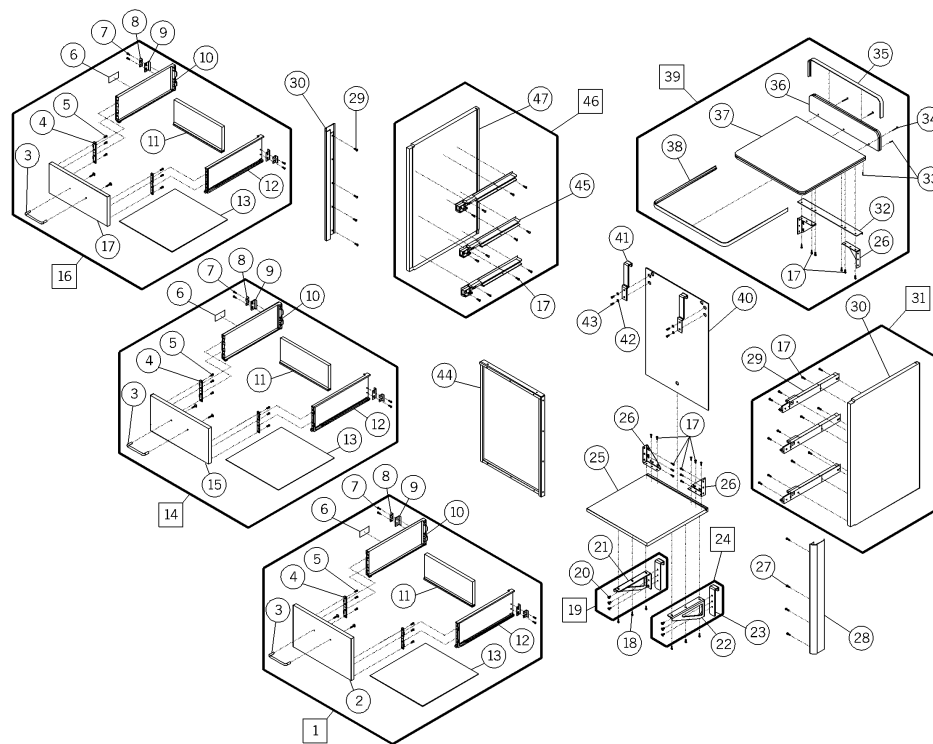

m044a049

Table 1-39. NurseMate—Horizon Mount Three Drawer (P101135) and Wall Mount Three Drawer (P101136)

Item Number	Part Number	Quantity	Description
1	56436*	1	Drawer assembly—lower
2	56425*	1	Drawer front, lower
3	56244	3	Pull with screws
4	53293	6	Mounting clip
5	50415	18	Screw
6	51158	3	Label—drawer removal
7	53397-03	12	Screw
8	51060	6	Latch—anti-withdrawal
9	50161	6	Latch plate
10	53288	3	Side—left-hand modified
11	53291	3	Back
12	53287	3	Side—right-hand modified
13	50412	3	Drawer bottom
14	56435*	1	Drawer assembly—middle
15	56424*	1	Drawer front—middle
16	56434*	1	Drawer assembly—upper
17	56423*	1	Drawer front, upper
18	50854-03	28	Pan head screw
19	54832-02	1	Bottom support—left-hand
20	54610-01	6	Screw
21	52482-02	1	Shelf mount—left-hand
22	52482-01	1	Shelf mount—right-hand
23	52563	2	Shelf hanger
24	54832-01	1	Bottom support—right-hand
25	51751*	1	Bottom panel
26	5044939	2	Corner bracket—light neutral
27	10595	8	Screw
28	54619	2	Trim
29	50385	3	Drawer runner, right-hand
30	56427*	1	End panel, right-hand

* Specify high pressure laminate color.


Item Number	Part Number	Quantity	Description
31	56432*	1	End panel assembly, right-hand
32	50456	1	Claw
33	27246	2	Power pin
34	52763	3	Screw
35	54794	1	T-moulding
36	53091*	1	Backsplash
37	51752*	1	Top
38	54524-03	1	Trim—shelf—light neutral
39	54827*	1	Top assembly
40	54829	1	Back panel
41	52673	2	Upper support
42	25283	4	Washer
43	4921	4	Screw
44	5177139	1	Frame assembly
45	50386	3	Drawer runner, left-hand
46	56433*	1	End panel assembly, left-hand
47	56426*	1	End panel, left-hand
48	5375302	8	Screw, flathead
49	3679	6	Screw
50	21045	2	Toggle bolt
51	50455	1	Hanger—NurseMate/wall cabinet
52	21046	2	Toggle wing
53	29325	2	Stud guide
54	53065-01	2	Washer
55	50853-01	2	Hex head bolt
56	15384	1	Toggle wing
57	34516	1	Stiffener plate
58	50167	1	Screw

* Specify high pressure laminate color.

NOTES:

Mobile Storage Cabinet—P101202 (Neonatal Horizon Headwall System—P1001)

Figure 1-40. Mobile Storage Cabinet—P101202 (Neonatal Horizon Headwall System—P1001)

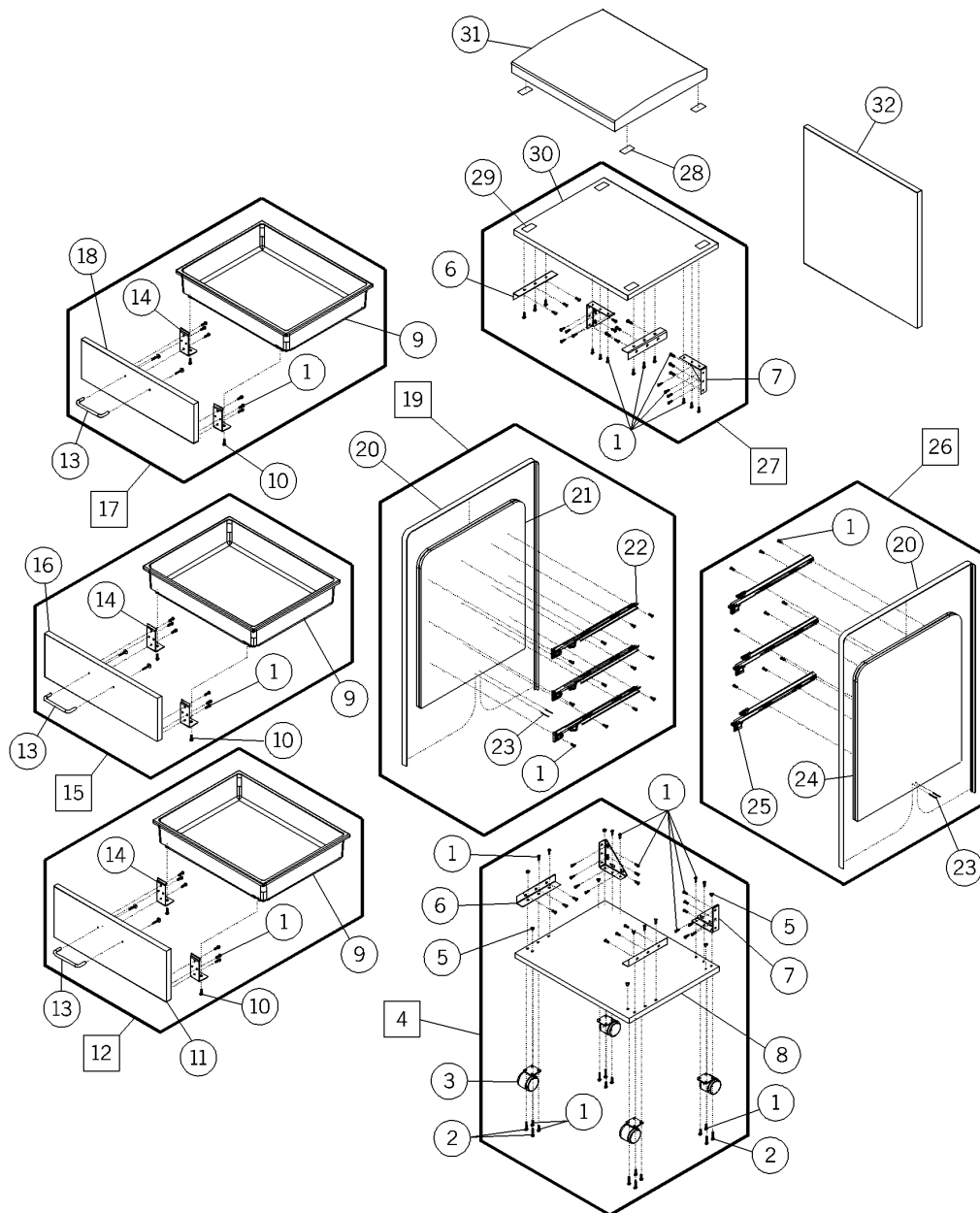

m044a052


Table 1-40. Mobile Storage Cabinet—P101202 (Neonatal Horizon Headwall System—P1001)

Item Number	Part Number	Quantity	Description
1	50854-03	58	Pan head screw
2	21230	8	Screw
3	53354	4	Caster
4	53359*	1	Bottom panel assembly
5	15250	8	Locknut
6	5224239	4	Angle bracket, light neutral
7	5044939	4	Corner bracket, light neutral
8	53336*	1	Bottom panel
9	13296s	3	Plastic drawer
10	23202	6	Screw
11	56455*	1	Drawer front
12	56456*	1	Drawer assembly
13	56244	3	Pull with screws
14	53347-39	6	Drawer mounting bracket
15	56454*	1	Drawer assembly
16	56453*	1	Drawer front
17	56452*	1	Drawer assembly
18	56451*	1	Drawer front
19	56449*	1	End panel assembly—left-hand
20	54794	2	T-moulding
21	56450*	1	End panel—left-hand
22	12577	3	Drawer guide left-hand
23	27246	4	Power pin
24	56448*	1	End panel—right-hand
25	12576	3	Drawer guide right-hand
26	56447*	1	End panel assembly—right-hand
27	53356*	1	Top panel assembly
28	53384	4	Hook fastener
29	53383	4	Loop arm fastener
30	53334*	1	Top panel
31	56919	1	Cushion
32	53333*	1	Back panel

* Specify high pressure laminate color.

Mobile Storage Cabinet—P101203 (Neonatal Horizon Headwall System—P1001)

Figure 1-41. Mobile Storage Cabinet—P101203 (Neonatal Horizon Headwall System—P1001)

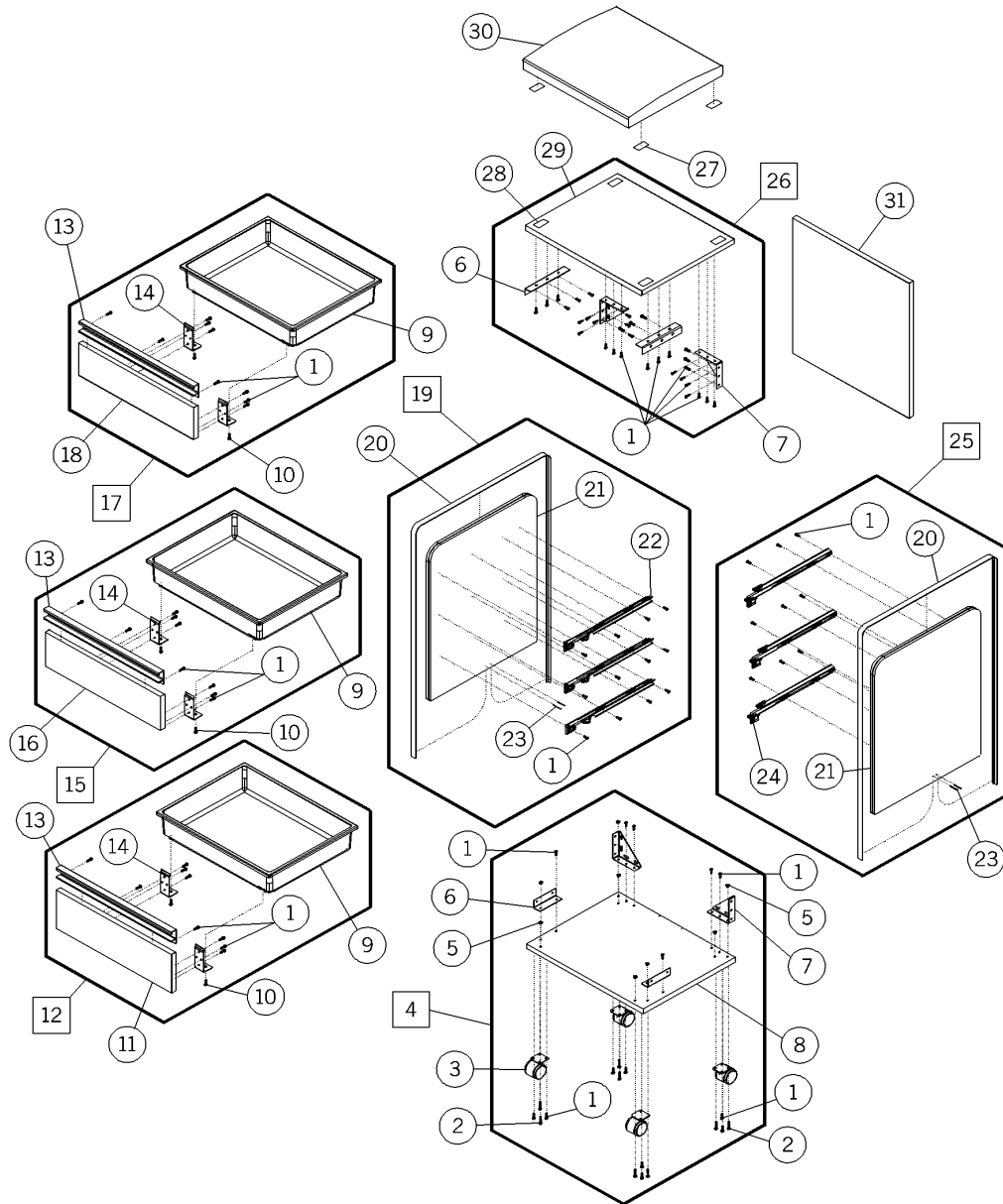


m044a053

Table 1-41. Mobile Storage Cabinet—P101203 (Neonatal Horizon Headwall System—P1001)

Item Number	Part Number	Quantity	Description
1	50854-03	73	Pan head screw
2	21230	8	Screw
3	53354	4	Caster
4	57415*	1	Bottom panel assembly
5	15250	8	Locknut
6	52223	4	Corner bracket
7	50449	4	Corner bracket
8	57411*	1	Bottom panel
9	13296s	3	Plastic drawer
10	58616	6	Screw
11	5333103*	1	Drawer front
12	57418*	1	Drawer assembly
13	57158	3	Pull, light neutral
14	53347-39	6	Drawer mounting bracket
15	57416*	1	Drawer assembly
16	5333102*	1	Drawer front
17	57414*	1	Drawer assembly
18	5333101*	1	Drawer front
19	57413*	1	End panel assembly left-hand
20	54794	2	T-moulding
21	56450*	2	End panel—left-hand
22	12577	3	Drawer guide left-hand
23	27246	4	Power pin
24	12576	3	Drawer guide right-hand
25	57412*	1	End panel assembly right-hand
26	57417*	1	Top panel assembly
27	53384	4	Hook fastener
28	53383	4	Loop arm fastener
29	53334*	1	Top panel
30	57420	1	Cushion
31	53333*	1	Back panel

* Specify high pressure laminate color.

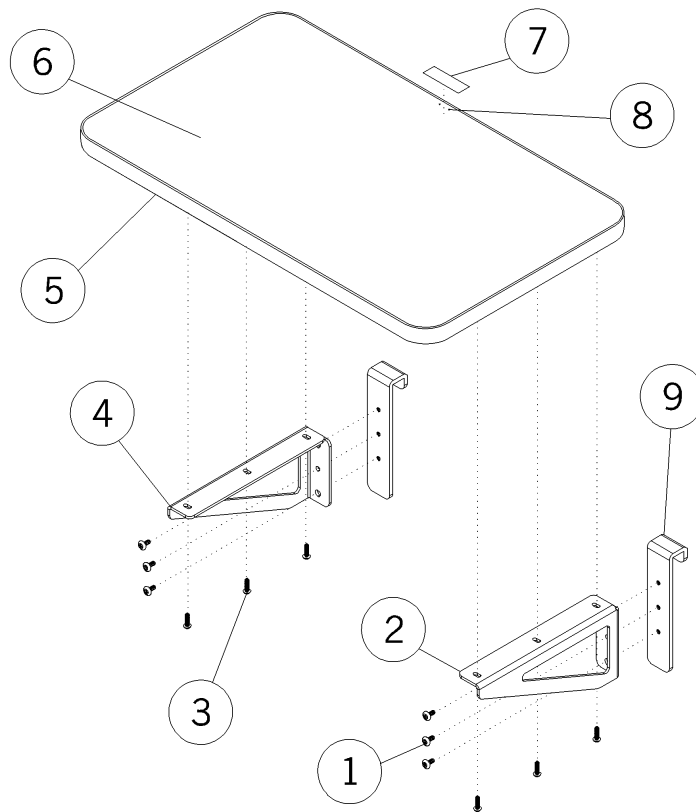Utility Shelf—20"—P101402 (Horizon Headwall System—P1000)**Figure 1-42. Utility Shelf—20"—P101402 (Horizon Headwall System—P1000)**

m044a054


Table 1-42. Utility Shelf—20"—P101402 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	54610-01	6	Screw
2	52482-01	1	Shelf mount—right-hand
3	50854-03	6	Pan head screw
4	52482-02	1	Shelf mount—left-hand
5	54524-02	1	Trim, shelf—light neutral
6	5317920*	1	Shelf, top
7	54607	1	Label, load
8	27246	2	Power pin
9	52563	2	Shelf hanger

* Specify high pressure laminate color.

Bed Bumper—P1016A2 (Horizon Headwall System—P1000)


Figure 1-43. Bed Bumper—P1016A2 (Horizon Headwall System—P1000)

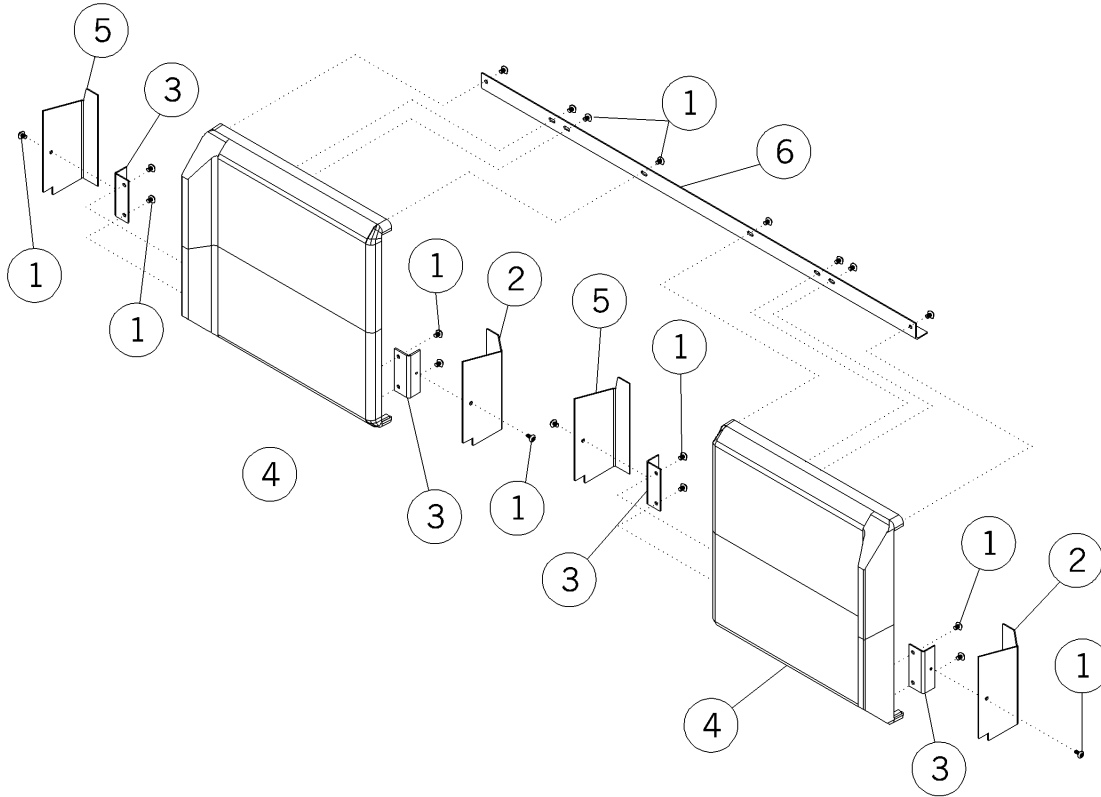

m044a055

Table 1-43. Bed Bumper—P1016A2 (Horizon Headwall System—P1000)


Item Number	Part Number	Quantity	Description
1	52555-04	16	Screw
2	54220-02	2	Locking end plate—left-hand
3	54221	4	Mounting angle—bumper
4	54624	2	Bumper
5	54220-01	2	Locking end plate—right-hand
6	53197-39	1	Bumper stretcher—top

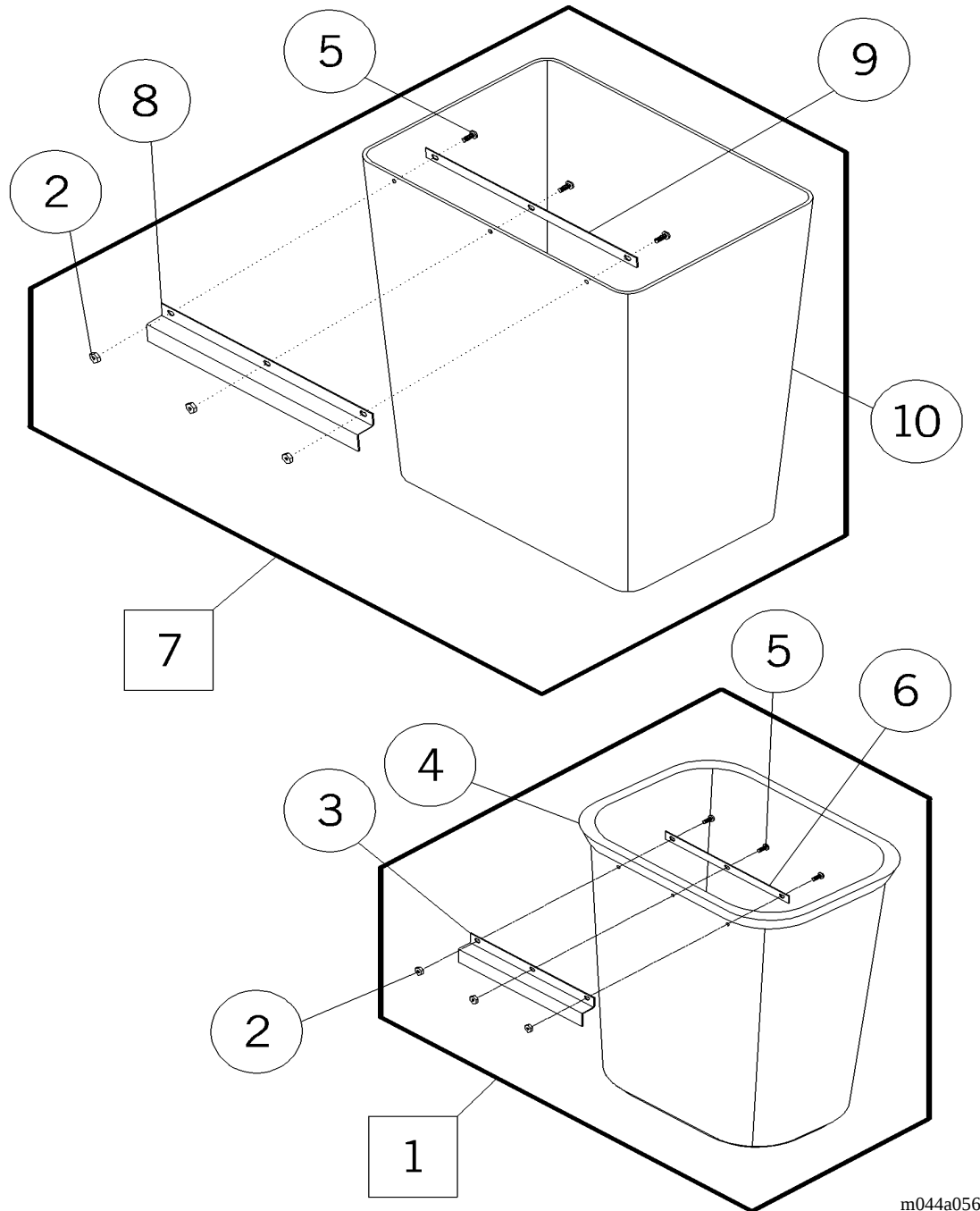Waste Receptacle Assembly—P101701, 02 (Horizon Headwall System—P1000)**Figure 1-44. Waste Receptacle Assembly—P101701, 02 (Horizon Headwall System—P1000)**

m044a056

Table 1-44. Waste Receptacle Assembly—P101701, 02 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	P101702	1	Waste receptacle assembly
2	309	3	Nut
3	57330	1	Hanger bracket
4	57325	1	Waste receptacle
5	50891-04	3	Screw
6	57329	1	Strap
7	P101701	1	Waste receptacle
8	52325	1	Hanger bracket—light neutral
9	56612-39	1	Strap
10	51909	1	Waste basket

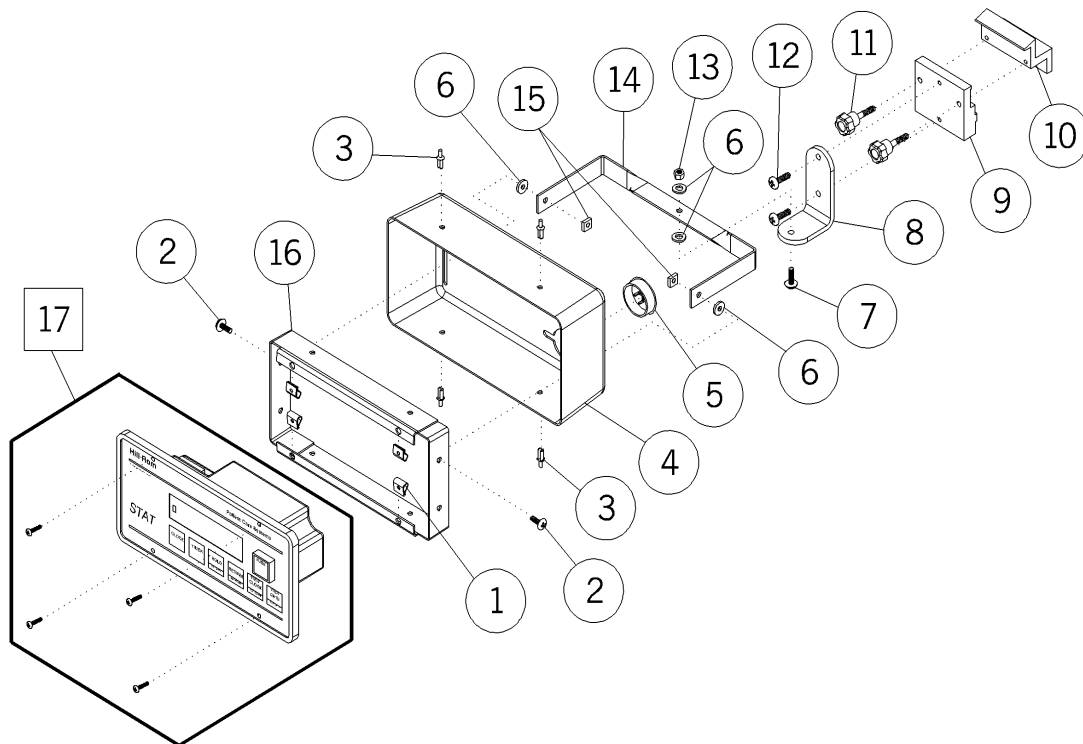Pod Mounted STAT Clock/Timer—P101811 (Horizon Headwall System—P1000)**Figure 1-45. Pod Mounted STAT Clock/Timer—P101811 (Horizon Headwall System—P1000)**


m044a057

Table 1-45. Pod Mounted STAT Clock/Timer—P101811 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	31992	4	Speed nut
2	51637	2	Screw
3	33719	4	Rivet
4	33701	1	Cover
5	33720	1	Hold plug
6	33721	4	Nylon washer
7	52396	1	Screw
8	53800	1	Arm—STAT clock
9	52571	1	Trough clamp—front
10	52572	1	Trough clamp—rear
11	52569	2	Knob
12	53258-02	2	Screw
13	33680	1	Locknut
14	33693	1	Bail
15	33653	2	Nut
16	33694	1	Pivot frame
17	967-00	1	STAT clock timer

Utility/Bottle Slide Mount—P1020A1 (Horizon Headwall System—P1000) and Bottle Slide—P1020N1 (Neonatal Horizon Headwall System—P1001AC)

Figure 1-46. Utility/Bottle Slide Mount—P1020A1 (Horizon Headwall System—P1000) and Bottle Slide—P1020N1 (Neonatal Horizon Headwall System—P1001AC)

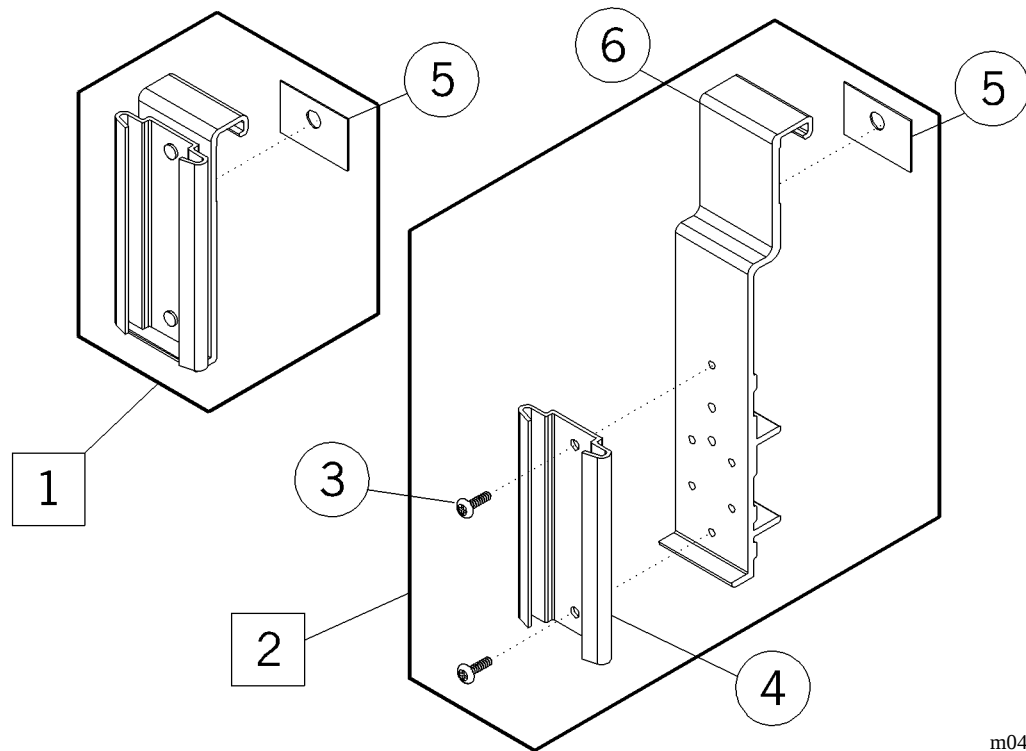


m044a058

Table 1-46. Utility/Bottle Slide Mount—P1020A1 (Horizon Headwall System—P1000) and Bottle Slide—P1020N1 (Neonatal Horizon Headwall System—P1001AC)

Item Number	Part Number	Quantity	Description
1	1020A1	1	Bottle slide
2	1020N1	1	Bottle slide
3	57018-04	2	Screw
4	50561	1	Bracket
5	52793	1	Skid pad
6	57260pl	1	Bird blender bracket

Bird Blender Mount—P1020B1 (Horizon Headwall System—P1000)

Figure 1-47. Bird Blender Mount—P1020B1 (Horizon Headwall System—P1000)

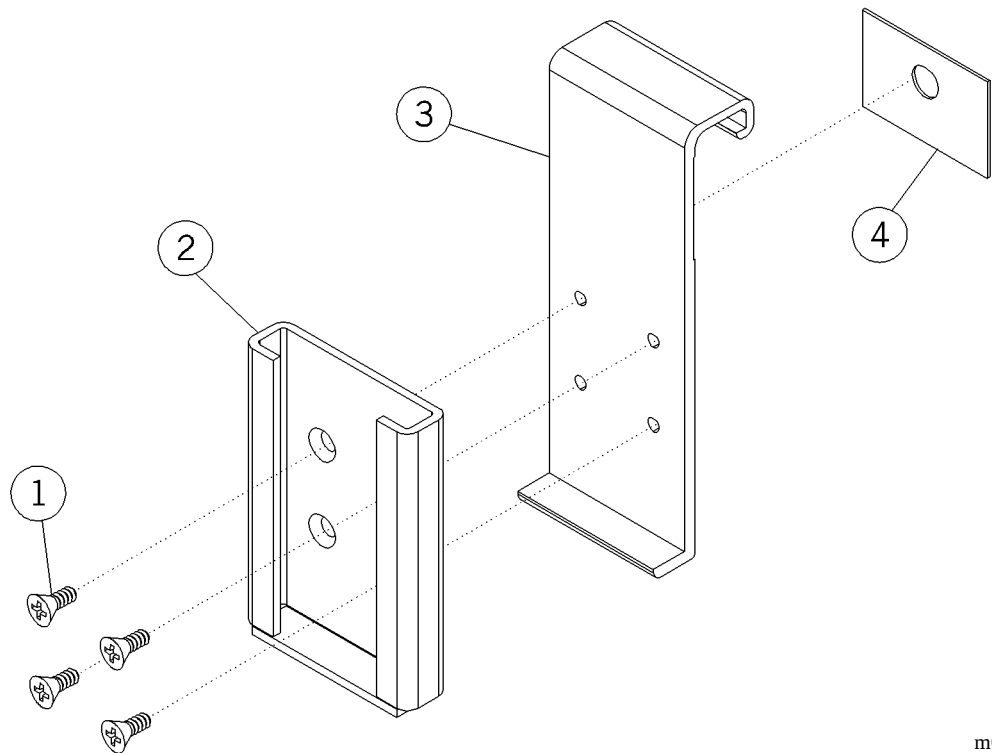


m044a059

Table 1-47. Bird Blender Mount—P1020B1 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	5487001	4	Short screw, flathead
2	56002	1	Dovetail bracket
3	55831pl	1	Hook, bird blender
4	52793	1	Skid pad

Opthalmoscope Mount—P1020S1, 2 (Horizon Headwall System—P1000) and P1020S3 (Horizon Headwall System—P1000, Console—P1002AC)

Figure 1-48. Opthalmoscope Mount—P1020S1, 2 (Horizon Headwall System—P1000) and P1020S3 (Horizon Headwall System—P1000, Console—P1002AC)

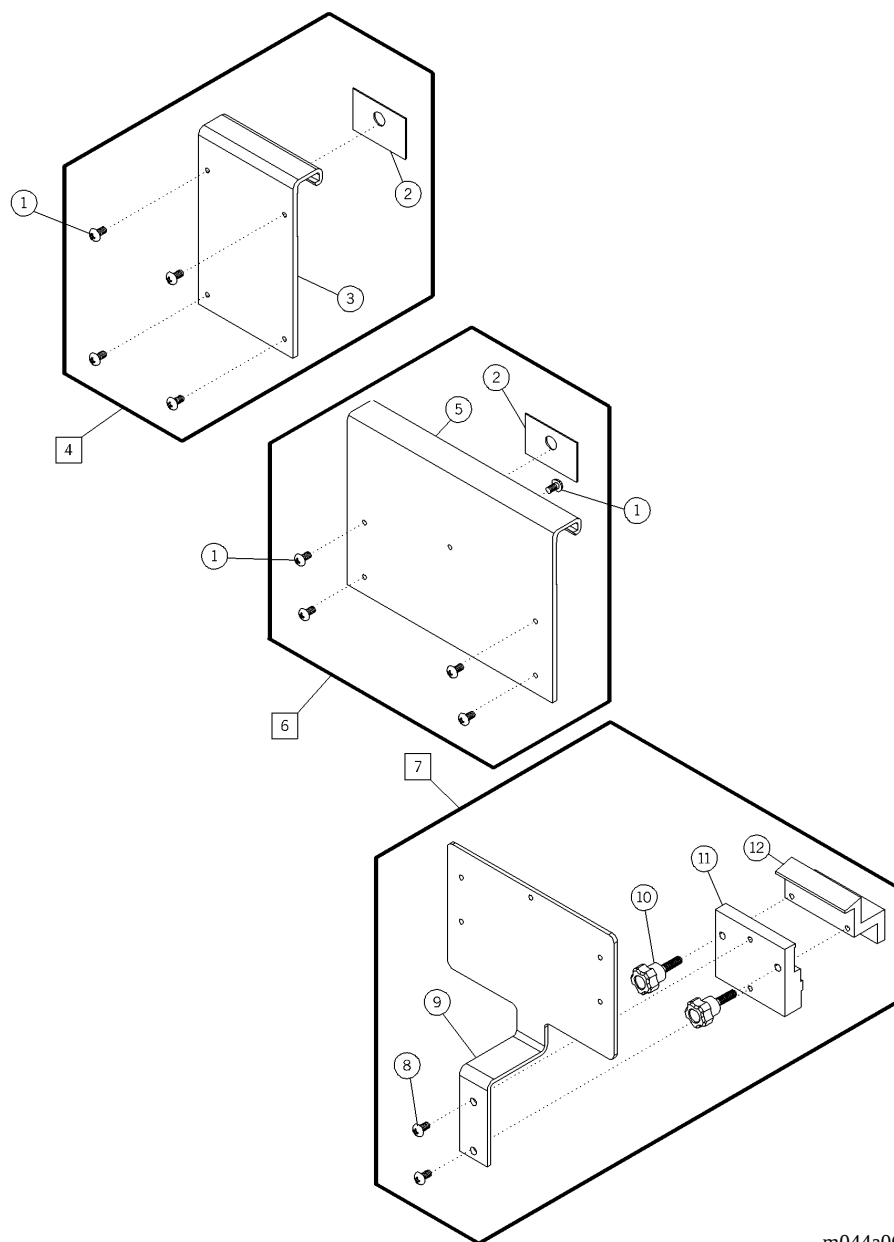


m044a060

Table 1-48. Ophthalmoscope Mount—P1020S1, 2 (Horizon Headwall System—P1000) and P1020S3 (Horizon Headwall System—P1000, Console—P1002AC)

Item Number	Part Number	Quantity	Description
1	52604-02	4 or 5	Screw
2	52793	1	Skid pad
3	55830pl	1	Hook, ophthalmoscope
4	P1020S1	1	Ophthalmoscope mount
5	57893pl	1	Hook, ophthalmoscope
6	P1020S2	1	Ophthalmoscope mount
7	P1020S3	1	Ophthalmoscope mount—Uniflex
8	5325801	2	Screw, pan head
9	5789539	1	Mount, ophthalmoscope
10	52569	2	Knob
11	52571	1	Trough clamp—front
12	52572	1	Trough clamp—rear

Chart Holder—P102101 (Horizon Headwall System—P1000)

Figure 1-49. Chart Holder—P102101 (Horizon Headwall System—P1000)

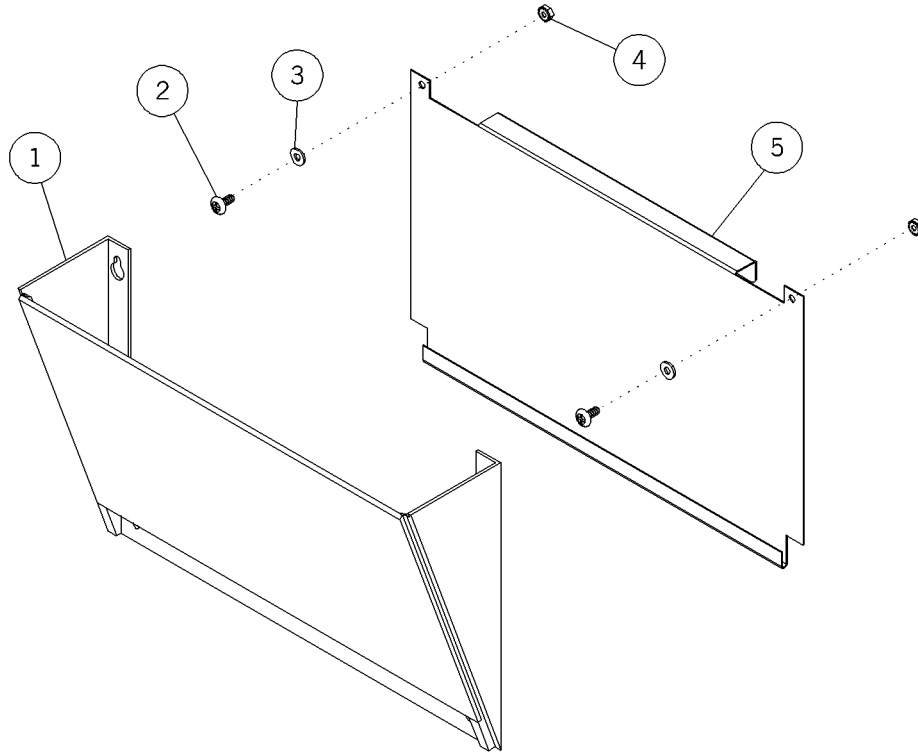

m044a061

Table 1-49. Chart Holder—P102101 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	51920	1	Filing pocket
2	17017	2	Screw
3	143	2	Washer
4	15250	2	Locknut
5	55937-39	1	Back

NOTES:


m044a062

Table 1-50. Resuscitator Bag Holder—P102201 (Horizon Headwall System—P1000) and P1022PH (Flatwall Headwall System—P670, Power Column—P950)

Item Number	Part Number	Quantity	Description
1	102201	1	Resuscitation bag holder
2	15338	2	Screw
3	54621	1	Latch
4	52457	2	Roll pin
5	54880	1	Accessory mount
6	54885	1	Accessory mount assembly
7	5610239	1	Bracket, resuscitator bag
8	54610-01	2	Screw
9	51886	1	Holder—manual resuscitator
10	1022ph	1	Holder—resuscitator bag
11	5640339	1	Support bracket, resuscitator bag
12	53274	1	Label—patent
13	33704	1	Bolt
14	56530	4	Washed, beveled nylon
15	50126	2	O-ring
16	34218	1	Compression nut
17	52386	1	Washer
18	53397-03	1	Screw

Universal Holder—P102301 (Horizon Headwall System—P1000, Neonatal Horizon Headwall System—P1001)

Figure 1-51. Universal Holder—P102301 (Horizon Headwall System—P1000, Neonatal Horizon Headwall System—P1001)

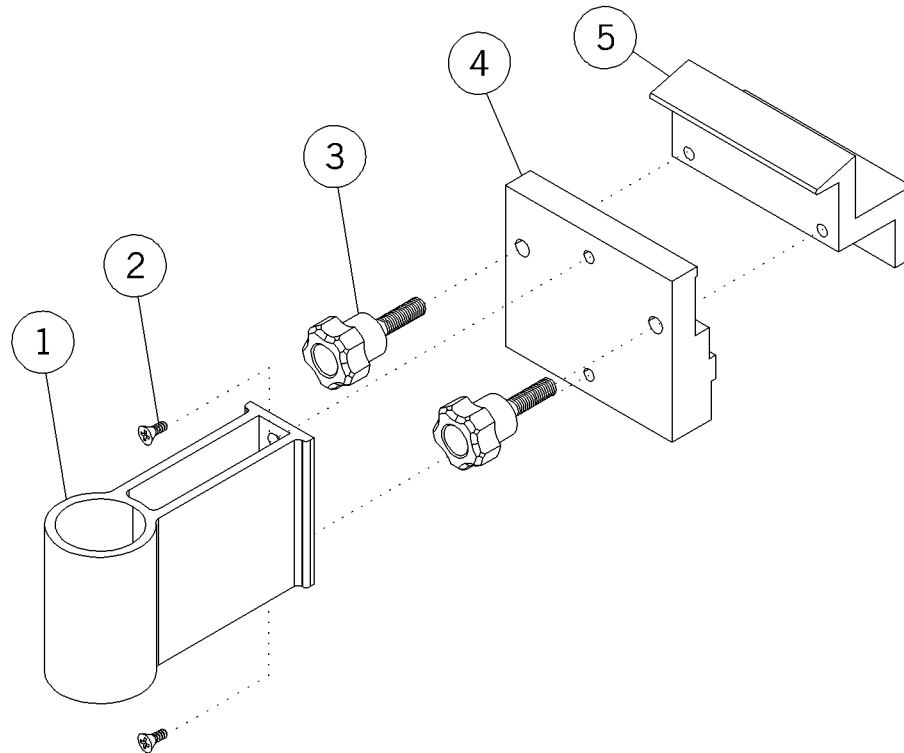

m044a063


Table 1-51. Universal Holder—P102301 (Horizon Headwall System—P1000, Neonatal Horizon Headwall System—P1001)

Item Number	Part Number	Quantity	Description
1	52570	1	Holder
2	34099	2	Screw
3	52569	2	Knob
4	52571	1	Trough clamp—front
5	52572	1	Trough clamp—rear

NOTES:

Sphygmomanometer Mount, Columnar (Mercurial)—P102402, Aneroid—P102502 (Horizon Headwall System—P1000)

Figure 1-52. Sphygmomanometer Mount, Columnar (Mercurial)—P102402, Aneroid—P102502 (Horizon Headwall System—P1000)

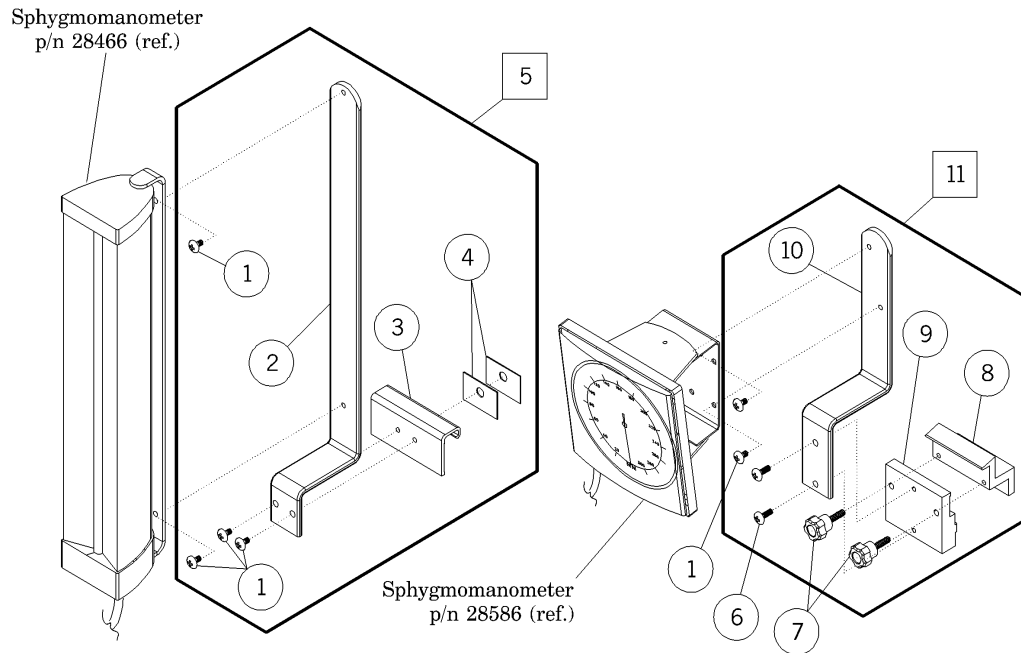


m044a074

Table 1-52. Sphygmomanometer Mount, Columnar (Mercurial)—P102402, Aneroid—P102502 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	54610-01	2 or 4	Screw
2	5881139	1	Bracket, mercury sphygmomanometer
3	5881839	1	Hook, sphygmomanometer mount
4	52793	1	Skid pad
5	P102402	1	Sphygmomanometer mount, columnar (mercurial)
6	51637	2	Screw
7	52569	2	Knob
8	52572	1	Trough clamp—rear
9	52571	1	Trough clamp—front
10	58812	1	Bracket, aneroid sphygomanometer
11	P102502	1	Sphygmomanometer mount, aneroid

Utility Hook—P102601 (Horizon Headwall System—P1000)

Figure 1-53. Utility Hook—P102601 (Horizon Headwall System—P1000)

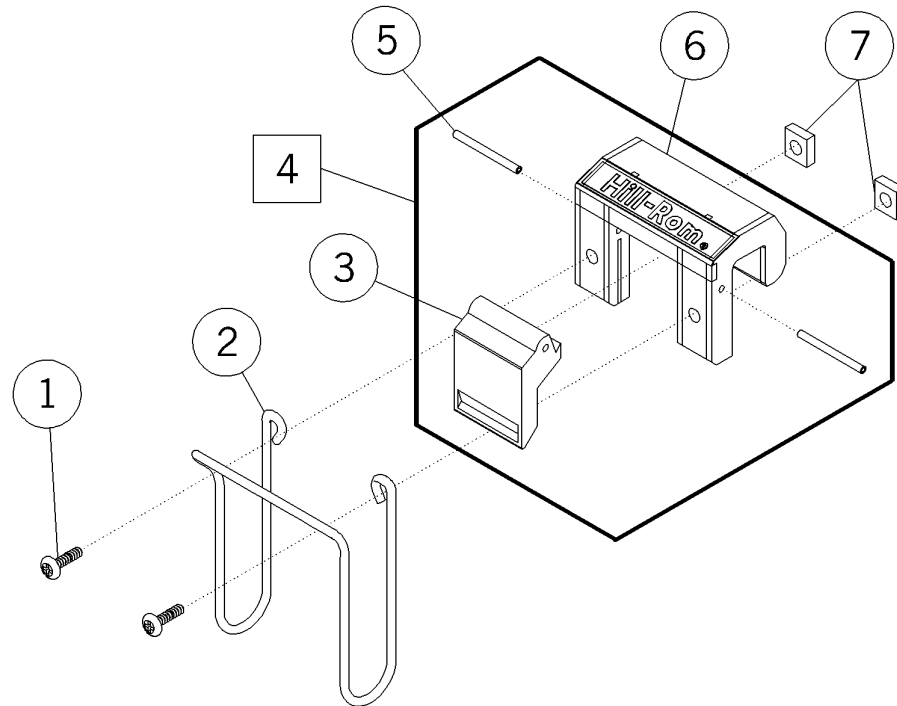

m044a065

Table 1-53. Utility Hook—P102601 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	15338	2	Screw
2	51781	1	Cuff clip
3	54621	1	Latch
4	54886	1	Accessory mount assembly
5	52457	2	Roll pin
6	54881	1	Accessory mount
7	33653	2	Nut

Utility Light—P102701 (Horizon Headwall System—P1000)

Figure 1-54. Utility Light—P102701 (Horizon Headwall System—P1000)

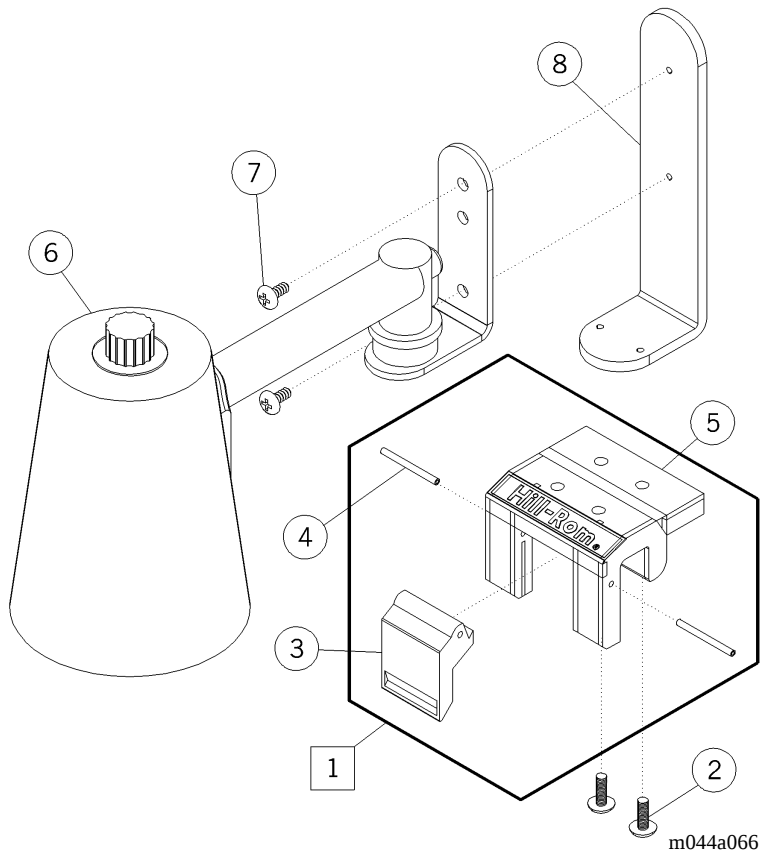


Table 1-54. Utility Light—P102701 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	54885	1	Accessory mount assembly
2	15338	2	Screw
3	54621	1	Latch
4	52457	2	Roll pin
5	54886	1	Accessory mount assembly
6	51571	1	Lamp—966 track light
7	54610-01	2	Screw
8	5649439	1	Bracket, utility light

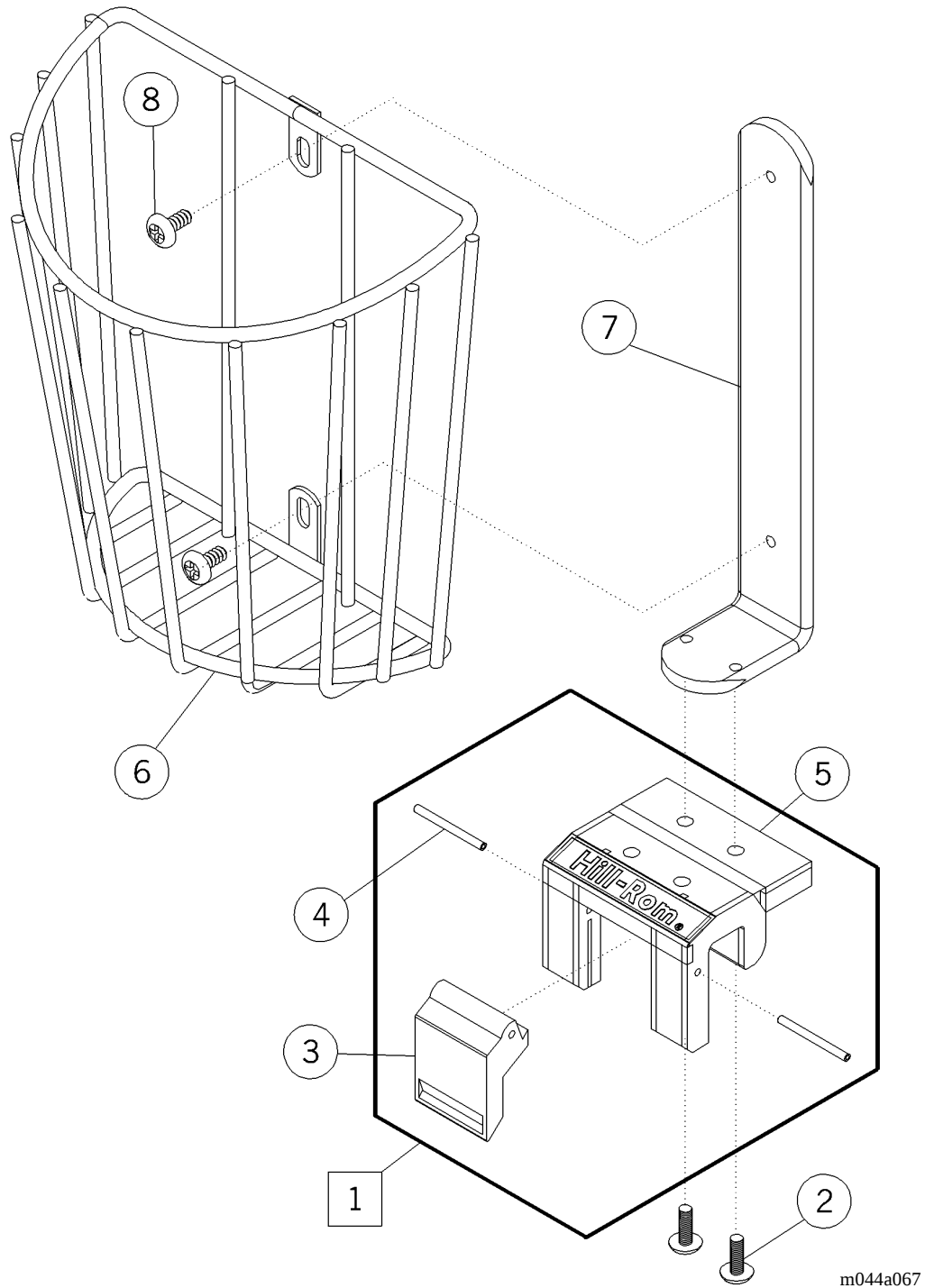
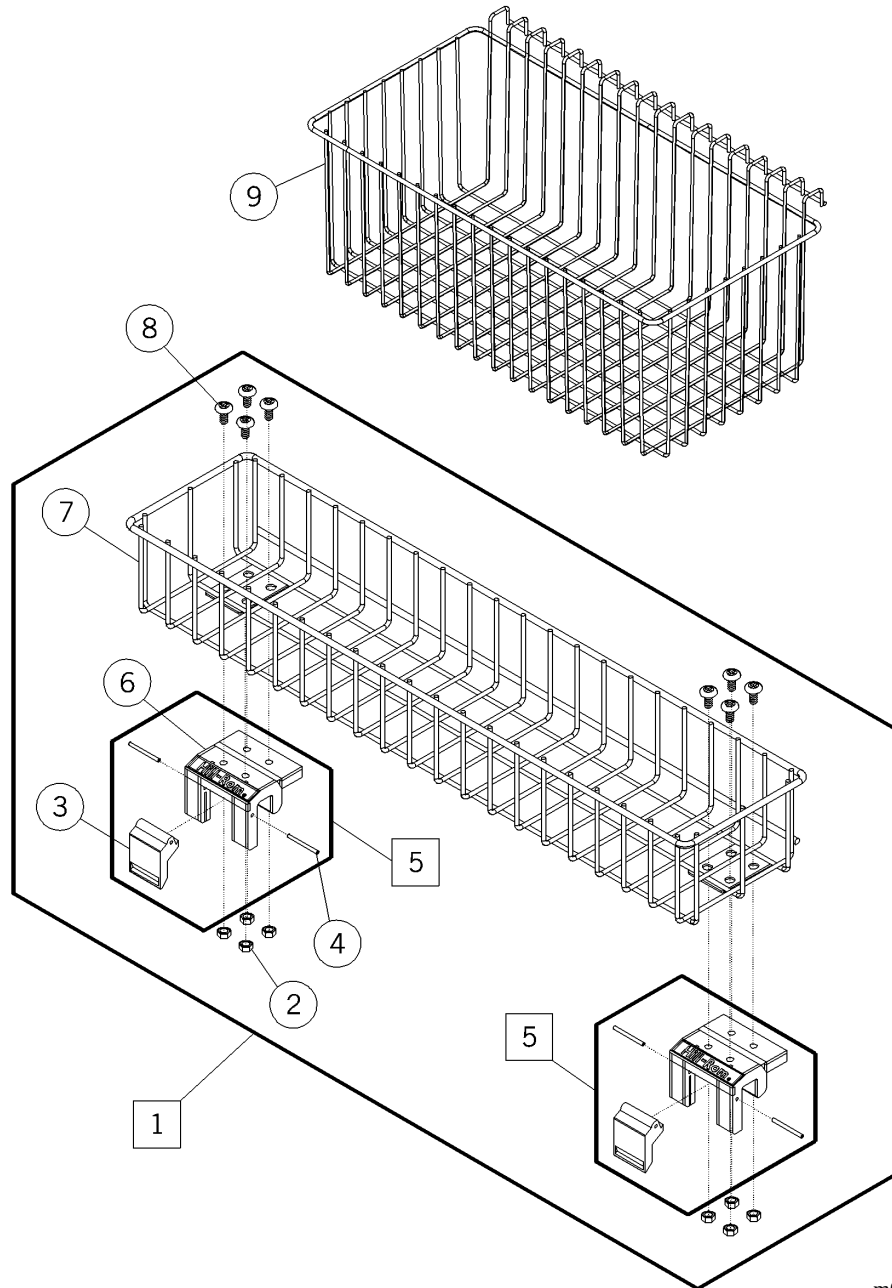
Utility Basket—P102801 (Horizon Headwall System—P1000)**Figure 1-55. Utility Basket—P102801 (Horizon Headwall System—P1000)**

Table 1-55. Utility Basket—P102801 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	54885	1	Accessory mount assembly
2	15338	2	Screw
3	54621	1	Latch
4	52457	2	Roll pin
5	54880	1	Accessory mount
6	56875	1	Cuff basket
7	57010-39	1	Bracket, cuff
8	54610-01	2	Screw

Small Storage Basket—P1029S2 and Large Storage Basket—P1029L2 (Horizon Headwall System—P1000)**Figure 1-56. Small Storage Basket—P1029S2 and Large Storage Basket—P1029L2 (Horizon Headwall System—P1000)**


m044a068

Table 1-56. Small Storage Basket—P1029S2 and Large Storage Basket—P1029L2 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	1029S2	1	Small storage basket
2	56400	8	Nut, shakeproof
3	54621	2	Latch
4	52457	4	Roll pin
5	54885	2	Accessory mount assembly
6	54880	2	Accessory mount
7	56921	1	Basket, small
8	54105	8	Pivot screw
9	P1029L2	1	Large storage basket

Bag Assembly—Port Cap, Air Manifold—P1032A1, Nitrogen Manifold—N1, Oxygen Manifold—O1, and Vacuum Manifold—V1 (Horizon Headwall System—P1000)

Figure 1-57. Bag Assembly—Port Cap, Air Manifold—P1032A1, Nitrogen Manifold—N1, Oxygen Manifold—O1, and Vacuum Manifold—V1 (Horizon Headwall System—P1000)

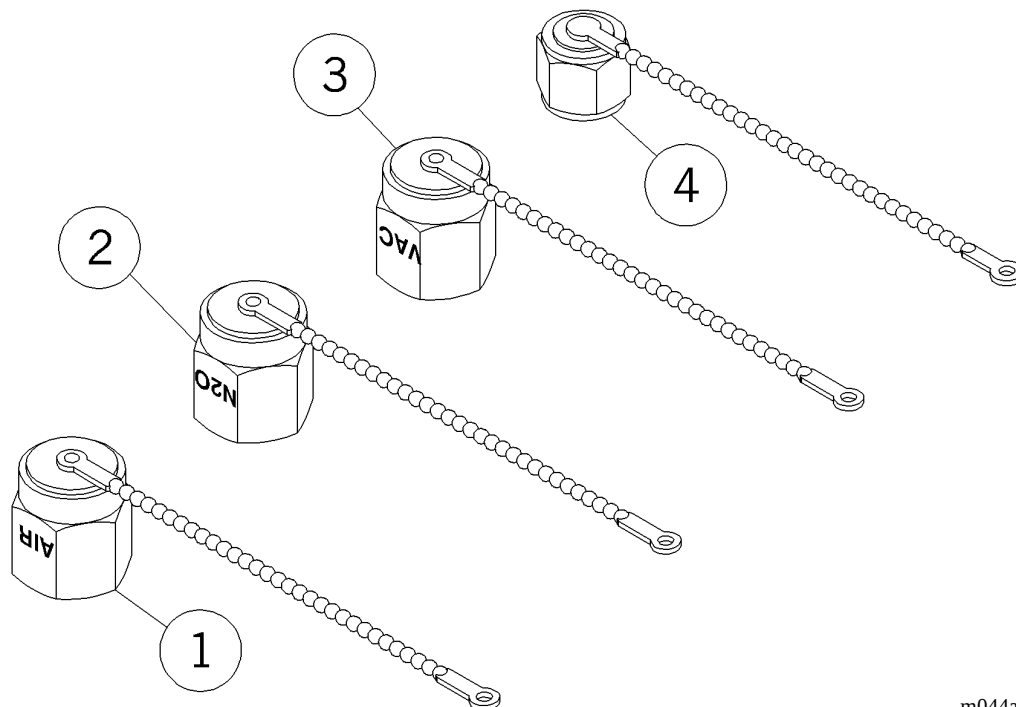

m044a069

Table 1-57. Bag Assembly—Port Cap, Air Manifold—P1032A1, Nitrogen Manifold—N1, Oxygen Manifold—O1, and Vacuum Manifold—V1 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	53109	1	Cap—air
2	53111	1	Cap—seal—nitrous oxide
3	53110	1	Cap—vacuum
4	53108	1	Cap—oxygen

Adapter, Air—P1032A7, 8, 9, Oxygen—P1032O7, 8, 9, and Vacuum—P1032V7, 8, 9

Figure 1-58. Adapter, Air—P1032A7, 8, 9; Oxygen—P1032O7, 8, 9; and Vacuum—P1032V7, 8, 9

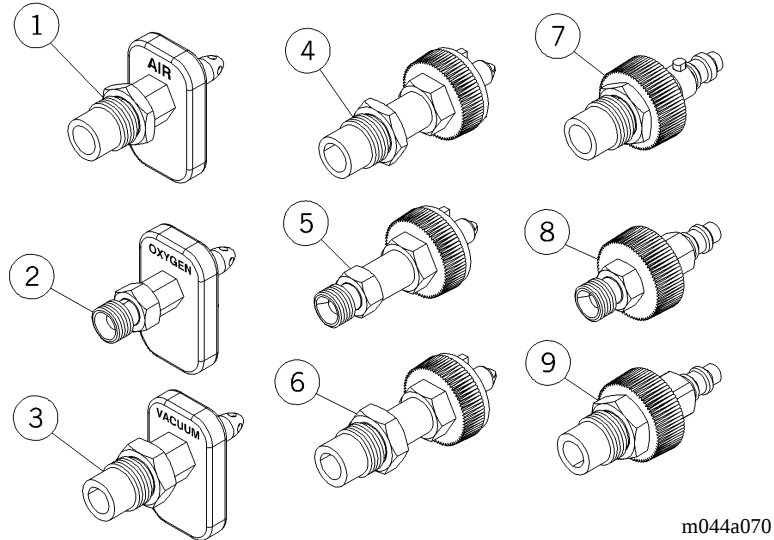


Table 1-58. Adapter, Air—P1032A7, 8, 9; Oxygen—P1032O7, 8, 9; and Vacuum—P1032V7, 8, 9

Item Number	Part Number	Quantity	Description
1	P1032A7	1	Adapter, air—Chemetron to DISS—male
2	P1032O7	1	Adapter, oxygen—Chemetron to DISS—male
3	P1032V7	1	Adapter, vacuum—Chemetron to DISS—male
4	P1032A8	1	Adapter, air—Ohio to DISS—male
5	P1032O8	1	Adapter, oxygen—Ohio to DISS—male
6	P1032V8	1	Adapter, vacuum—Ohio to DISS—male
7	P1032A9	1	Adapter, air—Puritan to DISS—male
8	P1032O9	1	Adapter, oxygen—Puritan to DISS—male

Item Number	Part Number	Quantity	Description
9	P1032V9	1	Adapter, vacuum—Puritan to DISS

Rough-In Box—P103402 (Horizon Headwall System—P1000)

Figure 1-59. Rough-In Box—P103402 (Horizon Headwall System—P1000)

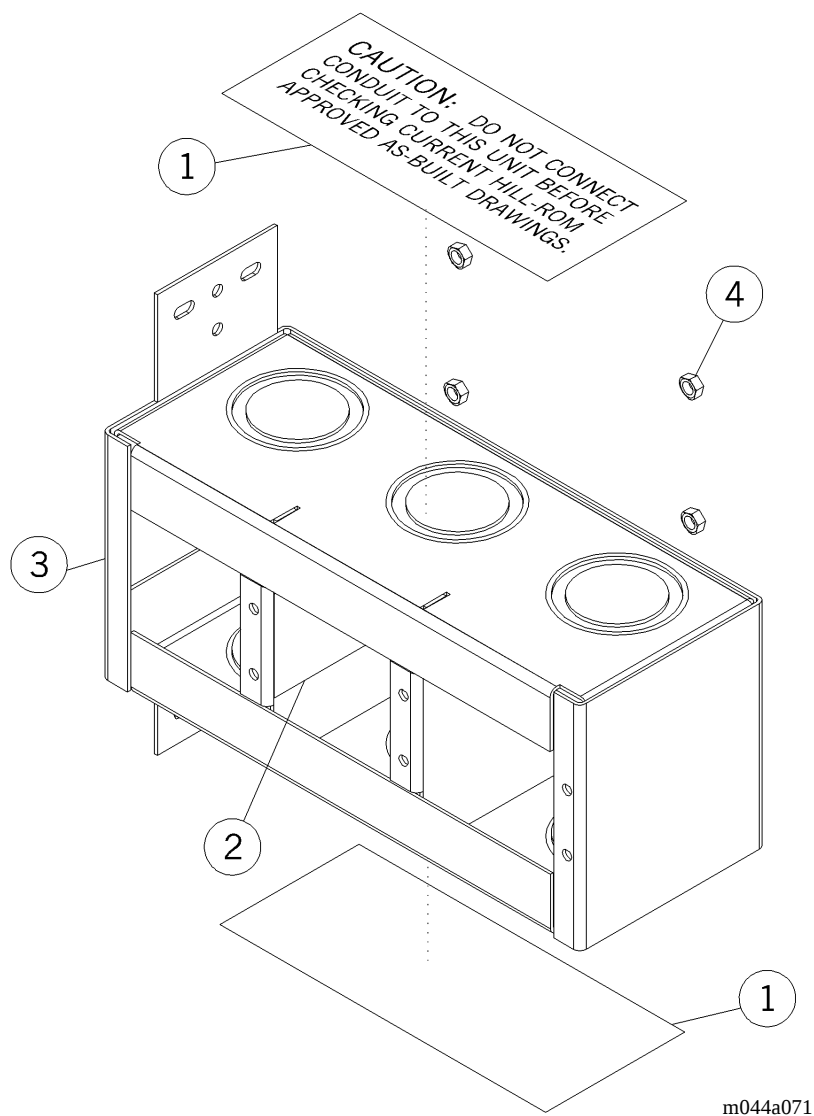



Table 1-59. Rough-In Box—P103402 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	55633	2	Label, caution
2	53056	1	Label—identification
3	55536pl	1	Wall box assembly
4	15250	4	Locknut

NOTES:

TV Arm Mount—RCA/Phillips—P103504, Bunting—P103505, and Phillips—P103506 (Horizon Headwall System—P1000)

Figure 1-60. TV Arm Mount—RCA/Phillips—P103504, Bunting—P103505, and Phillips—P103506 (Horizon Headwall System—P1000)

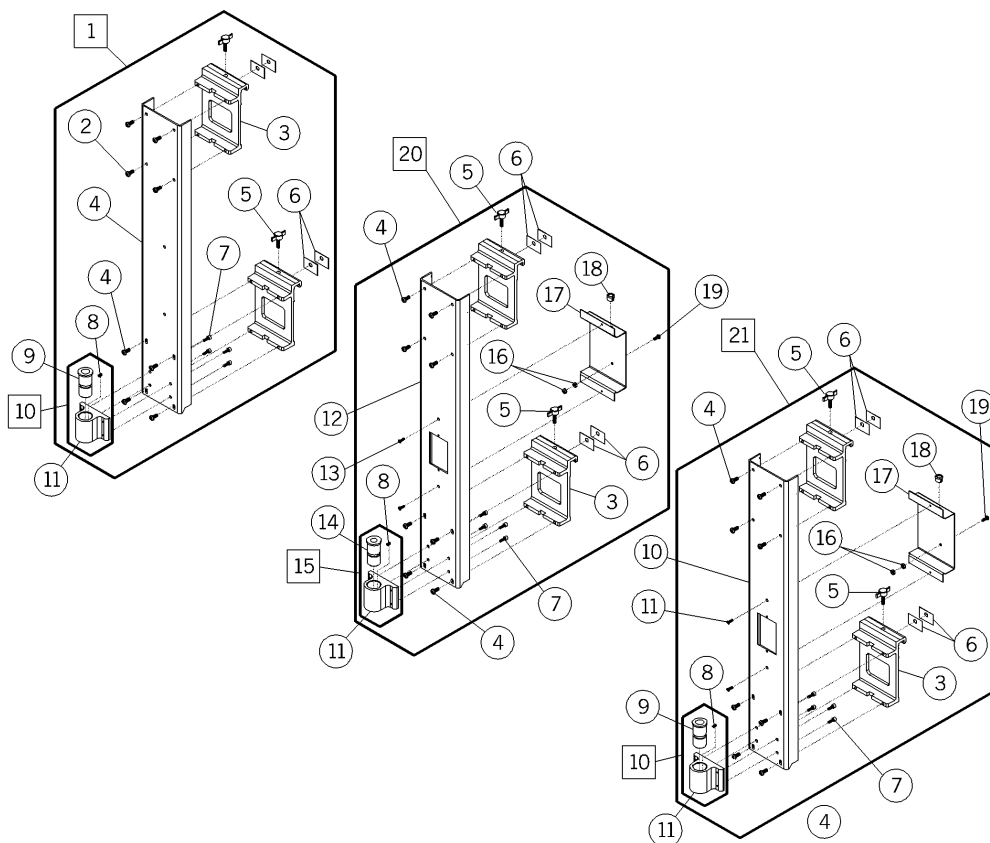

m044a072

TV Arm Mount—RCA/Phillips—P103504, Bunting—P103505, and Phillips—P103506 (Horizon Headwall System—P1000)


Chapter 1: Parts List

Table 1-60. TV Arm Mount—RCA/Phillips—P103504, Bunting—P103505, and Phillips—P103506 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	P103504	1	TV arm mount—RCA/Phillips
2	52396	8	Screw
3	52794	2	Hanger
4	527920139	1	Support TV arm
5	53913	2	Knob assembly
6	52793	4	Skid pad
7	51012	4	Screw
8	50060	1	Setscrew
9	52431	1	Adapter
10	52973-01	1	Pivot block assembly
11	52969	1	Pivot block
12	527920239	1	Support TV arm
13	50590	2	Screw
14	52301	1	Adapter
15	5297302	1	Pivot block assembly
16	15250	2	Locknut
17	5297039	1	Cover, jack
18	24925	1	Bushing
19	52243-03	1	Screw
20	P103505	1	TV arm mount, Bunting
21	P103506	1	TV arm mount, Phillips

Cable Organizer—P103601 (Horizon Headwall System—P1000)

Figure 1-61. Cable Organizer—P103601 (Horizon Headwall System—P1000)

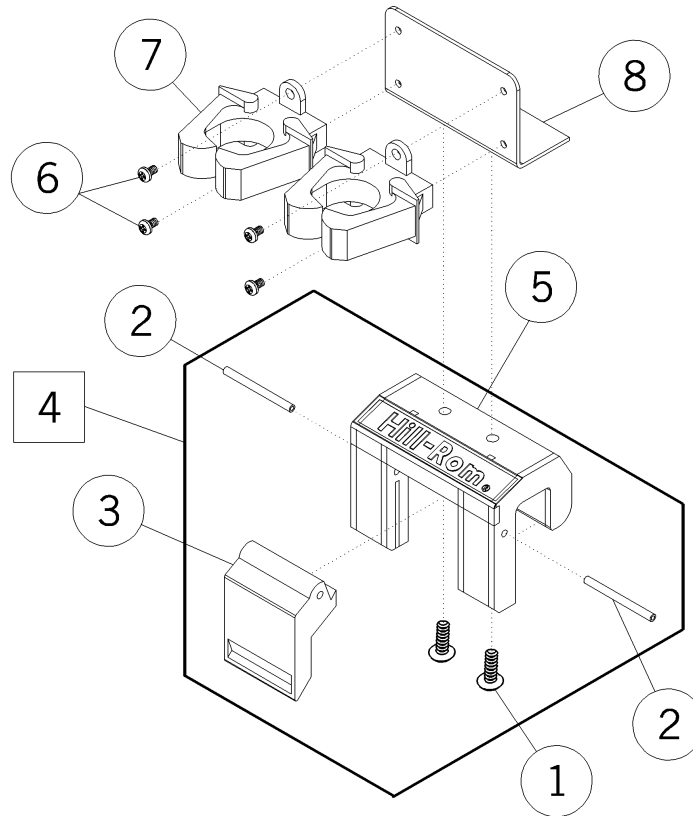

m044a073

Table 1-61. Cable Organizer—P103601 (Horizon Headwall System—P1000)

Item Number	Part Number	Quantity	Description
1	53397-03	2	Screw
2	52457	2	Roll pin
3	54621	1	Latch
4	54887	1	Accessory mount assembly
5	54882	1	Accessory mount
6	50891-02	4	Screw
7	53319	2	Cable holder

Item Number	Part Number	Quantity	Description
8	5331839	1	Bracket