

stryker[®]

TPS[™]

**Universal
Handswitch**

REF 5100-9

ENGLISH

日本語

ESPAÑOL

DEUTSCH

FRANÇAIS

ITALIANO

NEDERLANDS

SVENSKA

DANSK

SUOMI

PORTUGUÊS

NORSK

R_x ONLY

CE 0197

Stryker Instruments
4100 E. Milham
Kalamazoo, Michigan
(USA) 49001
1-269-323-7700
1-800-253-3210
www.stryker.com



Stryker France
ZAC Satolas Green Pusignan
Av. de Satolas Green
69881 MEYZIEU Cedex
France

11/07
5100-009-700 Rev-H

ENGLISH

For use with Total Performance System handpieces.



WARNING: Before using any TPS component, read and understand its instructions and review the important User/Patient Safety information provided in the TPS Console instruction booklet. Familiarization with the Total Performance System prior to use is important. If you have any questions, contact your Stryker Instruments representative or Stryker Customer Service at 1-800-253-3210.



WARNING: Prior to use, inspect for damage. Do not use if damage is apparent.



WARNING: Place handswitch in SAFE position while changing cutting accessory.

Important Information

Refer to your *TPS Cleaning, Maintenance, and Sterilization Recommendations* for instrument care information.

Installation

NOTE: Cord must be removed from handpiece to install handswitch.

1. Slide safety switch to SAFE position. Figure 1.
2. Align white dots and press handswitch band over end of handpiece. Figure 1.

NOTE: Handswitch band must be flush with handpiece so cord can seat properly. Figure 2.

3. Plug cord into handpiece. Figure 3.
4. Adjust finger rest as needed. Figure 3. The handpiece can be held like a pencil. Figure 4.

NOTE: When using the handswitch with the TPS Oscillating saw, the finger rest must be either fully extended or retracted.

RUN = run
LOAD = load

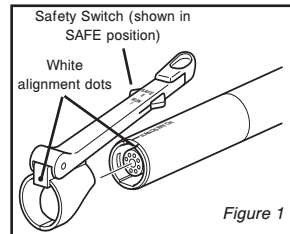


Figure 1

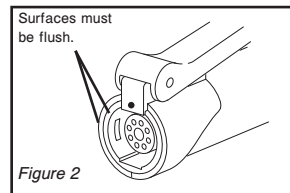


Figure 2

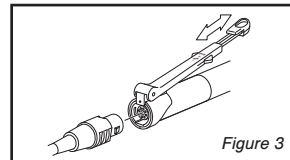


Figure 3

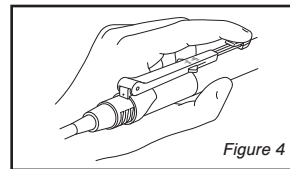


Figure 4

Instructions

1. Slide safety switch to RUN position.
2. Depress lever to regulate variable handpiece speed.

NOTE: Maximum speed setting is selected on console.



WARNING: Always put the safety switch into the safety position to prevent inadvertent running of the handpiece.

To remove:

1. Place safety switch in SAFE position.
2. Remove cord.
3. Grasp band portion of handswitch and slide off back of handpiece taking care not to pull back on lever.

Troubleshooting

Problem	Cause	Solution
Handswitch does not work.	Not properly seated on handpiece.	Reset.
	Safety switch in SAFE position.	Place safety switch in RUN position.
Motor does not reach full speed.	User may be sideloaded switch.	Depress lever centered against handpiece.

Stryker Instruments
4100 E. Milham
Kalamazoo, Michigan
(USA) 49001
1-269-323-7700
1-800-253-3210
www.stryker.com



Stryker France
ZAC Satolas Green Pusignan
Av. de Satolas Green
69881 MEYZIEU Cedex
France

stryker®