
Microstream[®]

ETCO₂ SAMPLING

FilterLine XL Directions for Use

FilterLine[®] XL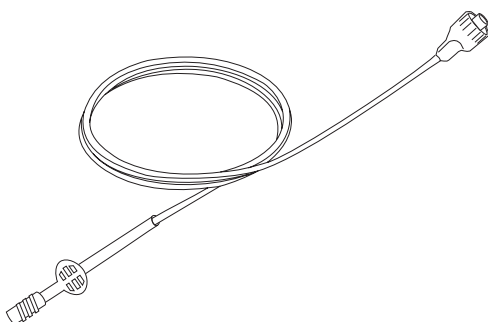

US Patents: www.covidien.com/patents.

© 2015 Oridion Medical 1987 Ltd.
All rights reserved.




PN 006256_RMS1F

Microstream® EtCO₂ Sampling

Instructions for use:

1. Place the monitor outside the MRI suite. There must be a hole in the wall of the suite (approximately 10 cm diameter).
2. Connect the FilterLine® XL **1** to the monitor by twisting the female luer connector securely but gently into the Microstream® input connector **2**.
3. Guide the FilterLine® XL through the hole in the wall of the MRI suite.
4. Attach the FilterLine® XL to an intubated or non-intubated Microstream CO₂ sampling line as appropriate for the patient **3**, by attaching the male luer at the end of the sampling line to the luer connector on the FilterLine XL.
5. Ensure that the tubing is not twisted or crimped.
6. Attach the CO₂ sampling line to the patient as described in its Directions for Use.
7. Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.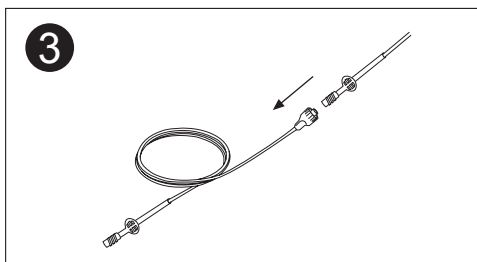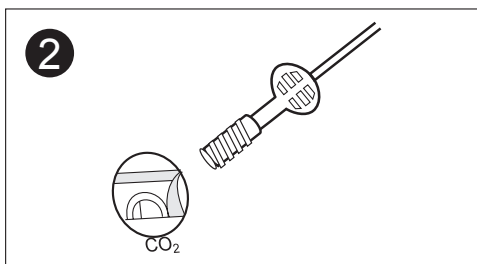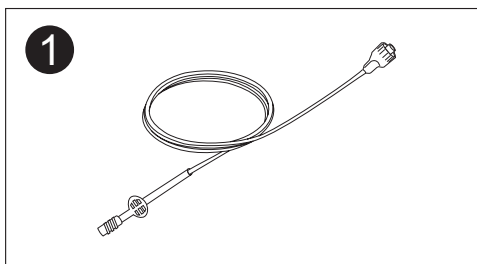

Microstream® EtCO₂ Sampling

FilterLine® XL for short-term use during MRI scanning

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Does not contain natural rubber latex

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Caution: Use of a CO₂ sampling line with H in its name (indicating that it is for use in humidified environments) during MRI scanning may cause interference. These sampling lines include FilterLine® H Set/Long, VitaLine™ H Set CapnoLine® H/Long, CapnoLine® H O₂, Smart CapnoLine® H/Long, Smart CapnoLine® H O₂, and Smart CapnoLine® H Plus O₂/Long. The use of non H sampling lines is advised.

Note: During nebulization or suction, in order to avoid moisture buildup and sampling line occlusion, remove the sampling line luer connector from the monitor.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Note: When connecting a sampling line to the monitor, insert the sampling line connector clockwise into the monitor CO₂ port and turn the connector until it can no longer be turned, to ensure that it is connected securely to the monitor. This will assure that there is no leak of gases during measurement at the connection point and that measurement accuracy is not compromised.

Note: Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.

Note: A longer delay time, slower response time and decreased frequency response will occur when using the FilterLine XL, due to its extended length.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

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 Single patient use	For use with a single patient
Rx ONLY	Rx Only
 Consult instructions for use	Consult instructions for use
 Caution, consult accompanying documents	Caution, consult accompanying documents
 -4°F -20°C Temperature limitation	Storage temperature
 This product does not contain natural rubber latex	Does not contain natural rubber latex
CE	CE Mark

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Oridion®, Microstream®, FilterLine®, Smart CapnoLine®, CapnoLine®, and Smart CapnoLine Guardian™ are trademarks or registered trademarks of Oridion Medical 1987 Ltd. ™* and ®* are trademarks of their respective owners.

Oridion Medical 1987 Ltd.
7 Hamarpe St.
P.O. Box 45025
Jerusalem 9777407, Israel
US Toll Free: 1-888-674-3466
1-888-ORIDION

Obelis S.A.
Bd Général Wahis 53
1030 Brussels
Belgium