Addendum

IMAGE 800 **Immunochemistry System Instructions for Use**

For *In Vitro* Diagnostic Use

C05164AD
March 2019

Beckman Coulter, Inc.
250 S. Kraemer Blvd.
Brea, CA 92821 U.S.A.

Addendum
IMMAGE 800 Immunochemistry System
Instructions For Use
PN C05164AD (March 2019)

© 2019 Beckman Coulter, Inc.
All rights reserved

Trademarks

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

All other trademarks are the property of their respective owners.

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Software version 2.0

Changes:

- Added six new languages: Russian, Slovak, Brazilian Portuguese, Dutch, Czech, and Traditional Chinese.

C05164AC, August 2018

Software version 2.0

Changes:

- **IMAGE 800 Symbols and Hardware Labels:** Added a symbol and a link to the website in the **California Proposition 65** statement.

C05164AB, March 2018

Software version 2.0

Changes:

- Changed the Chinese translation of Immunochemistry System.
- Added four new languages: French, Italian, German and Spanish.

C05164AA, March 2017

Software version 2.0. Initial Release

Safety Notice

Read all product manuals and consult with Beckman Coulter trained personnel before you operate the system. Do not perform any procedure before you carefully read all instructions. Always follow the product labels and the recommendation from the manufacturer. For more information, contact Beckman Coulter.

Alerts for Warning, Caution, Important, and Note

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could cause death or serious injury. Warning can indicate the possibility of erroneous data that could cause an incorrect diagnosis.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, can cause minor or moderate injury. Caution can also alert against unsafe practices, or indicate the possibility of erroneous data that could cause an incorrect diagnosis.

IMPORTANT Important indicates important information to follow.

NOTE Note indicates notable information to follow.

TIP Tip indicates information to consider.

Precautions And Hazards

Precautions

- Do not store or place a diskette near electrical motors, power supplies, generators, magnets, or magnetic fields.
- Hold a compact disk (CD) by the edges and replace it in its case after use. DO NOT place a CD in direct sunlight or excessive heat or humidity.
- Sample containers must contain an adequate volume of test specimen to ensure accurate aspiration.

Hazards

- Connect the three-pronged power plug from all system components of the IMAGE 800 Immunochemistry System to a three-wire grounded power source. DO NOT use an adapter to connect the power plug to a two-pronged outlet. If the electrical outlet will not accept the three-pronged plug, notify a qualified maintenance personnel.
UNDER ANY CIRCUMSTANCES, DO NOT OPERATE THE SYSTEM UNTIL AN ELECTRICAL GROUND IS PROVIDED AND THE POWER CORD IS PROPERLY CONNECTED TO GROUND.
- Close reagent and sample carousel covers and keep clear of all mechanical assemblies when booting up the system.
- Keep clear of both cranes while the instrument is running.
- Keep all covers and shields in place while the instrument is running.
- Confirm that the instrument status is *Standby* when adding or changing reagents, buffers, diluents, or dilution segments. The instrument status must be in *Standby* or "Pausing-OK to load

samples" when adding or removing samples. Keep reagent and sample carousel covers closed while the instrument is running.

- Confirm that the system is turned off before replacing any defective mechanical or electrical part in the system.
- DO NOT tamper with or remove the housing of any bar code reader.
- Dispose all waste liquids from the IMAGE 800 Immunochemistry System in an approved method for handling biohazardous material.
- If a hazardous substance such as blood is spilled onto the IMAGE 800 System, clean up the spill by using a 10% bleach solution, or use your laboratory decontamination solution. Then follow your laboratory procedure for disposal of hazardous materials. If the IMAGE 800 System needs to be decontaminated, call your Beckman Coulter Service Representative for assistance. Dispose of all contaminated disposable cleaning materials in accordance with your local regulations and acceptable laboratory practices.

- Observe all laboratory policies or procedures pertaining to the handling of biological samples that may contain pathogens.
- To **STOP**, turn the instrument main power switch off if the stop button on the screen is unavailable, and the instrument must be stopped immediately.
- **PRESERVATIVES:** Sodium azide preservative may form explosive compounds in metal drain lines. See National Institute for Occupational Safety and Health bulletin: Explosive Azide Hazards (8/16/76).
- Incineration of used reagent cartridges may produce toxic fumes.

IMAGE 800 Symbols and Hardware Labels

The following symbols and hardware labels are attached to the IMAGE 800. Use caution, observe and follow all warning labels. Do not cover or remove these labels. If the labels peel off or become illegible, contact Beckman Coulter to replace the labels.

- Orange labels identify areas of the system where hazards exist and where caution should be taken to avoid serious injury or death.
- Yellow labels indicate that there is a risk of personal injury, fire, or damage.
- Instruction Labels are affixed on the system at relevant locations to alert the operator to operate the system correctly.

Caution Read Manual Symbol

This symbol indicates that all operators must read the instrument's IFU and operation manuals for an explanation of the Caution or Warning.

Biohazard Symbol

This is the international symbol for biohazardous material. This label indicates the use of biohazardous materials in the area such as sample probes, mixer bars, sample racks, cuvettes, a sample probe wash well, and so forth. Use caution when working with possible infectious samples.

Wear Personal Protective Equipment (PPE) such as gloves, eye shields, and lab coats. Handle and dispose of biohazardous materials according to your laboratory procedures.

Moving Parts Symbol


The Moving Parts Symbol indicates that there are moving parts in the area and that the system should be operated only when all covers are in place and caution is used to reduce risks of personal injury. Failure to properly seat covers could lead to erroneous results. Moving parts may include the sample probe, reagent probes, parts of the syringes, mixer bars, and so forth.

Laser Radiation Symbol


The Laser Radiation Symbol indicates that there may be laser light radiation in the area. Take precautions to prevent exposure.

Caution Class 2 Laser Panel Marking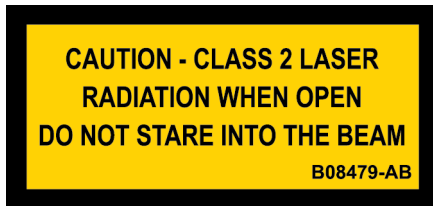

This Panel Marking indicates that there is Class 2 laser light radiation beyond the panel it is placed on. Use caution and do not stare into the beam when laser light is in the area.

Laser Certification Label

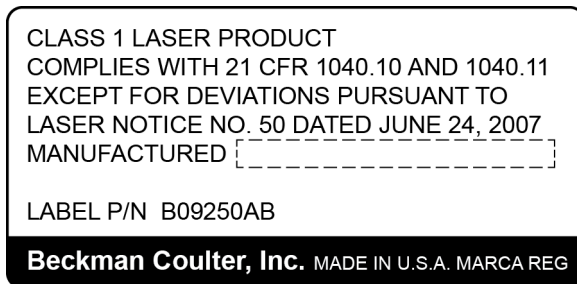The Class 1 Laser Product Label indicates that the product contains a Class 1 laser that complies with US requirements (21 CFR 1040.10 and 1040.11). This label is located on the right side panel.

CE Mark


The CE mark indicates that a product has been assessed before being placed on the market, and has been found to meet European Union safety, health, and environmental protection requirements.

Regulatory Compliance Mark

This Regulatory Compliance Mark (RCM) indicates compliance with the ACMA requirements (safety and EM) for Australia and New Zealand.

Recycling Label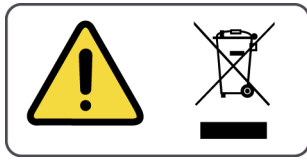

This label is required in accordance with the Waste Electrical and Electronic Equipment (WEEE) Directive of the European Union. The presence of this label indicates that:

1. The device was put on the European Market after August 13, 2005 and,
2. The device is not to be disposed of via the municipal waste collection system of any member state of the European Union.

Customers must understand and follow all laws regarding the correct decontamination and safe disposal of electrical equipment. For Beckman Coulter products bearing this label, contact your dealer or local Beckman Coulter office for details on the take-back program that facilitates the correct collection, treatment, recovery, recycling and safe disposal of these products.

For the Japan Market:

This system is considered an industrial waste, subject to special controls for infectious waste. Prior to disposal of the system, refer to the “Waste Disposal and Public Cleaning Law” for compliance procedures.

Restriction of Hazardous Substances (RoHS) Labels

These labels and materials declaration table (the Table of Hazardous Substance's Name and Concentration) meet People's Republic of China Electronic Industry Standard SJ/T11364-2006 “Marking for Control of Pollution Caused by Electronic Information Products” requirements.

RoHS Caution Label

This logo indicates that this electronic information product contains certain toxic or hazardous elements, and can be used safely during its environmental protection use period. The number in the middle of the logo indicates the environmental protection use period (in years) for the product. The outer circle indicates that the product can be recycled. The logo also signifies that the product should be recycled immediately after its environmental protection use period has expired. The date on the label indicates the date of manufacture.

RoHS Environmental Label

This logo indicates that the product does not contain any toxic or hazardous substances or elements. The “e” stands for electrical, electronic and environmental electronic information products. This logo indicates that this electronic information product does not contain any toxic or hazardous substances or elements, and is green and is environmental. The outer circle indicates that the product can be recycled. The logo also signifies that the product can be recycled after being discarded, and should not be casually discarded.

QNX License

This symbol indicates the license number for the QNX software included in the computer.

Consult Instructions For Use

This symbol indicates that the user should read the product’s Instructions For Use before using the system.

Manufacturer Identifier

This symbol indicates the location of the legal manufacturer.

Date of Manufacture

This symbol indicates the date of manufacture.

EC Representative

This symbol indicates the European Community Representative for the product.

CSA

This symbol indicates recognition by a nationally recognized testing laboratory that the instrument has met the relevant product safety standards for the US and Canada.

In Vitro Diagnostic Use

This symbol indicates that the instrument is an in vitro diagnostic medical device.

Keep Dry

This symbol indicates that the instrument must be kept dry.

Product or Reference Number

This symbol indicates the model number, catalog number or part number of the equipment.

Temperature Limitation

This symbol indicates storage requirement temperature limitations.

USB Port

This symbol indicates a USB port.

Computer Power Switch

This symbol, located above the computer power button, indicates On or Standby conditions. A green light indicates the power to the computer is ON.

Connection Between Computer and Mouse

This symbol is located beside the connection between the computer and the mouse.

Connection Between Computer and Keyboard

This symbol is located beside the connection between the computer and the keyboard.

Connection Between Computer and Monitor

This symbol is located beside the connection between the computer and the monitor.

Instrument or Uninterruptible Power Supply (UPS) Power Switch, ON

This symbol is located on the instrument and the UPS main power switches. When the portion of the switch that displays this symbol is depressed, the instrument is ON.

Instrument or UPS Power Switch, OFF


This symbol is also located on the instrument and the UPS main power switches. When the portion of the switch that displays this symbol is depressed, the instrument is OFF.

Monitor Power Switch

This symbol, located on the monitor power button, indicates On or Standby conditions. A green light indicates the power to the monitor is ON.

Network Port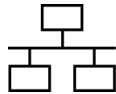

This symbol indicates the location of a network connection port on the computer.

California Proposition 65

This product may contain chemicals known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to <https://www.P65Warnings.ca.gov>.

Warranty and Service

Subject to the below exceptions and conditions, Beckman Coulter warrants to you that (i) the Equipment will perform in all material respects in accordance with the applicable operator manuals, which are available at the Beckman Coulter website, for 12 months upon validation by Beckman Coulter, (ii) Beckman Coulter reagents provided under this Agreement will conform and perform in all material respects in accordance with the applicable labeling for the lesser of the expiration date set forth on such label or 12 months from the delivery date if no date is specified on such label and (iii) Services will be performed in a workmanlike manner. If a Product defect develops under normal and proper use during the warranty period, Beckman Coulter will, at Beckman Coulter's option, and without charge, either repair, during Beckman Coulter's normal business hours, or replace the non-conforming Product. Your exclusive remedy for missing or defective Products will be, at Beckman Coulter's option, the repair or replacement by Beckman Coulter of the Products. The original warranty period will be in effect on any repaired or replaced Products. If Beckman Coulter replaces any part under this warranty or as a result of any Services performed, Beckman Coulter will own the replaced part. Your exclusive warranty for Services not performed in a workmanlike manner will be re-service of the applicable instrument. If a third party manufactured product is supplied to you pursuant to this Agreement (such as software, printers, or personal computers), Beckman Coulter assigns to you any rights that may exist under the warranty provided by the manufacturer. Beckman Coulter does not, however, warrant the performance of the third party manufactured product or provide any remedy for failure of the third party product to perform. Your exclusive remedy is any remedy that may exist under the warranty rights passed through to you under this section.

THE WARRANTIES IN THIS SECTION ARE PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ARE YOUR EXCLUSIVE REMEDIES RELATING TO PERFORMANCE OF THE PRODUCTS OR SERVICES. BECKMAN COULTER DISCLAIMS ALL OTHER WARRANTIES, INCLUDING WITHOUT LIMITATION, ANY WARRANTY ABOUT THE MERCHANTABILITY OF THE PRODUCTS, INFRINGEMENT OR THEIR FITNESS FOR A PARTICULAR PURPOSE. IF ANY IMPLIED WARRANTIES APPLY AS A MATTER OF LAW, THEY ARE LIMITED IN DURATION TO THE LENGTH OF THE TERM OF THE CUSTOMER AGREEMENT.

Please refer to the Customer Agreement for additional terms and conditions.

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