Bair Hugger* Model 505 Temperature Management Unit

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Bair Hugger Model 505 Temperature Management Unit

Operator's Manual

1 English

13 Deutsch

25 Français

37 Italiano

49 Español

61 Português

73 Nederlands

85 Dansk

97 Svenska

109 Norsk

121 Suomi

Table of Contents

3 Introduction

3 Description of the Total Temperature Management* System

3 Indications for Use

3 Contraindications

3 Warnings

3 Precautions

3 Important Information

4 Read Before Servicing Equipment

4 Setup and Operation

5 Temperature Management Unit

6 Control Panel Features of the Model 505 Temperature Management Unit

7 Mounting the Model 505 Temperature Management Unit

8 Attaching and Storing the Model 505 Hose

8 Blanket Dispenser

9 General Maintenance

9 Cabinet Cleaning

9 Technical Service and Order Placement

9 In-Warranty Repair and Exchange

9 When You Call for Technical Support

10 Specifications

12 Definition of Symbols

Introduction

Description of the Total Temperature Management* System

The Bair Hugger brand Total Temperature Management system consists of a Bair Hugger forced-air temperature management unit (with available rolling stand and sheet clip) and disposable components, including Bair Hugger forced-air blankets, Bair Paws* patient warming gowns, and the 241* blood/fluid warming set. The system and disposables are intended for use in clinical settings including the operating room.

This manual includes operating instructions and unit specifications for the Model 505 forced-air temperature management unit. For information about using Bair Hugger blankets, Bair Paws gowns, or the 241 blood/fluid warming set with Bair Hugger units, refer to the "Instructions for Use" included with each of these disposable components.

Indications for Use

The Bair Hugger brand Total Temperature Management system is intended to prevent and treat hypothermia and to provide warmth to cold or shivering patients. In addition, the system should be used whenever conditions exist that could cause patients to become cold.

Contraindication

Do not apply heat to lower extremities during aortic cross-clamping. Thermal injury may occur if heat is applied to ischemic limbs.

Warnings

- Do not use Bair Hugger units with any forced-air blanket or cover other than Bair Hugger blankets or Bair Paws gowns. Thermal injury may result.
- Do not use a forced-air warming device over transdermal medication. Increased drug delivery and patient injury or death may occur.
- Do not provide warming therapy when an intra-aortic balloon pump is in use; thermal injury may occur if heat is applied to ischemic limbs.
- Do not warm patients with the Bair Hugger unit hose alone. Thermal injury may result. Always attach the hose to a Bair Hugger blanket or Bair Paws gown before providing skin surface warming therapy.
- Do not continue therapy if the Over Heat warning light illuminates and the audible alarm sounds. Thermal injury may result. Turn the temperature management OFF and contact qualified technical personnel. If using 241 fluid warming, immediately stop fluid flow and discard the fluid warming set.
- Do not initiate therapy unless the Model 505 temperature management unit is securely mounted or injury may result.

Precautions

- Monitor the patient's temperature and vital signs regularly according to institutional protocol. Reduce air temperature or discontinue therapy when the therapeutic goal is reached or if vital sign instability occurs. Notify physician immediately of vital sign instability.
- To prevent suffocation from misuse, do not leave children or infants unattended when administering Bair Hugger therapy.
- See CONTRAINDICATION and WARNINGS before administering therapy. Read this Operator's Manual, blanket and gown instructions and 241 fluid warming set package instructions before use.

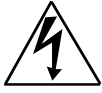
Important Information

EXPLOSION HAZARD

Do not use in the presence of flammable anesthetics.

ELECTRICAL SHOCK HAZARD

Do not disassemble the temperature management unit; refer to authorized technical personnel. There are electrically live parts within the unit when it is connected to the power source, even when the switches are in the OFF position.

**ELECTRICAL INTERFERENCE**

If radio frequency interference with monitoring equipment should occur, connect the temperature management unit to a different power source.

Read Before Servicing Equipment

The repair, calibration, and servicing of the temperature management unit requires the skill of qualified technical personnel who are familiar with good practice for medical device repair. If service is designated as not requiring the manufacturer's attention, the technical information is provided in the Service Manual or will be provided upon request by Arizant Healthcare Inc.

REFER TO SERVICE MANUAL

Perform all repairs and maintenance in accordance with the instructions in the Service Manual.

SAFETY INSPECTION

Perform a safety inspection after making repairs to the temperature management unit and before returning the unit to service. A safety inspection must include a test of the operating temperatures (described in the Service Manual), the Over Heat Alarm system, as well as a leakage current test.

PROPER USE AND MAINTENANCE

Arizant Healthcare Inc. assumes no responsibility for the reliability, performance, or safety of the equipment if:

- Modifications or repairs are performed by unauthorized personnel.
- The equipment is used in a manner other than that described in the Operation or Service Manuals.
- The equipment is installed in an environment that does not meet the relevant grounding requirements.

Setup and Operation

The Bair Hugger brand Total Temperature Management system is easy to set up and to use. Follow the instructions provided with each Bair Hugger blanket or Bair Paws gown for specific information.

- 1) Place the blanket or gown on the patient with the perforated side (the side with small holes) against the patient's skin.

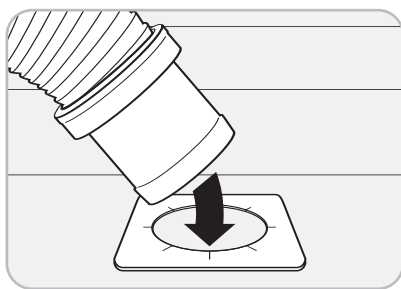
Figure A.

- 2) Insert the hose of the temperature management unit in the hose port on the blanket or gown. Use a twisting motion to ensure a snug fit (see Figure A).

- 3) Connect the unit to a properly grounded power source.
- 4) Press the System ON/OFF button to turn the unit ON and select the appropriate temperature setting.
- 5) You may place a cotton blanket over the blanket or gown for maximum effectiveness.
- 6) Monitor the patient's temperature at regular intervals according to institutional protocol and adjust the temperature setting of the unit as required.

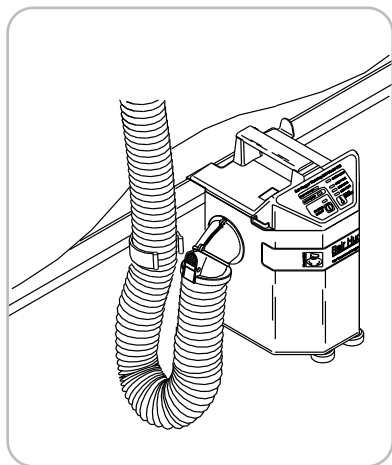
Figure B. Model 505 unit attached to a bedrail

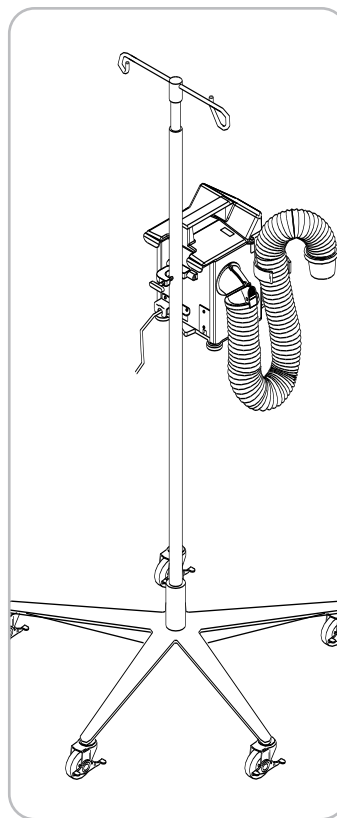
Figure C. Model 505 unit attached to an IV pole

Temperature Management Unit

The temperature management unit uses a high-efficiency motor, a heating element, and a solid-state temperature control to create a continuous flow of warm air to the blanket or gown. It is designed for safe use in all areas, including the operating room.

The Model 505 temperature management unit can be attached to an IV pole or to the railing on a bed.

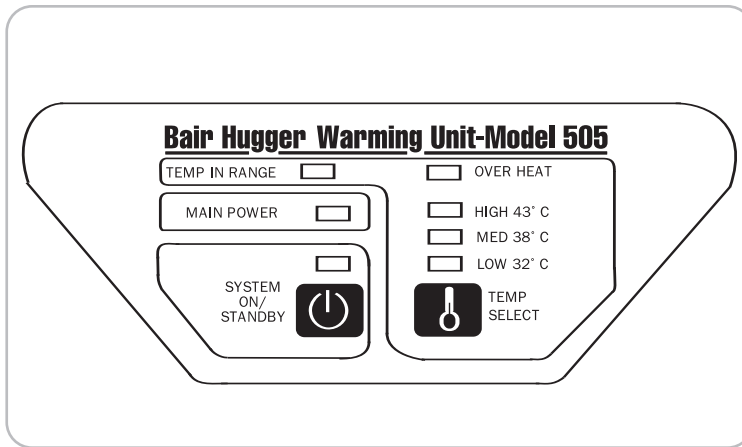
Figure D. Control panel of the Model 505 unit

Control Panel Features of the Model 505 Temperature Management Unit

TEMPERATURE IN RANGE INDICATOR

The temperature in range indicator illuminates when the output air temperature is within the range of the selected level.

MAIN POWER INDICATOR

The main power indicator illuminates when the unit is connected to a power source. This indicator must be illuminated for any functions to operate.

SYSTEM ON/STANDBY

Push this button to turn the unit either ON or OFF. The indicator directly above the switch illuminates when the unit is ON.

OVER HEAT INDICATOR

The Over Heat Indicator illuminates and an audible alarm sounds when an over-temperature condition is detected. To reset, turn the unit OFF and then ON, using the System ON/STANDBY button. (Also refer to the *Warnings* section of this manual.)

TEMPERATURE INDICATORS

The temperature indicators illuminate up to the selected temperature level. When the unit is initially turned on, none of these indicators are illuminated and ambient air will be delivered.

TEMPERATURE SELECT

Push this button to increase the temperature setting level by level to the desired setting. When the temperature setting is at HIGH, push the button again to return to delivery of ambient air.

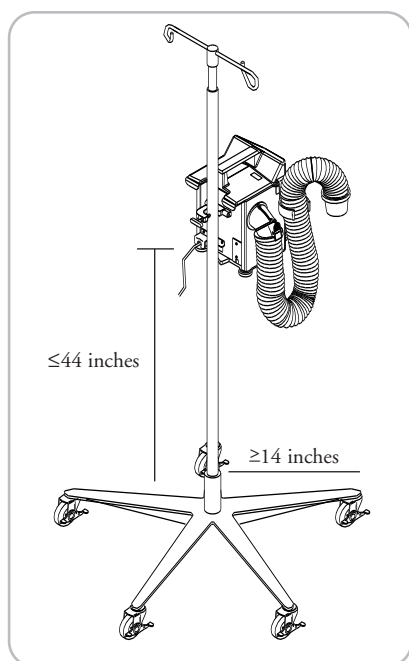
Figure E. Model 505 unit attached to an IV pole

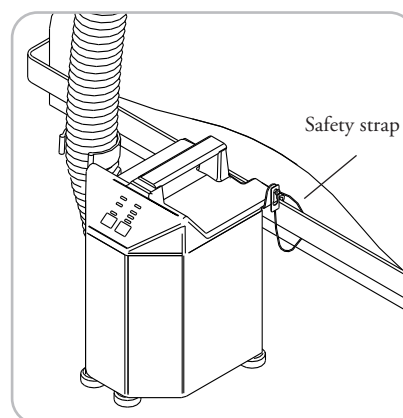
Figure F. Model 505 unit attached to a bedrail

Mounting the Model 505 Temperature Management Unit

USING AN IV POLE

The Model 505 unit clamps easily to an IV pole (see Figure E). Simply turn the handle clockwise to tighten the clamp onto an IV pole, counterclockwise to release.

Warning: To prevent tipping, clamp the Model 505 unit to an IV pole at a height that provides stability. We recommend clamping the unit no higher than 44" (112 cm) on an IV pole with a minimum 14" (35.6 cm) radius wheelbase. Failure to do so may result in IV pole tipping, catheter site trauma, and patient injury.

USING A BEDRAIL

The Model 505 unit can also hang on the edge of a bed. The safety strap is designed to loop around the bedrail, keeping the Model 505 unit safely suspended even if the unit is inadvertently dislodged from the bedrail (see Figure F).

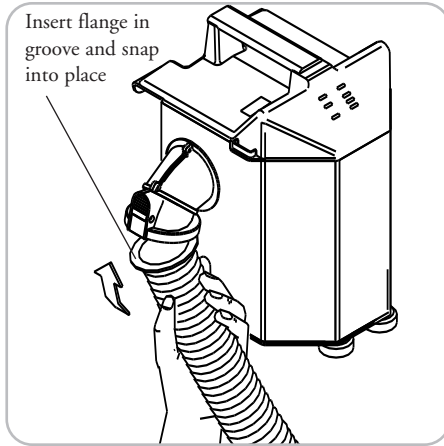
Figure G. Attaching the Model 505 unit hose

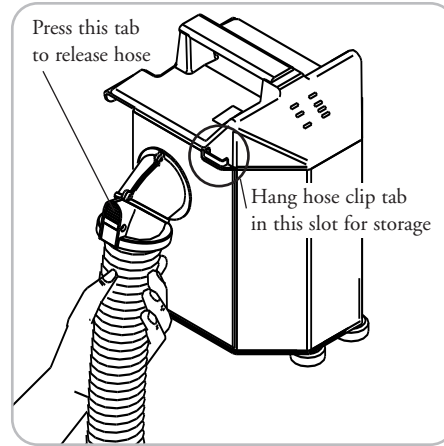
Figure H. Storing the Model 505 unit hose

Attaching and Storing the Model 505 Unit Hose

The Model 505 unit has a unique “snap-fit” hose. This extended-length swivel hose, adapted for 241 fluid warming, attaches by inserting the flange end at a 45° angle in the grooved blower outlet and “snapping” the hose into place.

Press the white tab on the blower outlet to release the hose.

When storing the Model 505 unit, insert the hose clip tab in the hanger slot near the blower outlet.

Blanket Dispenser

The blanket dispenser attaches to the unit and can store several Bair Hugger blankets. See the package instructions for details.

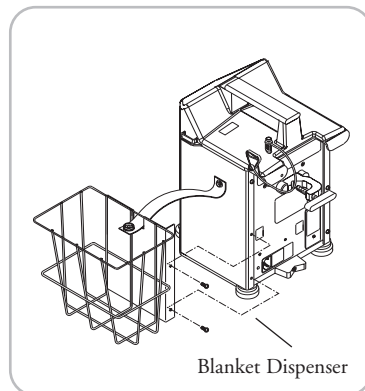
Figure I. Temperature management unit with blanket dispenser

General Maintenance

Cabinet Cleaning

- 1) Disconnect the temperature management unit from the power source before cleaning.
- 2) Use a damp soft cloth and a mild detergent to clean the unit cabinet. Dry with a separate soft cloth.

Caution

- *Do not use a dripping wet cloth to clean the cabinet. Moisture may seep into the electrical contacts, damaging the components.*
- *Do not use alcohol or other solvents to clean the cabinet. Solvents may damage the labels and other plastic parts.*

Technical Service and Order Placement

USA, WORLDWIDE

TEL: 1-952-947-1200
1-800-733-7775

FAX: 1-952-947-1400

WITHIN EUROPE

TEL: +49-4154-9934-0
0800-100-1236 (Toll-free in Germany)

FAX: +49-4154-9934-20
0800-100-1324 (Toll-free in Germany)

In-Warranty Repair and Exchange

Replacement parts to correct a problem are delivered at no charge. To return a device to Arizant Healthcare Inc. for service, first obtain a Return Authorization (RA) number from a technical service representative. Please use this number on all correspondence when returning a device for service. A shipping carton will be delivered to you at no charge, if needed. Call your local supplier or sales representative to inquire about loaner devices while your device is being serviced.

When You Call for Technical Support

Remember, we will need to know the serial number of your unit when you call us. On Model 505 units, the serial number label is affixed to the rear panel.

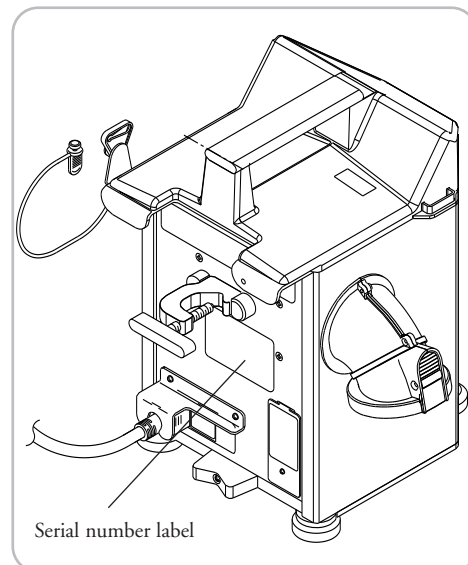
Figure J. Serial number label on Model 505

Specifications

Model 505

Physical Characteristics

DIMENSIONS	13 in. high x 10 in. deep x 11 in. wide 33 cm high x 25 cm deep x 28 cm wide
WEIGHT	13.6 lb; 6.2 kg
MOUNTING	IV pole clamp, bedrail hook with safety strap; can be placed on hard surface
RELATIVE NOISE LEVEL	53 decibels
HOSE	Detachable, flexible, washable; compatible with the 241 fluid warming system
FILTRATION SYSTEM	0.2µM level
RECOMMENDED FILTER CHANGE	Every 500 hours of use

Temperature Characteristics

TEMPERATURE CONTROL	Electronically controlled using a thermocouple sensor
HEAT GENERATED	1800 BTUs (average)
SYSTEM TIME TO 100°F (37.7°C)	~17 secs
OPERATING TEMPERATURES	<i>Air temperatures reaching the patient are approximately 2°C lower than the listed temperatures.</i> Average temperatures at the end of the hose: HIGH: 43° ± 3°C 109.4° ± 5.4°F MED: 38° ± 3°C 100.4° ± 5.4°F LOW: 32° ± 3°C 89.6° ± 5.4°F


Safety System

THERMOSTAT	Independent bulb and capillary
OVERCURRENT PROTECTION	Dual fused input lines
ALARM SYSTEM	Over-heat: flashing red light with audible alarm; heater shuts down
CERTIFICATIONS	UL 544, CSA C22.2 No. 125, IEC 601-1, IEC 601-1-2, EN55014, AS 3200.1990
CLASSIFICATION	Classified under IEC 601-1 Guidelines as Class I, Type BF, Ordinary equipment, Continuous operation

Electrical Characteristics

BLOWER MOTOR	Operating speed: 3150 rpm Airflow: 28-30 cfm
POWER CONSUMPTION	Peak: 1000W Average: 450W
LEAKAGE CURRENT	<100 μ A
HEATING ELEMENT	850W Resistive
POWER CORD	15-foot, SJT, 3 cond., 10A 4.6 meter, HAR, 3 cond., 10A
DEVICE RATINGS	110-120 VAC, 60 Hz, 9.5 Amperes, or 220-240 VAC, 50 Hz, 4.5 Amperes, or 100 VAC, 50/60 Hz, 10 Amperes
FUSES	10A, 200mA (110 - 120 VAC Units) 6.3A, 100mA (220 - 240 VAC Units) 15A, 160mA (100 VAC Units)
DIAGNOSTICS	Over-heat test can be performed by the biomedical group.

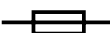Definition of Symbols

	ON/STANDBY
	ON (used on isolation switch)
	OFF (used on isolation switch)
	ON/OFF push button switch
	Temperature Control
	Equipotentiality plug (Ground)
	Fuse
	Attention (see appropriate documents)
	Non Explosion-Proof
	Dangerous Voltage
	Type BF Equipment (patient applied)
	Voltage, Alternating Current (AC)

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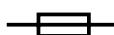
Sulake

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Authorized Representative in the European Community
(as defined in Article 14 of the Medical Device Directive: 93/42/EEC):
Actamed Limited, Calder Island Way, Wakefield, WF2 7AW,
United Kingdom • TEL + 44 1924 200550 • FAX + 44 1924 200518

©2003-2005 Arizant Healthcare Inc. All rights reserved.
10393 West 70th Street, Eden Prairie, MN 55344 USA
TEL 800-733-7775 • 952-947-1200 • FAX 800-775-0002 • 952-947-1400
www.arizanthealthcare.com

*Arizant, bright ideas that work, Bair Hugger, Bair Paws, 241, Total Temperature Management and the Bair Hugger and Arizant logos are trademarks of Arizant Healthcare Inc., registered or pending in the U.S. Patent & Trademark Office and in other countries.

Bair Hugger temperature management products may be covered by one or more of the following patents: US 6,355,915; 6,309,409; 6,309,408; 6,290,716; 6,287,327; 6,254,337; 6,241,755; 6,228,107; 6,210,428; 6,203,567; 6,176,870; 6,168,612; 6,146,412; 6,129,936; 6,126,681; 6,126,393; 6,036,722; 5,997,572; 5,968,084; 5,964,792; 5,928,274; 5,824,025; 5,800,489; 5,773,275; 5,733,318; 5,697,963; 5,674,269; 5,658,325; 5,620,482; 5,545,194; RE 38462; 5,350,417; 5,336,250; 5,324,320; 5,300,102; 5,300,101; 5,184,612; 5,044,364. Japan 2,561,326. Canada 1,325,484. EPO 0,311,336. Australia 756,900. Other patents pending.

200977C 04/05

Authorized Representative in the European Community
(as defined in Article 14 of the Medical Device Directive: 93/42/EEC):
Actamed Limited, Calder Island Way, Wakefield, WF2 7AW,
United Kingdom • TEL + 44 1924 200550 • FAX + 44 1924 200518

©2003-2005 Arizant Healthcare Inc. All rights reserved.
10393 West 70th Street, Eden Prairie, MN 55344 USA
TEL 800-733-7775 • 952-947-1200 • FAX 800-775-0002 • 952-947-1400
www.arizanthealthcare.com

*Arizant, bright ideas that work, Bair Hugger, Bair Paws, 241, Total Temperature Management and the Bair Hugger and Arizant logos are trademarks of Arizant Healthcare Inc., registered or pending in the U.S. Patent & Trademark Office and in other countries.

Bair Hugger temperature management products may be covered by one or more of the following patents: US 6,355,915; 6,309,409; 6,309,408; 6,290,716; 6,287,327; 6,254,337; 6,241,755; 6,228,107; 6,210,428; 6,203,567; 6,176,870; 6,168,612; 6,146,412; 6,129,936; 6,126,681; 6,126,393; 6,036,722; 5,997,572; 5,968,084; 5,964,792; 5,928,274; 5,824,025; 5,800,489; 5,773,275; 5,733,318; 5,697,963; 5,674,269; 5,658,325; 5,620,482; 5,545,194; RE 38462; 5,350,417; 5,336,250; 5,324,320; 5,300,102; 5,300,101; 5,184,612; 5,044,364. Japan 2,561,326. Canada 1,325,484. EPO 0,311,336. Australia 756,900. Other patents pending.

200977C 04/05