IVAC[®] PCAM[®] Syringe Pump

Directions For Use **en**





Page

Introduction
About This Manual
Features of the Pump
Controls & Indicators
Symbol Definitions
Operating Precautions
Getting Started
Basic Features
Alarms
Configured Options
Configured Options Record
Drugs and Protocols Record
Specifications
Maintenance
Trumpet Curves & Start-up Curves
Trumpet Curves & Start-up Curves continued
Service Contacts
Spare Parts

Introduction

The IVAC® PCAM® Syringe Pump (herein after referred to as "pump") system allows a patient to maintain a consistent level of pain relief by providing self administration of a clinician-prescribed dose of analgesic as and when it is required.

When the hand set is operated and the demand is within the parameters set by the clinician, the pump will automatically administer a precise bolus dose of analgesic.

For enhanced monitoring and management of post operative acute pain within the hospital, the pump provides convenient Patient Controlled Analgesia (PCA) and detailed information at the bed-side about the patients use of PCA.

Central to an effective pain service, the pump promotes improved pain management, more effective use of nursing resources, provides patient comfort and can contribute towards a quicker recovery.

Intended Use:

The pump is designed to meet the infusion requirements within the operating environment specified in this Directions For Use (DFU) including general wards, critical and intensive care, neonatal, operating rooms and accident and emergency rooms.

This pump is suitable for use by appropriately trained clinicians or nurses. The pump is suitable to deliver fluids and medications via intravenous, epidural and subcutaneous routes. Supporting fluid therapy, blood transfusions and parenteral feeding.

The pump is compatible with a wide range of standard, single-use, disposable Luer lock syringes. It accepts syringe sizes from 20 ml to 100 ml. See the 'Compatible Syringes' section for a full list of compatible syringes.

- User configured PCA protocols.
- Comprehensive history.
- Large graphics format display.
- Two key positions providing separation of nursing and programming procedures.
- 10 pre-set hospital PCA protocols.
- Unique electronic hand set with status indicator.
- Communications and nurse call interfaces.

About This Manual

It is important to ensure that you only refer to the most recent version of the Directions for Use and Technical Service Manual for your CareFusion products. These documents are referenced on www.carefusion.com. Copies can be obtained by contacting your local CareFusion representative.

The user must be thoroughly familiar with the IVAC PCAM Syringe Pump described in this manual prior to use.

All illustrations used in this manual show typical settings and values which may be used in setting up the functions of the pump. These settings and values are for illustrative use only. Where stated, a minimum infusion rate refers to a nominal rate of 1.0ml/h, and an intermediate infusion rate refers to a nominal rate of 5.0ml/h. The complete range of infusion rates, settings and values are shown in the Specifications section.

Features of the Pump



Controls & Indicators

Controls:

Symbol	Description
©	START button - Press to start the infusion. The green LED will flash during infusion.
	STOP button - Press to put the infusion on hold. The amber LED will be lit while on hold.
	PURGE button - Press and hold both buttons to purge the extension set during set up. See 'Basic Features' for further information.
	HISTORY button - Press to display PCA demands and drug infused history graphs, 24 hour review and event log.
	PRINT button - Press to print patient history. Note: A suitable printer must be connected to the pump.
	PLUS/MINUS buttons - Use to move cursor and to increase or decrease values shown on main display.
	ARROW buttons - Use as softkeys in conjunction with the prompts shown on the display.
OFF OFF SET RUN	 LOCK 1 - Insert key into LOCK 1 keyswitch and turn key to switch between OFF, SET and RUN positions. OFF - Turns the power off. SET - Use to select or modify protocols and to access configuration and test routines. RUN - Use to start the infusion. Note: Switching from RUN mode to SET mode without first pressing the STOP button automatically stops the infusion.
	LOCK 2 - Insert key into LOCK 2 and turn key clockwise to open the syringe cover. (This key lock is located on the left side of the pump)

Indicators:

Symbol	Description
+0-	BATTERY indicator - When illuminated the pump is running on the internal battery. When flashing the battery power is low with less than 30 minutes of use remaining.
€QI	AC POWER indicator - When illuminated the pump is connected to an AC power supply and the battery is being charged.

Symbol Definitions

Labelling Symbols:

Symbol	Description
	Attention (Consult accompanying documents)
\bigvee	Potential Equalisation (PE) Connector
	RS232/Nurse call Connector (Optional)
	Class II Equipment
	Type CF applied part (Degree of protection against electrical shock)
IPX4	Protected against splashing fluid (degree of protection against fluid ingress)
\sim	Alternating Current
C E 0086	Device complies with the requirements of Council Directive 93/42/EEC as amended by 2007/47/EC.
	Date of Manufacture
	Manufacturer
	Not for Municipal Waste
	Important information
EC REP	Authorised representative in the European Community

Main Display



Protocol Summary Screen



* These icons are not displayed when disabled.

Screen Icons:

Symbol	Description
	PUMPING PRESSURE icon - When enabled, this icon is shown on the Display. It provides a visual indicator of current pumping pressure and pressure level at which the alarm will operate.
	MAXIMUM DOSE icon - When enabled, this icon is shown on the Display. It provides a visual indication of the amount of drug administered during the limit period (as shown to the left of the icon). If the dose limit reaches the alarm level, the bottle icon will appear full, the pump will stop infusing and the message Max Dose Limit is displayed and the icon will flash until the dosing is less than the maximum dose limit. Clinician over-ride is always available however limited by the Max Dose and the time frame set up in the drug protocol.
•	BACK icon - Indicates the softkey to press to go back to previous screen.

Operating Precautions

include closing a tap in the extension set or activating a flow stop clamp.

Disposable Syringes and Extension Sets







removed from the pump before the extension set is properly isolated from the patient. Isolation may

This pump has been calibrated for use with single-use disposable syringes. To ensure correct and accurate operation, only use 3 piece Luer lock versions of the syringe make specified on the pump or

- The user must be thoroughly familiar with instructions in this Directions For Use and understand how to load and confirm the syringe on the Pump. Incorrect syringe loading may result in misidentification of the syringe type and size leading to significant under or over infusion.
- When combining several apparatus and/or instruments with extension sets and other tubing, for example via a 3-way tap, the performance of the pump may be impacted and should be monitored closely.

Mounting the Pump

Do not mount the pump in a vertical position with the syringe pointing upwards as this could lead to an infusion of air which may be in the syringe. To protect against the introduction of air the user should regularly monitor the progress of the infusion, syringe, extension set and patient connections and follow the priming procedure specified herein.

Operating Environment

- When using any infusion pump in conjunction with other pumps or devices requiring vascular access, extra care is advised. Adverse delivery of medication or fluids can be caused by the substantial variation in pressures created within the local vascular system by such pumps. Typical examples of those pumps are used during dialysis, bypass or cardiac assist applications.
- This pump is suitable for use in Hospital and clinical environments other than domestic establishments and those directly connected to the public single phase AC mains power supply network that supplies buildings used for domestic purposes. However, it may be used in domestic establishments under the supervision of Medical professionals with additional necessary appropriate measures. (Consult Technical Service Manual, appropriately trained technical personnel or CareFusion for further information).
- This pump is not intended to be used in the presence of a flammable anaesthetic mixture with air or oxygen or nitrous oxide.

Operating Pressure

- This is a positive pressure pump designed to achieve very accurate fluid administration by automatically compensating for resistance encountered in the infusion system.
- The pumping pressure alarm system is not designed to provide protection against, or detection of, IV complications which can occur.

Alarm Conditions

Several alarm conditions detected by this pump will stop the infusion and generate visual and audible alarms. Users must perform regular checks to ensure that the infusion is progressing correctly and no alarms are operating.





Electromagnetic Compatibility & Interference

- This pump is protected against the effects of external interference, including high energy radio frequency
 emissions, magnetic fields and electrostatic discharge (for example, as generated by electrosurgical and
 cauterising equipment, large motors, portable radios, cellular telephones etc.) and is designed to remain safe
 when unreasonable levels of interference are encountered.
- **Therapeutic Radiation Equipment:** Do not use the pump in the vicinity of any Therapeutic Radiation Equipment. Levels of radiation generated by the radiation therapy equipment such as Linear Accelerator, may severely affect functioning of the pump. Please consult manufacturer's recommendations for safe distance and other precautionary requirements. For further information, please contact your local CareFusion representative.
- Magnetic Resonance Imaging (MRI): The pump contains ferromagnetic materials which are susceptible to
 interference with magnetic field generated by the MRI devices. Therefore, the pump is not considered an MRI
 compatible pump as such. If use of the pump within an MRI environment is unavoidable, then CareFusion
 highly recommends securing the pump at a safe distance from the magnetic field outside the identified
 'Controlled Access Area' in order to evade any magnetic interference to the pump; or MRI image distortion.
 This safe distance should be established in accordance with the manufacturers' recommendations regarding
 electromagnetic interference (EMI). For further information, please refer to the product technical service
 manual (TSM). Alternatively, contact your local CareFusion representative for further guidance.
- Accessories: Do not use any non-recommended accessory with the pump. The pump is tested and compliant with the relevant EMC claims only with the recommended accessories. Use of any accessory, transducer or cable other than those specified by CareFusion may result in increased emissions or decreased pump immunity.
- This pump is a CISPR 11 Group 1 Class A device and uses RF energy only for its internal function in the normal product offering. Therefore, its RF emissions are very low and are not likely to cause any interference with the nearby electronic equipment. However, this pump emits a certain level of electromagnetic radiation which is within the levels specified by IEC/EN60601-1-2 and IEC/EN60601-2-24. If the pump interacts with other equipment, measures should be taken to minimise the effects, for instance by repositioning or relocation.



In some circumstances the pump may be affected by an electrostatic discharge through air at levels close
to or above 15kV; or by radio frequency radiation close to or above 10V/m. If the pump is affected by this
external interference the pump will remain in a safe mode; the pump will duly stop the infusion and alert the
user by generating a combination of visual and audible alarms. Should any encountered alarm condition
persist even after user intervention, it is recommended to replace that particular pump and quarantine the
pump for the attention of appropriately trained technical personnel. (Consult Technical Service Manual for
further information).

Hazards

- An explosion hazard exists if the pump is used in the presence of flammable anaesthetics. Exercise care to locate the pump away from any such hazardous sources.
- Dangerous Voltage: An electrical shock hazard exists if the pump's casing is opened or removed. Refer all servicing to qualified service personnel.
- Do not open the RS232 protective covering when not in use. Electrostatic discharge (ESD) precautions are required when connecting RS232. Touching the pins of the connectors may result in ESD protection failure. It is recommended that all actions must be taken by appropriately trained personnel.



If this pump is dropped, subjected to excessive moisture, fluid spillage, humidity or high temperature, or
otherwise suspected to have been damaged, remove it from service for inspection by a qualified service
engineer. When transporting or storing the pump, use original packaging where possible, and adhere to
temperature, humidity and pressure ranges stated in the Specifications section and on the outer packaging.

Getting Started

Initial Set-up



Before operating the pump read this Directions For Use manual carefully.

- 1. Check that the pump is complete, undamaged and that the voltage rating specified on the label is compatible with your AC power supply.
- 2. Items supplied are:
 - IVAC PCAM Syringe Pump
 - User Support CD (Directions For Use)
 - AC Power Cable (as requested)
 - Protective Packaging

3. Connect the pump to the AC power supply for 24 hours to ensure that the internal battery is fully charged (verify that the 🕫 is lit).



The pump will automatically operate from its internal battery if the pump is switched on without being connected to the AC power supply. Prior to use on battery power, verify the pump continues to function on battery power once disconnected from the AC power supply.

Should the pump fail to perform correctly, replace in its original protective packaging, where possible and contact a qualified service engineer for investigation.



Do not mount the pump with the AC power inlet or the syringe pointing upwards. This could affect the electrical safety in the event of a fluid spill or lead to the infusion of air which may be in the syringe.

Pole Clamp Installation

The pole clamp is supplied fitted to the rear of the pump and will provide secure fixing to standard I.V. poles of a diameter of up to 40mm.

The pole clamp can also be fitted in a choice of 4 fixing positions allowing the pump to be mounted to vertical and horizontal poles, equipment rails and hospital furniture in a variety of convenient operating orientations.

The pole clamp may be adjusted for use with horizontal fittings by using the existing fixings screws with the alternative fixing holes in the pole clamp.

The pole clamp may also be secured to the base of the pump in a choice of four positions.

Patient Hand Set

The patient hand set supplied with the pump is designed to be ambidextrous and suitable for both adult and paediatric use. The hand set provides an indicator light which clearly shows when the pump is available and can be configured to flash when a PCA dose is being delivered.



The indicator on the patient hand set will reflect the configuration of the pump and will provide feed-back on all, or just good demands, and the indicator light can be disabled should the clinical situation require.

Where appropriate the hand set can be configured so that the patient will not need to refer to the instrument to assess if PCA is being delivered, or is available.

The hand set is provided with a clip for attaching it to bedding or clothing.

The pump concept is that the patient can be instructed in the use of the hand set as it will carry all the information required by the patient using PCA. This design simplifies patient instruction and encourages a smooth transfer to alternative devices used to treat long term chronic pain, should this be indicated.



A latching (but non locking) connector makes the hand set easy to fit. To remove, hold the body of the connector and pull away from the pump.

An alarm warning will operate if the hand set is disconnected from the pump while it is in operation or the hand set is connected to the unit with the PCA button depressed. In addition, the pump can be operated in continuous or clinician over-ride modes without the hand set connected, should this be required.

Getting Started (continued)

Loading a Syringe

Warning: To securely load and confirm a syringe carefully follow the steps below. An incorrect loading of a syringe may result in misidentification of the syringe type and size. If then confirmed, this may lead to significant inaccuracy of the infusion rate and may also affect pump performance.

Only use a syringe of the type stated on the pump or in this manual. Using an incorrect syringe could adversely affect the accuracy of the infusion and the performance of the pump.

When initially loading the syringe, allow for the volume of fluid contained in the extension set and retained in the syringe at the end of infusion as this "dead-space" will not be infused.

Place the pump on a stable horizontal surface or secure as described above.

Prepare, load and prime the single-use disposable syringe and extension set using standard aseptic techniques.

- 1. Open the cover by turning key in Lock 2.
- 2. Squeeze the finger grips on the plunger holder and slide the mechanism to the left. Lift the syringe clamp and rotate clockwise.
- 3. Insert the syringe into the slots on the plunger holder.
- 4. Squeeze the finger grips on the plunger holder and slide the mechanism to the right until the syringe finger flanges locates in the V slot.



Gently advance the syringe until the finger flanges touch the front of the V slot closest to the syringe clamp. This is important to prevent delay at the start of the infusion.

- 5. Rotate the syringe clamp anticlockwise until it locks onto the syringe barrel to secure the syringe.
- 6. Check that the syringe plunger and finger flanges are correctly located in their slots.



7. Check that the syringe type and size being used matches display. If required, the make of syringe can be changed by pressing the CHANGE TYPE softkey. Press OK to confirm syringe.



To minimise the risk of incorrect confirmation of the syringe type it is recommended that the default syringe type should be locked to only the one brand type used in the hospital where possible.

Changing a Syringe during an infusion

When changing the syringe LOCK 1 should remain in the RUN position except when a change is required to the protocol.

- 1. Press the 🖫 button to halt the infusion and place the pump on hold.
- 2. Open the cover using LOCK 2.
- 3. Close the extension line to the patient.
- 4. Change the syringe, fitting the new syringe as per instructions above.
- 5. Follow steps 7 to 10 of 'Starting the Pump' section on the next page.







Starting the Pump

 AC POWER - Connect pump to AC power supply using the AC power cable. Note: the pump will operate on an internal battery when not connected to an AC power source for a limited time.



Prior to beginning an infusion, disconnect the pump from the AC power supply, confirm the pump continues to function on battery power. Then reconnect the pump to the AC power source.

- 2. SET Insert key into LOCK 1. Turn to SET position.
- 3. **NEW PATIENT?** Answering **NO** will retain all previous patient history, except for the PCA lockout time remaining which will be reset to PCA AVAILABLE. **YES** will automatically reset the patient history to zero. Check time and date is correct and answer **YES** or **NO**.
- SELECT/ MODIFY PROTOCOL Carefully check the protocol displayed. If required, press MODIFY PROTOCOL to adjust the current protocol, or, NEXT PROTOCOL to select an alternative pre-set protocol.
- 5. RUN Turn **LOCK 1** to the **RUN** position and remove the key from pump.
- 6. CONFIRM PROTOCOL Carefully check that protocol is correct. Press **OK**.
- 7. CONFIRM SYRINGE Check that the syringe type and size being used matches display. If required, the make of syringe can be changed by pressing the **CHANGE TYPE** softkey. Press **OK**.
- 8. PURGE (if required) The **PURGE** buttons can only be used when the cover is open and **LOCK 1** is in the **RUN** position. When the purge operation is complete close the cover.
- 9. CONNECT PATIENT Connect the PCA extension set to the patient access device. Recheck the protocol.
- 10. START Press the ^(C) button to commence pump operation. Either **PCA AVAILABLE** or **CONTINUOUS INFUSION** will be displayed with the rate, demand and drug totals. If selected, a loading dose will be delivered.
- 11. PROTOCOL Press **PROTOCOL** softkey at any time to display the protocol summary. To return to the main screen press **QUIT**.

The pump will automatically operate from its internal battery if the pump is switched on without being connected to the AC power supply. Each time the pump is switched ON, check that the alarm beeps twice and that all the segments of the display, the green and amber lights are illuminated during the self test routine. LOCK 1 should not be turned from OFF to SET whilst the syringe extension set is connected to the patient.











Basic Features

Purge

The 🖅 button allows the delivery of a limited volume of fluid in order to purge the extension set prior to being connected to a patient.

- 1. **PURGE** can only be operated with the cover opened and **LOCK 1** in the **RUN** position.
- 2. Press the 2 method buttons together until fluid flows and priming of the syringe extension set is complete. The audible alarm will operate during use of the 2 methods and the volume used during priming will be shown in the volume infused display.





The PURGE feature is not activated when the cover is closed. Ensure that the extension set is disconnected from the patient before purging the line. No alarms are disabled during the operation of the PURGE feature. During PURGE the pressure limit alarms are temporarily increased to their maximum level.

Clinician Over-ride

The clinician over-ride feature can be used in RUN mode to administer an additional bolus dose or continuous background infusion of a limited dose and duration, for example during the PCA lock out period. The clinician over-ride is a special feature which can be configured according to the specific clinical situation. Clinician over-ride can also be used in SET mode to allow modification of the pre-set PCA Protocol when this option has been disabled for normal use.

- 1. Turn **LOCK 1** to **RUN** position and ensure green light is illuminated on the ^(C) button.
- 2. Press and hold down the 🖾 button for 2 seconds.
- 3. Use B buttons and **NEXT** softkey to enter three figure pre-programmed clinician access code. See technical service manual.
- 4. Select BOLUS or CONTINUOUS.
- 5. Use 🗟 🖻 buttons to select the dose delivered, when the correct value has been entered press **OK**.
- 6. Use 🗟 😁 buttons to select period over which the dose is to be delivered. Press **OK** when correct time has been entered.
- 7. **BEGIN BOLUS? YES** Clinician bolus / continuous infusion will be delivered to the patient. **NO** Quit set up and return to normal operation.





The delivery of the clinician over-ride continuous infusion will automatically halt while a Patient or Clinician over-ride bolus is being administrated. To cancel clinician over-ride during delivery, press 🖺 button and press the YES softkey.

Basic Features (Continued)

Patient History

Each time the pump is switched ON it will ask if this is a new patient. Pressing **YES** will provide opportunity to re-set patient history. Pressing **NO** continues with the current protocol and retains all protocol records, event history, graphs etc. However, any remaining PCA lockout time will be cleared and a PCA demand will be immediately available.

The pump will retain the events in a rolling memory. Following selection of a new patient, it remains possible in technician mode to access previous patient(s) history still held in memory.

Patient history can be accessed at any time by pressing the im button. The pump provides a clear rolling 24 hour graphical representation of the PCA demand pattern and the drug administered to the patient. The graphs are updated when the history button is pressed and give values for each completed hour and the current hour. The cumulative counters on the Drug Infused history screen update in real-time.

24 Hour Review

An hour by hour record of the last 24 hours, showing good and total PCA demands along with the total dose and volume per hour. This information provides the accurate demand pattern and drug infused values from which the other graphs are derived.

- 1. To access the 24 Hour Review press the 🖮 button once.
- 2. To return to the previous screen press the **BACK** softkey.
- 3. To scroll through to the next History screen press the \fbox button.

HOUR	GOOD	(TOTAL)	MASS/h	VOL/h
13:00 - 13:59 14:00 - 14:59 15:00 - 15:59 16:00 - 16:59 17:00 - 17:59 18:00 - NOW	859756	(15) (7) (11) (14) (8) (7)	8.0 ug 5.0 ug 9.0 ug 7.0 ug 5.0 ug 6.0 ug	8.0 ml 5.0 ml 9.0 ml 7.0 ml 5.0 ml 6.0 ml
				<

PCA Demands

Provides a record of the last 24 hours good and failed PCA demands. The good demands are indicated by the shaded section of the graph and the failed demands by the clear section. The latest hour is shown at the right side of the display.

This graph provides a clear picture of good and bad PCA demands and pattern of the patients usage. Used in conjunction with the PCA Demands graph, this display helps to indicate if the PCA protocol needs modification and when to end treatment.

- 1. To access the PCA demand graph press the 🗎 button twice.
- 2. To exit the screen press the **BACK** softkey.
- 3. To scroll through to the next History screen press the 🖮 button.



Drug Infused

Record of the total amount of drug administered to the patient over the last 24 hours. The latest hour is shown at the right side of the display. This graph provides a clear picture of the actual drug administered, including loading dose, continuous background infusions, clinician over-rides and protocol changes. The left side of the display shows cumulative dose and time counters with the time, mass and volume infused since the patient session started or since the counters were last reset. To reset counters press **CLEAR** and then **CONFIRM**.

Used in conjunction with the PCA Demands graph, this display helps to indicate relative pattern of the demand pattern and the actual drug administered.

- 1. To access the Drug Infused graph press the 🖮 button three times.
- 2. To return to the previous screen press the **BACK** softkey.
- 3. To scroll through to the next History screen press the 🖮 button.



Event Log

Record of events since "**NEW PATIENT**" selected. Including, protocol selection and changes, patient demands etc. The event log will also record all alarms.

All events are recorded against date, clock and total drug infused.

- 1. To access the Event Log press the 🖮 button four times.
- 2. To return to the previous screen press the **BACK** softkey.
- 3. To return to the Protocol Summary press the 🖮 button again.



Printing

A printer fitted with a serial interface (or cable with parallel to serial converter) can be connected to the pump, either during normal PCA operation or following use. Printing patient history provides a permanent record and can be used for analysis away from the bedside.

All patient history, including protocols and the 24 hour demand pattern and drug dose administered graphs are available for printing.

When connected to the printer, the pump can also be configured to provide line by line continuous printing of all events, patient demands etc. as they occur at the bedside. See General Options.

Print	Connect Printer then	Information printed
Protocol Summary	• Turn LOCK 1 to SET position • Press	All protocol information will be printed with patient header.
Patient History	• Press 🖲 button	All protocol information, demand and drug totals, 24 hour graphs and records will be printed with patient header.
Event Log	 Press button until event log is displayed. Use buttons to position display at start point for events to be printed. Press button 	All events will be printed from information on screen forward with time, date and patient header.
Event Log at New Patient	• Press 🔳 button	All events will be printed from the patient event log.

Continuous Mode

Enable Continuous printing by selecting **YES** in General Options.

- 1. Connect printer.
- 2. All events will be printed as they occur.

Teach Learn Mode

By programming the configuration of one pump in the conventional way from the front panel buttons other pumps can have the configuration copied over using the "teach" and "learn" modes.

When set to "learn" mode the pump will accept information sets from a pre-configured pump set to "teach" mode.

When set to "teach" mode the pump sends out via the communications interface a sequence of all the information sets required to configure another pump. To fully configure a pump it is necessary to send 22 complete information sets as described in the protocols and the full cycle takes about 22 seconds.

The two pumps must be connected together using an RS232 Demonstration Cable. Both pumps must be of the same version software and revision and set to a common pump comms identification number (see General Options). The configured pump is set to "teach" mode and the pump to be configured is set to" learn" mode using the access codes listed in the Technical Service Manual. The pump in "learn" mode will display **PASS** or **FAIL** for each information set being received from the "teach" mode device.

The pumps must run through at least one complete sequence of the information sets and then switch off first the "learn" pump and then the "teach" pump. *After using this method it is the user's responsibility to check that the configuration has been copied over correctly.*

Pre-Set PCA Protocols

Operation of the pump is greatly simplified by the use of PRE-SET PCA protocols. When **LOCK 1** is turned to the **SET** position the pump will automatically display pre-set **PROTOCOL A** if **NEW PATIENT** has been selected or display the previous protocol in use if **NEW PATIENT** has not been selected.

With LOCK 1 in the SET position, it is possible for the user to modify the pre-set protocol using the MODIFY PROTOCOL button and select another pre-set protocol using the NEXT PROTOCOL button.

To modify a Pre-Set Protocol

- 1. Press MODIFY PROTOCOL indicated on the screen.
- 2. The protocol summary will be displayed. Use [△] buttons to highlight a field, press **ALTER** to enter that field and ^ᢙ [⊕] buttons to select desired values.
- 3. When field is correct press CONFIRM or CANCEL.
- 4. Display will show:

Δ

Protocol		
Drug name	Drug 'X'	
Drug conc.	1.0mg/ml	
PCA Dose	1.0mg	(1.0ml)
Lockout period	5 mins	
Occlusion level	4	
Continuous	0μg/h	(0.0ml/h)
Loading dose	Ομg	(0.0ml)
Max. limit	50mg	(50.0ml)
Limit duration	4 hours	
PCA delivery	STAT	

PROTOCOL DRUG X SUMMARY 1.0mg/ml PCA Dose Lockout Continuous Loading 1.0 ml 5 mins 2.0ml/h 2.0 ml MAX 50.0ml MODIFY NEXT PROTOCOL PROTOCOL IN 4h

5. Press **OK** to return to display.

Notes:

A modified protocol has no pre-set letter.

To utilise this function, "MODIFY PROTOCOL" must be enabled in GENERAL OPTIONS.

When changing a Drug name the protocol defaults associated with the original drug name remain, however the safety limits will change to those of the new drug name. This may cause the safety limits to be set to their widest parameters. The following warning message is displayed and the user must confirm this change. PROTOCOL DRUG NAME -- DRUG Y --WARNING: CHANGING DRUG NAME WILL ALTER SAFETY LIMITS FOR PROTOCOL. CONTINUE? CONFIRM QUIT

To change to another Pre-Set Protocol

Press **NEXT PROTOCOL** to display the step through the pre-set protocol.

Pre-set protocols are identified as A to J. When the desired protocol has been selected it can be used by turning **LOCK 1** to the **RUN** position, or, can be modified using the **MODIFY PROTOCOL**.

To confirm a Protocol

Whenever a new protocol is selected, modified or **LOCK 1** is turned to the **SET** position, the **CONFIRM PROTOCOL** display will appear. Pressing **OK** automatically records the protocol and any changes in the **EVENT LOG** history.



Volumes and rates are typically calculated to more decimal places than are displayed. To achieve this, the values are rounded down to prevent the impression of an over-infusion. The doses and dose rates are calculated from the volume of rate that the pump infuses. Thus, when configuring protocols some combinations of drug concentration and PCA dose may result in the displayed rate and the dose rate appearing inconsistent or the displayed dose appearing incorrect.

See examples on next page.

Pre-Set PCA Protocols (continued)

Example 1. Displayed rate and dose rate appear inconsistent.

Dose rate = 30µg/h Drug Concentration = 44µg/ml

This would give a rate of 0.68181818ml/h. The pump would round the calculation to two decimal places, giving an actual infusion rate of 0.68ml/h, whilst the screen will display a rate of 0.6ml/h and a dose rate of 29µg/h.

Example 2. Displayed dose does not appear correct.

PCA Dose = 720µg

Drug Concentration = 500µg/ml

For each PCA demand the pump would infuse 1.4399ml. For the purpose of the displayed volume, the first PCA demand is counted as 1.43ml (715 μ g) with the remaining 0.0099ml being carried over into the display of the next demand. The next demand would be displayed (accumulatively) as 2.87ml (1.4399ml + 1.4399ml = 2.8798ml) with the remaining 0.0098ml being carried over into the display of the next demand.

For the purpose of the displayed dose, the first PCA demand is calculated as 715 μ g (drug concentration 500 μ g/ml multiplied by calculated dose 1.43ml). The next demand would be displayed (accumulatively) as 1.43mg (500 μ g/ml multiplied by calculated dose 2.87ml = 1435 μ g = 1.435mg which is rounded down and displayed to two decimal places = 1.43mg).

If similar inconsistencies are experienced, contact your local CareFusion representative for clarification.

Alarms

Alarms are indicated by a combination of an audible alarm, flashing amber STOP light and a descriptive message in the display.

- 1. First press the **MUTE** softkey to silence the alarm for a maximum of 2 minutes, then check the display for an alarm message. Press 🗳 to cancel the alarm message.
- 2. If the infusion has stopped, rectify the cause of the alarm then press the \mathbb{C} button to resume the infusion.

Message	Cause and Troubleshooting Guide
DRIVE DISENGAGED	PUMP DRIVE DISENGAGED The drive system has been disengaged during operation. Use LOCK 2 to unlock and open the cover. Check the finger grips and the position of the syringe.
COVER OPENED	COVER OPENED DURING OPERATION The cover has been opened during operation. Check cover and LOCK 2 .
LINE OCCLUSION	EXCESSIVE PUMPING PRESSURE Pumping pressure has reached the alarm limit. Use LOCK 2 to unlock and open the cover, squeeze finger grips on the plunger holder to release the drive mechanism and relieve any excessive pressure in the syringe and patient line. Identify and remove the cause of the blockage in the drive, syringe, or administration system before restarting the infusion.
SYRINGE ERROR	SYRINGE SIZE ERROR / FITTED INCORRECTLY Incorrect size of syringe has been fitted, the syringe clamp has not been positioned correctly on the syringe or has been disturbed during operation or plunger is not fitted in plunger slot. Use LOCK 2 to unlock and open the cover, check syringe size, position of syringe clamp, syringe and plunger.
CHECK HANDSET	PATIENT HAND SET FAILURE Patient hand set has become faulty or disconnected during operation. Check operation and connection of the hand set to the pump. Press ^(C) to continue if operation without the hand set is required.
BATTERY LOW	BATTERY CHARGE LOW WARNING Battery charge low with 30 minutes operation remaining. Battery indicator will flash and after 30 minutes a continuous audible alarm will indicate that the battery is exhausted. Reconnect to AC power supply to continue operation and charge internal battery.
BATTERY EXHAUSTED	BATTERY EXHAUSTED Internal battery exhausted. To silence the alarm switch LOCK 1 to the OFF position and reconnect pump to AC power supply. Restart operation on AC power whilst charging the internal battery. Switch to the RUN position.
SYRINGE NEAR EMPTY	NEAR END OF SYRINGE WARNING Syringe almost empty with about 6% of its volume remaining. Press ^(P) to silence alarm and continue operation. Display will flash SYRINGE NEAR EMPTY . The alarm screen will also flash USE LOCK 2 TO OPEN COVER.
SYRINGE EMPTY	SYRINGE EMPTY - END OF INFUSION The pump has reached the end of the infusion. About 1% of the syringe volume will remain in the syringe helping to prevent the infusion of air bubbles into the PCA set. The alarm screen will flash USE LOCK 2 TO OPEN COVER.
AC POWER FAIL	AC POWER SUPPLY DISCONNECTED WARNING AC Power has been disconnected and the pump is operating on battery power. Reconnect AC power supply or press 🕄 to silence the alarm and continue battery operation. The display will light up ON BATTERY . The alarm will automatically cancel if the AC power supply is reconnected.
MALFUNCTION	INTERNAL MALFUNCTION The alarm system has detected an internal malfunction. Note the malfunction code. Remove pump from service for examination by a qualified service engineer.
MAX DOSE EXCEEDED	MAX DOSE LIMIT The maximum dose over time limit has been exceeded. Infusion is stopped when the alarm occurs. Press 聞 to cancel the alarm. The max dose icon will flash until dosing is less than the maximum dose limit. Note that this alarm can be disabled under General Options.
3 BEEPS (Alarm without screen prompt)	NURSE ATTENTION WARNING Pump left switched ON for more than 2 minutes without starting operation. Press 🗳 or any of the control buttons to silence the alarm for a further 2 minutes. To cancel this alarm for 15 minutes, press and hold the 🗳 button until 3 rapid, consecutive beeps are heard.

Configured Options

This menu comprises a list of options which are configurable by the user.

- 1. Whilst holding down the 🖱 button turn the pump **ON**.
- 2. The main display will show **000**. Enter the access code for Configured Options using the 🙆 🖻 buttons, pressing **NEXT** to move through the digits. A full list of access codes can be found in the Technical Service Manual.
- 3. When the complete code shows on screen, press ENTER. The Configured Options menu will be displayed.

General Options

- 1. Select **GENERAL OPTIONS** from the menu using the B buttons and press **ENTER**.
- 2. Select the option you wish to enable/disable or adjust and press MODIFY.
- 3. When all the desired modifications have been carried out press QUIT.
- 4. Either select the next configuration option from the menu or turn the pump **OFF**, returning it to operation as required.

1. ICONS on display YES - displays Pumping Pressure and Maximum Dose ICONS. NO - ICONS disabled. 2. Protocols in use PCA pre-set protocols to be available. Select number from 1 to 10. 3. Modify protocol YES - allows protocols to be modified in SET mode. NO - removes modify protocol so be modified in SET mode. 4. Handset mode MODE A B C BEEP GOO ALL ALL HAND SET LIGHT: PCA MSTOPPED OFF OFF OFF OF PCA DELIVERING FLASH ON ON ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the CB builton for 4 seconds. 6. Display Sleep YES - chipp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm is up to a calibable. 9. Loading doses YES - cloading dose option a ppears in protocols. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - continuous infusions are not available. <t< th=""><th>Option</th><th>Description</th><th></th><th></th><th></th></t<>	Option	Description			
NO - ICONS disabled. 2. Protocols in use PCA pre-set protocols to be available. SET mode. Nodify protocol YES - allows protocols to be modified in SET mode. NO - removes modify protocol option in SET mode. NO - removes modify protocol option in SET mode. 4. Handset mode MODE A B C BEEP GOOD ALL ALL HAND SET LIGHT: PCA MYALABLE ON ON PCA PCA DELIVENING FLASH ON FLASH PCA PCA IOCKOUT OFF ON ON ON S. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the Call button for 4 seconds. 6. Display Sleep YES - Chipp alarm is enabled, replacing the AC POWER FALL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FALL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FALL alarm and the SYRINGE NEAR EMPTY alarm. NO - Chirp alarm is disabled. The AC POWER FALL alarm and the SYRINGE NEAR EMPTY alarm. R. Continuous infusions YES - Continuous infusions are not available. NO - Chirp alarm is disabled. The AC POWER FALL alarm and the SYRINGE NEAR EMPTY alarm. No - Chirp alarm is disabled. <td< td=""><td>1. ICONS on display</td><td colspan="3">YES - displays Pumping Pressure and Maximum Dose ICONS.</td><td>imum Dose ICONS.</td></td<>	1. ICONS on display	YES - displays Pumping Pressure and Maximum Dose ICONS.			imum Dose ICONS.
2. Protocols in use PCA pre-set protocols to be available. Select number from 1 to 10. 3. Modify protocol YES - allows protocols to be modified in SET mode. A. Handset mode MODE A B C BEEP GOOD ALL ALL ALL HAND SET LIGHT: PCA MAILABLE ON OF OF PCA AVAILABLE ON ON PCA AUCK-OUT OF ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO call-back walls exameling on the seconds. 6. Display Sleep YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO call-back walls exameling on the seconds. 7. Chirp low alarms YES - clipplay goes blank after 2 minutes. NO - call-back alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - call-back alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 9. Loading dose YES - Continuous infusions are not available. Start were tog. 9. Loading dose YES - Continuous infusions are not available. Start were tog. 9. Loading dose rates YES - Continuous infusions are not available. Start were tog. <		NO - ICONS disabled.			
3. Modify protocol YES - allows protocols to be modified in SET mode. 4. Handset mode MODE A B C 4. Handset mode MODE A A L HAND SET LIGHT: PCA MOLLABLE ON ON PCA PCA LOCK-OUT OFF ON ON NO Stopped call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO Stoppage call-back YES - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the C all button for 4 seconds. 6. Display Sleep YES - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Ontinuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions option in protocols. NO - Continuous infusions option in protocols. NO - Loading dose are not available. <	2. Protocols in use	PCA pre-set protocols to	be availal	ble. Sel	ect number from 1 to 10.
NO - removes modify protocol option in SET mode. 4. Handset mode MODE A B C BEEP GOD ALL ALL HAND SET LIGHT: PCAM STOPPED OFF OFF PCA AVAILABLE ON ON PCA PCA DELIVERING FLASH ON PCA PCA DELIVERING FLASH ON PCA S. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back alarm can be delayed from 10 mins up to 90 mins. N.O - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms. press and hold the ⁽²⁾ / ₍₂₎ button for 4 seconds. 6. Display Sleep YES - Chirp alarm is insabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. N.O - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. N.O - Continuous infusions option in protocols. N.O - Continuous infusions are not available. 9. Loading doses YES - Continuous infusions are not available. 9. Loading doses rate. YES - Loading dose catito appears in protocols. N.O - Continuous infusion appears in protocols. NO	3. Modify protocol	YES - allows protocols to be modified in SET mode.			
4. Handset mode MODE A B C BEEP GOD ALL ALL HAND SET LIGHT: PCA MAILABLE OFF OFF PCA AVAILABLE ON ON PCA PCA AVAILABLE ON ON PCA PCA LOCK-OUT OFF OF ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the Call button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. NO - display stays on during operation. 7. Chirp low alarms YES - Continuous infusions option in protocols. NO - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the syrRINGE NEAR EMPTY alarm. No - Chirp alarm is enabled. The event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol alo includes the loading dose. Start the PCA. 10. Max. dose limits YES - Nose limit option appears in protocols. To activate this option to 60		NO - removes modify pro	otocol opt	ion in S	ET mode.
BEEP GOOD ALL ALL HAND SET LIGHT: PCA MATOPPED OFF OFF PCA MATOPPED OFF ON ON PCA AVAILABLE ON ON PCA PCA LOCK-OUT OFF ON ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the ⁽²⁾ / ₂ button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. NO - Loading doses are not available. NO - Dose limits option appears in protocols. 10. Max. dose limits YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate and there is no option to change when modifying the protocol. Dose rate can be either STAT rate and there is no option to change when modifying the protocol. Dose rate can be either STAT rate and there is no option to change when mo	4. Handset mode	MODE	А	В	С
HAND SET LIGHT: PCAM STOPPED OFF OFF OFF PCA AVAILABLE ON ON ON PCA DELIVERING FLASH ON FLASH PCA LOCK-OUT OFF OF ON S. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the B button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. YES - Loading dose option appears in protocols. 9. Loading doses YES - Dose limit option appears in protocols. NO - Dose limits are not available. NO - Dose limits are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose can be delivered over a period of time from 1 to 6		BEEP	GOOD	ALL	ALL
PCAM STOPPED OFF OFF OFF PCA AVAILABLE ON ON ON PCA DELIVERING FLASH ON FLASH PCA LOCK-OUT OFF ON ON S. Delayed call-back YES - call-back alarm can be delayed for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the B button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chrip alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. NO - Loading doses are not available. NO - Loading doses are not available. 10. Max. dose limits YES - Sole limit option appears in protocols. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. Dose rate can be either STAT rate (100m/h max) or the dose can be delivered over a period of time from 1 to 60 minutes. 11. Variable dose rates YES - Slows the dose rate to be varied, when modifying the protocol. Dose rate can be either S		HAND SET LIGHT:			
PCA AVAILABLE ON ON ON PCA DELIVERING FLASH ON FLASH PCA LOCK-OUT OFF ON ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm is disabled. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Chirp alarm is disabled. NO - Continuous infusions are not available. 9. Loading doses YES - Continuous infusions option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. NO - Loading doses are not available. 10. Max. dose limits YES - Obse limit option appears in protocols. NO - Dose limits are not available. NO - Back dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. Dose rate can be either STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use @ D buttons to set pump ident		PCAM STOPPED	OFF	OFF	OFF
PCA DELIVERING FLASH ON FLASH PCA LOCK-OUT OFF ON ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the D button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. NO - Loading doses are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use @ buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. 14. Nurse call YES - Nurse call concenter on abled.		PCA AVAILABLE	ON	ON	ON
PCA LOCK-OUT OFF ON ON 5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the D button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Loading dose option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. NO - Loading dose are not available. 10. Max. dose limits YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use D Stores I sub to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications enabled. 14. Nurse call YES - Nurse call connector enabled.		PCA DELIVERING	FLASH	ON	FLASH
5. Delayed call-back YES - call-back alarm can be delayed from 10 mins up to 90 mins. NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the CB button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. NO - Continuous infusions are not available. 9. Loading doses YES - Dose limit option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Dose limits are not available. NO - Dose limit option appears in protocols. 10. Max. dose limits YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100m//h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. 12. Comms identity number Use @ tuttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. NO - R5232 Communications enabled. NO - R5232 Communications disabled.		PCA LOCK-OUT	OFF	ON	ON
NO - call-back will be cancelled for up to 2 mins or extended to 15 mins. To extend call-back alarms, press and hold the ^{CB} button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms TES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Obse limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use @ buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. NO - R5232 Communications disabled.	5. Delayed call-back	YES - call-back alarm can	be delaye	ed from	10 mins up to 90 mins.
press and hold the Cal button for 4 seconds. 6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Ose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use W buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled. NO - RS232 Communications disabled.		NO - call-back will be ca	ncelled fo	r up to	2 mins or extended to 15 mins. To extend call-back alarms,
6. Display Sleep YES - display goes blank after 2 minutes. NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use 🗟 🐨 buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. NO - R5232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		press and hold the 🗳 b	utton for 4	l secon	ds.
NO - display stays on during operation. 7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Sose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use 🔘 D buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	6. Display Sleep	YES - display goes blank	after 2 mi	nutes.	
7. Chirp low alarms YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log. NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image: Start dose start dose and start dose and start dose. NO - RS232 Communications enabled. NO - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		NO - display stays on du	ring opera	ition.	
NO - Chirp alarm is disabled. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are active and recorded in the event log. 8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use @ buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - Nurse call connector enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	7. Chirp low alarms	YES - Chirp alarm is enabled, replacing the AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm. The AC POWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are not recorded in the event log.			
8. Continuous infusions YES - Continuous infusions option in protocols. NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use @ buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		NO - Chirp alarm is dis active and recorded in the	abled. Th	ne AC P og.	OWER FAIL alarm and the SYRINGE NEAR EMPTY alarm are
NO - Continuous infusions are not available. 9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	8. Continuous infusions	YES - Continuous infusio	ns option	in prot	ocols.
9. Loading doses YES - Loading dose option appears in protocols. To activate this option "NEW PATIENT" is confirmed. The protocol also includes the loading dose. Start the PCA. NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image: Start to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		NO - Continuous infusion	ns are not	availak	le.
NO - Loading doses are not available. 10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image: Disting the protocol of the set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	9. Loading doses	YES - Loading dose option The protocol also include	on appear es the load	rs in pr ding do	otocols. To activate this option "NEW PATIENT" is confirmed. ise. Start the PCA.
10. Max. dose limits YES - Dose limit option appears in protocols. NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image:		NO - Loading doses are r	not availat	ole.	
NO - Dose limits are not available. 11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image: Display the protocol between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	10. Max. dose limits	YES - Dose limit option a	ppears in	protoc	ols.
11. Variable dose rates YES - Allows the dose rate to be varied, when modifying the protocol. Dose rate can be either STAT rate (100ml/h max.) or the dose can be delivered over a period of time from 1 to 60 minutes. NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol. 12. Comms identity number Use Image: Display to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		NO - Dose limits are not	available.		
NO - Each dose will be delivered at the STAT rate and there is no option to change when modifying the protocol.12. Comms identity numberUse SImage: S13. Comms enabledYES - RS232 Communications enabled. NO - RS232 Communications disabled.14. Nurse callYES - Nurse call connector enabled. Here and the rest of the sector of the s	11. Variable dose rates	YES - Allows the dose rate (100ml/h max.) or the do	e to be vai ose can be	ried, wł delive	nen modifying the protocol. Dose rate can be either STAT rate red over a period of time from 1 to 60 minutes.
12. Comms identity number Use buttons to set pump identity (between 000 and 127) for use with remote communications. 13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.		NO - Each dose will be c the protocol.	delivered a	at the S	TAT rate and there is no option to change when modifying
13. Comms enabled YES - RS232 Communications enabled. NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	12. Comms identity number	Use 🗟 🗟 buttons communications.	to set p	oump	identity (between 000 and 127) for use with remote
NO - RS232 Communications disabled. 14. Nurse call YES - Nurse call connector enabled.	13. Comms enabled	YES - RS232 Communica	tions enak	oled.	
14. Nurse call YES - Nurse call connector enabled.		NO - RS232 Communicat	ions disat	oled.	
	14. Nurse call	YES - Nurse call connecto	or enabled	ł.	
NO - Nurse call connector disabled.		NO - Nurse call connecto	or disablec	١.	

Configured Options (Continued)

General Options (continued)

Option	Description
15. Nurse call inverted	YES - Nurse call inverted enabled.
	NO - Nurse call inverted disabled
16. Continuous Print	YES - Allows printing of events as they happen.
	NO - Continuous printing disabled.
17. Default Syringe	Use 🗟 🖻 buttons to select the default syringe brand.
18. Lock syringe type	YES - Syringe type locked to default syringe.
	NO - Syringe type not locked to default syringe.
19. Quiet mode	YES - Quiet mode enabled.
	NO - Quiet mode disabled.
20. Generic Drug Enabled	YES - Allows Protocols to be set up to use a Generic Drug, which is preset to maximum safety limits.
	NO - Disallows use of Generic Drug.
21. Max Dose Limit Alarm	YES - Alarm is generated when max dose limit is reached.
	NO - Alarm is not generated when max dose limit is reached.
22. Mix Mass & Vol Modes	YES - Allow a mix of mass and volume dose mode for drugs and protocols.
	NO - All drugs and protocols use mass dose mode.



To minimise the risk of incorrect confirmation of the syringe type it is recommended that the default syringe type should be locked to only the one brand type used in the hospital where possible.

Clock Set

- 1. Select **CLOCK SET** from the Configured Options menu and press **ENTER**.
- 2. Use the B buttons to adjust the date and time displayed, pressing **NEXT** to access the next field.
- 3. When the correct time and date are displayed press **OK** to return to the Configured Options menu.



The internal clock is the reference against which the pump stores patient history and events. Changing the clock will automatically reset the dates against which all new patient history is stored in the pump. After changing the clock, the pump will force a YES response the next time the NEW PATIENT screen appears. This will clear the patient history. Therefore, patient history should always be recorded and, if required, printed prior to changing the clock.

Pre-Set Protocol Set-Up

- 1. Select **PROTOCOL DEFAULT SET-UP** from the Configured Options menu and press **ENTER**.
- 2. Press **MODIFY PROTOCOL** to display current protocol summary. Use the 🙆 🗟 buttons to highlight a field, press **ALTER** to access the field and the 🗟 🗟 buttons to select required values.
- 3. When the field is correct press **CONFIRM**. Press **OK** to return to the protocol summary.



One of the ten drugs programmed in the Drug Names and Safety Limits are selectable for each protocol. Additionally a default drug with limits set to extremes may be chosen if Generic Drug is enabled in General Options. Default drugs are named "MASS DRUG" and "VOL DRUG". The names indicate the underlying dose mode.

Hospital Name

This option allows the user to programme in the name of the hospital, ward or department. This will appear during the power-up display sequence and the 'Display Sleep' screen (if enabled in General Options).

- 1. Whilst holding down the \bigcirc button turn the pump **ON**.
- 2. The main display will show **000**. Enter the access code for Hospital Name using the 🙆 🖻 buttons, pressing **NEXT** to move through the digits. A full list of access codes can be found in the Technical Service Manual.
- 3. When the complete code shows on screen, press **ENTER**. The Hospital Name Option will be displayed.
- 4. Use the 🐵 🗟 buttons to adjust the character displayed, pressing **NEXT** to access the next position.
- 5. When the correct name is displayed turn LOCK 1 to the the OFF position.

Drug Names and Safety Limits

7

If enabled in General Options, a drug can be configured in either Mass or Volume dosing mode. If this option is disabled, all drugs and protocols are computed in dose units of mass.

On all drug parameter screens except DRUG name, press BACK to return to the previous parameter.

- 1. Turn **LOCK 1** to **SET** position while pressing down ^(C) button.
- 2. Enter the access code using the B buttons.
- 3. Select DRUG NAMES AND SAFETY LIMITS from the menu. Press ENTER.



4. Press **MODIFY DRUG** to modify the drug summary. Use 🗟 🗟 buttons to select desired values. When field is correct press **OK** to store the selection.



- 5. NAME: Use 🗟 🗟 buttons to set highlighted letter. Press NEXT for next character (up to twelve letters). Press OK when complete.
- 6. **DOSE MODE:** Use 🗟 🗟 buttons to change dose mode. Changing dose mode resets Drug parameters to defaults and also resets Protocols that use this drug. To change dose mode, press **CONFIRM**. Press **OK** when complete. **Note: DOSE MODE** will not be displayed if Mix Mass & Volume Modes is disabled in General Options.
- 7. MIN DRUG CONC: Use 🗟 🗟 buttons to set minimum concentration. For volume mode, concentration can be set to OFF, the lowest value. Press OK when complete.
- 8. MAX DRUG CONC: Use 🐵 🗟 buttons to set maximum concentration. Press OK when complete. If min Drug conc is set to OFF, this parameter does not appear.
- 9. MIN LOCKOUT PERIOD: Use 🙆 🖻 buttons to set minimum lockout period. Press OK when complete.
- 10. MAX LOCKOUT PERIOD: Use 🙆 🗟 buttons to set maximum lockout period. Press OK when complete.
- 11. MIN PCA DOSE: Use 🙆 😁 buttons to set minimum PCA dose. Press OK when complete.
- 12. MAX PCA DOSE: Use 🙆 🕑 buttons to set maximum PCA dose. Press OK when complete.
- 13. MAX CONTINUOUS: Use 🖄 🕑 buttons to set maximum continuous rate. Press OK when complete.
- 14. MAX LOADING DOSE: Use 🙆 🕑 buttons to set maximum loading dose. Press OK when complete.
- 15. MAX DOSE LIMIT: Use 🐵 🖻 buttons to set maximum dose limit. Press OK when complete.
- 16. MAX CLINICIAN BOLUS: Use 🐵 🗟 buttons to set the maximum clinician bolus. Press OK when complete.
- 17. **NEXT DRUG** to display the next drug name and the safety limits. The ten pre-set drug protocols are identified as 1 to 10.
- 18. Press **QUIT** to exit and return to configuration menu.
- 19. When set-up is complete, turn LOCK 1 to OFF position to save selection and to turn pump off.

General Options

Configured Options Record

ns Enter the pump-specific information for your records on a copy of this page.

Option	Rai	nge	Default	Setting
ICONS ON DISPLAY	YES/NO		NO	
PROTOCOLS IN USE	1 - 10		5	
MODIFY PROTOCOL	YES/NO		YES	
HANDSET MODE	A / B / C		A	
DELAYED CALLBACK	YES/NO		YES	
DISPLAY SLEEP	YES/NO		YES	
CHIRP LOW ALARMS	YES/NO		NO	
CONTINUOUS INFUSIONS	YES/NO	·	YES	
LOADING DOSES	YES/NO		YES	
MAX DOSE LIMITS	YES/NO		YES	
VARIABLE DOSE RATES	YES/NO		YES	
COMMS PUMP IDENTITY	000 - 127		001	
COMMS ENABLED	YES/NO		YES	
NURSE CALL	YES/NO		NO	
NURSE CALL INVERTED	YES/NO		NO	
CONTINUOUS PRINT	YES/NO		NO	
DEFAULT SYRINGE	BD PLASTIPAK IVAC TERUMO B. BRAUN OMNIFIX MONOJECT R.R PRONTO BD WORLDWIDE ONCE	FRESENIOUS INJECT. RAPIJECT PHARMA-JECT BD PRECISE BRAUN PERFUSOR* JANPOL* * with options kit fitted	BD PLASTIPAK	
LOCK SYRINGE TYPE	YES/NO		NO	
QUIET MODE	YES/NO		NO	
GENERIC DRUG ENABLED	YES/NO		YES	
MAX DOSE LIMIT ALARM	YES/NO		YES	
MIX MASS & VOL MODES	YES/NO		NO	

Syringe Type	Enabled
UNIVERSAL	
BRAUN PERFUSOR	
JANPOL	

Hospital Name	Serial No.		Software Version
Approved by		Configured by	
Date		Date	

Drugs and Pro	tocols Reco	rd										
Enter the pump-spe	cific informa	ition for your r	ecords	Hos	spital/Insti-	tution:						
Drug names and	Safety Limit:	S		Dé	epartment,	/Ward:						
Drug number	Mass Range	Volume Range	1	2	m	4	2	6	7	8	6	10
Drug Name (12 characters)												
Dose Mode	Mass	Volume										
Minimum Drug Concentration	1µg/ml - 99.9mg/m	Off, 1 µg/ml - 99.9mg/ml										
Maximum Drug Concentration	lm/gmg.99.9mg/ml	1 h.g/ml - 99.9mg/ml										
Minimum Lockout Period	0 - 180 minutes	0 - 180 minutes										
Maximum Lockout Period	0 - 180 minutes	0 - 180 minutes										
Minimum PCA Dose	0µg - 99.9mg	0.0ml - 99.9ml										
Maximum PCA Dose	0µg - 99.9mg	0.0ml - 99.9ml										
Maximum Continuous	d/gm0.999.0mg/h	0.0ml/h - 35ml/h										
Maximum Loading Dose	0µg - 99.9mg	0.0ml - 99ml										
Maximum Max Limit	0µg - 999mg	0.0ml - 999ml										
Maximum Clinician Bolus	1µg - 99.9mg	0.1ml - 99.9ml										
Protocol Default	Set Up											
Protocol number	A	В			٥	Ш	ш	ט	н		_	-
Drug Name												
Drug Concentration												
PCA Dose												
Lockout Period												
Occlusion Level												
Continuous												
Loading Dose												
Max Limit												
Limit Duration												
PCA Delivery												
Moc	del:											
Serial Numb	er:			Cont	figured by:				Date			
Software Versic	.uc			Ap	proved by:	•			Date	:a		

Drugs and Protocols Record

1000DF00451 Issue 6

CONCENTRATION RANGE:

1µg/ml - 999µg/ml in 1µg/ml steps

1.0mg/ml - 99.9mg/ml in 0.1mg/ml steps

VOLUME MODE:

Concentration can also be set OFF, in which case no mass data is displayed.

PCA DOSE RANGE:

Mass Mode: 0.0μg - 99 9μg in 1μg steps 1mg - 99.9mg in 0.1mg steps

Volume Mode: 0.0ml - 99.9ml in 0.1ml steps

PCA DELIVERY RATE:

100ml/h max. STAT rate for 30ml, 50ml and 100ml syringes and 80ml/h for 20ml syringes.

(Option to set duration from 1 to 60 mins in 1 min steps to minimum rate of 0.1ml/h and maximum of the STAT rate).

RATE CONVERSION FACTOR:

When pump is programmed in Mass units the conversion factor is:- ml/h = (dose/concentration)/(time in minutes/60).

LOCKOUT INTERVAL:

0 - 180 minutes in 1 minute steps

LOADING DOSE RANGE:

	DAN/65
Volume Mode:	0.0ml - 99.9ml in 0.1ml steps
	0.0mg - 99.9mg in 0.1mg steps (Delivered at STAT rate)
Mass Mode:	0μg - 999μg in 1μg steps

CONTINUOUS RATE RANGE:

Mass Mode:	0μg/h - 90μg/h in 10μg/h steps
	0.0mg/h - 999.0mg/h in 0.1mg/h steps
Volume Mode:	0.0ml/h - 35.0ml/h in 0.1ml/h steps.
MAX DOSE LIMIT:	
Mass Mode:	off, 1µg - 999µg in 1µg steps
	1mg - 999mg in 1mg steps
Volume Mode:	off, 0.1ml to 999ml in 0.1ml steps
1 - 8 hours duration	in 1 hour steps.

PURGE RATE:

100ml/h

SYSTEM ACCURACY:

Drive Linearity:	+/- 1%
Volumetric:	+/- 2% (nominal)

(Volumetric accuracy is +/-2% typical by volume at the STAT PCA rate and above when the pump is used with the recommended syringes. Differences in factors such as size and plunger force in compatible syringes can cause variations in accuracy and trumpet curves.)

Important:

System accuracy is +/-2% typical by volume as measured using the trumpet curve test method defined in EN60601-2-24:1998 at rates of 1.0ml/h and above when the pump is used with the recommended syringes. Differences in factors such as size and plunger force in compatible syringes can cause variations in accuracy and trumpet curves. Also see trumpet curves section.

Environmental Specifications -

Operating Temperature	+10°C - +40°C
Operating Relative Humidity	30% - 75%
Operating Atmospheric Pressure	700hPa - 1060hPa
Transport & Storage Temperature	-20°C - +50°C
Transport & Storage Relative Humidity	5% - 95%
Transport & Storage Atmospheric Pressure	600hPa - 1060hPa

OPERATION MODE:

Continuous

CRITICAL VOLUME:

The maximum over infusion which can occur in the event of a single fault condition is 0.8ml for 20ml, 30ml and 50ml syringes and 1.5ml for a 100ml syringe.

ALARM CONDITIONS:

Pressure Limit exceeded	Low Battery Warning
Drive Disengaged	Battery Exhausted
Syringe Almost Empty	Cover Open during operation
Syringe Empty	Hand-set Disconnected
Max Dose Limit (optionally alarmed)	Internal Malfunction
Nurse Attention/Call Back	Syringe Error

PUMPING PRESSURE / ALARM LEVEL:

375mmHg (nominal) default alarm level (L-4) with 11 user selectable alarm levels (L-0 to L-10). Syringes may limit below level 10.

(The maximum pressure that can be developed by the system at the maximum user selectable alarm level is 1100mmHg).

CLINICIAN OVER-RIDE:

Bolus or continuous infusion in **RUN** mode.

(User selectable from $1\mu g$ - 99.9mg or 0.1ml to 99.9ml (volume mode) bolus dose administered at the STAT rate (100ml/h) or over 1 to 180 minutes delivery period).

Modify PCA Protocol in SET mode.

(When option to disable MODIFY PROTOCOL has been selected).

BATTERY OPERATION:

6 hours operation from a fully charged battery at 5.0ml/h and 20°C under normal conditions.

BATTERY TYPE AND RECHARGE TIME:

Rechargeable sealed lead acid type. 10 hours from discharge to 80% charge, 24 hours from discharge to 100% charge.

EVENT HISTORY:

2000 events rolling memory.

MEMORY RETENTION:

All calibration and set up information will be retained in the pump memory for a minimum of 3 years.

BOLUS VOLUME ACCURACY*:

The following table provides an indication of the accuracy with which a bolus infusion will be delivered. Test carried out as specified in IEC/EN60601-2-24.

Bolus Volume	No. of	Max. Positive	Max. Negative	Mean
(ml)	Samples	(%)	(%)	(%)
0.1	25	12	-14	-5
2	25	2	2	1
5	25	2	2	0.8

* - Using BD Plastipak 50ml syringe at 5ml/h under normal conditions (95% confidence / 95% of pumps).

Maximum Infusion Maximum Time to Nominal Occlusion Maximum Bolus Rate occlusion alarm Alarm Level Volume Alarm Pressure Pressure (ml/h) (min:sec) (mmHg) (mmHg) (ml) 0 1 2:00 0 50 0.1 1 8:00 92 0.2 1 110 2 1 20:00 184 220 0.3 3 0.5 1 33:00 276 330 4 450 0.7 1 52:00 368 5 0.9 1 65:00 460 560 6 1 85:00 552 670 1 7 1 102:00 780 1.2 664 1 8 736 890 1.6 120:00 9 1 1.8 140:00 828 1000 10 1 155:00 920 1100 2 5 0 1:00 0 50 0.1 1 5 2:00 92 110 0.2 2 5 5:00 184 220 0.3 3 5 7:00 276 330 0.5 4 5 450 0.7 10:00 368 5 5 12:00 460 560 0.9 6 5 15:00 552 670 1 7 5 17:00 644 780 1.2 5 8 20:00 736 890 1.6 9 5 24:00 828 1000 1.8 10 5 26:00 920 1100 2

The following tables show the worst case values for line pressure, time to alarm and bolus volume that can be expected in the event of an occlusion when the IVAC 50ml syringe and G40020B extension set are selected.

Compatible Syringes

The pump is calibrated and labelled for use with single-use disposable Luer lock syringes. Only use the size and type of syringe specified on the pump display. The full list of permitted syringe models is dependent on the software version of the pump.

Syringe Range	Syringe Type	20ml	30ml	50ml	100ml
UNIVERSAL	IVAC			✓	✓
	BD Plastipak	✓	✓	✓	
	B Braun Omnifix	✓	✓	✓	
	Terumo	✓	✓	✓	
	Rapiject			✓	
	BD Worldwide	✓	✓	✓	
	BD Precise	✓		✓	
	Pharma-Ject			✓	
	Once			✓	
	Fresenius Injectomat			✓	
	Monoject*	✓	✓	✓	
	RR Pronto	✓	✓	✓	
BRAUN PERFUSOR	B Braun Perfusor			✓	
JANPOL	Janpol			✓	

* Ξ TYCO / Healthcare KENDALL - MONOJECT.

Compatible Extension Sets

The pump uses standard, single-use, disposable extension sets and syringes with Luer lock connectors. The user is responsible for verifying the suitability of a product used if it is not recommended by CareFusion.

Standard Sets	
G40015	Standard PVC Syringe Extension Set (150 cm). Priming Volume: 2.6ml
G40020B	Standard PVC Syringe Extension Set (200 cm). Priming Volume: 1.5ml
G402EP	Extension set, Luer lock connectors. Kink resistant DEHP free PVC yellow striped tubing. Bore 1mm. Length 200cm. Priming volume 1.6ml.

Light Protecte	d Sets	
G40215	ISAmber Polyethylene Syringe Extension Set (150 cm).Priming Volume: 1.2ml	
	₽⊂Ĵ	
G40320	White PVC Syringe Extension Set (200 cm). Priming Volume: 3.6ml	
		₽₽

Low Sorbing Sets	
G40615	Polyethylene Syringe Extension Set (150 cm). Priming Volume: 1.5ml
	₽C////////////////////////////////
G40620	Polyethylene Syringe Extension Set (200 cm). Priming Volume: 2ml
G40720	Polyethylene Lined Syringe Extension Set with clamp. (200 cm). Priming Volume: 1.5ml
04105010509	Polyethylene Syringe Extension Set (100 cm). Priming Volume: 1ml



For availability please contact your local CareFusion representative because new sets are continuously being developed for our customers.

It is recommended that extension sets are changed in accordance with the Directions for Use. Carefully read the Directions For Use supplied with the extension set prior to use.

Please note these drawings are not to scale

Compatible Extension Sets (Continued)

The pump uses standard, single-use, disposable extension sets and syringes with Luer lock connectors. The user is responsible for verifying the suitability of a product used if it is not recommended by CareFusion.

Patient Controlled Analgesia (PCA) Sets			
30822	PVC Syringe Extension Set with clamp (152 cm).		
	Priming Volume: 0.5ml		
30832	PVC 'Y' Syringe Extension Set with back check valve and 2 clamps (178 cm).		
30842E	PVC Syringe Extension Set with back check value. SmartSite™ Needle-Free Value Port and clamp (30 cm)		
30042L	Priming Volume: 1.4ml		
30852	PVC 'Y' Syringe Extension Set with anti-siphon valve, back check valve and 2 clamps (183 cm).		
	Priming Volume: 1.8ml		
30862	PVC Syringe Extension Set with anti-siphon valve and clamp (156 cm).		
	₽ ⊑]		
04102215162	PVC Syringe Extension Set with rotating luer. (150 cm).		
	Priming Volume: 2.9ml		
04100010162	PVC Syringe Extension Set (105 cm).		
	Priming Volume: 7.2ml		



For availability please contact your local CareFusion representative because new sets are continuously being developed for our customers.

It is recommended that extension sets are changed in accordance with the Directions for Use. Carefully read the Directions For Use supplied with the extension set prior to use.

Please note these drawings are not to scale

Maintenance

Routine Maintenance Procedures

To ensure that this pump remains in good operating condition, it is important to keep it clean and carry out the routine maintenance procedures described below. All servicing should only be performed by a qualified service engineer with reference to the Technical Service Manual (TSM).

Circuit diagrams and component parts lists and all other servicing information which will assist the qualified service engineer in performing repair of the parts designated as repairable are available upon request from CareFusion.

If the pump is dropped, damaged, subjected to excessive moisture or high temperature, immediately take it out of service for examination by a qualified service engineer. All preventative and corrective maintenance and all such activities shall be performed at a compliant work place in accordance with the information supplied. CareFusion will not be responsible should any of these actions be performed outside the instructions or information supplied by CareFusion.

Refer to the Technical Service Manual for the access code for technical service features.

Interval	Routine Maintenance Procedure
As per Hospital Policy	Thoroughly clean external surfaces of the pump before and after prolonged period of storage.
At least once per year	1. Inspect AC power supply plug and cable for damage.
(Refer to TSM for	2. Perform functional tests as outlined in the Technical Service Manual.
identification of parts,	3. Operate the pump on battery power until the battery low alarm then charge the battery to confirm battery operation and charging.

Please refer to Technical Service Manual for calibration procedures. The units of measurement used in the calibration procedure are standard SI (The International System of Units) units.

It is important to ensure that you only refer to the most recent version of the Directions for Use and Technical Service Manual for your CareFusion products. These documents are referenced on www.carefusion.com. Copies can be obtained by contacting your local CareFusion representative.

Replacing the AC Fuses

If the pump continually illuminates the battery symbol and the AC power indicator light does not illuminate when the pump is connected to the AC power supply and switched ON, either the power supply fuse in the AC plug, if fitted, or the internal fuses have blown.

First check the power supply fuse in the AC mains plug, if fitted. If the AC power indicator light does not illuminate remove the pump from service.

It is recommended that only a qualified service engineer replaces the AC fuses. For further information regarding the replacement of internal AC fuses refer to the Technical Service Manual.

ſ	
L	łJ
_	

If the fuses continue to blow, suspect an electrical fault and have the pump and power supply checked out by a qualified service engineer.

Battery Operation

The internal rechargeable battery allows continued operation when the AC power is unavailable, for example during patient transfer or AC power failure. A fully charged battery will provide over 6 hours of operation at typical infusion rates. From the battery low alarm it will take about 24 hours to fully recharge when reconnected to the AC power supply, whether the pump is in use or not.

The battery is maintenance free, sealed lead acid type and requires no routine servicing. However, to achieve optimum operation, ensure that the battery is fully recharged after full discharge, before storage, and at regular 3 month intervals during storage.

CareFusion recommend verification that the pump operates on battery power once the pump has been removed from the AC power supply, refer to 'Starting the Pump' section.

Charge retention will eventually degrade. Where retention is critical the internal battery should be replaced every 3 years.

It is recommended that only a qualified service engineer replaces the battery. For further information regarding the replacement of batteries refer to the Technical Service Manual.

Test Routines

The test routines are designed to allow confirmation of many of the pump functions, defaults and calibrations without requiring internal inspection. They do not represent a full calibration check.



See the Technical Service Manual for a complete list of the test procedures, access codes and calibration procedures.

Cleaning and Storage

Before the transfer of the pump to a new patient and periodically during the use, clean the pump by wiping over with a lint-free cloth lightly dampened with warm water and a standard disinfectant / detergent solution.

Recommended cleaners are:

Brand	Concentration
Hibiscrub	20% (v/v)
Virkon	1% (w/v)

The following products were tested and are acceptable for use on the Pump if used in accordance with the specified manufacturer's guidelines.

- Warm soapy water
- Mild detergent in water (e.g. Young's Hospec)
- 70% Isopropyl Alcohol in water
- Chlor-Clean
- Clinell Sporicidal wipes
- Hibiscrub
- TriGene Advance
- Tristel Fuse sachets
- Tristel Trio wipes system
- Tuffie 5 wipe
- Virkon Disinfectant
- Virusolve+ (Ready To Use)
- Virusolve+ (Wipes)

Do not use the following disinfectant types:

- Disinfectants which are known to be corrosive to metals must not be used, which include:
 - NaDcc (such as Presept),
 - Hypochlorites (such as Chlorasol),
 - Aldehydes (such as Cidex),
 - Cationic Surfactants (such as Benzalkonium Chloride).
- Use of lodine (such as Betadine) will cause surface discoloration.
- Concentrated Isopropyl alcohol based cleaners will degrade plastic parts.

The syringe and extension sets are disposable single use items and should be discarded after use according to their manufacturers' instructions.

If the pump is to be stored for an extended period it should be first cleaned and the internal battery fully charged. Store in a clean, dry atmosphere at room temperature and, if available, employ the original packaging for protection.

Once every 3 months during storage, carry out functional tests as described in the Technical Service Manual and ensure that the internal battery is fully charged.



Before cleaning always switch off and disconnect from the AC power supply. Never allow liquid to enter the casing and avoid excess fluid build up on the pump. Do not use aggressive cleaning agents as these may damage the exterior surface of the pump. Do not steam autoclave, ethylene oxide sterilise or immerse this pump in any fluid.

Disposal

Information on Disposal for Users of Waste Electrical & Electronic Equipment

This X symbol on the product and/or accompanying documents means that used electrical and electronic products should not be mixed with municipal waste.

If you wish to discard electrical and electronic equipment, please contact your CareFusion affiliate office or distributor for further information.

Disposing of this product correctly will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling.

Information on Disposal in Countries outside the European Union

This X symbol is only valid in the European Union. The product should be disposed of taking environmental factors into consideration. To ensure no risk or hazard, remove the internal rechargeable battery and the Lithium battery from the control board and dispose of as outlined by the local country regulations. All other components can be safely disposed of as per local regulations.

RS232 / Nurse Call Feature

The RS232 / Nurse call feature is a feature on Alaris[™] Syringe Pumps. It allows the pump to be monitored remotely and/or controlled via a suitable central monitoring or computer system.

When the pump is started by a command from the serial interface, communication must take place over the serial interface. A communication must take place every 15 seconds or the pump will alarm, display communications failure and stop infusing. This failure protects against failure of the communications, including the removal of the RS232 cable.

\mathcal{V}^{-}

The nurse call interface provides a remote backup to the internal audible alarm. It should not be relied upon to replace monitoring of the internal alarm.

Refer to the Technical Service Manual for further information regarding the RS232 interface. Since it is possible to control the syringe pump using the RS232 interface at some distance from the pump and hence remote from the patient, responsibility for the control of the pump is vested in the software run on the computer control system.

The assessment for the suitability of any software used in the clinical environment to control or receive data from the pump lies with the user of the equipment. This software should include detection of the disconnection or other failure of the RS232 cable. The protocol is detailed in the Technical Service Manual and is for general information only.

Any connected analogue and digital components are required to meet IEC/EN60950 for data processing and IEC/EN60601 for medical devices. Anyone connecting additional devices to the signal input or output is a system configurator and responsible for meeting the requirements of the system standard IEC/EN60601-1-1.

RS232 / Nurse Call Connection Data

Nurse call Specification -

-	
Connector	D Type - 9 Pin
TXD/RXD	EIA RS232-C Standard
TXD Output Voltage Range	Minimum: -5V (mark), +5V (space)
	Typical: -7V (mark), +7V (space) with 3KΩ load to ground
RXD Input Voltage Range	-15V to +15V max.
RXD Input Thresholds	Low: 0.6V minimum / High: 3.0V maximum
RXD Input Resistance	3KΩ minimum
Baud Rate	9600 Baud
Start Bits	1 Start Bit
Data Bits	8 Data Bits
Parity	Odd Parity
Stop Bits	1 stop bit
Nurse Call Relay Contacts	Pins 6 & 7

IBM COMPATIBLE	PUMP	IBM COMPATIBLE
(9 PIN)		(25 PIN)
PIN 3 (TXD)	PIN 2 (RXD2)	PIN 2 (TXD)
PIN 2 (RXD)	PIN 3 (TXD2)	PIN 3 (RXD)
PIN 5 (GND)	PIN 5 (GND)	PIN 7 (GND)
PIN 7 (RTS)		PIN 4 (RTS)
PIN 8 (CTS)		PIN 5 (CTS)
PIN 4 (DTR)	= -	PIN 20 (DTR)
PIN 6 (DSR)		PIN 6 (DSR)

PUMP

PIN1(ALM1)

RESP ALARM

PUMP	PRINTER
	(25 PIN)
PIN 9 (TXD1)	PIN 3 (RX)
PIN 4 (PWR1)	PIN 5 (CTS)
PIN 5 (GND)	PIN 7 (GND)
PIN 8 (CTS1)	PIN 20 (DTR)



Trumpet Curves & Start-up Curves

In this pump, as with all infusion systems, the action of the pumping mechanism and variations in individual syringes cause short-term fluctuations in rate accuracy.

The following curves show typical performance of the system in two ways: 1) the delay in onset of fluid flow when infusion commences (start-up curves), and 2) the accuracy of fluid delivery over various time periods is measured (trumpet curves).

The start-up curves represent continuous flow versus operating time from the start of the infusion. They exhibit the delay in onset of delivery due to mechanical compliance and provide a visual representation of uniformity. Trumpet curves are derived from the second hour of this data. Tests performed per IEC/EN60601-2-24 standard.

Trumpet curves are named for their characteristic shape. They display discrete data averaged over particular time periods or 'observation windows', not continuous data versus operating time. Over long observation windows, short term fluctuations have little effect on accuracy as represented by the flat part of the curve. As the observation window is reduced, short term fluctuations have greater effects as represented by the "mouth" of the trumpet.

Knowledge of system accuracy over various observation windows may be of interest when certain drugs are being administered. Short term fluctuations in rate accuracy may have clinical impact depending on the half-life of the drug being infused, therefore the clinical effect cannot be determined from the trumpet curves alone.

Start-up and trumpet curves may not be indicative of operation under negative pressure.

Differences in factors such as size and plunger force in compatible syringes produced by other manufacturers can cause variations in accuracy and trumpet curves as compared to those represented. Additional curves for compatible syringes are available upon written request.

For applications where flow uniformity is a concern, rates of 1.0 ml/h or above are recommended.



Trumpet curve values show minimum and maximum percentage rate error at 2 minutes and long term accuracy.

Trumpet Curves & Start-up Curves continued



Performance with syringes : BD Plastipak, B Braun Omnifix, Monoject, Once. 2 minute minimum/maximum +/- 5% @ 5.0ml/h. Long term mean accuracy +/- 2% is in addition to the trumpet curve percentage error.

Performance with syringes : IVAC, Zeneca, Terumo, Nipro, Fresenius, B Braun Perfusor & JMS. 2 minute minimum/maximum +/- 7.5% @ 5.0ml/h. Long term mean accuracy +/- 2% is in addition to the trumpet curve percentage error.

Evaluation over the range -100mmHg to +100mmHg equivalent to water height +/- 1.3m, will produce trumpet curves with the limits shown above.

Service Contacts

For service contact your local Affiliate Office or Distributor.

AE	DE	НИ	РТ
CareFusion, PO Box 5527, Dubai, United Arab Emirates.	CareFusion, Tullastr. 8-12 69126 Heidelberg, Deutschland.	CareFusion, Döbrentei tér 1, H-1013 Budapest, Magyarország.	CareFusion, Avda. São Miguel, 296 Atelier 14 2775-751 Carcavelos, Lisboa Portugal
Tel: (971) 4 28 22 842	Tel: (49) 6221 305 0	Tel: (36) 1 488 0232 Tel: (36) 1 488 0233	Tel: +351 219 152 593
Fax: (971) 4 28 22 914	Fax: (49) 6221 305 216	Fax: (36) 1 201 5987	Fax: +351 219 152 598
AU	DK	IT	SE
CareFusion, 3/167 Prospect Highway, PO Box 355 Seven Hills, NSW 2147, Australia.	CareFusion, Firskovvej 25 B, 2800 Lyngby, Danmark.	CareFusion, Via Ticino 4, 50019 Sesto Fiorentino, Firenze, Italia.	CareFusion, Hammarbacken 4B, 191 46 Sollentuna, Sverige.
Tel: (61) 1800 833 372	Tlf. (45)70 20 30 74	Tél: (39) 055 30 33 93 00	Tel: (46) 8 544 43 200
Fax: (61) 1800 833 518	Fax. (45)70 20 30 98	Fax: (39) 055 34 00 24	Fax: (46) 8 544 43 225
BE	ES	NL	US
CareFusion, Erembodegem-Dorp 86 B-9320 Erembodegem Belgium.	CareFusion, Edificio Veganova, Avenida de La Vega, nº1, Bloque 1 - Planta 1, 28108 Alcobendas, Madrid, España.	CareFusion, De Molen 8-10, 3994 DB Houten, Nederland.	CareFusion, 10020 Pacific Mesa Blvd., San Diego, CA 92121, USA.
Tel: +32 (0) 2 267 38 99	Tel: (34) 902 555 660	Tel: +31 (0)30 2289 711	Tel: (1) 800 854 7128
Fax: +32 (0) 2 267 99 21	Fax: (34) 902 555 661	Fax: +31 (0)30 2289 713	Fax: (1) 858 458 6179
CA	FR	NO	ZA
CareFusion, 235 Shields Court, Markham, Ontario L3R 8V2, Canada.	CareFusion, Parc d'affaire le Val Saint Quentin 2, rue René Caudron 78960 Voisins le Bretonneux France	CareFusion, Fjordveien 3 1363 HØVIK Norge.	CareFusion, Unit 2 Oude Molen Business Park, Oude Molen Road, Ndabeni, Cape Town 7405, South Africa.
Tel: (1) 905-752-3333	Tél: (33) 01 30 02 81 41	Tel: (47) 64 00 99 00	Tel: (27) (0) 860 597 572 Tel: (27) 21 510 7562
Fax: (1) 905-752-3343	Fax: (33) 01 30 02 81 31		Fax: (27) 21 5107567
СН	FI	NZ	
CareFusion, A-One Business Centre Zone d'activitiés Vers-la-Pièce n° 10 1180 Rolle / Switzerland	CareFusion, Kuortaneenkatu 2, 00510 Helsinki	CareFusion, 14B George Bourke Drive, Mt Wellington 1060, PO Box 14-518, Panmure 1741, Auckland, New Zealand	
Ph.: 0848 244 433	Tel: +358 207871 090	Tel: 09 270 2420 Freephone: 0508 422734	
Fax: 0848 244 100		Fax: 09 270 6285	
CN	GB	PL	
康尔福盛(上海)商贸有限公司 地址:上海市浦东新区张杨路 500号24楼E.F.G.H单元	CareFusion, The Crescent, Jays Close, Basingstoke, Hampshire, RG22 4BS, United Kingdom.	Becton Dickinson Polska Sp. z o.o. ul. Osmańska 14 02-823 Warszawa Polska.	
电话: +86-21-60369369 400 878 8885	Tel: (44) 0800 917 8776	Tel: (48) 22 377 11 00	
传真: +86-21-60369399	Fax: (44) 1256 330860	Fax: (48) 22 377 11 01	Rev. O

Spare Parts

A comprehensive list of spare parts for this pump is included within the Technical Service Manual.

The Technical Service Manual (1000SM00017) is now available in electronic format on the World Wide Web at :-

www.carefusion.co.uk/alaris-technical/

A username and password are required to access our manuals. Please contact local customer services representative to obtain login details.

Part Number	Description
0000EL00004	Internal Battery Pack
1001FAOPT91	AC Power Lead - UK
1001FAOPT92	AC Power Lead - European
5000SP00008	Printer cable 9 to 25 pin
5000SP00011	Printer cable 9 to 26 pin
1000SP01008	Comms Cable (9 pin to 9 pin)
1000SP01015	Pole Clamp Assembly

Page Intentionally Left Blank

Alaris, PCAM, IVAC, CareFusion, CareFusion logo and SmartSite are trademarks or registered trademarks of CareFusion Corporation or one of its affiliates. All rights reserved. All other trademarks are property of their recording ourgoar respective owners.

© 2016 CareFusion Corporation or one of its affiliates. All rights reserved.

This document contains proprietary information of CareFusion Corporation or one of its affiliates, and its receipt or possession does not convey any rights to reproduce its contents, or to manufacture or sell any product described. Reproduction, disclosure, or use other than for the intended numeroe without procife written authorization of purpose without specific written authorization of CareFusion Corporation or one of its affiliates is strictly forbidden.



CareFusion Switzerland 317 Sarl, A-One Business Centre, Z.A Vers –La-Pièce n° 10, CH-1180, Rolle



CareFusion UK 305 Ltd., The Crescent, Jays Close, Basingstoke, Hampshire, RG22 4BS, UK

1000DF00451 Issue 6



carefusion.com