GE Healthcare

Optima EMS ECG Management System Database Backup and Recovery Instruction

Software Version 1.0
2069773-007 Revision B

Optiam EMS
English
© 2012 General Electric Company.
All Rights Reserved.

Publication Information

The information in this manual applies only to Optima EMS ECG Management System Version 1.0. It does not apply to earlier product versions. Due to continuing product innovation, specifications in this manual are subject to change without notice.

MUSE and MAC are trademarks owned by GE Medical Systems *Information Technologies*, Inc., a General Electric Company going to market as GE Healthcare. All other trademarks contained herein are the property of their respective owners.

The document part number and revision appear at the bottom of each page. The revision identifies the document's update level. The revision history of this document is summarized in the following table.

Revision	Date	Comments
A	1 November 2012	Internal Release
B	10 December 2012	Initial release for customer

To access other GE Healthcare Diagnostic Cardiology manuals, go to the Common Documentation Library (CDL), located at http://www3.gehealthcare.com/en/Support/Support_Documentation_Library, and click **Cardiology**.

To access Original Equipment Manufacturer (OEM) manuals, go to the device manufacturer's Web site.

Service Manual Language Information

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Contents

1	Introduction	
	Manual Information	11
	Intended Audience	11
	Manual Purpose	11
	Document Conventions	11
	Responsibility of the Purchaser/Customer	12
	Additional Assistance	12
	Overview	13
2	Database Backup	
	Backing Up the Database Manually	15
	File Copy	15
	SQL Server Management Studio.....	17
	Backing Up the Database Automatically	19
3	Database Recovery	
	File Copy	31
	SQL Server Management Studio	33
4	Verifying Backup and Recovery	
	Verifying Database Backup	37
	Verifying Database Recovery	38

1

Introduction

This chapter provides general information required for the proper use of the system and this manual. Familiarize yourself with this information before using the system.

Manual Information

This section provides information for the correct use of this manual.

Keep this manual with the equipment at all times and periodically review it. You should request training assistance from GE Healthcare, if needed.

Intended Audience

This manual is intended for use by the user who is responsible for backing up and recovering database for the Optima EMS system (also referred to as "the system" throughout this manual).

Manual Purpose

This document provides a detailed description on how to backup and recovery the database of the Optima EMS System.

Document Conventions

This manual uses the following conventions.

Typographical Conventions

Convention	Description
Bold Text	Indicates keys on the keyboard, text to enter, or hardware items such as buttons or switches on the equipment.
<i>Italicized-Bold Text</i>	Indicates software terms that identify menu items, buttons or options in various windows.
CTRL+ESC	Indicates a keyboard operation. A plus (+) sign between the names of two keys indicates that while holding the first key, you should press and release the second key. For example, Press CTRL+ESC means to press and hold the CTRL key and then press and release the ESC key.

Convention	Description
<space>	Indicates that you must press the spacebar. When instructions are given for typing a precise text string with one or more spaces, the point where you must press the spacebar is indicated as <space>. This ensures that the correct number of spaces is inserted in the correct positions within the literal text string. The purpose of the < > brackets is to distinguish the command from the literal text within the string.
Enter	Indicates that you must press the Enter or Return key on the keyboard. Do not type Enter .
>	The greater than symbol, or right angle bracket, is a concise method to indicate a sequence of menu selections. For example, the statement "From the main menu, select System > Setup > Options to open the Option Activation window" replaces the following: <ol style="list-style-type: none"> 1. From the main menu, select System to open the System menu. 2. From the System menu, select Setup to open the Setup menu. 3. From the Setup menu, select Options to open the Option Activation window.

Illustrations

All illustrations in the manual are provided as examples only. Depending on system configuration, screens in the manual may differ from the screens on your system.

All patient names and data are fictitious. Any similarity to actual persons is coincidental.

Notes

Notes provide application tips or additional information that, while useful, are not essential to the correct operation of the system. They are called out from the body text through a flag word and indentation, as follows:

NOTE:

The tip or additional information is indented below the **NOTE** flag word.

Responsibility of the Purchaser/Customer

The customer is responsible for providing appropriate desks, chairs, electrical wall outlets, network connections, analog phone lines, and for locating any of the Optima EMS System components described in this manual in compliance with all local, state or national codes.

Additional Assistance

GE Healthcare maintains a trained staff of application and technical experts to answer questions and respond to issues and problems that may arise during the installation, maintenance, and use of this system.

Contact your local GE Healthcare representative to request additional assistance.

Overview

The Optima EMS system supports two methods for database backup and recovery:

- **Manual**
- **Automatic**

For **Manual** database backup, you may use the following methods:

- **File Copy**
- **SQL Server Management Studio**

NOTE:

Both of these methods may be used in database recovery as well.

For **Automatic** database backup, the **SQL Management Studio** may be used to define the backup policy, frequency, duration and media.

If you only have the **SQL Server Express**, GE Healthcare recommends that the system database is backed up manually.

The recommended media for backups is DVD disk or other similar storage media.

2

Database Backup

Backing Up the Database Manually

There are two ways to back up database manually:

- *File Copy*
- *SQL Server Management Studio*


File Copy

Use the following procedure to back up the system database manually through *File Copy*.

1. On the desktop, select **Start > Run**.
The **Run** window opens.
2. Type **services.msc** in the **Open** field.

- Click **OK**.
The **Services** window opens.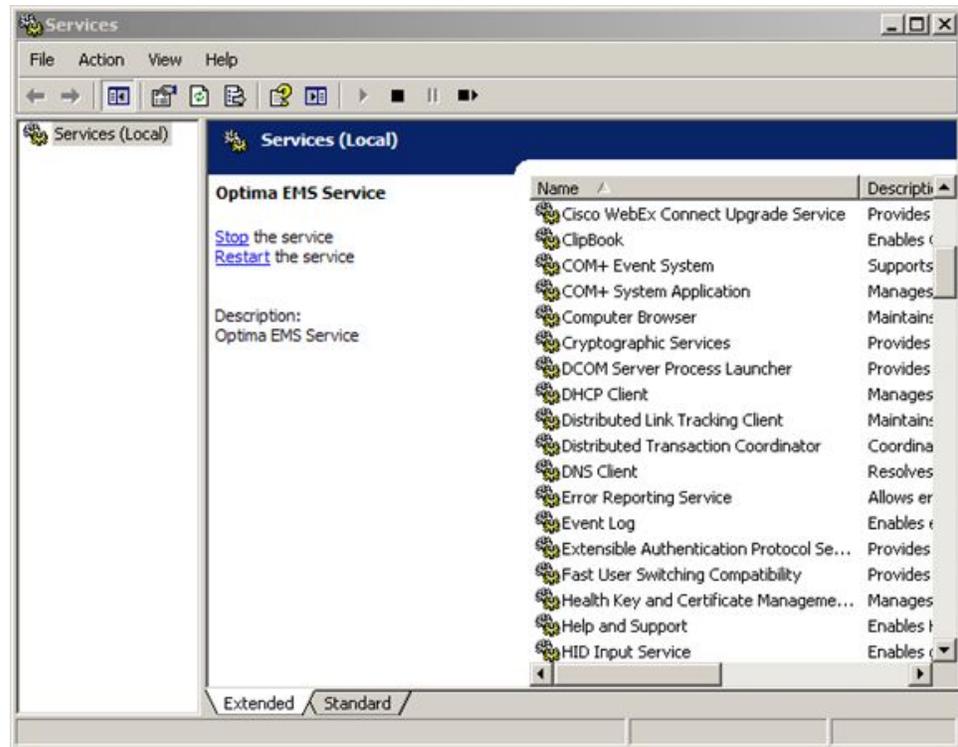


- In the **Services (Local)** list, right-click **SQL Server (MSSQLSERVER)** and select **Stop**.

The **SQL Server (MSSQLSERVER)** service has stopped.

- Open **My Computer** and navigate to the SQL Server installation folder.
The SQL Server installation folder is located in **C:\Program Files\Microsoft SQL Server\MSSQL10_50.MSSQLSERVER\MSSQL\DATA** by default.

6. Select *macit.mdf* and *macit_log.ldf* at the same time.

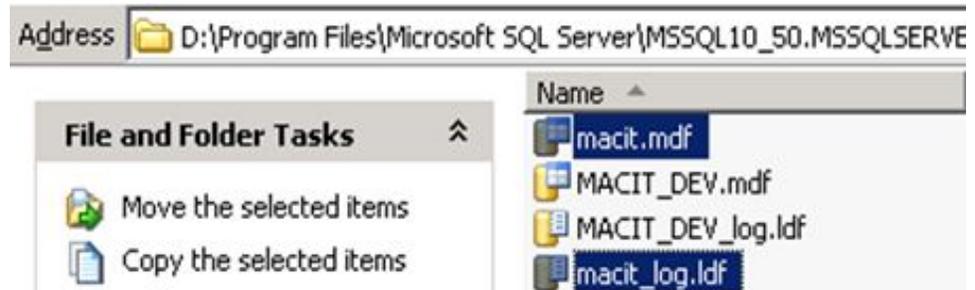7. Copy these two files to a database backup folder. For example *D:\DbBackup*.
8. Repeat Steps 1 to 3 to open the **Services** window.
9. In the **Services (Local)** list, right-click **SQL Server (MSSQLSERVER)** and select **Start**.


The **SQL Server (MSSQLSERVER)** service starts and the manual backup is completed.

SQL Server Management Studio


Use the following procedure to back up the system database manually through **SQL Server Management Studio**.

1. On the desktop, select **Start > All Programs > Microsoft SQL Server 2008 R2 > SQL Server Management Studio**.
2. Click **SQL Server Management Studio** to open it.
The **Connect to Server** window opens.

3. Log on to the SQL Server 2008 R2.
You must enter your password while logging on.

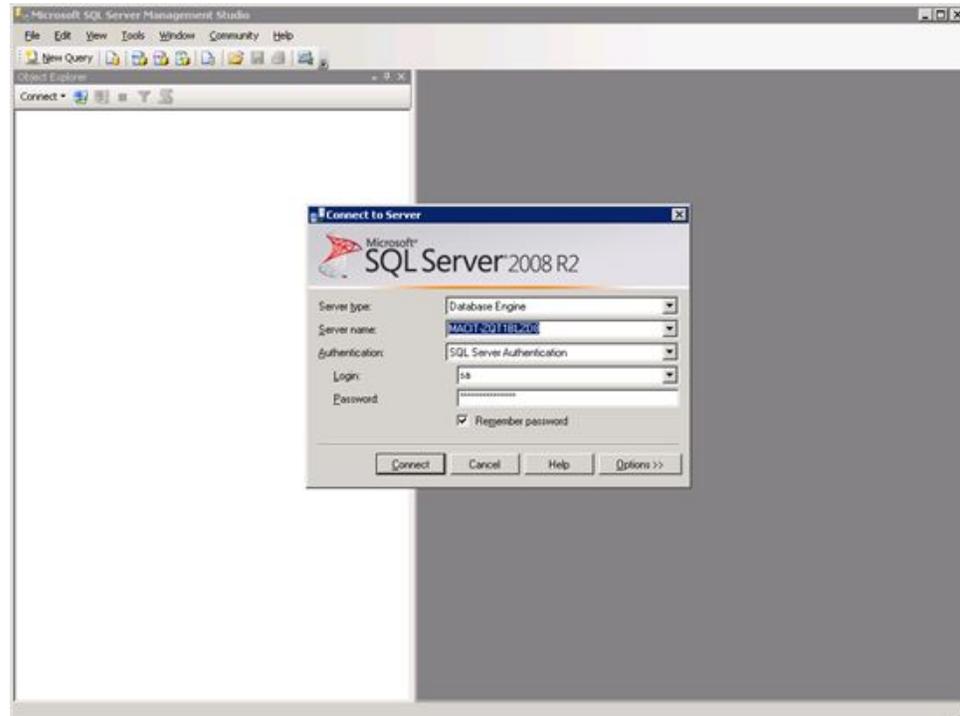
4. Click **Connect**.
5. Expand the **Database** by clicking the + icon.
6. Right-click on **macit** and select **Tasks > Back Up...**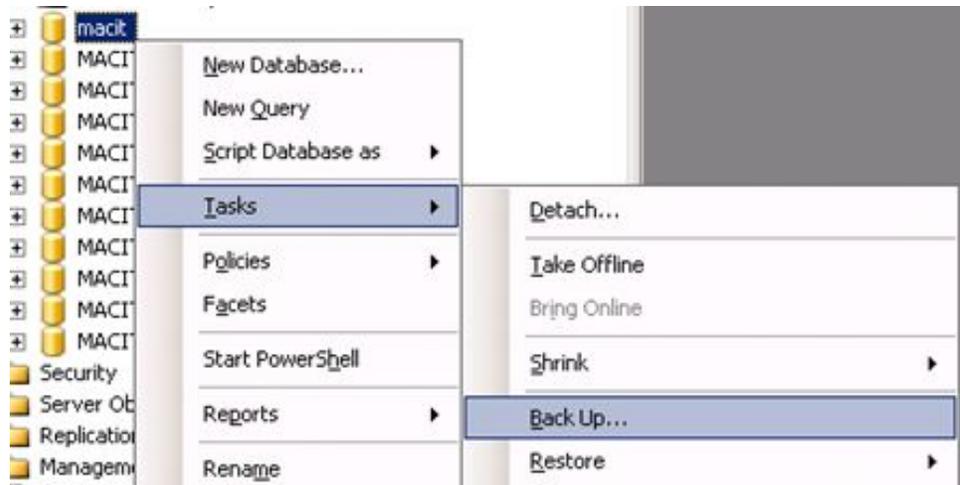

The **Backup Database - macit** window opens.

- Click **Add** to specify the backup destination.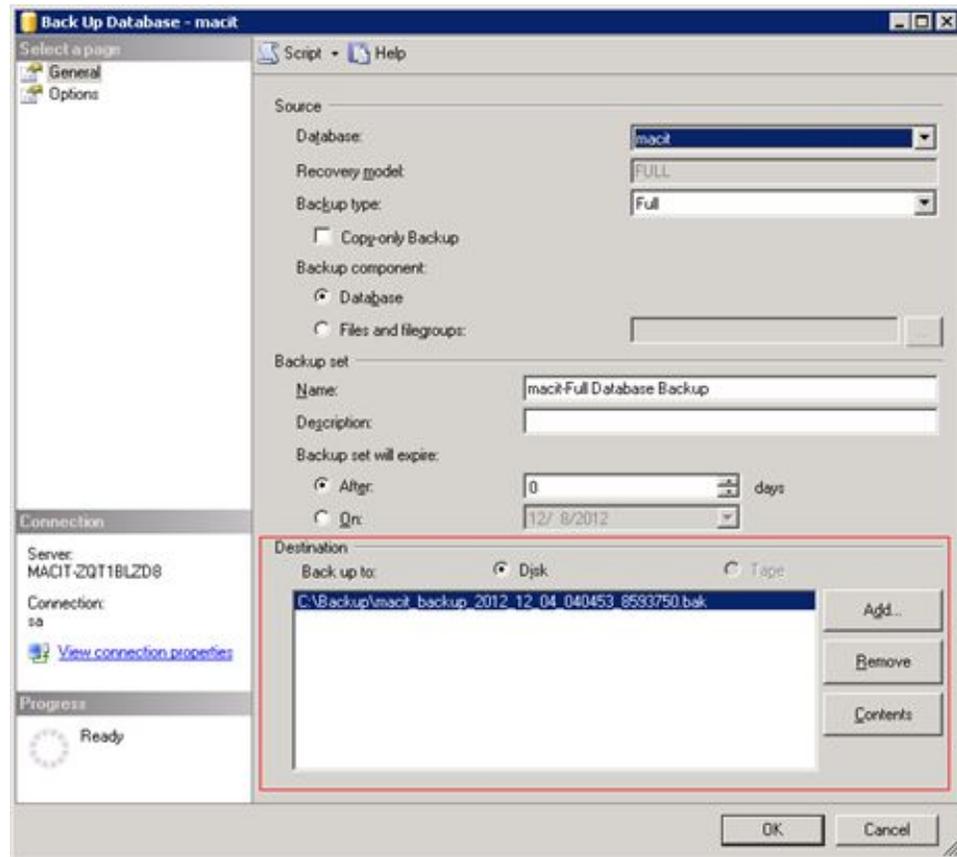


- Click **OK** to complete the backup.

Backing Up the Database Automatically


Use the following procedure to back up the system database automatically using the *Maintenance Plans* in *SQL Server Management Studio 2008*.

- On the desktop, select **Start > All Programs > Microsoft SQL Server 2008 R2 > SQL Server Management Studio**.
- Click **SQL Server Management Studio** to open it.
The **Connect to Server** window opens.

3. Log on to the SQL Server 2008 R2.
You must enter your password while logging on.

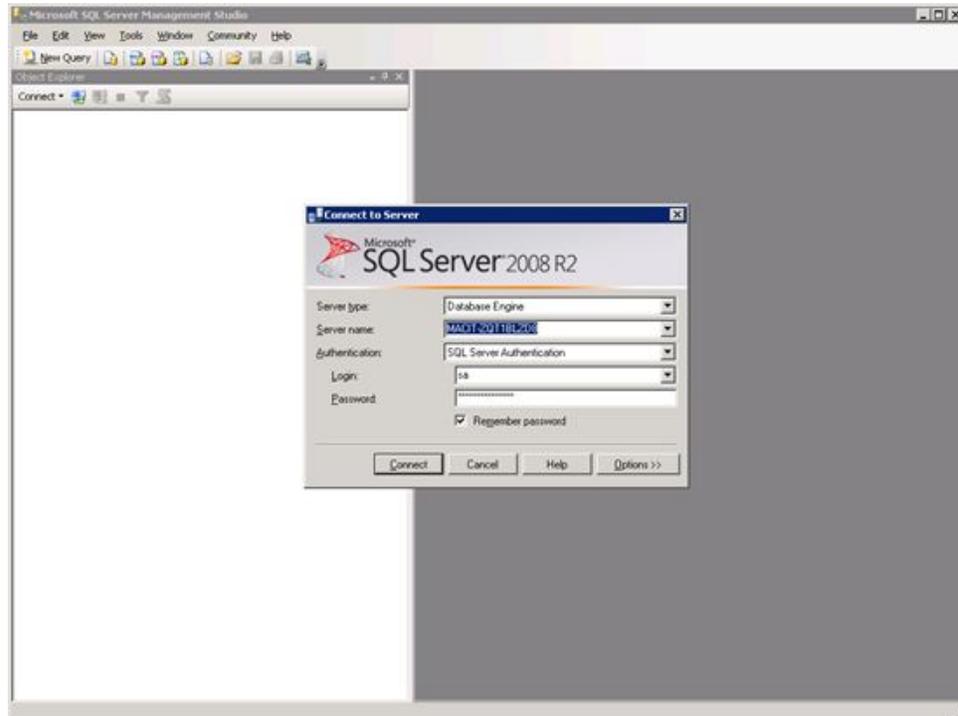
4. Click **Connect**.
5. Click + to expand the **Management** folder.
6. Right-click **Maintenance Plans** and select **New Maintenance Plan...**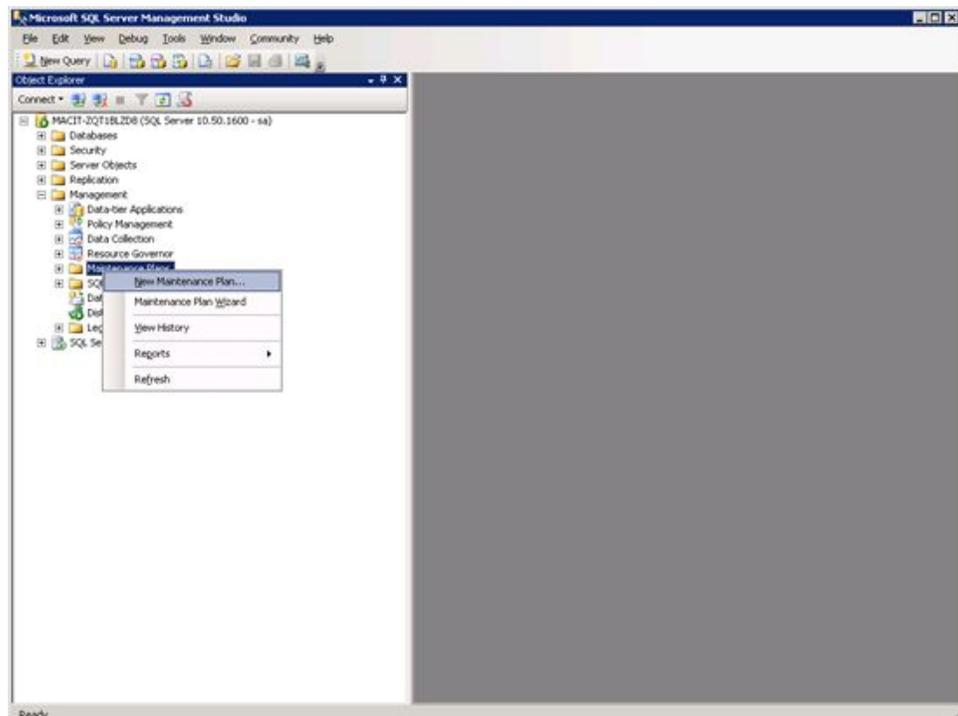


The **New Maintenance Plan** naming window opens.


7. Type the maintenance plan name and click **OK**.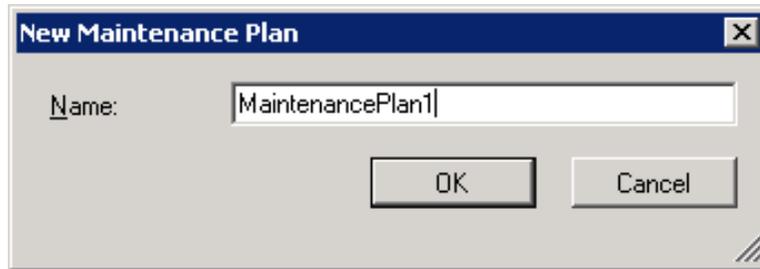


8. From **Maintenance Plan Tasks** panel, drag **Back Up Database Task** and **Maintenance Cleanup Task** to the **Data Backup Maintenance Plan - [Design]** panel to indicate the task execution order.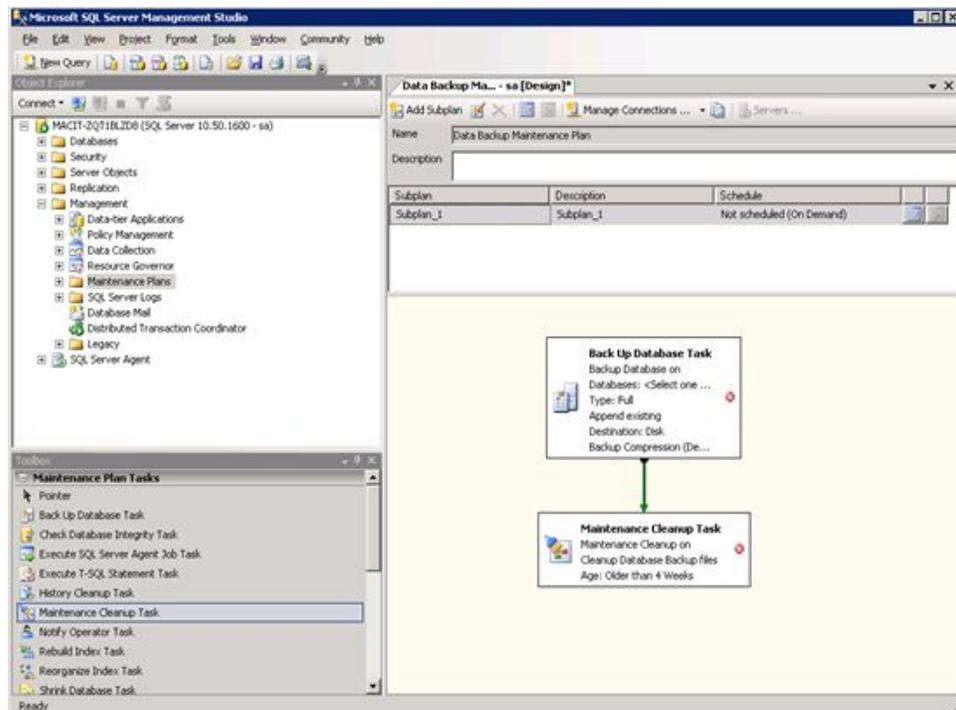

9. Right-click **Back Up Database Task** and select **Edit**.

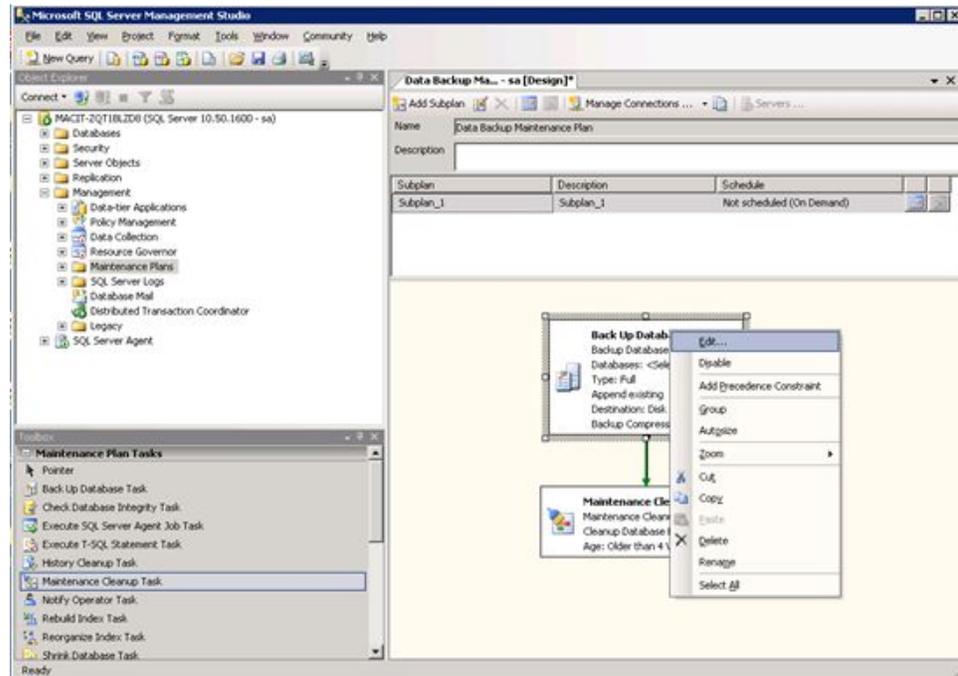
The **Back Up Database Task** window opens.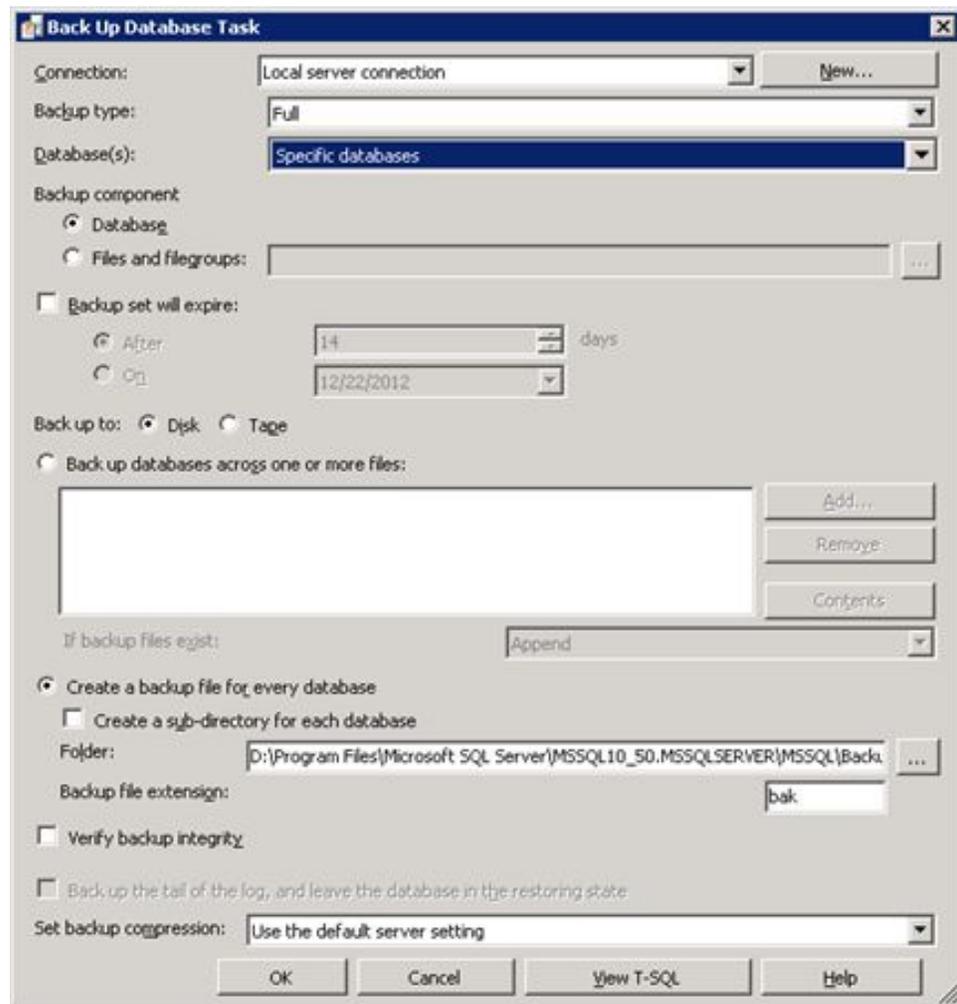


10. In **Databases(s)**, select **Specific databases**.
The **specify databases** window opens.


11. Make sure **These databases** is selected. Select **macit** and click **OK**.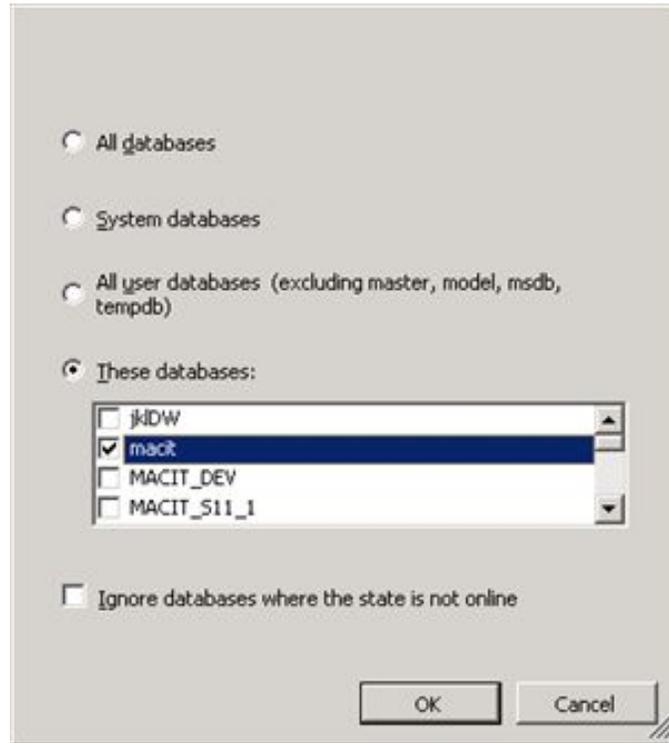

12. In the **Create a backup file for every database**, in the **Folder** box, enter the location where you want the backup written.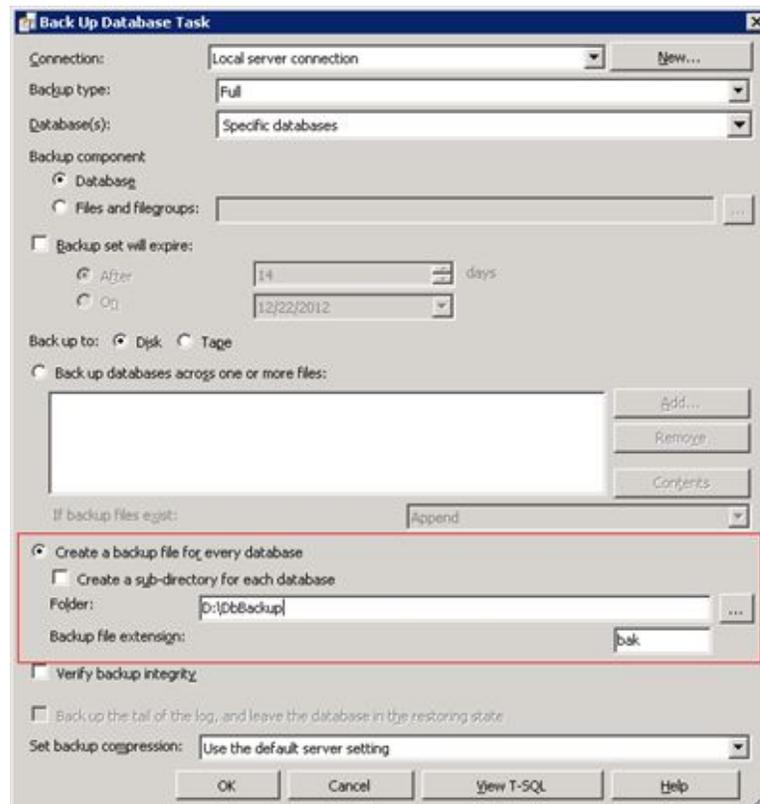

13. Click **OK** to complete the backup database task configuration
14. Right-click **Back Up Database Task** in **Data Backup Maintenance Plan - [Design]** panel and select **Edit**.

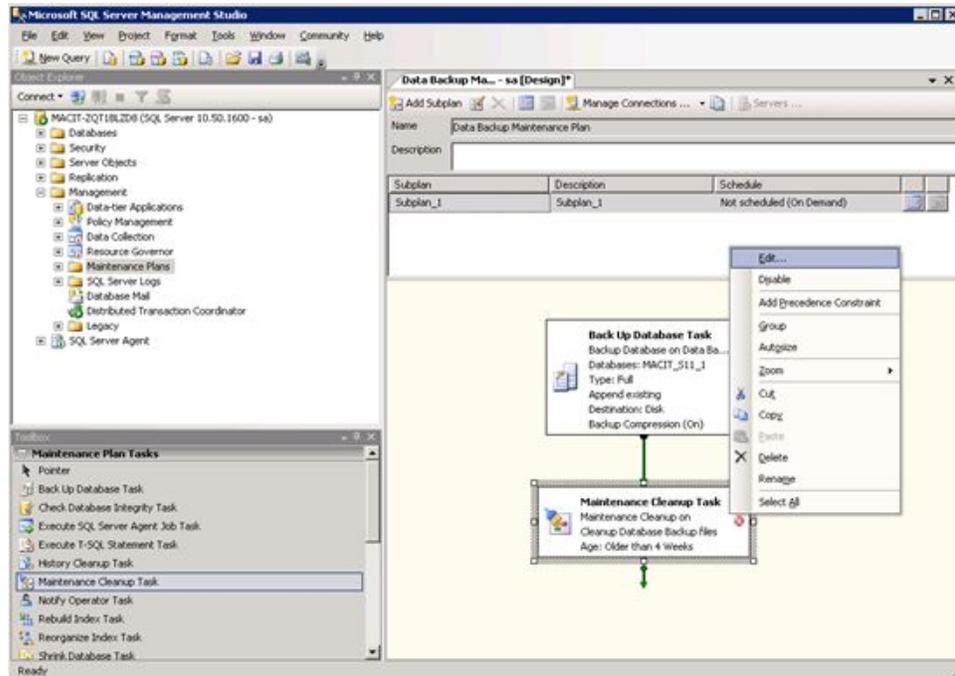The *Maintenance Cleanup Task* window opens.

15. In the *Search folder and delete files based on an extension*, next to the *Folder* box, click the
16. Select the *data backup* folder.

17. Type the **File extension** and set the **File age** as shown:

Maintenance Cleanup Task

Connection: Local server connection New...

Delete files of the following type:

- Backup files
- Maintenance Plan text reports

File location:

- Delete specific file
- Search folder and delete files based on an extension

File name: [] ...

Folder: D:\DbBackup ...

File extension: bak

Include first-level subfolders

File age:

- Delete files based on the age of the file at task run time


Delete files older than the following:

4 Week(s)


OK Cancel View T-SQL Help

18. Click **OK** to complete **Maintenance Cleanup Task** configuration.

- In the **Data Backup Maintenance Plan [Design]** panel, click the **Calendar** icon.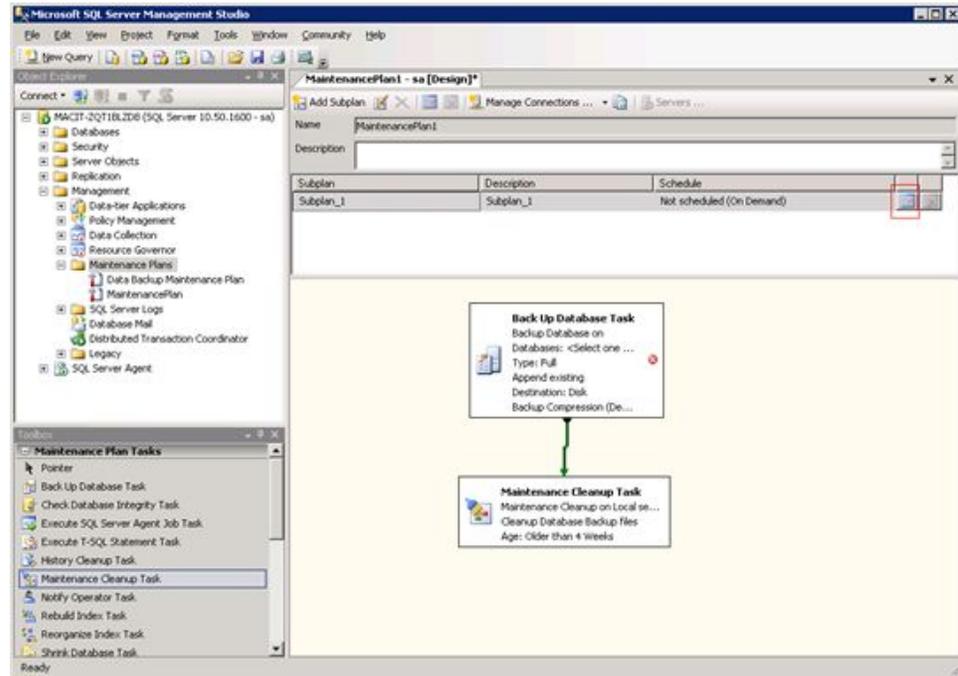

The **Job Schedule Properties** window opens.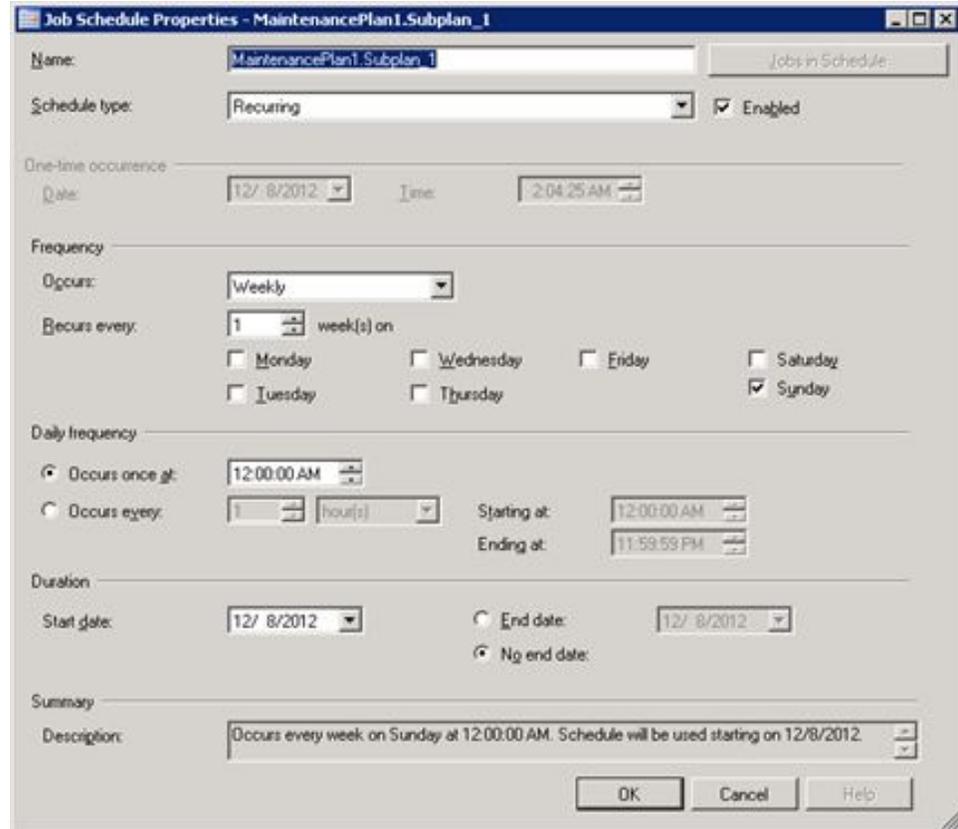

20. Enter the **Name**, **Frequency**, **Daily frequency**, and **Duration** information to define the automatic backup.

The following is an example for your reference only:

- **Name:** Daily Backup
- **Frequency:** Daily
- **Daily Frequency:** Occurs once at 12:00 am
- **Duration:** Start date and no end date

Job Schedule Properties - MaintenancePlan1.Subplan_1

Name: Jobs in Schedule

Schedule type: Enabled

One-time occurrence

Date: Time:

Frequency

Occurs:

Repeat every: day(s)

Daily frequency

Occurs once at:

Occurs every: hour(s)

Starting at:

Ending at:

Duration


Start date: End date:

No end date

Summary

Description:

21. Click **OK** to complete the **Job Schedule Properties** configuration.
22. In the **New Maintenance Plan** window, click the **Save** icon to complete the automatic backup configuration.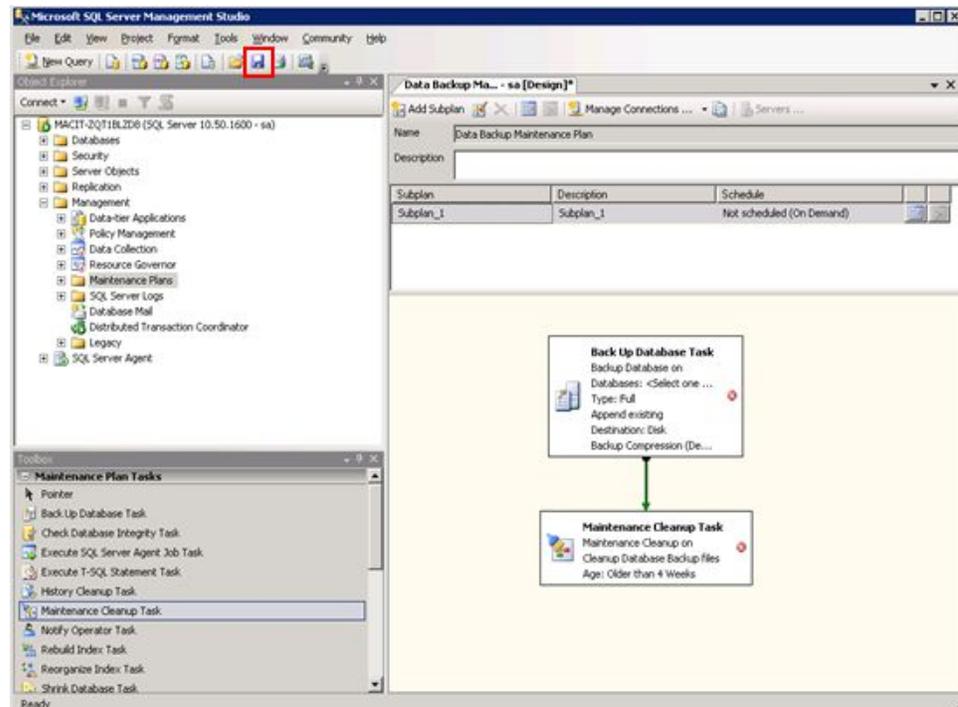

3

Database Recovery


There are two methods to manually recover the database:

- *File Copy*
- *SQL Server Management Studio*


File Copy

Use the following procedure to manually recover the system database using *File Copy*.

1. On the desktop, select **Start > Run**.
The **Run** window opens.
2. Type **services.msc** in the **Open** field.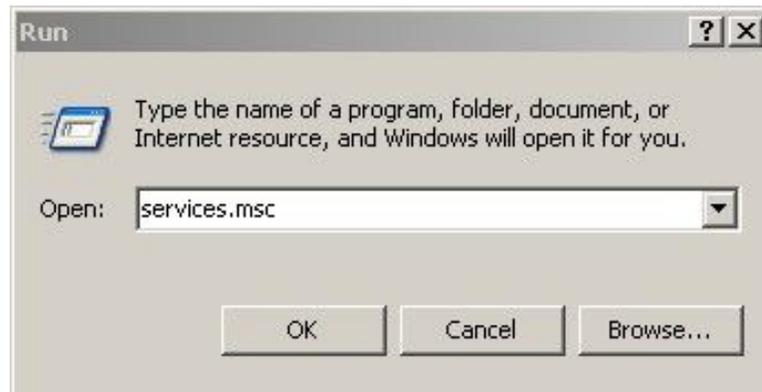

3. Click **OK**.
The **Services** window opens.

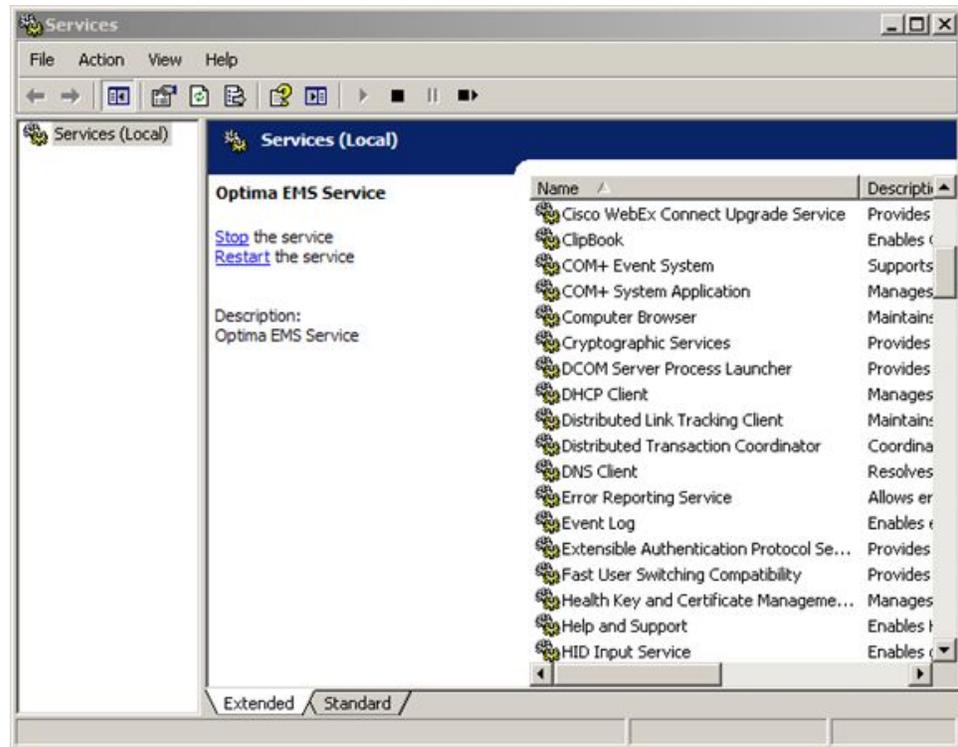
4. In the **Services (Local)** list, right-click **SQL Server (MSSQLSERVER)** and select **Stop**.

The **SQL Server (MSSQLSERVER)** service has stopped.

- Copy the **macit.mdf** and **macit_log.ldf** files from the database backup folder, and paste them to the SQL Server installation folder

The SQL Server installation folder is located in **C:\Program Files\Microsoft SQL Server\MSSQL10_50.MSSQLSERVER\MSSQL\DATA** by default.

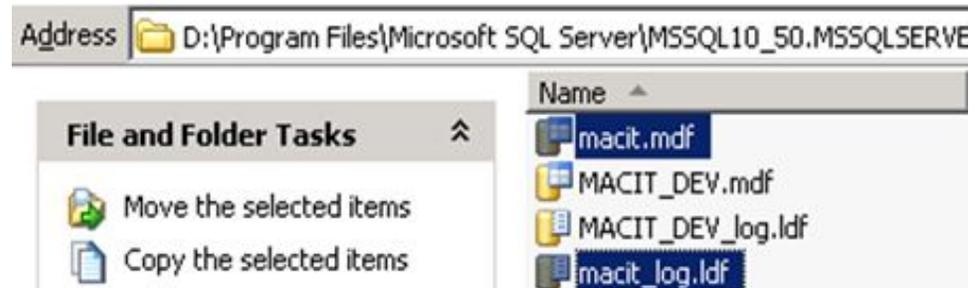- Repeat Steps 1 to 3 to open the **Services** window.
- In the **Services (Local)** list, right-click **SQL Server (MSSQLSERVER)** and select **Start**.


The **SQL Server (MSSQLSERVER)** service starts and the manual database recovery is completed.

SQL Server Management Studio


Use the following procedure to manually recover the system database using **SQL Server Management Studio**.

- On the desktop, select **Start > All Programs > Microsoft SQL Server 2008 R2 > SQL Server Management Studio**.
- Click **SQL Server Management Studio** to open it.
The **Connect to Server** window opens.

3. Log on to the SQL Server 2008 R2.
You must enter your password while logging on.

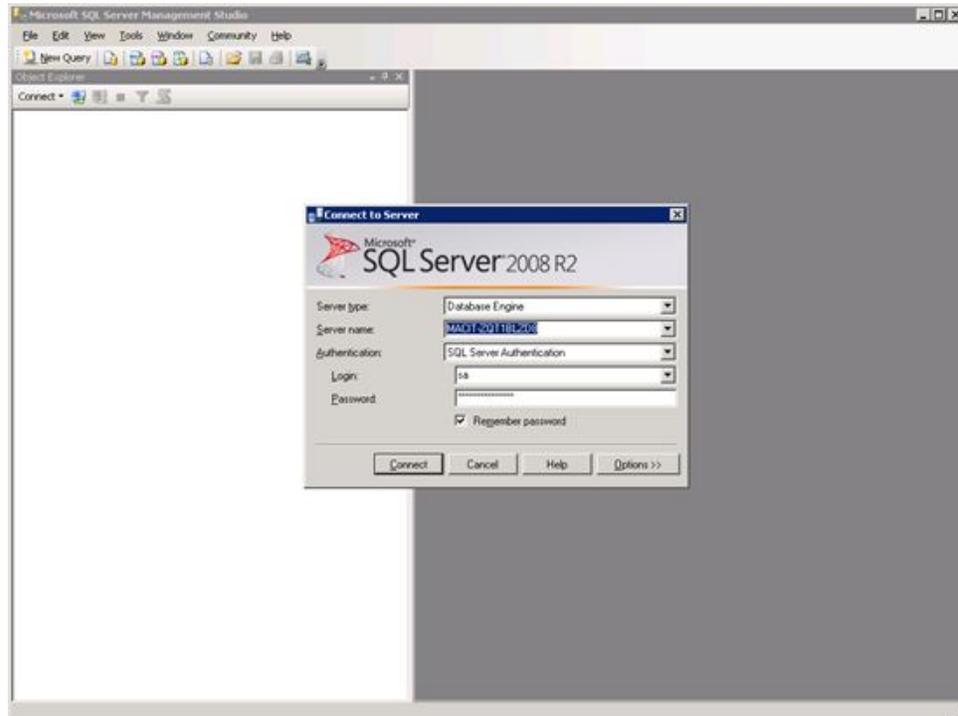
4. Click **Connect**.
5. Expand the **Database** by clicking the + icon.
6. Right-click **macit** and select **Tasks > Restore > Database....**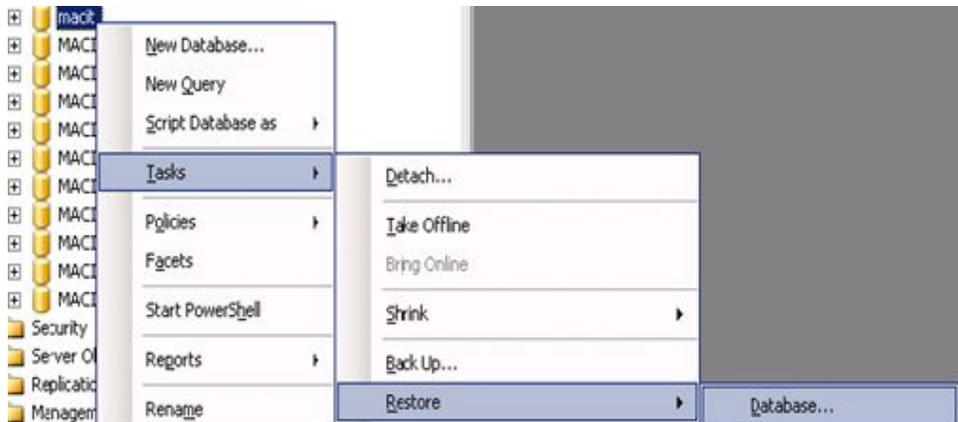


The **Restore Database - macit** window opens.

7. In **Service for restore**, from device click the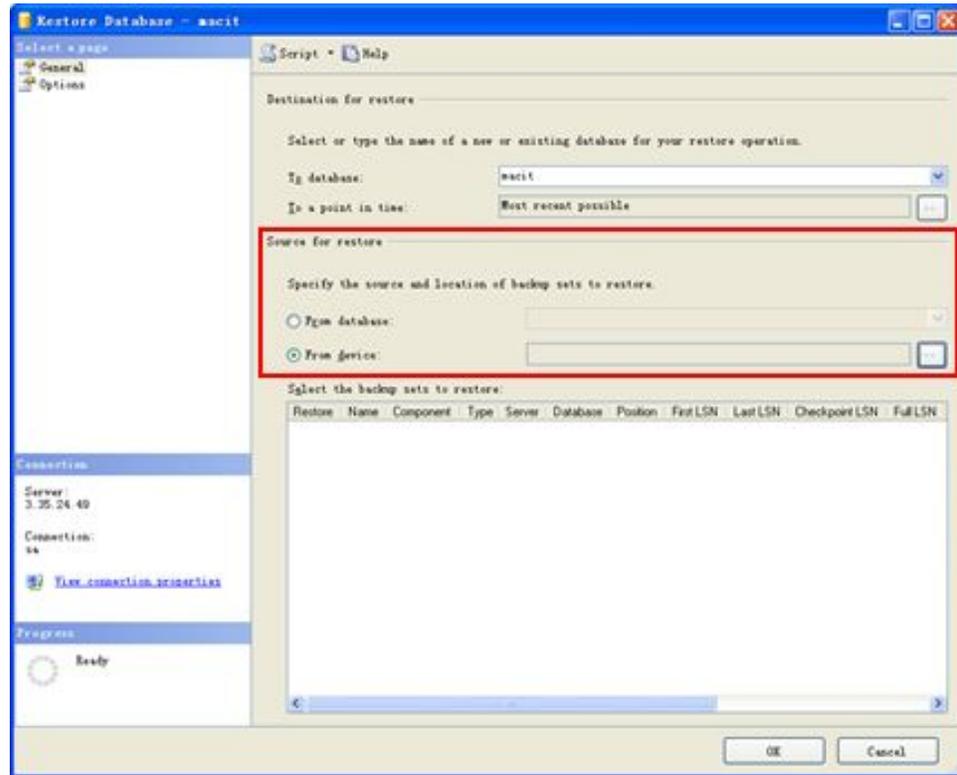

The **Specify Backup** window opens.

8. Click Add to select the path of backup file, and click **OK** to save.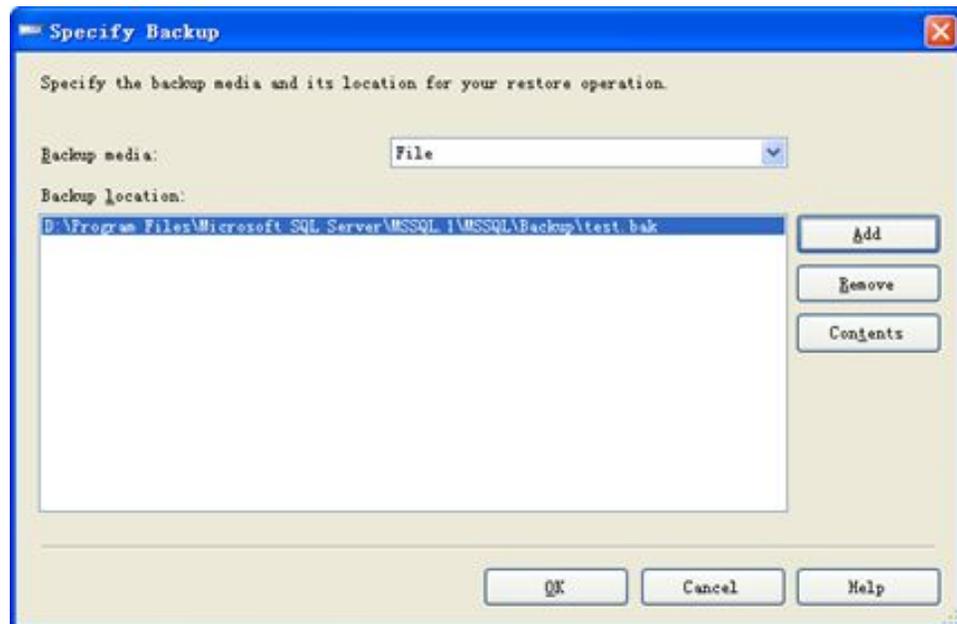


9. In the **Restore Database - macit** window, click **OK** to complete the database recovery.

4

Verifying Backup and Recovery


Verifying Database Backup

1. Open My Computer and navigate to the backup folder. For example, **D:\DbBackup**.
2. Check whether the database backup file exists.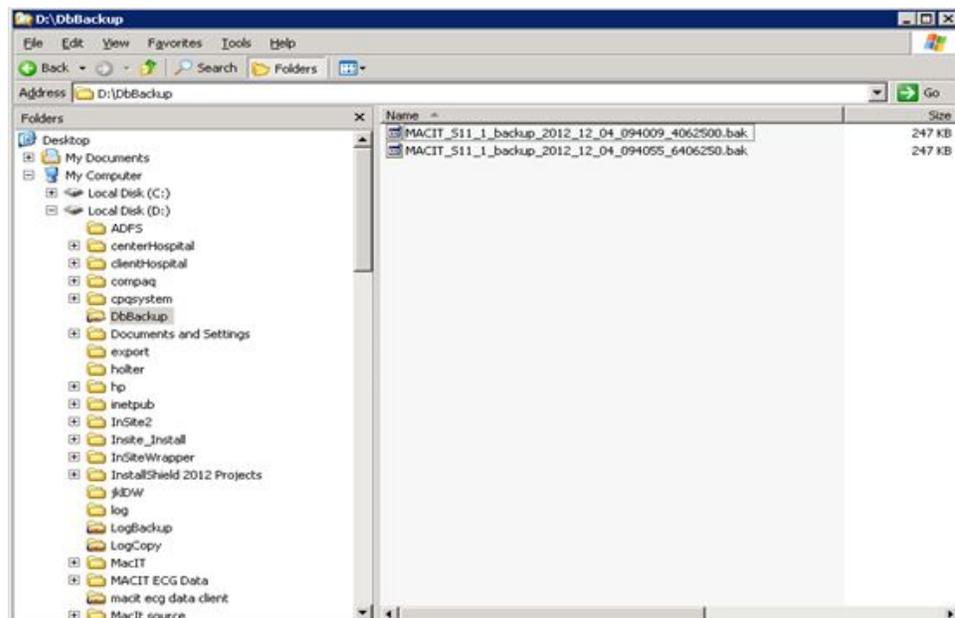


Verifying Database Recovery

1. On the desktop, select **Start > Run**.
The **Run** window opens.
2. Type **services.msc** in the **Open** field.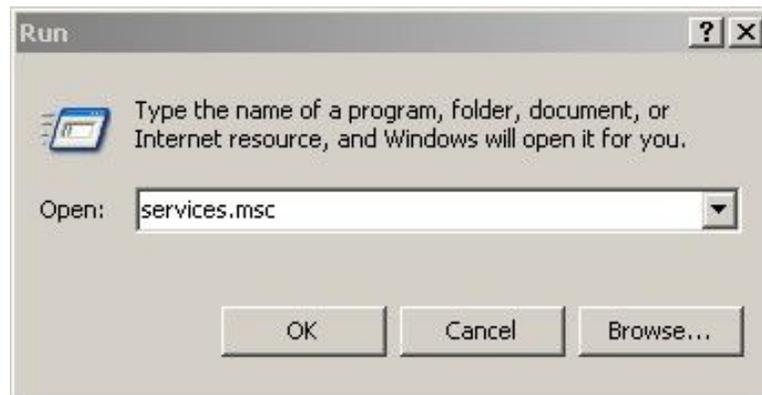

3. Click **OK**.
The **Services** window opens.

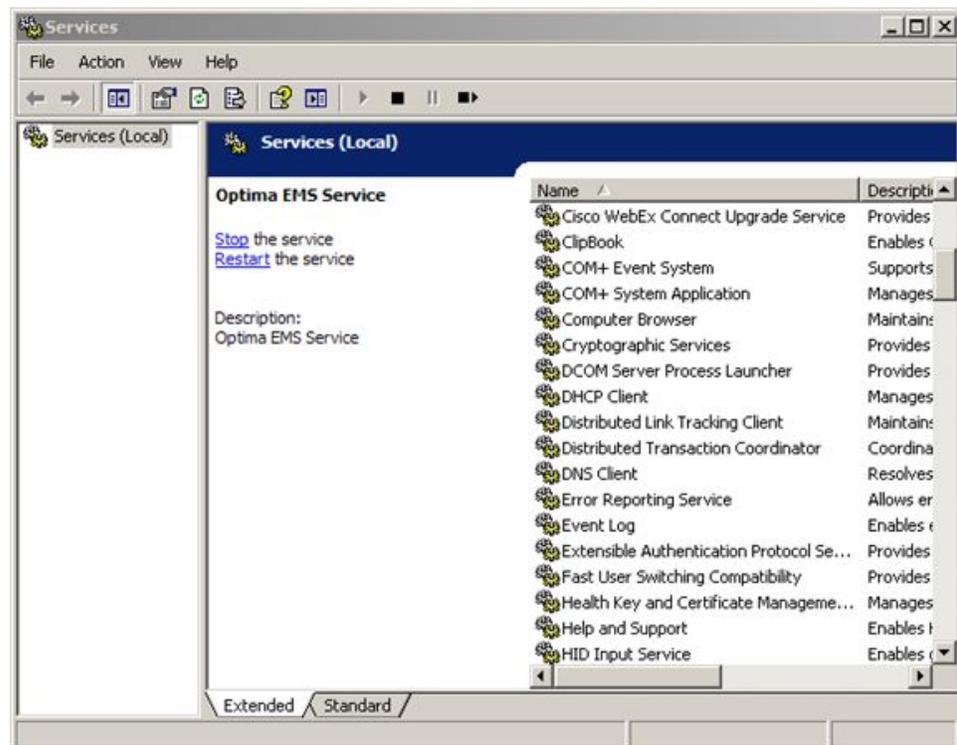4. Find **SQL Server (MSSQLSERVER)** and **Optima EMS Service** in the **Service (Local)** list, and check whether the status of these two services is **Started**.
If the status is not **Started**, right-click on the item and select **Start**.

Name	Description	Status	Startup Type	Log On As
Security Accounts Manager	The startup of this ser...	Started	Automatic	Local System
Server	Supports file, print, an...	Started	Automatic	Local System
Shell Hardware Detection	Provides notifications f...	Started	Automatic	Local System
Smart Card	Manages access to sm...		Manual	Local Service
Special Administration Console...	Allows administrators t...		Manual	Local System
SQL Active Directory Helper S...	Enables integration wit...		Disabled	Network S...
SQL Server (MSSQLSERVER)	Provides storage, proc...	Started	Automatic	Local System
SQL Server Agent (MSSQLSER...	Executes jobs, monitor...	Started	Automatic	Network S...
SQL Server Browser	Provides SQL Server co...		Disabled	Local Service
SQL Server VSS Writer	Provides the interface ...	Started	Automatic	Local System
System Event Notification	Monitors system event...	Started	Automatic	Local System
Task Scheduler	Enables a user to confi...	Started	Automatic	Local System
TCP/IP NetBIOS Helper	Provides support for th...	Started	Automatic	Local Service
Telephony	Provides Telephony AP...	Started	Manual	Local System
Telnet	Enables a remote user ...		Manual	Local Service
Terminal Services	Allows users to connec...	Started	Manual	Local System
Terminal Services Session Dire...	Enables a user connect...		Disabled	Local System
Themes	Provides user experien...		Disabled	Local System
TightVNC Server			Manual	Local System
Uninterruptible Power Supply	Manages an uninterrup...		Manual	Local Service
Virtual Disk Service	Provides software volu...		Manual	Local System
Visual Studio 2008 Remote De...	Allows members of the ...		Disabled	Local System
VisualSVN Server	VisualSVN Server	Started	Automatic	Network S...

Name	Description	Status	Startup Type	Log On As
Net Logon	Maintains a secure cha...		Manual	Local System
Net.Tcp Port Sharing Service	Provides ability to shar...		Manual	Local Service
NetMeeting Remote Desktop S...	Enables an authorized ...		Manual	Local System
Network Connections	Manages objects in the ...	Started	Manual	Local System
Network DDE	Provides network trans...		Disabled	Local System
Network DDE DSDM	Manages Dynamic Data...		Disabled	Local System
Network Location Awareness (...)	Collects and stores net...	Started	Manual	Local System
Network Provisioning Service	Manages XML configur...		Manual	Local System
NT LM Security Support Provider	Provides security to re...		Manual	Local System
Optima EMS Service	Optima EMS Service	Started	Automatic	Local System
Performance Logs and Alerts	Collects performance d...		Automatic	Network S...
Plug and Play	Enables a computer to ...	Started	Automatic	Local System
Portable Media Serial Number ...	Retrieves the serial nu...		Manual	Local System
Print Spooler	Manages all local and n...	Started	Automatic	Local System
Protected Storage	Protects storage of se...	Started	Automatic	Local System
QQBrowser Software Updater	QQ浏览器自动升级服务	Started	Automatic	Local System
QUESTRA SERVICE AGENT		Started	Automatic	Local System
Remote Access Auto Connecti...	Creates a connection t...		Manual	Local System
Remote Access Connection Ma...	Creates a network con...	Started	Manual	Local System
Remote Desktop Help Session ...	Manages and controls ...		Manual	Local System

5. Double-click the Optima EMS System icon on the desktop.
Check whether you can log on the system successfully.

GE Medical Systems
Information Technologies, Inc.
8200 West Tower Avenue
Milwaukee, WI 53223 USA
Tel: +1 414 355 5000
+1 800 558 7044 (US Only)
Fax: +1 414 355 3790

Asia Headquarters

GE Medical Systems
Information Technologies, Inc.
Asia; GE (China) Co., Ltd.
1 Huatuo Road
Zhangjiang Hi-tech Park Pudong
Shanghai, People's Republic of China 201203
Tel: +86 21 3877 7888
Fax: +86 21 3877 7451

GE Medical Systems *Information Technologies, Inc.*, a General Electric Company, going to market as GE Healthcare.

www.gehealthcare.com