

GE Healthcare

AMX-4+ Documentation (Model 2169360, 2236420 & 2275938 Series)



Direction 2166910-100, Revision 24
Copyright© 1996 By General Electric Co.
All Rights Reserved

THIS PAGE INTENTIONALLY LEFT BLANK.

REVISION HISTORY

REV	DATE	REASON FOR CHANGE
A	October, 1996	Initial documentation.
0	Dec. 13, 1996	Initial release.
1	Mar. 7, 1997	Changes to 2173221-100, 2173222-100, 2173224-100, 2173225-100, 2173227-100, 2173228-100, 2173229-100. Deleted 46-017538.
2	Mar. 28, 1997	Change to 2173228-100.
3	June 24, 1997	Changes to 2173225-100, 2173227-100, 2173228-100, 2173229-100.
4	Aug. 14, 1997	High Impact Inspection: Changes to 2173221-100, 2173222-100, 2173223-100, 2173224-100, 2173225-100, 2173227-100, 2173228-100, 2173229-100. Added 46-013288, 2190775-100 and 2196272-100.
5	Sept. 16, 1997	Added reference to Keithly Non-Invasive Divider to 2173221-100 and 2173223-100. Updated 2173221-100 and 2173223-100.
6	Dec. 1, 1997	Changes to 2173228-100.
7	Dec. 10, 1997	Changes to 2173222-100 and 2173228-100.
8	Sept. 23, 1998	Changes to 2173228-100.
9	Dec. 7, 1998	Updated 2173228-100.
10	Apr. 12, 1999	Added 2237257-100. Revised 2173221-100, 2173222-100, 2173223-100, 2173224-100, 2173225-100, 2173227-100, 2173229-100, 46-013288.
11	May 6, 1999	Revised 2173224-100 and 2237257-100.
12	July 14, 1999	Revised 2173224-100.
13	Mar. 7, 2000	Revised 2173228-100.
14	Aug. 8, 2000	Revised 2173228-100 and 2237257-100.
15	Nov. 8, 2000	Added 2275938-100 Series to 2173221-100, 2173222-100, 2173223-100, 2173224-100, 2173225-100, 2173227-100, 2237257-100, 2173229-100. Corrected part # of spanner wrench to 46-404295G1 in 2196272-100.
16	Dec. 13, 2001	Revised 2173228-100 and 2237257-100.
17	Aug. 08, 2002	Revised 2173227-100.
18	Sept. 11, 2002	Revised 2173225-100.
19	Sept. 16, 2002	Revised 2237257-100.
20	July 30, 2003	Revised 2173222-100.
21	Nov. 20, 2003	Revised 2173221-100 and 2237257-100.
22	Feb. 2, 2004	Revised 2173224-100 and 2237257-100.
23	Oct. 21, 2004	Revised 2173221-100, 2173225-100 and 2237257-100 to include new collimator 46-270615P3, new Display Control Bd. 2409241 and Display Bd. 2345236. Revised 2173229-100 to include new Display Control Bd. and Display Bd.
24	Dec. 05, 2006	Revision numbers removed from 2166910-100 Table of contents, in order to minimize the effects caused by changes to the documents in the TABs.

THIS PAGE INTENTIONALLY LEFT BLANK.

TABLE OF CONTENTS

TAB	NUMBER	TITLE
1	2173221-100	AMX-4+ RATINGS AND SPECIFICATIONS (MODEL 2169360, 2236420 & 2275938 SERIES)
2	2173222-100	AMX-4+ INSTALLATION (MODEL 2169360, 2236420 & 2275938 SERIES)
3	2173223-100	AMX-4+ CALIBRATION (MODEL 2169360, 2236420 & 2275938 SERIES)
4	46-013288	BLEEDER, HIGH-VOLTAGE DUAL TYPE T8005G AND C1515A CONNECTION.... APPLICATIONS
5	2173224-100	AMX-4+ FUNCTIONAL CHECKS (MODEL 2169360, 2236420 & 2275938 SERIES)
6	2173225-100	AMX-4+ SERVICE (MODEL 2169360, 2236420 & 2275938 SERIES)
7	2196272-100	HIGH VOLTAGE CABLE INSTALLATION AND TROUBLESHOOTING PROCEDURES
8	2173227-100	AMX-4+ PERIODIC MAINTENANCE (MODEL 2169360, 2236420 & 2275938 SERIES)
9	2190775-100	STRANDED STEEL CABLE INSPECTION
10	2173228-100	AMX-4+ RENEWAL PARTS (MODEL 2169360 SERIES)
	2237257-100	AMX-4+ RENEWAL PARTS (MODEL 2236420 & 2275938 SERIES)
11	2173229-100	AMX-4+ SCHEMATICS (MODEL 2169360, 2236420 & 2275938 SERIES)

THIS PAGE INTENTIONALLY LEFT BLANK.



GE Healthcare

GE Healthcare — Telex 3797371
P.O. Box 414, Milwaukee, Wisconsin 53201 U.S.A.
(Asia, Pacific, Latin America, North America)

GE Healthcare — Europe: Telex 698626
283, rue de la Minière, B.P. 34, 78533 Buc Cedex
France