

PARTS LIST MANUAL

PRIMA™ Prevention Mattress Series From Hill-Rom



**Product No. P576/P577/P578/P708/P709/
P710/P711/P712/P713**

For Parts Or Technical Assistance
USA (800) 445-3720 Canada (800) 267-2337
International: Contact your distributor.

man162ra

PRIMA™ Prevention Mattress Series Parts List Manual

Revisions

Revision Letter	Pages Affected	Date
Original Issue		March, 1998
A	All	September, 2000

man162ra

© 2000 by Hill-Rom Company, Inc. ALL RIGHTS RESERVED.

No part of this text shall be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information or retrieval system without written permission from Hill-Rom Company, Inc. (Hill-Rom).

Second Edition

First Printing 1998

Printed in the USA

Anti-Shear™ is a trademark of Hill-Rom, Inc.

Hill-Rom® is a registered trademark of Hill-Rom, Inc.

HY•TEX Bi-Stretch™ is a trademark of Hill-Rom Company, Inc.

HY•TEX Nylon™ is a trademark of Hill-Rom Company, Inc.

PRIMA™ is a trademark of Hill-Rom Company, Inc.

PRIMA SHEAR-LESS™ is a trademark of Hill-Rom, Inc.

PRIMA SHEAR-LESS PLUS™ is a trademark of Hill-Rom, Inc.

Swiftach® is a registered trademark of Dennison Manufacturing Company.

The information contained in this manual is subject to change without notice. Hill-Rom makes no commitment to update or keep current, the information contained in this manual.

The only product warranty intended by Hill-Rom is the express, written warranty accompanying the bill of sale to the original purchaser. Hill-Rom makes no other warranty, express or implied, and in particular, makes no warranty of merchantability or fitness for a particular purpose.

Additional copies of this manual can be obtained from Hill-Rom.

To order additional copies of this manual, call (800) 445-3720, and place a parts order for part number man162ra.

Chapter 1

Parts List

Chapter Contents

Model Identification	1 - 3
Warranty	1 - 7
Service Parts Ordering	1 - 9
Exchange Policy	1 - 11
In-Warranty Exchanges	1 - 11
Out-of-Warranty Exchanges	1 - 11
Recommended Spare Parts	1 - 12
PRIMA SHEAR-LESS PLUS™ Prevention Mattress and PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—P576, P577, and P578	1 - 14
PRIMA SHEAR-LESS™ Prevention Mattress—P708, P709, and P710.	1 - 18
PRIMA™ Prevention Mattress—P711, P712, and P713	1 - 22
Installation	1 - 25
General Cleaning and Inspection	1 - 27
Mattress Disinfection	1 - 27
Exterior Mattress Inspection.	1 - 27
Interior Mattress Inspection (PRIMA SHEAR-LESS PLUS™ Prevention Mattress, PRIMA SHEAR-LESS PLUS™ II Prevention Mattress, and PRIMA SHEAR-LESS™ Prevention Mattress Only)	1 - 28
Flame Barrier (PRIMA SHEAR-LESS PLUS™ Prevention Mattress, PRIMA SHEAR-LESS PLUS™ II Prevention Mattress, and PRIMA SHEAR-LESS™ Prevention Mattress with Flame Barrier Only)	1 - 30
Installation	1 - 30
Flame Barrier (Standard Model PRIMA™ Prevention Mattress with Flame Barrier Only).	1 - 32
Installation	1 - 32

NOTES:

Model Identification

For PRIMA SHEAR-LESS PLUS™ Prevention Mattress model identification, see table 1-1 on page 1-3.

Table 1-1. PRIMA SHEAR-LESS PLUS™ Prevention Mattress Model Identification

Part Number	Description
P576CA1	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P576EA1	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier
P577CA1	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P577EA1	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier
P578CA1	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P578EA1	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier

For PRIMA SHEAR-LESS PLUS™ II Prevention Mattress model identification, see table 1-2 on page 1-4.

Table 1-2. PRIMA SHEAR-LESS PLUS™ Prevention Mattress Model Identification

Part Number	Description
P576CA3	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P576EA3	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier
P577CA3	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P577EA3	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier
P578CA3	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover
P578EA3	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Bi-Stretch™ Cover with flame barrier

For PRIMA SHEAR-LESS™ Prevention Mattress model identification, see table 1-3 on page 1-4.

Table 1-3. PRIMA SHEAR-LESS™ Prevention Mattress Model Identification

Part Number	Description
P708CA1	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover
P708EA1	76" (193 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover with flame barrier
P709CA1	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover
P709EA1	80" (203 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover with flame barrier
P710CA1	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover
P710EA1	84" (213 cm), Anti-Shear™ Inner Liner, HY•TEX Nylon™ Cover with flame barrier

For PRIMA™ Prevention Mattress Series model identification, see table 1-4 on page 1-5.

Table 1-4. PRIMA™ Prevention Mattress Series Model Identification

Part Number	Description
P711CA1	76" (193 cm), standard model, sewn (and zippered) vinyl/nylon ticking
P711EA1	76" (193 cm), standard model, sewn (and zippered) vinyl/nylon ticking with flame barrier
P711CA2	76" (193 cm), psychiatric model, sewn vinyl/nylon ticking
P711EA2	76" (193 cm), psychiatric model, sewn vinyl/nylon ticking with flame barrier
P712CA1	80" (203 cm), standard model, sewn (and zippered) vinyl/nylon ticking
P712EA1	80" (203 cm), standard model, sewn (and zippered) vinyl/nylon ticking with flame barrier
P712CA2	80" (203 cm), psychiatric model, sewn vinyl/nylon ticking
P712EA2	80" (203 cm), psychiatric model, sewn vinyl/nylon ticking with flame barrier
P713CA1	84" (213 cm), standard model, sewn (and zippered) vinyl/nylon ticking
P713EA1	84" (213 cm), standard model, sewn (and zippered) vinyl/nylon ticking with flame barrier
P713CA2	84" (213 cm), psychiatric model, sewn vinyl/nylon ticking
P713EA2	84" (213 cm), psychiatric model, sewn vinyl/nylon ticking with flame barrier

NOTES:

Warranty

PRIMA™ Prevention Mattress Series LIMITED WARRANTY

Hill-Rom, Inc. (Hill-Rom) has a long tradition of providing superior products and service to our customers. Our goal is “Total Customer Satisfaction”. In that spirit, Hill-Rom is proud to offer the following warranty.

SPECIFIC WARRANTY

This warranty applies only to defects in workmanship and materials and only to damages arising from normal usage on a non-prorated basis for a period of five (5) years for foam and internal components and three (3) years for cover and external components from the date of original purchase. Any damages that arise from abnormal use such as those caused by improper handling, cleaning, or negligent use are excluded from this warranty. Hill-Rom’s obligation under this warranty is expressly limited to supplying replacement parts or replacing, at its option, any mattress which is, in the sole discretion of Hill-Rom, found to be defective. No returns, allowances, credits, discounts, charge backs, or other deductions will be made without Hill-Rom’s prior written authorization. This warranty cannot be modified by any employee or representative of Hill-Rom or any agent thereof except in writing signed by an officer of Hill-Rom.

Shipping and handling charges incurred in connection with this warranty during the first year will be paid by Hill-Rom, but thereafter are the purchaser’s sole responsibility.

The purchaser is hereby advised that some fabric will stretch, and all foam will lose some of its resiliency during its life. This is normal and is not included in this or any other warranty applicable to this product. This warranty is void if the manufacturer’s tag is removed.

TO OBTAIN PARTS AND SERVICE

In the United States, call Hill-Rom Technical Support Department at (800) 445-3720, Monday through Friday. In Canada, call Hill-Rom Technical Support Department at (800) 267-2337, Monday through Friday. Outside the United States and Canada, call your authorized Hill-Rom Distributor. In order to expedite service, we request you furnish the following information: customer identification number, product model number, serial number, and description of problem. A qualified specialist will provide, via telephone (United States and Canada), or FAX (Outside the United States and Canada), troubleshooting assistance and provide necessary parts to make repairs.

OUT OF WARRANTY EXCHANGE POLICY

After the expiration of the original warranty, upon request, Hill-Rom will ship replacement components. Please call Hill-Rom Technical Support Department for current pricing.

PARTS AVAILABILITY POLICY

Hill-Rom will offer parts for the PRIMA Prevention Mattress Series for ten (10) years from date of sale.

Note: Some original component parts and assemblies may not be available; functional equivalents may be substituted.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES AND IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. HILL-ROM’S OBLIGATION UNDER THESE WARRANTIES SHALL NOT INCLUDE ANY LIABILITY FOR LOSS OF PROFITS, DIRECT, INDIRECT OR CONSEQUENTIAL DAMAGES OR DELAYS. Some states, provinces, or countries do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusion or limitation may not apply. These warranties provide specific legal rights; but, there may be other available rights, which vary from state to state, province to province, or country to country.

Revised March 12, 1998

(war006)

Hill-Rom Company, Inc., 1069 State Route 46 E, Batesville, IN 47006-9167

Warranty


Chapter 1: Parts List

NOTES:

Service Parts Ordering

Using the parts lists in this manual, identify the part number(s) you require. Find the product number and serial number on the product identification label (A) (see figure 1-1 on page 1-9).

Figure 1-1. Product Identification Label Location

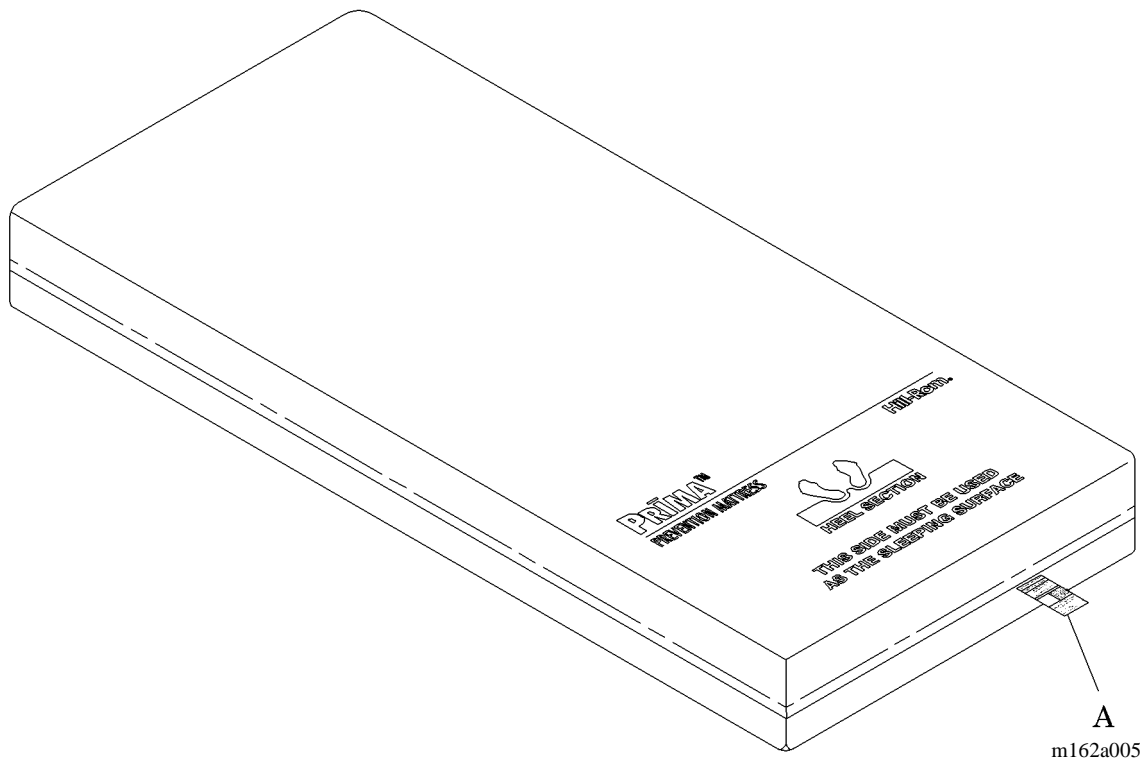

Call Hill-Rom Technical Support at (800) 445-3720 with the following information:

- Six-digit customer account number
- Purchase order number
- Product number
- Serial number
- Part number(s)

To promptly order parts, request part prices and availability, or follow up on a service order, use the following Hill-Rom fax number:

(812) 934-8472

Terms:

- Net 30 days
- F.O.B. Batesville, IN
- Prepaid shipping charges added to invoice
- All orders shipped UPS ground unless specified

Address all inquiries to:

ATTN TECHNICAL SUPPORT—PARTS
HILL-ROM COMPANY, INC.
1069 STATE ROUTE 46 E
BATESVILLE IN 47006-9167

Address all return goods to:

ATTN SERVICE STORES
DISTRIBUTION CENTER DOOR D23
HILL-ROM COMPANY, INC.
COUNTY ROAD 300E
BATESVILLE IN 47006-9167

NOTE:

To eliminate possible delays or incorrect billings, **do not** return any items without a Return Material Authorization (RMA) number. When a return is requested, an RMA packet is included with each order. This packet includes an RMA number, instructions, and a shipping label. If an RMA number is not available, obtain one by phoning Hill-Rom Technical Support at (800) 445-3720.

Exchange Policy

The following are policies for in-warranty and out-of-warranty exchanges from Hill-Rom.

In-Warranty Exchanges

In some cases, Hill-Rom will request that parts/products be returned for inspection. When this occurs, you are expected to return parts/products within 30 days of receipt of the exchange part. If you fail to return the inoperative parts/products within the 30 day period, Hill-Rom will invoice your facility for the full selling price of the parts/products.

NOTE:

The preceding billing procedure pertains **only** to parts/products that Hill-Rom requests to be returned.

In some cases, the invoice accompanying the parts will show the full selling price (only for internal use at Hill-Rom). Do not confuse this price with your price.

Do not return any parts without an RMA number. When parts/products have been requested to be returned, Hill-Rom will include an RMA packet with the parts/products shipment. If an RMA number is not available, obtain one by phoning Hill-Rom Technical Support at (800) 445-3720.

Out-of-Warranty Exchanges

You are expected to return the inoperative parts/products within 30 days of receipt of the exchange part. Hill-Rom will include an RMA packet with the parts/products shipment. If an RMA number is not available, obtain one by phoning Hill-Rom Technical Support at (800) 445-3720. Hill-Rom will invoice your facility for the full selling price of the parts/products. Upon return of the inoperative parts/products, Hill-Rom will issue a credit to your facility for **the difference between the exchange price and the full selling price of the parts/products**.


Recommended Spare Parts

There are no recommended spare parts for the PRIMA™ Prevention Mattress Series.

NOTES:

PRIMA SHEAR-LESS PLUS™ Prevention Mattress and PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—P576, P577, and P578

Figure 1-2. PRIMA SHEAR-LESS PLUS™ Prevention Mattress and PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—P576, P577, and P578

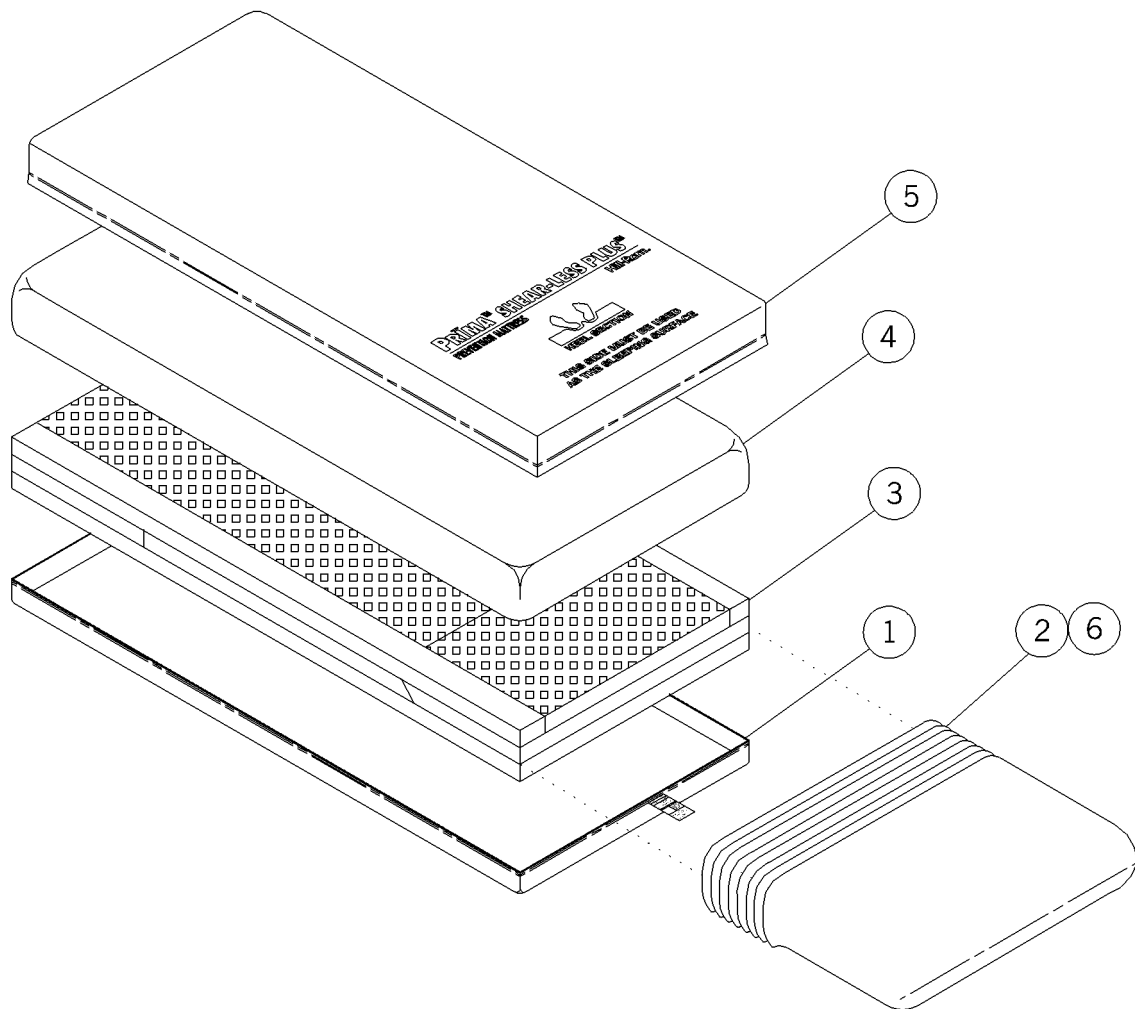

m162a003

**Table 1-5. PRIMA SHEAR-LESS PLUS™ Prevention Mattress—
P576CA1 and P576EA1**

Item Number	Part Number	Quantity	Description
1	100-005-0103	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P576EA1 only)
3	070-009-0001	1	Mattress core
4	100-005-0162	1	Shear cover, 76" (Anti-Shear™ Inner Liner)
5	100-005-0097	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P577EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-6. PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—
P576CA3 and P576EA3**

Item Number	Part Number	Quantity	Description
1	100-005-0230	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P576EA3 only)
3	070-009-0019	1	Mattress core
4	100-005-0228	1	Shear cover, 76" (Anti-Shear™ Inner Liner)
5	100-005-0226	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P577EA3 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-7. PRIMA SHEAR-LESS PLUS™ Prevention Mattress—
P577CA1 and P577EA1**

Item Number	Part Number	Quantity	Description
1	100-005-0104	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P577EA1 only)
3	070-009-0002	1	Mattress core
4	100-005-0077	1	Shear cover, 80" (Anti-Shear™ Inner Liner)
5	100-005-0098	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P577EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-8. PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—
P577CA3 and P577EA3**

Item Number	Part Number	Quantity	Description
1	100-005-0231	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P577EA3 only)
3	070-009-0020	1	Mattress core
4	100-005-0229	1	Inner shear cover (80") (Anti-Shear™ Inner Liner)
5	100-005-0227	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P577EA3 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-9. PRIMA SHEAR-LESS PLUS™ Prevention Mattress—
P578CA1 and P578EA1**

Item Number	Part Number	Quantity	Description
1	100-005-0105	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P578EA1 only)
3	070-009-0003	1	Mattress core
4	100-005-0082	1	Shear cover, 84" (Anti-Shear™ Inner Liner)
5	100-005-0099	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P578EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.


**Table 1-10. PRIMA SHEAR-LESS PLUS™ II Prevention Mattress—
P578CA3 and P578EA3**

Item Number	Part Number	Quantity	Description
1	100-005-0264	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P578EA3 only)
3	070-009-0021	1	Mattress core
4	100-005-0262	1	Shear cover, 84" (Anti-Shear™ Inner Liner)
5	100-005-0263	1	Urethane coverlet (HY•TEX Bi-Stretch™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P578EA3 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

PRIMA SHEAR-LESS™ Prevention Mattress—P708, P709, and P710

Figure 1-3. PRIMA SHEAR-LESS™ Prevention Mattress—P708, P709, and P710

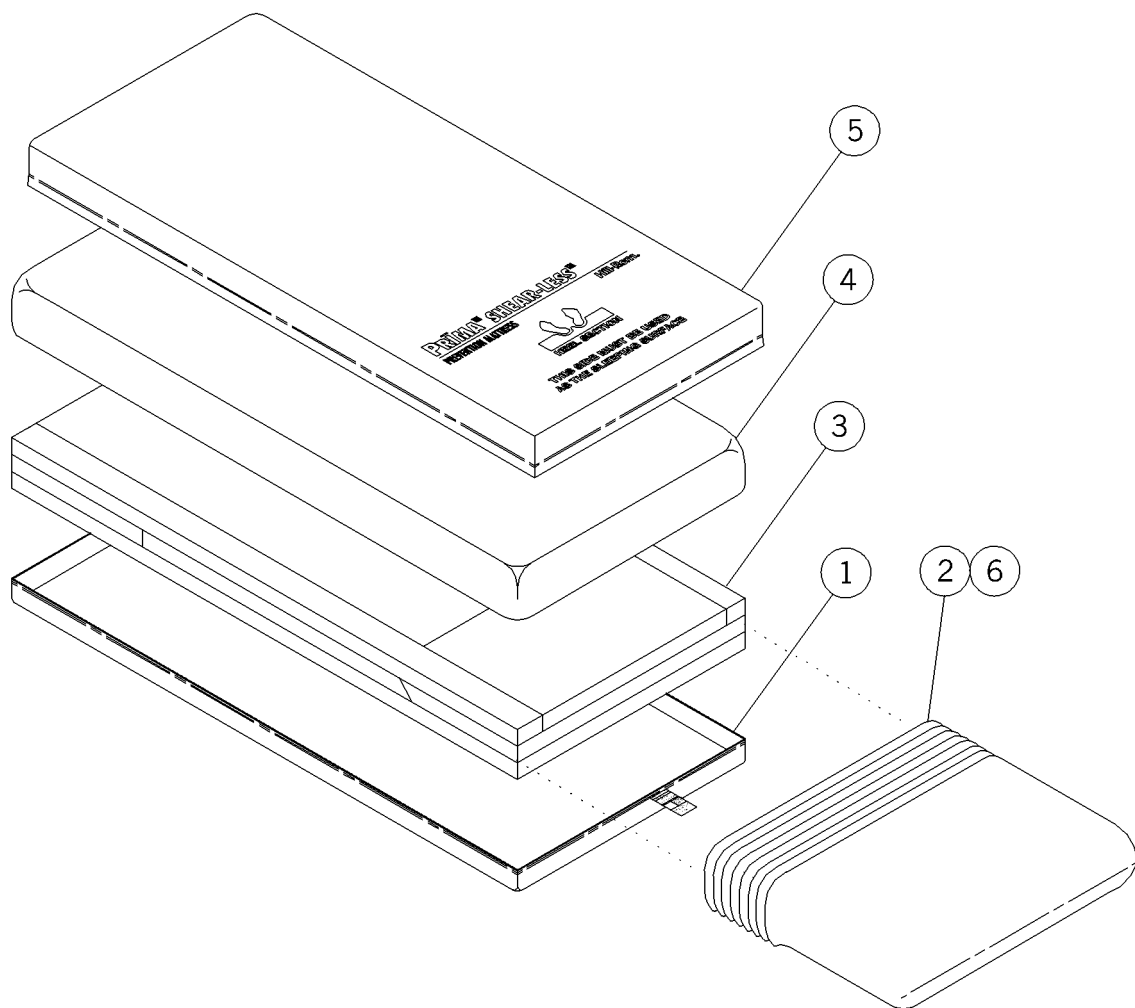

m162a001

**Table 1-11. PRIMA SHEAR-LESS™ Prevention Mattress—
P708CA1 and P708EA1**

Item Number	Part Number	Quantity	Description
1	100-005-0103	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P708EA1 only)
3	070-009-0004	1	Mattress core
4	100-005-0162	1	Shear cover, 76" (Anti-Shear™ Inner Liner)
5	100-005-0100	1	Nylon coverlet (HY•TEX Nylon™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P708EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-12. PRIMA SHEAR-LESS™ Prevention Mattress—
P709CA1 and P709EA1**

Item Number	Part Number	Quantity	Description
1	100-005-0104	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P709EA1 only)
3	070-009-0005	1	Mattress core
4	100-005-0077	1	Shear-cover, 80" (Anti-Shear™ Inner Liner)
5	100-005-0101	1	Nylon coverlet (HY•TEX Nylon™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P709EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-13. PRIMA SHEAR-LESS™ Prevention Mattress—
P710CA1 and P710EA1**


Item Number	Part Number	Quantity	Description
1	100-005-0105	1	Bottom cover
2	100-005-0069	1	Fire barrier mattress (P710E models only)
3	070-009-0006	1	Mattress core
4	100-005-0082	1	Shear cover, 84" (Anti-Shear™ Inner Liner)
5	100-005-0102	1	Nylon coverlet (HY•TEX Nylon™ Cover)
6	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P578EA1 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

NOTES:

PRIMA™ Prevention Mattress Series—P711, P712, and P713

Figure 1-4. PRIMA™ Prevention Mattress Series—P711, P712, and P713

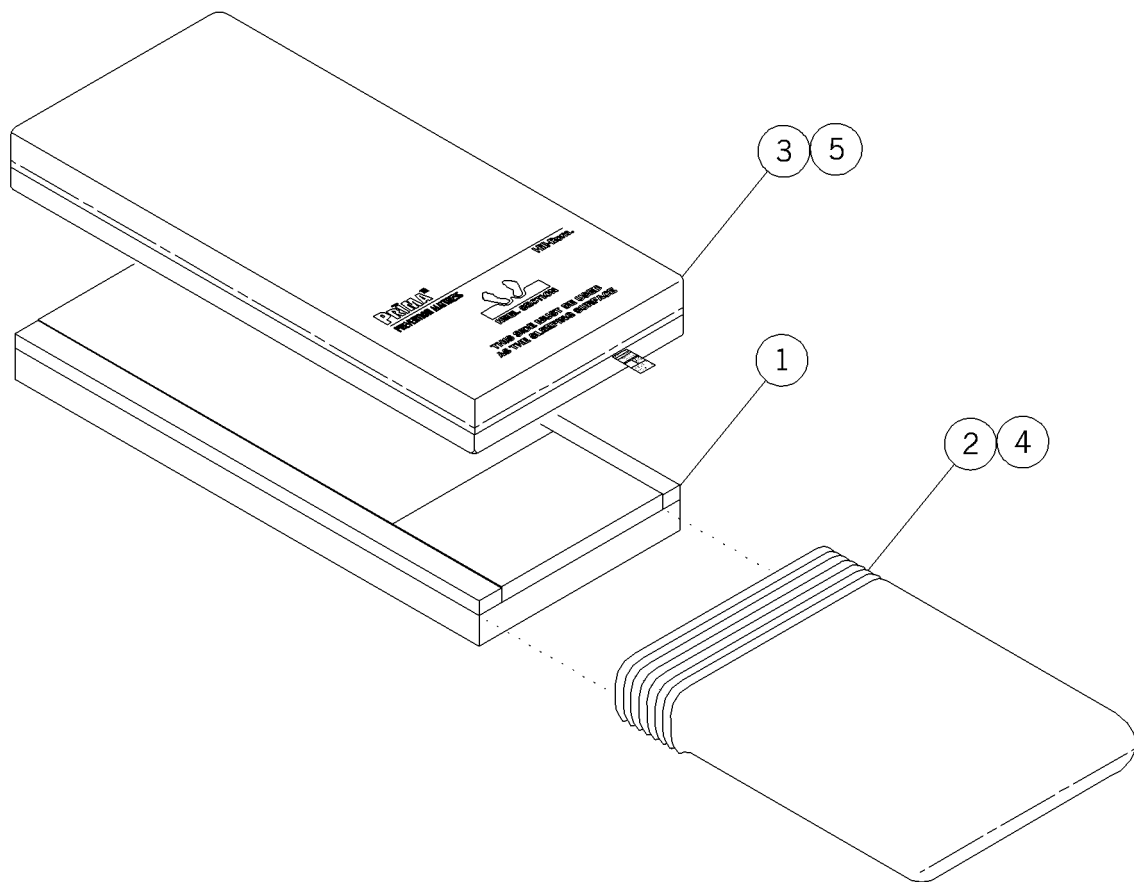

m162a002

**Table 1-14. PRIMA™ Prevention Mattress Series—
P711CA1/P711EA1/P711CA2/P711EA2**

Item Number	Part Number	Quantity	Description
1	100-005-0069	1	Fire barrier mattress (P711EA1 only)
2	070-009-0007	1	Mattress core
3	100-005-0112	1	Ticking assembly (P711CA1 and P711EA1 only)
4	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P711EA1 only)
5	100-005-0192	1	Psychiatric ticking assembly (P711CA2 and P711EA2 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-15. PRIMA™ Prevention Mattress—
P712CA1/P712EA1/P712CA2/P712EA2**

Item Number	Part Number	Quantity	Description
1	100-005-0069	1	Fire barrier mattress (P712EA1 only)
2	070-009-0008	1	Mattress core
3	100-005-0113	1	Ticking assembly (P712CA1 and P712EA1 only)
4	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P712EA1 only)
5	100-005-0193	1	Psychiatric ticking assembly (P712CA2 and P712EA2 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

**Table 1-16. PRIMA™ Prevention Mattress—
P713CA1/P713EA1/P713CA2/P713EA2**

Item Number	Part Number	Quantity	Description
1	100-005-0069	1	Fire barrier mattress (P713EA1 only)
2	070-009-0009	1	Mattress core
3	100-005-0114	1	Ticking assembly (P713CA1 and P713EA1 only)
4	001-005-0066	As required	Fastener, attachment 1" Swiftach® ^a (P713EA1 only)
5	100-005-0194	1	Psychiatric ticking assembly (P713CA2 and P713EA2 only)

a. Swiftach® is a registered trademark of Dennison Manufacturing Company.

1.1 Installation



CAUTION:

The mattress may be exposed to extreme temperatures during shipping. Allow the mattress to adjust to the room temperature before unpacking. Failure to do so could result in equipment damage.

1. Before unpacking, allow the mattress to adjust to the room temperature.




WARNING:

Ensure the mattress is installed with the foot symbol facing up at the foot end of the bed frame. Failure to do so could result in personal injury or equipment damage.

2. Install the mattress on the bed frame with the foot symbol (A) facing up at the foot end of the bed frame (see figure 1-5 on page 1-25).

Figure 1-5. PRIMA™ Prevention Mattress Series Installation

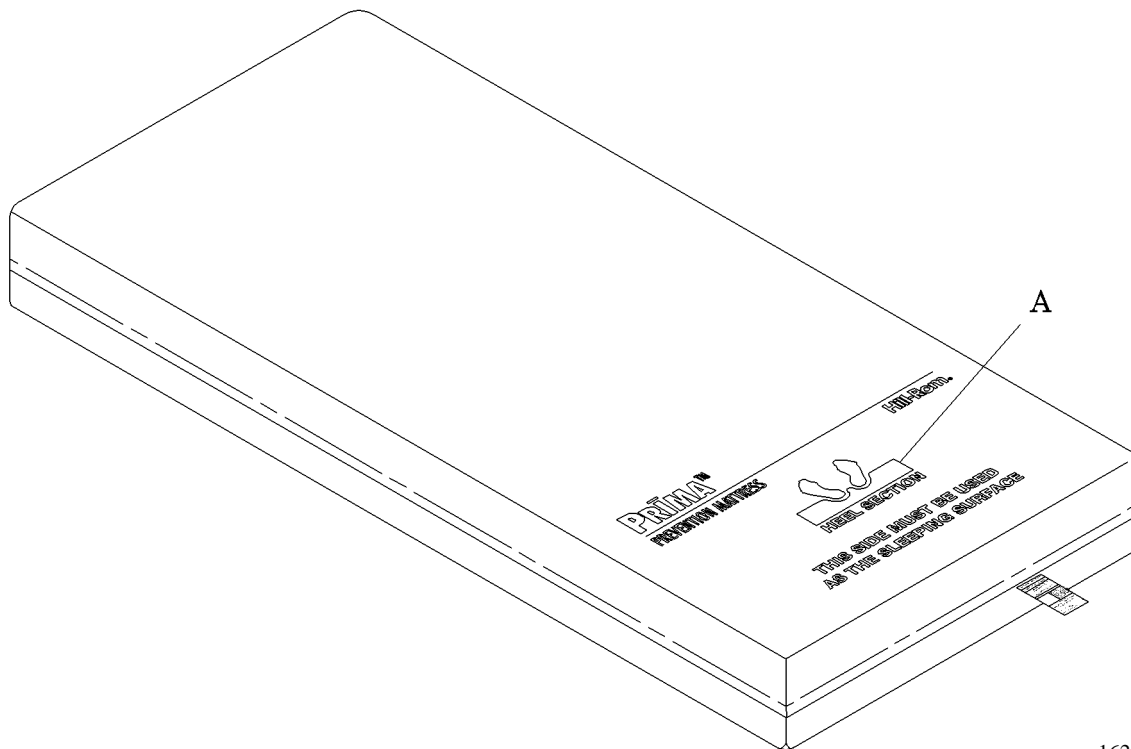

m162a004



WARNING:

Allow any folded mattress to renew its shape for 1 to 2 hours before placing a patient on the surface. Failure to do so could result in equipment damage or patient discomfort.

3. After the mattress is installed, allow the mattress 1 to 2 hours to renew its shape.
4. Install a sheet over the mattress. Ensure the sheet fits the mattress properly.

1.2 General Cleaning and Inspection

Ensure compliance with Occupational Safety and Health Administration (OSHA) regulations and Centers for Disease Control and Prevention (CDC) recommendations relating to bloodborne pathogens. To disinfect the mattress, use an intermediate-level disinfectant registered with the Environmental Protection Agency (EPA). When disinfecting, follow the standard precautions and the manufacturer's instructions.

Mattress Disinfection

**CAUTION:**

Do not expose the mattress to excessive moisture. Equipment damage may occur.

1. Remove visible soil from the mattress surface by wiping the mattress with a soft cloth or paper towel dampened with soapy water.
2. Disinfect the mattress with an EPA-registered, intermediate-level disinfectant.
 - a. Follow the manufacturer's recommendations for use and dilution.
 - b. Do not saturate the material or allow cleaning solutions to pool on the mattress.
3. Allow the surface to air dry.

Exterior Mattress Inspection

**CAUTION:**

Do not use a mattress with damaged components. Equipment damage could occur.

1. Inspect the cover for cuts, rips, cracks, or tears. Do not use a mattress with damaged components.
2. If the cover is damaged or has a stain that cannot be removed, set the mattress aside, and notify Hill-Rom according to your facility's call-in policy.

Interior Mattress Inspection (PRIMA SHEAR-LESS PLUS™ Prevention Mattress, PRIMA SHEAR-LESS PLUS™ II Prevention Mattress, and PRIMA SHEAR-LESS™ Prevention Mattress Only)




WARNING:

When unzipping, keep the Anti-Shear™ Inner Liner from becoming entangled with the zipper. Failure to do so could result in equipment damage or personal injury.

1. Unzip the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A) from the bottom cover (B) (see figure 1-6 on page 1-28).
 - a. Pull the zipper with one hand.
 - b. Use the other hand as a barrier between the Anti-Shear™ Inner Liner (C) and the sides of the bottom cover (B) and HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A).

Figure 1-6. Mattress Interior Inspection

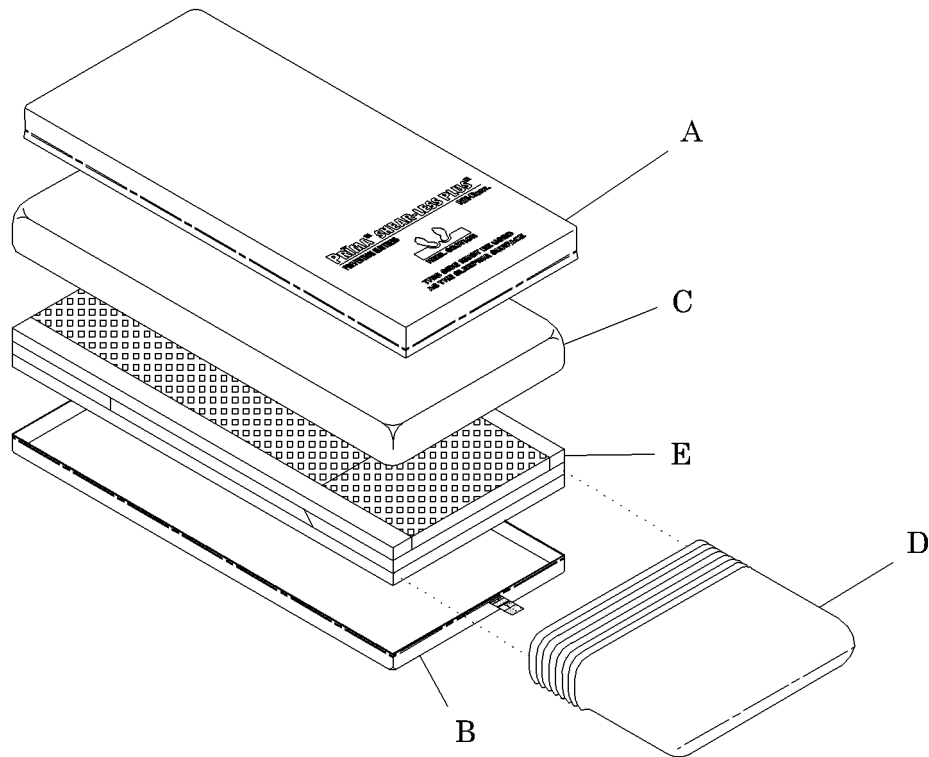

m162a006

2. Inspect the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A), Anti-Shear™ Inner Liner (C), and bottom cover (B) for soiling or damage.

3. If any components are soiled or damaged, set the mattress aside, and notify Hill-Rom according to your facility's call-in policy.

**CAUTION:**

Do not remove the flame barrier (if present) from the mattress core. Equipment damage may occur.

4. Inspect the flame barrier (D) (if present) for soiling or damage.
 - a. If the flame barrier (D) is soiled or damaged, set the mattress aside, and notify Hill-Rom according to your facility's call-in policy.
 - b. If the flame barrier (D) is not present, inspect the mattress core (E) for soiling or damage.
5. If the mattress is damaged or has soiling that cannot be removed, set the mattress aside, and notify Hill-Rom according to your facility's call-in policy.

**WARNING:**

Install the mattress core yellow foam section face up at the foot end of the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover. Failure to do so could result in equipment damage or personal injury.

6. Install the mattress core (E) yellow foam section face up at the mattress foot end in the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A).
7. Assemble in reverse order.

1.3 Flame Barrier Installation (PRIMA SHEAR-LESS PLUS™ Prevention Mattress, PRIMA SHEAR-LESS PLUS™ II Prevention Mattress, and PRIMA SHEAR-LESS™ Prevention Mattress with Flame Barrier Only)

Installation




WARNING:

When unzipping, keep the Anti-Shear™ Inner Liner from becoming entangled with the zipper. Failure to do so could result in equipment damage or personal injury.

1. Unzip the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A) from the bottom cover (B) (see figure 1-7 on page 1-30).
 - a. Pull the zipper with one hand.
 - b. Use the other hand as a barrier between the Anti-Shear™ Inner Liner (C) and the sides of the bottom cover (B) and HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A).

Figure 1-7. PRIMA SHEAR-LESS PLUS™ Prevention Mattress and PRIMA SHEAR-LESS™ Prevention Mattress Flame Barrier Installation

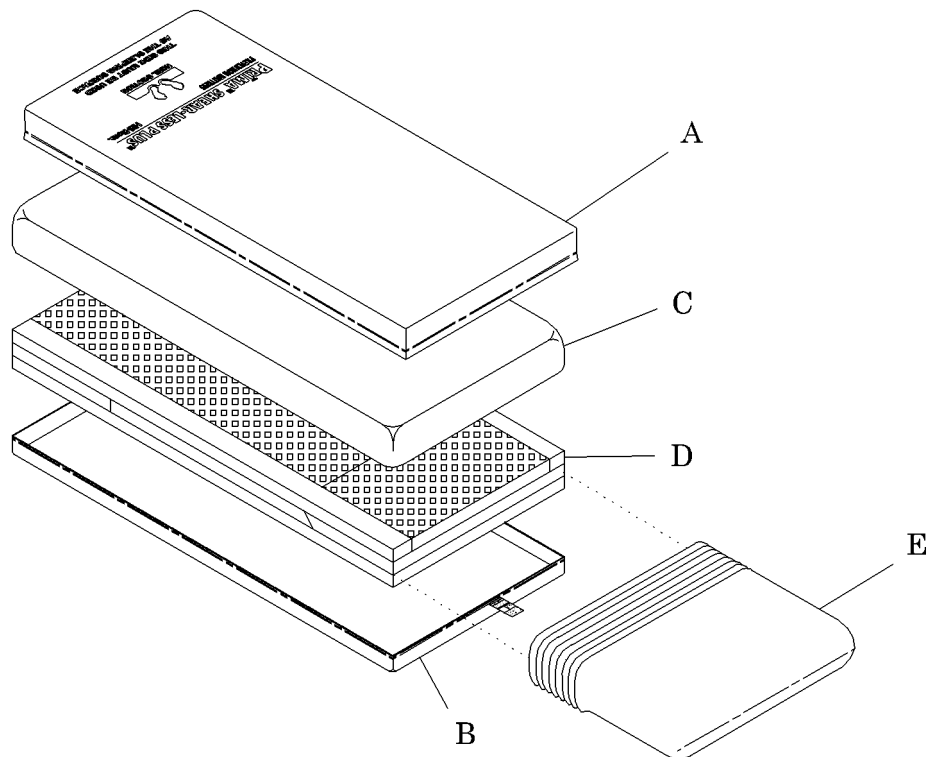

m162a007

2. Remove the Anti-Shear™ Inner Liner (C) from the mattress core (D).
3. Install the flame barrier (E) around the mattress core (D).
 - a. Roll the flame barrier (E) at the head end of the mattress core (D).
 - b. Unroll the flame barrier (E) around the mattress core (D) until it closes at the foot end.
 - c. Ensure any excess flame barrier (E) is under the foot end of the mattress core (D).



WARNING:

Use caution when fastening the Swiftach®¹ attachment fasteners. Do not attach the Swiftach® attachment fasteners to anything other than the flame barrier. Equipment damage or personal injury may occur.

4. Secure the flame barrier (E) foot end around the mattress core (D) by fastening the Swiftach® attachment fasteners to the flame barrier (E) only.



WARNING:

Install the mattress core yellow foam section face up at the foot end of the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover. Failure to do so could result in equipment damage or personal injury.

5. Insert the mattress core (D) yellow foam section face up at the foot end of the HY•TEX Bi-Stretch™ Cover or HY•TEX Nylon™ Cover (A).
6. Assemble in reverse order.


1. Swiftach® is a registered trademark of Dennison Manufacturing Company.

1.4 Flame Barrier Installation (Standard Model PRIMA™ Prevention Mattress with Flame Barrier Only)

Installation

1. Unzip the ticking assembly (A) along the bottom (see figure 1-8 on page 1-32).

Figure 1-8. PRIMA™ Prevention Mattress Flame Barrier Installation

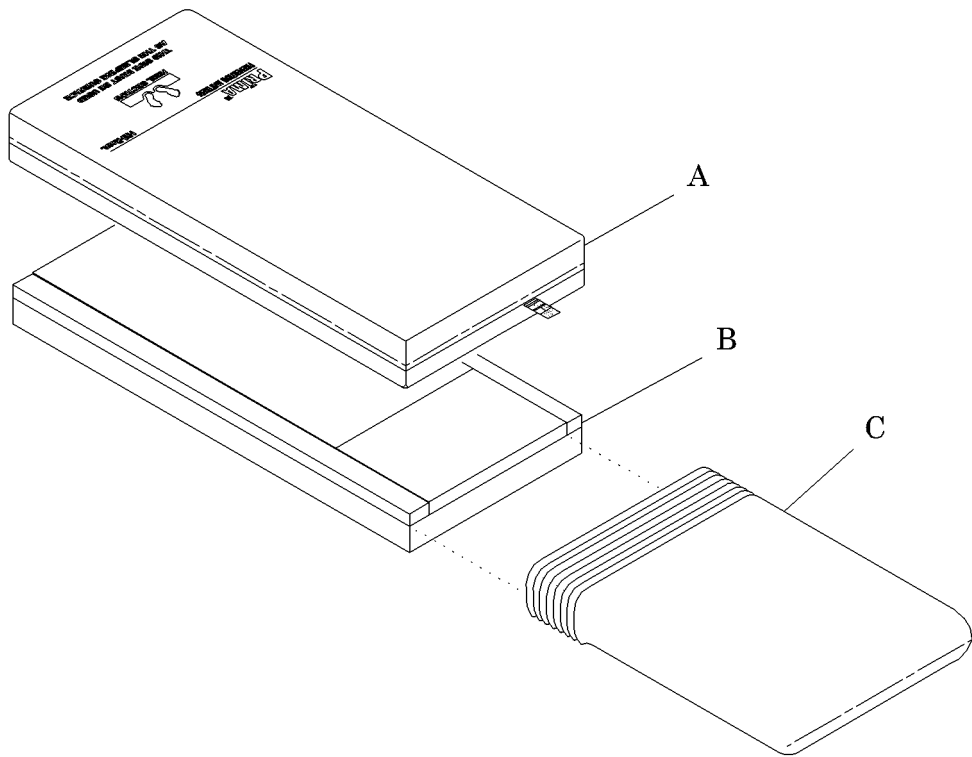

m162a008

2. Remove the ticking assembly (A) from the mattress core (B).
3. Install the flame barrier (C) around the mattress core (B).
 - a. Roll the flame barrier (C) at the head end of the mattress core (B).
 - b. Unroll the flame barrier (C) around the mattress core (B) until it closes at the foot end.
 - c. Ensure any excess flame barrier (C) is under the foot end of the mattress core (B).



WARNING:

Use caution when fastening the Swiftach®¹ attachment fasteners. Do not attach the Swiftach® attachment fasteners to anything other than the flame barrier. Equipment damage or personal injury may occur.

4. Secure the flame barrier (C) foot end around the mattress core (B) by fastening the Swiftach® attachment fasteners to the flame barrier (C) only.



WARNING:

Install the mattress core yellow foam section face up at the foot end of the ticking assembly. Failure to do so could result in equipment damage or personal injury.

5. Install the mattress core (B) yellow foam section face up at the foot end of the ticking assembly (A).
6. Assemble in reverse order.

1. Swiftach® is a registered trademark of Dennison Manufacturing Company.

NOTES:
