
OmniVentLine™ Set
Intubated Adult and Pediatric Patients
Directions for Use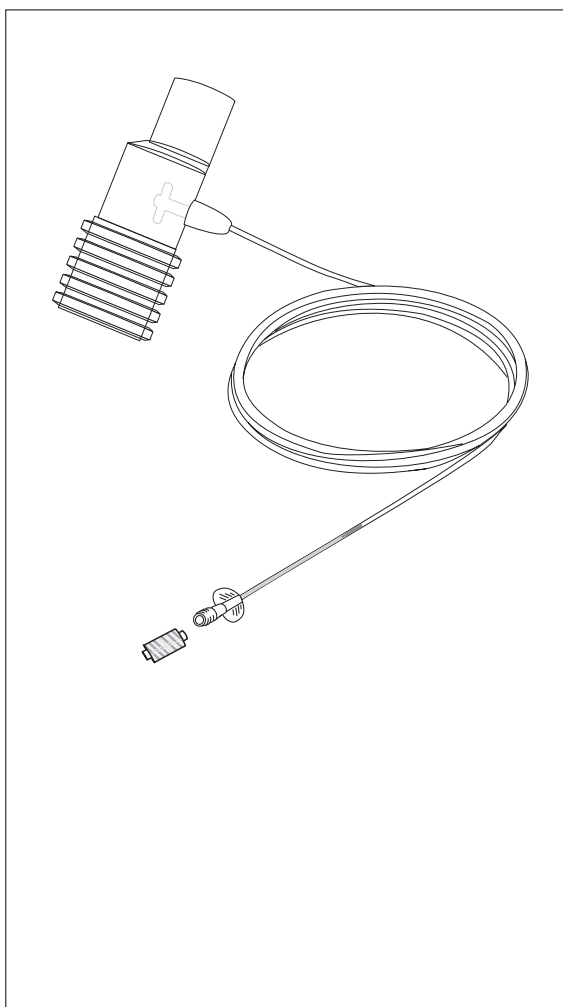

US Patents: www.covidien.com/patents.

© 2015 Oridion Medical 1987 Ltd.
All rights reserved.

PN 012808E

OmniVentLine™

Adult-Pediatric CO₂ Sampling Line and Airway Adapter

Omnistream™ EtCO₂ Sampling
for Intubated Patients
Adult and Pediatric Patients

Airway Adapters for Adult to Pediatric patients:

- Added dead space <6.6 cc.
- To be used with an endotracheal tube > 4.5mm bore.
- Suitable for use with 22mm and 15mm patient tubing.

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Does not contain natural rubber latex

Warning: When used with closed suction system, do not place the airway adapter between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Carefully route the OmniVentLine to reduce the possibility of patient entanglement or strangulation.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Note: During nebulization or suction, in order to avoid moisture buildup and sampling line occlusion, remove the sampling line luer connector from the monitor.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.



Note: When connecting a sampling line to the monitor, insert the sampling line connector with its luer connector clockwise into the monitor CO₂ port and turn the connector until it can no longer be turned, to ensure that it is connected securely to the monitor. This will assure that there is no leak of gases during measurement at the connection point and that measurement accuracy is not compromised.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

Omnistream™ EtCO₂ Sampling for Intubated Patients

Instructions for use:

1. Firmly connect the small-end (male) of the Omnistream™ Airway Adapter **1** to the female-end of the Wye piece (ventilation source) .
2. Firmly connect the patient's endotracheal tube connector into the large-end (female) **2** of the Omnistream™ Airway Adapter.
3. Ensure that the tubing is not twisted or crimped.
4. The male-male luer adapter/connector **4** will already be attached to the sampling line **3**. Twist the sampling line with luer connector (3 and 4, attached) securely but gently into the capnograph input connector **5** .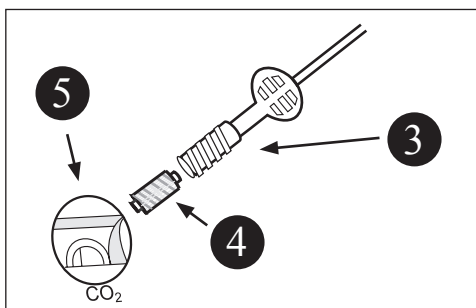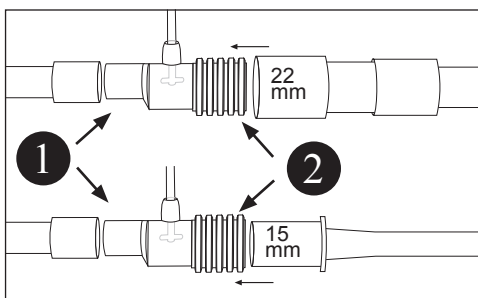


Omnistream™ EtCO₂ Sampling with a Closed Suction System

Instructions for use with a closed suction system:

1. Set up the parts of the system as seen in the sketch below. See legend.
2. Connect the system as seen in the sketch at top right as follows:
3. Firmly connect the small-end (male) of the Omnistream™ Airway Adapter ⑥ to the female-end of the Wye piece (ventilation source) ⑦.
4. Firmly connect the large-end (female) of the Omnistream™ airway adapter ⑥ to the ventilator connector on the closed suction system ⑧, so that the airway adapter does not interfere with the functioning of the suction tubing ⑨.
5. The setup seen in the sketch at bottom right is incorrect; if set up in this way, the airway adapter will interfere with the functioning of the suction tubing.


Setup of Omnistream™ EtCO₂ Sampling with a closed suction system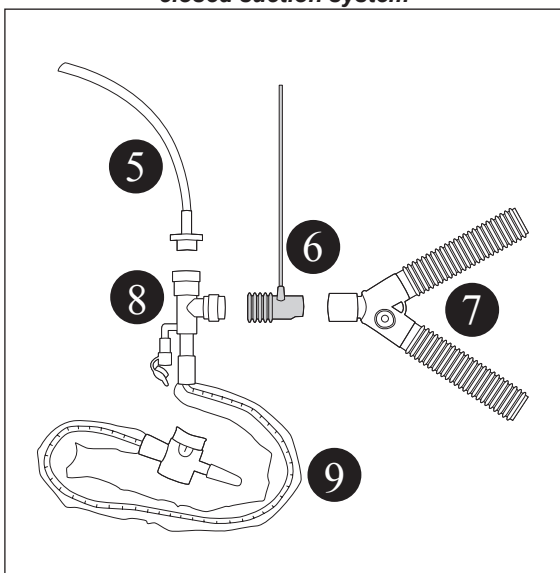


Legend

1. Male-end of airway adapter
2. Female end of airway adapter
3. Female luer connector
4. Omnistream™ input port
5. Endotracheal tube
6. OmniVentLine Set with airway adapter
7. Wye piece (ventilation source)
8. Ventilator connector on closed suction system, with lavage
9. Suction tubing

***Correctly Connected
Omnistream™ EtCO₂ Sampling
with a closed suction system***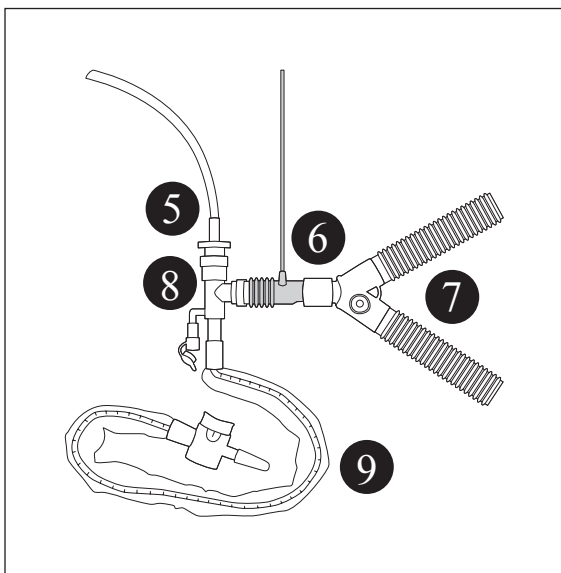

***Incorrectly Connected
Omnistream™ EtCO₂ Sampling
with a closed suction system***

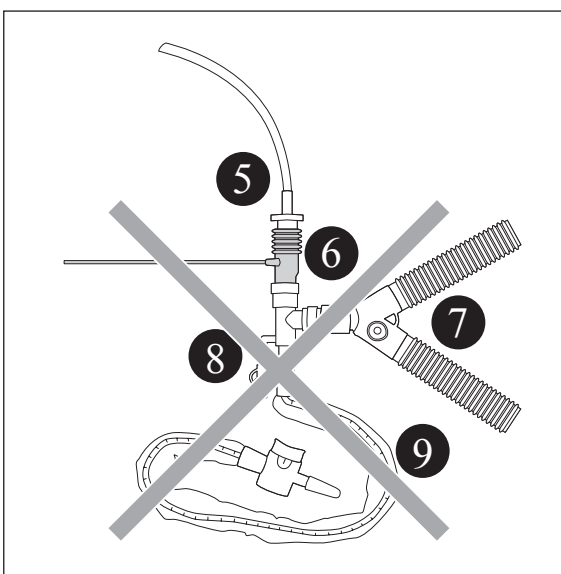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 Single patient use	For use with a single patient
Rx ONLY	Rx Only
 Consult instructions for use	Consult instructions for use
 Caution, consult accompanying documents	Caution, consult accompanying documents
 Temperature limitation	Storage temperature
 This product does not contain natural rubber latex	Does not contain natural rubber latex
CE	CE Mark

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Oridion®, Microstream®, FilterLine®, Smart CapnoLine®, CapnoLine®, Smart CapnoLine Guardian™ Omnistream™ and OmniVentLine™ are trademarks or registered trademarks of Oridion Medical 1987 Ltd.™* and ®* are trademarks of their respective owners.

Oridion Medical 1987 Ltd.
7 Hamarpe St.
P.O. Box 45025
Jerusalem 9777407, Israel
US Toll Free: 1-888-674-3466
1-888-ORIDION

Obelis S.A.
Bd Général Wahis 53
1030 Brussels
Belgium